

Q-Optics

Through the Lens (TTL) Loupes



USER MANUAL

PLEASE READ AND FOLLOW THESE INSTRUCTIONS PRIOR TO
USING YOUR Q-OPTICS TTL STYLE LOUPES

Table of Contents

Saddle / Nose Pad Adjustment.....	3
Ear Tip Adjustment	3
Cleaning and Disinfecting	4
Prescription Service – Rx	4
Attach or Remove the Rx Clip.....	5
Remove Name Plate	5
Attaching a Headlight	6
Accessories	6
Optional Accessories and Replacement Parts	7

Q-Optics

Product Support Info

Toll-Free U.S. and Canada: 1-800-858-2121

Phone: (972) 298-2669

Fax: (972) 298-6592

Web Site: www.q-optics.com

Email: support@q-optics.com



P.O. Box 382120 • Duncanville, TX 75138
Phone: (972) 298-2669 • 1-800-858-2121 • Fax: (972) 298-6592

Wearing the Frame

Frames should rest high on the nose.



Correct



Incorrect

Saddle / Nose Pad Adjustment:


To adjust the nose pads or saddle so the loupes will sit higher on the nose gently press the pads/saddle closer together.

To adjust the nose pads or saddle so the loupes will sit lower on the nose spread the pads/saddle further apart.



To raise loupes on face press pad or saddle closer together

Raise Loupes

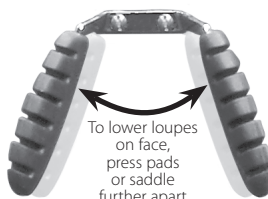

To lower loupes on face, press pads or saddle further apart

Lower Loupes

Ear tips:



Bend ear tips for a more secure, comfortable fit.

Note: You may also visit your local optometrist or eyeglass shop for an adjustment.

Cleaning / Disinfecting

Use Q-Optics cleaning solution. It is OK to rinse your loupes under fast moving room temperature water.

NEVER SOAK LOUPES.

It is OK to disinfect loupes using alcohol, Lysol® spray, Cavi-Wipes™, Opti-Cide® or other surface disinfectant.



Warning



Never attempt to sterilize loupes by any method!
Never soak loupes in any solution – not even water!
Never place loupes in an ultrasonic cleaner!

Prescription Service – Rx

Q-Optics loupes provide a magnified view for improved performance. Q-Optics do not correct vision. If you work with prescription lenses or you normally work with reading glasses then you should use Q-Optics loupes with your prescription installed. Most single strength and bifocal prescriptions can be placed in the frame by Q-Optics at a nominal charge with the purchase of a new set of loupes. For ease of use, progressive lenses and trifocal prescriptions are typically converted to flat top bifocals. Prescriptions for Q-Optics may also be obtained and placed by your local optometrist or eyeglass provider.





Attach or Remove the RX Clip



Attach the RX Clip

Line up the slot located on the left side of the RX clip with the mounts located on the inside of the frame as shown above.

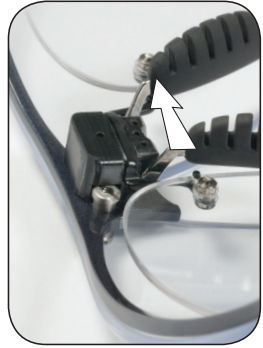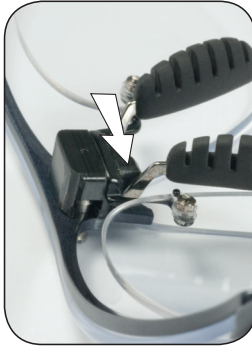

Remove the RX Clip

Slide the clip off the frame as shown above.

Remove the Name Plate

1 Apply pressure to the bottom center as shown

2 Slide name plate up and off to expose mounting screws



Attaching a Headlight

1 Align slot in headlight base with mounting screw as shown

2 Slide headlight over the mounting screws



Accessories (included)

- Side shields
- Micro fiber cleaning cloth
- Additional nose pad set or saddle
- Lens cap covers
- Sport strap or elastic band



Accessories will vary depending on frame style and telescope type.

Optional Accessories & Replacement Parts

**Styles may vary*



Combo Case
Loupe & Headlight



Loupe Case



Sport Strap
Cloth End



Sport Strap
Silicone End



Replacement Nose Pads



Cleaning Solution



Cleaning Cloth

Q-Optics

For more information contact:

P.O. Box 382120 • Duncanville, TX 75138
1-800-858-2121 • (972) 298-2669 • Fax: (972) 298-6592
www.q-optics.com
support@q-optics.com

www.q-optics.com

Have a better look[®].

© 2012 Q-Optics. All rights reserved. Product specifications are subject to change at any time without prior notice.
UI308-10003-01, ECN 1260