



Hand Hygiene Training and Management Information System User Manual

GLANTA Ltd

The Tower, Trinity Enterprise Centre,

Pearse St, Dublin 2

IRELAND

www.surewash.com

+353 (0)1 677 8894

1 TABLE OF CONTENTS

2	Setting up the SureWash system	4
3	Using the SureWash system	5
3.1	Location selection	5
3.2	User login.....	6
3.3	Training mode.....	15
3.4	Assessment mode	18
4	SureWash administration	21
4.1	Logging in as an administrator.....	21
4.2	View records	23
4.3	Administrator management.....	24
4.3.1	Register new Administrator ID card	26
4.4	Change location	27
4.5	Generate report.....	28
4.6	Reports	30
4.6.1	User report	31
4.7	Configuration	32
4.8	Content customisation	33
4.9	Questionnaire.....	34
4.10	System upgrade	35
4.11	Licence update	36
4.12	System diagnostics.....	37
4.13	User data backup	38
4.13.1	Delete user data	38

4.14	Device Settings	39
5	Support and troubleshooting	41
6	Legal notices	43
7	CE compliance.....	44

2 SETTING UP THE SUREWASH SYSTEM


To set up the SureWash system, plug in the power supply and press the on/off button located on the right-hand side of the system. The SureWash system will boot up automatically.



To turn off the system, press and hold the on/off button until the system shuts down. When the system has completely shut down (this takes approximately 30 seconds) the unit may be unplugged from the mains power. SureWash should not be unplugged from the mains power before being shut down.

3 USING THE SUREWASH SYSTEM

3.1 LOCATION SELECTION


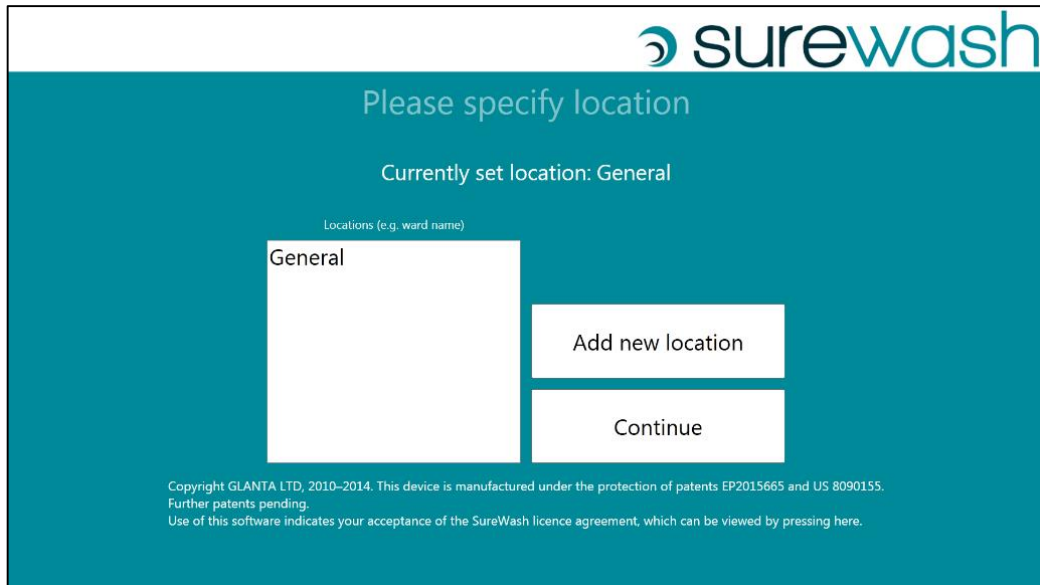


The screenshot shows the SureWash interface for location selection. At the top right is the SureWash logo. The main heading is "Please specify location". Below this, it states "Currently set location: General". A text input field labeled "Locations (e.g. ward name)" contains the text "General". To the right of the input field are two buttons: "Add new location" and "Continue". At the bottom of the screen, there is a small copyright notice: "Copyright GLANTA LTD, 2010-2014. This device is manufactured under the protection of patents EP2015665 and US 8090155. Further patents pending. Use of this software indicates your acceptance of the SureWash licence agreement, which can be viewed by pressing here."

After the system has been switched on, you will be prompted to specify the location where the SureWash Training Unit is currently operating. This could be, for example, the name of a hospital department. You may select an existing location from the list or you may enter a new location. If you select 'Add new location', an on-screen keyboard will appear allowing you to enter the name of the location. Once the location has been selected press 'Continue' to start SureWash.

The "Add new location" option may be disabled by the Administrator. See 4.4 Change Location.

3.2 USER LOGIN

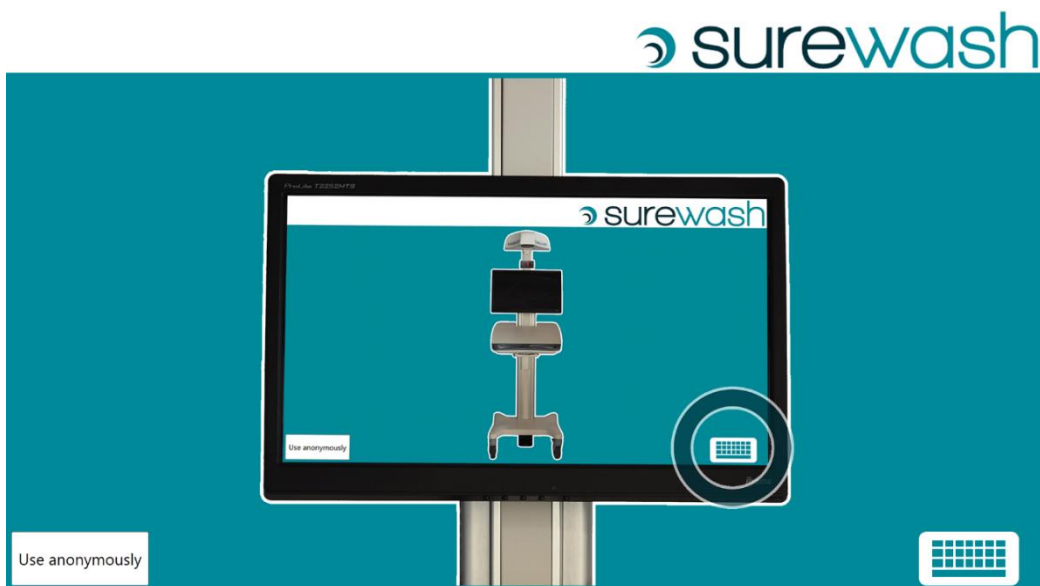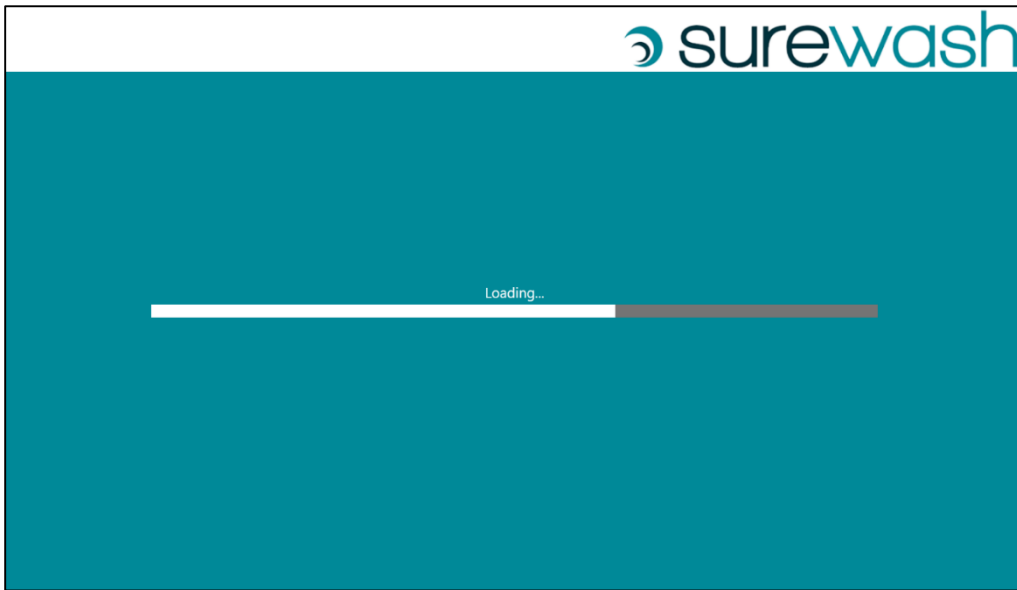


Next, the system initialises and the login screen appears. Depending on the configuration of the SureWash unit a user can log in to the system in three ways:

1. Anonymously

Press the anonymous button and use SureWash for Training, doing the Quiz or for Assessment. All interactions are counted as Anonymous.


2. Manually

Press the keyboard icon and using your SureWash User ID (this can be a number and is indicated by Administrator) log in to SureWash for Training, doing the Quiz or for Assessment. All interactions are counted under individual usernames.

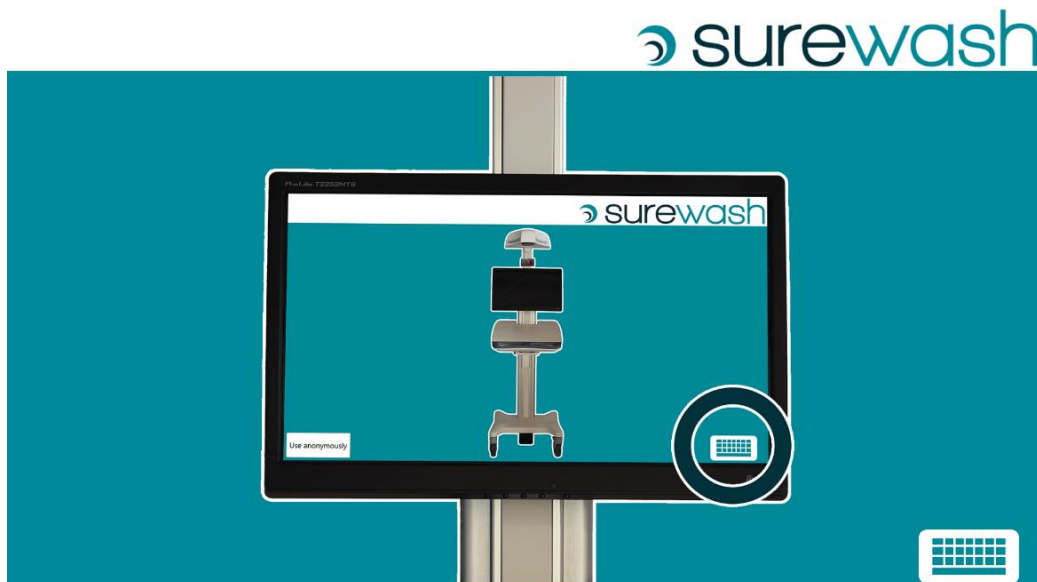

3. Using Finger Print Reader (FPR)

Using FPR as indicated (use finger or thumb as indicated by Administrator) on your unit. SureWash recommends that Roles and Departments are preloaded by the Administrator to ensure better quality reporting. (Contact SureWash for further information if you wish to automate this process.)

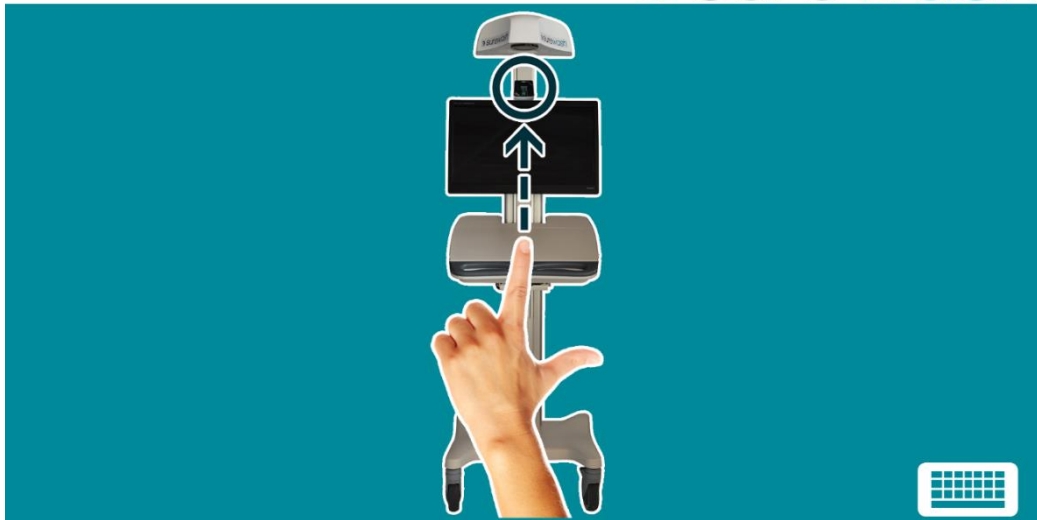

When logging in manually or by FPR for the first time a user will be asked to complete the following information.

Welcome! This is the first time you have logged in.

Press OK to proceed and your ID will be registered.

Press Cancel if you are already registered and try to log in again.

OK Cancel

A registration form with a teal background. It contains four input fields: 'First name', 'Last name', 'Role', and 'Department'. The 'Role' and 'Department' fields have dropdown arrows on their right sides. At the bottom, there are three buttons: 'Next', 'OK', and 'Cancel'.

An on-screen keyboard interface with a teal background. At the top, it says 'Enter first name' and has a language dropdown set to 'English'. Below this is a text input field containing 'Mohammed'. The keyboard has several rows of keys: numbers 1-0 with a '+' key, letters q-w-e-r-t-y-u-i-o-p with a '.' key, letters a-s-d-f-g-h-j-k-l with a '/' key, CAPS, z-x-c-v-b-n-m, BACKSPACE, and a '' key. The bottom row has CANCEL, SPACE, ENTER, and a '-' key. The 'd' key is highlighted in blue.

Enter last name English

Nassar

1	2	3	4	5	6	7	8	9	0	+
q	w	e	r	t	y	u	i	o	p	.
a	s	d	f	g	h	j	k	l	/	
CAPS	z	x	c	v	b	n	m	BACKSPACE	'	
CANCEL		SPACE					ENTER		-	

First name

Last name

Role

Department

A screenshot of a user registration form with a teal background. The form contains four input fields: 'First name' with 'Mohammed', 'Last name' with 'Nassar', 'Role' with 'Administration', and 'Department' with 'Administration'. A dropdown menu is open for the 'Department' field, showing a list of departments: 'Audiologist', 'Biomedical scientist', 'Building works', 'Catering', 'Clinical psychologist', and 'Clinical scientist'. A 'Next' button is visible at the bottom left of the form.

Roles can be pre-loaded by SureWash Support Team or amended on-site (see Customising your System). Drop down list will be alphabetical.

A screenshot of the same user registration form, but with the 'Department' dropdown menu closed. The 'Department' field now displays 'Cardiology'. At the bottom of the form, there are three buttons: 'Next', 'OK', and 'Cancel'.

Place finger on the sensor



Back

OK

Cancel



Place finger on the sensor



Back

OK

Cancel

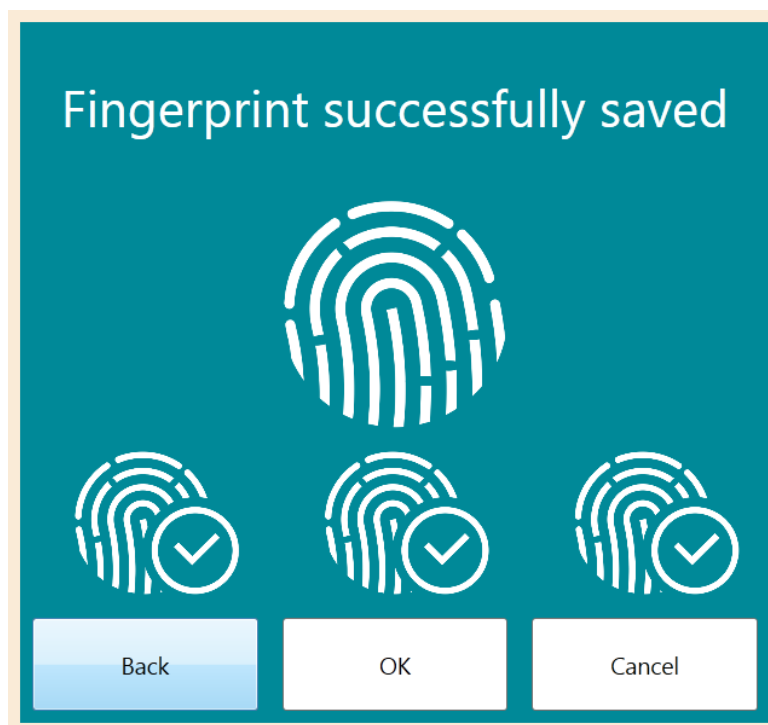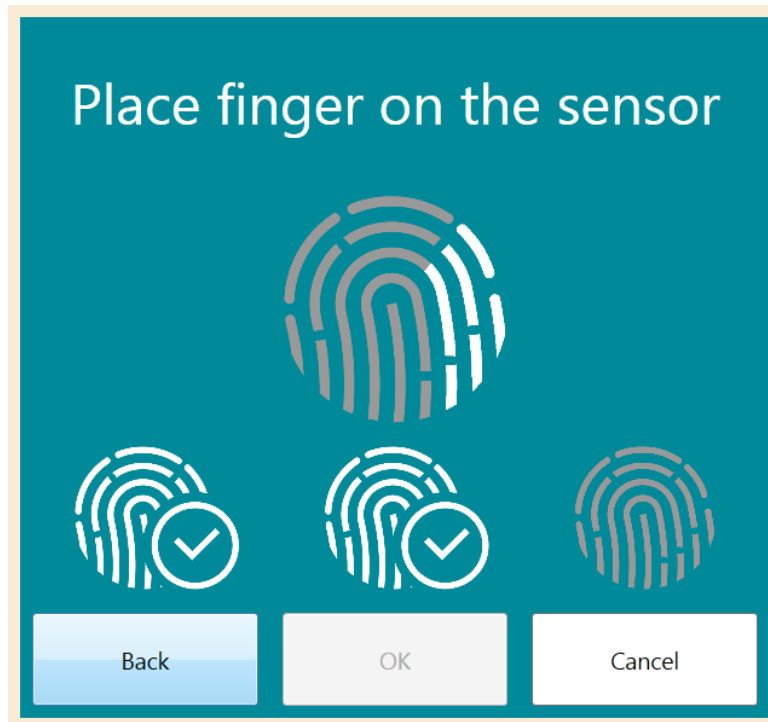

A user only completes this process the first time they log in.

Log out

Logged in
Mohammed Nassar

 surewash

Main Menu

SureWash is a hand hygiene training and assessment tool in accordance with the WHO hand hygiene protocol.



Training: The training aims to help you ensure that every time you wash your hands, they are cleaned fully with no missed spots.

Assessment: The assessment determines your score in cleaning your hands according to the hand hygiene poses of the WHO protocol.

Training

Quiz

Assessment

Help

When logged in, your first, last name will be displayed in the top left hand corner. The user can choose between the three modes that are offered by the SureWash system: **Training**, **Quiz** and **Assessment**. (Pressing the **Help** button presents the user with information about how to use SureWash.)

Please note: User must be in **Assessment** mode to be recorded in Reports.


3.3 TRAINING MODE

The Training mode contains general information about hand hygiene (Why, When, How) and also allows the users to practise their hand hygiene skills.




By pressing the on-screen arrows on the e-learning material, a user can navigate through the content. Users can also press the images to go to training material that is specific to a pose.


Training for each of the seven hand hygiene poses happens in two steps: firstly, some information about this particular pose is presented to the user, e.g., what makes this particular pose so important. Additionally, a video of the pose being completed correctly is shown.

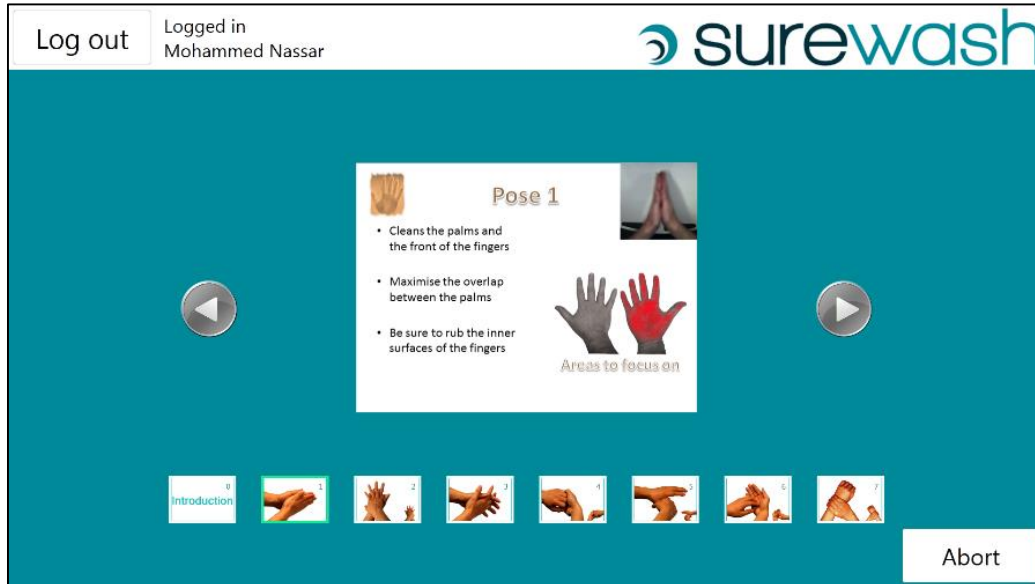


By pressing the right arrow, the user has the opportunity to perform this particular pose. The system camera checks if the user is performing the pose correctly. If they are, the progress bar shown at the bottom of the screen starts to grow. When the pose is completed, the Training screen moves to the next pose and the procedure begins again.

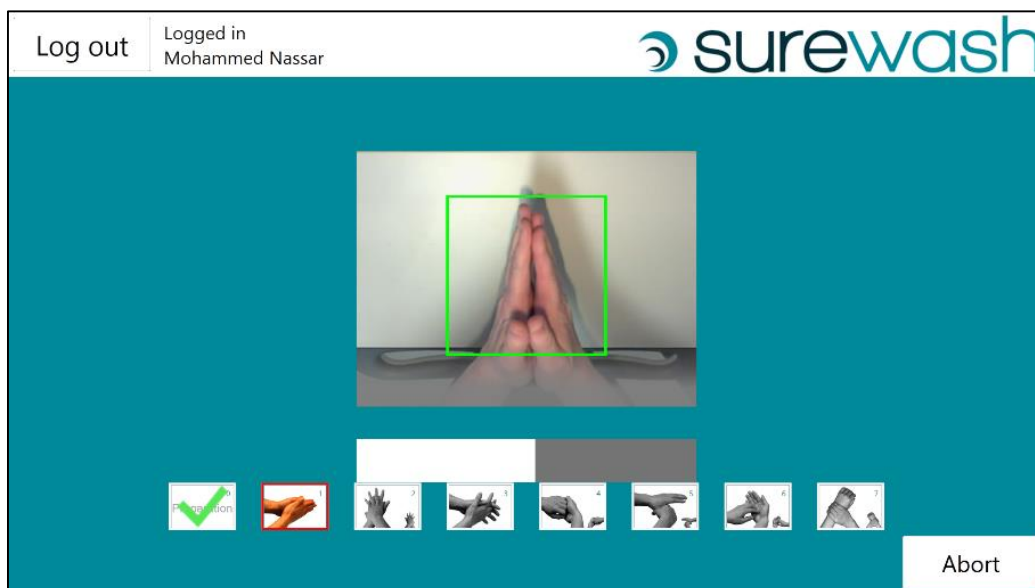

If a user does not manage to finish a pose within a specific time, an information box will pop up. The popup helps the user to pass this pose by showing the mistakes that are commonly made. After the popup, there is still some time left to finish the pose successfully.

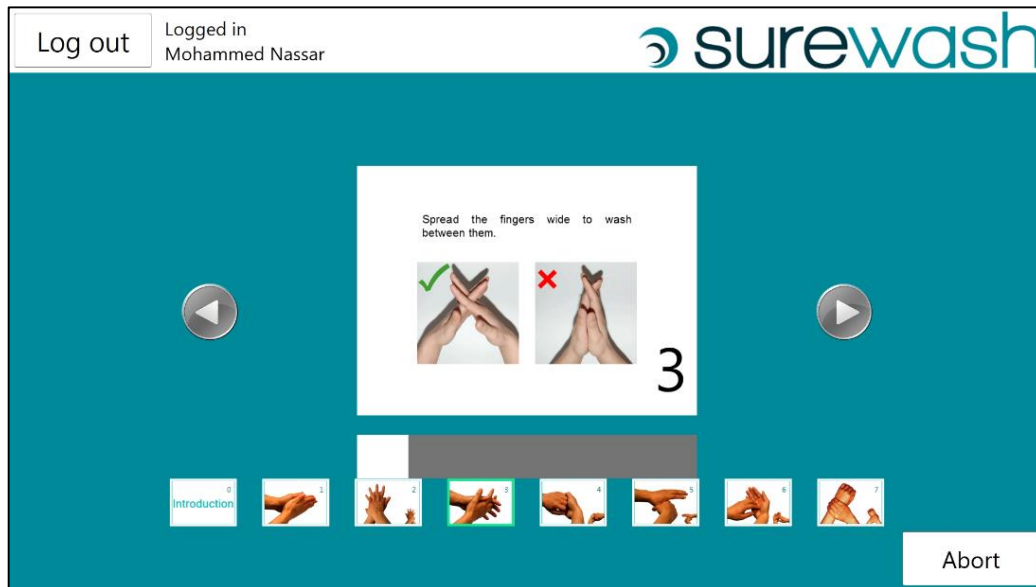

While performing a hand hygiene pose, the hands must be placed in the box overlaid on the video. The box turns green when the hands are in the right place. Furthermore, the user must roll their sleeves up to the elbow. If the sleeves are visible in the video, the system will recognise this and will give a warning of "Roll up your sleeves". A large watch or bracelet may also cause this warning.

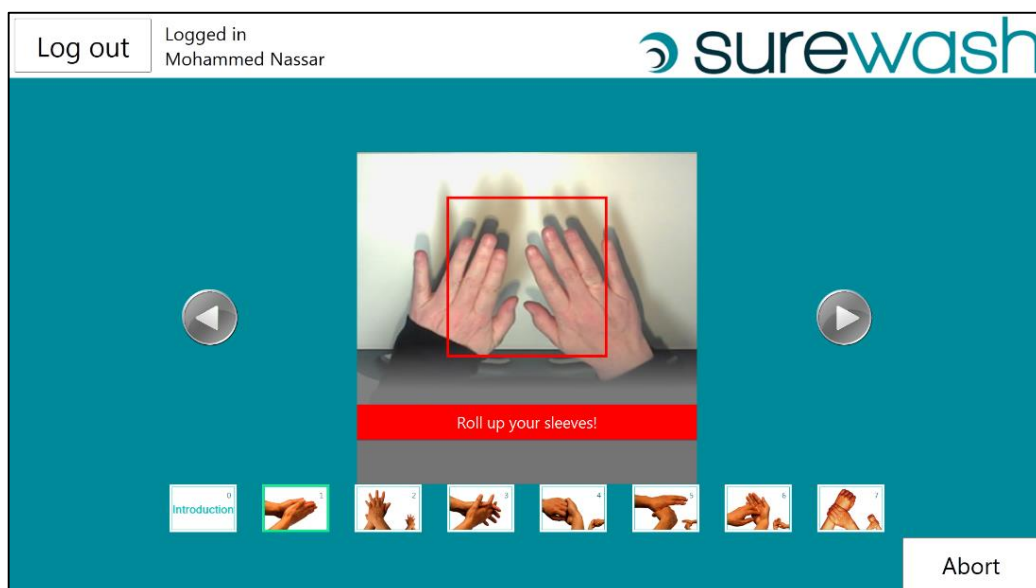


3.4 ASSESSMENT MODE

In Assessment mode a user has to perform all seven hand hygiene poses one after the other in sequence. As the user passes each pose the system moves automatically to the next one.


Before the user can start hand hygiene assessment, the hands need to be presented within the silhouette outlined on the video screen. This image of the hands is saved and will go into the report so that administrators can check if people are wearing accessories such as jewellery, false nails, bracelets, etc. The user may need to lift their hands up or down to fit the silhouette and turn the outline to green.



The user must follow the poses in the correct sequence, the system moves on to the next pose after a certain length of time.

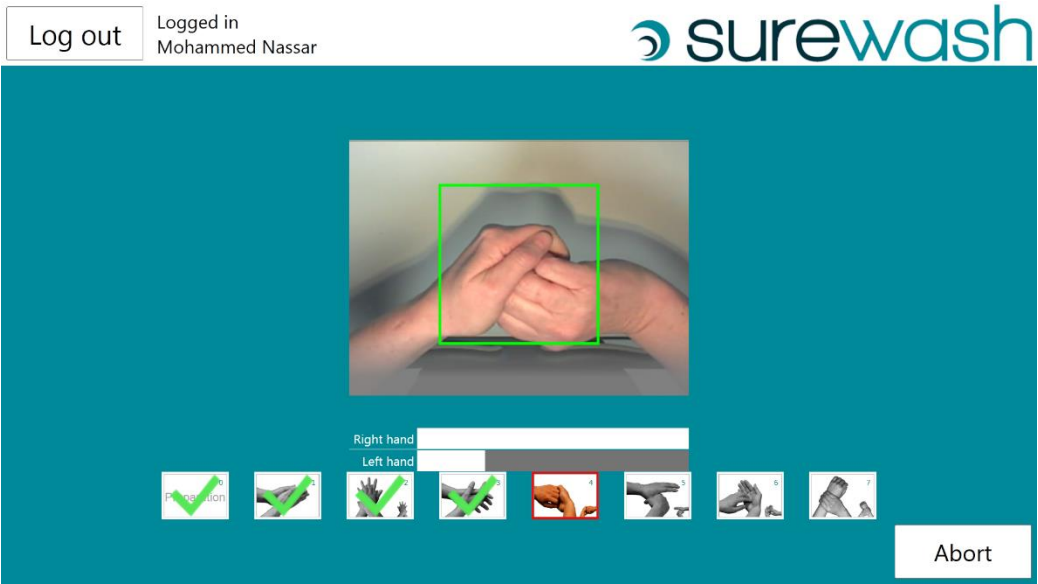

After a user has finished an assessment, a score will be presented. Additionally, each pose image will show a green tick if the user passed the particular pose, and a red X otherwise. The user can choose to return to the main menu, retry the assessment or enter the Training mode. (Within Training they can select any of the seven poses by pressing the pose images, as described above.)

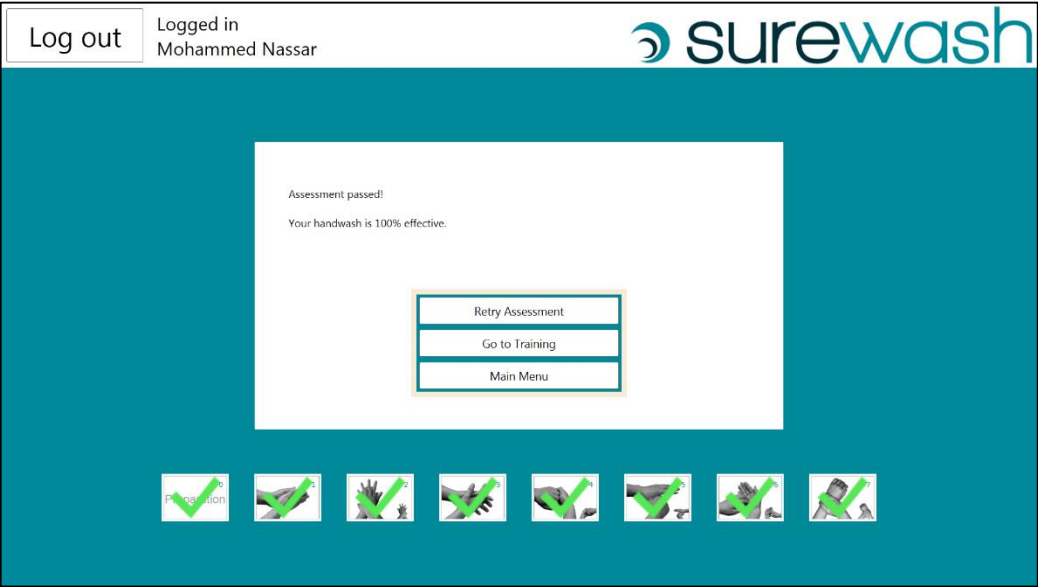

Log out

Logged in
Mohammed Nassar



Assessment not passed!
Your handwash is 75% effective.

- Retry Assessment
- Go to Training
- Main Menu




4 SUREWASH ADMINISTRATION

4.1 LOGGING IN AS AN ADMINISTRATOR

The Administrator screen of the SureWash Training Unit provides access to extra features of the system. In order to access the Administrator screen, press the keyboard icon on the main login screen. Type the username 'admin123'. (Ensure that the username is entered in lowercase; press the CAPS button on the on-screen keyboard if necessary.) You will be prompted to enter your password. If this is the first time that the Administrator screen has been accessed, you should enter the default password ('admin123'). You must change the password before proceeding.

When logged in as an administrator, the following screen appears.

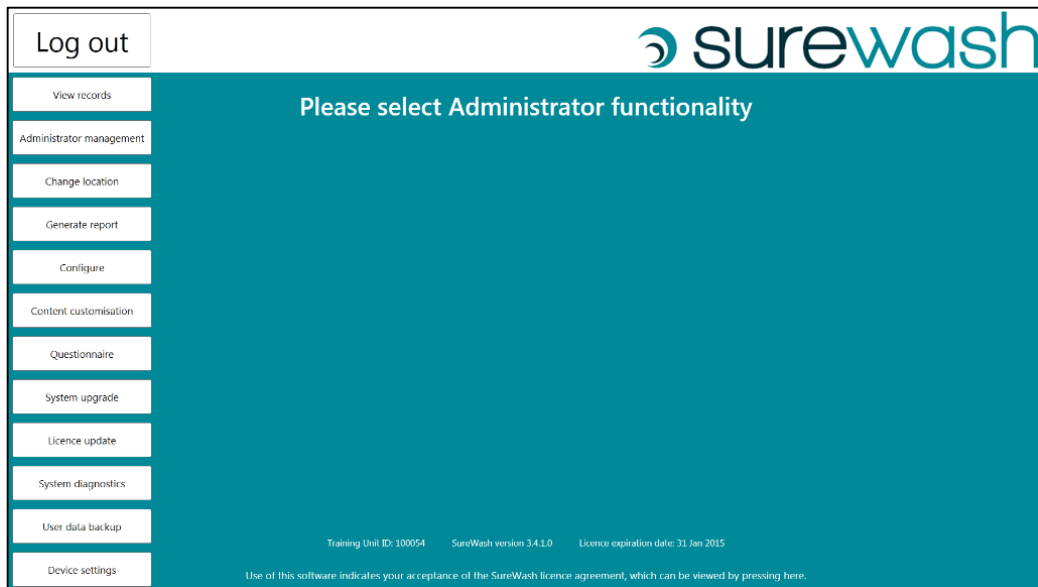

The different administrator features are presented on the left-hand side of the screen:

- View records (section 4.2)
- Administrator management (section 4.3)
- Change location (section 4.4)
- Generate report (section 4.5)
- Configuration (section 4.7)
- Content customisation (section 4.8)
- Questionnaire (section 3.9)
- System upgrade (section 4.9)
- Licence update (section 4.11)
- System diagnostics (section 4.12)
- User data backup (section 4.13)
- Device Setting (section 3.14)

4.2 VIEW RECORDS

The 'view records' functionality allows you to have an overview of the usage of the system, presented as a spreadsheet. The columns of the table are:

- User: The ID or name of the individual
- # Trainings: The number of training sessions performed by this user
- # Assessments: The number of assessments performed by this user
- Passed: A 'Yes' indicates that this user passed at least one assessment, while a 'No' means that this user did not pass an assessment yet.

The user with the ID '000000000000' is the anonymous user: whenever somebody logs in using the 'Anonymous' button instead of using a fingerprint or username, the records are saved with this ID.


The screenshot shows the 'View records' page in the SureWash administration interface. The page has a teal header with the 'surewash' logo and a 'Log out' button. A sidebar on the left contains navigation links: 'View records', 'Administrator management', 'Change location', 'Generate report', 'Configure', 'Content customisation', 'Questionnaire', 'System upgrade', 'Licence update', 'System diagnostics', 'User data backup', and 'Device settings'. The main content area is titled 'View records' and contains a table with the following data:

User	# Trainings	# Assessments	Passed
000000000000	2	1	No
user20141211151825	5	8	Yes

At the bottom of the page, there is a footer with the following text: 'Training Unit ID: 100054 SureWash version 3.4.1.0 Licence expiration date: 31 Jan 2015'. Below this, a small notice states: 'Use of this software indicates your acceptance of the SureWash licence agreement, which can be viewed by pressing here.'

4.3 ADMINISTRATOR MANAGEMENT

The administrator management feature allows you to change the current administrator's password and to register an administrator user. Press 'Administrator management' and the following screen will appear.

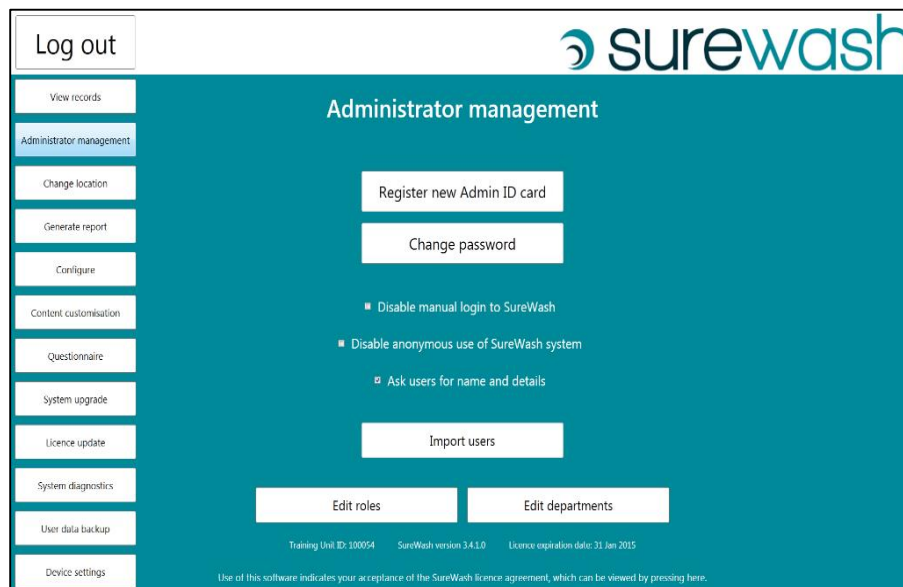

Manual login can be enabled or disabled on this screen.

Anonymous use can be enabled or disabled on this screen.


Roles and Departments can be created or disabled on this screen.

Import Users: If you have a high number of users, it is recommended that user profiles i.e. unique user id, names, role, department are preloaded. This will provide quality reporting and simplify the initial logging in process as staff will not have to enter their own profile data. User profile data can be uploaded using a SUREWASH USB key or the SureWash technical team can upload remotely if WIFI is available locally. SureWash technical department can provide a template which can be populated for upload.

When a finger print user is in use, data can be preloaded against a unique identifier such as a staff number or a card number and have this data verified against the finger print when they first use the system.

Change password


Press 'Change password' to enter a new password for the current administrator. The following screen will appear.

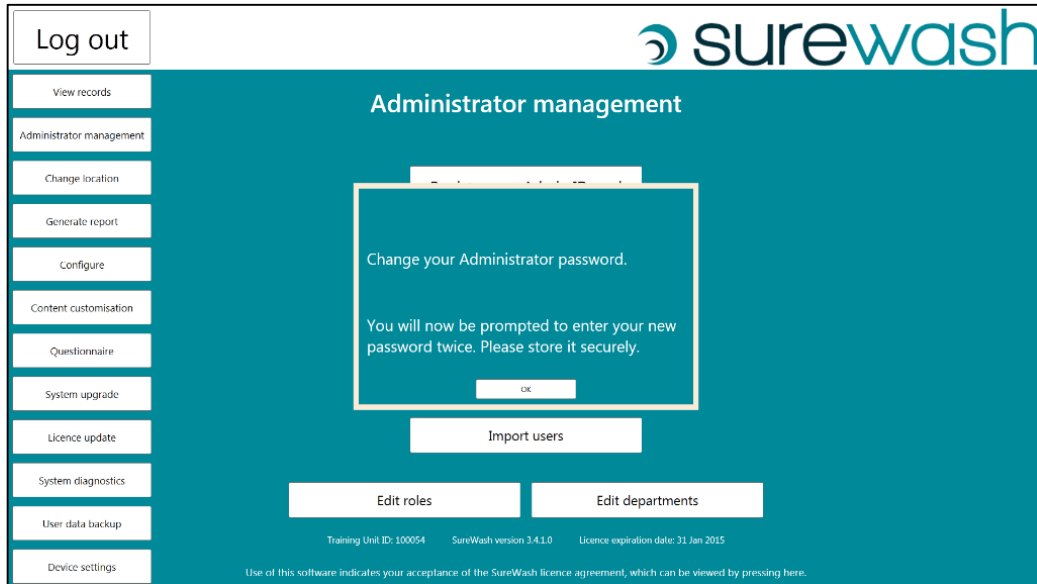

You will be prompted to enter your new password twice.

4.3.1 REGISTER NEW ADMINISTRATOR ID CARD

If you press 'Register new Admin ID card' you will see the following message box.

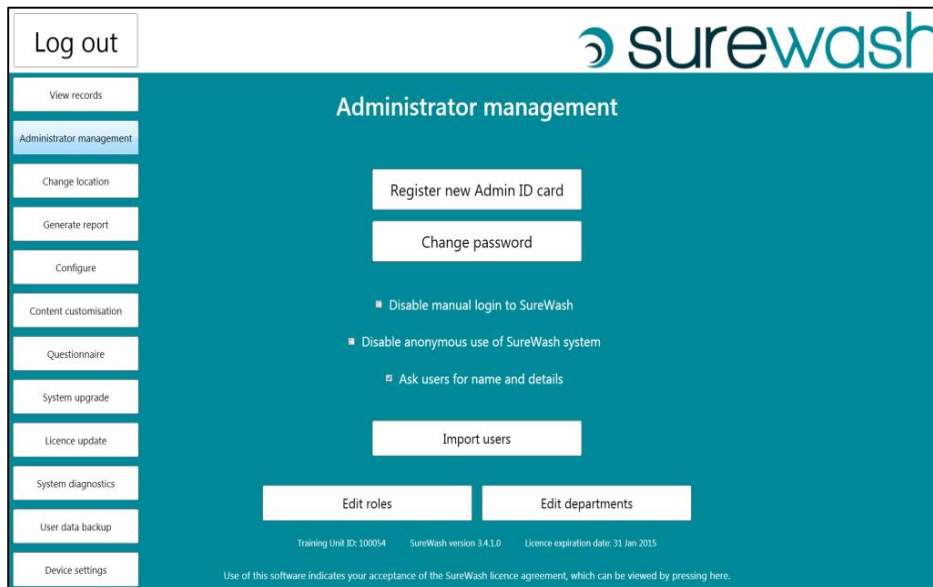

If your SureWash Training Unit has an ID card reader, simply swipe the ID card that you want to register as an administrator card. The message box will disappear; the card is now registered as an administrator card. The password for the card will initially be set to the default password ('admin123').

4.4 CHANGE LOCATION

Pressing the ‘Change location’ button allows you to specify the location where the machine is currently being used. A screen similar to the one shown at system start-up (section 3.1) will appear.


The location specified could be, for example, the name of a hospital department. You may select an existing location from the list or you may enter a new location.

SureWash recommends that the “Prevent changes to locations at system start up” button is ticked as the default setting.

4.5 GENERATE REPORT

The SureWash Training Unit allows detailed reports to be created that show each user's performance during hand hygiene assessment. The reports are broken down by department and can be limited to a specific time period.

First, select 'Generate report' from the left-hand side of the administrator interface. The

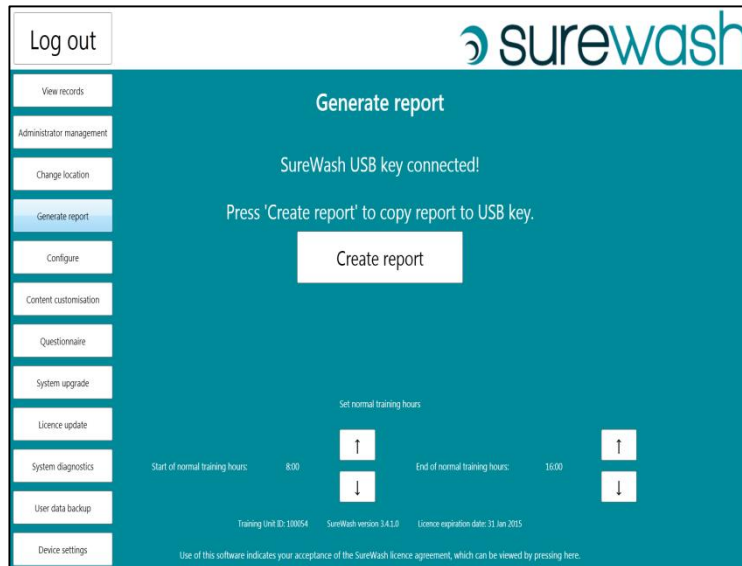

following screen will appear. The 'Create report' button cannot be pressed unless the licence for the system is valid (see section 4.11) and until the SureWash USB key is connected to the system. After plugging in the SureWash USB key, the 'Create report' button can be pressed.

Pressing the 'Create report' button brings up the following screen.



Choosing the default settings and pressing 'Generate report' will create a single report that covers the entire time period that the system has been in use. The report will also include all locations stored in the system (section 4.4).

After creating a report, the report may be copied onto the attached SureWash USB key along with an Excel file (CSV format) containing all the records of the system.



SureWash encourages the use of an eight digit alpha numeric password i.e. Abc12345 which will be required to open the report on another computer. The information within the report is encrypted so it cannot be read by unauthorised persons.




4.6 REPORTS

The reports generated by SureWash begin with general information about all of the sessions in the reporting period.



This shows the dates which the report covers (defaults to previous six months but can be customised), the usage of the unit and summary by role, time used and overall hospital/department pass rate.

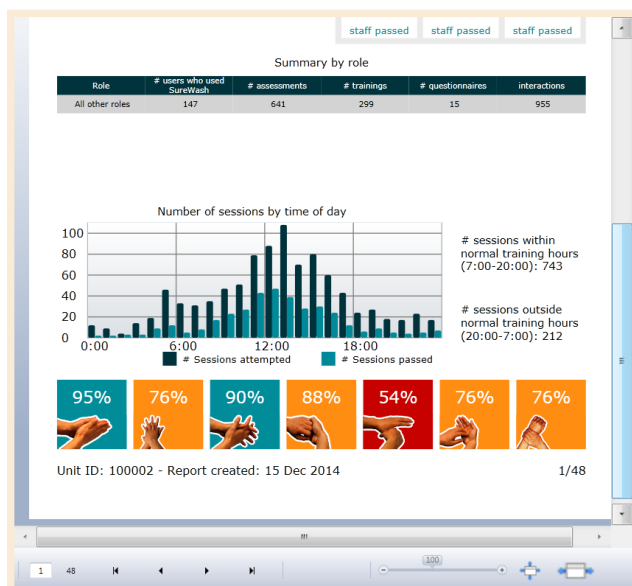

4.6.1 USER REPORT

The screenshot shows the SureWash user report interface. At the top, the SureWash logo is visible. Below it, the text '107 *' indicates the number of users. The main part of the interface is a table with the following columns: User ID, Name, Outcome, Last session, # Sessions, and seven performance metrics (represented by small orange icons). The table lists 25 users with their respective scores and outcomes. At the bottom, there is a navigation bar with a page number '7' and a search bar containing '100'.

User ID	Name	Outcome	Last session	# Sessions	1	2	3	4	5	6	7
21071	AUSTIN FERGUSON	passed	13 Mar 2014	4	75%	75%	75%	75%	75%	75%	75%
0809	LORNA HIGGINS	passed	12 Mar 2014	2	100%	100%	100%	100%	100%	100%	100%
3985	SHARON HORNE	passed	12 Mar 2014	2	100%	100%	100%	100%	100%	100%	100%
8493	JOANNE MCDOUGALL	passed	13 Mar 2014	7	85%	42%	71%	85%	28%	85%	28%
5259	HELGA MORROW	passed	12 Mar 2014	2	100%	100%	100%	100%	100%	100%	100%
3091	BARRY O DOWD	passed	12 Mar 2014	2	100%	100%	100%	100%	100%	100%	100%
2358	LYNN PAXTON	passed	12 Mar 2014	1	100%	100%	100%	100%	100%	100%	100%
6168	RUDI REIMANN	passed	12 Mar 2014	3	66%	66%	33%	66%	66%	66%	66%
2762	AMANDA SORLEY	passed	12 Mar 2014	2	100%	100%	100%	100%	100%	100%	100%
DIANNE	DIANNE COLEMAN	failed	11 Mar 2014	3	66%	0%	100%	33%	0%	0%	0%
1875	CATHERINE FERGUSON	failed	11 Mar 2014	6	50%	16%	16%	16%	0%	0%	0%
1709	CHRIS FINDLAY	failed	7 Mar 2014	1	100%	100%	100%	100%	100%	0%	100%
1071	CLAIRE HENDERSON	failed	8 Mar 2014	3	66%	66%	33%	33%	0%	33%	33%
AMANDA	AMANDA MCCLELLAND	failed	9 Mar 2014	5	80%	40%	20%	40%	0%	40%	40%
9955	RAVI RAVINDRAN	failed	7 Mar 2014	2	50%	50%	50%	0%	50%	0%	50%
1089	EMMA RUTHERFORD	failed	12 Mar 2014	6	83%	83%	83%	83%	33%	50%	66%
ELAINE	ELAINE SIBBALD	failed	7 Mar 2014	1	100%	0%	100%	100%	0%	100%	0%
ELAINE	ELAINE SIBBALD	failed	10 Mar 2014	1	100%	0%	0%	0%	0%	0%	0%
2107	BOB WHITE	failed	10 Mar 2014	5	100%	100%	80%	100%	20%	60%	80%
6043	ROBERT ADAMSON	not assessed		0	0%	0%	0%	0%	0%	0%	0%
6131	MARY EVANS	not assessed		0	0%	0%	0%	0%	0%	0%	0%
2805	LOUISE GRAHAM	not assessed		0	0%	0%	0%	0%	0%	0%	0%
7545	RENOS IOANNOU	not assessed		0	0%	0%	0%	0%	0%	0%	0%

Users are listed by User ID and report shows if they have passed (achieved 100% in the assessment in the time period of the report), failed (not attempted or achieved 100% in the assessment during the time period of the report) or not assessed (have not used the machine in the time period of the report).

4.7 CONFIGURATION

Pressing 'Configure' allows you to change the time users must spend on a particular pose before they pass. The parameters can be adjusted for each pose individually using the '+' and '-' buttons. After adjusting the parameters, they must be saved by pressing the 'Save all changes' button. The parameters can also be reset to the default values by pressing 'Reset all poses to default value 40'.

The screenshot displays the 'Pose configuration' interface. On the left is a sidebar with a 'Log out' button and a list of menu items: 'View records', 'Administrator management', 'Change location', 'Generate report', 'Configure', 'Content customisation', 'Questionnaire', 'System upgrade', 'Licence update', 'System diagnostics', 'User data backup', and 'Device settings'. The main content area has a teal header with the 'surewash' logo and the title 'Pose configuration'. Below the header, it says 'Adjust the value for a pose to let users spend more or less time at this pose'. There is a table with 8 rows, each representing a pose. Each row has a small image of the pose, a numerical value (all are 40), and two buttons: a plus sign (+) and a minus sign (-). To the right of the table are two buttons: 'Reset all poses to default value 40' and 'Save all changes'. Below the table, there are two checkboxes: 'Require both left- and right-handed versions of pose 4' (checked) and 'Activate pose 7' (checked). At the bottom, there is a footer with the text: 'Training Unit ID: 100054 SureWash version 3.4.1.0 Licence expiration date: 31 Jan 2015' and a link: 'Use of this software indicates your acceptance of the SureWash licence agreement, which can be viewed by pressing here.'

Greater than 40: increased difficulty


Less than 40: easier to pass

4.8 CONTENT CUSTOMISATION

The 'Content customisation' feature allows an administrator to replace the first four slides in the Training mode of the SureWash system (Why, When, How, etc. – see section 3.3) with their own e-learning content.

Contact the SureWash Support Team to convert your custom slides to a format suitable for use with the system. Copy the file you receive to the SureWash USB key and plug the USB key into the system.

The 'Customise content' button cannot be pressed unless the licence for the system is valid (see section 4.11). When the button is pressed, a file selection dialog appears.

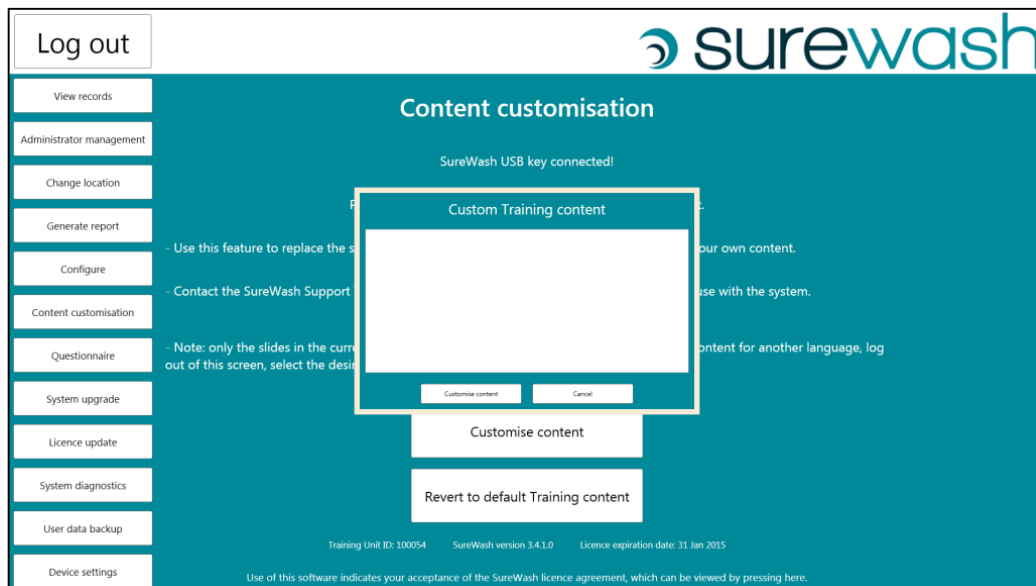


Choose the correct custom content file from the list and press the 'Customise content' button. If your SureWash Training Unit offers multiple interface languages, ensure that you are logged in to the Administrator interface in the correct language, i.e., that language for which you wish to customise the Training content.


By pressing the 'Revert to default Training content', any custom content for the current language will be removed from the Training screens.

4.9 QUESTIONNAIRE

SureWash also comes with multiple choice type questionnaire functionality.

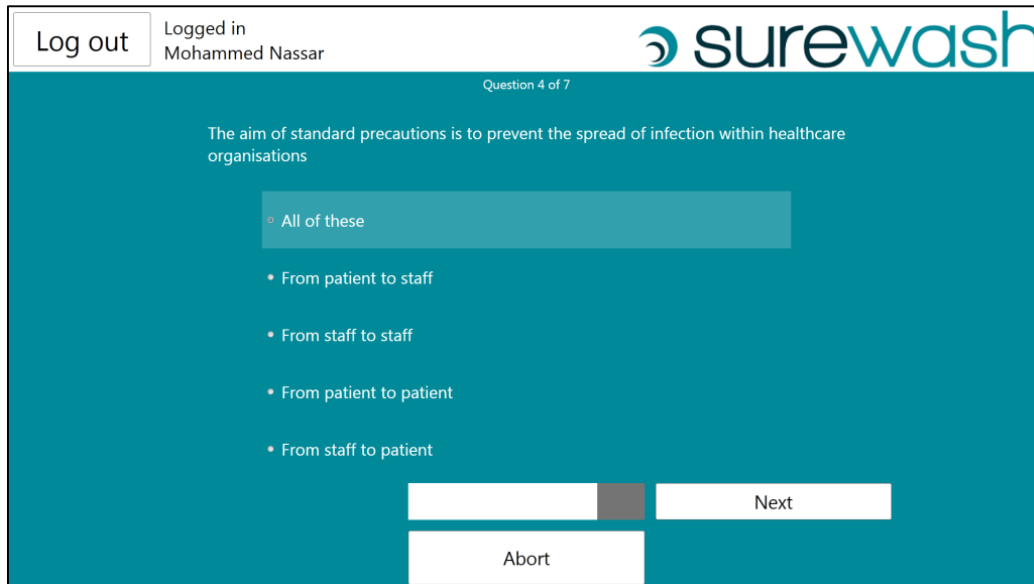


SureWash will provide a standard questionnaire, but it is also possible to have many quizzes on a SureWash machine, which can be activated as required through the Questionnaire configuration screen.

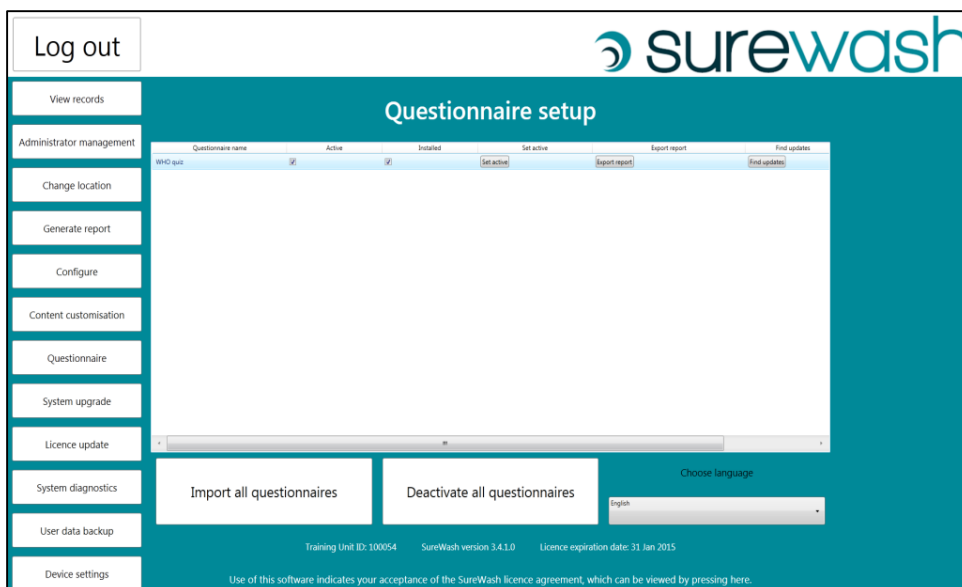

4.10 SYSTEM UPGRADE

The 'System upgrade' feature allows an administrator to upgrade the SureWash software on the Training Unit. Such upgrades will be notified to you from time to time by the SureWash Support Team. Once you have downloaded the upgrade file from the internet, copy it to the SureWash USB key and connect the USB key to the system.

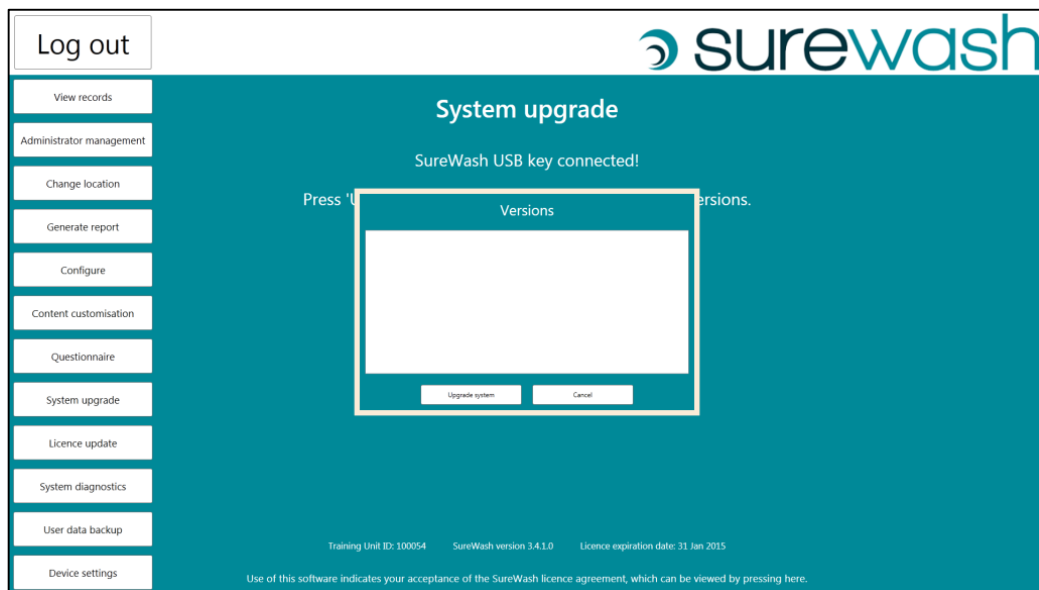


Pressing the 'Upgrade system' button brings up a file selection dialog.

Choose the correct SureWash upgrade file from the list and press the 'Upgrade system' button. Once the new version of the software has been copied to the system, you will be prompted to restart the system. The upgrade will not take effect until the restart is performed.

Note: no user data will be deleted by upgrading the SureWash software.

4.1.1 LICENCE UPDATE

A valid licence is required to use certain features of the SureWash software. If the licence for the Training Unit has expired, or is about to expire, contact the SureWash Support Team.

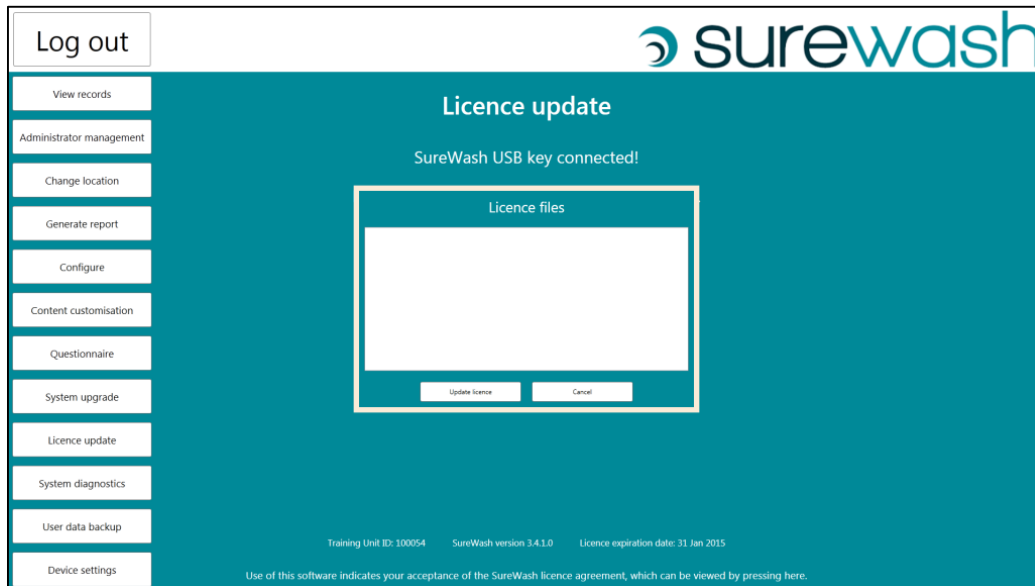


Pressing the 'Update licence' button brings up a file selection dialog.


Choose the correct SureWash licence file from the list and press the 'Update licence' button. If the licence is valid, all features of the SureWash software will be enabled.

4.12 SYSTEM DIAGNOSTICS

The 'System diagnostics' feature gathers diagnostic information about the SureWash Training Unit that can prove useful for troubleshooting. Connect the SureWash USB key to the system and press the 'Gather diagnostic information' button.

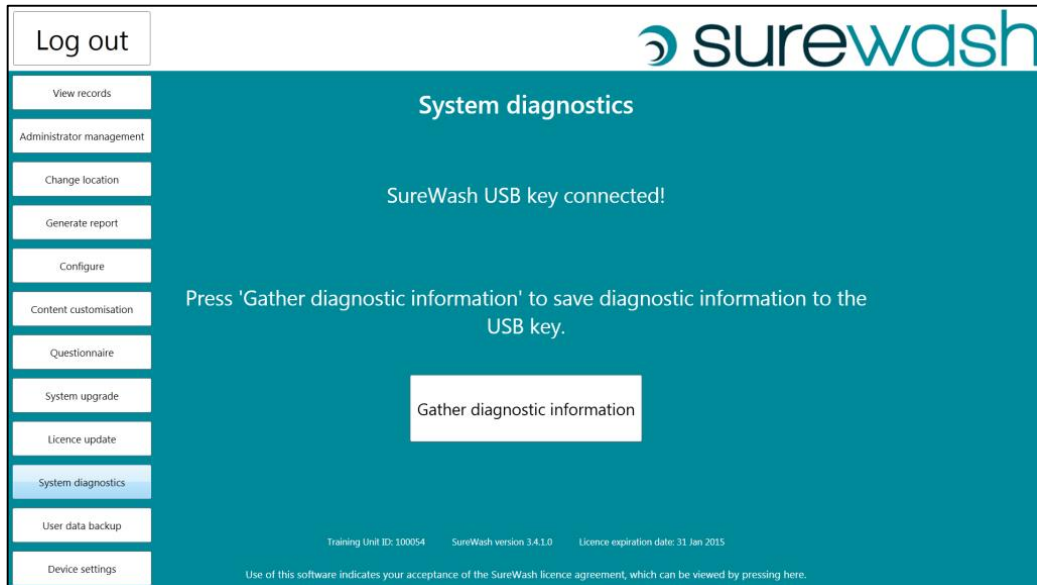


A file containing diagnostic information called 'SureWashDiagnostics.zip' will be copied to the USB key. Please email this file to the SureWash Support Team with a description of the problem you are encountering.

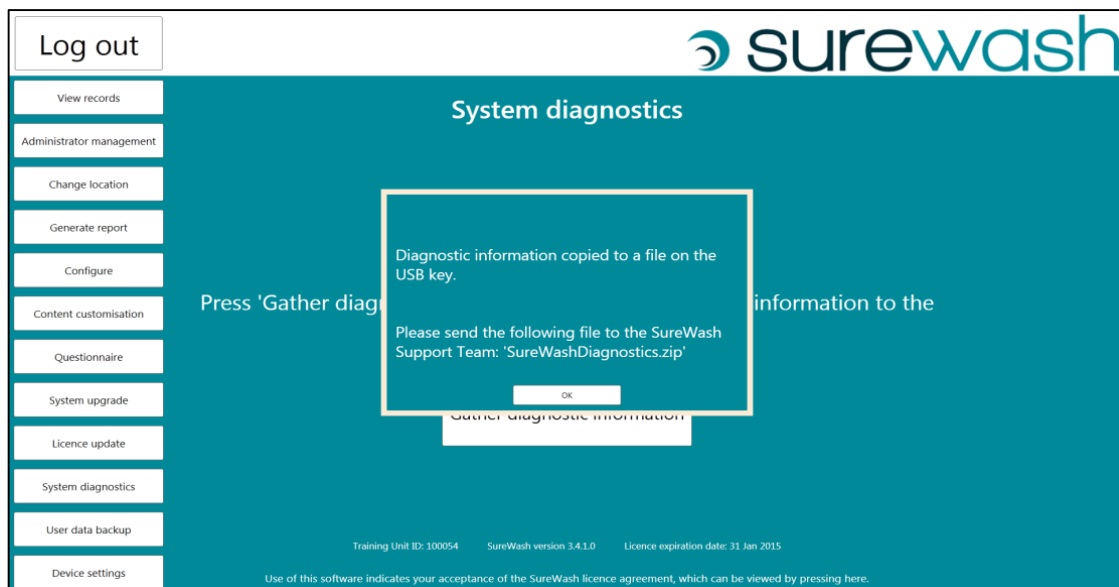

4.13 USER DATA BACKUP

The 'User data backup' feature allows an administrator to copy the user data from the SureWash Training Unit to the USB key. This procedure should be carried out on a regular basis to protect against data loss.

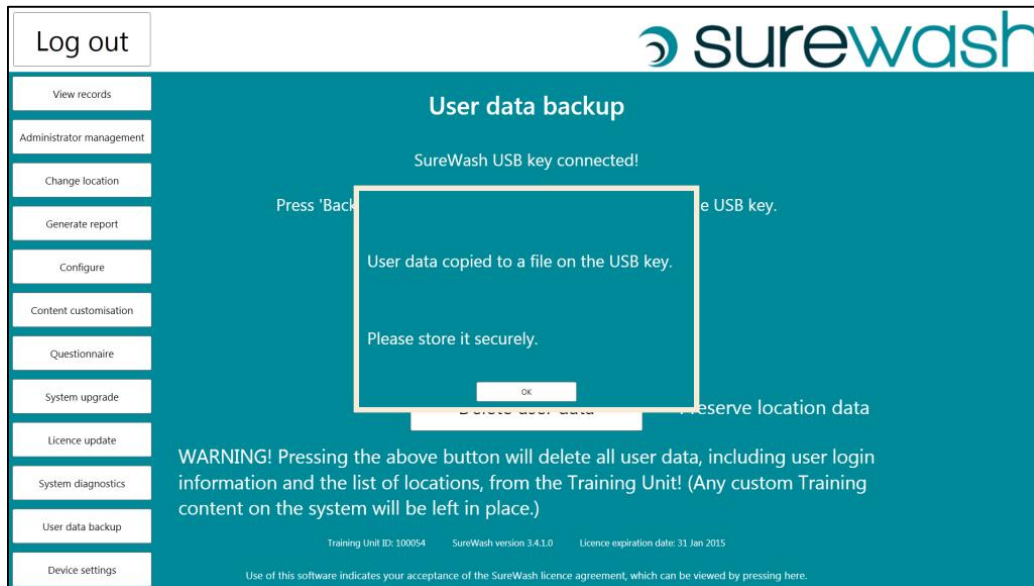


Connect the SureWash USB key to the system and press the 'Back up user data' button. The user data will be copied to the USB key. Please store the data securely.

4.13.1 DELETE USER DATA

Occasionally, it may be desirable to delete all user data from the system. Pressing the 'Delete user data' button will remove all user data from the system, except for administrator login details and any custom Training content that is present (section 4.8).

WARNING! Once the user data has been deleted, it cannot be restored to the system!

4.14 DEVICE SETTINGS

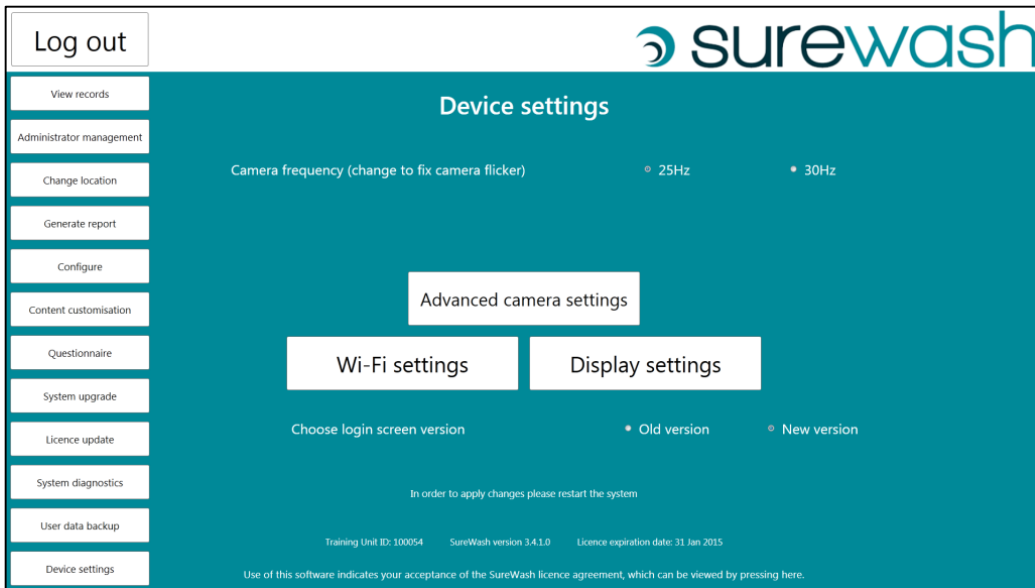

4.14.1 CAMERA FREQUENCY

The camera frequency must be set according to the electricity supply frequency of the country in which the SureWash system is being used:

- Saudi Arabia: 30Hz
- United Arab Emirates: 25Hz
- Qatar: 25Hz
- Bahrain: 25Hz
- Europe, Ireland, UK: 25Hz
- North America: 30Hz

It is necessary to restart the system if you change the camera frequency.

4.14.2 ADVANCED CAMERA SETTINGS

It is only necessary to change these settings if no image is coming from the camera, in which case you should contact SureWash.

4.14.3 WI-FI SETTINGS

This allows the SureWash unit to connect to a Wi-Fi network, to facilitate updates to the software and troubleshooting. Pressing the button brings up a Wi-Fi connections screen,

from where it is possible to connect to any available Wi-Fi network. (A password may be required. Please contact your IP department in such cases.)

Please ensure that you do not accidentally close any other programs when working with the Wi-Fi connections screen.

4.14.4 DISPLAY SETTINGS

It is possible to connect a second screen or a projector to the SureWash system (for group demonstrations, etc.). A VGA connector is available on the back of the unit. Once the screen has been connected, you may have to press *display settings* and configure the screens:

- Under "Multiple displays" choose "Duplicate these displays"
- Press "Apply" and check that the same image appears on both screens

Please ensure that you do not accidentally close any other programs when working with the display settings screen. This settings change must be made every time the SureWash system is restarted.

4.14.5 LOGIN SCREEN VERSION

It is possible to enable an old version of the login screen (without any animations showing how to use the ID card reader/fingerprint reader, but we recommend that the new version be used in most cases.

5 SUPPORT AND TROUBLESHOOTING

The SureWash Training Unit is extremely reliable. However, if the unit becomes unresponsive, the following troubleshooting steps should be tried:

1. Power down the unit by pressing and holding the ON/OFF button. Once the unit has been shut down for 10 seconds, restart the unit by pressing the ON/OFF button. The system should restart successfully.



2. If the screen is dark/blank please ensure that the screen has not been powered off via the screen power button by mistake.



Location of screen ON/OFF button. It is not necessary to turn off the screen separately and it should not be used in normal operation

3. If these steps fail please follow the instructions in section 4.12 (“System diagnostics”) and send an email to support@surewash.com with a description of the problem. We will be in contact with you within one working day.

6 LEGAL NOTICES

This user manual and all supporting SureWash documentation is copyright © GLANTA Ltd, 2011–2015, all rights reserved.

The SureWash Training Unit software is copyright © GLANTA Ltd, 2010–2015, all rights reserved. The device is manufactured under the protection of patents EP2015665 and US 8090155. Further patents pending.



Declaration of Conformity

We:

**Glanta Ltd
The Tower
Trinity Enterprise Center
Pearse St
Dublin 2**

declare under our sole responsibility that the product,

Surewash Training

to which this documentation relates, is in conformity with the provisions of the following directives:

**2001/95/EC General Product Safety
2006/95/EC Low Voltage Directive (LVD)
2004/108/EC Electromagnetic Compatibility Directive (EMC)**

The product has been tested to the following standards,

**EN55022:2010, EN55024:2010
EN 61000-3-2:2006+A1+A2:2009
EN 61000-3-3:2008**

The technical file is maintained at:

**Glanta Ltd
The Tower
Trinity Enterprise Center,
Pearse St., Dublin2 , Ireland**

Date of Issue: 15th Dec 2011

Place of Issue: Dublin, Ireland

Name: **Seán Bay**

Title: **CEO**