

General information regarding the safe use of our products

Introduction to using our products

- Please note that people with special needs, reading difficulties and impaired vision must ask a parent, career or technical advisor to read out or explain this technical documentation so that safe usage of our products can be guaranteed.
- We would like to point out that only those spare parts and accessories from our company may be used and installed so as not to endanger the safety of the users. In individual cases, our products may be combined with products by other manufacturers upon clarification by your specialist retailer or us, the manufacturer.
- Settings and adjustments to the product or accessories may only be made by people who have been given the necessary instructions by a medical product advisor. Please ensure that none of the user's extremities are in the respective area when making adjustments to avoid the risk of injury.
- The respective product should be reduced to its smallest dimensions for transportation. For example, reduce the seat, the armrests, the back etc. to the smallest dimensions. Remove parts of the product that can be easily removed, e.g. the wheels of the TODD standing frame, the handlebar of the MOMO tricycle and therapy bike etc. Fold up the foot rest on products with a foot rest. Please see the User Manual belonging to the respective product for more detailed information.



General information about cleaning and hygiene

Cleaning guidelines by material

- **Wood:** Wooden parts should be cleaned with a damp cloth. When using detergents, please consult the manufacturer's instructions regarding detergent concentration. Caustic household detergents or corrosive substances are not suitable. Never leave natural wood in a damp or wet condition over long periods, as it will take on water and can change in shape (swelling). As such, only ever dampen to wipe and clean (e.g. with wrung-out cloths). Test a small, non-visible area for compatibility.
- **Powder-coated parts:** Aluminium parts should be cleaned with a sponge or damp cloth using normal antibacterial household detergents.
- **Stainless steel:** Stainless steel parts should be cleaned using a sponge or damp cloth with a mild soap. Bleach must not be used.
- **Covers / Fabric:** Covers and fabric should be washed at 30° C in the washing machine using normal detergent. Do not tumble dry.
- **Plastic:** Plastics should be cleaned with a sponge or damp cloth.
- **PU (polyurethane):** PU parts should only be cleaned using a sponge or damp cloth with a mild soap. Bleach must not be used.

Hygiene information for our products

- **What?** All products from our range
- **Who?** Retail service staff and other authorised institutions
- **When?** CLEANING: As required (maintenance /repairs /general care)
CLEANING + DISINFECTION: Prior to passing on to other users!
Damaged frames or components must first be removed or serviced.
- **With what?** We recommend wearing protective gloves (e.g. made from butyl rubber)! Avoid the penetration of liquids in the Bowden cables, frame tubes etc. Clean visible dirt using a wet sponge cloth and a little washing-up liquid if required. Do not use any corrosive or auxiliary agents! Disinfect surfaces using non-alcohol-based disinfection solution "mikrozid® sensitive liquid" as a spray disinfectant or a sponge from Schülke & Mayr GmbH. Please note the solution instructions if using other solutions!
- **How?** To CLEAN, wipe all accessible surfaces with a damp sponge cloth and a fat-dissolving washing-up liquid if required. Then dry the surfaces with a cotton or disposable towel. Hard-to-access places can also be dried using a hairdryer on its lowest setting.
To DISINFECT apply the "mikrozid® sensitive liquid" undiluted on the surfaces and leave for 1 minute (leave for 30 minutes if there is suspicion of contamination with the Noro virus). Ensure a thorough application. If necessary, remove excess solution with an untinted disposable towel.
- **Questions?** Schuchmann GmbH & Co. KG
Dütestraße 3 · 49205 Hasbergen
Tel. +49 (0)5405 909-0 · Fax +49 (0) 5405 909-109
www.schuchmann-reha.com · info@schuchmann-reha.de