

HKS-08 user manual




Warning

Dear user:

Thanks for using our company's products, in order to fully make use of the products, we sincerely suggests follow:

- ◆◆◆ Please read the instruction in detail and keep it well for further reading and consulting.
- ◆◆◆ Please follow the instruction to fit and operate the machine correctly.
- ◆◆◆ Please do not remove and change any accessories of the machine.
- ◆◆◆ It forbids others to open and remove the mainframe except the authorized people in the company.
- ◆◆◆ It forbids others to change and maintain the machine except the professional people.
- ◆◆◆ Please contact us if there is any trouble during operation.

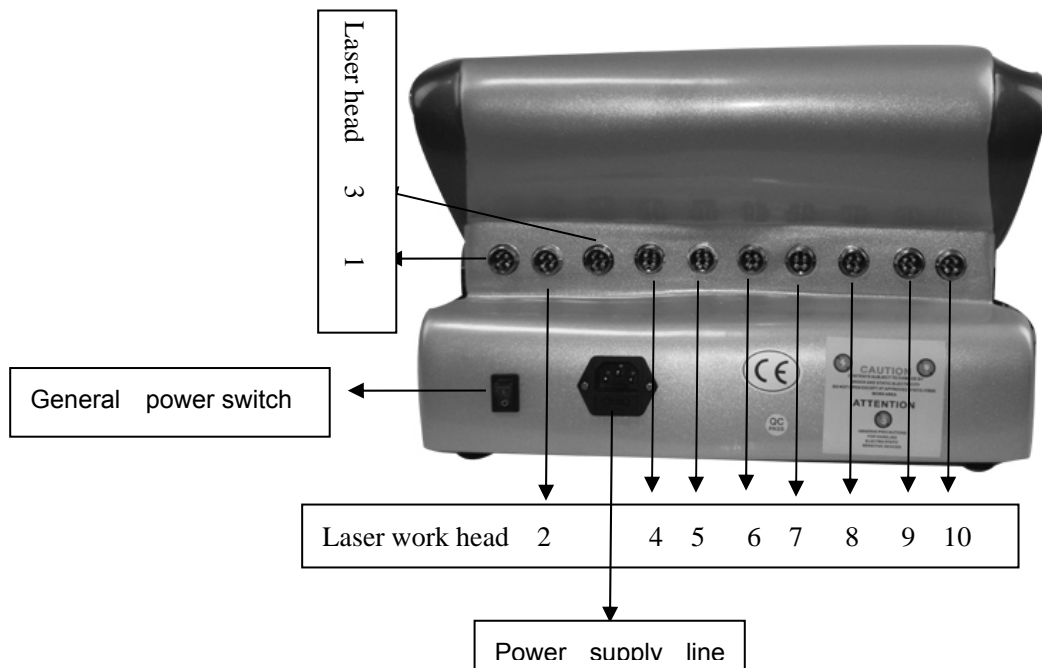

Laser head×8 *10



glasses



bandage for tied laser heads 5 pcs

Basal Operation

- 1.Put the accessories well and connect them to the host.
- 2.Insert power supply line to machine, turn on the General power
3. As shows, input default "SET" enter interface .
- 4.Click any place on screen to enter function interface as shows.
- 5.you also can set the language English or Spanish.



Laser Function

1.Operation step of Laser 8X

- (1)Clean the skin, and according the treatment part to choose the suit band ties and tied them up.
- (2)Put the Laser 8X work heads into the band ties, the side with Led contact to skin directly.

2 lase×8r can tied on the arms,4 laser×8 can tied on the big legs 4 laser×8 can tied on the belly.

- (3)Set intensity of every Laser 8X in interface shows.the range is 1(weak)~3(strong).

Operation steps: When the cursor "V" on the location "channel 1 1HZ",click" ←"and" →"to set the intensity of laser, the range is 1HZ~1000HZ. OFF means that close channel 1.

Then move the cursor "V" to the "channel 2 1HZ", click" ←"and" →" to set the intensity of laser, the range is 1HZ~1000HZ. OFF means that close channel 2.

- (4)Setting work time .Click "Time" to enter interface . Click" ←"and" →" "one time to change one minute, click the number of time directly to change ten minutes every time.30 minutes of every treatment part.



- (5)click "||" "to start work.

- (6)The machine will pause automatically if the setting time is over. Beautician take the Laser 8X work head out and put them well, then loosen the band ties.

Normally,fat is more, the frequency is higher.

after four treatment,the frequency is lower.

After operation,ask client to drink more water.

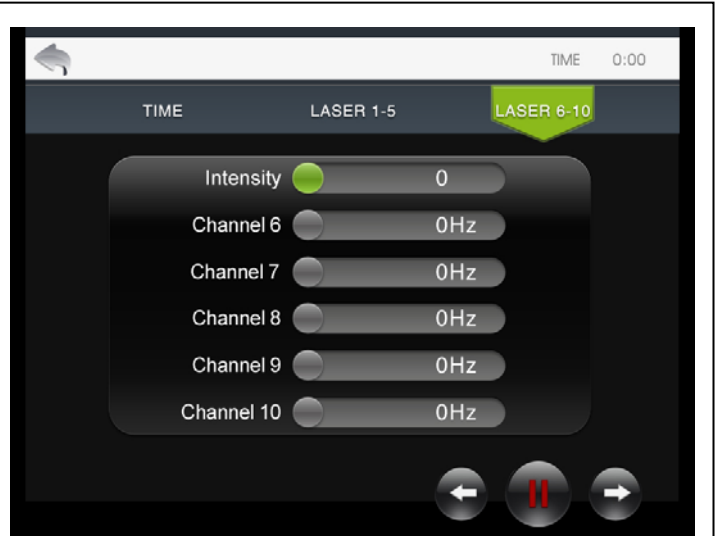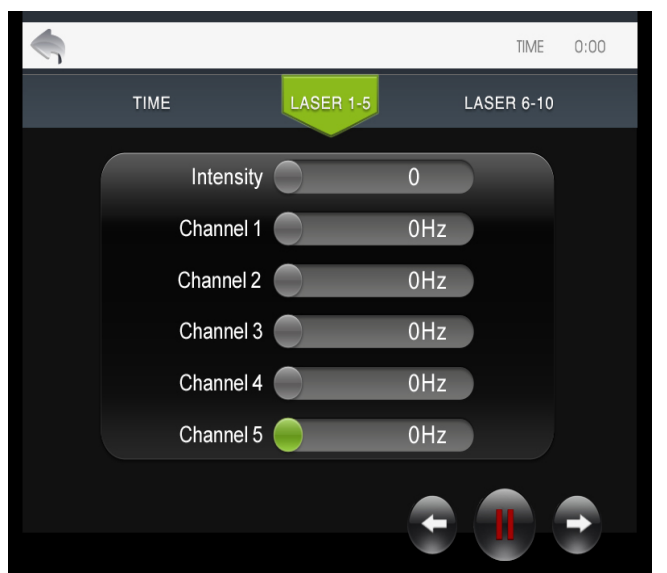

Forbidden group

1. People with heart disease or high blood pressure, or configured cardiac pacemaker.
2. Patients with acute inflammation, asthma, deep vein thrombosis, thyroncus, cancer.
3. People with hemorrhagic disease, trauma or who is bleeding.
4. Pregnant women
5. Medical Plastic parts, or parts with artificial in filling inside.
6. People with metal inside the body(include metallic tooth)
7. Patient with skin inflammation or with edema.
8. People with an abnormal immune system.
9. Numb or insensitive to heat.

Technical parameters

Input: AC110V/60Hz

Power \leq 150VA

Accessories list

| | |
|----------------------|--------|
| 1. Host | 1 pc |
| 2. Power supply line | 1 pc |
| 3. User manual | 1 pc |
| 4. Laser8X work head | 10 pcs |
| 5. glasses | 1 pc |
| 6. Bind ties | 5 pcs |
| 7. Accessory frame | 2 pcs |
| 8. Hexagonal screws | 8 pcs |
| 9. Hexagon wrench | 1 pc |