

Bringing
Innovation
to the Rehab
World!™



ISO 13485 Certified

TTET 200

Traction Table



User Manual
Model 6870

TABLE OF CONTENTS

TTET 200 Traction Table®

Foreword	1
Precautionary Instructions	2
Nomenclature	4
Specifications	5-6
Table Dimensions	5
Table Specifications	5
Electric Motor Specifications	6
Installation	7
Installing the Traction Stand	7
Operation	8-10
Using the Model 6870	8
Adjusting the Table Height	9
Relocating the Table	10
Troubleshooting	11
Maintenance	12
Cleaning	12
Factory Service	12
Warranty	13

Thank you for purchasing the Chattanooga Group TTET 200 Traction Table.

This manual contains general safety, operating, maintenance and care instructions for the owners and operators of the TTET 200 Traction Table Model 6870.

At the time of publication the information contained herein was current and up to date. However, due to continual technological improvements and Chattanooga Group's policy of continual improvement, Chattanooga Group reserves the right to make periodic changes and improvements to their equipment and documentation without any obligation on the part of Chattanooga Group.

Read, understand and follow the information contained in this manual.


Observe all applicable operating instructions and precautionary measures for use of traction tables.


Never transport a patient on a TTET 200 Traction Table. This table is not designed to support a patient during transport.


PRECAUTIONARY INSTRUCTIONS


TTET 200 Traction Table®


The precautionary instructions found in this section and throughout this manual are indicated by specific symbols. Understand these symbols and their definitions before operating this equipment. The definitions of these symbols are as follows:

 = **CAUTION-** Text with a “CAUTION” indicator will explain possible Safety infractions that could have the potential to cause minor to moderate injury or damage to equipment.

 = **WARNING-** Text with a “WARNING” indicator will explain possible Safety infractions that will potentially cause serious injury and equipment damage.

 = **DANGER-** Text with a “DANGER” indicator will explain possible Safety infractions that are imminently hazardous situations that would result in death or serious injury.

 = **EXPLOSION HAZARD-** Text with an “Explosion Hazard” indicator will explain possible safety infractions if this equipment is used in the presence of flammable anesthetics.

 = **LOCKING KNOB-** This symbol is located on the Locking Knob underneath the table, which serves to prevent the table from moving during therapy.

NOTE:- Throughout this manual “NOTE” may be found. These notes are helpful information to aid in the particular area or function being described.

CAUTION

- Read, understand and practice the precautionary and operating instructions found in this manual. Know the limitations and hazards associated with using any traction table. Observe any and all precautionary and operational decals placed on the unit.
- Do not exceed the table weight capacity of 400 lbs (182 kg).
- Securely lock the sliding section on the TTET 200 Traction Table before the patient is placed on, or removed, from the table’s surface.
- When removing the Power Supply Cord from the table’s motor, make sure you do not pull out the green ground wire that is attached to both the female end of the plug and the frame of the table.

WARNING

- Children must be supervised by a responsible adult when near this equipment.
- When making adjustments to the table sections, make certain the patient weight is removed before adjusting.
- Do not transport patients on the table. This table is not designed to support a patient during transport.
- Only use a Chattanooga Group Pedestal Mounted Traction units with the TTET 200 Traction Table

PRECAUTIONARY INSTRUCTIONS

TTET 200 Traction Table®

WARNING

- Move hands and feet away from the working mechanism of the table when making any and all adjustments to height or table sections.
- Make certain that the unit is electrically grounded by connecting only to a grounded electrical service receptacle conforming to the applicable national and local electrical codes.

This table should only be operated by trained personnel familiar with the precautionary measures and operational functions associated with the table being used.

DANGER



- Explosion hazard if used in the presence of flammable anesthetics. The warning symbol for this hazard is prominently displayed on the serial number decal.

NOMENCLATURE

TTET 200 Traction Table®





SPECIFICATIONS

TTET 200 Traction Table®

Table Dimensions

Specifications


Model.....	6870
Table Weight.....	140 lbs
Traction Stand Weight.....	30 lbs
Carton and Skid Weight.....	30 lbs
Total Shipping Weight.....	200 lbs

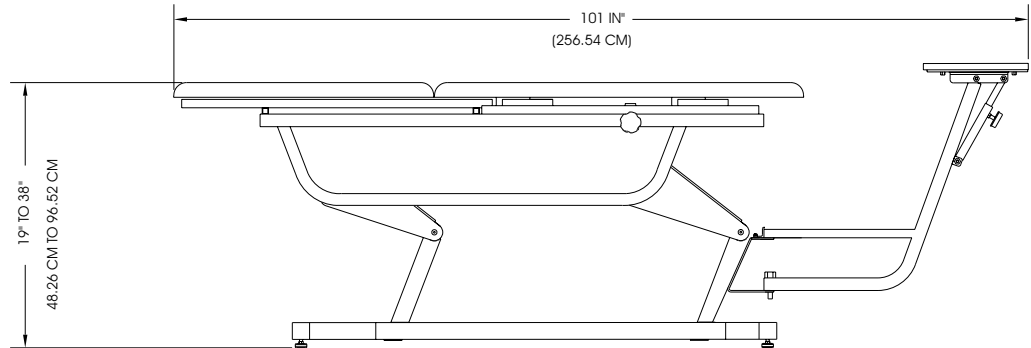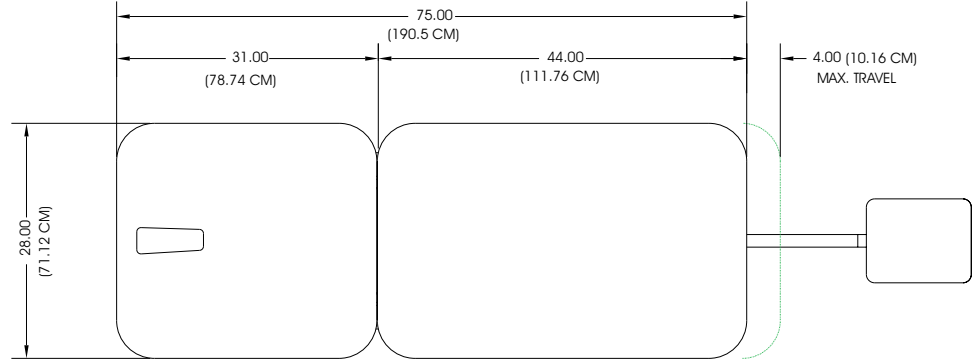

SPECIFICATIONS

TTET 200 Traction Table®

Electric Motor

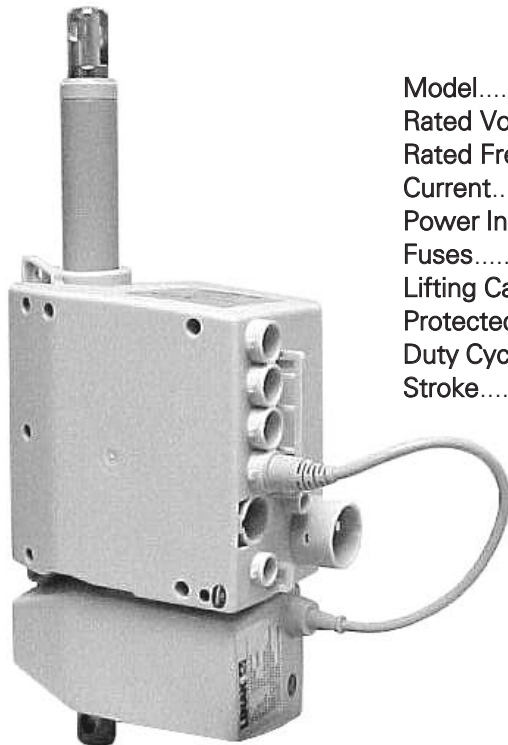


120 Volt Motor

Model.....Linak LA-31
Rated Voltage.....120V
Rated Frequency.....60 hz
Current.....1.6 A
Power Input.....200 W
Fuses.....1.6A (Non-serviceable)
Lifting Capacity.....400 lbs (182 kg)
Protected.....Class I, Type B
Duty Cycle.....10%
Stroke.....7.875" (20 cm)

230 Volt Motor


Model.....Linak LA-31
Rated Voltage.....230V
Rated Frequency.....50-60 hz
Current.....1.0 A
Power Input.....230 W
Fuses.....1.0A (Non-serviceable)
Lifting Capacity.....400 lbs (182 kg)
Protected.....Class I, Type B
Duty Cycle.....10%
Stroke.....7.875" (20 cm)

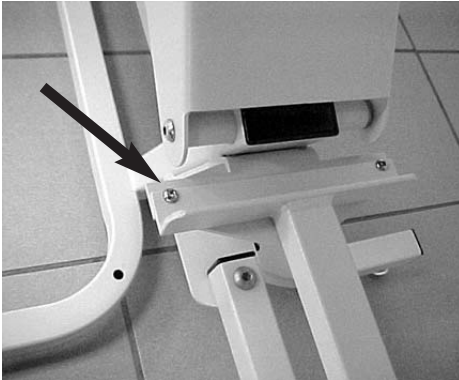

Installing the Traction Stand

All screws and nuts required to install the Traction Stand were shipped to you in the original packaging.

Use the 1/4 - 20 x 3/4 round head Phillips screw with the 1/4 - 20 ESNA nut (as shown) to secure the top section of the traction stand to the base bracket of the table.

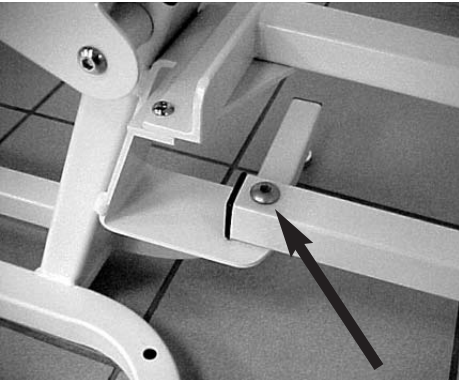

To secure the bottom section of the traction stand to the lower base bracket, use the 1/2 - 13 x 2 button head screw with a 1/2 - 13 ESNA nut (as shown).

OPERATION

TTET 200 Traction Table®



Unlocking the Table

Turn the Locking Knob either clockwise or counter clockwise until the table's foot section moves freely.



Sliding Sections


Slide the sections apart and adjust as necessary.



Locking Sections

Turn the Locking Knob so that the knob indication label is straight up to lock the traction table and begin the traction therapy.


NOTE: Always make certain that the sections are locked before moving a patient on or off of the table.

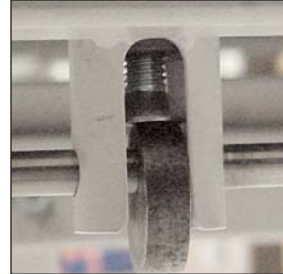

Increasing Tension

The Tension Screw controls the resistance of the cushions on the table.

To adjust the tension of the brake plunger against the cushion, unlock the table so that the Locking Knob is not straight up.

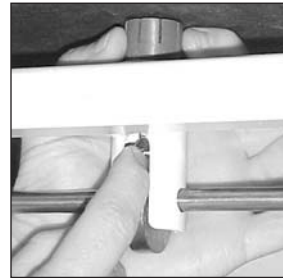

Hold the plunger securely with one hand, and with the other hand, turn the Tension Screw counter clockwise. The plunger will adjust outward.

Re-lock the table (so that the Locking Knob is straight up).

If the foot section is secure, the adjustment is complete.

If there is still some "play" in the foot section, repeat the steps listed above.

WARNING

- When making adjustments to the table sections, make certain the patient weight is removed before adjusting.
- Move hands and feet away from the working mechanism of the table when making any and all adjustments to height or table sections.



Adjusting Table Height

Step on the corresponding side of the foot pedal to raise and lower the table to the desired height.

WARNING

- Do not transport patients on the TTET 200 Traction Table. This table is not designed to support a patient during transport.



Relocating the TTET 200 Traction Table® with Optional Retractable Casters

The retractable casters are optional equipment that can be purchased separately. The casters are not shipped with the standard TTET 200 Traction Table.

Step on the Caster Lock Lever and push it completely down with your foot. This will bring the swivel caster down to allow the table to be moved to another location.



To park the table, first move it to the desired location. Raise the Caster Lock Lever with the toe of your shoe. This will raise the swivel caster and cease table movement.

TROUBLESHOOTING

TTET 200 Traction Table®

Before calling for service, carefully review this operator's guide. In the event you are still unable to correct the problem, contact your Chattanooga Group dealer for all repair service. Be certain to specify your model number, serial number, and a detailed description of the issue you encountered.

Problems	Possible Causes
1. Table will not raise or lower when using the foot pedal.	• The Power Supply Cord (male end) is not connected to the electrical outlet.
	• The electrical outlet is not operable. Try connecting another electrical appliance known to be operational (such as a lamp) to the same outlet. Check the circuit breaker or fuse at the building's power panel.
	• If your electrical outlet is fitted with a GFI (Ground Fault Interrupt) device, it may require resetting.
	• The Power Supply Cord (female end) is not fully engaged in the Power Supply Receptacle.

CLEANING

After each use, clean the table using a soft cloth dampened with water and a mild antibacterial detergent.

SERVICE

Should the TTET 200 Traction Table require service, contact the selling dealer or Chattanooga Group Service Department. All units returned to the factory for service must include the following:

WARRANTY REPAIR/OUT OF WARRANTY REPAIR

- 1.** Written statement containing the following information;
 - RGA Number - Obtain from Factory
 - Unit Model Number
 - Unit Serial Number
 - Contact person with Phone and Fax Numbers
 - Billing Address (for Out of Warranty Repair)
 - Shipping Address (Where to Ship Unit after Repair)
 - Detailed Description of Problem or Symptoms
- 2.** Copy of original invoice issued at purchase of the unit.
- 3.** Ship unit to Factory in the original container with all accessories and information as required in item 1 above to:

Chattanooga Group
4717 Adams Road
Hixson, TN 37343
Phone: USA: (800) 592-7329
Outside USA Phone: +1 (423) 870-7200
FAX: (423) 875-5497
Outside USA FAX: +1 (423) 870-2046

Service to these units should be performed only by Service Technicians Certified by Chattanooga Group.

WARRANTY

TTET 200 Traction Table®

Chattanooga Group ("Company") warrants that the TTET 200 Traction Table ("Product") is free of defects in material and workmanship. This warranty shall remain in effect for one year (12 months) from the date of original consumer purchase of this Product and extends to any owner of the Product during the warranty period. If this Product fails to function during the one year warranty period because of defect in material or workmanship, Company or the selling dealer will repair or replace this Product without charge within a period of thirty (30) days from the date on which the defective Product is returned to the Company or the dealer. Company or the dealer will ship the replacement or the repaired Product to the consumer's facility.

All repairs to the Product must be performed by a service center authorized by the Company. Any modifications or repairs performed by unauthorized centers or groups will void this warranty.

To participate in warranty coverage, this Product's warranty registration card (included with Product) must be filled out and returned to the Company by the original owner within ten (10) business days of purchase.

This Warranty Does Not Cover:

Replacement parts or labor furnished by anyone other than the Company, the selling dealer or a certified Company service technician.

Defects or damage caused by labor furnished by someone other than Company, the selling dealer or a certified Company service technician.

Any malfunction or failure in the Product caused by product misuse, including, but not limited to, the failure to provide reasonable and required maintenance or any use that is inconsistent with the Product User's Manual.

COMPANY SHALL NOT BE LIABLE IN ANY EVENT FOR INCIDENTAL OR CONSEQUENTIAL DAMAGES TO PROPERTY OR BUSINESS.

Some states do not allow the exclusion or limitation of incidental or consequential damages, so the above limitation or exclusion may not apply to you.

To Obtain Service From Company or the selling dealer under this warranty:

1. A written claim must be made within the warranty period to the Company or the selling dealer. Written claims made to the Company should be sent to:

4717 Adams Road
P.O. Box 489
Hixson, TN 37343 USA
Telephone: (800) 592-7329
Facsimile: (423) 875-5497

2. The Product must be returned to the Company or the selling dealer by the owner.

This warranty gives you specific legal rights and you may also have other rights which vary from state to state.

The Company does not authorize any person or representative to create for it any other obligation or liability in connection with the sale of the Product.

Any representation or agreement not contained in the warranty shall be void and of no effect.

**THE FOREGOING WARRANTY IS IN LIEU OF ALL OTHER WARRANTIES, EXPRESSED OR IMPLIED,
INCLUDING ANY WARRANTY OR MERCHANTABILITY OR FITNESS FOR A PARTICULAR PURPOSE.**

More Trusted Products from Chattanooga Group

Achiever™

Supports

Adapta®

Treatment Tables

A.E.R.® Boot

Auto Edema Reduction Boot

Baseline® Measurement Instruments

Dynamometers, Goniometers, etc.

boo-boo pac™

Child Size Bear-Shaped Cold Pack

Cambion®

Shock Dampening Foot Care Products

Carpal-Trac™

Carpal Traction Accessory

Cervical Traction System

Clinical Cervical Traction

ColPac®

Chilling Units and Reusable Cold Therapy Products

Conductor™ Gel

Highly Conductive Ultrasound Gel

Contracture Products

Contracture Management Orthotic Products

Chattanooga Exerciser™

Exercise Equipment

CTS®

Carpal Tunnel Stretching Device

DURA-STICK® Electrodes

Self-Adhesive Electrodes

EMG Retrainer® & EMG Retrainer® IR

Dual Channel Surface EMG

Flexi-PAC®

Reusable Hot and Cold Compresses

Fluidotherapy®

Dry Heat Whirlpool Therapy Units

Forte™ CPS

Electrotherapy and Ultrasound

Gel Medex™

Gel Mattress Overlay

HANDISIZER™

Hand Exerciser

Hydrocollator®

Heating Units and HotPacs™

Intelect® Legend

Ultrasound and Electrotherapy Products

Magneciser™

Exercise Equipment

Myassage®

Massage Lotion

Nylatex®

Elastic Wraps

OptiFlex® & OptiFlex® S

Continuous Passive Motion

Opti-Ice™

Cold Therapy System

Para-Care®

Paraffin Wax Unit

Pillow Perfect™

Cervical Pillow Line

Pivotal Therapy System™

Orthotics for the Spine

PR™ and PR II™

Pressure Reduction Wheelchair Cushions

ProPower Pillow™

Power Massage Pillow

PresSsion®

Intermittent Compression

Pron Pillo®

Positioning Pillow

QuikWrap™

Deluxe Universal Belt System

SensaFlex™

Hot and Cold Compress

SPORT-PAC™

Soccer Ball Shaped Cold Pack

Theratherm™

Digital Moist Heating Pad

Therma-Wrap™

Hot and Cold Compression

Triton®

Treatment and Traction Equipment

Tru-Trac®

Traction Equipment

TX®

Treatment and Traction Equipment

Vectra™ Series

Electrotherapy products.

VersaBath Seat™

Aid to Daily Living

Wellness 1st™

Back Support

Women's Contour Back Support

Back Support

Bringing
Innovation
to the Rehab
World!™



4717 Adams Road
P.O. Box 489
Hixson, TN 37343 U.S.A.
1-423-870-2281
1-800-592-7329 U.S.A.
1-800-361-6661 CANADA
+ 1-423-870-2046 OUTSIDE U.S.A. FAX
www.chattgroup.com

78550A

© 2003 Encore Medical