USER MANUAL
BEDIENUNGSANLEITUNG

FORTRESS-CALYPSO

English 4

Deutsch 52

English

© 2009 Handicare

All rights reserved.

The information contained in this manual may not be reproduced and/or published in any form, by print, photoprint, microfilm or by any other means, whether electronically or mechanically, without the prior written authorisation of Handicare.

The information provided is based on the general data relating to the design of the product known at the time of publication of this manual. Handicare has a policy of continuous improvement and reserves the right to make changes and modifications to its products.

The information provided applies to the standard version of the product. Handicare can not be held liable for any damage resulting from product specifications which do not conform to the standard configuration.

The information made available has been compiled with the greatest care. However, Handicare can not be held liable for any possible errors in the information or for the consequences thereof. Handicare accepts no liability for loss as a result of work carried out by third parties.

Under regulations concerning the protection of trade names, the names and trade names etc. used by Handicare may not be regarded as openly available.

Introduction.....	7
About this manual	7
Product labels	8
Symbols used in this manual	10
Service and technical support.....	10
Identification plate.....	10
Identifying the product.....	11
Approvals	11
CE declaration.....	11
Used scooters and the environment.....	11
Use for intended purposes ⁽¹⁾	11
Terms of warranty.....	12
Liability provisions	14
1 General safety rules and regulations	15
1.1 Labels and instructions on the scooter.....	15
1.2 Technical specifications	15
1.3 Adaptations	15
1.4 Safety.....	15
2 General description	18
3 Main components	19
4 Adjustment options	21
4.1 Adjusting the seat	21
4.1.1 Adjusting the seat height	21
4.1.2 Adjusting the seat depth (sliding system)	21
4.1.3 Adjusting the backrest	22
4.1.4 Adjusting the armrest.....	22
4.1.5 Adjusting the headrest	23
4.2 Adjusting the steering column.....	23
4.3 Adjusting the rear suspension.....	23
5 Operation.....	24
5.1 Charging the batteries.....	24
5.2 Driving the Fortress Calypso.....	25
5.2.1 Checks before use.....	25
5.2.2 Mounting and dismounting.....	26
5.3 Controls	26
5.3.1 Control panel	27
5.3.2 Switching on	29
5.3.3 Charging point	29
5.4 Manoeuvring	30
5.4.1 Forward drive and reverse drive	30
5.4.2 Corners.....	31
5.4.3 Slopes.....	31
5.4.4 Obstacles.....	33
5.4.5 Automatic cut-out on overload	34
5.4.6 Switching off	34
5.4.7 Parking	34
5.5 Pushing the scooter	35
5.5.1 The scooter in neutral.....	35
5.6 Dismantling the scooter for storage and/or transportation	36

Table of contents

5.6.1	Removing the seat.....	36
5.6.2	Dismantling the scooter	36
5.6.3	Collapsing the steering column.....	40
5.6.4	Lifting and loading	40
5.6.5	Reassembling the scooter	40
5.7	Transporting the scooter in its entirety	40
6	Maintenance	42
6.1	Maintenance table.....	42
6.1.1	Batteries	42
6.1.2	Replacing the batteries	43
6.1.3	Cleaning the batteries.....	44
6.2	Tyres.....	44
6.2.1	Inflating the tyres	44
6.3	Cleaning the scooter	44
7	Troubleshooting.....	46
7.1	Troubleshooting table	46
7.2	Automatic circuit breaker	47
7.3	Electromagnetic interference	48
8	Technical specifications	49
8.1.1	Fortress Calypso product specifications	49
8.2	Authorised service and technical support	51

Introduction


Congratulations on choosing the Handicare Fortress Calypso electric scooter, and welcome to the growing group of people who use a Handicare scooter. Handicare is a name that stands for reliability and innovative technology, with high-quality, user-friendly products as a result.

About this manual


This manual will help you to use and maintain the product safely. If in doubt, always contact your dealer.

The words “left”, “right”, “front” and “rear” are used in this manual to refer to specific parts of the scooter, as seen from the user’s perspective while sitting on the scooter.

The serial number for the scooter, a description of the model and the name of the authorised Handicare dealer can all be found on the back cover of this manual. This information will be useful to you if you should need assistance with your product in the future.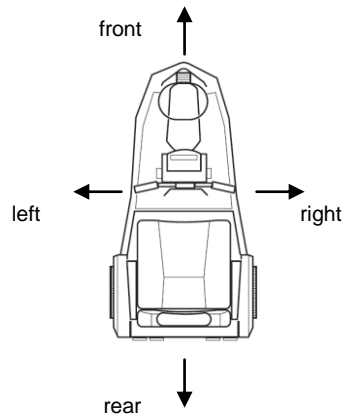


Product labels

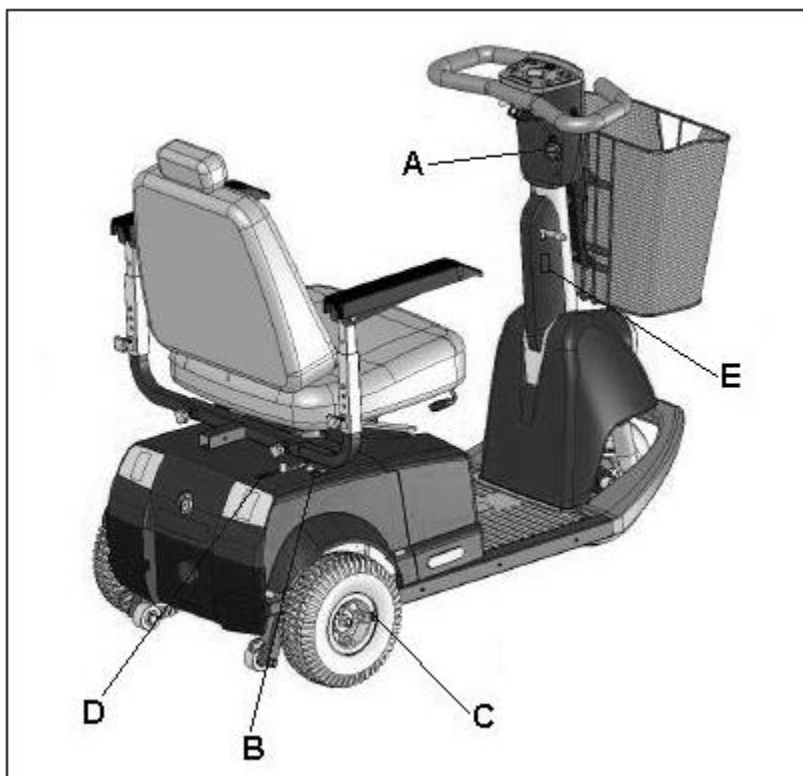
The following stickers and labels can be found on the product:

- A. Charging point
- B. Circuit breaker
- C. Tyre pressure
- D. Neutral position
- E. Adjustment of steering column angle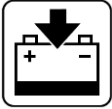


A. Charging point

See the chapter on 'Charging the batteries', section 5.1, for how to charge the batteries.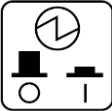

B. Circuit breaker

The scooter is equipped with a fuse which interrupts the electric current on overload. See 'Automatic circuit breaker', section 7.2.

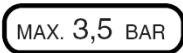
C. Tyre pressure

See the 'product specifications' for the correct tyre pressure.

D. Neutral position – Free wheel facility

1. Lever fully forward ("BRAKE applied"): When the motor drive is engaged the scooter can be driven electrically and the brakes are also applied when the scooter is switched off.
2. When the lever is in the lowest position the motor drive is disengaged. The scooter can then be pushed manually.

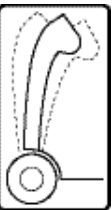E. Adjusting the steering column angle

The steering column can be adjusted to the position you find most comfortable when driving. See 'Changing the position of the steering column', section 4.1.6.

Symbols used in this manual

CAUTION

Operations or manoeuvres which are not carried out correctly may result in damage to the product or the environment, or in user injury.

ATTENTION!

Suggestions for making specific operations or manoeuvres easier to carry out.

First consult the source(s) of information given.

Disconnect the charging cable from the charging point on the scooter before carrying out any maintenance on the scooter.

Service and technical support

Your dealer should be contacted for information about specific adjustments, maintenance or repair and will be happy to help you.

When contacting your dealer, please ensure you have the following information to hand:

- Model
- Serial number

This information can be found on the information plate. See the illustration below.

Identification plate


Product information is recorded on plate (A). See: 'Identifying the product'.

A

Identifying the product

- A. Model
- B. Date of manufacture
- C. Serial number / identification number
- D. Usage: Indoor use, outdoor use or both
- E. Maximum load in kg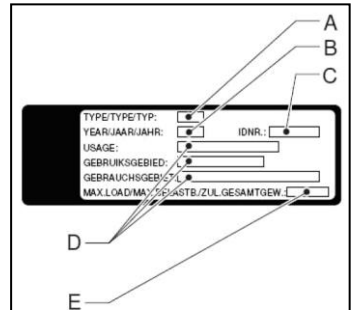

Approvals

The product meets the following requirements:

- EN12184; 1999 Electrically powered wheelchairs and scooters, class B.
- ISO7176-8 Requirements for static, impact and fatigue strengths.
- ISO7176-9 Climatic tests for electric wheelchairs and scooters.
- ISO7176-14 Requirements and test methods for power and control systems for electric wheelchairs.
- ISO7176-16 Requirements for resistance to ignition of upholstered parts.

The product has been EMC-approved (for electromagnetic compatibility) in accordance with EN12184 (1999).

CE declaration

The product complies with the regulations of the Medical Devices Directive and therefore bears the CE mark.

Used scooters and the environment

If your scooter is no longer required or needs to be replaced, it will usually be possible to arrange for your dealer to take it back. If this should not be possible, consult with your local authority to find out if the scooter can be recycled or if the materials used in it can be processed in an environmentally friendly way.

Various plastics and metals were used in the manufacture of the scooter. The scooter also contains electronic components which should be disposed of accordingly. The batteries constitute chemical waste.

Use for intended purposes ⁽¹⁾

The Fortress Calypso scooter was developed for:

- The transportation of people weighing up to 125 kg
- Use on pavements, footpaths and cycle paths
- Use in and around the house

Introduction

The dealer must give you clear instructions before you start operating the product independently.

Your first test drives of the Fortress Calypso are best carried out under the supervision of an experienced dealer.

The Fortress Calypso scooter is not classified as a road vehicle and therefore may not be used on the road (crossing over the road being the only exception).

The Fortress Calypso has a maximum speed of 10 or 12 km/h (6 or 7,5 mph).

ENSURE THAT YOU ARE FULLY ACQUAINTED WITH THE CONTENTS OF THIS MANUAL BEFORE STARTING TO DRIVE THE FORTRESS CALYPSO.

If you use the scooter in a dangerous fashion, or use the scooter for purposes other than those intended, Handicare will accept no liability for any personal injury or damage to property caused by such misuse.

(1) Intended use, as specified by EN 292-1, means the use for which the technical product is suitable according to the manufacturer's declaration, including the manufacturer's instructions in the sales brochure. In the event of doubt, use is taken to mean the use as construed from the design, construction and function of a product. The instructions contained in the user manual must also be followed within the scope of intended use.

Terms of warranty

The following words used in the terms of warranty and liability are defined as follows:

Product: The manual or electric wheelchair or scooter manufactured and supplied by Handicare.

Customer: The person procuring a Product directly from Handicare.

Dealer: The person who supplies a Product procured from Handicare to third parties.

User: The person who uses a Product manufactured by Handicare.

Without prejudice to the provisions with respect to guarantees contained in the general terms and conditions applicable to the Product, the following applies in any case with regard to guarantees:

1. Unless otherwise specified in the following conditions, Handicare guarantees to the Customer the reliability of the Product for the purpose for which it is intended – as variously described in this manual – and the quality of the materials from which the Product is made, and the manner in which it has been manufactured.
2. Repairs to or replacement of Product components becoming necessary as a consequence of defects caused by defective materials or manufacturing errors will be carried out free of charge, providing that the defects developed within one year of the date of delivery of the Product to the Customer. The components to be replaced shall be sent to Handicare, carriage paid. The costs for removal or mounting of these components shall be born by the Customer. The following therefore do not qualify for free-of-charge repair or replacement as referred to in the previous sentence:
 - repair or replacement necessary due to defects developing later than one year from the date on which the Product was delivered to the Customer;
 - repair or replacement necessary due to defects originating from improper or careless use of the Product or due to use of the Product for purposes other than those for which it is intended, in which case, if the Customer is a Dealer,

the Dealer shall indemnify Handicare against any claims from Users or other third parties resulting from defects originating from improper or careless use of the Product;

- components which are liable to wear whereby the necessity for repair or replacement of such components is the actual result of normal wear.
3. With respect to electric Products and without prejudice to the provisions under 2, a guarantee is given with respect to the battery constituting a component of the Product only in the case of battery failure or inadequate performance when these are the demonstrable, direct result of material or manufacturing faults. A battery failure or inadequate performance of a battery as the result of normal wear is not covered by the guarantee as referred to in these terms of warranty. Failures or inadequate performance are similarly not covered by the guarantee if they are the result of improper or incompetent use of the Product or of the battery comprising a part thereof, including the incorrect charging of the battery and neglecting to carry out proper maintenance in good time, in connection with which, in cases in which the Customer is a Dealer, the Dealer shall similarly indemnify Handicare against any claims from Users or other third parties resulting from improper or incompetent use of the Product or of the battery comprising a part thereof as referred to above.
 4. The guarantees given under the preceding terms are in any case rendered null and void if:
 - the guidelines for the maintenance of the Product given by Handicare are not followed or are insufficiently followed;
 - necessary repairs or replacement of components are the result of neglect of, damage to or overloading of the Product, or if the Product was used for purposes other than that for which it was intended;
 - components of the Product have been replaced by components originating from a source other than the source used by Handicare and/or components of the Product have been replaced without Handicare's authorisation.
 5. The guarantees given in provisions 1 to 3 inclusive are also rendered null and void in the case of the Product being used by a new user within the period of warranty when reuse necessitated adaptations to the product in any way whatsoever, and these adaptations were not made by Handicare or by order of and/or under the instructions of Handicare.
 6. In order to retain rights under the guarantees stated above, the Customer must contact Handicare as soon as possible in the case of loss events or other calamities and provide Handicare with as many details as possible. The opportunity for the Customer to assert a claim under the terms of the guarantees described above lapses in any case after 20 working days from the loss event or from the calamity giving cause for a claim under the guarantee.
 7. Replacement of a component or the repair or reconditioning of the Product within an ongoing period of guarantee does not extend the period of guarantee.
 8. Handicare gives no guarantees with respect to repairs to or reconditioning of the Product not carried out by order of or on the instructions of Handicare. When repairs or reconditioning have been carried out by or by order of and/or on the instructions of a Customer, the Customer shall indemnify Handicare against claims from third parties resulting in the broadest sense from such repairs or reconditioning.

Liability provisions

Without prejudice to the provisions with respect to liability contained in the general terms and conditions applicable to the Product, the following in any case applies with regard to liability:

1. Having regard to the following provisions, Handicare accepts liability only for damage/loss through death or physical injury as a result of a defect in a Product for which Handicare is responsible and for damage/loss to another object which is part of the private property of the user of the Product, providing that the damage/loss is the direct result of a Product defect. Handicare's liability is in any case limited to the amount paid by Handicare's liability insurance for the case in question.
2. Handicare accepts no other or no further liability than that described under 1. In particular, Handicare accepts no liability for resulting damage of any kind.

1 General safety rules and regulations

Handicare accepts no liability for loss or injury caused by not following the safety rules and regulations fully, or otherwise for loss or injury as a result of negligence during use or when cleaning the scooter and any accessories. Supplementary safety regulations may apply depending on the specific circumstances for operation or on the accessories used. Please contact your dealer immediately if you should ascertain any potential hazards when using the product.

The user of the scooter (see 'Intended use') is at all times responsible for following local safety rules and guidelines.

1.1 Labels and instructions on the scooter

The markings, symbols and instructions on the scooter constitute a part of the safety guidelines for the scooter. For this reason they may never be covered or removed and must be present and clearly legible for the entire period of time the scooter is in existence.

- Illegible or damaged labels, symbols and instructions must be replaced or repaired immediately. Contact your dealer in such cases.

1.2 Technical specifications

No changes may be made to the technical specifications.

1.3 Adaptations

Adaptations to components of this product are not permitted.

1.4 Safety

It is of great importance that attention be paid to the following safety regulations for the prevention of accidents and undesirable situations.

Take extra care when driving on slopes:

- **Never remove safety devices, such as the anti-tip wheels.**
- **Never drive the Fortress Calypso up slopes with a gradient greater than that specified in section 5.4.3 of this manual.**
- **Always drive up slopes slowly and take extra care.**
- **Never descend a slope at full speed.**
- **Do not drive down slopes covered with loose gravel or with a sandy surface, as one of the back wheels may slip.**
- **Do not turn on slopes.**
- **Never take corners at full speed. Reduce speed before taking a corner.**
- **When driving up a slope, adopt a position which promotes stability, as described in section 5.4.3 of this manual.**

General safety rules and regulations

- Ensure that clothing does not trail as this can become entangled in the wheels.
- Ensure that your fingers do not become trapped in the mechanism for adjusting the position of the steering column.

You must adjust your driving to the prevailing conditions:

- Drive carefully on roads which have become slippery due to rain, black ice or snow!
- Reduce speed in busy surroundings.
- Do not use the scooter on unsurfaced roads.
- Ensure that the Fortress Calypso A does not come into contact with salt water. Salt water is corrosive and can damage the scooter.
- Ensure that the Fortress Calypso does not come into contact with sand. Sand can find its way into the moving parts of the scooter, resulting in these parts wearing out unnecessarily quickly.
- Never drive the scooter while under the influence of drugs, alcohol or prescription medications which could affect your driving ability.
- Your eyesight must be adequate for you to drive the scooter safely.
- If visibility is not ideal, then it is obligatory for the lights to be turned on. The scooter is equipped with brake lights which illuminate on braking.
- Use the indicator lights only to indicate a change in driving direction.
- Only use the horn for warning pedestrians or other road users of possibly hazardous situations.

- Never place metal components on top of the batteries. Doing so could cause a short circuit, leading to damage.
- Do not transport any passengers on the scooter. The scooter has been specifically designed to transport you alone.
- Do not drive the scooter with the seat backrest angled too far to the rear. This can affect weight distribution and the stability to the rear of the scooter, in particular when driving up slopes or over obstacles.
- Ensure that your scooter does not transport a greater load than fits in the basket. Never hang anything on the tiller.

- Never use the scooter for pulling a trailer. The scooter has not been designed for such use and doing so can cause serious damage to your scooter.
- Do not stand on the platform in order to reach up to objects.
- Do not position your feet too close to the front wheel and fork when driving. Always keep your feet on the rubber mat and not up against the front tiller cover.

The scooter is electronically driven. Specific parameters have been factory-set. These settings have been made in order to ensure that use of the Fortress Calypso is comfortable and efficient and they can not be changed.

The standard version of your scooter has been tested in accordance with the most stringent requirements for electromagnetic interference. If you use a mobile telephone near to a specially adapted scooter, it is advisable to switch the scooter off first.

Your scooter can affect electromagnetic fields, such as alarm systems. If the scooter's electronics are not well shielded, sensitive electrical devices, such as shop alarm systems and garage door openers, can be affected. The scooter has been tested for such interference. Please report any problems of this nature to your dealer immediately.

- Beware of ultraviolet light. Ultraviolet light can cause premature wear of materials such as rubber, plastic and enamel.
- Pay attention to objects sticking out from the scooter. These can cause damage to your surroundings or to the scooter itself.
- Keep the scooter away from open flames.
- Avoid extreme weather conditions or extremely wet conditions. Maintain and store the scooter in a clean and dry state.

2 General description

The Fortress Calypso is an electrically driven scooter for general indoor and outdoor use.

The seat height can be adjusted to the length of the user's legs. The scooter's seat can be swivelled and the armrests can be folded away to facilitate sitting down on the scooter or getting up from the scooter.

The steering column is adjustable for a comfortable driving position.

Extra caution must be paid when driving at higher speeds. Pay attention to your speed and slow down, particularly on pavements and in pedestrian areas.

The following points should always be remembered when using the scooter:

- In terms of traffic regulations, a scooter is not a motor vehicle. The rights and obligations of a scooter user are the same as those of a pedestrian.
- You are always responsible for your scooter being in perfect working order so that the scooter can be driven safely.
- The scooter is a technical product. For this reason it is important that repairs and all maintenance are carried out by qualified personnel. If your scooter needs attention, then you must contact your dealer

3 Main components


The scooter is equipped with, or can be equipped with, the following main components:

- A. Seat.
This is where the user sits.
- B. Backrest.
Supports the user's back.
- C. Headrest
Supports the head.
- D. Armrest.
Supports the arms.
- E. Steering column
The control panel and all of the control keys and levers are located on the steering column.
- F. Lights
The lights should be switched on when driving in the dark.
- G. Indicator lights
Indicate changes in direction while driving.
- H. Detachable basket
For the transportation of personal belongings.
- I. Front wheel
Front wheel with suspension.
- J. Front bumper
Protects the front wheel on collision.
- K. Rear wheels
Rear wheels with suspension.
- L. Anti-tip wheels
Wheels to prevent the scooter from tipping over backwards on slopes.
- M. Attachment points
Points of attachment for securing the scooter.

(Consult the next page for the location of these components on the scooter)

Various elements and components on the scooter can be adjusted for optimal seating comfort.

Main components

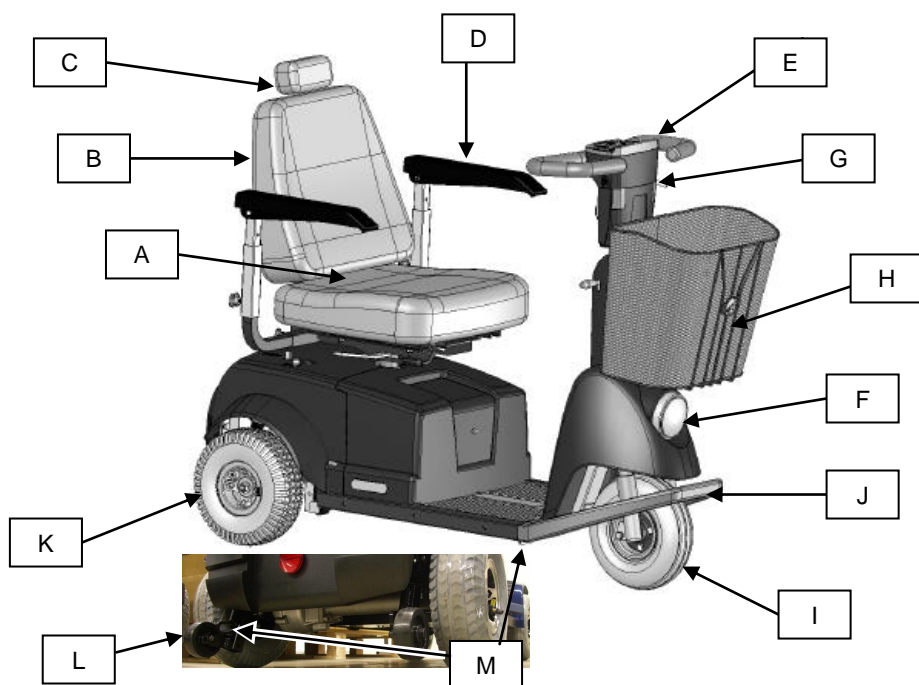4-wheel model

4 Adjustment options

There are a number of adjustments which can be made to the Fortress Calypso in order to improve its seating and driving comfort levels.

Some adjustments are permanent adjustments made by the dealer using tools.

Other adjustments can be made by the user without the use of tools.

Adjustments can be made to the following by your dealer:

- Seat height
- Rear suspension

Adjustments can be made to the following by the user:

- Seat depth
- Backrest angle
- Armrest width
- Armrest height
- Headrest
- Steering column

4.1 Adjusting the seat

4.1.1 Adjusting the seat height

The entire seat can be adjusted in height in order to obtain the optimal seating position. This adjustment is carried out by the dealer.

4.1.2 Adjusting the seat depth (sliding system)

(seat may vary from the illustration)

The seat depth can be adjusted as follows:


- Lift and hold handle (A).
- Slide the seat forwards or backwards.
- Release the handle once the seat is in the required position. Now slide the seat slightly forwards or backwards so that it locks into place.
- The seat depth has now been adjusted.

4.1.3 Adjusting the backrest

The backrest angle can be adjusted as follows:

- Lift handle (A) whilst sitting on the seat. The backrest now automatically moves forwards.
- Push the backrest to the rear by leaning against it until it is in the position you find the most comfortable.
- Once adjusted, release the handle. The backrest remains in the position you selected.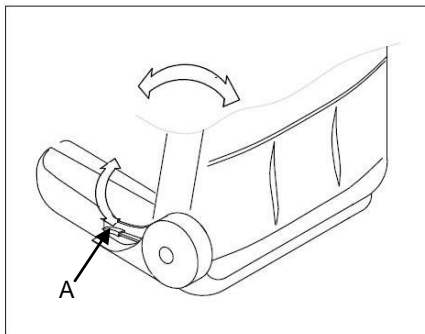

Caution: Exercise caution when using the handle when no one is sitting on the seat. The backrest is equipped with a spring mechanism which causes it to be driven quickly forwards with some force.

The backrest can be adjusted to provide good back support while driving.


If the backrest is adjusted to an angle too far to the rear, the driver's seating position is then less stable, particularly on slopes.

It is therefore important never to drive your scooter if the backrest is inclined too far back.

4.1.4 Adjusting the armrest

The armrests can be adjusted as follows:

- Rotary knobs (A) for adjusting the distance between the armrests are located at the back of the seat.
- Turn the knob in an anti-clockwise direction to loosen the armrest.
- Once you have adjusted the distance between the armrests, tighten the rotary knobs by turning in a clockwise direction. Then check to see whether the armrests are firmly in place.
- The rotary knobs (B) for adjusting the height of the armrests are located on the rods supporting the armrests.
- Turn the knob in an anti-clockwise direction to loosen the armrest.
- Press the nipple and slide the armrest upwards or downwards into one of the three positions.
- Once you have adjusted the height, tighten the rotary knobs by turning in a clockwise direction. Then check to see whether the armrests are firmly in place.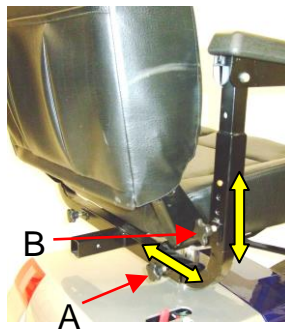

4.1.5 Adjusting the headrest

The headrest can be adjusted to various heights.

The headrest height is adjusted as follows:

- Press the plastic button at the point where the headrest is attached to the seat, at the same time moving the headrest to the required height. The headrest can also be removed if required.
- Release the button and move the headrest slightly so that it locks into one of the positions.


4.2 Adjusting the steering column

The steering column can be adjusted to make driving more comfortable and to make it easier for you to mount and dismount from the scooter.

The steering column is adjusted as follows:

- Using one hand, pull handle (A) downwards while pulling the steering column towards you with the other hand until it is in the most comfortable position.
- Release the handle. Then release the steering column.

If you release the steering column while still pulling the handle down, the pneumatic spring will cause the steering column to jump into the most forward position automatically.

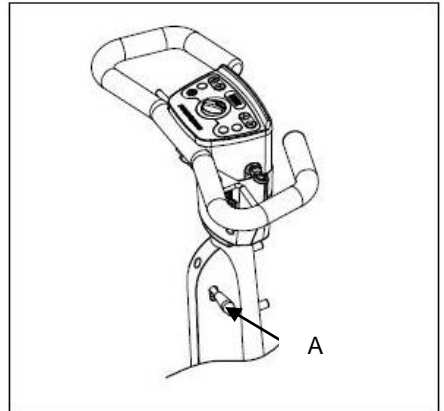Exercise caution while adjusting the steering column and ensure that your fingers do not become trapped.

4.3 Adjusting the rear suspension

The rear suspension of the scooter can be adjusted to provide optimal comfort. This adjustment is carried out by the dealer.

5 Operation

5.1 Charging the batteries

Consult the following documentation on charging the batteries:

Regulations relating to batteries

The battery charger manual, or the instructions on the battery charger.

The scooter uses gel batteries which are fully encased and sealed and require no maintenance.

Under normal use the batteries need to be charged every night.

Charge the batteries as follows:

- Switch off the scooter.
- Plug the charging cable into the charging point. See 'Charging point'.
- Plug the battery charger into the wall socket.
- Turn on the battery charger (if the battery charger has an on/off switch).

Use only maximum 8 A battery chargers.

Safety:

The scooter can not be used once the charging cable plug has been connected to the charging point.

The battery charger has been designed to ensure that batteries can not be too highly charged. If the batteries are no longer charging, the charging current is drastically reduced until charging is terminated - the batteries are fully charged. The minimum duration for charging gel batteries is approximately 8 hours. Most batteries are 80% charged after 8-12 hours of charging.

Once the batteries are charged you must:

- Turn off the battery charger, if applicable.
- Unplug the battery charger from the wall socket.
- Disconnect the charger from the charging point on the scooter.

The scooter is now ready for use.

Always remove the charging cable once the batteries have been charged. This prevents the batteries from slowly running down.

5.2 Driving the Fortress Calypso

The following matters are covered in this manual in order to enable you to drive your scooter:

- Checks before use
- Mounting and dismounting
- Control panel
- Driving
- Pushing
- Dismantling

5.2.1 Checks before use

The following checks should be carried out each time before the scooter is used:

- Check whether the seat adjustments are locked in place.
- Check whether all lights and indicators, both front and rear, are in good working order.
- Check whether tyre pressures are correct. See 'Product specifications' (3.5 bar).
 - Tyres which are poorly inflated make driving more difficult and less pleasant.
 - The scooter uses more power and the batteries run down more quickly if tyres are poorly inflated.
 - Poorly inflated tyres cause the tyres to wear unnecessarily.
- Check whether the batteries are sufficiently charged: this is indicated by the green section of the battery indicator. See 'Control panel'

The capacity of the batteries is reduced in winter. When there is a light frost, battery capacity is reduced to approximately 75%, and when temperatures are under -5°C, capacity drops to around 50% of the normal capacity. This reduces the scooter's range.

- Check whether the neutral handle is in the 'Brake applied' position. See 'The scooter in neutral'.
- Check whether the brakes are in good working order. See 'Manoeuvring'.

If the scooter does not react in a predictable fashion, immediately release the throttles and allow the scooter to come to a complete standstill. Remove the ignition key from the control panel and then re-insert it to restart the scooter.

If there are no problems, then the scooter can be driven.

5.2.2 Mounting and dismounting

(seat may vary from the illustration)

Before mounting or dismounting, ensure that the scooter is turned off by removing the ignition key. The automatic parking brake must also be applied.

The armrest can be lifted and the seat can be swivelled to the side before mounting or dismounting.

Mounting:

- Push the seat lock handle (A) forwards and swivel the seat by a quarter of a turn to the left or to the right. When you release the seat lock handle the seat locks into place automatically. The handle clicks back into place by itself.
- You can now be seated.
- Swivel the seat back into place in the same way as described above.

When the seat is in the normal position again, it clicks into place automatically. This means that the seat can not start to swivel while driving.


The seat has folding armrests, allowing you to mount or dismount from the side. After mounting or dismounting, always check whether the armrests have been folded back down.

Before dismounting, follow the same steps but in reverse order.


5.3 Controls

The scooter has the following controls:

- Control panel with all of the control keys
- Forward or reverse levers
- Fuse for the lights
- Charging point. See 'Charging the batteries'
- Handle for adjusting the angle of the steering column.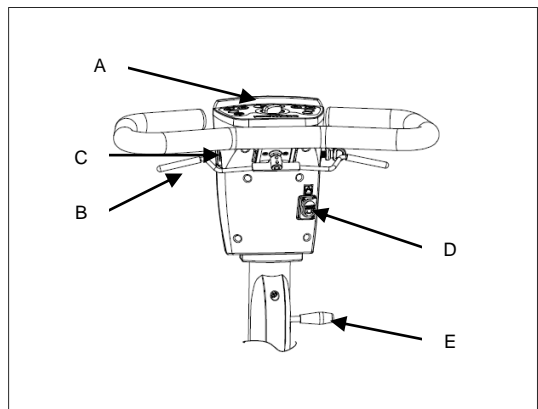

5.3.1 Control panel

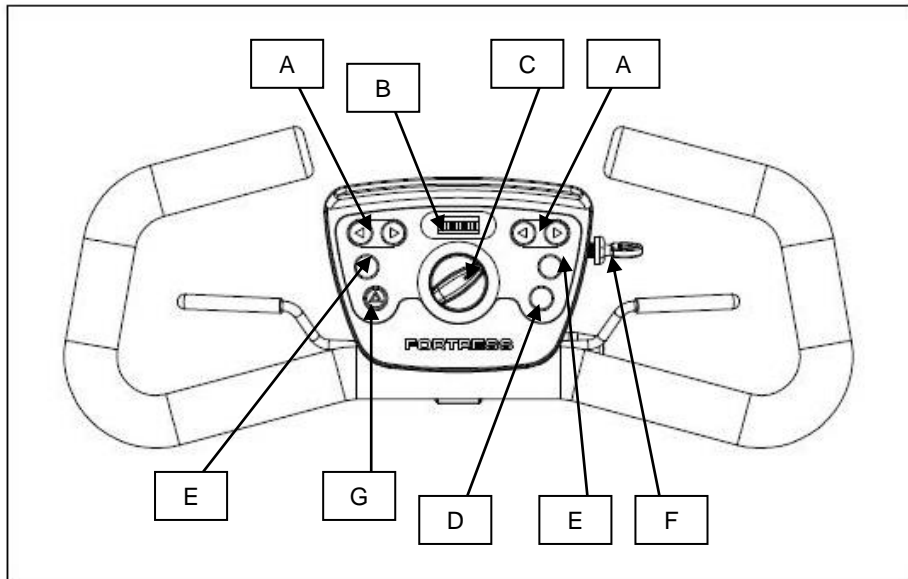The control panel is equipped with the most advanced technology, offering functions for operating your scooter which are both useful and reliable.

- A. Indicators, left and right *
- B. Battery indicator
- C. Speed regulator
- D. Lights
- E. Horn*
- F. Ignition key
- G. Hazard lights

*These keys can be found on both the left-hand and right-hand side of the control panel.

A. Indicators, left / right

On pressing the key (on the left or right-hand side) the indicator light starts flashing in order to indicate that you are going to change direction:

- press on the left-hand key for turning left
- press on the right-hand key for turning right.

A bleeping sound can be heard when the indicator is switched on.

Once you have turned left or right, press the same key again to turn off the indicator.

B. Battery indicator

The battery indicator provides a general indication of the state of the batteries. From left to right, there are three red indicator lights, three orange lights and four green lights on the battery indicator. The batteries are fully charged when all the lights are illuminated. As the battery runs down, the lights go out one by one, starting with the green lights on the right-hand side. The indicator lights indicate the battery voltage available for the control unit. It is normal for the lower indicator to go out when the scooter accelerates. This is because extra power is needed momentarily, leading to a decrease in available voltage. This momentary dip is not a true indication of the battery capacity. When the scooter is used for the first time, the battery indicator may show that the batteries are fully charged, even if this is not the case. This is a feature of the batteries. Therefore, the most accurate indication of the battery level is to be obtained when driving on a flat surface.

If the RED lights only are illuminated on the battery indicator, then it is important that the batteries be recharged as soon as possible. **Ensure that the batteries never run down completely; this reduces the lifespan of the batteries and can damage them.**

C. Speed regulator

This knob allows you to set the desired maximum speed for your scooter. The maximum speed is increased by turning the knob in a clockwise direction. Turning the knob in an anti-clockwise direction decreases the maximum speed. Adjust the speed regulator before driving. Adjust the maximum speed to the surroundings and to traffic conditions (to a limited amount of space or to a room full of people, for example).

D. Lights

By pressing this key once, the front and rear lights are turned on. On pressing once more, the lights are turned off again.

E. Horn

By pressing one of the horn keys, a warning sound is emitted to warn others of hazardous situations.
The horn continues to sound as long as the key is pressed.

F. Ignition key

The ignition key needs to be inserted in the ignition lock in order to switch on the scooter.
All of the Fortress Calypso's functions can be operated only when the ignition key has been inserted in the ignition lock, with the exception of the lights and the hazard lights.
The hazard lights and lights can always be operated, even when the ignition key has not been inserted in the lock.
If the ignition key is inserted in the lock and the scooter is not used for a period of time, a beeping sound is emitted after 20 minutes.


G. Hazard lights

Pressing this key turns on the hazard lights.
The hazard lights should be used if you suspect that you will not be seen by other road users, or if the scooter is at a standstill due to a defect.
Pressing this key once more turns off the hazard lights. A beeping sound is emitted when the hazard lights are turned on.

5.3.2 Switching on

The scooter is switched on as follows:


- Insert the ignition key as far as possible in the lock (A).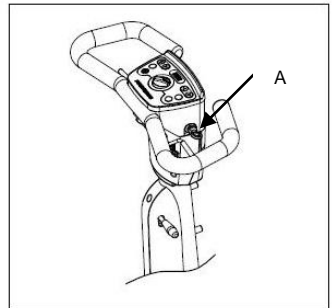

5.3.3 Charging point

- The charging point (A) is located on the steering column, under the control panel. This is the point to which the battery charger cable can be connected.

All electronics must be switched off while the batteries are charging. Remove the ignition key before charging the batteries.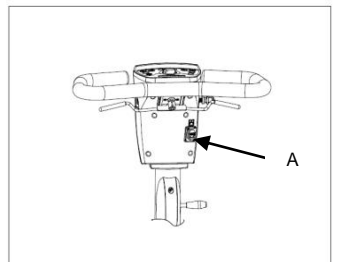

5.4 Manoeuvring

The scooter needs to be adjusted to your personal needs before you begin driving it. After carrying out all of the checks, take your seat on the scooter. See 'Mounting and dismounting'.

You can now begin driving. Use the lap strap if applicable.


As the driver of the scooter, it is important to remember that other people may not always notice you. Always pay sufficient attention to the people and traffic around you.

5.4.1 Forward drive and reverse drive

There are two levers on the tiller, one for forward drive and one for reverse drive. The levers are different colours.

Green: forward drive (A)

Yellow: reverse drive (B)

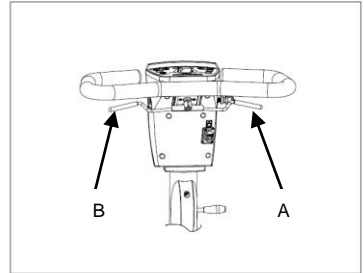Forward drive

Forward drive with finger control:

- Switch the scooter on. See 'Switching on'
- Pull the forward-drive lever (A) slowly back using your fingers. The further you pull the lever towards you, the faster you will drive.

Reverse drive

Reverse drive with finger control:

- Switch the scooter on. See 'Switching on'
- Pull the reverse-drive lever (B) slowly back using your fingers. The further you pull the lever towards you, the faster you will drive in reverse.

- **Check to see that the space behind you is free before driving in reverse.**
- **The maximum speed for driving in reverse is half the maximum speed for forward drive.**

The speed regulator allows you to limit the maximum speed for both forward and reverse drive.

Braking and stopping when in forward or reverse drive

- On slowly releasing the forward-drive or reverse-drive lever, the Fortress Calypso brakes and comes to a standstill.
On braking, the lights and brake lights illuminate automatically.
- If you need to stop suddenly while driving (to perform an emergency stop), the drive lever should be immediately released.

Under normal circumstances the drive lever can be released slowly if you wish to stop.

Once the scooter has come to a standstill, the parking brake is automatically applied.

Remarks:

The brakes can be checked by releasing the drive lever suddenly.

- **When driving at high speeds caution must be exercised, particularly on pavements and in pedestrian areas. In such cases it is advisable to set a lower maximum speed using the speed regulator.**
- **Do not cross over any roads until you can handle the scooter and the controls.**

5.4.2 Corners

Going around corners; Turning left or right

- If you turn the tiller to the left or to the right, the scooter changes direction and will drive to the left or to the right.


- **When turning, always remember to make a visual check first and to use the indicators.**
- **Always take corners at a safe speed, i.e. at a reduced speed.**

5.4.3 Slopes

- Although the Fortress Calypso can bear a maximum weight of 125 kg, it is important that the following safety regulations be observed when driving up slopes.
- When driving up slopes you must adopt a specific body position which increases stability. This is achieved by leaning your upper body forwards. This renders the scooter more stable. The position for improved stability is illustrated opposite. In addition, the scooter's backrest should never be inclined too far back and the seat must be pushed forward.
- Driving up slopes with gradients greater than those given in the following table is not permitted.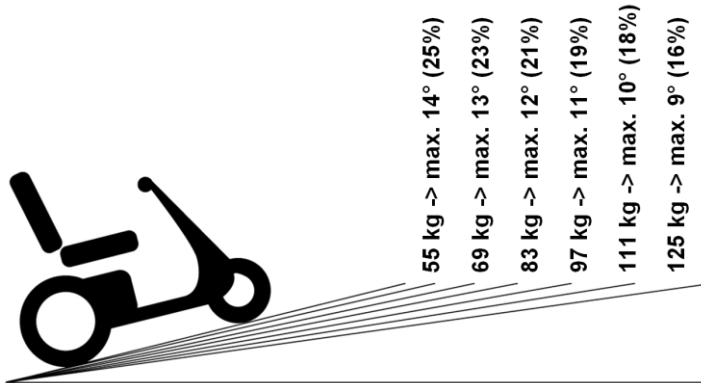

Observe these regulations. Not doing so may lead to scooter instability and to the scooter tipping over, which may result in personal injury and/or damage to your scooter.

Instructions for driving up slopes

- Adopt the position for improved stability by leaning your upper body forwards. This renders your scooter more stable. Your position can be further improved by sliding the seat further forward.
- Drive up slopes at half your normal speed.
- Maintain a constant speed while driving up slopes.
- Avoid sudden movements and jolts, such as sudden braking or accelerating.
- Do not change direction and do not attempt to turn when driving up a slope.
- Slopes should be driven up at a perpendicular angle. Do not attempt to turn and/or to drive up the slope diagonally.
- Never attempt to drive up a slope when there are possible hazards, such as slopes covered in snow, black ice, mown grass or wet leaves.
- If you notice that the scooter's speed decreases significantly when mounting a slope, you are advised to select a route which is less steep. This is to avoid the danger of the motor overheating.
- Driving up a slope for too long can cause the motor to overheat.
 - In such cases the electronics are switched off in order to prevent motor failure.
 - Remove the ignition key from the lock and allow the motor to cool down for a while.
- It is always advisable to select a less steep route wherever possible.

Non-observance of these regulations may lead to scooter instability and to the scooter tipping over. This may result in personal injury and/or damage to your scooter.


Driving down slopes

When driving down slopes, drive as slowly as possible and with the utmost control.

- Turn the speed regulator fully to the left (to the lowest speed) before driving down a slope.
- Lean your upper body to the rear and, when braking, brake gently. Braking too hard can cause the scooter to tip forwards.

5.4.4 Obstacles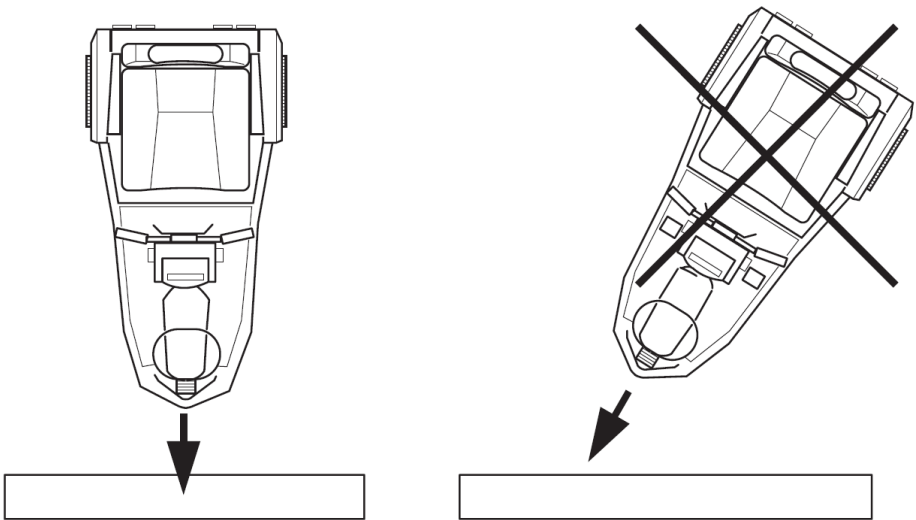

Driving over obstacles

- Drive straight towards the kerb or other obstacle and stop when the front wheel is almost touching the obstacle.
- Then press the drive lever and drive onto the pavement without changing direction.
- Once the front wheel is on the pavement, the same speed must be maintained so that the rear wheels can also mount the pavement.
- If it is not possible to drive onto the pavement, look for a location where the kerb is lower.
- Obstacles must always be approached at right angles.
- It is always advisable to make use of kerbs lowered for vehicles rather than driving over high kerbs.
- Do not drive over obstacles which are higher than 7.6 cm.

Practise by driving over low obstacles, slowly increasing heights until you are able to drive over obstacles of the maximum height. Allow yourself plenty of time for this.

Descending from obstacles

- Exercise extreme caution when descending from obstacles.
- Drive straight towards the kerb and stop at the kerb's edge.
- Carefully press the drive lever and allow the scooter to drive off of the pavement without changing direction.

- **Driving down stairs or steps is not permitted. Doing so is extremely dangerous. The scooter is equipped with anti-tip wheels offering extra stability and safety. These small wheels may hit an obstacle as you are descending from it.**
- **Ensure that the rear wheels of the scooter are at equal height at all times. The scooter may become unstable if they are not.**

5.4.5 Automatic cut-out on overload

- On overload caused by improper use or long-term use in high temperatures, the circuit breaker is activated and the scooter automatically stops. See 'Automatic circuit breaker'.
- Remove the ignition key from the lock and allow the motor to cool down.
- Incorrect use may also cause unnecessary defects and damage to the scooter.

5.4.6 Switching off

The scooter should be switched off completely after every drive. This prevents the batteries from running down unnecessarily and having to be recharged.

- **Always remove the ignition key from the lock when not driving the scooter, even if you are planning to stay seated on the scooter while it is parked. This prevents you from accidentally setting the scooter in motion by inadvertently touching the drive lever.**

5.4.7 Parking

Having parked, remove the ignition key from the lock so that it is not possible for anyone else to use your Fortress Calypso without your permission.

Once the scooter has been turned off (the ignition key has been removed from the lock), the scooter's parking brake remains applied, even if the batteries are removed. If the scooter is put into neutral, the automatic parking brake is NOT applied!

- **If the scooter is brought to a standstill on a slope, then the automatic parking brake must be applied.**
- **For this reason never put the scooter in neutral when on a slope.**

5.5 Pushing the scooter

In the event of a defect, or if the battery capacity is too low for driving the scooter, the scooter can also be pushed by hand.

Pushing the scooter can be useful when parking or when manoeuvring the scooter into a small space for storage.

- Turn the scooter off by removing the ignition key from the lock.
- Put the neutral handle in the 'scooter in neutral' position.


If the scooter is pushed too quickly, a built-in safety mechanism applies the brakes, causing the scooter to slow down.

5.5.1 The scooter in neutral

In order to enable the Fortress Calypso to be pushed, the automatic parking brake must be disengaged as follows:

- Push the neutral handle (A), which is located under the seat on the right-hand side of the cover, into the NEUTRAL POSITION (C). This disengages the automatic parking brake.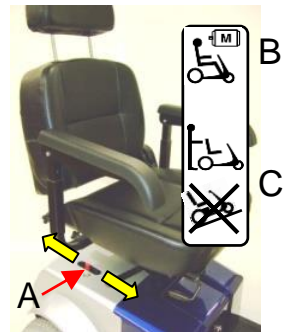

The Fortress Calypso's automatic parking brake can be reapplied by placing the neutral handle back into the BRAKE APPLIED position, (B).

Precautionary measures in neutral

The scooter is equipped with a unique safety feature which prevents the scooter from rolling too quickly when in neutral. If the scooter starts rolling when in neutral, the motor brake automatically brakes the scooter until it comes to a standstill.

- **Ensure that the neutral handle is in the BRAKE APPLIED position before being seated on the scooter.**
- **NEVER touch the neutral handle while driving.**
- **Do not touch the drive lever on the tiller while operating the neutral handle.**

The neutral handle is only to be used when the scooter needs to be pushed. By putting the scooter in neutral, the motor is mechanically disengaged, as a result of which the automatic parking brake no longer works. For this reason it is important for the handle to be put back into the 'Brake applied' position immediately after pushing the scooter, so that the automatic parking brake is re-engaged.

Caution:
NEVER put the scooter in neutral when parked on a slope. The scooter will then roll down the slope due to gravity.

Operation

When the Fortress Calypso is in neutral:

- It is not possible to drive the scooter.
- The electronics can be turned on, but the motor can not drive the scooter.

For this reason the electronics should be switched off in such situations.

The following steps need to be taken in order to be able to drive the Fortress Calypso again:

- Put the neutral handle in the 'Brake applied' position.
- Turn the scooter on by inserting the ignition key in the lock.

You can now drive the Fortress Calypso again.

5.6 Dismantling the scooter for storage and/or transportation

The Fortress Calypso can be dismantled so that it can be stored in a small space or transported in a small car. The scooter takes up less room in its dismantled state. Take the following steps in order to dismantle your Fortress Calypso.

5.6.1 Removing the seat

(seat may vary slightly from the illustration)

The seat is removed as follows:

- Push the release lever forward to release the seat's swivel mechanism.
- Lift the seat off of the seat post.
- The seat can be easily lifted by swivelling it slightly while lifting.
- The size of the seat is reduced by folding the backrest fully forwards, making it easier to lift the seat off of the seat post.

5.6.2 Dismantling the scooter

The Fortress Calypso has been designed to make it fast and easy to dismantle and reassemble without the use of tools. If you have problems in dismantling the scooter or in putting it back together, consult these instructions and then try again.

REMARK: Ensure that power is turned off before dismantling or assembling the scooter.

Dismantling

- Remove the battery cover by loosening the elastic belt on the front of the cover and the Velcro on either side of the battery cover.

- Loosen the rear cover from the rear frame by turning the rotary knob in an anti-clockwise direction and by loosening the Velcro on both sides inside the cover.

- Carefully tip the rear cover towards the rear of the scooter, so that the mechanism becomes visible. Look for the connector plug for the rear lights and disconnect them. The rear cover can now be completely removed.


Operation

- Disconnect the red and black battery cables.

- Undo the belt holding the battery in its place.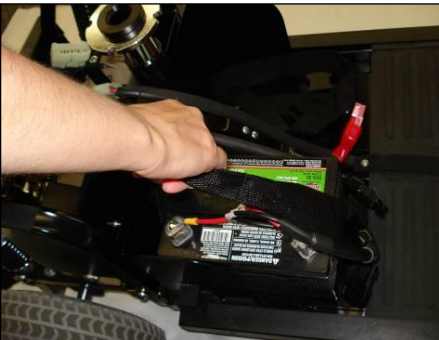

- Remove the batteries by lifting them up using the straps.

CAUTION – the batteries are much heavier than they may seem at first sight!

- Disconnect the main cable before dismantling the scooter's frame components.

- Locate the release clip needed for separating the front frame from the rear frame. This can be found underneath the battery section.
- Press and hold the button to the side of the clip and pull the clip upwards. Release the clip from the hook on the rear frame.

- Take hold of the seat post with one hand and the tiller with the other hand. Now carefully push the seat post backwards until the rear frame of the scooter is resting on the rear bumper.
- Now hold on to the tiller with one hand and take hold of the middle of the front frame platform with the other hand. Lift the front frame upwards until it comes away from the rear frame. Roll the front frame away from the rear frame. While doing so maintain balance between the steering column and the front wheel.


5.6.3 Collapsing the steering column

The steering column can be collapsed as follows:

- Pull handle (A) downwards, at the same time pulling the steering column downwards.

Exercise caution when collapsing the steering column; ensure that your fingers do not become trapped.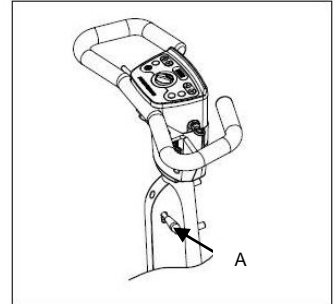

5.6.4 Lifting and loading

- We recommend that you always get someone to help you to load the dismantled scooter into a vehicle.
- When loading the dismantled scooter into a car, ensure that the neutral handle to the rear of the scooter has been set to the “Brake applied” position.
- Once the front of the scooter has been loaded, the batteries can be replaced in the battery compartment and fastened in place.
- You are responsible for ensuring that all of the components of the dismantled scooter are secured in the car and that they will pose no hazard in the event of a collision.

5.6.5 Reassembling the scooter

The above steps can be followed for reassembling the scooter, but then in reverse order. In doing so, attention should be paid to the following points:

- When placing the front frame on the J-hooks of the rear frame, the rear bumper must be resting on the ground.
- The release mechanism and the clip lever must be firmly in place on connecting the front frame to the rear frame.
- The batteries must be firmly held in place by the belts.
- All of the electrical connections must be properly connected up.

5.7 Transporting the scooter in its entirety

The following guidelines should be observed when transporting the Fortress Calypso in its fully assembled state by car suitable for this purpose:


- **If the scooter is to be lifted into the car, never lift the scooter by its plastic covers and do not attempt to lift the scooter on your own. If your scooter fits into a car in a fully assembled state, do NOT use the scooter as a passenger seat in the car. You must be seated in a normal car seat, even if the car in question has been adapted for the transportation of scooters. The reason for this is that the Fortress Calypso can not offer the same level of safety as a normal car seat, irrespective of how well the scooter is secured in the car.**
- **Having positioned the scooter in the car, you must check that the scooter is not in neutral/Free wheel mode.**

- **The scooter must be secured with an approved securement system. Handicare does not issue any recommendations concerning the suitability of specific securement systems. Moreover, Handicare is unable to foresee the situations which may arise when use is made of third party transportation services. For instructions on how to use specific securement systems, please refer to the manufacturer and the user instructions in question. When making use of third party transportation services, contact should be taken up with the service in question.**

The scooter features securement points on the rear and underneath (at the front of the platform). The securement points are exclusively designed for facilitating securement. We would like to point out that any dismantled scooter components which are not secured in a vehicle may cause damage if the vehicle should make any abrupt movements.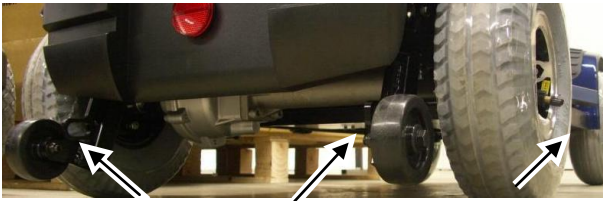

6 Maintenance

6.1 Maintenance table

In order to ensure its best performance, durability and lifespan, your scooter should undergo regular maintenance to be carried out by your dealer.

The following table indicates what checks should be made, how often these checks should be made and who should make them.

Frequency	Description	User
Daily	<ul style="list-style-type: none">• Recharge batteries after each full day of use	X
Weekly	<ul style="list-style-type: none">• Check tyre pressure	X
	<ul style="list-style-type: none">• Check for oil leaks under scooter	X
Monthly	<ul style="list-style-type: none">• Clean scooter (see cleaning procedures)	X
	<ul style="list-style-type: none">• Clean upholstery (if required)	X
Every three months	<ul style="list-style-type: none">• Grease the seat's swivel mechanism: the seat needs to be removed for this. See 'Removing the seat'. Lithium grease is recommended.	X

It is recommended that your dealer carries out a service on your scooter at least once a year. If the scooter is used intensively, the service should be carried out every six months.

In principle, we recommend that you allow your dealer to carry out all required maintenance on your scooter. The maintenance that you will be able to carry out yourself is indicated in the above table.

If you should find traces of oil underneath your scooter, you should immediately notify your dealer of this. In such cases do not drive your scooter.

6.1.1 Batteries

Consult the following documentation on battery maintenance:

Current regulations relating to batteries

The user manual for the battery charger or the instructions on the battery charger

The scooter is equipped with gel batteries. These batteries are fully encased and sealed and require no maintenance.

A sticker indicates how the batteries are to be connected. This sticker is located on the inside of the battery compartment cover.

- **Ensure that the batteries are always well charged.**
- **Do not use the scooter if the batteries are almost flat. Doing so may damage the batteries and means you run the risk of coming to an unexpected standstill.**
- **The use of 'wet' batteries is not permitted. If the batteries need to be replaced, then gel batteries must be used.**

6.1.2 Replacing the batteries

If battery capacity is so low that the scooter can only make short trips, or can not be driven at all, then the batteries have reached the end of their lifespan. Replace the batteries as soon as possible.

Contact your dealer: your dealer will know exactly which batteries are most suitable for your scooter.

The batteries are replaced as follows:

- Turn off the scooter (remove the ignition key from the lock).
- Remove the seat. See 'Removing the seat'.
- Remove the fastener on the front of the battery cover.
- Remove the red and black caps from the battery.
- Loosen the belts which hold the batteries in place by pressing the clasp.
- Unscrew the battery clips from the cables using a 13 mm spanner.
- Remove both batteries.

The new batteries are positioned in reverse order, as follows:

- Position the new batteries.
- Secure the batteries using the belts.
- Connect the cables in accordance with the following table:

No.	Cable colour	Text on label	Connect to
B	Black	Battery 1	Negative terminal on battery 1 (left)
C	Red	Battery 1	Positive terminal on battery 1 (left)
D	Black	Battery 2	Negative terminal on battery 2 (right)
E	Red	Battery 2	Positive terminal on battery 2 (right)

- Tighten the battery clips.

- **Ensure that the cables have been correctly connected to the batteries. The labels indicate which connections are correct.**
- **Connections may NOT be swapped around.**
- **Ensure that the battery terminals and battery clips are fairly clean.**

- Grease the battery clips in order to prevent rusting. Use acid-free Vaseline for this purpose.
- Replace the cover.

- **Once the new batteries have been put in place they will need to be charged. See 'Charging the batteries'.**
- **Avoid metal parts coming into contact with the battery terminals. This could lead to a short circuit which can have serious consequences.**

- See 'Used scooters and the environment' for the correct way to dispose of batteries.

6.1.3 Cleaning the batteries

Gel batteries are maintenance-free. However, attention may be paid to the following matters:

- Ensure that the batteries remain clean and dry: dirt and water can cause leaks, as a result of which battery capacity can be reduced.
- Clean the battery terminals and then grease them using acid-free Vaseline.

Ensure that the batteries are never completely flat. This can seriously damage the batteries and shorten their lifespan.

6.2 Tyres

To ensure that your scooter performs well, it is of great importance for the tyres to be kept at the right pressure.

6.2.1 Inflating the tyres

The tyres are equipped with a car tyre valve. You can have the tyres inflated by your dealer, or at your local petrol station. You can also inflate the tyres yourself using a hand or foot pump. Remove the caps from the valves before inflating the tyres. If the tyres are not adequately inflated, the range of the scooter may be reduced and the tread may wear more quickly. See the 'product specifications' for the correct tyre pressure.

When inflating tyres, never exceed the maximum recommended tyre pressure which is stated on the tyres (3.5 bar).

See the 'Maintenance table' regarding tyre checks.

Never forget to replace the valve cap after inflating the tyres. The valve cap prevents dirt and sand from getting into the valve.

6.3 Cleaning the scooter

Removing dry dirt

The upholstery, metal parts and frame components can usually be cleaned with ease using a soft, dry cloth.

Removing mud and/or other wet dirt

Components which have become dirty due to wet dirt are best removed by first cleaning with a damp cloth and then with a soft, dry cloth.

Upholstery

Cleaning with a damp cloth. Once the dirt has been removed, the parts which have been cleaned need to be dried using a soft, dry cloth.

- **Never use caustic cleaning products. These can be damaging to the scooter. Do not use organic solvents, such as thinners, chemical-cleaning naphtha or methylated spirits.**
- **Do not spray or pour water onto your scooter.**
- **Exercise caution when using water near to the electronic system.**
- **Upholstery: do not chemically clean, iron or spin-dry.**

7 Troubleshooting

If your Fortress Calypso is not working even though the batteries are fully charged, the following checks can be carried out before contacting your dealer.

1. Check whether all of the battery clips are firmly in place.
2. Check whether the neutral handle is in the 'BRAKE APPLIED' position (and not in the neutral position).
3. Check whether the automatic circuit breaker lever is in the tripped position. If it is, reset the circuit breaker by pressing the lever back into position.
If the circuit breaker trips again, contact your dealer.

7.1 Troubleshooting table

If your scooter is not working, or is not working as it should, go through the following list of possible problems before contacting your dealer. You may be able to solve the problem yourself.

Possible cause	Action	Action to be taken by:
<ul style="list-style-type: none">• The ignition key has been incorrectly inserted in the lock.	<ul style="list-style-type: none">• Insert the ignition key in the lock correctly.	User
<ul style="list-style-type: none">• The connector cables in the motor compartment are not properly connected to the control mechanism, or are loose.	<ul style="list-style-type: none">• Check the connector cables and ensure that they are properly connected to the control mechanism.	Dealer
<ul style="list-style-type: none">• The battery clips are not properly attached.	<ul style="list-style-type: none">• Check the battery clips.	User
<ul style="list-style-type: none">• The scooter is in neutral.	<ul style="list-style-type: none">• Move the neutral handle to the BRAKE APPLIED position and remove the ignition key from the lock. Reinsert the ignition key to restart the scooter.	User
<ul style="list-style-type: none">• The automatic circuit breaker has tripped.	<ul style="list-style-type: none">• See 'Circuit breaker'• Remove the ignition key from the lock.• Press the automatic circuit breaker back into position and restart the scooter using the ignition key.	User
<ul style="list-style-type: none">• There is a loose connection.	<ul style="list-style-type: none">• Solve the connection problem if possible, or contact your dealer.	User + Dealer

Possible cause	Action	Action to be taken by:
<ul style="list-style-type: none"> The battery voltage is too low. 	<ul style="list-style-type: none"> Check the battery voltage. See 'Battery indicator'. Recharge the batteries for eight hours. Check whether the battery charger is working properly. 	User
<ul style="list-style-type: none"> The scooter drives too slowly. 	<ul style="list-style-type: none"> Turn the speed regulator knob to the right. 	User
<ul style="list-style-type: none"> The motor has overheated. 	<ul style="list-style-type: none"> Remove the ignition key from the lock and allow the scooter to cool down. 	User

Problems can sometimes be solved by turning the scooter off and then turning it back on again.

If you are not able to solve a problem by consulting the above list, then you should contact your dealer.

7.2 Automatic circuit breaker

The scooter is equipped with an automatic circuit breaker (A).

Under normal circumstances the automatic circuit breaker is not tripped and the scooter is ready for use. In the event of overload or of a short circuit, the automatic circuit breaker switches off the electrical system in order to prevent damage.

Overload can occur if the motor becomes overheated. If the automatic circuit breaker switches off the scooter's electrical circuit, the circuit breaker trips.

The automatic circuit breaker is located on the rear cover, under the seat.

If you press the automatic circuit breaker lever back into position, you can then try restarting the scooter. If the automatic circuit breaker trips again, wait for 15-20 minutes so that the motor can cool down, and then try pressing the circuit breaker lever back into position again.

If it is possible to restart the scooter after following this procedure, then it is no problem to carry on driving. However, take care to avoid obstacles and hills.

If the automatic circuit breaker trips again once the motor has cooled down, then there may be a short circuit. In such cases, do not attempt to drive the scooter. Instead you must contact your dealer.

7.3 Electromagnetic interference

Radio waves can have a negative effect on the capacity of an electric scooter. The scooter itself can also interfere with the operation of electromagnetic fields, such as alarm systems or electronic circuits in shops.

Powered wheelchairs and scooters can be affected by sources of radio waves, such as radio and TV transmitters, amateur radio stations, lifts, transmitting equipment, stereo radios and mobile telephones.

The following advices are intended to prevent the electric scooter from being driven unintentionally, which could lead to serious injury.

1. Do not turn on any manually controlled equipment for personal communication, such as a radio transmitter or mobile telephone, while the electric scooter is switched on.
2. Keep at a distance from transmitter equipment, such as radio and TV stations.
3. If the scooter starts making unintended movements, or if the brake goes into neutral, turn off the powered wheelchair or scooter by removing the ignition key from the lock.
4. Be aware of the fact that any accessories, components or adaptations installed on the scooter may increase the effect of radio waves.

Remarks:

There is no easy way of testing the effects of radio waves on the general immunity of powered wheelchairs or scooter.

5. All cases of unintended movements or of the scooter spontaneously going into neutral should be reported to your dealer or to the scooter manufacturer. When reporting, please indicate whether a source of radio waves was nearby at the time.

Remarks:

The Fortress Calypso's immunity level is 21 volt/metre.

8 Technical specifications

Handicare B.V.:	Vossenbeemd 104 5705 CL, Helmond The Netherlands
-----------------	--

8.1.1 Fortress Calypso product specifications

Model	Fortress Calypso 3 wheels (3W) Fortress Calypso 4 wheels (4W)
Maximum user weight	125 kg (19.7 stone)

Description		3W	4W
Total length	mm (inches)	1207 (47.5)	1214 (47.8)
Total width	mm (inches)	622 (24.5)	622 (24.5)
Minimum height*	mm (inches)	710 (28)	710 (28)
Total weight excluding batteries:	kg (pounds)	70 (154.3)	75 (165.3)
Total weight including batteries:	kg (pounds)	100 (220.5)	105 (231.5)
Weight of heaviest component	kg (pounds)	27 (59.5)	32 (70.5)
Static stability in the downward direction	°	15 (27%)	15 (27%)
Static stability in the upward direction	°	15 (27%)	15 (27%)
Lateral static stability	°	15 (27%)	15 (27%)
Distance range (ISO 7176-4)**	km (miles)	40 (24.9)	40 (24.9)
Dynamic stability (max. safe slope)	°	9 (16%)	9 (16%)
Maximum speed forwards	km/h (mph)	10 (6)	10 (6)
Back angle, Captain Seat	°	0-103-126	0-103-126
Seat depth, Captain Seat	mm (inches)	445 (17.5)	445 (17.5)
Seat width, Captain Seat	mm (inches)	457 (18)	457 (18)
Back height (excl. headrest), Captain Seat	mm (inches)	406 (16)	406 (16)
Seat height (to platform), Captain Seat	mm (inches)	432-490 (17-19.3)	432-490 (17-19.3)
Armrest height, Captain Seat	mm (inches)	229-273 (9-10.8)	229-273 (9-10.8)
Distance between armrests, Captain Seat	mm (inches)	457-610 (18-24)	457-610 (18-24)
Back angle, Master Seat	°	85-180	85-180
Seat depth, Master Seat	mm (inches)	430 (16.9)	430 (16.9)
Seat width, Master Seat	mm (inches)	500 (19.7)	500 (19.7)
Back height (excl. headrest), Master Seat	mm (inches)	550 (21.7)	550 (21.7)
Seat height (to platform), Master Seat	mm (inches)	432-510 (17-20.1)	432-510 (17-20.1)
Armrest height, Master Seat	mm (inches)	150-340 (5.9-13.4)	150-340 (5.9-13.4)
Distance between armrests, Master Seat	mm (inches)	500 (19.7)	500 (19.7)

Technical specifications

Description		3W	4W
Turning radius (ISO 7176-5)	mm (inches)	1170 (46.1)	1600 (63)
Obstacle height (max. user weight)	mm (inches)	76 (3)	76 (3)
Ground clearance (max. user weight)	mm (inches)	101 (4)	101 (4)

Test data	
Test weight	125 kg (19.7 stone)

Operating force	
Drive lever operation	< 60 N
Resetting the automatic fuse (circuit breaker)	< 60 N
Electronic switches	< 15 N
Connecting the charger plug	< 60 N

Technical specifications for wheels		
Front wheel diameter (3W)	mm (inches)	260 x 85 (10.2 x 3.3)
Front wheel diameter (4W)	mm (inches)	260 x 85 (10.2 x 3.3)
Rear wheel diameter	mm (inches)	260 x 85 (10.2 x 3.3)
Tyre pressure 4W, front wheel	bar (psi)	3.5 (50)
Tyre pressure 4W, rear wheel	bar (psi)	3.5 (50)
Tyre pressure 3W, front wheel	bar (psi)	3.5 (50)
Tyre pressure 3W, rear wheel	bar (psi)	3.5 (50)

Batteries		3W	4W
Maximum dimensions of batteries (lxbxh)	mm (inches)	197x168x175 (7.8x6.6x6.9)	197x168x175 (7.8x6.6x6.9)
Max. battery capacity	Ah	60 (C20)	60 (C20)
Maximum permitted charging current	Ampère	8	8

* height of steering column, collapsed, excluding seat

** range dependent on user weight, condition of tyres, type of terrain, condition of battery and weather conditions.

8.2 Authorised service and technical support

In the event of problems or questions relating to this product, please contact your dealer.

Handicare can be contacted for information regarding dealers in your area.

Handicare:	Handicare B.V. Vossenbeemd 104 5705 CL Helmond The Netherlands
------------	---

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