

Sparky

UK

AGM# 088 609 661

Illustrated *P*arts *B*reakdown

NOTA IMPORTANTE:
INDICAR SIEMPRE MODELO Y NUMERO DE SERIE



Pride®

Your new Pride Illustrated Parts Breakdown (IPB) Manual has been compiled from the latest product information available at the time of publication. Pride[®] Mobility Products reserves the right to make changes and updates to this manual and to the product(s) it represents. Any changes, updates, or improvements to the product(s) may result in slight differences between this manual and the product you purchased.

An IPB is an illustrated parts listing of all product components which have maintenance significance. IPB's utilize "exploded" drawings to detail the individual components for each assembly and an itemized list of reference numbers, part numbers, descriptions, and quantities per assembly. Some parts may be included in an assembly but may not be available for purchase separately. Refer to the parts list to locate the part numbers for assemblies and components for ordering what you need.

To achieve proper identification and relationship of detail parts and next higher assemblies, parts listings are presented in a convenient alphanumeric reference system. Reference numbers for parent assemblies identify one part number that can be used to order all of the components listed within it. Components of an assembly will have the same alpha character as its parent assembly.

- For example, the parent assembly part number using reference number of A1 — A4 is the only number you need to place an order for parts with references of A1, A2, A3, and A4. Instead of ordering 4 separate parts, use the parent assembly part number to get the complete assembly.

Master assembly part numbers have reference numbers that include several subassemblies. By ordering parts using master assembly part numbers, you can save a lot of time and reduce the chance for error.

- For example, A1 — G10, a master assembly reference number, includes all components of assemblies A, B, C, D, E, F, and G. Use caution when ordering assemblies. Check to see if a master assembly part number exists for the components that you need. If an entire master assembly is not needed, then the individual assembly part number (i.e. for reference number G1 — G10) will be sufficient to use for ordering.

Asterisks (*) and NOTES are used throughout this service manual to point out important information regarding assembly structure and contents.

- For example, in some illustrations, only the left OR right side is shown. In other drawings, only the front OR back is depicted. The part numbers for ALL components are listed. An asterisk will also be used to indicate which components are not illustrated. Illustrated items without callouts are shown for assembly purposes.

Updates to the contents of this IPB will be made on a regular basis. Providers may retrieve updates immediately upon their release by going to the Pride Service Website. The web address is www.prideservice.com. We welcome any questions or comments regarding the new Pride IPB Manual that you may have.

Thank you for choosing products from Pride[®] Mobility Products.

SECTION I.

Main Frame and Battery Box Assembly	4
---	---

SECTION II.

Caster Assembly	6
-----------------------	---

SECTION III.

Anti-tip and Suspension Assembly	8
--	---

SECTION IV.

Drive Motor Assembly	10
----------------------------	----

SECTION V.

Electronics Assembly	12
----------------------------	----

SECTION VI.

Controller Assembly	14
---------------------------	----

SECTION VII.

Charger Assembly	15
------------------------	----

SECTION VIII.

Legrest Assembly	
Pediatric Swing-away	16
ELR 4" x 6" Footplate	18
Legrest Hangers	20

SECTION IX.

Drive Wheel Assembly	21
----------------------------	----

SECTION X.

Actuator Assembly	
Manual Recline	22
Power Tilt	24

SECTION XI.

Seat Assembly	
Seat Frame	26
Seat Back, Tension Adjustable	28
Seat Interface	30
Seat Cushion	32

SECTION XII.

Headrest Assembly	34
-------------------------	----

SECTION XIII.

Armrest Assembly	36
------------------------	----


SECTION XIV.

Owner's and Decal Packages	38
----------------------------------	----

APPENDIX A.

Electrical System Diagram	39
---------------------------------	----

Pride®
Mobility Products Corp.

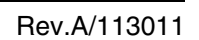


I. Main Frame and Battery Box Assembly

REF #	PART #	DESCRIPTION	QTY
A1 - A7	FRMASMB17501	FRAME,ASSY,MAIN,W/SIDE RAILS,WHITE,SPARKY	1
A1	FRMINDV1614	FRAME,WELDMENT,MAIN,BLACK,(ANI: PD10D-A-008-T)	1
A2	FRMINDV1615	FRAME,WELDMENT,SIDE RAILS,WHITE,(ANI: PD10D-A-007-T)	1
A3	SCRSHFH1166	SCREW,SHFH,BLACK ZINC,M8-1.25 X 24MM,W/NYLON PATCH	8
A4	WASMETR1029	WASHER,FLAT,BLACK ZINC,8.4MM ID X 16MM OD X 1.6MM TH,METRIC,GB96-8	8
A5	SCRASBH1280	SCREW,BHSCS,BLACK ZINC,M8-1.25 X 38MM,W/NYLON PATCH	2
A6	WASMETR1142	WASHER,FLAT,BLACK ZINC,8.4MM ID X 26MM OD X 3MM TH	2
A7	NUTNYLK1011	NUT,NYLOCK,BLACK ZINC,M8-1.25	10
B1 - B4	FRMASMB17502	FRAME,ASSY,BATTERY TRAY,50AH BATTERIES,SPARKY	1
B1	FRMINDV1616	FRAME,WELDMENT,BATTERY TRAY,BLACK,(ANI: OS12S-A-006-C)	1
B2	SCRSHCS1096	SCREW,SHCS,BLACK ZINC,M4-.7 X 20MM	2
B3	SCRASBH1289	SCREW,BHSCS,BLACK ZINC,M8-1.25 X 35MM	1
B4	NUTNYLK1011	NUT,NYLOCK,BLACK ZINC,M8-1.25	1
C1 - C5	PLSASMB2201	BATTERY BOX,ASSY,50AH BATTERIES,SPARKY	1
C1	PLSASMB2202	BATTERY BOX,LID,ASSY,50AH BATTERIES,SPARKY,(ANI: OS12S-P-059-C)	1
C2	ELEASMB7000	ELECTRONIC,ASSY,CIRCUIT BREAKER,50 AMP,STRAIGHT TERMINALS,W/PROTECTIVE COVER,(ANI: DE16S-P-116-C)	1
*C3	INDPART2148	COVER,12 X 1.25T,BREAKER BUTTON, W/NUT	1
C4	PLSINDV1314	BATTERY BOX,BASE,50AH BATTERIES,SPARKY,(ANI: OS12S-P-060-C)	1
C5	ACCSTRP1076	SECUREMENT STRAP,BATTERY BOX,BLACK,(ANI: OS12S-P-071-C)	1

Notes :

*C3 The item is a quantity of one but is illustrated as two parts for assembly purposes.

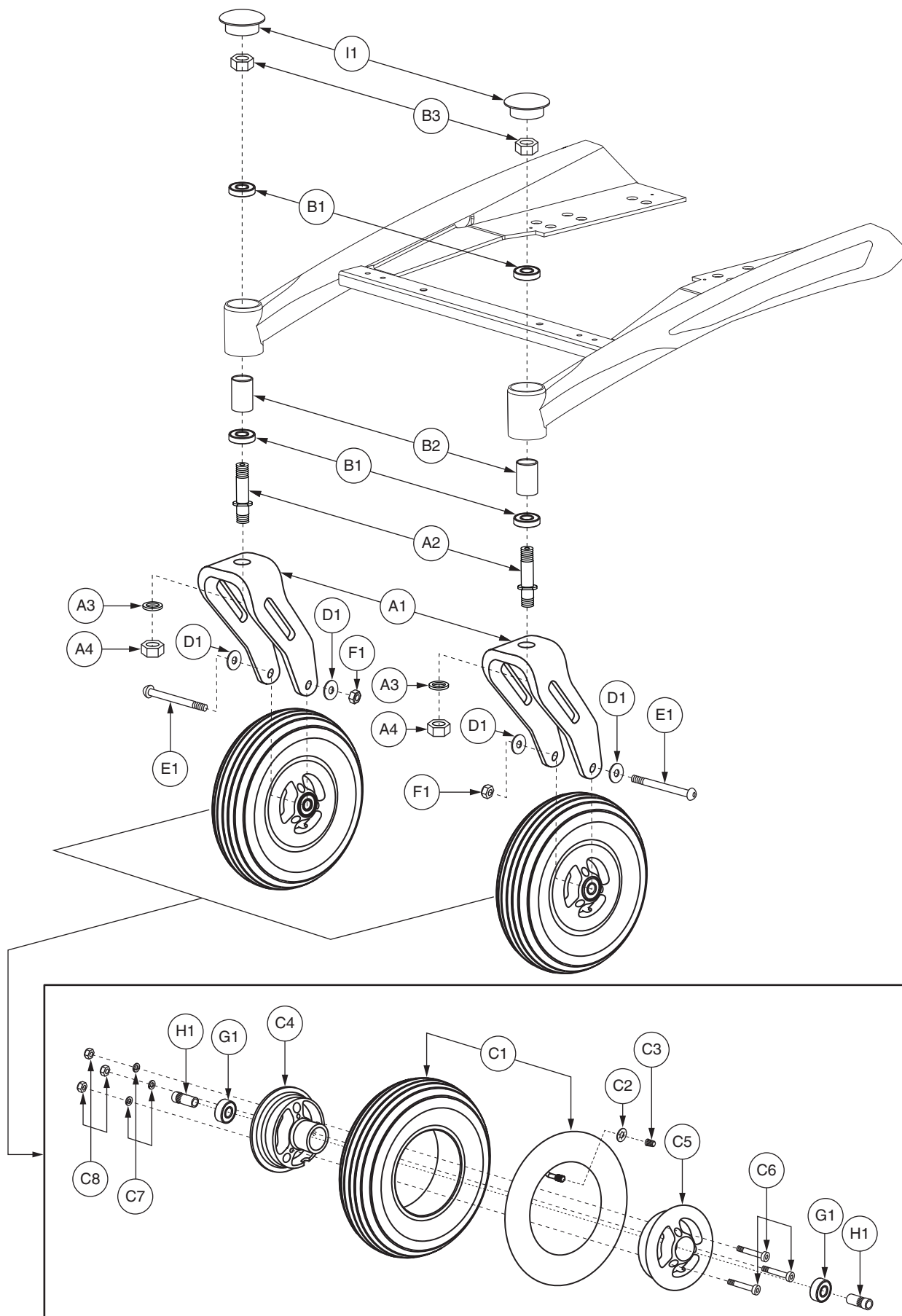


II. Caster Assembly

REF #	PART #	DESCRIPTION	QTY
A1 - I1	FRMASMB17504	CASTER,ASSY,PNEUMATIC,8" WHEEL,W/FORK,SPARKY,PAIR	1
*A1 - I1	FRMASMB17506	CASTER,ASSY,PNEUMATIC,8" WHEEL,W/FORK,SPARKY,RIGHT	1
*A1 - I1	FRMASMB17505	CASTER,ASSY,PNEUMATIC,8" WHEEL,W/FORK,SPARKY,LEFT	1
A1 - A4	FRMASMB17503	FORK,ASSY,CASTER,8",SPARKY	1
A1	FRMINDV1617	FORK,WELDMENT,8",CASTER,(ANI: DE16S-E-007-C)	1
A2	HDWINDV1255	PIVOT STUD,CASTER FORK,ZINC,M12-1.25(TOP),M12-1.25(BOTTOM),(ANI: TD16S-P-121-C)	1
A3	WASMETR1162	WASHER,FLAT,ZINC,12.2MM ID X 24MM OD X 1.5MM TH	1
A4	NUTNJAM1033	NUT,NYLOCK JAM,ZINC,M12-1.25	1
B1 - B3	HDWASMB1186	ANTI-FLUTTER,ASSY,CASTER WHEEL,SPARKY	1
B1	BRGSLV1035	BEARING,BALL,12MM ID X 28MM OD X 8MM TH,6001RS	2
B2	SPCINDV1005	SPACER,ZINC,STEEL,13.5MM ID X 16MM OD X 36MM TH,(ANI: TD16S-P-120-C)	1
B3	NUTNYLK1066	NUT,NYLOCK,BLACK ZINC,M12-1.25,GRADE 8,(MTS: 53212007)	1
C1 - C8	WHLASMB2228	WHEEL,ASSY,PNEUMATIC,SILVER RIM/BLACK TIRE,8"	1
C1	TIRPNEU1090	TIRE,PNEUMATIC,BLACK,8",200 X 50,W/TUBE & VALVE CAP,(ANI: OS12S-P-008-C)	1
C2	DWR7700A010	SLIP COVER,VALVE STEM,35PSI,2.4 BAR,MAX RATING,A-7700-010	1
C3	PLSINDV1003	CAP,BLACK,VALVE	1
C4	WHLRIMS1250	WHEEL,RIM,SILVER,INNER,(ANI: OS12S-P-074-C)	1
C5	WHLRIMS1251	WHEEL,RIM,SILVER,OUTER,(ANI: OS12S-P-073-C)	1
C6	SCRSHCS1048	SCREW,SHCS,ZINC,M6-1.0 X 20MM	3
C7	WASLOCK1003	WASHER,LOCK,ZINC,M6	3
C8	NUTNYLK1045	NUT,NYLOCK,ZINC,M6-1.0	3
D1	WASMETR1055	WASHER,FLAT,BLACK ZINC,8.2MM ID X 16MM OD X 1.0 MM TH,(MTS: 53108008)	2
E1	SCRASBH1317	SCREW,BHSCS,BLACK ZINC,M8-1.25 X 86MM	1
F1	NUTNYLK1011	NUT,NYLOCK,BLACK ZINC,M8-1.25	1
G1	BRGSLV1035	BEARING,BALL,12MM ID X 28MM OD X 8MM TH,6001RS	2
H1	BSHFLNG1086	SPACER,STEEL,ZINC,8MM ID X 12MM OD X 31MM TH X 14MM FOD X 18MM FTH,KNURLED,(ANI: DE16S-P-126-C)	2
I1	PLSINDV1315	COVER,BLACK,1.25" DIA,CASTER CAP,FITS 1.125" DIA TUBE,(ANI: TD16S-P-127-C)	1

Notes :

*A1 - I1 Both assemblies contain identical parts. There are left and right sides for assembly purposes.


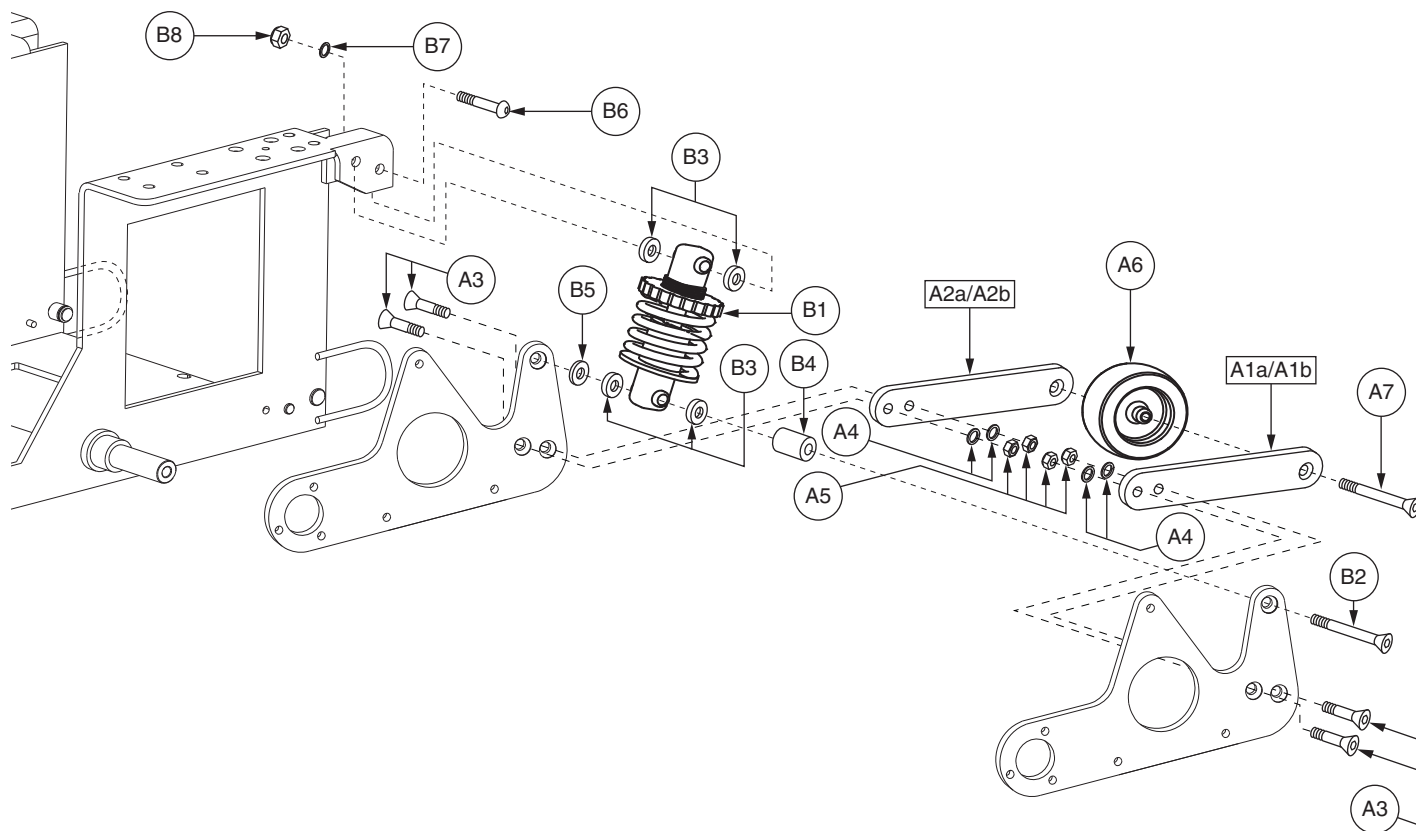


III. Anti-tip and Suspension Assembly

REF #	PART #	DESCRIPTION	QTY
A1 - A7	FRMASMB17507	ANTI-TIP,ASSY,SPARKY,PAIR	1
A1a - A7	FRMASMB17509	ANTI-TIP,ASSY,W/HARDWARE,SPARKY,RIGHT	1
A1b - A7	FRMASMB17508	ANTI-TIP,ASSY,W/HARDWARE,SPARKY,LEFT	1
A1a	FRMINDV1620	ANTI-TIP,WELDMENT,OUTSIDE,RIGHT,R40 PED,(ANI: PD10D-P-082-T)	1
A1b	FRMINDV1618	ANTI-TIP,WELDMENT,OUTSIDE,LEFT,R40 PED,(ANI: PD10D-P-080-T)	1
A2a	FRMINDV1621	ANTI-TIP,WELDMENT,INSIDE,RIGHT,R40 PED,(ANI: PD10D-P-083-T)	1
A2b	FRMINDV1619	ANTI-TIP,WELDMENT,INSIDE,LEFT,R40 PED,(ANI: PD10D-P-081-T)	1
A3	SCRSHFH1170	SCREW,SHFH,BLACK ZINC,M8-1.25 X 25MM,W/NYLON PATCH	4
A4	WASMETR1143	WASHER,FLAT,BLACK ZINC,8.4MM ID X 14MM OD X 1MM TH	4
A5	NUTNYLK1011	NUT,NYLOCK,BLACK ZINC,M8-1.25	4
A6	WHLATIP1017	WHEEL,ANTI-TIP,BLACK RIM/GRAY TIRE,3",(ANI: DE16D-P-101-T)	1
A7	SCRSHFH1174	SCREW,SHFH,BLACK ZINC,M8-1.25 X 45MM,W/NYLON PATCH	1
*B1 - B8	FRMASMB16112	SUSPENSION,ASSY,STRUT,REAR,QUANTUMR40	2
B1	FRMASMB16096	STRUT WELDMENT,W/BUSHINGS & BLACK SPRING,250 LBS/IN,(ANI: DE16D-P-018-C)	1
B2	SCRSHFH1172	SCREW,SHFH,BLACK ZINC,M8-1.25 X 60MM,W/NYLON PATCH	1
B3	WASNYLN1166	WASHER,WHITE,NYLON,11.5MM ID X 18MM OD X 5MM TH	4
B4	SPCINDV1003	SPACER,BLACK,ALUMINUM,8MM ID X 17MM OD X 16MM TH	1
B5	WASMETR1144	WASHER,FLAT,BLACK ZINC,8.4MM ID X 17MM OD X 7MM TH	1
B6	SCRASBH1283	SCREW,BHSCS,BLACK ZINC,M8-1.25 X 45MM,W/NYLON PATCH	1
B7	WASMETR1143	WASHER,FLAT,BLACK ZINC,8.4MM ID X 14MM OD X 1MM TH	1
B8	NUTNYLK1011	NUT,NYLOCK,BLACK ZINC,M8-1.25	1

Notes :

Top-level assemblies will include components designated with corresponding lower-case letters. Parts without lower-case letters will be included in all top-level assemblies.

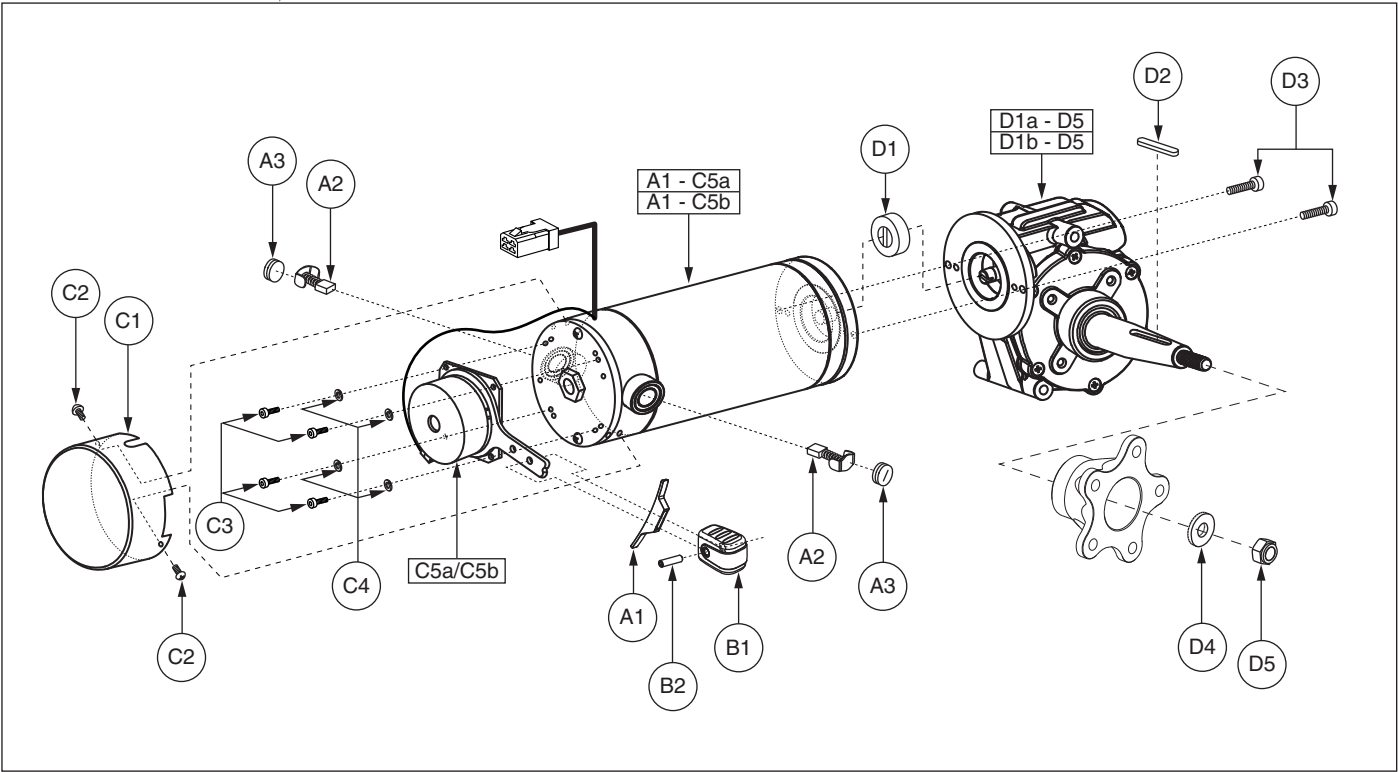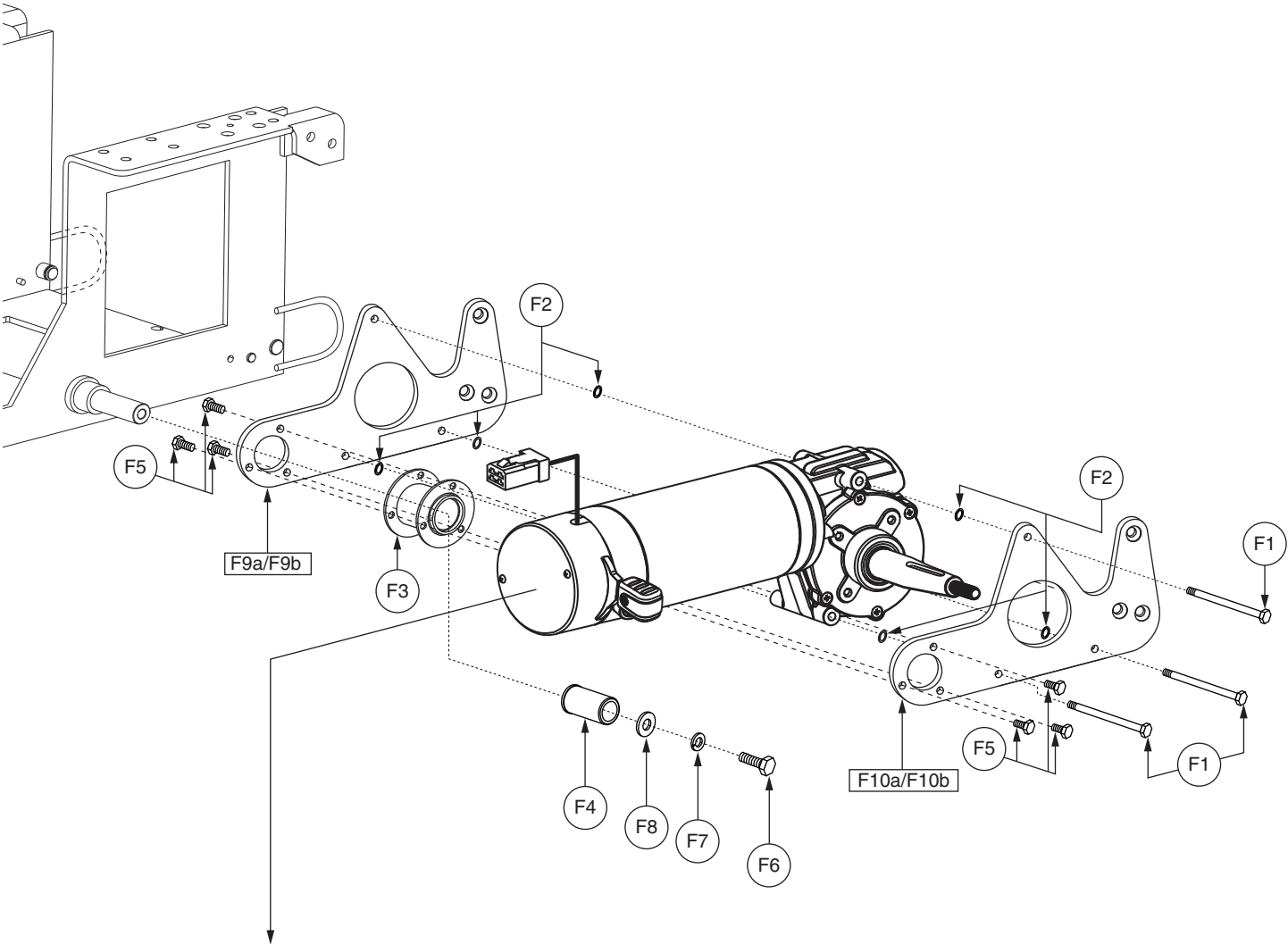


IV. Drive Motor Assembly

REF #	PART #	DESCRIPTION	QTY
A1 - F10	DRVASMB7110002	MOTOR,ASSY,DRIVE,6KMH,350W,STRAIGHT SHAFT,W/GEARBOX & INDIVIDUAL BRAKE RELEASE,PAIR,SPARKY,VR2	1
A1 - F10a	DRVASMB7110004	MOTOR,ASSY,DRIVE,6KMH,350W,STRAIGHT SHAFT,INDIVIDUAL BRAKE LEVER,W/MOTOR MOUNT,RIGHT,SPARKY,VR2	1
A1 - F10b	DRVASMB7110003	MOTOR,ASSY,DRIVE,6KMH,350W,STRAIGHT SHAFT,INDIVIDUAL BRAKE LEVER,W/MOTOR MOUNT,LEFT,SPARKY,VR2	1
A1 - E1a	DRVASMB2495	MOTOR,ASSY,DRIVE,6KMH,350W,STRAIGHT SHAFT,W/GEARBOX,INDIVIDUAL BRAKE RELEASE,RIGHT,VR2 CONNECTOR,H-1210-108,(CTE: 07T9AI015)	1
A1 - E1b	DRVASMB2494	MOTOR,ASSY,DRIVE,6KMH,350W,STRAIGHT SHAFT,W/GEARBOX,INDIVIDUAL BRAKE RELEASE,LEFT,VR2 CONNECTOR,H-1210-107,(CTE: 07T9AI016)	1
A1 - C5a	DRVMOTR1456	MOTOR,DRIVE,6KMH,350W,W/BRAKE,RIGHT,VR2,H-1210-058,(CTE: 08TPR0003)	1
A1 - C5b	DRVMOTR1455	MOTOR,DRIVE,6KMH,350W,W/BRAKE,LEFT,VR2,H-1210-057,(CTE: 08TPR0004)	1
A1	DWR1215D001	LEVER,BRAKE SLOT SHIELD,D-1215-001	1
A2	DRVBRSH1037	MOTOR,BRUSH,350W,(CTE: 212440701)	2
A3	PLSINDV1286	BRUSH CAP,MOTOR,350W,(CTE: 212400701)	2
B1 - B2	FRMASMB8691	FREEWHEEL,ASSY,RELEASE HANDLE GRIP,YELLOW,B-1160-051	1
B1	DWR1160C017	HANDLE,GRIP,YELLOW,C-1160-017	1
B2	SCRSETS1099	SCREW,SET,BLACK ZINC,M4-.7 X 16MM	1
C1 - C5a	DRVASMB2343	MOTOR,ASSY,BRAKE,INDIVIDUAL RELEASE,RIGHT,QUANTUM R40	1
C1 - C5b	DRVASMB2249	MOTOR,ASSY,BRAKE,INDIVIDUAL RELEASE,LEFT,QUANTUM R40	1
C1	INDPART2839	CAP,MOTOR,BRAKE,BLACK,INDIVIDUAL RELEASE,(CTE: 107960002)	1
C2	SCRCPH1003	SCREW,CPH,BLACK ZINC,M4-.7 X 6MM	2
C3	SCRSHCS1309	SCREW,SHCS,ZINC,M4-.7 X 10MM	4
C4	WASLOCK1086	WASHER,LOCK,SPLIT,ZINC,M4	4
C5a	DRV1069	BRAKE,ASSY,24VDC,9.2W,RIGHT,(ELECTROCRAFT: 0002-9425)	1
C5b	DRV1068	BRAKE,ASSY,24VDC,9.2W,LEFT,(ELECTROCRAFT: 0002-9426)	1
D1a - D5	DRVASMB2453	MOTOR,ASSY,GEARBOX,STRAIGHT 65MM SHAFT,RIGHT,25:1 GEAR RATIO,H-1210-094,(CTE: 02TAI3001)	1
D1b - D5	DRVASMB2452	MOTOR,ASSY,GEARBOX,STRAIGHT 65MM SHAFT,LEFT,25:1 GEAR RATIO,H-1210-093,(CTE: 02TAI3002)	1
D1	INDPART2840	MOTOR COUPLER,(CTE: 107700003)	1
D2	KEYSTL1025	AXLE KEY,STEEL,3/16" X 3/16" X 29MM,(CTE: 204123010)	1
D3	SCRSHCS1048	SCREW,SHCS,ZINC,M6-1.0 X 20MM	2
D4	WASSTND1148	WASHER,FLAT,ZINC,12.80MM ID X 27.80MM OD X 2.00MM TH	1
D5	NUTNYLK1057	NUT,NYLOCK,ZINC,M10-1.5	1
*E1a	DWR1235L870	DECAL,MOTOR PART NUMBER BAR CODE,DRVASMB2495,RIGHT,L-1235-870	1
*E1b	DWR1235L869	DECAL,MOTOR PART NUMBER BAR CODE,DRVASMB2494,LEFT,L-1235-869	1
F1 - F10a	FRMASMB16132	MOTOR MOUNT,ASSY,W/HARDWARE,RIGHT,QUANTUMR40	1
F1 - F10b	FRMASMB16131	MOTOR MOUNT,ASSY,W/HARDWARE,LEFT,QUANTUMR40	1
F1	SCRHHCS1675	SCREW,HHCS,BLACK ZINC,M6-1.0 X 58MM	3
F2	WASMETR1036	WASHER,FLAT,BLACK ZINC,6.35MM ID X 13MM OD X 1MM TH,METRIC	6
F3	FRMINDV1463	FRAME,WELDMENT,MOTOR PIVOT SPACER,BLACK,(ANI: PG16D-A-012-T)	1
F4	BSHFLNG1075	BUSHING,FLANGE,BRONZE,18.00MM ID X 24.75MM OD X 49.00MM TH X 25.90MM FOD X 2.50MM FTH,B-1010-205	1
F5	SCRHHCS1676	SCREW,HHCS,ZINC,M6-1.0 X 15MM,W/NYLON PATCH	6
F6	SCRHHCS1190	SCREW,HHCS,ZINC,M8-1.25 X 20MM,GRADE 8.8	1
F7	WASLOCK1039	WASHER,LOCK,BLACK ZINC,M8,SPLIT	1
F8	WASMETR1142	WASHER,FLAT,BLACK ZINC,8.4MM ID X 26MM OD X 3MM TH	1
F9a	FRMINDV1470	FRAME,MOTOR MOUNT,RIGHT INNER,(ANI: PD16D-P-028-C-R)	1
F9b	FRMINDV1468	FRAME,MOTOR MOUNT,LEFT INNER,(ANI: PD16D-P-028-C-L)	1
F10a	FRMINDV1471	FRAME,MOTOR MOUNT,RIGHT OUTER,(ANI: PD16D-P-027-C-R)	1
F10b	FRMINDV1469	FRAME,MOTOR MOUNT,LEFT OUTER,(ANI: PD16D-P-027-C-L)	1

Notes :


All items with lower-case letters represent right/left specific components. For all items referenced with an "***", items are not shown.

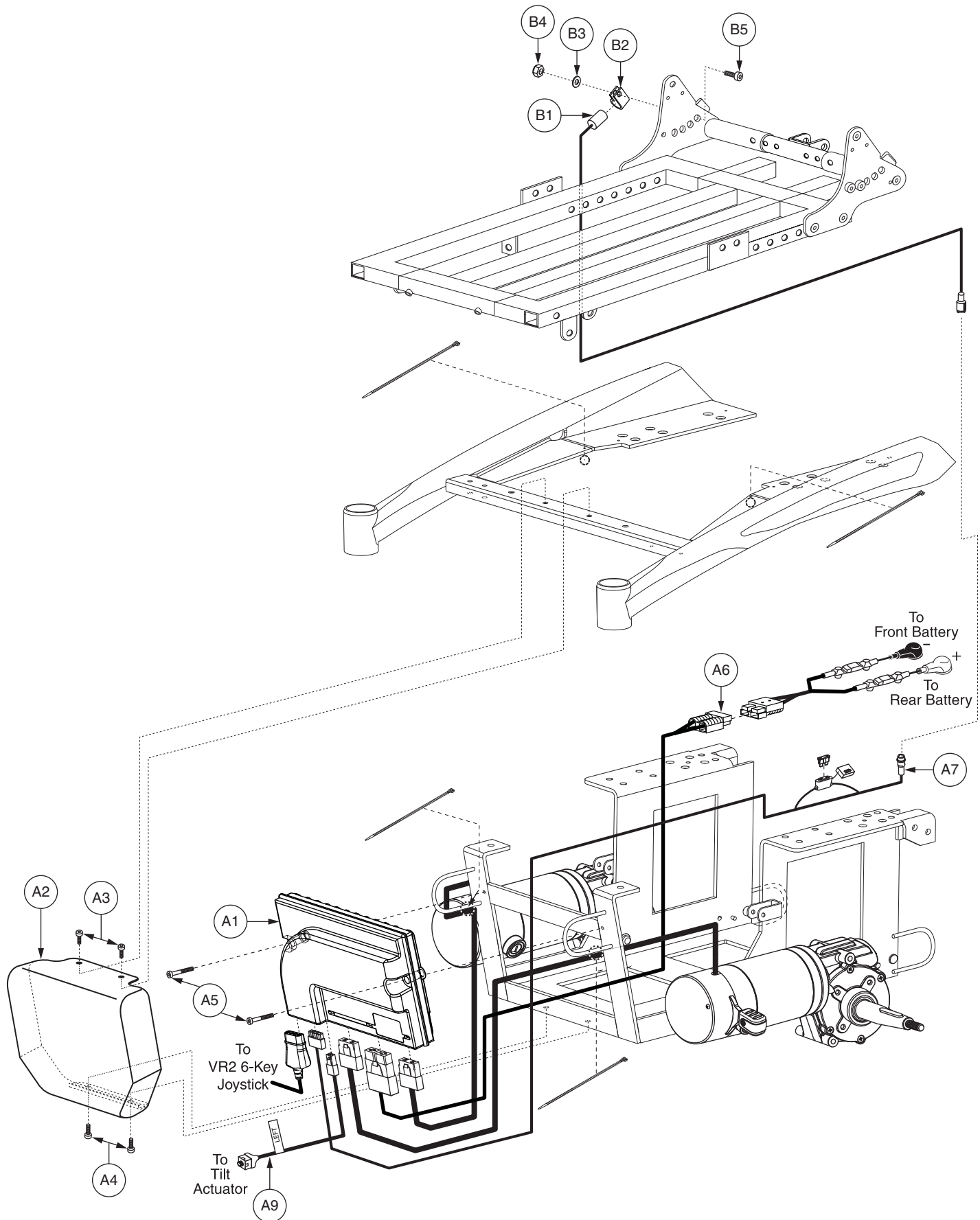


REF #	PART #	DESCRIPTION	QTY
A1 - A9	ELEASMB7110025	ELECTRONIC,ASSY,VR2,70 AMP,6KMH,350W MOTORS,SPARKY,EURO 1 ACTUATOR	1
A1	CTLASMB7110015	CONTROLLER,VR2,ASSY,70 AMP,W/6KMH,350W PROGRAM,SPARKY,1 ACTUATOR	1
A2	PLSINDV1316	SHROUD,BLACK,POWER MODULE COVER,PEDIATRIC,(ANI: PD10D-P-097-T)	1
A3	SCRSHCS1092	SCREW,SHCS,BLACK ZINC,M4-.7 X 12MM	2
A4	SCRSHCS1196	SCREW,SHCS,BLACK ZINC,M4-.7 X 16MM	2
A5	SCRSHCS1314	SCREW,SHCS,ZINC,M4-.7 X 30MM,GRADE 8.8	2
A6	DWR1056H023	ELECTRONIC,HARNESS,POWER CABLE,VR2,JET3ULTRA/600 SERIES,H-1056-023	1
A7	DWR1265H030	ELECTRONIC,HARNESS,POWER TAKE OFF(PTO)/INHIBIT,VR2,H-1265-030	1
*A8	DWR1361H038	ELECTRONIC,HARNESS,RUN PLUG,SURE SEAL,SYNERGY TILT,H-1361-038	1
A9	HAR8110003	HARNESS,TILT ACTUATOR,VR2,SPARKY,(ANI: PD10D-P-105-T)	1
B1 - B5	ELEASMB7385	ELECTRONIC,ASSY,VR2,TILT ONLY,SPARKY SEAT	1
B1	DWR1265H035	ELECTRONIC,HARNESS,MERCURY SWITCH,HIGH AMP TOGGLE,VR2,H-1265-035	1
B2	CLPCBLC1025	CLAMP,CABLE,STEEL,0.50" DIA,RUBBER COATED	1
B3	WASSTND1205	WASHER,FLAT,BLACK ZINC,8.2MM ID X 14MM OD X 1MM TH	1
B4	NUTNYLK1011	NUT,NYLOCK,BLACK ZINC,M8-1.25	1
B5	SCRSHCS1219	SCREW,SHCS,BLACK ZINC,M8-1.25 X 20MM	1

Notes :


*A8 Not shown.

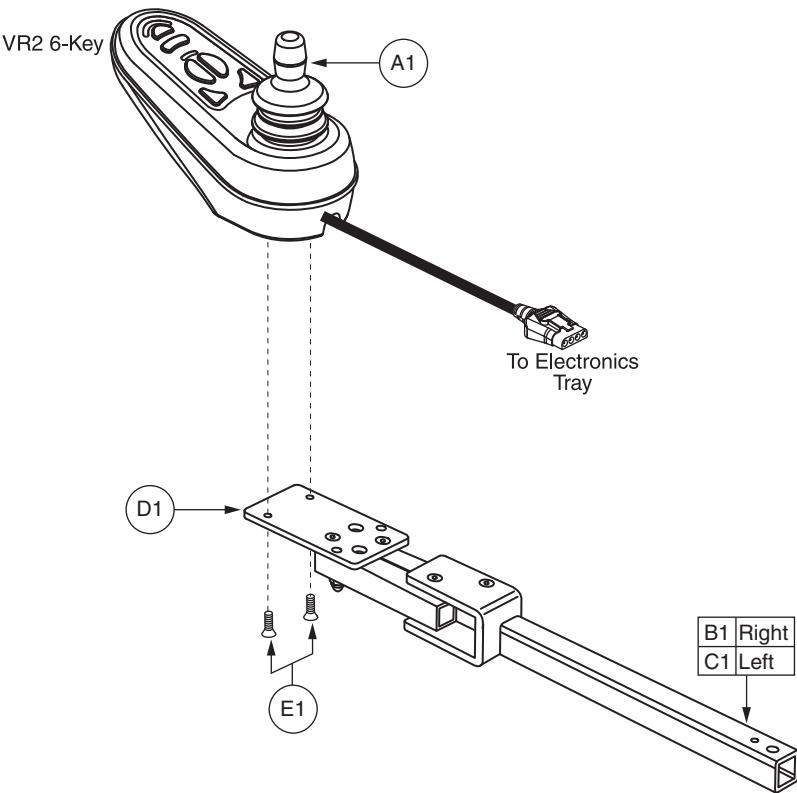

REF #	PART #	DESCRIPTION	QTY
A1	CTLDC1498	CONTROLLER,JOYSTICK MODULE,6 KEY,VR2,H-1111-063,(P&G: D50680)	1
B1	FRMASMB16152	SWING AWAY JOYSTICK MOUNT,ASSY,QUANTUMR40 BRACKET,DOUBLE LINK,INLINE,RIGHT	1
C1	FRMASMB16151	SWING AWAY JOYSTICK MOUNT,ASSY,QUANTUMR40 BRACKET,DOUBLE LINK,INLINE,LEFT	1
D1	BRKINDV1210	MOUNTING BRACKET,CONTROLLER MOUNTING PLATE,(ANI: PD16D-P-034-T)	1
E1	SCRPFH1081	SCREW,PFH,BLACK ZINC,M5-.8 X 22MM	2

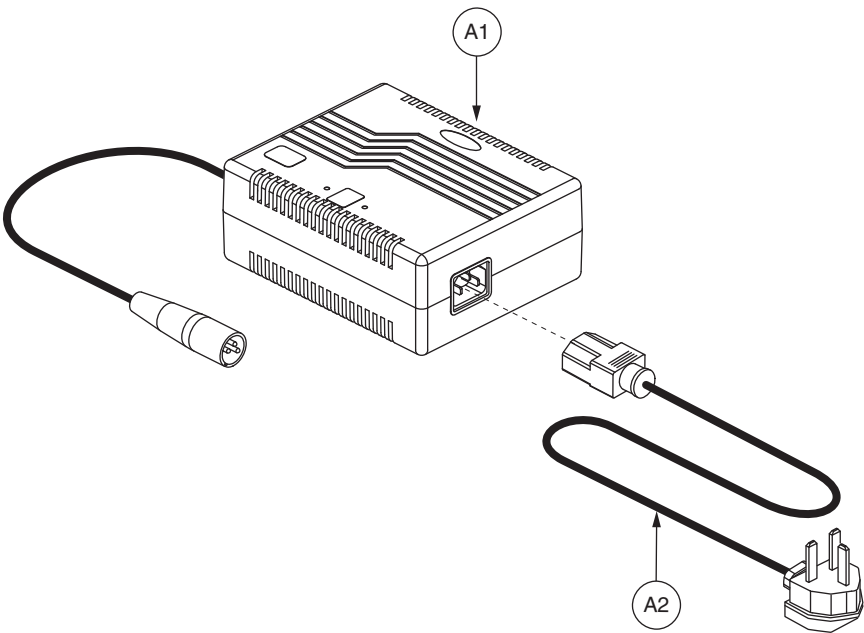


REF #	PART #	DESCRIPTION	QTY
A1 - A2	ELEASMB6294	CHARGER,ASSY,OFFBOARD,100V-240V,5 AMP,BATTERY MAXIMIZER,W/UNITED KINGDOM POWERCORD	1
A1	ELECHG1021	CHARGER,OFFBOARD,100V-240V,5 AMP,H-1010-142,(BATTERY MAXIMIZER: A24050-2)	1
A2	HARUSHD2041	ELECTRONIC,HARNESS, POWER CORD,AC W/UK PLUG,H-1020-049	1

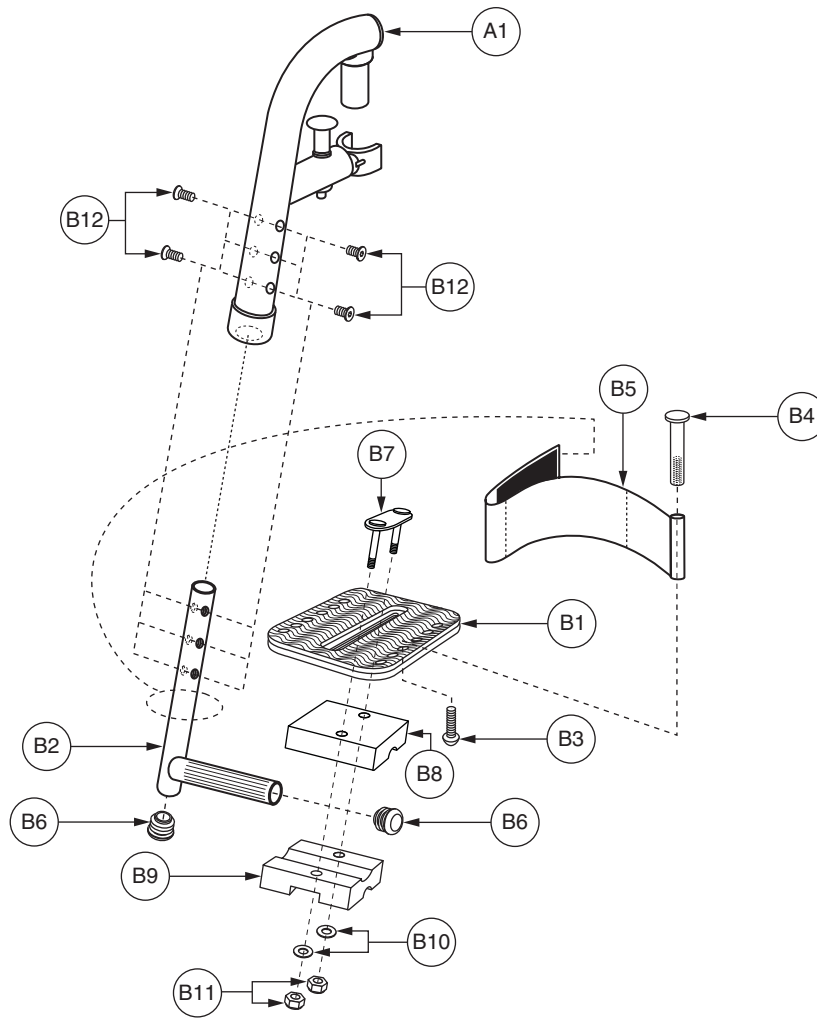

VIII. Legrest Assembly — Pediatric Swing-away

REF #	PART #	DESCRIPTION	QTY
A1 - B12	FRMASMB17563	LEGREST,ASSY,PEDIATRIC S/A,4" X 6" FOOTPLATE,SPARKY SEAT,PAIR	1
A1	FRMASMB17565	LEGREST,ASSY,PEDIATRIC S/A UPPER,SPARKY SEAT	2
B1 - B12	FRMASMB17566	LEGREST,ASSY,PEDIATRIC S/A,LOWER,SPARKY SEAT	2
B1	PLSINDV1317	FOOTPLATE,PEDIATRIC,4" X 6,W/MAT,(ANI: PD10D-P-036-T)	1
B2	FRMINDV1639	LEGREST EXTENSION WELDMENT,LOWER PEDIATRIC,LEFT/RIGHT,(ANI:PD10D-A-010-T)	1
B3	SCRASBH1133	SCREW,BHSCS,BLACK ZINC,M6-1.0 X 35 MM	1
B4	BLTSEX1006	BOLT,SEX,BLACK ZINC,M6-1.0 X 50MM,FEMALE,(ANI: TD16S-P-111-C)	1
B5	ACCSTRP1067	HEEL LOOP,345MM,ADJUSTABLE,(ANI: VA16S-P-043-C)	1
B6	PROECAP1150	END CAP,ROUND,PLASTIC,FITS 16MM HOLE,DOMED	2
B7	FRMINDV1640	CLAMP PLATE,W/THREADED STUDS	1
B8	FRMINDV1641	CLAMP,FOOTPLATE MOUNTING BLOCK	1
B9	FRMINDV1642	CLAMP,FOOTPLATE MOUNTING BLOCK,LOWER	1
B10	WASMETR1108	WASHER,FLAT,BLACK ZINC,6.5MM ID X 13MM OD X 1MM TH	2
B11	NUTNJAM1021	NUT,NYLOCK JAM,BLACK ZINC,M6-1.0	2
B12	SCRSHFH1173	SCREW,SHFH,ZINC,M6-1.0 X 12MM,W/NYLON PATCH	4

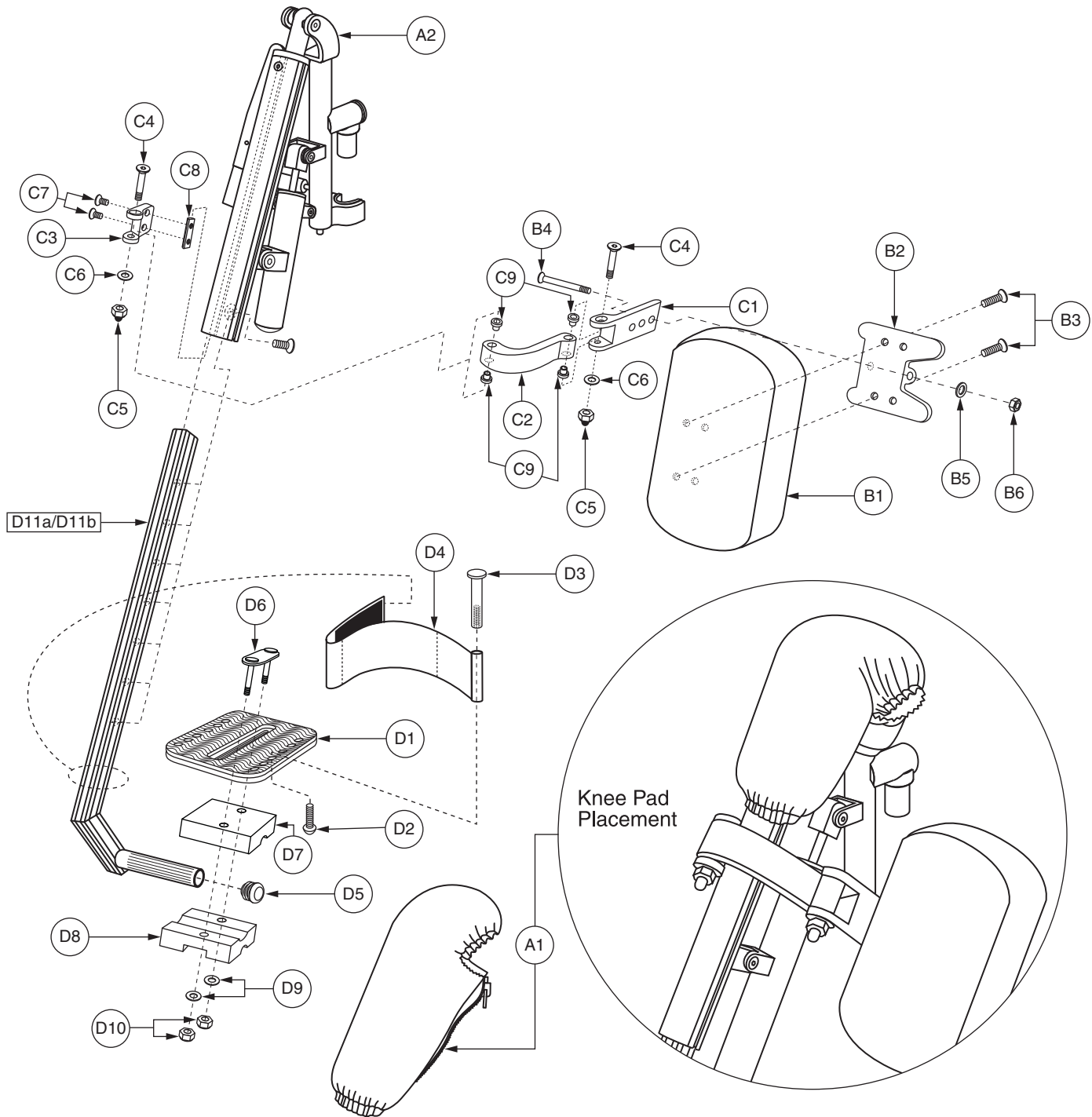


VIII. Legrest Assembly — ELR 4" x 6" Footplate

REF #	PART #	DESCRIPTION	QTY
A1 - D11	FRMASMB17564	LEGREST,ASSY,ELR,4" X 6" FOOTPLATE,W/CALF PADS,SPARKY SEAT,PAIR	1
A1 - D11a	FRMASMB17568	LEGREST,ASSY,ELR UPPER,SPARKY SEAT,RIGHT	1
A1 - D11b	FRMASMB17567	LEGREST,ASSY,ELR UPPER,SPARKY SEAT,LEFT	1
A1	INDPART2865	PAD,KNEE PIVOT COVER,POWER ELR,(ANI: OS16D-P-175-C)	1
A2	FRMASMB17572	LEGREST,ASSY,ELR UPPER,LEFT/RIGHT,SPARKY SEAT	1
*B1 - C9	FRMASMB17574	CALF PAD,ASSY,BLACK,4"W X 6.25"L,W/MOUNTING BRACKET,RIGHT	1
*B1 - C9	FRMASMB17573	CALF PAD,ASSY,BLACK,4"W X 6.25"L,W/MOUNTING BRACKET,LEFT	1
B1	INDPART2909	CALF PAD,BLACK,4"W X 6.25"L,(ANI:BL12S-P-023-C)	1
B2	INDPART2861	CALF PAD MOUNTING BRACKET,BLACK,PLASTIC,(ANI: TD18D-P-029-C)	1
B3	SCRSHFH1184	SCREW,SHFH,BLACK ZINC,M6-1.0 X 21MM,W/NYLON PATCH	2
B4	SCRSHFH1185	SCREW,SHFH,BLACK ZINC,M8-1.25 X 65MM	1
B5	WASMETR1146	WASHER,FLAT,BLACK ZINC,8.4MM ID X 16MM OD X 1.6MM TH	1
B6	NUTNYLK1011	NUT,NYLOCK,BLACK ZINC,M8-1.25	1
C1 - C9	FRMASMB17571	MOUNTING BRACKET,ASSY,CALF PAD,3 HOLE	1
C1	INDPART2910	CALF PAD BRACKET,DEPTH ADJUSTABLE,3 HOLE,(ANI:TD18D-P-003-C)	1
C2	INDPART2863	CALF PAD BRACKET,FLIP UP,(ANI: TD18D-P-002-C)	1
C3	INDPART2864	MOUNTING BRACKET,CALF PAD,(ANI: TD18D-P-004-C)	1
C4	SCRSHFH1187	SCREW,SHFH,BLACK ZINC,M6-1.0 X 38MM	2
C5	NUTNYLK1084	NUT,NYLOCK,BLACK ZINC,M6-1.0,W/BLACK CAP	2
C6	WASSTND1053	WASHER,FLAT,ZINC,6MM ID X 13MM OD X 1MM TH	2
C7	SCRSHFH1188	SCREW,SHFH,BLACK ZINC,M6-1.0 X 10MM	2
C8	HDWINDV1225	T-NUT,ALUMINUM,M6-1.0,2 HOLE,(ANI: TD18D-P-036-C)	1
C9	BSHFLNG1079	BUSHING,FLANGED,ALUMINUM,6MM ID X 7.9MM OD X 10MM TH X 12MM FOD X 1MM FTH,(ANI: TD18D-P-036-C)	4
D1 - D11a	FRMASMB17570	LEGREST,ASSY,ELR LOWER,W/4" X 6" FOOTPLATE,SPARKY SEAT,RIGHT	1
D1 - D11b	FRMASMB17569	LEGREST,ASSY,ELR LOWER,W/4" X 6" FOOTPLATE,SPARKY SEAT,LEFT	1
D1	PLSINDV1317	FOOTPLATE,PEDIATRIC,4" X 6,W/MAT,(ANI: PD10D-P-036-T)	1
D2	SCRASBH1133	SCREW,BHSCS,BLACK ZINC,M6-1.0 X 35 MM	1
D3	BLTSEX1006	BOLT,SEX,BLACK ZINC,M6-1.0 X 50MM,FEMALE,(ANI: TD16S-P-111-C)	1
D4	ACCSTRP1067	HEEL LOOP,345MM,ADJUSTABLE,(ANI: VA16S-P-043-C)	1
D5	PROECAP1150	END CAP,ROUND,PLASTIC,FITS 16MM HOLE,DOMED	1
D6	FRMINDV1640	CLAMP PLATE,W/THREADED STUDS	1
D7	FRMINDV1641	CLAMP,FOOTPLATE MOUNTING BLOCK	1
D8	FRMINDV1642	CLAMP,FOOTPLATE MOUNTING BLOCK,LOWER	1
D9	WASMETR1108	WASHER,FLAT,BLACK ZINC,6.5MM ID X 13MM OD X 1MM TH	2
D10	NUTNJAM1021	NUT,NYLOCK JAM,BLACK ZINC,M6-1.0	2
D11a	FRMINDV1644	LEGREST EXTENSION,BLACK,ALUMINUM,ELR,RIGHT,(ANI:PD10D-P-100-C)	1
D11b	FRMINDV1643	LEGREST EXTENSION,BLACK,ALUMINUM,ELR,LEFT,(ANI:PD10D-A-018-C)	1

Notes :

Top-level assemblies will include components designated with corresponding lower-case letters. Parts without lower-case letters will be included in all top-level assemblies.


*B1 - C9 Both assemblies contain identical parts. There are right and left sides for assembly purposes.

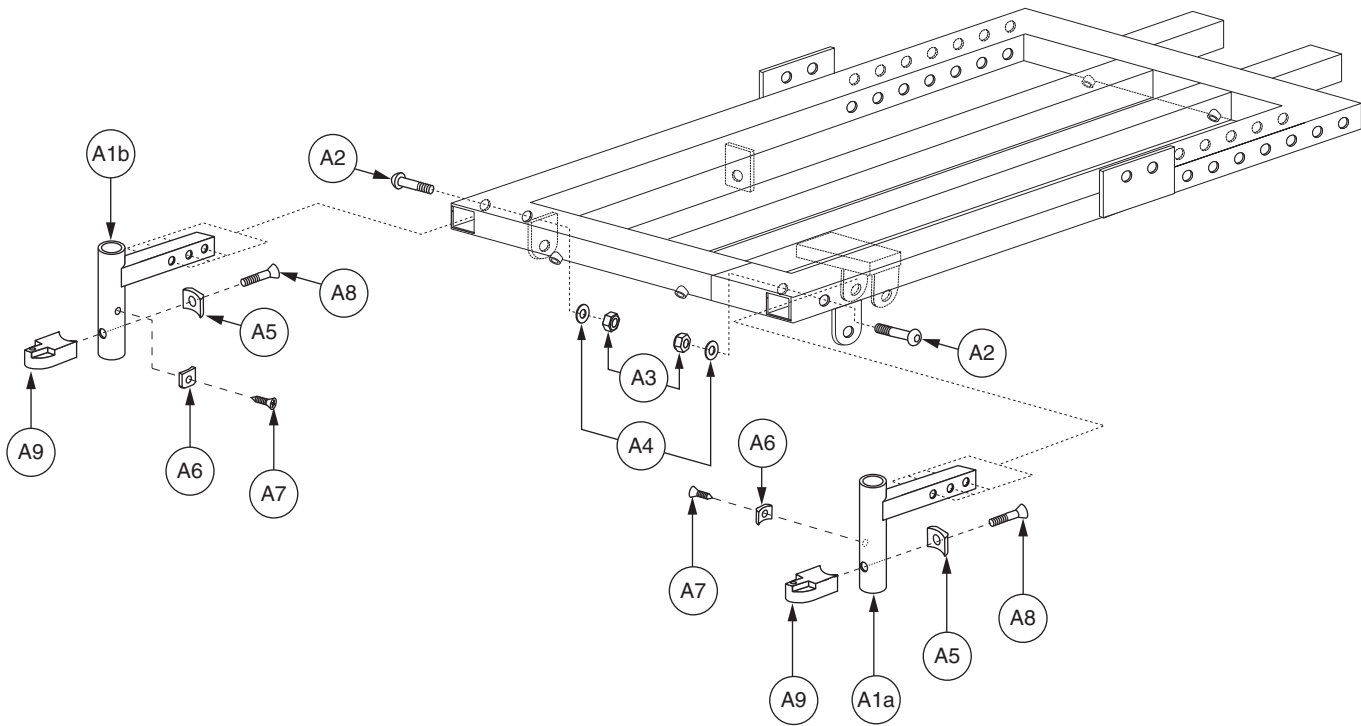


REF #	PART #	DESCRIPTION	QTY
A1 - A9	FRMASMB17560	LEGREST HANGER,ASSY,ELR/SA,STRAIGHT,SPARKY SEAT,PAIR	1
A1a - A9	FRMASMB17562	LEGREST HANGER,ASSY,ELR/SA,STRAIGHT,SPARKY SEAT,RIGHT	1
A1b - A9	FRMASMB17561	LEGREST HANGER,ASSY,ELR/SA,STRAIGHT,SPARKY SEAT,LEFT	1
A1a	FRMINDV1638	LEGREST HANGER WELDMENT,STRAIGHT,RIGHT,SPARKY SEAT,(ANI: PD10D-A-009-T-R)	1
A1b	FRMINDV1637	LEGREST HANGER WELDMENT,STRAIGHT,LEFT,SPARKY SEAT,(ANI: PD10D-A-009-T-L))	1
A2	SCRASBH1289	SCREW,BHSCS,BLACK ZINC,M8-1.25 X 35MM	1
A3	NUTNYLK1011	NUT,NYLOCK,BLACK ZINC,M8-1.25	1
A4	WASSTND1205	WASHER,FLAT,BLACK ZINC,8.2MM ID X 14MM OD X 1MM TH	1
A5	INDPART2830	CLAMP,HANGER MOUNTING BLOCK,SILVER,ION SEAT,(ANI: TD16S-E-004-C)	1
A6	INDPART2831	CLAMP,LATCH,LEGREST HANGER,SILVER,ION SEAT,(ANI: VA16S-E-002-C)	1
A7	SCRPF5M1006	SCREW,PFH-SHEET METAL,BLACK ZINC,M5 X 16MM	1
A8	SCRSHFH1178	SCREW,SHFH,BLACK ZINC,M8-1.25 X 35MM	1
A9	INDPART2843	STOP BLOCK,SILVER,15MM SQUARE,(ANI: VA16S-E-003-C)	1

Notes :


Top-level assemblies will include components designated with corresponding lower-case letters. Parts without lower-case letters will be included in all top-level assemblies.

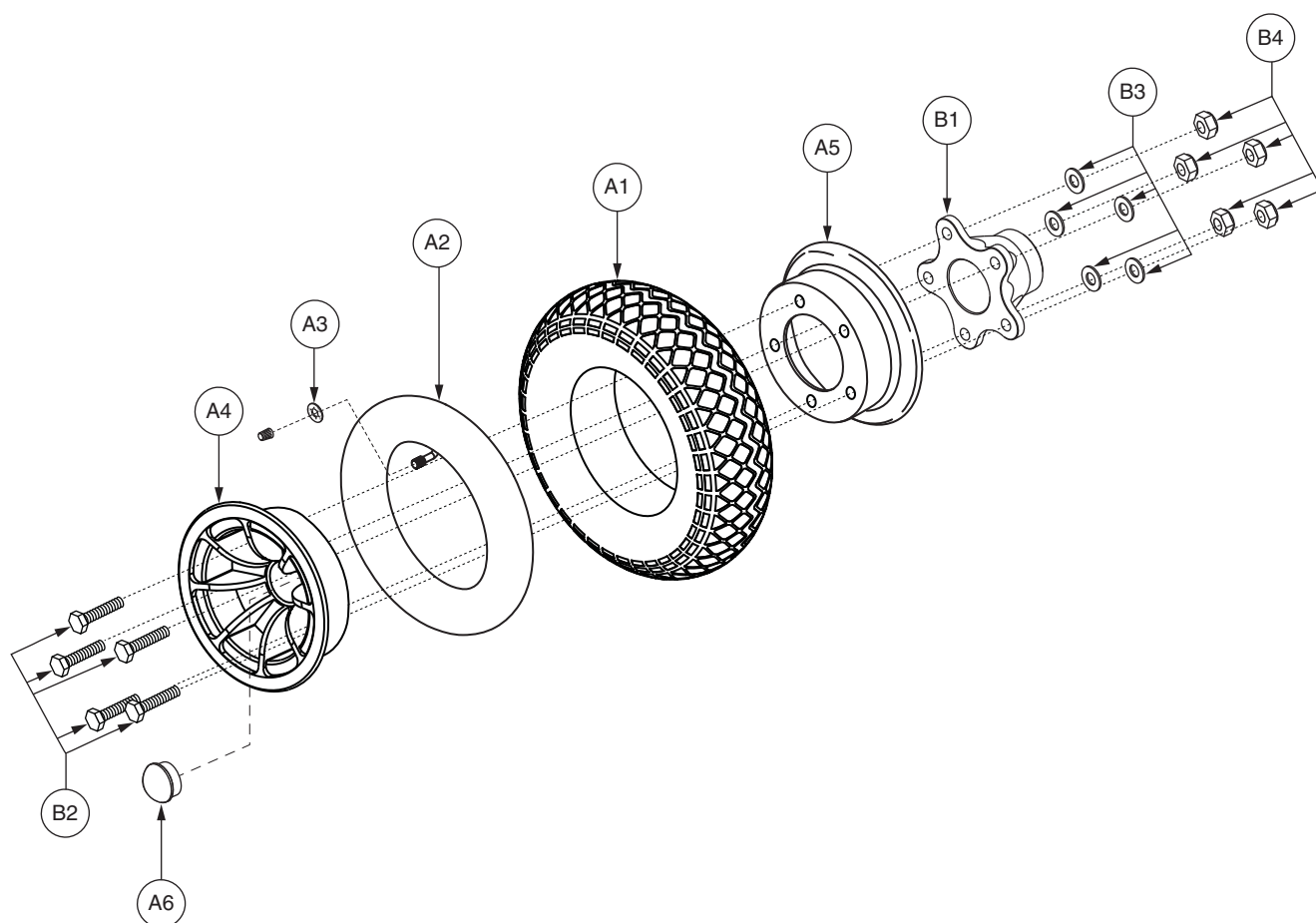


REF #	PART #	DESCRIPTION	QTY
A1 - B4	WHLASMB2229	WHEEL,ASSY,PNEUMATIC,SILVER RIM/BLACK TIRE,4.00-5,C-248-5,W/LUGNUTS	1
A1	TIRPNEU1091	TIRE,PNEUMATIC,BLACK,4.00-5 X 12",(ANI: OS12S-P-009-C)	1
A2	TUBSTND1014	TUBE,4.00-5,ITM4005-7	1
A3	DWR7700A010	SLIP COVER,VALVE STEM,35PSI,2.4 BAR,MAX RATING,A-7700-010	1
A4	WHLRIMS1252	WHEEL,RIM,SILVER,ALUMINUM,OUTER,(ANI: OS12S-E-003-C)	1
A5	WHLRIMS1253	WHEEL,RIM,BLACK,ALUMINUM,INNER,(ANI: OS12S-E-004-C)	1
A6	PLSINDV1318	COVER,DUST CAP,PLASTIC,WHEEL,(ANI: DR16S-P-052-C)	1
B1 - B4	WHLASMB2227	WHEEL,ASSY,HUB,REAR,STRAIGHT,SPARKY	1
B1	HUBSTND1005	WHEEL,HUB,REAR,STRAIGHT,(ANI: OS12S-P-011-C)	1
B2	SCRHHCS1695	SCREW,HHCS,ZINC,M8-1.25 X 25MM,GRADE 12.9	5
B3	WASMETR1163	WASHER,FLAT,ZINC,8.2MM ID X 11MM OD X 1MM TH	5
B4	NUTNYLK1012	NUT,NYLOCK,BLACK ZINC,M8-1.25	5

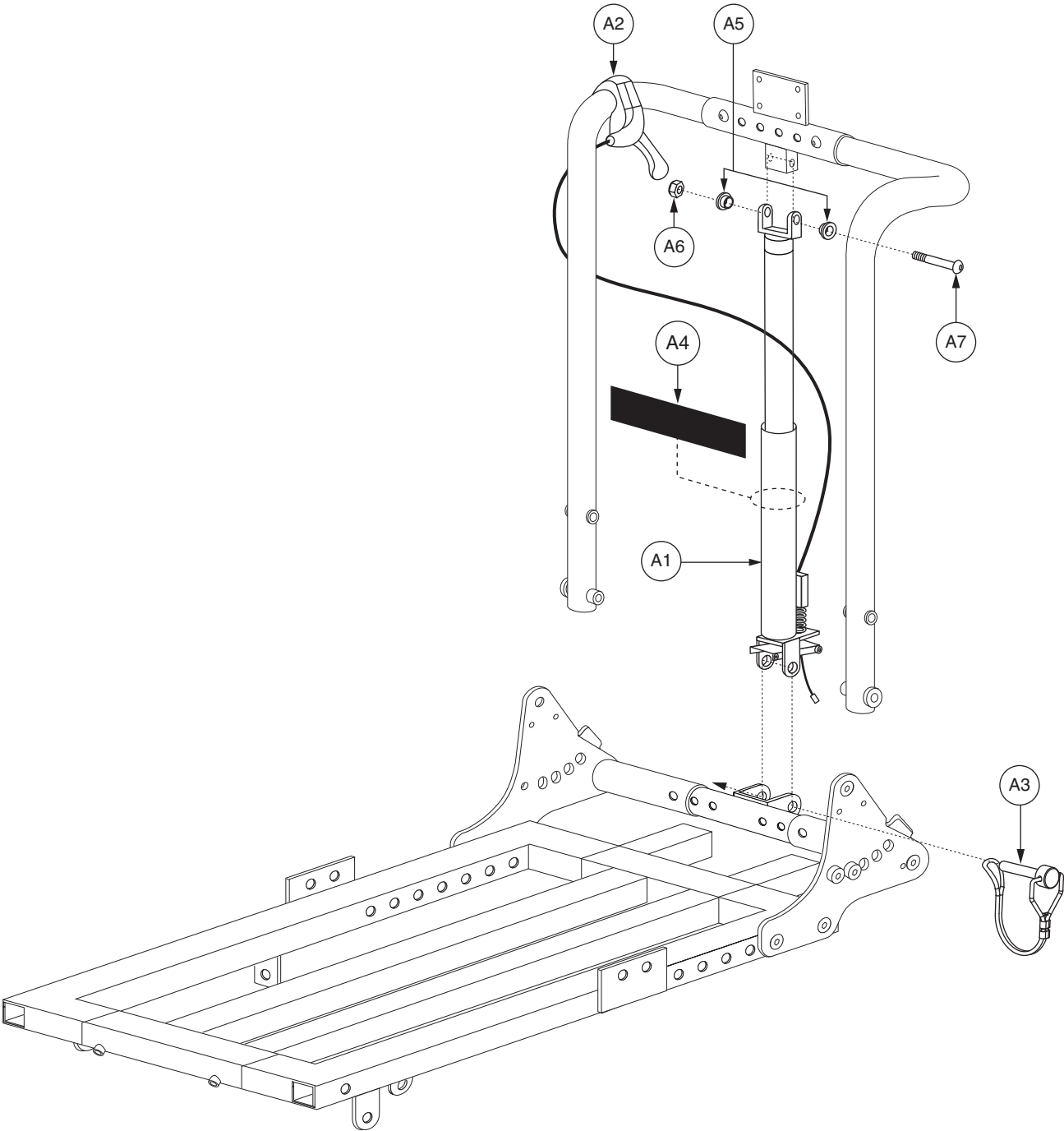

REF #	PART #	DESCRIPTION	QTY
A1 - A7	FRMASMB17543	MANUAL RECLINE,ASSY,W/CABLE & RELEASE LEVER,SPARKY SEAT	1
A1	ACCACTR1060	ACTUATOR,LOCKING GAS SPRING,10KG FORCE,290-370MM STROKE,SPARKY SEAT,(ANI: KA16S-P-014-C)	1
A2	FRMINDV1628	MOUNTING BRACKET,MANUAL ACTUATOR LEVER MOUNT W/CABLE & RELEASE LEVER,(ANI: BS16S-P-013-C)	1
A3	HDWINDV1211	PIN,SAFETY SNAP,8.25MM X 60MM GRIP LENGTH,(ANI: DU16D-P-033-C)	1
A4	ACCSTRP1077	STRAP,BLACK,7 1/2" L X 2" W	1
A5	BSHFLNG1087	BUSHING,FLANGED,BRONZE,8 MM ID X 12MM OD X 5MM TH X 16MM FOD X 1.75MM FTH	2
A6	NUTNYLK1011	NUT,NYLOCK,BLACK ZINC,M8-1.25	1
A7	SCRASBH1289	SCREW,BHSCS,BLACK ZINC,M8-1.25 X 35MM	1

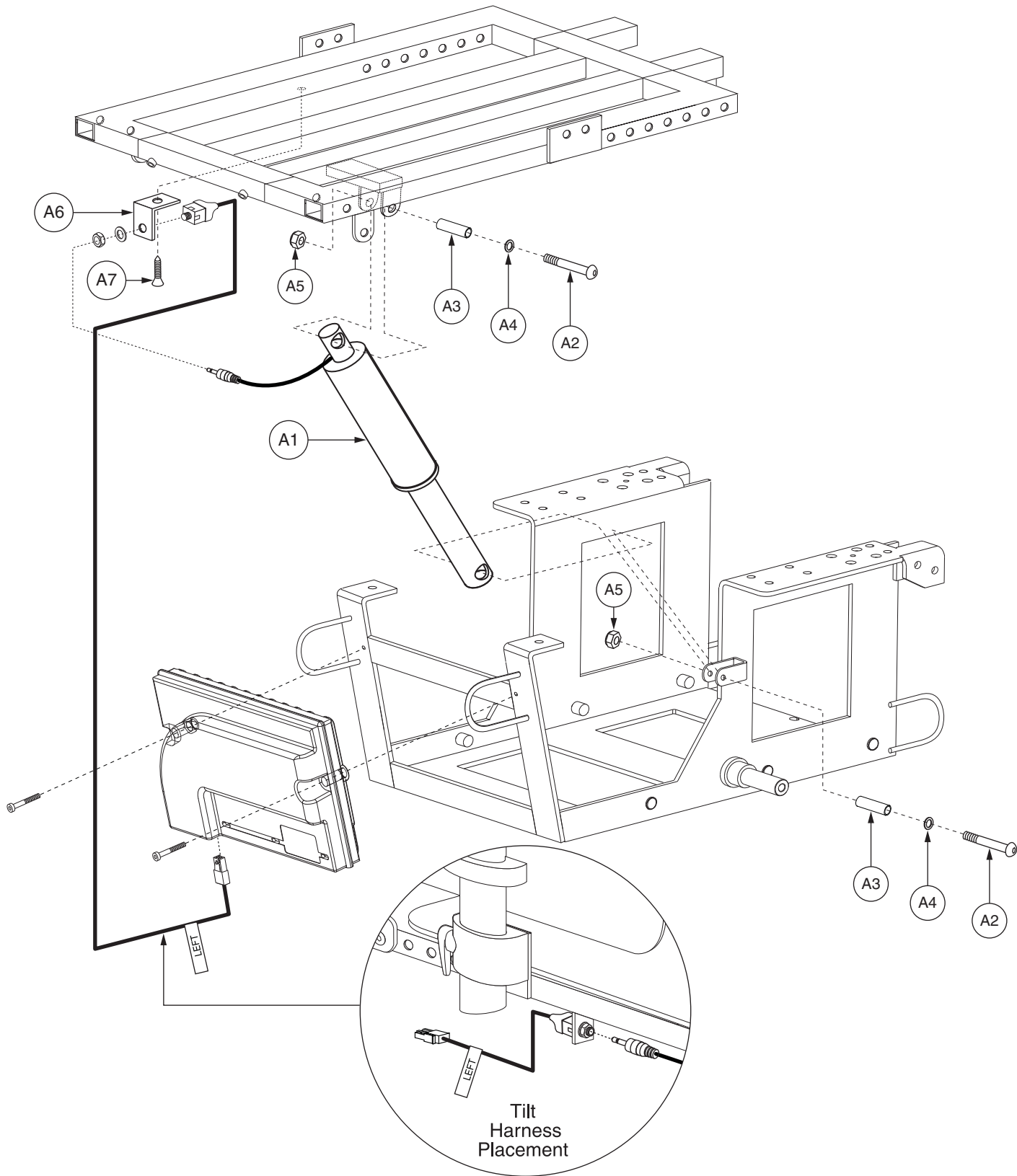

REF #	PART #	DESCRIPTION	QTY
A1 - A7	FRMASMB17544	ACTUATOR,ASSY,POWER TILT,SPARKY SEAT	1
A1	ACCACTR1061	ACTUATOR,POWER TILT,100MM STROKE,1000N,INLINE,(ANI: PD10D-P-044-T)	1
A2	SCRASBH1282	SCREW,BHSCS,BLACK ZINC,M8-1.25 X 42MM	2
A3	HDWINDV1258	SPACER,BLACK ZINC,8MM ID,10MM OD,18MM TH,(ANI: PD10D-P-044-T)	2
A4	WASSTND1205	WASHER,FLAT,BLACK ZINC,8.2MM ID X 14MM OD X 1MM TH	2
A5	NUTNYLK1012	NUT,NYLOCK,BLACK ZINC,M8-1.25	2
A6	FRMINDV1629	L-BRACKET,POWER TILT ACTUATOR HARNESS,SPARKY SEAT	1
A7	SCRPF5M1006	SCREW,PFH-SHEET METAL,BLACK ZINC,M5 X 16MM	1

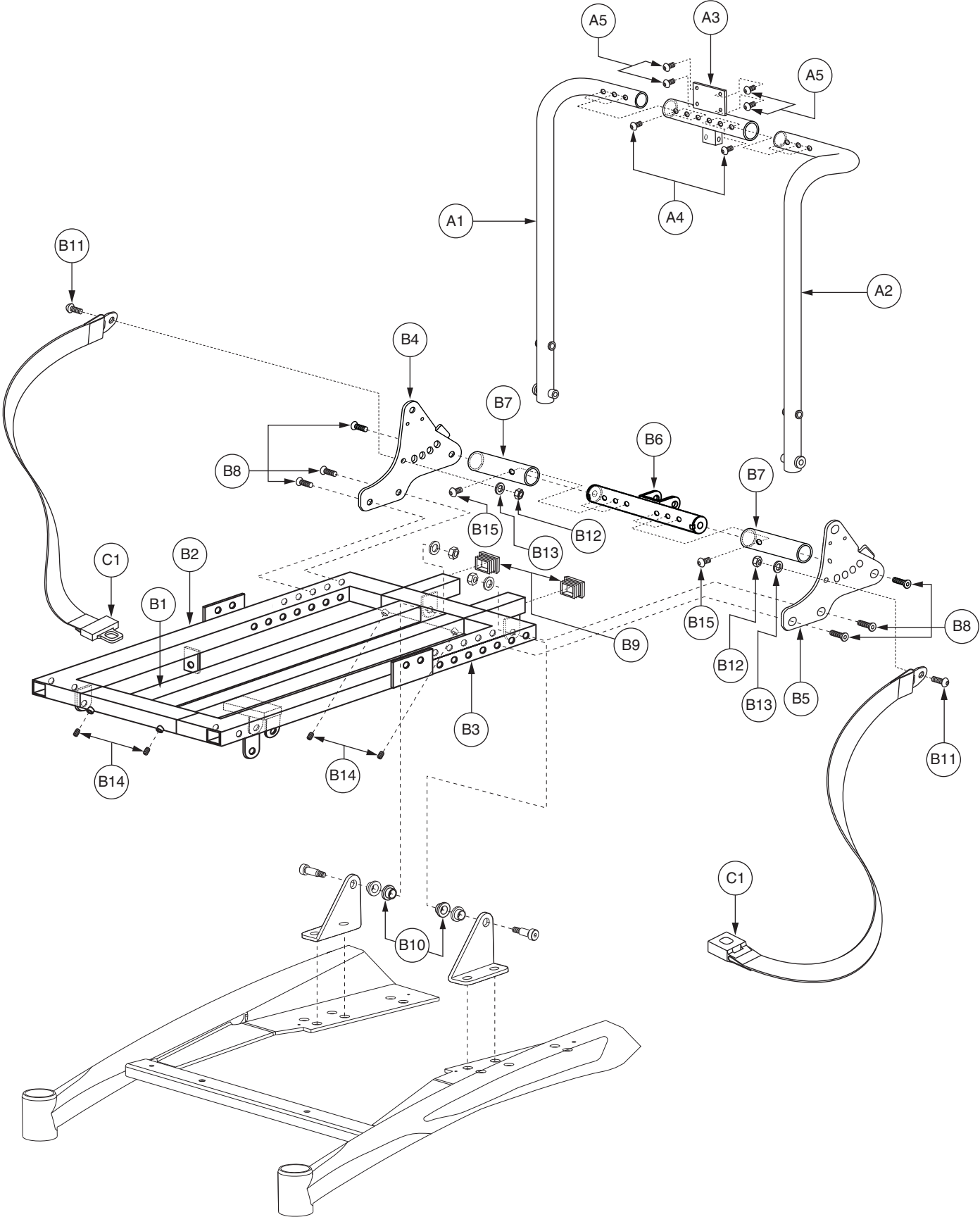


XI. Seat Assembly — Seat Frame

REF #	PART #	DESCRIPTION	QTY
A1 - B15	FRMASMB17542	SEAT FRAME,ASSY,MAIN,10"W-14"W,ADJUSTABLE,SPARKY SEAT	1
A1 - A5	SETASMB7105	SEAT BACK,ASSY,FRAME,16"W,14"H,SPARKY SEAT	1
A1	SETBACK2630	SEAT BACK,FRAME,10"W-14"W,14"H,RIGHT,(ANI: PD10D-A-011-T)	1
A2	SETBACK2629	SEAT BACK,FRAME,10"W-14"W,14"H,LEFT,(ANI: PD10D-A-001-T)	1
A3	SETBACK2631	SEAT BACK,CENTER ADJUSTMENT BAR,10"W-14"W,14"H,(ANI: PD10D-A-002-T)	1
A4	SCRASBH1291	SCREW,BHSCS,BLACK ZINC,M8-1.25 X 12MM	2
A5	SCRASBH1058	SCREW,BHSCS,BLACK ZINC,M6-1.0 X 12MM,GRADE 8.8	4
B1 - B15	SETASMB7109	SEAT BASE,ASSY,MAIN,10"D-14"D,SPARKY SEAT	1
B1	FRMINDV1622	SEAT FRAME WELDMENT,(ANI: PD10D-P-022-T)	1
B2	FRMINDV1624	SEAT FRAME WELDMENT,10"D-14"D,RIGHT,(ANI: PD10D-A-006-T-R)	1
B3	FRMINDV1623	SEAT FRAME WELDMENT,10"D-14"D,LEFT,(ANI: PD10D-A-006-T-L)	1
B4	FRMINDV1626	SEAT FRAME,PIVOT BAR,RIGHT,(ANI: PD10D-A-012-T)	1
B5	FRMINDV1625	SEAT FRAME,PIVOT BAR,LEFT,(ANI: PD10D-A-004-T)	1
B6	FRMINDV1627	SEAT FRAME,CENTER ADJUSTMENT PIVOT BAR,(ANI: PD10D-A-003-T)	1
B7	FRMINDV1478	PIVOT BRACKET,ION SEAT,(ANI: PD16D-A-011-T)	2
B8	SCRSHFH1170	SCREW,SHFH,BLACK ZINC,M8-1.25 X 25MM,W/NYLON PATCH	6
B9	PROECAP1149	END CAP,SQUARE,BLACK,PLASTIC,FITS 22MM SQ TUBE,(ANI:UG500S-P-152-C)	2
B10	BSHFLNG1078	BUSHING,FLANGED,BRONZE,9.7MM ID X 12MM OD X 6.5MM TH X 15MM FOD X 1.5MM FTH,(ANI: PD16D-P-077-T)	2
B11	SCRASBH1319	SCREW,BHSCS,BLACK ZINC,M8-1.25 X 22MM	2
B12	NUTNYLK1011	NUT,NYLOCK,BLACK ZINC,M8-1.25	2
B13	WASSTND1205	WASHER,FLAT,BLACK ZINC,8.2MM ID X 14MM OD X 1MM TH	2
B14	SCRSETS1145	SCREW,SET,BLACK ZINC,M8-1.25 X 9MM,SOFT POINT	4
B15	SCRASBH1291	SCREW,BHSCS,BLACK ZINC,M8-1.25 X 12MM	2
*C1	ACCBELT1025	LAP BELT,47",(ANI: DE16S-P-052-C)	1


Notes :

*C1 The item is a quantity of one but is illustrated as two parts for assembly purposes.

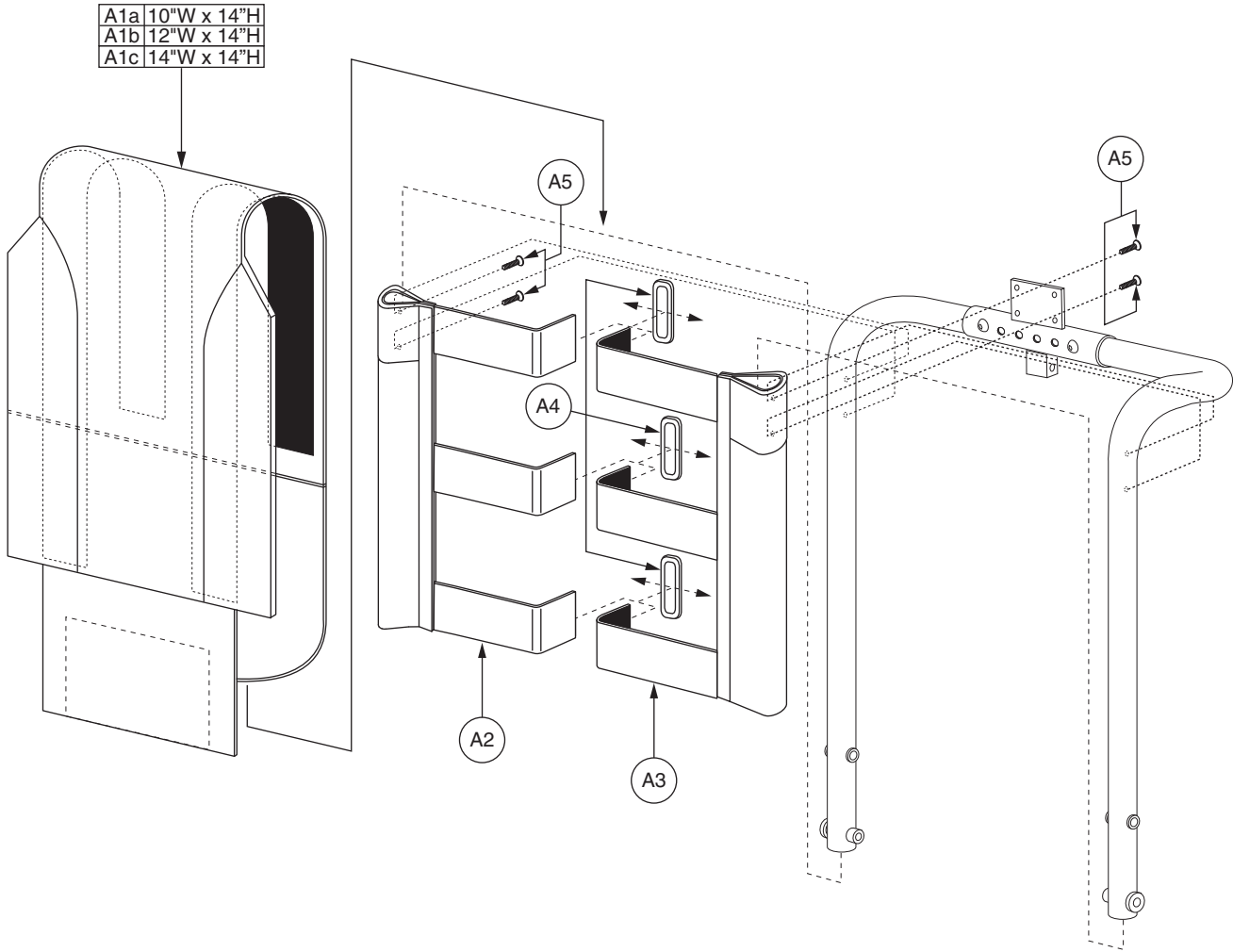


XI. Seat Assembly — Seat Back, Tension Adjustable

REF #	PART #	DESCRIPTION	QTY
A1a - A5	SETASMB7106	SEAT BACK,ASSY,TENSION ADJUSTABLE,10"W,14"H,SPARKY SEAT	1
A1a	SETBACK2632	SEAT BACK,TENSION ADJUSTABLE OVERLAY,10"W,SPARKY SEAT,(ANI: PD10D-P-006-T)	1
A1b - A5	SETASMB7107	SEAT BACK,ASSY,TENSION ADJUSTABLE,12"W,14"H,SPARKY SEAT	1
A1b	SETBACK2633	SEAT BACK,TENSION ADJUSTABLE OVERLAY,12"W,SPARKY SEAT,(ANI: PD10D-P-007-T)	1
A1c - A5	SETASMB7108	SEAT BACK,ASSY,TENSION ADJUSTABLE,14"W,14"H,SPARKY SEAT	1
A1c	SETBACK2634	SEAT BACK,TENSION ADJUSTABLE OVERLAY,14"W,SPARKY SEAT,(ANI: PD10D-P-008-T)	1
A2	SETBACK2636	SEAT BACK,TENSION ADJUSTABLE,10"W-14"W,14"H,SPARKY SEAT,RIGHT,(ANI: PD10D-P-010-T)	1
A3	SETBACK2635	SEAT BACK,TENSION ADJUSTABLE,10"W-14"W,14"H,SPARKY SEAT,LEFT,(ANI: PD10D-P-009-T)	1
A4	INDPART2911	D-RING,BLACK ZINC,SPARKY SEAT,(ANI:X3-DR16S-D-002-C)	3
A5	SCRPFH1081	SCREW,PFH,BLACK ZINC,M5-.8 X 22MM	4

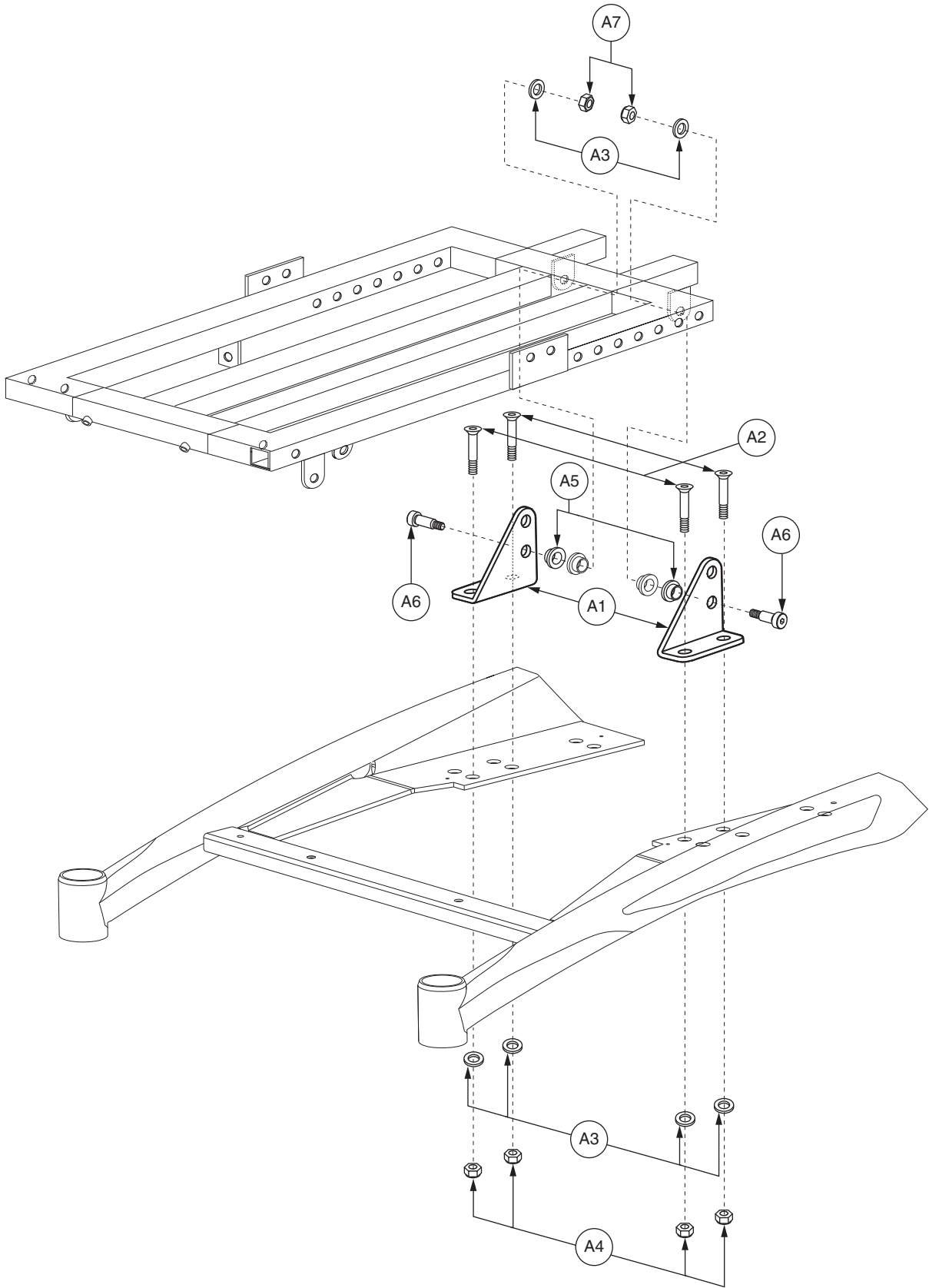


XI. Seat Assembly — Seat Interface

REF #	PART #	DESCRIPTION	QTY
A1 - A7	FRMASMB16114	SEAT INTERFACE,ASSY,R40,ION SEAT	2
A1	FRMINDV1479	SEAT INTERFACE BRACKET,(ANI: PD16D-P-030-C)	1
A2	SCRSHFH1180	SCREW,SHFH,BLACK ZINC,M8-1.25 X 25MM	2
A3	WASSTND1123	WASHER,FLAT,BLACK ZINC,8.4MM ID X 16MM OD X 1MM TH	3
A4	NUTNYLK1012	NUT,NYLOCK,BLACK ZINC,M8-1.25	2
A5	BSHFLNG1078	BUSHING,FLANGED,BRONZE,9.7MM ID X 12MM OD X 6.5MM TH X 15MM FOD X 1.5MM FTH,(ANI: PD16D-P-077-T)	1
A6	BLTASHL1217	BOLT,SHOULDER,BLACK ZINC,10MM X 12MM,(ANI: PD16D-P-006-C)	1
A7	NUTNYLK1011	NUT,NYLOCK,BLACK ZINC,M8-1.25	1

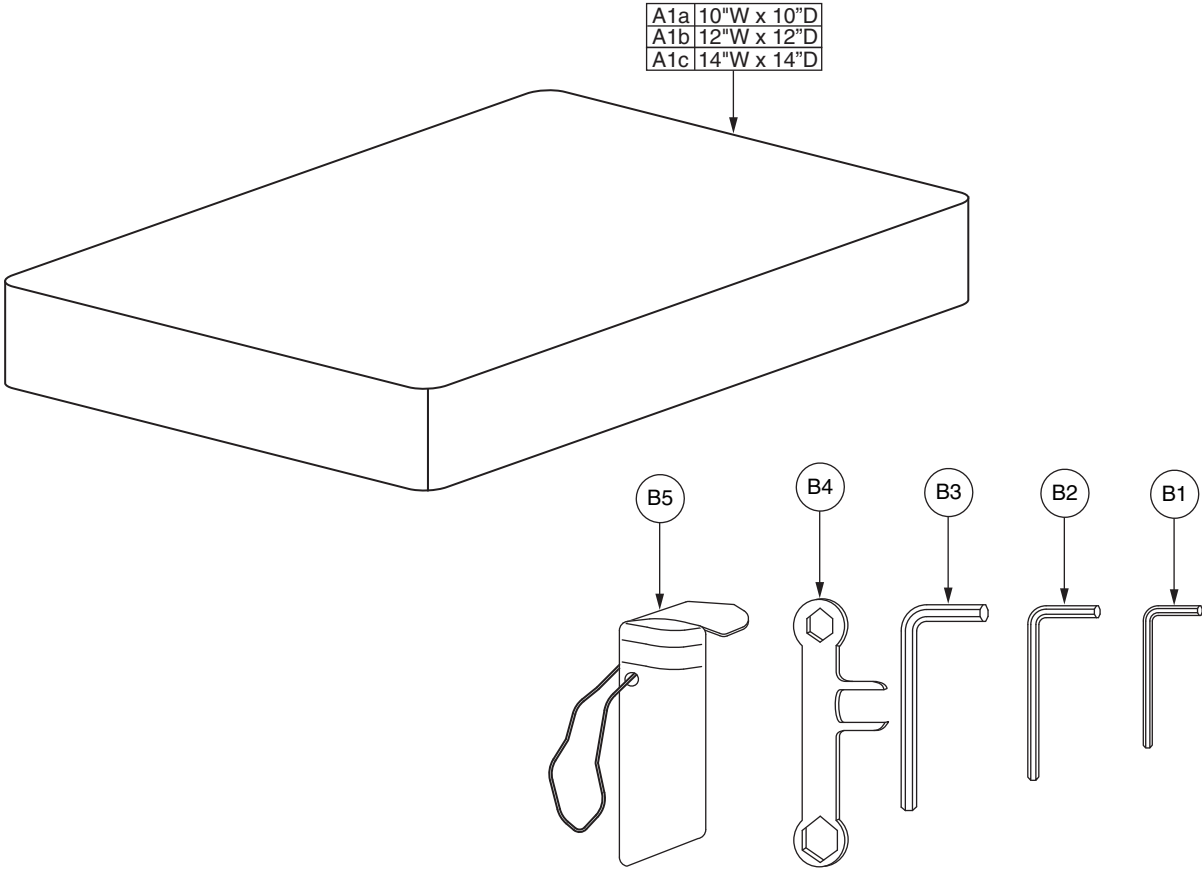

XI. Seat Assembly — Seat Cushion

REF #	PART #	DESCRIPTION	QTY
A1a	CUSINDV1204	CUSHION,SEAT,10"W,10"D,BASIC,(ANI:PD10D-P-025-T)	1
A1b	CUSINDV1205	CUSHION,SEAT,12"W,12"D,BASIC,(ANI:PD10D-P-026-T)	1
A1c	CUSINDV1206	CUSHION,SEAT,14"W,14"D,BASIC,(ANI:PD10D-P-027-T)	1
B1 - B5	HDWASMB1171	TOOL,ASSY,KIT,METRIC,ION SEAT,(ANI: TD16S-P-160-C)	1
B1	HDWINDV1058	HEX KEY,BLACK,4MM	1
B2	HDWINDV1059	HEX KEY,BLACK,5MM	1
B3	HDWINDV1080	HEX KEY,BLACK,6MM	1
B4	HDWINDV1205	TOOL,WRENCH,10MM X 13MM,COMBO,(ANI: TD16S-P-164-C)	1
B5	PROPBAG1073	BAG,GRAY,PLASTIC,6"L X 2 1/4"W,TOOL KIT	1

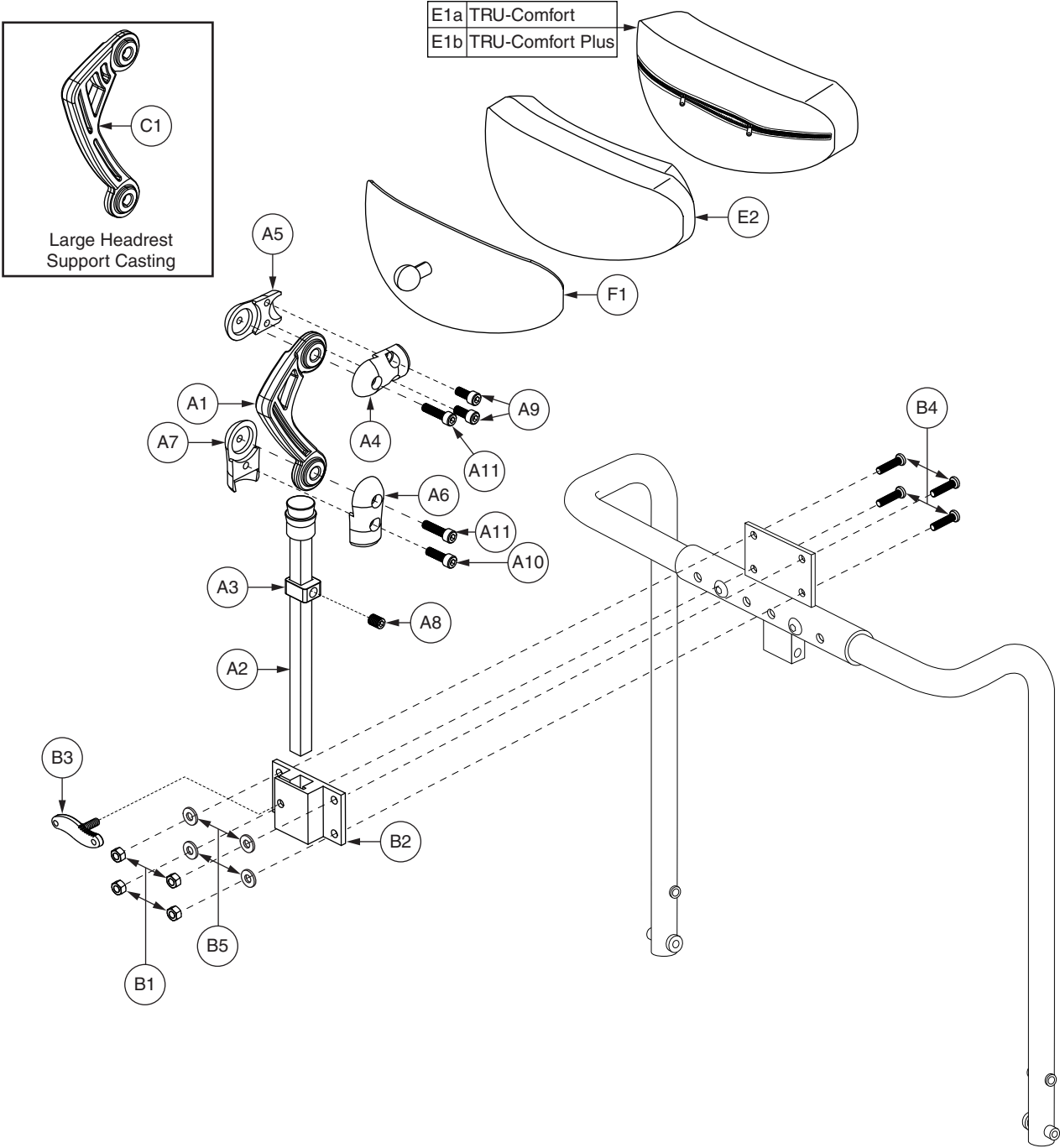


XII. Headrest Assembly

REF #	PART #	DESCRIPTION	QTY
A1 - D1	HDWASMB1168	HARDWARE KIT,ASSY,HEADREST MOUNT,TRU-COMFORT,W/REMOVABLE MOUNTING BRACKET	1
A1 - A11	FRMASMB14425	HARDWARE KIT,ASSY,HEADREST MOUNT,TRU-COMFORT,W/MOUNTING HARDWARE,D-1010-381	1
A1	DWR1010C418	BRACKET,HEADREST SUPPORT CASTING,SMALL,TRU-COMFORT HEADREST,C-1010-418	1
A2	DWR1010C364	HEADREST,POST WELDMENT,MAIN SUPPORT TUBE,TRU-COMFORT HEADREST,C-1010-364	1
A3	DWR1010C367	BRACKET,POST SET SLEEVE,TRU-COMFORT HEADREST,C-1010-367	1
A4	DWR1010E825	CLAMP,BOTTOM POST ATTACHMENT,THREADED,TRU-COMFORT HEADREST,E-1010-825	1
A5	DWR1010E826	CLAMP,TOP BALL,COUNTER BORE,TRU-COMFORT HEADREST,E-1010-826	1
A6	DWR1010E827	CLAMP,TOP BALL,THREADED,TRU-COMFORT HEADREST,E-1010-827	1
A7	DWR1010E828	CLAMP,BOTTOM POST ATTACHMENT,COUNTER BORE,TRU-COMFORT HEADREST,E-1010-828	1
A8	SCRSETS1032	SCREW,SET,3/8-24 X 0.375"	1
A9	SCRSHCS1134	SCREW,SHCS,BLACK ZINC,1/4-20 X 0.625",W/NYLON PATCH	2
A10	SCRSHCS1295	SCREW,SHCS,BLACK,ZINC,1/4-20 X 0.875",ASTM F574	1
A11	SCRSHCS1120	SCREW,SHCS,BLACK ZINC,1/4-20 X 1.00",W/NYLON PATCH	2
B1 - B6	FRMASMB15855	MOUNTING BRACKET,ASSY,TRU-COMFORT HEADREST	1
B1	NUTNYLK1001	NUT,NYLOCK,BLACK ZINC,10-32	4
B2	BRKINDV1169	MOUNTING CLAMP BRACKET,HEADREST	1
B3	DWR1306C005	HEADREST KNOB,LOW PROFILE,C-1306-005	1
B4	SCRASBH1040	SCREW,BHSCS,10-32 X 0.875"	4
B5	WASSAE1005	WASHER,FLAT SAE,ZINC,#10,0.2812" ID X 0.50" OD X 0.0468" TH	4
*B6	HDWINDV1220	HEX KEY,BLACK,3/16",LONG ARM	1
C1	DWR1010C355	BRACKET,HEADREST SUPPORT CASTING,LARGE,TRU-COMFORT HEADREST,C-1010-355	1
*D1	INFINFB2479	MANUAL,INSTALLATION INSTRUCTIONS,QUANTUM TRU-COMFORT HEADREST	1
E1a - F1	FRMASMB15669	HEADREST PAD,ASSY,MEDIUM,8",TRU-COMFORT,W/COVER & FOAM	1
E1b - F1	FRMASMB15670	HEADREST PAD,ASSY,MEDIUM,8",TRU-COMFORT PLUS,W/COVER & FOAM	1
E1a - E2	FABASMB1417	HEADREST COVER,ASSY,OUTER,MEDIUM,TRU-COMFORT,W/FOAM	1
E1b - E2	FABASMB1418	HEADREST COVER,ASSY,OUTER,MEDIUM,TRU-COMFORT PLUS,W/FOAM	1
E1a	DWR1040E748	HEADREST COVER,OUTER,VINYL,MEDIUM,TRU-COMFORT,E-1040-748	1
E1b	DWR1040E747	HEADREST COVER,OUTER,DRY-TEK,MEDIUM,TRU-COMFORT,E-1040-747	1
E2	DWR1040D184	HEADREST,FOAM BASE,MEDIUM,TRU-COMFORT,D-1040-184	1
F1	DWR1010D383	HEADREST,WELDMENT BRACE,MEDIUM,TRU-COMFORT HEADREST,D-1010-383	1

Notes :

Top-level assemblies will include components designated with corresponding lower-case letters. Parts without lower-case extensions will be included in all top-level assemblies.

*B6 Not shown.
*D1 Not shown.

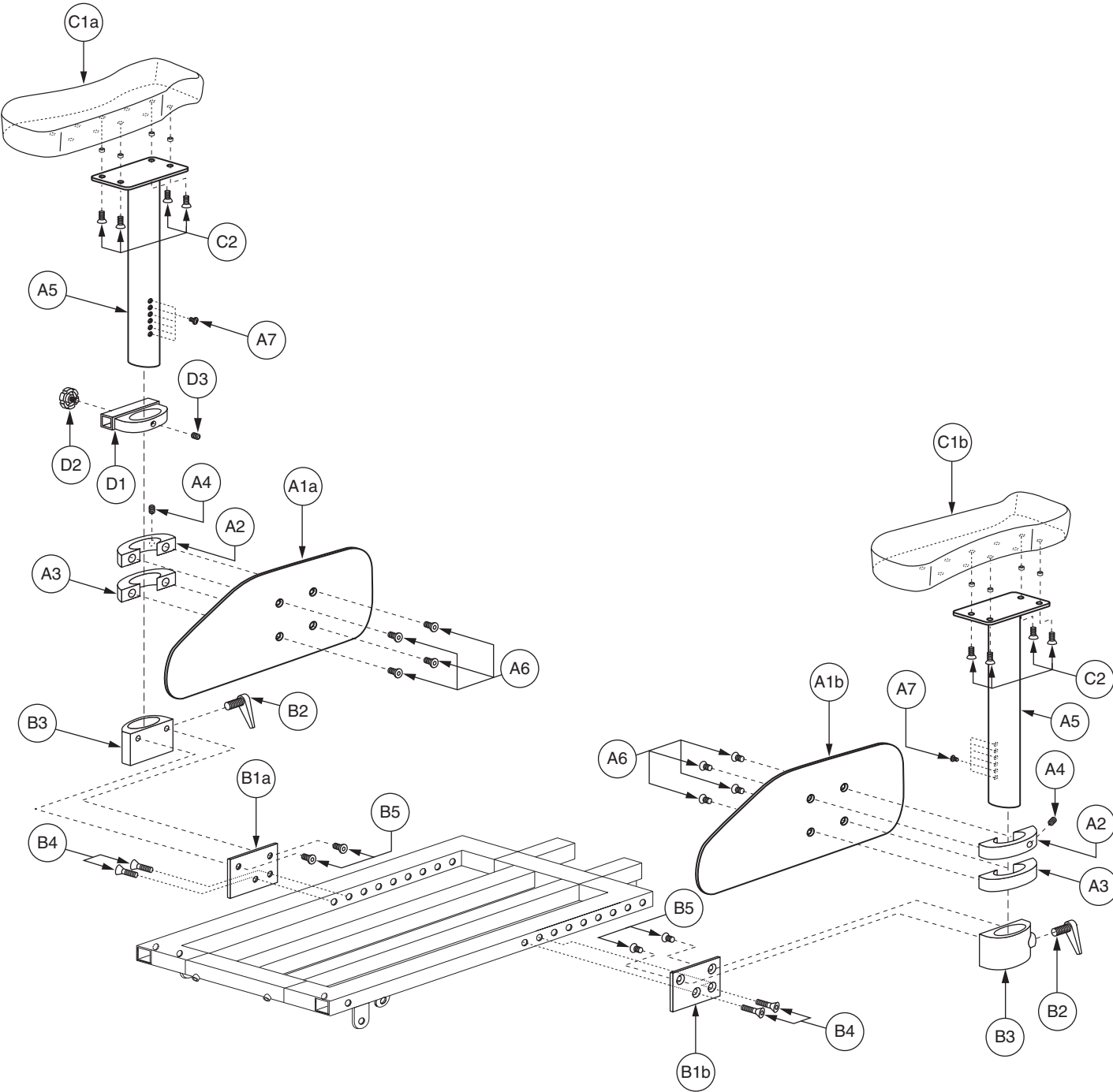

XIII. Armrest Assembly

REF #	PART #	DESCRIPTION	QTY
A1a - B5	FRMASMB17551	ARMREST,ASSY,HEIGHT ADJUSTABLE,FULL LENGTH,PEDIATRIC,RIGHT,SPARKY SEAT	1
A1b - B5	FRMASMB17552	ARMREST,ASSY,HEIGHT ADJUSTABLE,FULL LENGTH,PEDIATRIC,LEFT,SPARKY SEAT	1
A1a - A7	FRMASMB17545	ARMREST,ASSY,HEIGHT ADJUSTABLE,FULL LENGTH,RIGHT,SPARKY SEAT	1
A1b - A7	FRMASMB17546	ARMREST,ASSY,HEIGHT ADJUSTABLE,FULL LENGTH,LEFT,SPARKY SEAT	1
A1a	FRMINDV1635	ARMREST,CLOTHING GUARD,RIGHT,SPARKY SEAT,(ANI:OS12S-P-0260C)	1
A1b	FRMINDV1636	ARMREST,CLOTHING GUARD,LEFT,SPARKY SEAT,(ANI:OS12S-P-0260C)	1
A2	FRMINDV1633	ARMREST BRACKET,UPPER,CLOTHING GUARD,(ANI:DE16S-E-049-C)	1
A3	FRMINDV1465	ARMREST BRACKET,LOWER,CLOTHING GUARD	1
A4	SCRSETS1117	SCREW,SET,BLACK ZINC,M8-1.25 X 8MM	1
A5	FRMINDV1632	ARMREST,UPRIGHT WELDMENT,(ANI: OS12S-A-007-C-R)	1
A6	SCRSHFH1189	SCREW,SHFH,BLACK ZINC,M6-1.0 X 15MM,W/NYLON PATCH	4
A7	SCRCPH1004	SCREW,CPH,BLACK ZINC,M5-.8 X 6MM,W/NYLON PATCH	1
B1a - B5	FRMASMB17547	ARMREST,ASSY,RECEIVER BLOCK,HEIGHT ADJUSTABLE,FULL LENGTH,RIGHT,SPARKY SEAT	1
B1b - B5	FRMASMB17548	ARMREST,ASSY,RECEIVER BLOCK,HEIGHT ADJUSTABLE,FULL LENGTH,LEFT,SPARKY SEAT	1
B1a	FRMINDV1631	ARMREST,RECEIVER BLOCK,RECEIVER BAR,RIGHT,(ANI: PD10D-P-060-C)	1
B1b	FRMINDV1630	ARMREST,RECEIVER BLOCK,RECEIVER BAR,LEFT,(ANI: PD10D-P-099-T)	1
B2	BRKINDV1265	LEVER,CLAMPING CAM,(ANI:DE16S-E-002-C)	1
B3	FRMINDV1634	ARMREST,RECEIVER BLOCK,LOWER,(ANI: OS12S-P-0310C)	1
B4	SCRSHFH1170	SCREW,SHFH,BLACK ZINC,M8-1.25 X 25MM,W/NYLON PATCH	2
B5	SCRSHFH1189	SCREW,SHFH,BLACK ZINC,M6-1.0 X 15MM,W/NYLON PATCH	4
C1 - C2	UPSASMB1034	ARMPAD,ASSY,MOLDED,BLACK,SPARKY SEAT,PAIR	1
C1a - C2	UPSASMB1036	ARMPAD,ASSY,MOLDED,BLACK,SPARKY SEAT,W/HARDWARE,RIGHT	1
C1b - C2	UPSASMB1035	ARMPAD,ASSY,MOLDED,BLACK,SPARKY SEAT W/HARDWARE,LEFT	1
C1a	SETLRAM1203	ARMPAD,MOLDED,BLACK,RIGHT,SPARKY SEAT,(ANI: BL12S-P-001-T-R)	1
C1b	SETLRAM1202	ARMPAD,MOLDED,BLACK,LEFT,SPARKY SEAT,(ANI: BL12S-P-001-T-L)	1
C2	SCRPFH1081	SCREW,PFH,BLACK ZINC,M5-.8 X 22MM	4
D1 - D3	FRMASMB16127	MOUNTING BRACKET,ASSY,JOYSTICK RECEIVER MOUNT BLOCK,ION SEAT	1
D1	FRMINDV1467	ARMREST,RECEIVER BLOCK,UPPER,(ANI: OS16D-A-024-C)	1
D2	PLSKNCP1063	KNOB,PLASTIC,(ANI: OS16D-P-120-C)	1
D3	SCRSETS1117	SCREW,SET,BLACK ZINC,M8-1.25 X 8MM	1

Notes :

Top-level assemblies will include components designated with corresponding lower-case letters. Parts without lower-case letters will be included in all top-level assemblies.

XIV. Owner's and Decal Packages

REF #	PART #	DESCRIPTION	QTY
*N/A	INFMANU4200	MANUAL,OWNER'S,SPARKY,UNITED KINGDOM	1
*N/A	INFINFB2745	MANUAL,PRODUCT SPECIFICATION SHEET,SPARKY,UNITED KINGDOM	1
*N/A	INFINFB1137	MANUAL,WARRANTY CARD,ENGLISH	1
*N/A	PROZBAG1003	BAG,10"L X 12"W X 2MM,ZIPLOCK	1
*N/A	PLSTIE1014	CABLE TIE,BLACK,NYLON,16"L	6
*N/A	DWR1235L519	HANG TAG,ROUND,GENERIC,COUNTRY OF ORIGIN,L-1235-519	1
*N/A	DWR1020L003	HANG TAG,FULLY RECHARGE BATTERY BEFORE INITIAL USE,6-1/4" X 3-3/8",L-1020-003	1
*N/A	HDWASMB1171	TOOL,ASSY,KIT,METRIC,ION SEAT,(ANI: TD16S-P-160-C)	1


Notes :

*N/A Not applicable.

REF #	PART #	DESCRIPTION	QTY
*N/A	DECSERN1035	DECAL,SERIAL NUMBER,2" X 1.25",BARCODE,MADE IN CHINA,W/PRIDE LOGO	1
*N/A	DWR1121L003	DECAL,PRIDE MOBILITY PRODUCTS,WEBSITE,L-1121-003	1
*N/A	DWR1235L389	DECAL,ICON,MAXIMUM WEIGHT CAPACITY,120LBS/54KGS,L-1235-389	1
*N/A	DWR1234L091	DECAL,ICON,JAZZY FREEWHEEL,UP-DRIVE/DOWN-NEUTRAL,L-1234-091	2
*N/A	DWR1234L007	DECAL,ICON,DO NOT STAND/NOT A STEP,L-1234-007	1
*N/A	DWR1234L012	DECAL,ICON,1.5",READ OWNERS MANUAL,L-1234-012	1
*N/A	DWR1235L521	DECAL,WARNING,CONTROLLER CORD PLACEMENT,L-1235-521	1
*N/A	DWR1234L799	DECAL,ICON,1" X 2.5",WEATHER/MANUAL/LOOSE GARMENT,L-1234-799	1
*N/A	DWR1234L811	DECAL,ICON,DO NOT ADJUST ANTI-TIP,1.25" X 0.75",JAZZYSELECT,L-1234-811	2
*N/A	DWR1234L957	DECAL,WARNING,DEFLATE TIRE COMPLETELY,L-1234-957	2
*N/A	DWR1235L512	DECAL,WARNING,EMI-RFI,20V/M,L-1235-512	1
*N/A	DWR1235L515	DECAL,WARNING,BATTERY WARNINGS,L-1235-515	1
*N/A	DWR1235L520	DECAL,CAUTION,EXTENSION CORD/GROUNDING PRONG,L-1235-520	1
*N/A	DWR1235L655	DECAL,BATTERY CONNECTION,1.75" X 2.50",R40,L-1235-655	1
*N/A	DWR1235L850	DECAL,DATA PLATE,SPARKY,UNITED KINGDOM,L-1235-850	1
*N/A	DWR1235L832	DECAL,NAME,SPARKY,BLACK,LEFT,L-1235-832	1
*N/A	DWR1235L831	DECAL,NAME,SPARKY,BLACK,RIGHT,L-1235-831	1
*N/A	DWR1235L834	DECAL,NAME,SPARKY,BLUE,RIGHT,L-1235-834	1
*N/A	DWR1235L833	DECAL,NAME,SPARKY,BLUE,LEFT,L-1235-833	1
*N/A	DWR1235L836	DECAL,NAME,SPARKY,WHITE,RIGHT,L-1235-836	1
*N/A	DWR1235L835	DECAL,NAME,SPARKY,WHITE,LEFT,L-1235-835	1

Notes :

*N/A Not applicable.

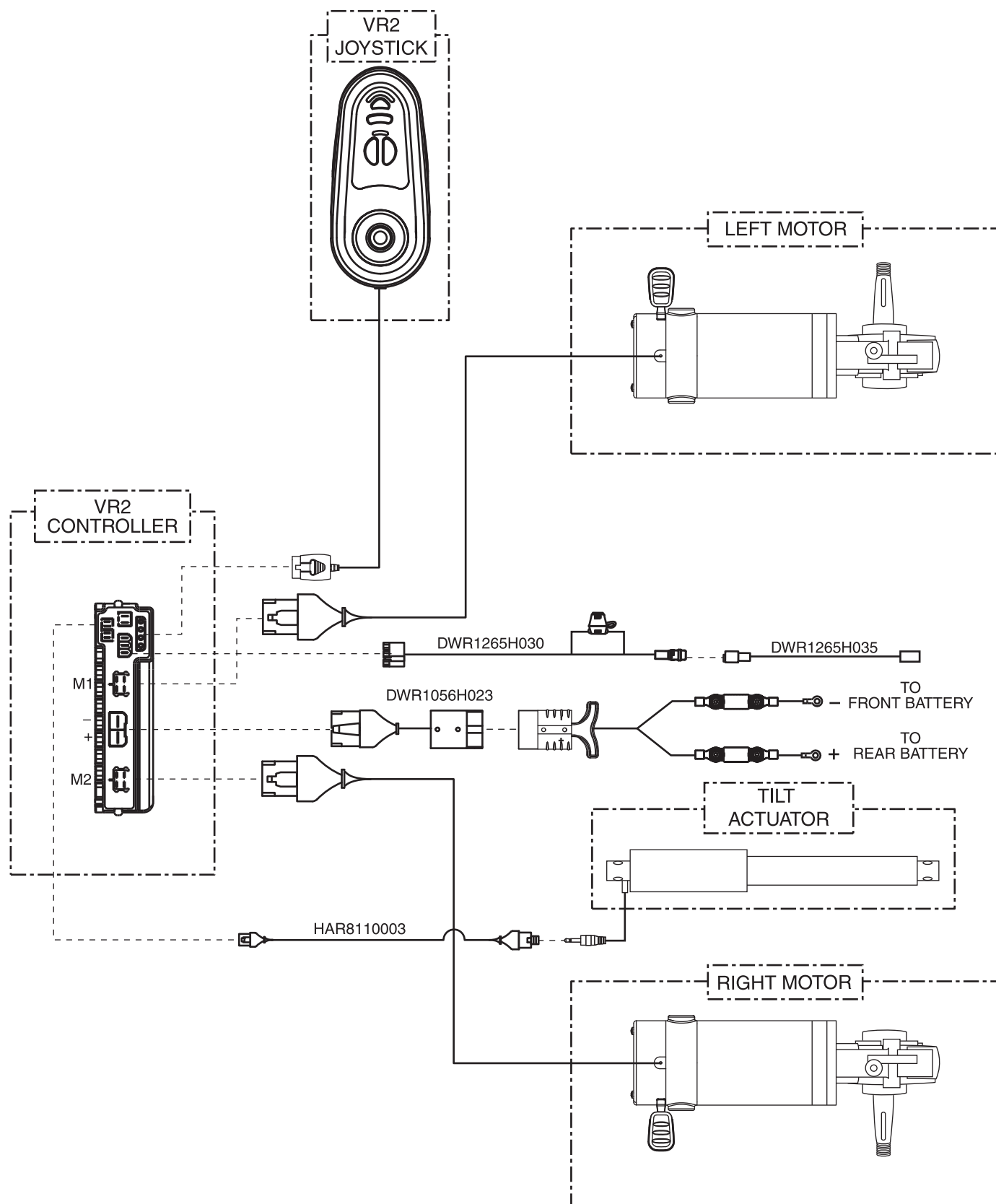



Pride®

Mobility Products Corp.

www.pridemobility.com

Pride® Mobility Products Corporation

182 Susquehanna Avenue
Exeter, PA 18643
USA

Pride® Mobility Products Company

380 Vansickle Road, Unit 350
St. Catharines, Ontario L2R 6P7
Canada

Pride® Mobility Products Ltd.

32 Wedgwood Road
Bicester, Oxon OX26 4UL
UK

Pride® Mobility Products Australia Pty. Ltd.

20-24 Apollo Drive
Hallam, Victoria 3803
Australia

Pride® Mobility Products Italia S.r.l.

Via del Porgresso - ang. Via del Lavoro
Loc. Prato della Corte
00065 - Fiano Romano (RM)

Pride® Mobility Products Europe B.V.

Castricummer Werf 26
1901 RW Castricum
The Netherlands