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# BEATLE – PUMA

**Y·e·S** Series

 **handicare**

9002433E

<b>English .....</b>	<b>5</b>
<b>Svenska .....</b>	<b>39</b>

---

---

# English

© 2008 Handicare

All rights reserved.

The information provided herein may not be reproduced and/or published in any form, by print, photoprint, microfilm or any other means whatsoever (electronically or mechanically) without the prior written authorisation of Handicare.

The information provided is based on general data concerning the constructions known at the time of the publication of this manual. Handicare executes a policy of continuous improvement and reserves the right to changes and modifications.

The information provided is valid for the product in its standard version. Handicare cannot be held liable for possible damage resulting from specifications of the product deviating from the standard configuration.

The available information has been prepared with all possible diligence, but Handicare cannot be held liable for possible errors in the information or the consequences thereof. Handicare accepts no liability for loss resulting from work executed by third parties.

Names, trade names, etc. used by Handicare may not, as per the legislation concerning the protection of trade names, be considered as being available.

2008-01

<b>Preface</b> .....	<b>7</b>
This manual .....	7
Service and technical support.....	7
Identification plate.....	7
Approval .....	8
CE declaration.....	8
Warranty .....	8
Liability .....	10
Used wheelchairs and the environment.....	10
<b>1 Safety</b> .....	<b>11</b>
1.1 General safety instructions .....	11
1.2 Decals and instructions on the wheelchair.....	11
1.3 Electronic safety .....	11
<b>2 General description</b> .....	<b>12</b>
2.1 Product description Beatle/Puma .....	12
2.2 Main components .....	13
2.3 Controller 'DX'.....	14
2.4 Controller 'SHARK' .....	15
<b>3 Operating instructions</b> .....	<b>16</b>
3.1 Inspection before use .....	16
3.2 Getting in and out .....	16
3.3 Freewheel lever .....	17
3.4 Switching on the DX .....	17
3.5 Switching on the Shark .....	17
3.6 Driving .....	18
3.7 Driving forward and backward .....	19
3.8 Improper use .....	20
3.9 Curves .....	20
3.10 Obstacles.....	21
3.11 Slopes.....	21
3.12 Reducing the size of wheelchair .....	23
3.13 Transport .....	24
3.14 Electric adjustments.....	25
<b>4 Adjustment options</b> .....	<b>28</b>
4.1 Seat height adjustment.....	28
4.2 Seat belt .....	28
4.3 Adjustment 'Swing-away' controller .....	28
4.4 Adjustment 'Fixed' controller.....	28
<b>5 Maintenance</b> .....	<b>30</b>
5.1 Maintenance table .....	30
5.2 Batteries .....	30
5.3 Checking and inflating the tyres.....	31
5.4 Cleaning the wheelchair .....	31
<b>6 Troubleshooting</b> .....	<b>32</b>
6.1 Troubleshooting table 'DX'.....	32
6.2 Troubleshooting table 'Shark' .....	33
<b>7 Technical specifications</b> .....	<b>34</b>

## Preface

### This manual

With this manual you can safely use and maintain (clean) the wheelchair. This manual should be used in combination with Handicare's Sedeo seating system manual (supplement). In case of doubt always contact your dealer.

**Strictly follow the procedures with a caution symbol! These procedures can - if they are not executed properly - result in human injury or damage to the product or the environment.**

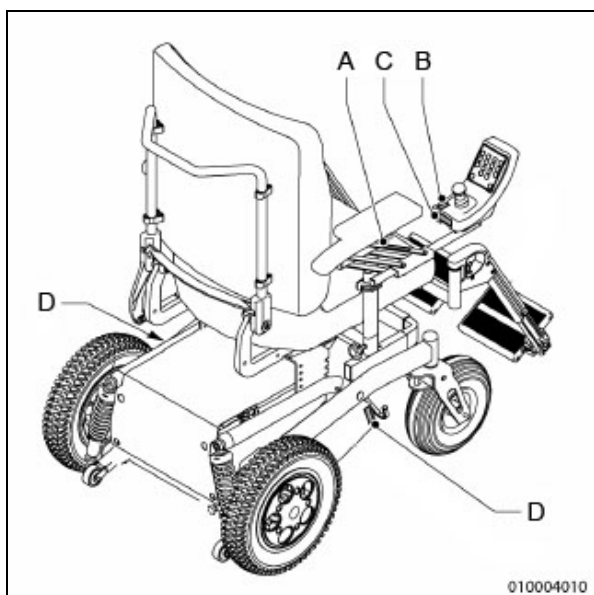
Figure 0.1

### Sticker markings on the wheelchair

- A. Beware of fracture!
- B. Speed sticker.
- C. Reference to manual.
- D. Neutral, left and right.

### Service and technical support

For information concerning specific settings, maintenance or repair work, please contact your dealer. Make sure you always mention the model, year of manufacture and identification number. This information is provided on the identification plate of the product (fig. 0.2).

Spare parts are available for the Beetle and Puma. A service manual, listing the spare parts among other items, is available for the dealer.

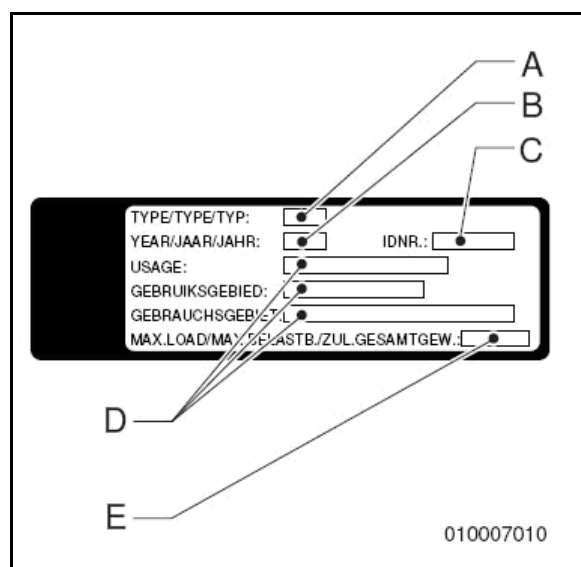
Figure 0.2

### Identification plate

- A. Model
- B. Year of manufacture
- C. Identification number
- D. Usage is indoors or outdoors
- E. Maximum load in kg

### Approval

The product meets the following standards:

1. The applicable requirements as specified in Annex 1 of the EU directive 93/42 EEC.
2. The requirements of NEN-EN 12182 (1999) Technical tools for the handicapped – General standards and testing methods October 1997 - specified in NEN-EN 12184 (1999).
3. NEN-EN 12184 (1999) Electrically driven wheelchairs, scooters and associated charging equipment – Standards and testing methods, April 1999.
4. The Beatle is classified according to EN12184 (1999) in class B.
5. The Puma is classified according to EN12184 (1999) in class B.
6. ISO 7176-8 Standards and test methods for impact, static, and fatigue strength, July 1998.
7. ISO 7176-14 Standards and test methods for controller systems of electrically driven wheelchairs 1997.
8. ISO 7176-9 Climate test for electrically driven wheelchairs.
9. ISO 7176-16 Standards for resistance to fire, May 1997.
10. The Beatle/Puma has been subjected to a dynamic crash test and meets the requirements imposed in ISO7176-19 in this regard.

The quality is guaranteed by an independent institution who controls the fulfilment of the above mentioned standards.

### CE declaration

The product is in conformity with the provisions of the

Medical Devices Directive and thus has CE marking.

### Warranty

In the following warranty and liability stipulations the terms and definitions as summed up hereafter must be explained as follows:

- **Product:** The hand-operated or electric wheelchair or scooter manufactured and delivered by Handicare.
- **Customer:** The person who directly obtains a Product from Handicare.
- **Dealer:** The person who delivers a Product obtained from Handicare to customers or third parties.
- **User:** The person who uses a Product manufactured by Handicare.

Irrespective of what is determined concerning warranty conditions applicable to the Product, in any case the following applies with regard to the warranty:

1. Except insofar as described otherwise Handicare guarantees the Product for its suitability for the purpose for which the Product is intended - all of these points as described in this manual - and for the quality of the material of which the Product is made and the manner in which the Product is manufactured.
2. Repairs or replacement of parts of the Product that may be necessary as a result of faults that are based on qualitatively faulty material or manufacturing errors will be executed free of charge, as long as such faults occurred within one (1) year after the date of delivery of the Product to the Customer. The parts to be replaced must be shipped post-paid to Handicare. Disassembly or assembly of these parts is at the expense of the

Customer. Therefore the following cases are not eligible for free repair or replacement as meant above:

- The repair or replacement that is necessary in connection with faults that arise after one (1) year after the date of delivery of the Product to the Customer;
  - The repair or replacement that is required in connection with faults due to improper or careless use of the Product or that are based on the Product being used for another purpose than the one for which it is intended, in which regard it applies that if the Customer is a Dealer, this Dealer will indemnify Handicare against possible claims from Users or other third parties for faults based on an incorrect or careless use of the Product;
  - Parts that are subject to wear, and the need for repair or replacement of the parts are the actual consequence of normal wear.
3. Irrespective of that stipulated under 2, it applies as far as an electric Product is concerned that with regard to the battery that forms part of the Product warranty is only given in case of faults or non-functioning of the battery that are demonstrably the direct consequence of material or manufacturing errors. A fault or non-functioning of the battery as a result of normal wear is not covered by the warranty as meant in these warranty stipulations. Similarly not covered by the warranty are faults or non-functioning that are the consequence of improper or unprofessional use of the Product or the battery that is part of the Product, including the incorrect charging of the battery and the failure to carry out timely and good maintenance, in which context it also applies that if the Customer is a Dealer, this Dealer will indemnify Handicare against possible claims from Users or other third parties that are based on the above mentioned improper or unprofessional use of the Product or the battery that is part of the Product.
  4. The warranty conditions as mentioned in above stipulations in any case become null and void if:
    - The guidelines of Handicare for the maintenance of the Product are not, or insufficiently, followed;
    - A necessary repair or replacement of parts is based on neglect, damage or abuse of the Product or a use of the Product for another purpose than the one for which it was intended;
    - Parts of the Product are replaced by parts of another origin than those which Handicare uses and/or parts of the Product are replaced without the permission of Handicare.
  5. The warranties as mentioned in stipulations 1 through 3 also become null and void if what is involved is re-used by a new user within the warranty period and that such re-use necessitates adaptations to the product and those adaptations are carried out without the instructions and/or at the order of Handicare.
  6. To retain rights under the above delineated warranties the Customer must, in case of damage or other calamities, as rapidly as possible contact Handicare and inform them as fully as possible. The possibility of taking recourse to the above-mentioned warranty

conditions becomes null and void for the Customer in any case after 20 workdays after the claim situation or the calamity arises that was the reason for the recourse to the guarantee.

7. The replacement of a part or the repair or the reconditioning of the Product within a current warranty period does not extend the warranty period.
8. Handicare gives no warranty on repair to or reconditioning of the Product carried out other than under order of and/or at the instructions of Handicare. If repairs and/or reconditioning are executed by or on behalf of a Customer, the Customer indemnifies Handicare with respect to the claims of third parties who result, in the broadest sense of the word, from such repairs or reconditioning.

## Liability

Irrespective of what is determined regarding liability in the general conditions applicable to the Product, with regard to liability in any case the following applies:

1. Taking into consideration the following stipulations, Handicare only accepts liability for loss due to death or physical injury that is the result of a defect in the Product for which Handicare is responsible and for damage to another object that is the private property of the user of the Product, as long as said loss is the direct result of a fault in the Product.
2. Handicare accepts no other or further liability than delineated under 1. In particular Handicare accepts no liability for

consequential damage, in any form whatsoever.

## Used wheelchairs and the environment

If your wheelchair has become superfluous or needs to be replaced, it can usually be taken back by your dealer. If this is not possible, please contact your local authorities for the possibilities of recycling or an environmentally friendly way of disposing of the used materials.

For the production of the wheelchairs several plastics and metals have been used. The wheelchair also contains electronic components that must go to electronic waste. Used batteries belong to chemical waste.

# 1 Safety

Handicare accepts no liability whatsoever for damage or injury caused by the failure to (strictly) adhere to the safety guidelines and instructions or else due to carelessness during the use and cleaning of the wheelchair and possible accessories. Depending on the specific work circumstances or the accessories used, additional safety instructions may be required. Please contact your dealer immediately if you observe a potential danger during the use of the product.

## 1.1 General safety instructions

- **The user of the wheelchair is at all times fully responsible for the fulfilment of the locally applicable safety regulations and guidelines.**
- **The technical specifications must not be altered.**
- **Modifications to (parts of) the wheelchair are not permitted.**

## 1.2 Decals and instructions on the wheelchair

- **Signs, symbols and instructions placed on this wheelchair comprise part of its safety facilities. They must never be covered or removed and must remain present and clearly legible throughout the entire lifespan of the wheelchair.**
- **Immediately replace or repair illegible or damaged signs, symbols and instructions.**

**Contact your dealer with regard to this.**

## 1.3 Electronic safety

- The standard version of your electric wheelchair has been tested according to the applicable EMC requirements. Mobile telephones have no influence on the driving behaviour of the wheelchair. When using a mobile telephone in the vicinity of an electric wheelchair with special adaptations, you are advised to first switch off the electric wheelchair.
- Your electric wheelchair may influence electromagnetic fields such as alarm systems.
- If the electronics of the wheelchair is not properly screened off, this may affect sensitive electrical appliances, such as shop alarms and garage openers. The electric wheelchair has been tested for this. Should any problems of this nature occur, please report these to your dealer immediately.

## 2 General description

### 2.1 Product description Beatle/Puma

The Beatle/Puma is an electrically driven wheelchair with joystick control for usage under various circumstances, for use indoors and outdoors.

The seating on the Beatle / Puma is securely attached by means of a link frame (also referred to as the interface) to the undercarriage (also referred to as the carrier) in which the entire electrical installation is contained. The drive consists of two electric motors, one on each drive wheel. The steering of the Beatle / Puma on curves takes place by the change in the speed of one of the motors, making the wheelchair turn left or right.

The range of settings provides optimal support of the body, making the Beatle / Puma a pleasant and comfortable product for active lifestyles.

The Beatle/Puma has been designed for:

- Transport of persons to a maximum weight of 160 kg;
- Use on paved roads, pavements, foot- and bicycle paths.
- Use in and around the house.

Your dealer must give you good driver's instructions before you can independently operate the product and participate in traffic.

You must be able to correct the consequences of actions while driving the wheelchair.

Consider operating the wheelchair under the supervision of an experienced dealer during first operations.

The wheelchair is not a motorised vehicle in the sense of traffic regulations.

The Beatle/Puma has a maximum speed of 6 or 10km/h.

If you use the wheelchair for other purposes than those for which it is intended, Handicare accepts no liability whatsoever for damage or injury resulting from such use other than for which the wheelchair was developed and designed.

**Please study this manual carefully before you use the wheelchair.**

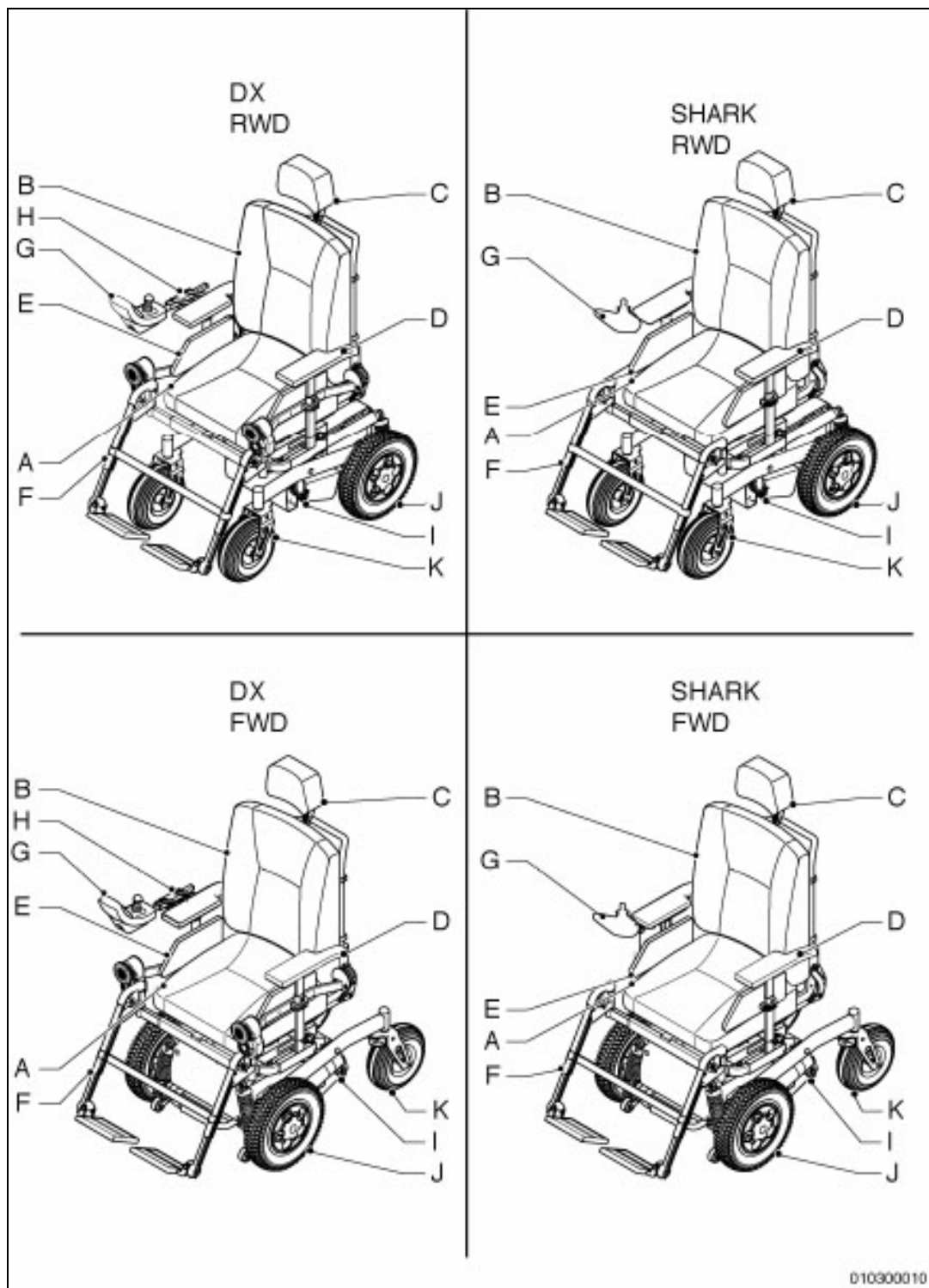
Figure 2.1

## 2.2 Main components

- A. Seat
- B. Backrest
- C. Headrest (option).
- D. Arm-rest
- E. Side panel.
- F. Leg rest with footrest
- G. Controller, type 'Shark' or "DX"
- H. Controller attachment, fixed or swing away.
- I. Free wheel lever
- J. Drive wheels
- K. Swivel castors

### 2.3 Controller 'DX'

The controller contains the following main components:

- Control panel (A in fig. 2.2).
- Joystick (B in fig. 2.2).
- Charge connection (C in fig. 2.2), see 'charging batteries'.

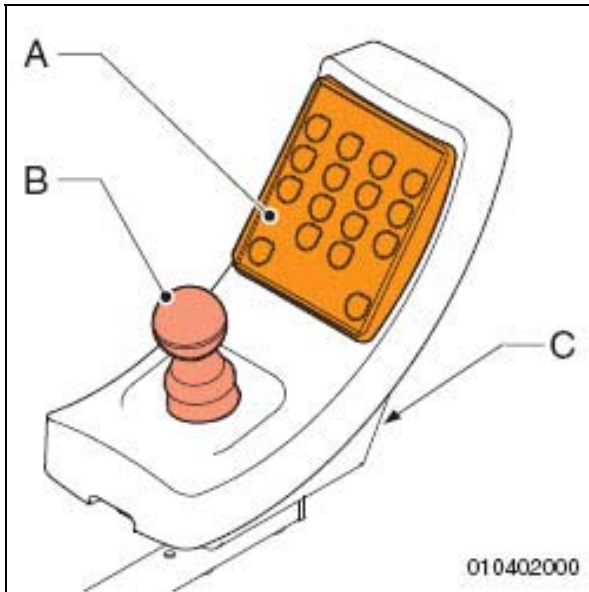
Figure 2.2

All functions that can be installed on the wheelchair can be found on the control panel. This includes all standard functions and all other optional functions.

The joystick steers the wheelchair and adjusts various settings, if these are installed.

The charging connection connects the battery charger to charge the batteries.

### Control panel

The control panel contains the following functions (see fig. 2.3):

- A. Panel pad
- B. On/off key
- C. Speed (drive profile)
- D. Battery meter
- E. Horn
- F. Lighting (option)
- G. Turn signal left (option)
- H. Turn signal right (option)
- I. Hazard lights (option)
- J. High/low seat adjustment (option)
- K. Electrical tilt adjustment of the seat (option)
- L. Electrical adjustment of the angle of the backrest (option)
- M. Electrical adjustment of the left leg-rest (option)
- N. Electrical adjustment of the right leg-rest (option)

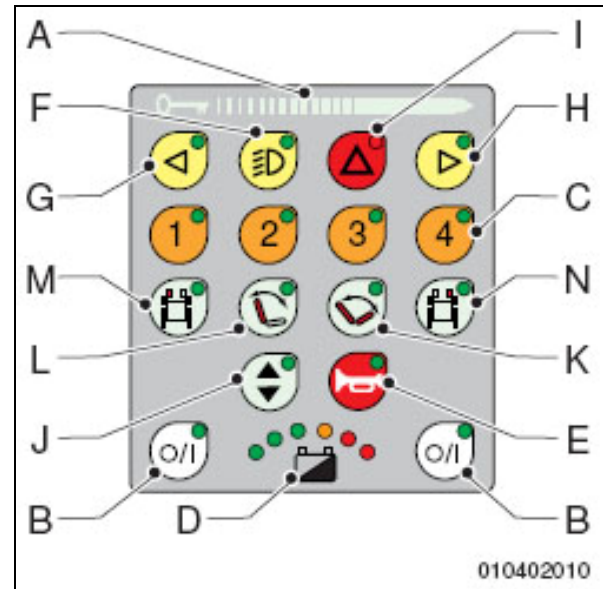
Figure 2.3

## 2.4 Controller 'SHARK'

The controller contains the following components (See fig. 2.4, 2.5 and fig. 2.6):

- A. On/off key
- B. Joystick
- C. Charge connection
- D. Horn
- E. Battery indicator
- F. Speed control
- G. Indicators
- H. Lightning
- I. Warning lights
- J. Actuators/adjustment options

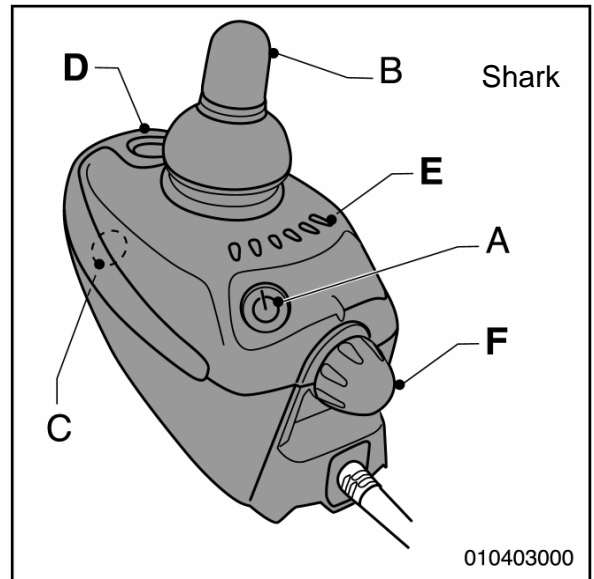
Figure 2.4

**Performance adjustments should only be made by professionals conversant with the adjustment process and the operator's capabilities. Incorrect settings could cause injury to the operator or bystanders, or damage to the vehicle or surrounding property.**

The joystick is for driving the wheelchair and adjusting the various electrical options, if installed.

### Battery indicator

The battery indicator shows the Condition of the batteries. When all the lights are on, the batteries are fully charged. As the batteries are depleted, lights from right to left will go out.

**If only the red lights are on, the Batteries must be charged immediately.**

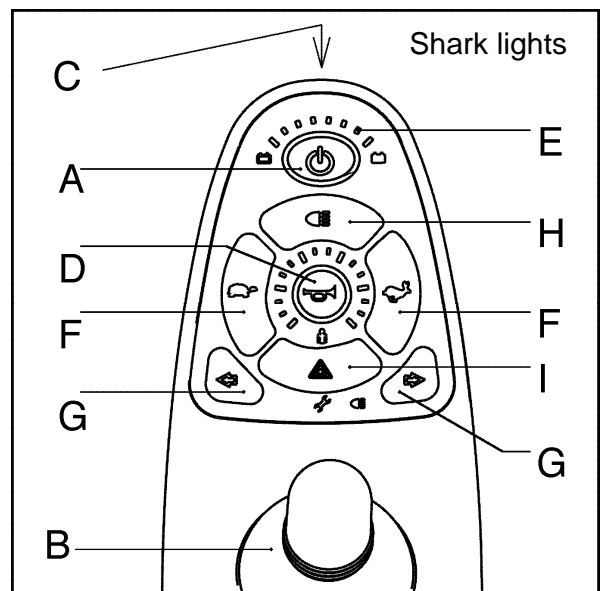
Figure 2.5

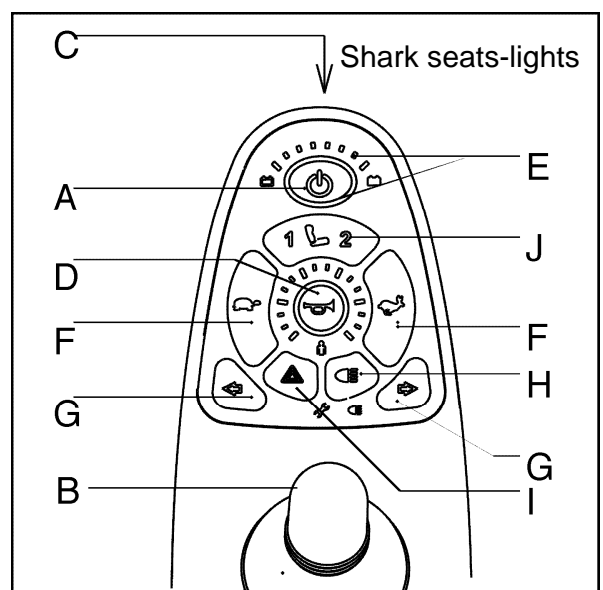
Figure 2.6

## 3 Operating instructions

### 3.1 Inspection before use

It is important to check the following points before you start driving the wheelchair:

- If the lights and indicators are working properly.
- If the tyres are sufficiently inflated.
- If the batteries are charged sufficiently. The green battery indicator lights must be on.

**In winter, the batteries have a lower capacity. In case of light frost, the capacity will drop to approximately 75% of the normal capacity. At a temperature of less than -5° the capacity will be approximately 50% of the normal capacity. This will affect the range.**

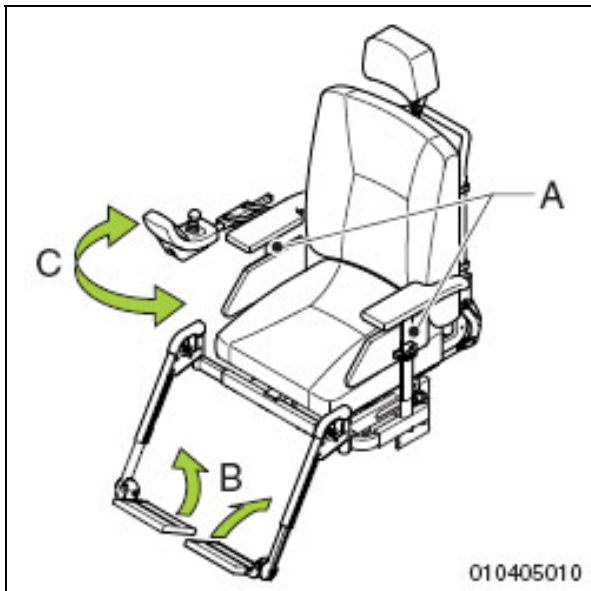
Figure 3.1

### 3.2 Getting in and out

1. Make sure the wheelchair is switched off electrically.
2. Put the free wheel lever in the Drive position.

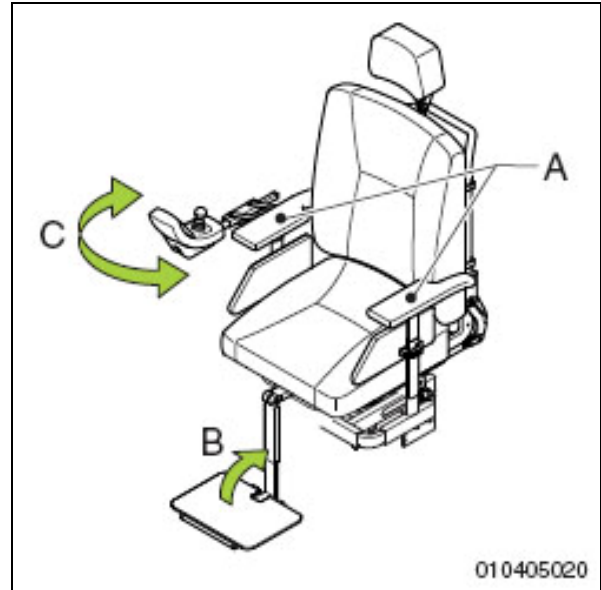
Figure 3.2

#### Transfer forwards

1. Fold up the foot support (B in fig. 3.1 and fig. 3.2)
2. Swing away (if possible) the leg rests
3. Seat yourself in the chair.

#### Transfer sideways

1. Loosen the star knob and remove the armrest (A in fig. 3.1 and fig. 3.2) .
2. Seat yourself in the chair.

Depending on the version of the controller and the location of the transfer, it can be necessary that the controller be swung away (C in fig. 3.1 and fig. 3.2).

**Switch off the wheelchair electrically before you get in or out! This way you will avoid that the wheelchair starts to move when you accidentally move the joystick.**

### 3.3 Freewheel lever

Put both freewheel levers (A in fig. 3.3) in the Drive position to be able to drive the wheelchair electrically.

To be able to push the wheelchair, put both freewheel levers in the **PUSH** position.

The freewheel levers must be operated by the attendant.

- **Never put the wheelchair in the Drive position when you are parked on a slope. When the wheelchair is parked on a slope that is too steep, it can start moving down the slope.**
- **After pushing, the free wheel lever should immediately be returned to the Drive position.**
- **The free wheel lever should only be put in the Push position if the wheelchair is to be pushed. In the Drive position the automatic parking brake is activated.**

### 3.4 Switching on the DX

Activate the wheelchair by moving the magnetic key from left to right across the panel (A in fig. 3.4).

Press the ON/OFF key (B in fig. 3.4) to switch the wheelchair on.

### 3.5 Switching on the Shark

To switch on the wheelchair, push the on/off key (A in fig. 3.5).

**When an electronics error is detected, the battery indicator will start flashing. The electronics are blocked.**

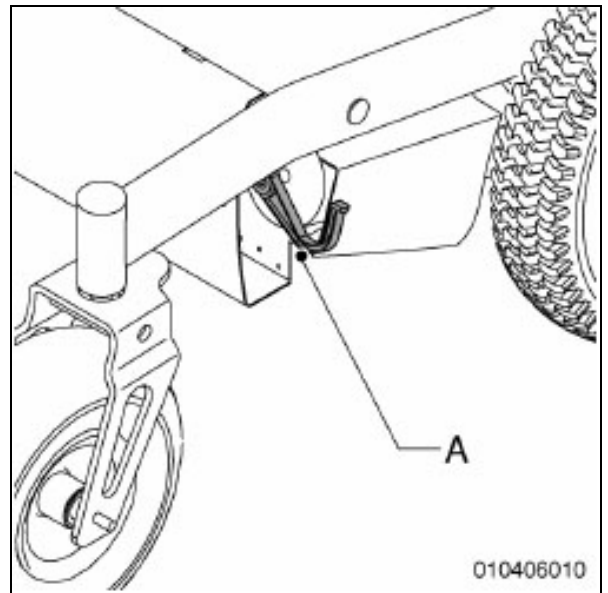
Figure 3.3

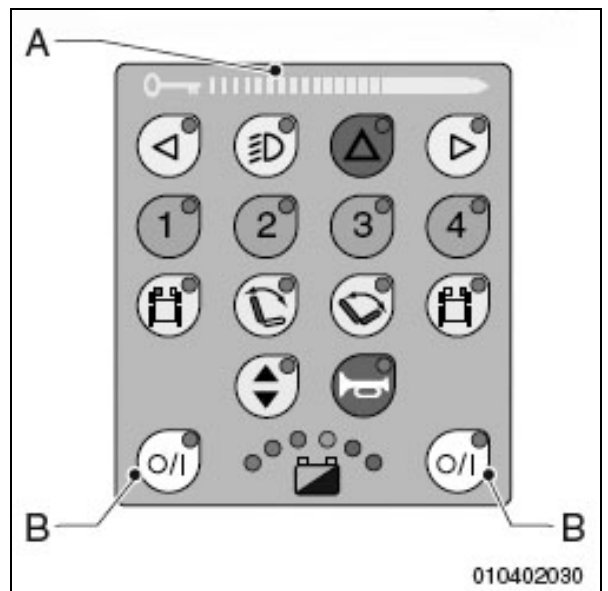
Figure 3.4

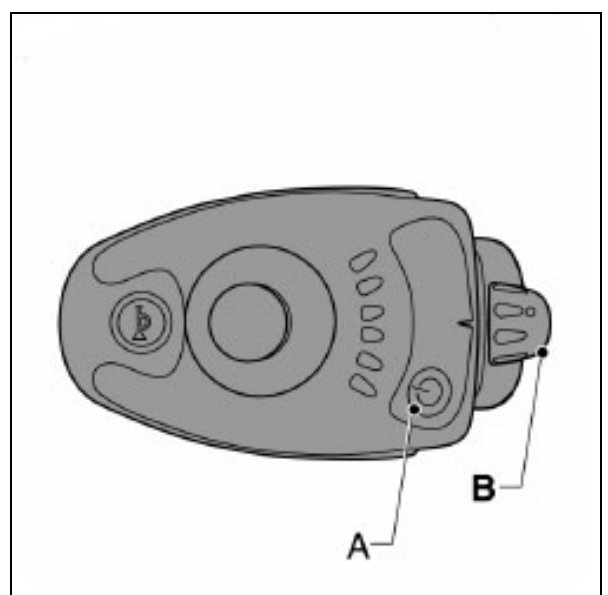
Figure 3.5

### 3.6 Driving

**As a wheelchair user you could be vulnerable in traffic. When travelling in or across a road, keep in mind that other traffic participants may not always notice you. Keep to the applicable traffic regulations.**

**Avoid out-of-the way routes, so that help can be on its way quickly when needed.**

#### **Inspection before use**

It is important to check the following points before you start driving the wheelchair:

- If the lights and indicators are working properly.
- If the tyres are sufficiently inflated.
- If the batteries are charged sufficiently. The green battery indicator lights must be on.
- If the free wheel lever is in its Drive position

- **The joystick must be completely in the zero position (centred), before switching the wheel-chair on. Otherwise the electronic system will malfunction. As an indication the lights of the battery light up.**
- **To correct this fault the wheelchair must be switched off and then back on, with the joystick in the 'zero position'. This safety fuse feature prevents the wheelchair from unintended movement.**

### 3.7 Driving forward and backward

- Switch on the wheelchair
- Choose the desired speed value:
  - With key 1, 2, 3 or 4 (A in fig. 3.6) for controller 'DX'.
  - **⚠ Maintain a maximum of 3 indoors or in pedestrian areas.**
  - With the rotary knob (D in fig. 3.7) for controller 'Shark'.
  - With the speed keys (F in fig. 3.8) on the 'Shark seats/lights & actuators' controller.
- Push the joystick carefully forward or backward, depending on which direction you want to go, and the wheelchair will begin to move.

**With the joystick you can also control the speed of the wheelchair. The more you push the joystick forward from the neutral position, the faster the wheelchair goes forward or backward.**

- **The wheelchair reacts very quickly to force on the joystick.**
- **A smooth operating characteristic is obtained by even force on joystick.**

#### Braking

When you release the joystick or move it into the central position, you will brake automatically.

#### Emergency stop

1. When driving forward pull the joystick all the way backward with one move. When reversing, push the joystick all the way forward with one move.
2. Immediately release the joystick when the wheelchair has come to a standstill.

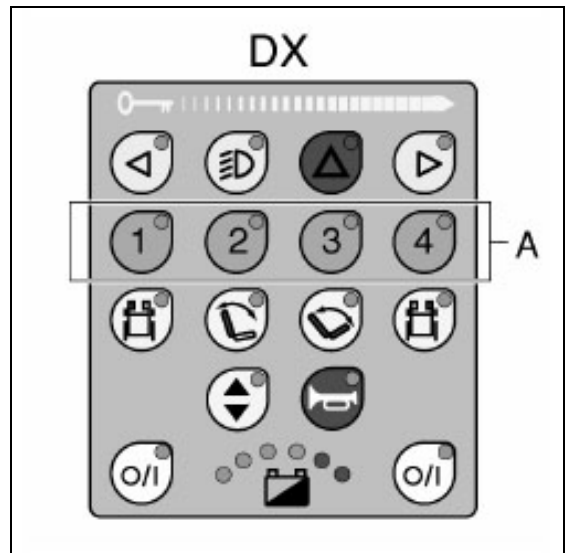
Figure 3.6

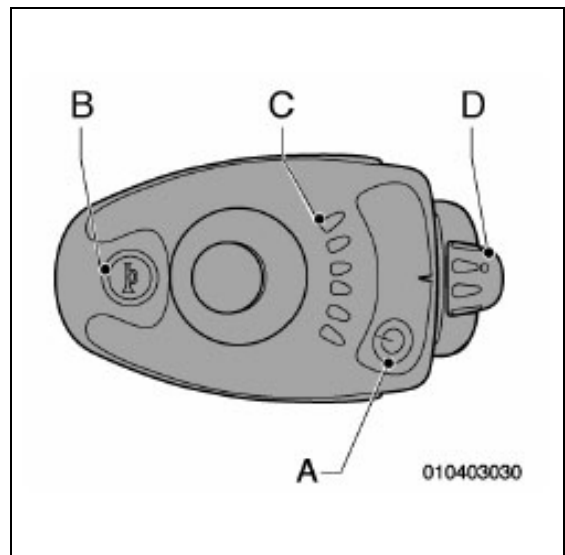
Figure 3.7

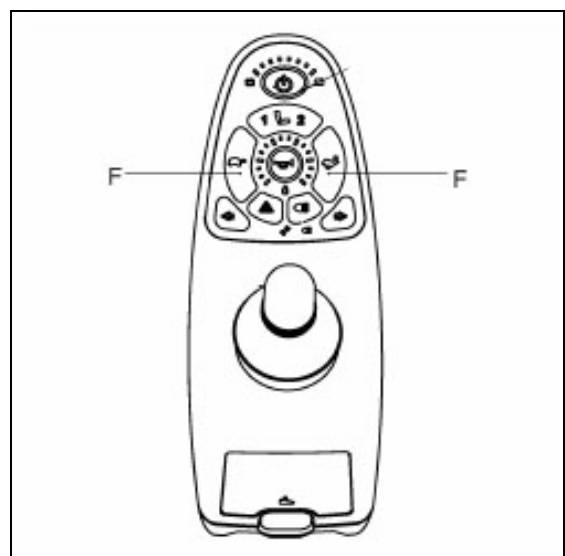
Figure 3.8

### 3.8 Improper use

Follow the instructions with a caution symbol and consider suggestions! Cautions concerning improper use of the product have been included in the chapter on the concerned subject where possible. Cautions for improper use can be recognized by the caution symbol:

#### Miscellaneous improper use

- Drive carefully on slippery roads, resulting from rain, ice or snow!
  - Prevent the wheelchair from coming into contact with seawater: seawater is caustic and may damage the wheelchair.
  - Prevent the wheelchair from coming into contact with sand: sand may affect the moving parts of the wheelchair, resulting in unnecessarily rapid wear.
  - Do not put the wheelchair in direct sunlight: the upholstery may become too hot, which may cause burning or an allergic reaction of the skin.
  - Do not drive the wheelchair when you are under the influence of drugs, alcohol or medication that could affect your ability to drive.
  - You must have sufficient eye sight to be able to safely drive the wheelchair.
  - It is imperative to turn on the lights in case of reduced visibility.
  - When driving at a high speed you must be extra careful. Select a lower maximum speed indoors, on pavements and in pedestrian areas.
- Be sure that no items of clothing hang loose. These could be caught between the wheels.
  - When driving on a slope, always make sure that all adjustment options are in the lowest position and that the leg rests are in the lowest position.
  - Do not add weight to the wheelchair without obtaining advice from an expert. You could inadvertently affect the stability of the product.
  - Do not push and/or drag objects with the wheelchair.
  - Do not open doors with the leg rests.
  - Do not use the footrests when transferring.
  - Do not drive off high obstacles.
  - Do not alter the electrical circuit.
  - Do not seat more than one person on the wheelchair.

### 3.9 Curves

- Never take curves at full speed. Decrease speed before taking a curve.
- Use the indicators when changing direction.

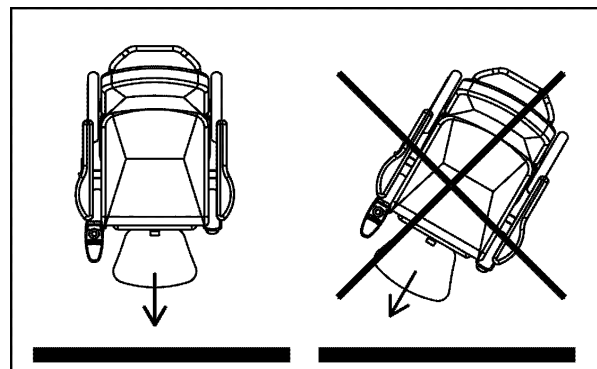
Figure 3.10

### 3.10 Obstacles

#### Driving up obstacles

1. Drive straight up to the kerb (fig. 3.10).
2. Slowly move the joystick forward. Drive up the pavement without changing direction.
3. As soon as the front wheels are on the pavement, you must keep your speed for the rear wheels to be able to move up the pavement as well.  
If you cannot drive up onto the pavement, find a lower place where you can.

#### Driving down obstacles

1. Drive straight up to the edge of the pavement.
2. Slowly move the joystick forward. Drive carefully down the pavement without changing direction.

**Do not drive down stairs!**

### 3.11 Slopes

1. **Always drive slowly and with awareness on slopes.**
2. **Avoid sudden and jerky movements.**
3. **Avoid directional changes on slopes.**
4. **Do not turn on slopes.**
5. **Only drive up a slope with all adjustment options in the lowest position and the leg rests in the lowest position.**
6. **It can be very dangerous to reverse down a slope.**
7. **Do not drive down slopes with loose gravel or a sandy surface, as one of the driving wheels might slip.**
8. **If you notice a strong decrease in speed when driving up a slope, take a less steep route.**
9. **Prevent the wheelchair from gaining too much speed.**
10. **Driving on a slope too long can cause overheating.**
11. **When driving forward down a slope with a FWD wheelchair we always advise the use of an anti-tip device.**

## Operating instructions

### Safe slope

The Puma and Beatle meet the requirements of a class B (indoor/outdoor) wheelchair in accordance with the European standard EN12184 (1999). The Puma and Beatle are available in various configurations. Different configurations have different specifications. With regard to the maximum safe slope, a distinction is made between front-wheel driven wheelchairs (FWD) and rear-wheel driven wheelchairs (RWD). See the table below.

Slopes tested in accordance with EN12184 (1999) (Class B Indoor/Outdoor)	
	Maximum safe slope
Puma/Beatle <b>FWD</b> Front-wheel driven	6°
Puma/Beatle <b>RWD</b> Rear-wheel driven	10°

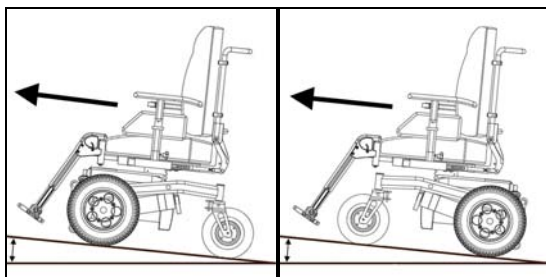
Figure 3.11 Driving up a slope; left FWD, right RWD.

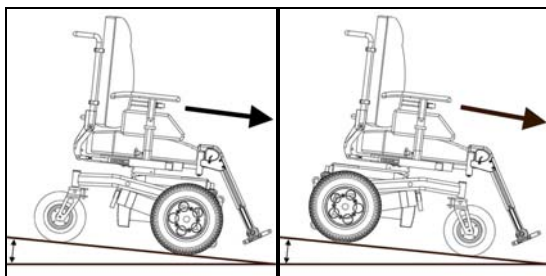
Figure 3.12 Driving down a slope; left FWD, right RWD.

Slopes steeper than the maximum safe slope.

The Puma and Beatle are capable of tackling slopes steeper than the maximum safe slope. This capability ensures optimal everyday use of the wheelchair.

Driving on slopes steeper than the maximum safe slope can engender safety risks with regard to stability. This requires the user to exert considerable caution and full control of the wheelchair. Always drive carefully and never experiment! Follow the instructions with a caution symbol!

The stability and performance of a wheelchair depend on many variables. Wheelchairs are adapted to the individual user. Consequently, the variables will vary from wheelchair to wheelchair. Therefore, have your dealer properly inform you about the use of the wheelchair and whether the specific settings or personal adaptations will affect the driving behaviour of the wheelchair.

### 3.12 Reducing the size of wheelchair

#### Removing the seat

1. Pull up the seat (A in fig. 3.13) in the front.
2. When replacing the seat, the clips (B in fig. 3.13) must click around the frame (C in fig. 3.13) of the chair.

**Check if the seat is clicked in place securely.**

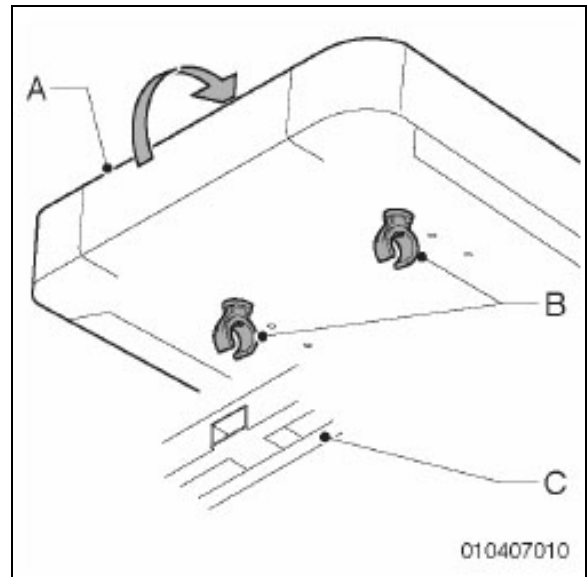
Figure 3.13

#### Folding down the backrest

1. Flip the lock clip (A in fig. 3.15) off the adjusting bar.
2. Pull the lock clip from the hinged joint.
3. Fold down the backrest.

#### Removing the armrest

Loosen the cap and remove the armrest.

**The controller remains connected to the wheelchair with the cable.**

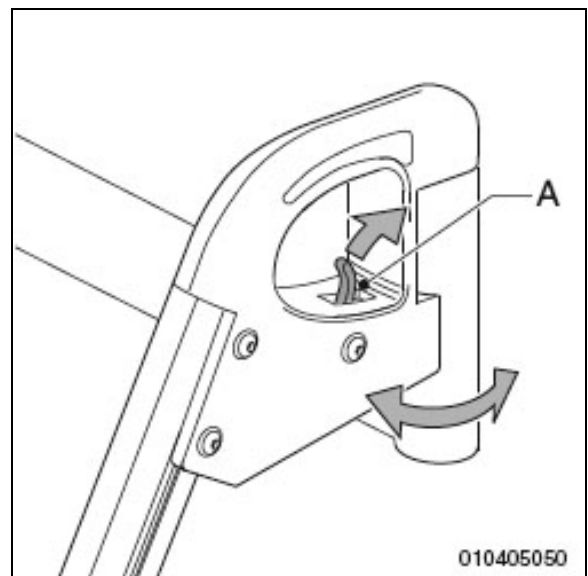
Figure 3.14

#### Removing the legrest

1. Push the catch (A in fig. 3.15) backward to unlock the leg rest.
2. Swing the legrest (B) outward.
3. Raise the legrest and remove it from the guide.

**Make sure you hear a click when you put the legrest back in place. The legrest is now securely locked.**

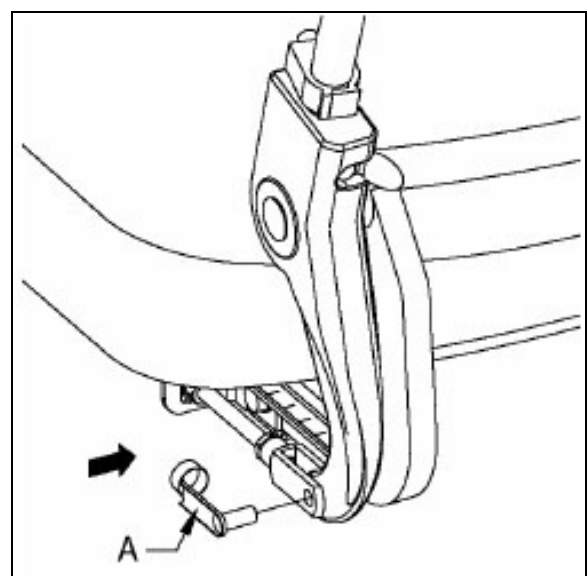
Figure 3.15

### 3.13 Transport

The wheelchair on its own can be transported in a car. The best way to do this, is by removing the armrests and legrests (see chapter 3.12). If your car is small, there is also the possibility of reducing the size of the chair by flipping the backrest down (see chapter 3.12). Use steel ramps to place the wheelchair in the car.

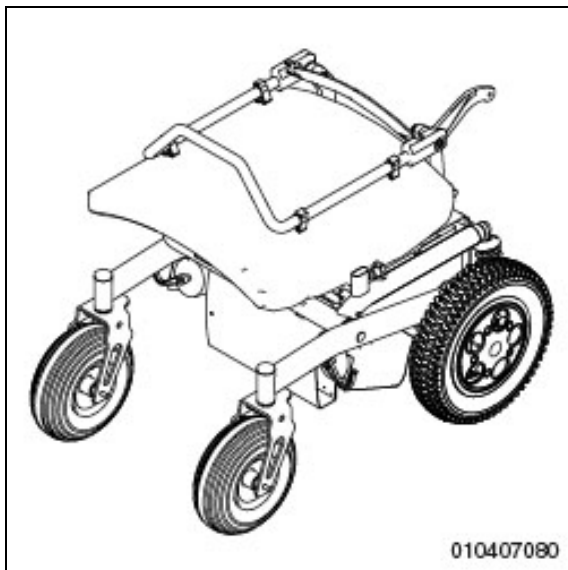
Figure 3.16

Handicare recommends wheelchair users to travel in a regular car seat whenever possible. A wheelchair is not designed as a car seat and cannot offer the level of safety that is offered by standard car seats, no matter how well the wheelchair is fixed to the vehicle concerned.

It is not possible for every wheelchair user to make a transfer to a regular car seat. The wheelchair needs to be secured by using a tie-down system that is tested and approved according to ISO10542 with a max weight suitable for the wheelchair.

The weight of the wheelchair depends on the configuration of the wheelchair. The overview below shows how several options influence the total weight of the wheelchair. To be sure

you use the right tie-down system, always weigh the total wheelchair.

	Beatle	Puma
<b>Total weight</b> Basic configuration without batteries	73kg (160.9Lbs)	74,4kg (164Lbs)
<b>Batteries (per two)</b> 40Ah	+29,2kg (64.4l.)	
56Ah		+43,6kg (96.1l.)
73Ah		+47,2kg (104.1l.)
<b>Wheels (per two)</b> Puncture proof drive	+2,4kg (5.3l.)	+3,8kg
Puncture proof castors	+0,8kg (1.76l.)	+2,8kg (6.17l.)
Suspended castor forks		+2,5kg (5.5l.)
<b>Electrical options</b> Elec. High/Low		+23kg (50.7l.)
Elec. Tilt in space		+24kg (52.9l.)
Elec. Legrests (per two)		+4kg (8.8l.)
<b>Worktop</b>		+2,6kg (5.7l.)

- **The Beatle/Puma has been subjected to a dynamic crash test and meets the concerned requirements stipulated in ISO7176-19.**
- **You should always ask for confirmation from the transporter that the vehicle is suitably designed, insured and equipped to transport a passenger in a wheelchair.**
- **Parts of the wheelchair that can be easily detached (accessories) should be removed and stored elsewhere during transportation.**
- **The wheelchair should always be transported in a forward facing direction.**
- **Handicare recommends a tie-down system that is tested and approved according to ISO10542 with a max weight suitable for the Puma and Beatle.**

- There are two commonly used restraint systems, one using hooks/karabiners and the other uses straps. When a restraint system with straps is used, your wheelchair needs to be equipped with heavy duty brackets. The use of the strap based tie-down system without using these brackets can lead to excessive wear of the straps with the risk of breakage.
- The wheelchair should be secured only at the tie-down points on the frame (figure 3.17), not onto other parts of the wheelchair or onto any accessories (armrests, anti-tip brackets etc).
- Handicare recommends that the wheelchair should be supplied with a headrest and that this should be used during transportation.
- During transportation it is essential that you are secured by a three point lap belt, which is anchored to the vehicle wall and floor rails. A seat belt on your wheelchair must not be considered as an alternative to the occupant restraint system.

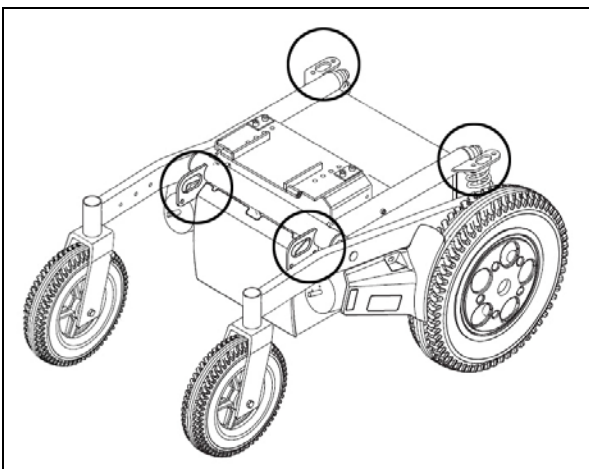
Figure 3.17: Tie-down points for safe transport. The tie-down points are the same for front wheel and rear wheel drive.

### 3.14 Electric adjustments

The Beatle / Puma may be equipped with electric (optional) adjustments, which can be operated from the controller.

The following electric adjustments may have been provided:

- Seat with electric height adjustment
- Seat with electric tilt adjustment
- Backrest with electric angle adjustment
- Leg-rest with electric angle adjustment

With the DX version, the keys intended for the operation of the electric adjustments are standard mounted on the controller panel.

With the 'Shark' version, available with maximum two options, you have to push the adjustment key (J in fig. 2.6) several times to select the various adjustment options.

**Remark:**

**When you drive up or down a slope of more than 4 degrees, you must put all electric adjustments back into the original position.**

#### Seat with electric height adjustment

The height of the whole seating system of the Beatle / Puma you can adjust electrically yourself.

As soon as the seating system has been raised a couple of centimetres, the light in the on/off switch will start flashing. This is to indicate that the wheelchair can now only be driven at half speed.

Therefore, while you are adjusting the seating system, do not confuse the blinking on/off light with the diagnostic function of this lamp.

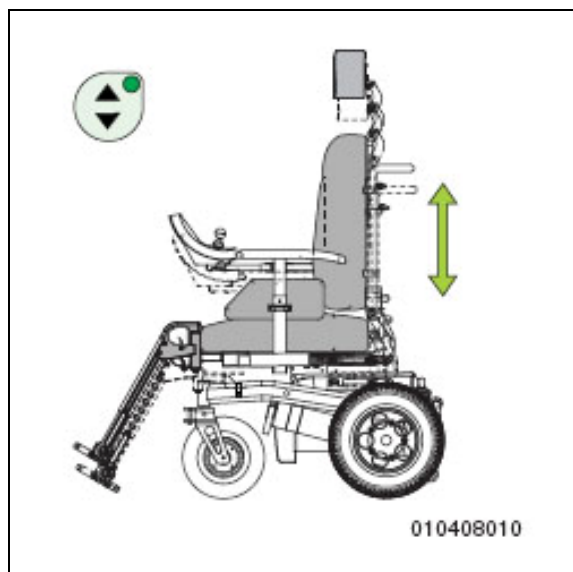
Figure 3.17

### Adjusting with 'DX' control:

1. Switch on the electrical wheelchair.
2. On the control panel of the controller press the symbol for the height adjustment of the seating system. The light of the key should now go on. The wheelchair can no longer drive.
3. Now move the joystick to the front: the seating system will be raised. Or move the joystick to the back; the seating system will move down.
4. Press the symbol once more to switch on the function as soon as the desired height has been reached.

### Adjusting with 'Shark' control:

1. Switch on the wheelchair.
2. Push the adjustment key (J in fig. 2.6). The indicator light of the key will go on. You can no longer drive the wheelchair.
3. Move the joystick forward: the adjustment option moves forward, or move the joystick backward: the adjustment option moves backward.
4. Push the adjustment key (J in fig. 2.6) to switch off the adjustment function.

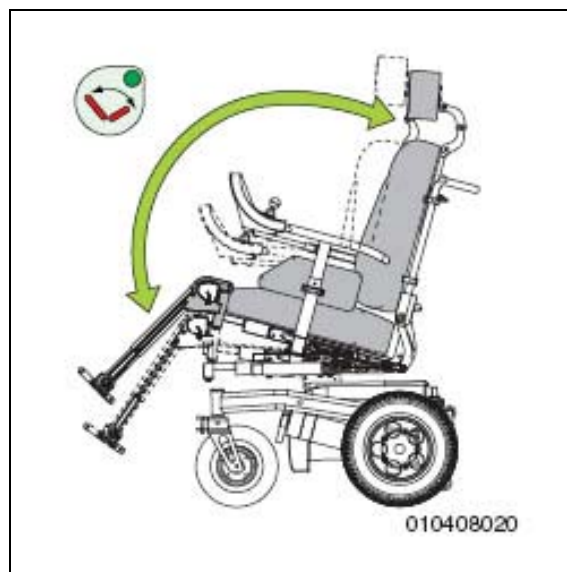
Figure 3.18

### Seat with electric tilt adjustment

The seat part of your wheelchair can be set at an angle that is comfortable for you. In this adjustment, the angle of the entire seating system is tilted with respect to the bottom part of the chair. Also, this adjustment can be used to tilt the seating system backward, to facilitate getting out.

**For each degree that the seat is tilted backward, the seat height is raised by approximately 5 mm (1/4").**

### Adjusting with 'DX' control:

1. Switch on the electrical wheelchair.
2. On the control panel of the controller press the symbol for the tilting adjustment of the seating system. The light of the key should now go on. The wheelchair can no longer drive.
3. Now move the joystick to the front: the seating system will tilt forward. Or move the joystick to the back: the seating system will tilt backward.

4. Press the symbol once more to switch on the function as soon as the desired tilting angle has been reached.

### Adjusting with 'Shark' control:

See 'Seat with electric height adjustment'.

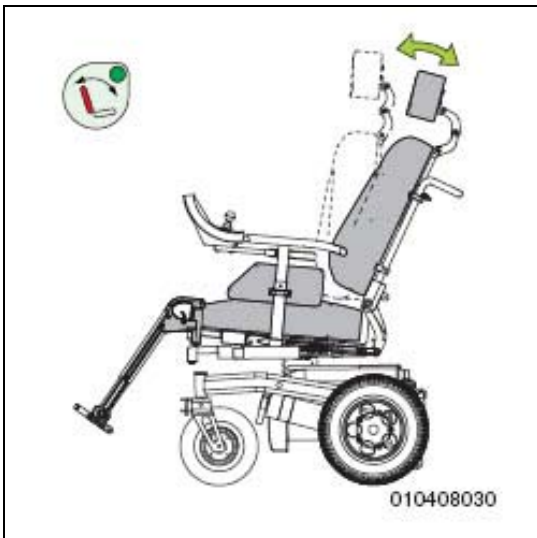
Figure 3.19

### Backrest with electric angle adjustment

For optimal seating comfort, you can adjust the angle of the backrest yourself. This is done using continuous adjustment.

### Adjusting with 'DX' control:

1. Switch on the electrical wheelchair.
2. On the control panel of the controller press the symbol for the angle adjustment of the backrest of the seating system. The light of the key should now go on. The wheelchair can no longer drive.
3. Now move the joystick to the front: the backrest will tilt forward. Or move the joystick to the back: the backrest will tilt backward.
4. Press the symbol once more to switch on the function as soon as the desired angle has been reached.

### Adjusting with 'Shark' control:

See 'Seat with electric height adjustment'.

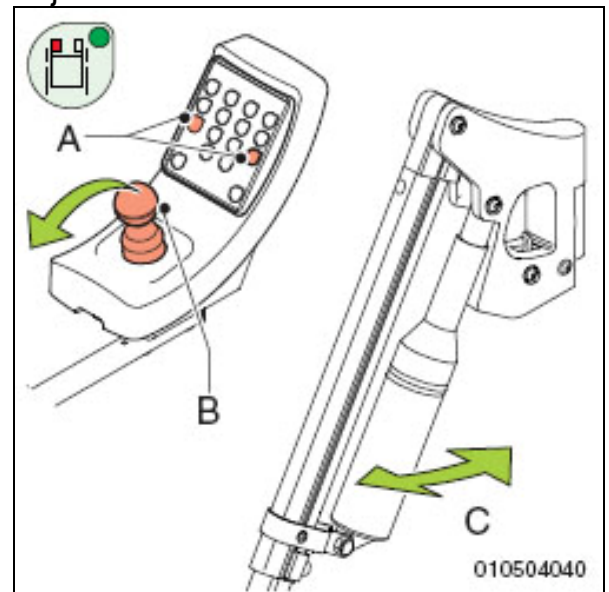
Figure 3.20

### Leg rest with electrical motor angle adjustment

The comfort leg-rests with electrical set motor can be independently adjusted. The motor on each leg-rest provides enough power to lift the leg rest without difficulty.

The adjustment of the angle of the leg rests goes like this:

- Switch the electrical wheelchair on.
- On the control panel of the controller, press the symbol for angle adjustment of the leg rest (A in fig. 3.13) of which the angle (C in fig. 3.20) has to be adjusted. The light of the pressed symbol must now be on.
- Now move the joystick (B) forward to adjust the leg-rest (C) upward or backward to bring the leg-rest down.
- Press the relevant symbol again to switch it off, or press on the symbol of the other leg-rest to adjust this one as well.
- Once again press the lit symbol, to switch off the function of the angle adjustment.

### 4 Adjustment options

#### 4.1 Seat height adjustment

The seat height of the Beatle / Puma can be adjusted to five levels in steps of 25 mm by your dealer.

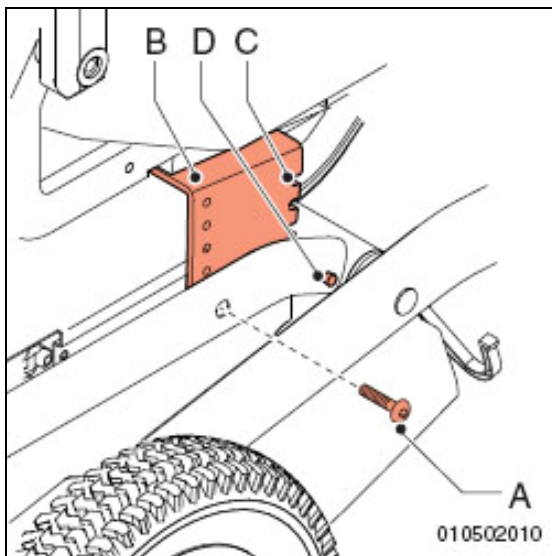
Figure 4.2

#### 4.2 Seat belt

It is possible to equip the wheelchair with a seat belt. The seat belt gives extra sitting stability during normal use. Ask your dealer for information.

For the assembly of the seat belt do the following:

- Fasten the belt buckle (A in fig. 4.2) with the aid of a bolt plus nut.

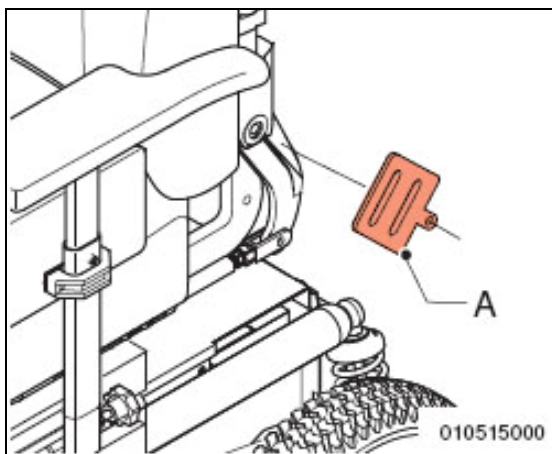
Figure 4.2

#### 4.3 Adjustment 'Swing-away' controller

##### Height

1. Loosen the bolt (A in fig. 4.3) a few turns with a 3 mm Allen key.
2. Slide the controller with the swing-away mechanism in the desired position.
3. Tighten the bolt securely.

##### Depth

1. Loosen the bolt (B in fig. 4.3) a few turns with a 4mm Allen key.
2. Slide the controller in the desired position.
3. Tighten the bolt securely.

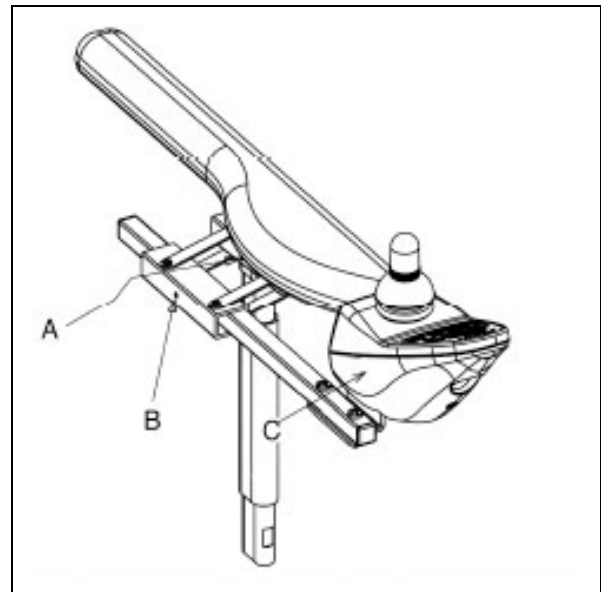
Figure 4.3

#### 4.4 Adjustment 'Fixed' controller

##### Depth:

1. Loosen the bolt (B in fig. 4.4) a few turns with a 4mm Allen key.
2. Slide the controller (C in fig. 4.4) in the desired position.
3. Tighten the bolt securely.

### Height:

1. Loosen the bolt (A in fig. 4.4) a few turns with a 3 mm Allen key.
2. Slide the controller (C in fig. 4.4) in the desired position.
3. Tighten the bolt securely.

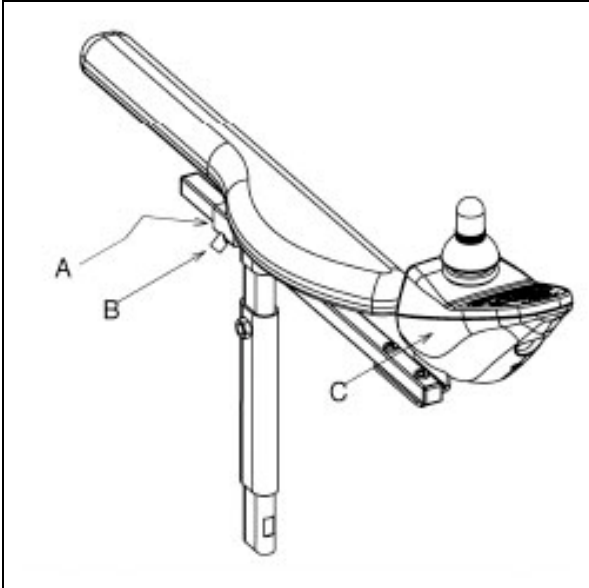
Figure 4.4

## 5 Maintenance

### 5.1 Maintenance table

It is recommended to have your wheelchair serviced by your dealer once a year or, in case of intensive use, every six months. The table below shows the maintenance activities you can perform yourself.

Frequency	Description
Daily	Charge the batteries after each use. See chapter 5.2.
Weekly	Check tyre pressure and inflate the tyres if necessary. See chapter 5.3.
Monthly	Clean the wheelchair. See chapter 5.4. Check the batteries (in summer: once every fortnight; in winter: once a month). See chapter 5.2.

### 5.2 Batteries

The wheelchair has 'dry' gel batteries. These dry batteries are entirely closed and maintenance free.

**The use of 'wet' batteries is not permitted.**

#### Charging the batteries

Consult the owner's manual of the battery charger to check for compatibility with the batteries used.

In case of normal use, the batteries should be charged every night.

1. Switch off the wheelchair.
2. Put the plug of the charge cord in the charge connection (A).
3. Put the plug of the battery charger in the wall outlet of the electricity mains.

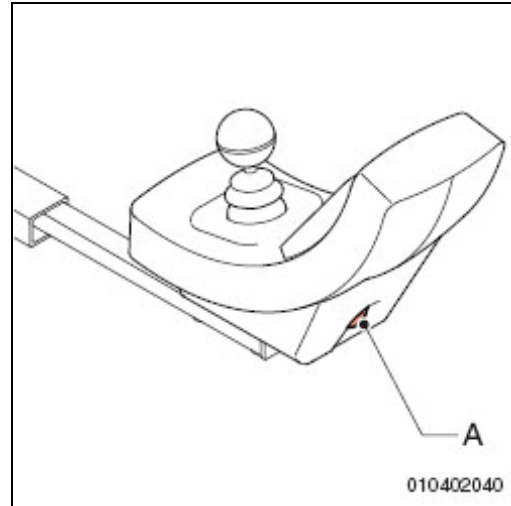
Figure 5.1

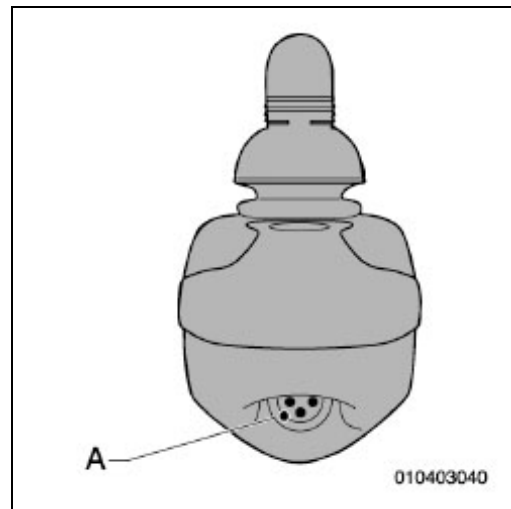
Figure 5.2

When the batteries have been charged:

1. Remove the plug from the charge connection.
2. Remove the plug of the battery charger from the wall outlet of the electricity mains.

- **As soon as the plug of the charge cord is connected to the charge connection, the operation of the wheelchair is switched off.**
- **Always remove the charge cord when the batteries have been charged. This way, you will prevent slow depletion of the batteries.**

### Maintenance of the batteries

Refer to the battery instructions.

- **Make sure that the batteries are always well charged.**
- **Do not use the wheelchair if the batteries are almost depleted and never completely deplete the batteries. This can seriously damage the batteries and you run the risk to come to an unintended standstill.**

### Cleaning the batteries

Keep the batteries clean and dry. Dirt and water can cause a leak, reducing the capacity of the batteries. After cleaning grease the poles with acid-free Vaseline.

### Replacing the batteries

If the capacity of the batteries is reduced to the extent that the wheelchair can only make very short trips, the batteries are at the end of their lifespan. The batteries must be replaced as soon as possible. Contact your dealer for this.

### 5.3 Checking and inflating the tyres

- **Make sure the tyres are kept at the correct pressure.**
- **The tyre pressure may never exceed the maximum value indicated on the tyre.**

Inflate the tyres with a foot pump or a bicycle pump, in combination with the reducing nipple included in the delivery. You can also have your tyres inflated at your local petrol station.

- **Tire pressure driving wheel:**  
Puma: 1.75 bar  
Beate: 2.7 bar
- **Tire pressure swivel castor:**
  - Outdoor type: 3.5 bar
  - Indoor type: 2.5 bar

### 5.4 Cleaning the wheelchair

#### Removing dry dirt

Upholstery, metal parts and frame parts can usually easily be cleaned with a dry soft cloth.

#### Removing mud and other wet dirt

The dirty parts can best be cleaned by first wiping them with a wet sponge and then wiping them dry with a dry soft cloth.

- **Never use abrasive or aggressive cleansers. They can scratch the wheelchair.**
- **Do not use organic solvents such a thinner, benzene or white spirit.**
- **Be careful with water in view of the electronic installation.**

## 6 Troubleshooting

### 6.1 Troubleshooting table 'DX'

Signal	Possible cause	Remedy
ON/OFF light is not on; the wheelchair does not drive.	The plug of the controller is not properly placed in the 'power module'.	Place the plug properly in the "power module" (DX BUS). The DX power module is behind the cover that is on the outside under the chair against the battery tray.
	The battery charger is still connected to the controller.	Take the plug of the charge cord from the charging connection.
	Neutral is still on.	<ul style="list-style-type: none"> <li>• Switch the wheelchair off electrically.</li> <li>• Switch off neutral.</li> <li>• Switch the wheelchair on.</li> </ul>
	The batteries are not properly connected.	Check the connections.
	A connection of the batteries is loose.	Secure the connection.
ON/OFF light flashes 7 times	The battery is too low.	Charge the batteries.
	The batteries are empty.	Charge the batteries.
ON/OFF light flashes 8 times	The battery voltage is too high, so that it exceeds 32 V.	This can only occur at charging.
ON/OFF light flashes 12 times.	System does not 'fit'. System modules are not compatible.	After checking press the ON/OFF key back to start driving.

In all other cases, contact your dealer. The contact information of your dealer can be found on the last page of this manual.

## 6.2 Troubleshooting table 'Shark'

If the wheelchair will not function, while the batteries are fully charged, check the following points before contacting your dealer.

- Switch off the wheelchair and then switch it on again. Check if the problem has been solved.
- Check if all charge plugs are properly connected and secured.
- Check if the wheelchair is in the Drive position (not in Neutral).
- Check if the position of the speed control has not changed.

In case of a malfunction the battery indicator lights will be flashing on the (Shark controller). On the Shark lights and Shark seats-lights controllers the spanner symbol will be flashing (see X

in fig. 6.1). Use the table below to find out the cause and report this information to your dealer.

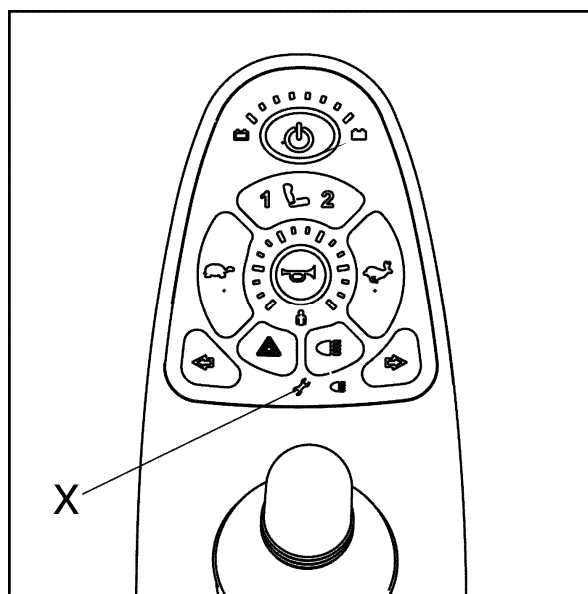
Figure 6.1

Flashing frequency indicator lights	Fault description	Action
5	Motor 1 is in the Neutral position	Put the free wheel lever of motor 1 in the Drive position
6	Motor 2 is in the Neutral position	Put the free wheel lever of motor 2 in the Drive position

In all other cases, contact your dealer. The contact information of your dealer can be found on the last page of this manual.

## 7 Technical specifications

Manufacturer:	Handicare B.V.
Address:	Vossenbeemd 104 5705 CL Helmond The Netherlands


Model:	Beatle/Puma
Maximum user weight	160 kg (25.2 st)

	Description	Min.	Max.
D	Total length including leg- rest	1050 mm	1100 mm
B	Total width Beatle	600 mm	
B	Total width Puma	650 mm	
	Total weight including batteries	122 kg (19.2 st)	
	Transport weight of the heaviest part	122 kg (19.2 st)	
	Maximum safe slope	6°	
	Static stability downward	15°	
	Static stability upward	15°	
	Static stability sideways	15°	
	Range (with max. user weight 160 kg. (352.7 Lbs))	36 km (22 miles) (56 Ah batteries)	
	Climbing cap. for obstacles Beatle	60 mm RWD / 60 mm FWD	
	Climbing cap. for obstacles Puma	80 mm RWD / 70 mm FWD 100 mm (RWD with kerbclimber)	
	Max. speed forward Beatle	6 km/h (4 mph)	
	Max. speed forward Puma	6 km/h (4mph)	10 km/h (6mph)
	Seat angle	2°	7°
	Tilt adjustment mechanical	2°	18°
C	Effective sitting depth (adjustable)	440 mm	520 mm
A	Effective sitting width (adjustable)	380 mm	525 mm
E	Sitting height Beatle	470 mm	550 mm
E	Sitting height Puma	490 mm	570 mm
	Back angle	89°	118°
	Back height	520 mm	570 mm
	Lower leg length	370 mm	520 mm
	Leg-rest angle	6°	70°
	Arm-rest height	220 mm	290 mm
	Arm-rest depth	60 mm	120 mm
	Minimal turning radius	600 mm (FWD)	960 (RWD)
	Ground clearance Beatle	60 mm	
	Ground clearance Puma	80 mm	
	Turning space	1220 mm	
G	Total height	1050 mm	

Test data	
Weight test dummy	160 kg (25.2 st)

Operating force	
Joystick	15 N
Swinging away drive box	58 N
Electronic switches	13 N
Parking brake	58 N
Placing charging plug	58 N

Batteries	
Maximum dimensions batteries	290 x 360 x 230 mm (bxdxh)
Weight wheelchair without batteries	82 kg (12.9 st)
Battery capacity Puma	40 Ah, 56 Ah or 73 Ah
Battery capacity Beatle	40 Ah
Maximal permissible charging voltage	13,6 V
Maximal charging voltage	12 A
Connector type	A DIN 72311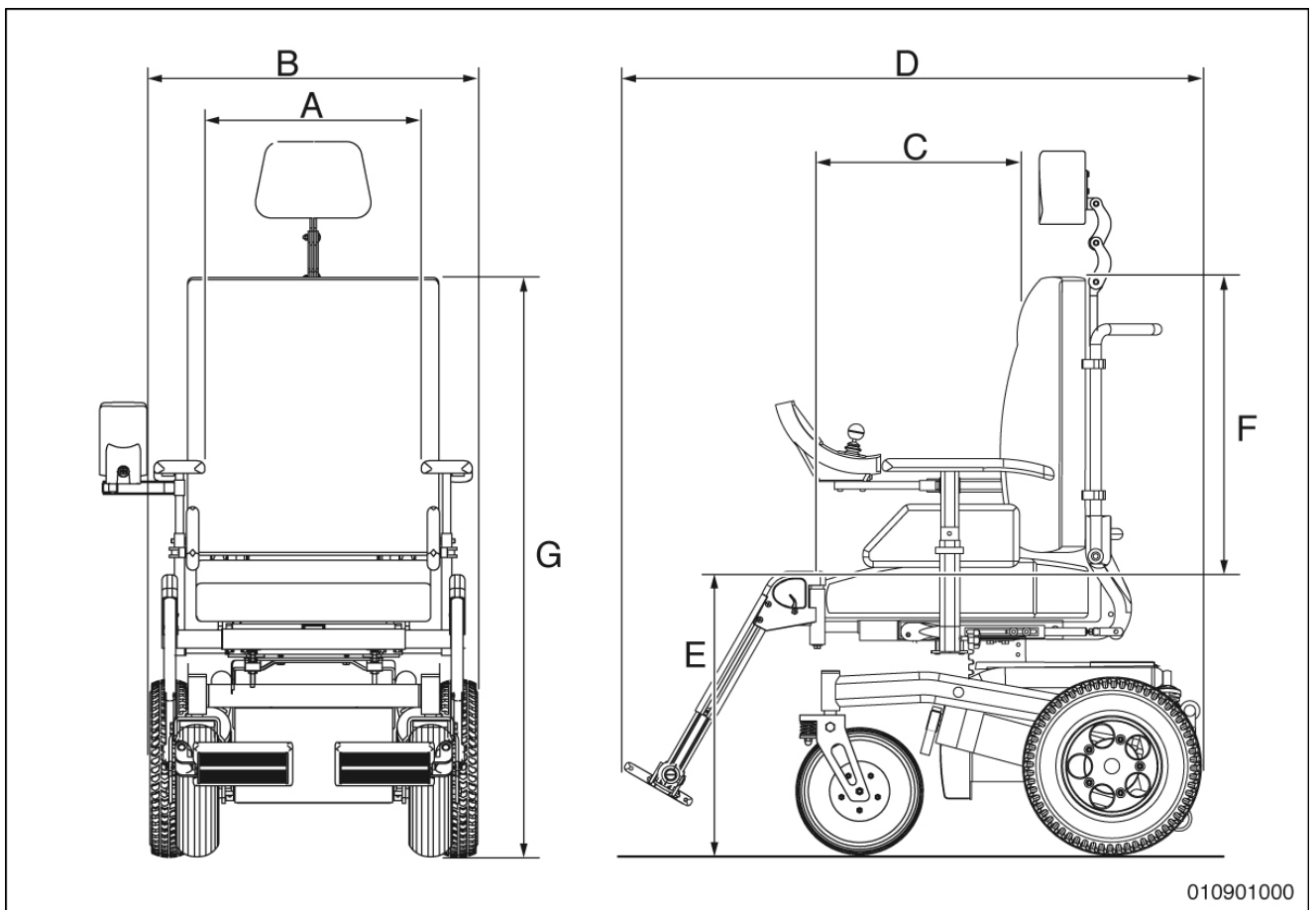

010901000

Figure 7.1

## Technical specifications

### Electrical wiring diagram

The electrical wiring diagram of the electronic installation is affixed to the interior of the battery box lid, wrapped in a plastic bag.

### Electrical wiring diagram 'DX'

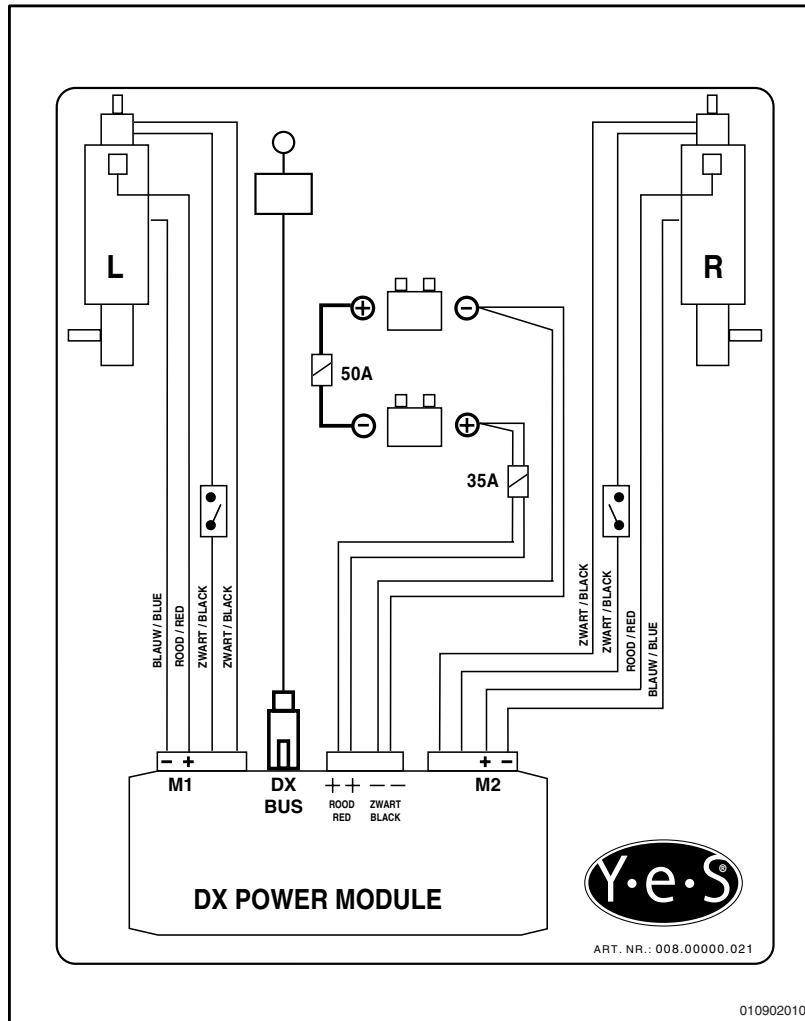
Figure 7.2

### Electrical wiring diagram 'Shark'

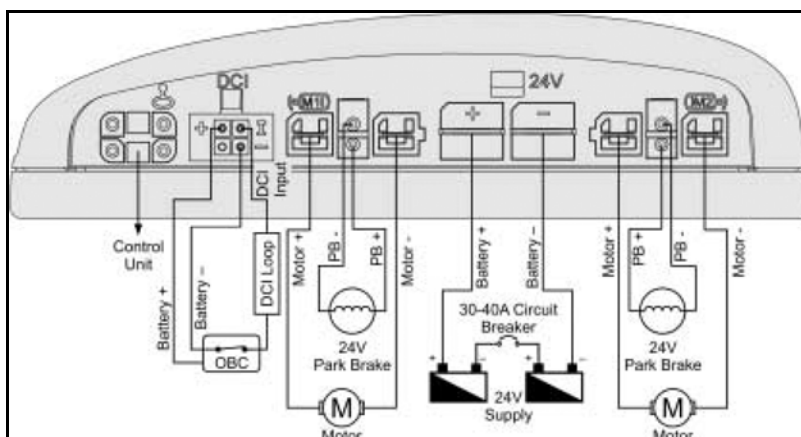
Figure 7.3

## Connecting the battery charger

The joystick's standard configuration includes a '3-pin connection' (see also figure 7.4). Ensure that the battery charger is properly connected so that the 'negative pole' (see figure 7.4 'Battery -') and the 'inhibit' (see figure 7.4 'Inhibit') are connected, enabling the system to prevent the wheelchair from moving when the battery is being charged.

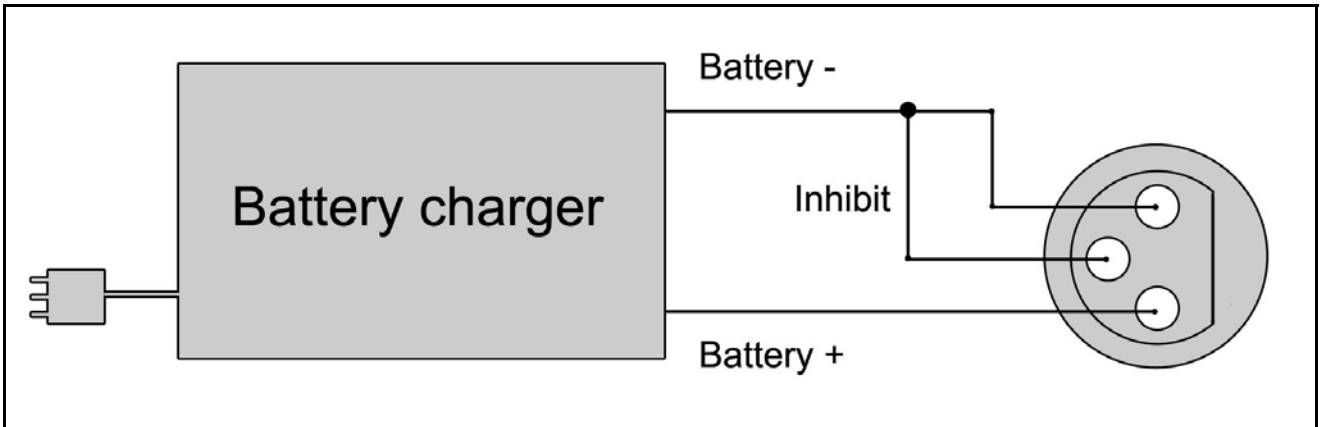
Figure 7.4

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