
AMT Liposuction Supplies & Equipment

iposuction Disposables	3
Cannulas	5
Body & Facial Cannula Sets	7
Tumescent Infusion Handles	8
Fumescent Infusion Needles & Pumps	9
Fat Transfer Cannulas	10
Aspirators	11
LS2 Aspirator System	12
K-Pump Infusion pump specs	13
Tumescent Measuring Device	14
Airbrush Liposculpture System	15

AESTHETIC MEDICAL TECHNOLOGIES

Improving the Business of Cosmetic Plastic Surgery

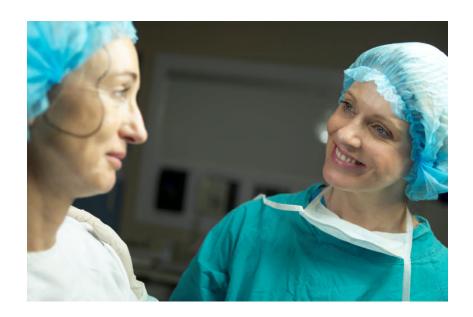







AESTHETIC MEDICAL TECHNOLOGIES.

Tel: 1-800-AMT-2937 Fax: 1-888-255-1011 orders@aestheticmt.com

LIPOSUCTION SUPPLIES AND ACCESSORIES

Aspiration Tubing

MicroAire® Compatible





Item #AMT1159 (10/box)

High-flow, aspiration tubing withstands the powerful demands of Power Assisted Lipoplasty. Proprietary "memory-free" tubing provides for maximum freedom of movement. **9' length.**

9' Standard Aspiration Tubing



Item #AMT37050 (20/box)

Lightweight, flexible Easy grip ends to insure a secure fit to your machine & handle. Tubing that is ideal for facial and other delicate areas. 9' length.

High Flow Aspiration Tubing



Item #AMT1160 (10/box) (Compare to PT5558 & B5558)

High-flow, aspiration tubing withstands the powerful demands of lipo-aspiration. Made from a special polymer mixture. Proprietary tubing reduces "memory", and has non-stick coating which which improves movement. 10' length.

12' Standard Aspiration Tubing



Item #AMT37060 (20/box)

Lightweight, flexible Easy grip ends to insure a secure fit to your machine & handle. Tubing that is ideal for facial and other delicate areas. 12' length.

Infiltration Tubing

Infiltration Tubing for Peristaltic Pump (Compare to "Klein Pump or AFTU-DS" tubing)



Item #AMT1168 (10/box)

For use with Autofuse peristaltic or Klein type pumps. Single Spike, kink resistant, memory free. Luer Lock connector designed for maximum flow. 9' length.

Single Spike Infiltration Tubing (LAM) Double Spike Infiltration Tubing (LAM-Y)



Item #AMT1163 -Single Spike (10/box)

Item #AMT1161 - Double Spike (10/box)

Single spike or Double Spike large bore, kink resistant and memory free. Luer lock connector, designed for maximum flow. Both come in 8' length.

Infiltration Tubing / BPI Pumps (Compare to CG-INF-T)



Item #AMTSoftube (10/box)

Large bore, Soft formulated tubing for use with BPI infiltrator pumps. 13ft length. Spiked with Clamp. Luer Lock Connector.



AESTHETIC MEDICAL TECHNOLOGIES.

Tel: 1-800-AMT-2937 Fax: 1-888-255-1011 orders@aestheticmt.com

LIPOSUCTION SUPPLIES AND ACCESSORIES

Canisters

1500cc Quick-Fit Suction Liner - Item #AMT1504 (10/box) With Attached Lid

2000cc Abbott Liners for use with Abbott, Byron PSI
Tech & Wells Johnson reusable hard canisters
Item# AMT2000L (10/box)

2000cc Implosion Proof Hard Canister - (MD Resource)

Item #AMTLS7020 (10/box)

3000cc Quick-Fit Suction Liner - Item #AMT3004 (10/box) With Attached Lid

1500cc Reusable Outer Canister - Item #AMT1500 01 SC with attached stopcock bracket

2000cc Abbott Reusable Outer Canister - Item# AMT2000can

3000cc Reusable Outer Canister - Item #AMT3000 01 SC with attached stopcock bracket

2000cc Abbott Liners w/ Over Flow Trap - Item# AMT2000LOT













Aspirator Filter

Item #AMTHFIFD



The H.E.P.A "High Alpha" 0.3 micron aspiration filter is a superior high flow filter.



LIPOSUCTION SUPPLIES AND ACCESSORIES

Cannulas & MicroCannulas


AMT offers the highest quality stainless steel cannulas at a competitive price point. Each cannula is precisely machined, polished and assembled to your exact specifications.

We have included some of the more common cannula choices in our catalog. However, *Please let us know if there is something that you would like that you don't see here.*

To Order a Cannula, please specify the following:

- 1. Tip Shape
- 2. Tip Style
- 3. Diameter
- 4. Length
- 5. Handle Type
- 6. Straight or Curved
- 7. Vent Hole on Handle Yes or No?
- 8. Perfect Alignment to indicate orientation of cannula. I.E. For one sided orifice opening.
- 9. Autoclave Tray or Set Please let us know if you would like to put together an entire set?

Cannula Tips

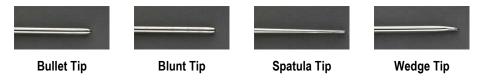

Cannula Styles



Grater-Round

Sattler-Blugerman (24 Holes, 4 rows)

Sattler-Blugerman Draining Cannula (12 Holes)



LIPOSUCTION SUPPLIES AND ACCESSORIES

Diameters		Outside Diameters	(not to scale)
1.2mm	•	18g	
1.4mm	٥	17g	Standard Lengths
1.6mm	٥	16g	7cm - 9cm - 10cm - 15cm - 23cm - 30cm - 35cm
2mm	0	14g	
2.7mm	0	12g	Curve Preference Choice
3mm	0	11g	
3.4mm	0	10g	Straight - Convex Curve - Concave - Swan Neck (Pictured here)
4mm	\bigcirc	8g	

Cannula Handle Options

5mm

We have included some of the more common cannula handle choices in our catalog. However, please let us know if there is something that you would like that you don't see here.

One Piece Aluminum Handles, Contour Handles (With half the weight and more control), Sure Fit Luer lock handles, Two Piece Quick Screw (Delrin) handles and NEW One Piece Handles with a swivel barb to allow easier turning of the handle. (A Tube Swivel Adaptor is also pictured)







NEW One-Piece Swivel Barb Handle



NEW Tube Swivel Adaptor

Thank you for your interest in our Cannula product offering. Please contact an AMT Sales representative to help with a custom order. Again, please let us know if there is something that you would like that you don't see here.



AMT Body Liposuction SET

Designed for standard body aspiration procedures

SIZES

- (1) 3-Port Radial (Mercedes)
- (1) 3-Port Radial (Mercedes) 3mm x 23cm
- (1) 3-Port Radial (Mercedes) 4mm x 30cm
- (1) Accelerator 3mm x 23cm

Choose All One Piece Handles or two piece design with a choice one of the following:

- (2) Delrin Handles
- (2) Luer Lock Handles
- (1) Infusion Handle, luer lock, with on/off control
- (1) 3.0mm x 23cm infusion needle, luer lock
- (1) 2.0mm x 17cm infusion needle, luer lock







Item No. AMT-BodyLipo Includes (1) Autoclavable Tray

AMT Face & Neck Liposuction SET

Designed for small area aspiration procedures

SIZES

- (1) Accelerator 2mm x 15cm
- (1) Accelerator 2mm x 23cm
- (1) Accelerator 3mm x 15cm
- (1) Spatula 2mm x 15cm

Choose All One Piece Handles or two piece design with a choice of one of the following:

- (2) Delrin Handles
- (2) Luer Lock Handles
- (1) Infusion Handle, luer lock, with on/off control
- (1) 1.5mm x 15cm infusion needle, luer lock
- (1) 1.5mm x 10cm infusion needle, luer lock



Item No. AMT-FaceLipo Includes (1) Autoclavable Tray



Tumescent Accessories

Liposuction Infusion Handle Options:



INFUSION HANDLE WITH ON/OFF CONTROL LUER LOCK CONNECT

The on/off valve workings have been improved by sealing systems that will better withstand the high autoclave temperatures. This improved handle allows for fingertip control during the infusion process. Lightweight Aluminum one piece design.

Order No. AMT1040-INFU-C



INFUSION HANDLE WITH STOP COCK CONTROL

The level handle shows flow direction for easy visual checks, quarter turn shot-off. The vale is made of brass and chrome-plated finish. The stop cock control can be rotated to any direction by releasing the handle cap and tight the cap back after you choose what direction you want. Two Piece design. May be used with out stopcock.

Order No. AMT1540-INFU-SCC



LIPOSUCTION SUPPLIES AND ACCESSORIES

Tumescent Accessories

Infiltration/Infusion Needles with Sure Fit Luer Lock

Available in single,multi-hole and Garden hose style versions. Infusion cannulas are strong, slightly flexible and resilient (all on Luer Lock Hubs). Standard diameters are 1.5mm, 2.0mm, and 3.0mm.

Standard lengths are 17cm, 25cm and 32cm.

Custom Configurations are available.

Standard Spiral (6 Hole) & Garden Hose Style (4 Holes at 90 Degrees) Shown Here.



Infiltration Pumps & Infiltration Syringes



K Pump Infusion pump (110 or 230 Volt), 1000 ml/min. Also works with TMD.

Item# AMTKP



Tumescent Measuring Device

The TMD is a 24volt system and plugs directly into an outlet in the kpump itself. It can also be powered separately with a separate power supply. It mounts directly onto the LS systems IV pole or at the top of other IV poles where the bag hangers can be removed.

Item# AMTTMD



Bi-Phasic Infiltrator Pump

0 to 6 ml per minute, dual pneumatic footswitch, 15 psi max, 24v power adaptor. Also works with TMD.

Item# AMTBPI







TMD with K Pump

The TMD with the K Pump is also offered as a system.

Item# AMTTMDK

LIPOSUCTION SUPPLIES AND ACCESSORIES

Fat Transfer Instrumentation

AMT Microcannulas for Fat Transfer are made from the highest quality stainless steel. The cannulas are specifically designed to gently harvest fat without damage. Similar in design to the Coleman style cannulae. All Fat Transfer microcannulas come with a Premium Sure Fit Hub. Please specify the desired diameter, tip shape, tip style, curve preference and length.

*All 16g, 17g & 18g come with a cleaning trocar. Please let us know if you would also like an autoclavable tray to store and sterilize your microcannula set.

Type I - Blunt End



Type II - Partially Closed


Micro-infiltration cannula

Custom made to order. Common requests 16g or 18g Multi-Hole infiltration. 10cm, 16cm, 22cm lengths.

Micro-aspiration (harvesting) cannula

Custom made to order. Common requests 12g or 14g aspirator. 9cm, 15cm, 21cm lengths.

Micro-injection cannula

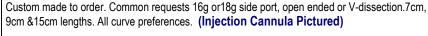

Type III - Half Open



V - Notched-Dissector



Syringe Assisted Liposuction Accessories

60cc Toomey Syringe Cannula

All AMT Toomey syringe cannula are custom made to order all have sure fit leur lock connections.

Syringe Clicker Locks

Allows syringe to be locked in various increments and maintain vacuum. Item # AMTCLI

Male & Female Transfers 60cc (Pictured) Item AMT60T

60cc Power Adapter Handle (Pictured) Item # AMTPAH

Punch Awls (Available not pictured) Item # AMTPUN

Disposable **Infiltration Syringes**

10cc Repetitive injection syringes QTY. 10/box Item # AMTRIS



Micro - Injection Guns

All stainless steel construction and ergonomically correct. Designed to fit the BD syringes, the industry standard. The guns are available in the following sizes: 1, 3, 5, 10, 30 and 60 cc. Item # AMTMG









Syringe Racks

Made of a medical grade plastic that will withstand the high sterilization temperatures. No more wire racks that rust after only a few uses. Light weight and stackable if need be. Available to fit syringes in sizes: 5, 10, 20, 30 and 60 cc. The ideal device to aid in transfers and re-injection. Item # AMTSR





AESTHETIC MEDICAL TECHNOLOGIES.

Tel: 1-800-AMT-2937 Fax: 1-888-255-1011 orders@aestheticmt.com

LIPOSUCTION SUPPLIES AND ACCESSORIES

Aspirators



LS2 Dual Port

Designed for practical use. The LS2 has many available options to customize it to your needs. Available in single & dual port models. We build each system based on customer needs. Comes with a power outlet for additional components such as the PAL console shown, and k pump. Call for more information.

Item # AMTLS2D



LS2 Single Port

Same as above. Note units come with IV pole which holds both the dual canister holder as well as the IV bags themselves. The new units can also include a TMD (see) on that same pole. Rotary vane pumps, pneumatic foot pedal, static proof neoprene wheels.

Item # AMTLS2S



LS/PAL System

Pictured (left) is the assembly starting (from bottom) with the LS2-DP, the PAL console & console cover, the K Pump infiltrator (resting above it), the standard dual canister holders and at the top is the new Tumescent Measuring Device (TMD). The footprint for this entire assembly is 15 square inches.

The TMD with the K
Pump (shown right) is
also offered as a system

Item # AMTTMDK



RM2000

Whisper quiet, strong and reliable power. Two self-healing 1/3 hp rotary vane pumps like the LS2. Adjustable from 0 to 29" hg. Dual footswitch control. Features one size-adjustable canister holder. Quietest strong aspirator available.

Item # AMTRM2000



Touch Up Aspirator

Transportable power aspirator with fullsized features for secondary lipoplasty procedures. 0 to 27in Hg.

Item # AMTTU



RM2000 TITAN

Quiet power aspirator. Low-profile design. Two self-healing 1/3 hp rotary vane pumps. Adjustable from 0 to 29" hg. Dual footswitch control. Features two size-adjustable canister holders.

Item # AMTRM2000T



NX

Single 1/3 hp piston pump system similar to KMI Phantom I and HK systems. Adjustable from 0 to 29" hg. Dual footswitch control. Features one canister holder.

Item # AMTNX

ASPIRATION SYSTEM

The LS2 continues our line of durable reliable liposuction systems.

The LS2 is virtually maintenance free because it uses oil-less rotary vane pumps, and is amazingly quiet. The tower design provides convenience and simplicity. As shown from the bottom: the aspirator, Power Assisted Lipoplasty console by MicroAire®, the PAL console cover, infiltration pump, dual canister mount, and on top, the TMD (Tumescent Measuring Device) to measure tumescent fluid output.

A built in hospital grade power outlet in back means only one power cord from this unit to wall power, so less power cords across the Operating Room floor. And the entire system takes up only 15in. X 15in. of Operating Room footprint.

The extension pole is extendable up and down, with a grip at the base to easily move the system about. Once in place, the heavy duty static free casters can be locked. The systems bright digital display is easily visible even in a dimly lit Operating Room.

Includes either single or optional dual pneumatic footswitch; single or dual port/dual valve.



- 0 to 29+ vacuum in 6 seconds
- Digital and graphical LED displays

Specifications

Display...... 20-segment, 3-color LED bar graph; two-digit LED numeric vacuum readout

staging

Vacuum...... Adjustable from 0 to 29 Hg at sea level

Sound Level <50 dBa

Grounding...... Double insulated

 Chassis
 Steel frame with vibration-free parallel motor mounts

 Enclosure
 Powder coated sheet aluminum with sound insulation

Thermal Protector..... Yes Warranty...... 6 years

Ordering Options

LS2..... Power aspirator with single output port. Includes power cord, user/service manual, pneumatic footswitch, extension pole with canister mount, biofilter and canisters.

LS2-DP LS2 Dual Port model adds one additional output port and valve.



AESTHETIC MEDICAL TECHNOLOGIES.

the K-Pump

PERISTALTIC PUMP

Ideal use for tumescent infiltration for liposuction or vein ablation. K-Pump provides infusion/irrigation rates from 20 to 1,020 ml/min.

The resilient Kydex chassis is built to last and allows for quiet, vibration-free operation.

A rotary speed control allows the circulating nurse to precisely control the flow. A pneumatic foot-switch turns the pump on/off. Dual foot pedal connectors provide the capability to place a foot-switch on either side of the OR table. The lever shown shuts off the rollers when opened as a safety feature. It has connections to connect



- Easy to load, simple to operate
- Uses disposable tubing

Specifications

Panel Indicators......Power-on green LED

Speed ControlRotary control knob

Drive Speed......10-600 rpm

Additional Protection...... Over-speed protector; automatic safety cut-off switch

Operating Temperature......0° to 40° C

Weight.......Approximately 5.0 lbs (without accessories)

Ordering Options

KPUMP...... K-Pump with "hospital grade" power cord, user manual, & pneumatic footswitch.

AFTD...... Disposable infiltration tubing.

AFTDY...... Disposable infiltration tubing with Y connector.



Please call 800-268-2937 or email orders@AestheticMT.com for more information.

the TMD

TUMESCENT MEASURING DEVICE

An innovative design, for tumescent lipoplasty procedures or to precisely fill Saline breast implants.

Its convenient pole top mount delivers effortless measurement of total tumescent delivery. Weight lost from the hanging tumescent fluid bag is digitally displayed with +/- 1 ml accuracy.

The very low current circuitry enables the TMD to be "piggy backed" to our **K Pump** or it can be used as a "stand-alone" device.

SPECIFICATIONS

Display	4-digit LCD with blue backlight. 60 degree viewing window visible with 1 inch digits.
Start Switch	Once for 'on', once for 'off'. Starts or restarts at 0 (zero).
Power	Supplied AC adapter provides 24 VDC @ 1 A. Operates with DC power supply of 8 to 40 volts @ 100 mA.
Hook	Stainless steel hook accommodates up to 3 saline bags.
Load Limit	Up to 4,000 ml (4 kg) maximum advised load.
Mounting	Mounts on adjustable height pole with 7/8(.875)" O.I







AESTHETIC MEDICAL TECHNOLOGIES.

Please call 800-268-2937 or email orders@AestheticMT.com for more information.



NEW: Airbrush Liposculpture® System

A patented Twin-Cannula design which allows fast, effortless, less traumatic, and more precise fat removal.



Twin- Cannula Design Unleashes the Artist in the Surgeon

The Twin Cannula design is the defining feature which <u>eliminates the physical labor</u> of the procedure and allows the surgeon to <u>concentrate on smooth</u>, even fat removal and feathering



Not Another Power Assisted Liposuction Device

Airbrush Liposculptor® is a much more elegant procedure which allows the physician to concentrate on their artistic vision.

- <u>The Inner Cannula</u> captures fat as the device is moved over the target site.
- <u>The Capture Zone</u> is shielded so that the physician may turn the opening towards the skin.
- <u>May Significantly Reduce Touch Ups</u> because the device itself is precisely removing fat with each pass.
- <u>Eliminates the Battering Ram effect</u>-that an unprotected reciprocating cannula tends to have on a patient.
- <u>Twin Cannula design</u> may lessen tissue trauma and reduce bruising, swelling, and shorten convalescence.







MT AESTHETIC MEDICAL TECHNOLOGIES

We are currently scheduling Airbrush Liposculptor® evaluation requests from qualified Physicians. Call (800) AMT– 2937 to reserve a demonstration or an evaluation.

Phone 1-800-AMT-2937 • Fax 888-255-1011 • Email: orders@aestheticmt.com

LIPOSUCTION SUPPLIES AND ACCESSORIES

AMT, Inc. - Policy on Returned Goods

- 1. All AMT orders are carefully inspected before shipment. Any errors in the shipment must be reported to customer service within five (5) business days of receipt. Any shipping damage should be reported to the carrier and AMT customer service at the time the package is received.
- 2. NO RETURNS OR EXCHANGES WILL BE ACCEPTED UNLESS YOU HAVE RECEIVED A RETURN OF GOODS AUTHORIZATION (RGA) NUMBER FROM AMT. This number must be on the return shipment box and all written communication.
- 3. Credit will be only issued if the merchandise is in saleable condition. If the product(s) has been used in surgery, the product must be CLEANSED and STERILIZED and a written statement confirming such must accompany the product(s).
- 4. Full credit will be issued for any item returned in 30 days that is undamaged and saleable condition.
- 5. A restocking charge of 25% will be charged to any product returned between 31-90 days. NO CASH REFUNDS ARE PERMITTED AFTER 30 DAYS OF THE INVOICE DATE. A credit memo, based on dollar amount, will be issued to the account.
- 6. OPENED PACKAGES OR BOXES containing implants or sterile products are not returnable for credit or exchange. Each package or sales unit specifically states "Not Returnable if Package Seal is Broken."
- 7. Custom products are NOT RETURNABLE. Since special tooling and time are dedicated to these products which are manufactured to precise specifications of the surgeon, they cannot be sold to another customer.
- 8. NO CREDIT WILL BE ISSUED ON ANY ITEM RETURNED AFTER 90 DAYS.
- 9. Compliance with FDA regulations on sterile and implantable products accepted for return require more extensive controls and are subject to refusal and/or higher handling charges.

PRICES SUBJECT TO CHANGE WITHOUT NOTICE