CHROMOPHARE® C 570
CHROMOPHARE® C 571

Service-Manual (D, E)

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**Table of contents
for Service-Manual
C 570/C 571**

Contens	Page
Introduction	4
Technical Data	7 – 8
Functional Description valid up to Serial no. 152.000	9 – 11
Connecting Diagram valid up to. Serial no. 152.000	12
Functional Description of light intensity regulation board valid from serial no. 152.000	13
Wiring Diagram valid from serial no. 152.000	14
Spare Part List	15 – 20
Graphic spare part description	21 – 37
Valid up to serial no. 152.000	21
Valid from serial no. 152.000	22
Valid for all types	23 – 29
LFN	29 – 30
EndoLite®	31
CFWP	32 – 34
Mobile (SO)	35
Wall mounted (W)	36 – 37
Testpoints – halogen bulb supply voltage	37
Compare Table for max. halogen bulb supply voltage valid from Serial no. 152.000	38
Measuring Points – trouble shooting	39 - 43
Adjustment of the weight compensation on the Acrobat Arm	44 - 45
Adjustment of the height movement on the Acrobat Arm	45
Electrical Diagram of C 570/C 571 Transformers and emergency relays mounted on the ceiling tube	46
Electrical Diagram for C 570/C 571 Transformers and emergency relais mounted in a wall box on side of building	47
Fuses	48

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The BERCHTOLD CHROMOPHARE® is the latest in the new generation of surgical lights, and features many new technological advances.

Many advantages are offered by utilizing the latest electronic technics, a computer designed polygon-reflector, and cardanic suspension of the light head. They include:

- Freedom from shadows in the operating field
- High luminous power which generates the high illumination and high colour temperature similar to daylight
- Easier hygiene due to smooth styling of the light body - the absence of control knobs on the outer surface, and airtight sealing of the light head
- Larger range of height adjustment; greater mobility of the light
- Easy replacement of defective halogen bulbs by OP staff
- The construction of mechanics and control functions will be described in detail on the following pages.
- Diagrams and electrical specs accompany the descriptions. An installation and mounting instruction manuals are included to aid in the mounting and electrical installation.
- Modern braking system; the suspension arm can be moved inward in the direction of the operating site, from the outmost dead center position, and overcomes the dead point

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Technical Data BERCHTOLD CHROMOPHARE® Surgical Lights C 570/C 571

Light technical data	C 570		C 571	
Reflector system	Polygonreflector			
Colour temperature	4500 K	3600 K	4500 K	3600 K
Intensity Ec at 1 m [klx]	75	90	100	120
Intensity control [klx]	45 - 75	55 - 90	45 - 100	55 - 120
Total radiant power at max intensity	280 W/m ²	335 W/m ²	370 W/m ²	447 W/m ²
Total radiant power/intensity Ec [mW/m ² lux]	3,7			
Colour rendering index R _a	94			
Light field diameter	20 - 28 cm		17 - 28 cm	
d ₁₀ Light field dia. at 10% of max intensity	20 cm		17 cm	
d ₅₀ Light field dia. at 50% of max intensity	10 cm		8,5 cm	
Remaining illuminance when beam is obstructed by one mask	26 %		24 %	
Remaining illuminance when beam is obstructed by two mask	45 %		45 %	
Remaining illuminance inside at the bottom of a standardised tube.	100 %		100 %	
Remaining illuminance inside at the bottom of a standardised tube. Beam obstructed by one mask	26 %		24 %	
Remaining illuminance inside at the bottom of a standardised tube. Beam obstructed by two mask	45 %		45 %	
Depth of illumination L1 + L2	130 cm		130 cm	
Electrical data				
Primary voltage of transformer	110 / 120 / 130 V A.C. 220 / 230 / 240 V A.C.			
Power consumption	250 VA or 190 VA at 28 V D.C.			
Rated capacity of transformer / frequency	250 VA / 50/60 Hz			
Safety Category	1			
Protection provided by enclosure	IP 53			
General data				
Halogen bulb Order no.	24 V / 150 W CZ 908-24			
Life time of halogen bulb	1000 h			
Automatic switch over to reserve bulb	yes			

Dimensions	C 570	C 571
Diameter of light body	57 cm	
Diameter of Polygon Reflectors	50 cm	
Light emission surface (glass surface)	2092 cm ²	
Max. swivel radius	226 cm	
Lowest position of light body	128 cm	
Highest position of light body	224 cm	
Clearance (single light)	200 cm at 265 cm finished ceiling	
Weight / Torque		
Weight with. transformer	61 kg	
Max. torque	278 Nm	
Certificate	<p>CE conform with 93/42/ EEC</p> <p>UL listed</p>	

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**Functional Description of the
BERCHTOLD CHROMOPHARE®
C 570 / C 571 Operating Light valid
up to serial no. 152.000**

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Transformers and Emergency Power Switch Over

The current is sent from a mains switch and the fuses F 12 and F 13, to the T 1 transformer. A secondary cable runs the current from the transformer to the K 9 relay. This circuit is secured by the F 11 fuse. Relay K 1 is activated and in turn switches on the contacts of the transformer's secondary circuit, which switches on the light. Should the power supply fail, relay K 9 switches into the off-position connecting the light to the emergency power supply. This circuit is secured by the fuses F 14 and F 15.

ON - OFF Switching of the light

Circuit Board 1

The A 7 voltage supplied from the contacts X 1 and X 2 is rectified by the bridge connected rectifier V 1 and flattened by capacitor C 1. This voltage operates the bistable relay K 1. If the push button switch S 1 (on key) allows voltage to reach coil I of the relay K 1, relay contact k 1 closes, and current flows through the halogen bulb H 1, the light is switched on. The residual magnetism in relay K 1 holds relay contact k 1 closed. If push button switch S 2 (off key) allows voltage to flow to coil II of relay K 1, the residual magnetism disappears and relay contact k 1 returns into its rest position, the circuit to the halogen bulb is cut, and the light is switched off. The electrical current for relay K 1 is protected by the F 1 fuse.

Regulating the Light Intensity

Circuit Board 1

If the light is switched on, the A.C. working voltage is rectified by bridge connected rectifier V 4 and flattened by capacitor C 2 and adjusted to 12 Volt by voltage control IC 2 and stabilized.

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The capacitors C 7 and C 8 avoid disturbing effects to read voltage control IC 2. The circuit is switched over the the M 1 driving motor by the control switch S 3 (lower light intensity), or by control switch S 4 (higher light intensity). Then, S 3 and S 4 switch on different polarities on the M 1 driving motor, and determine its turning direction. Motor M 1 in turn drives the potentiometer R 8 to regulate the current of the halogen bulb. This is how the light intensity is regulated, by activating the M 1 motor. Electrical current to M 1 motor is protected by the F 2 fuse.

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Switching on the H2 Reserve Bulb

Circuit Boards 1 and 10
The working voltage rectified by bridge connected rectifier V 4 and capacitor C 2 is regulated to 24 Volts by voltage control IC 1 and stabilized. The capacitors C 3 and C 4 avoid disturbing effects to read voltage control IC 1. When the light is turned on, transistor V 8, regulated by capacitor C 6 and resistor R 6, turns resistor R 4 to minus. In this way, transistor V 9 cuts the voltage supply of relay K 2. Relay contact K 2 stays open, reserve lamp H 2 receives no voltage. Capacitor C 5 is charged by resistor R 3. As long as halogen bulb H 1 lights, the phototransistor V 30 on circuit board 10 is kept at a low impedance, and prevents the voltage increase on capacitor C 5. Transistor V 7 remains insulating and transistor V 8 remains conductive, transistor V 9 remains insulating. If halogen bulb H 1 becomes dark because of a defect, phototransistor V 30 on board 10 switches to a high ohm output, and voltage increases on capacitor C 5. By this transistor V 7 switches the base of transistor V 8 to minus. V 8 insulates, transistor V 9 obtains a single wire resistor R 5 and switches to minus. The relay K 2 is activated, the relay contact K2 closes, and current for stand-by lamp H 2 can flow.

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Parallel to relay K 2, the illuminated read out V 12/V 13 (on board 2) is switched to minus via resistor R 7 and so lights up to signal „bulb defective“. This warning lights as long as the defective halogen bulb H 1 is exchanged for a new one.

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
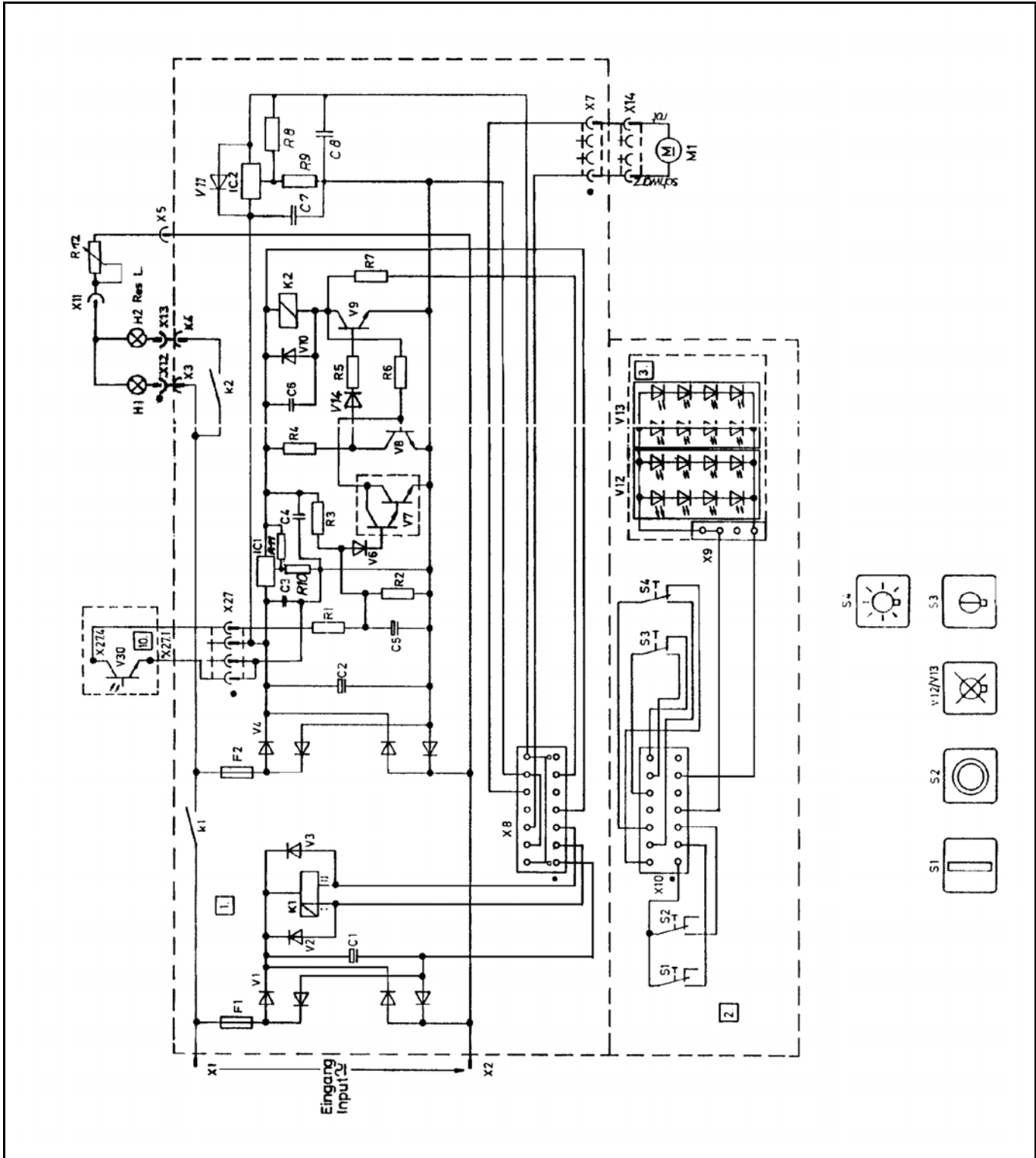
**Functional Description Light intensity control board
C 570 / C 571 valid from serial no. 152.000**

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The input voltage of 25,5 V AC – or 27,5 V DC (measured at the screw joint on top of the ceiling tube with max. load) is connected via the orange and grey cables to the connector X 2 at pin 5 (orange) and pin 6 (grey) to the OR-light head. The relay K 1 (bistable) is supplied from the rectifier V 5 and the capacitor C 8 if switch S 1 (on) will be closed. If the contact of S 1 will be closed the relay K1 will come and will stay in hold position. If the contact of S 2 will be closed the relay will cut off. The contact of relay K 1 will switch the input voltage to the power rectifier V1. The rectifier V 1 rectifies the input voltage for the electronic.

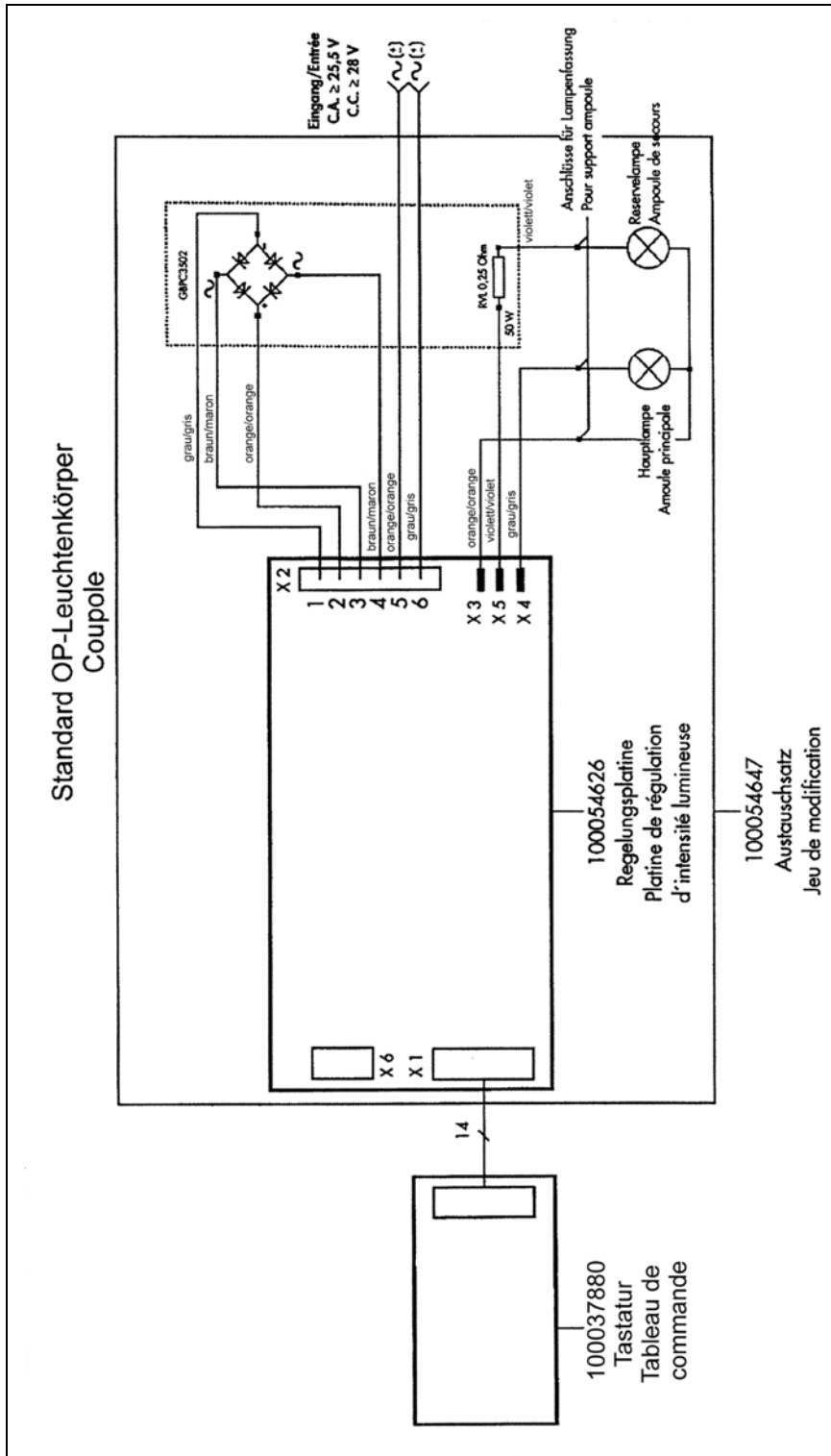Stromlaufplan der C 570/C 571 gültig ab Serien Nr. 152.000
 Wiring Diagram of C 570 / C 571 valid from serial no. 152.000

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Beschreibung description	Teile Nr. part no.	Abbildung auf Seite description on page
<i>Gültig bis Serien Nr. 152.000</i> <i>Valid up serial no. 152.000</i>		
Steuerplatine control board	42775	21
Lichtreglergruppe, komplett light intensity regulator group, cpl.	49695	21
Photo-Transistor photo transistor	37882	21
<i>Gültig ab Serien Nr. 152.000</i> <i>Valid from serial no. 152.000</i>		
Lichtregelungsplatine light intensity regulation board	54626	22
Widerstand auf Winkel resistor on fixation	54614	22
Gleichrichter Rectifier	47971	22
<i>Gültig für alle Ausführungen</i> <i>Valid for all types</i>		
Tastaturplatine key board	37880	23
Mikroschalter micro switch	37801	23
Tastaturfolie key board foil	37837	23
Fassungshalter socket holder	59880	23
Kabelverbinder cable connector	39903	23
Bremsschraube friction screw	38821	23
Bremsschraube friction screw	38827	23
Bremsschraube friction screw	64103	23

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Lichtreglergruppe komplett, Teile Nr. 49695

Light intensity regulator group, compl. Part no. 49695

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Photo transistor for switching over to reserve lamp with wiring
 harness, part no. 37882

Bildliche Ersatzteildarstellung – gültig ab Serien Nr. 152.000
Graphic spare part description – valid from to serial no. 152.000

Lichtregelungsplatine, Teile Nr. 54626
 light intensity regulation board, board no. 54626

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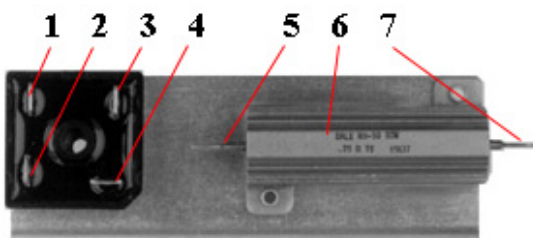
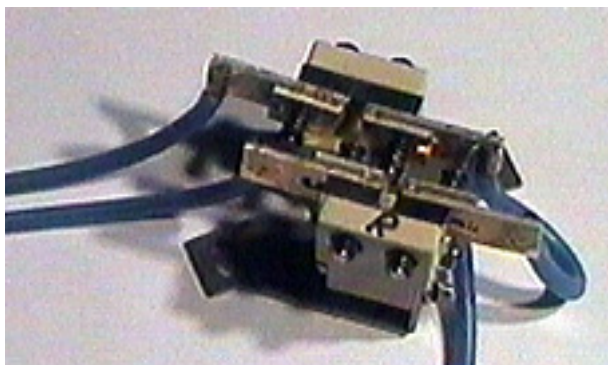
Keyboard foil, self adhesive, for standard OR lights C-570 and C-571, part no. 37837

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Socket holder with mounted main- and reserve bulb holder, with cable connections and flat plugs, part no. 59880

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Cable connector, part no. 39903 to filterglas / focussystem

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Friction screw, 34 mm length, part no. 38821
 Friction screw, 26 mm length, part no. 38827

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Contact block for sliding contacts at central column to horizontal
suspension arm, part no. 39693

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Cover for joint at spring balanced arm, part no. 20012

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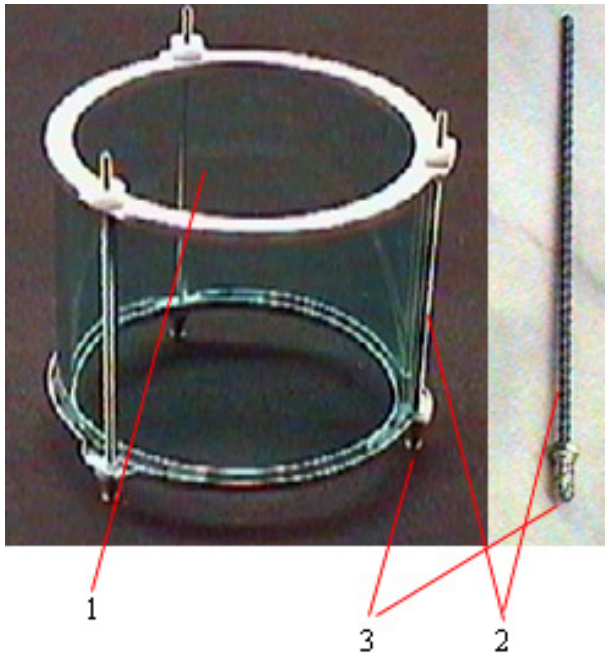
Male sliding contact at horizontal suspension arm to vertical tube,
part no. 20001

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Female sliding contact at horizontal suspension arm to vertical tube,
part no. 20002


1. Filterglasgruppe, 4500 Kelvin ohne ThermoSorb, Teile Nr. 38734
 1. Filterglass assembly, 4500 Kelvin, without ThermoSorb, part no. 38734

Filterglasgruppe, 3600 Kelvin, Teile Nr. 41984
 Filterglass assembly, 3600 Kelvin, part no. 41984

Filterglasgruppe, ThermoSorb, Teile Nr. 41984
 Filterglass assembly, ThermoSorb, part no. 41984

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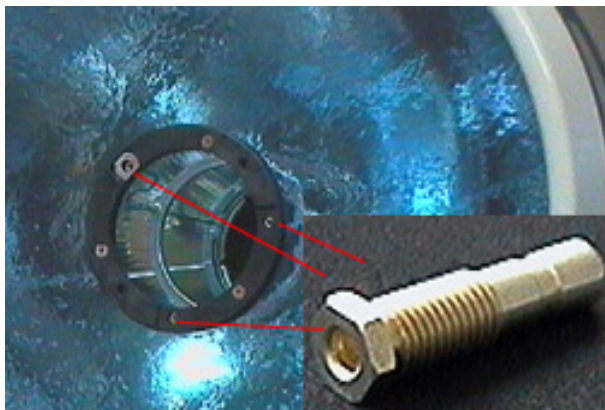Connector socket for banana plug, part no. 38580

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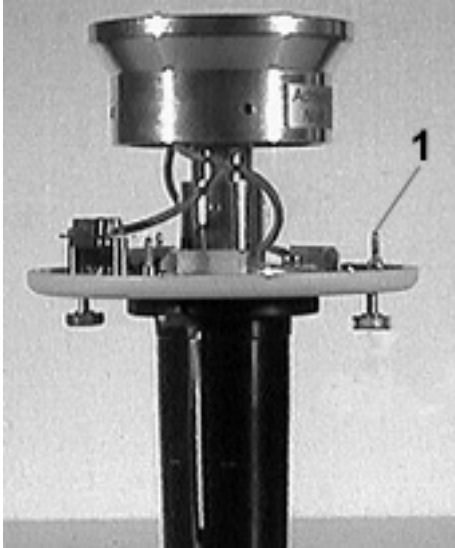
Testadapter for check of the halogen bulb supply voltage and check of the switch over function to reserve bulb, part no.50574

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Fixation for sterilizable handle and camera for C 570, focus not changable, only for cameras of the productline ChromoVision with fixed focus lens, without halogen bulbs, part no. 54466

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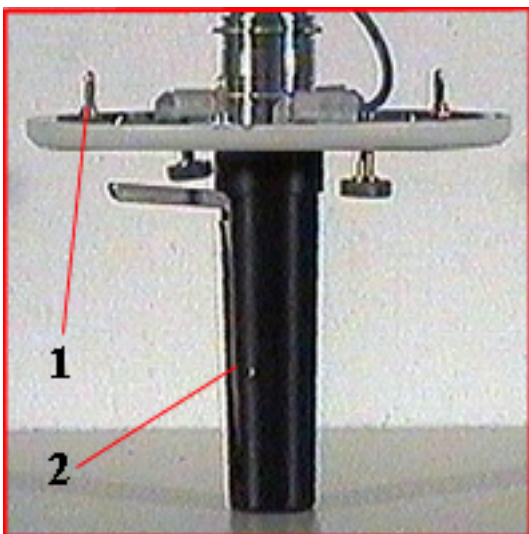
Focus mechanism for C 570 standart, without halogen bulbs, part no. 54403

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Halogen bulb reflector with bulb holder, part no. 54930

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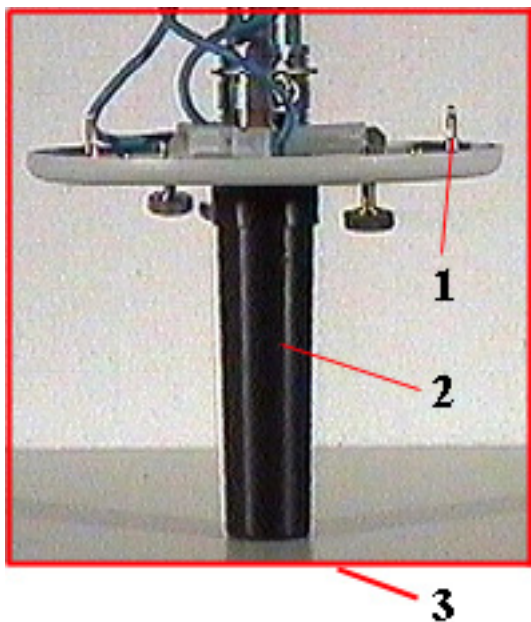
Focus mechanism for C 571 standard, without halogen bulbs, part no. 54538

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Halogen bulb reflector with bulb holders and cable connectors, part no. 54930

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2. Fixation mechanism for sterilizable handle with lock system, part no. 40769

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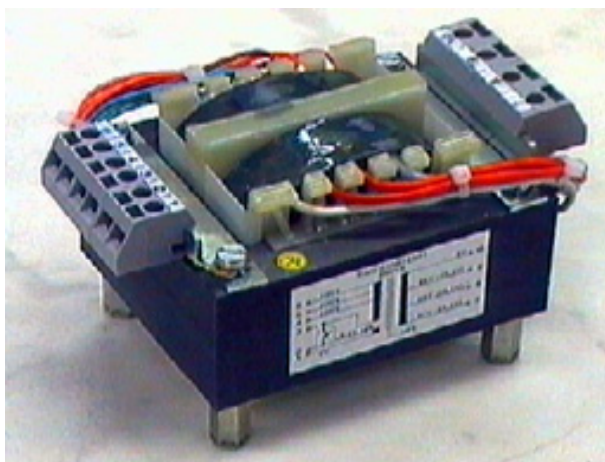
2. counter nut for guiding screw at focus system, part no. 38596

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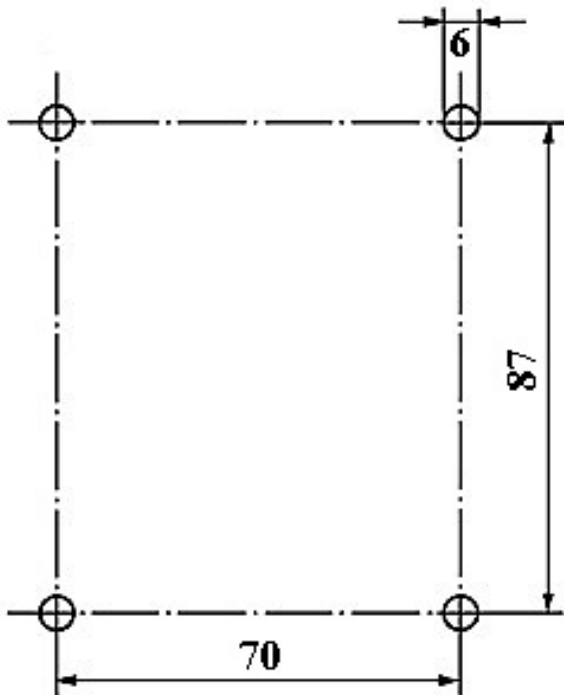
Security lock - spring balanced arm, part no. 20010

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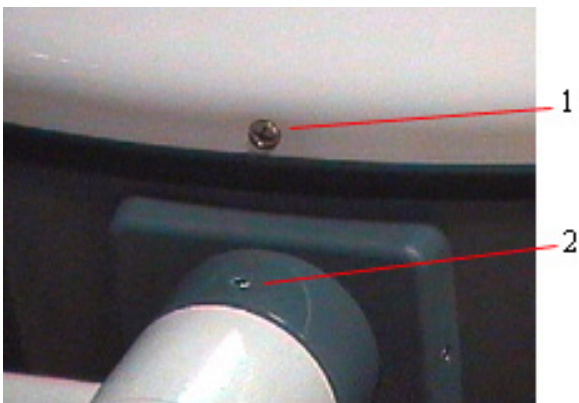
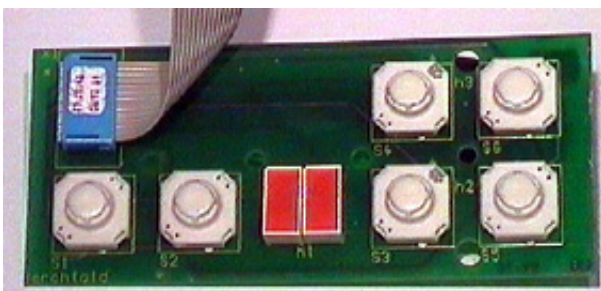
Fixation ring for ceiling cover for Ø 125 mm ceiling tube, part no. 50177

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
keyboard foil, self adhesive, for C-571 LFN, part no. 54814

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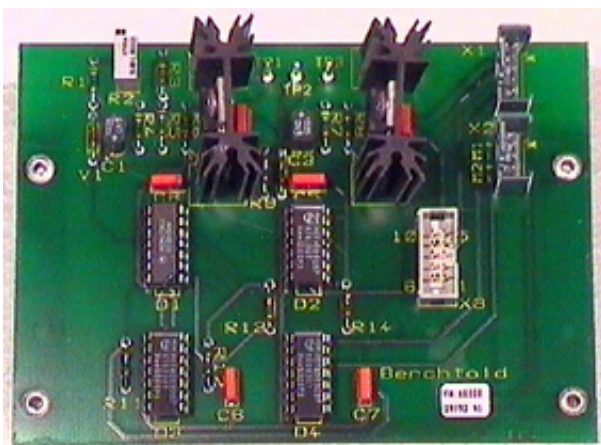
keyboard for C-571 LFN, with flat cable and connector, part no. 54827

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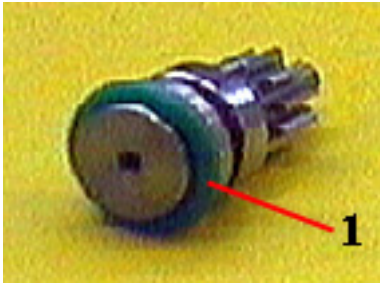motor- and focus control board for C-571 LFN, part no. 54882

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
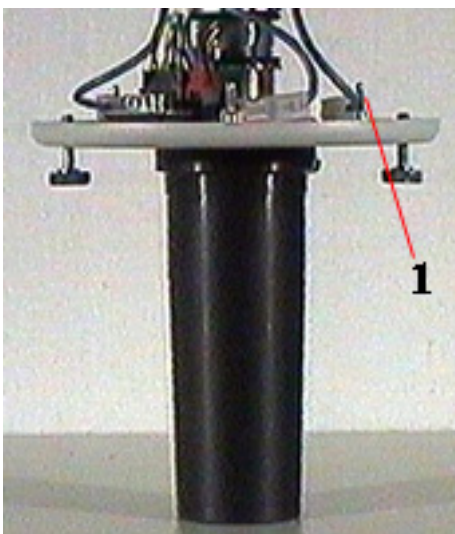
halogen bulb reflector with bulb holders and cable connectors,
part no. 54930

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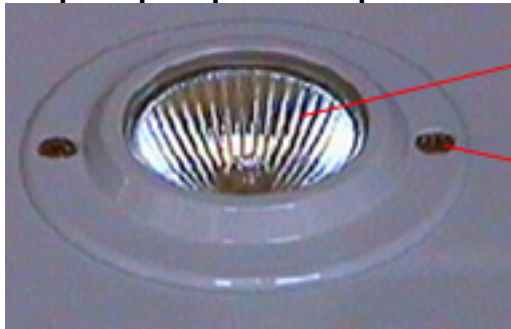
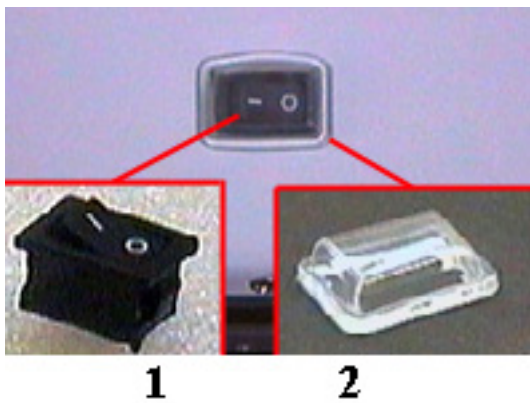
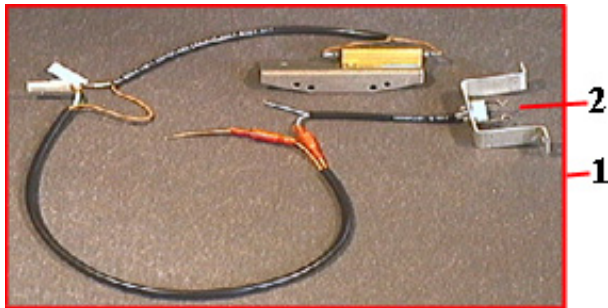
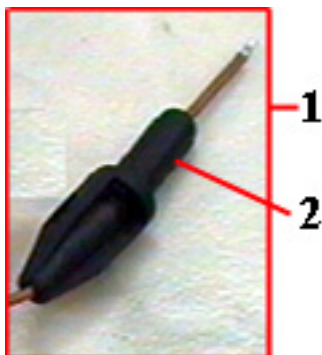
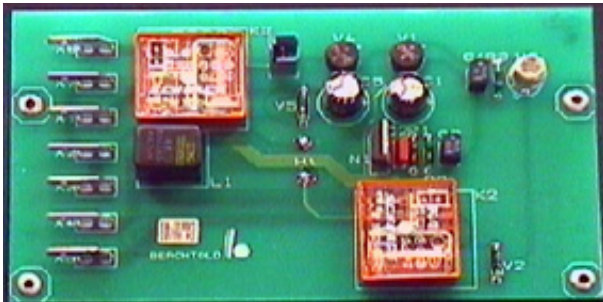
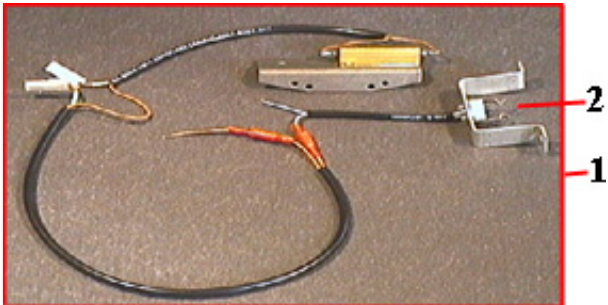
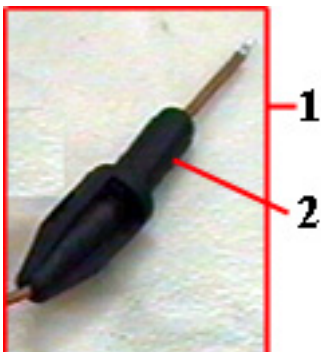
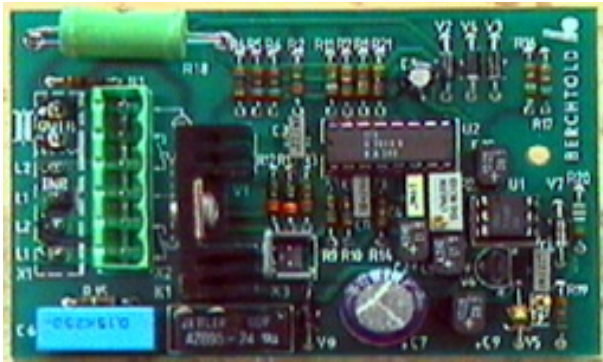
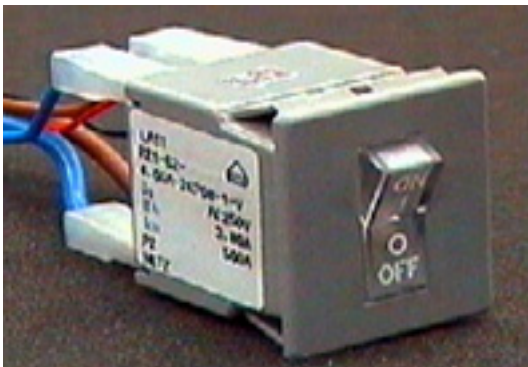
dimming p.c. board for CFWP - wall box,
220/230 Volt, part no. 63671
110/120 Volt, part no. 54535

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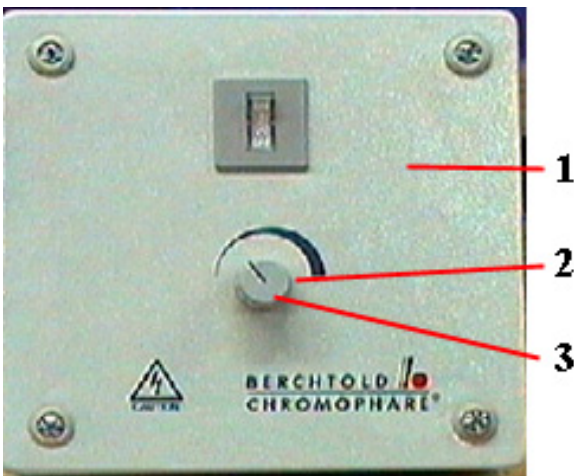automatic fuse and circuit braker, illuminated, only for 110/120
Volt, part no. 54493


1. CFWP - Kasten für 1-fache OP-Leuchte, 110/120 Volt Netz-
spannung, ohne Notstromschialtung, Teile Nr. 54526
1. CFWP - box for single OR.light, without emergency backup
system, for 110/120 Volt mains supply only, part no. 54526

wie oben - für 2-fache OP-Leuchten, Teile Nr. 54527
as above - for double OR-light, part no. 54527

wie oben - für 3-fache OP-Leuchten, Teile Nr. 54528
as above - for tripple OR-light, part no. 54528

2. Verdreherschutz, Teile Nr. 54604
2. wiring protector, part no. 54604

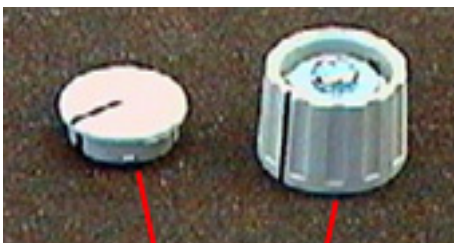3. Drehreglerknopf, Teile Nr. 54502
3. regulator button, part no. 54502


1. Drehknopfdeckel, Teile Nr. 38647

1. cover of regulation button, part no. 38647

2. Drehreglerknopf, Teile Nr. 54502
2. regulator button, part no. 54502


1

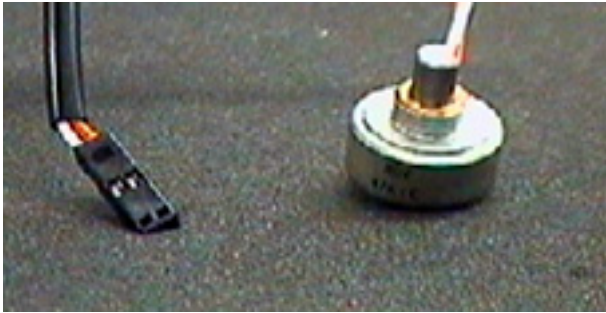2

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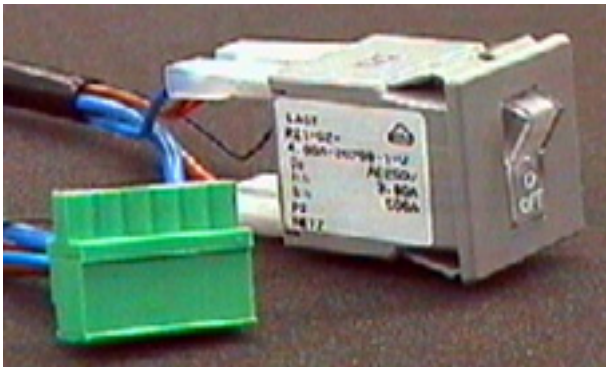
potentiometer with wiring harness, part no. 54530

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wiring harness with circuit breaker and connector, part no. 54533

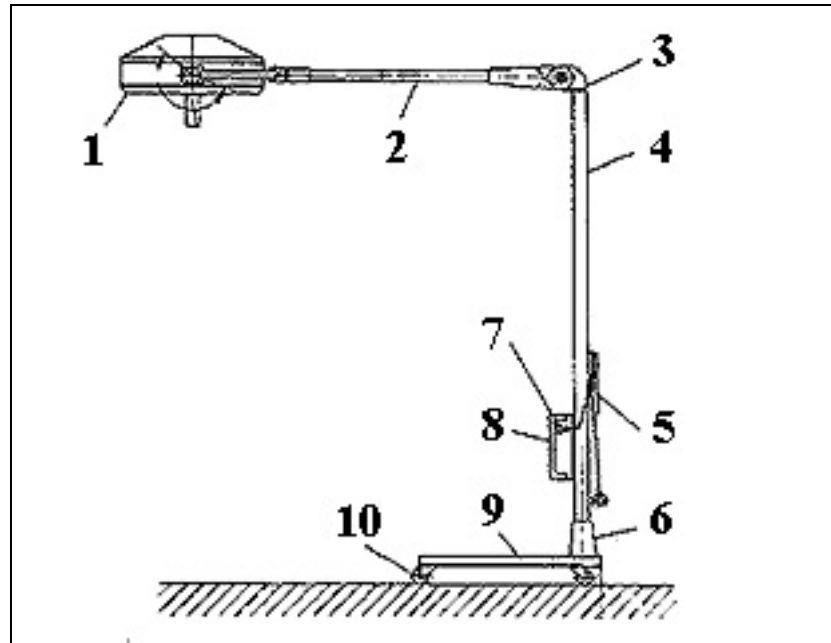OP- und Untersuchungsleuchte CHROMOPHARE® C 570 / C 571 SO
Surgical and Examination Lights CHROMOPHARE® C 570 / C 571 SO

1. Leuchtenkörper C-570 SO, Teile Nr. 38749
 1. Leuchtenkörper C-571 SO, Teile Nr. 54630
 1. light head C-570 SO, part no. 38749
 1. light head C-571 SO, part no. 54630

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
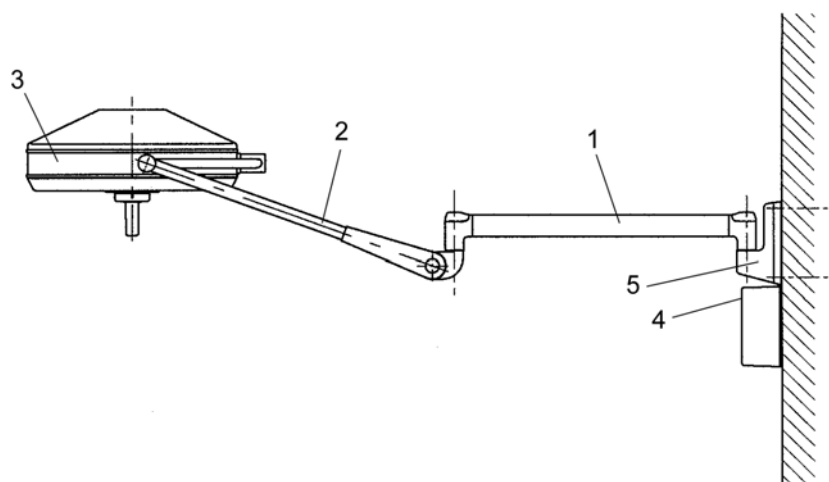
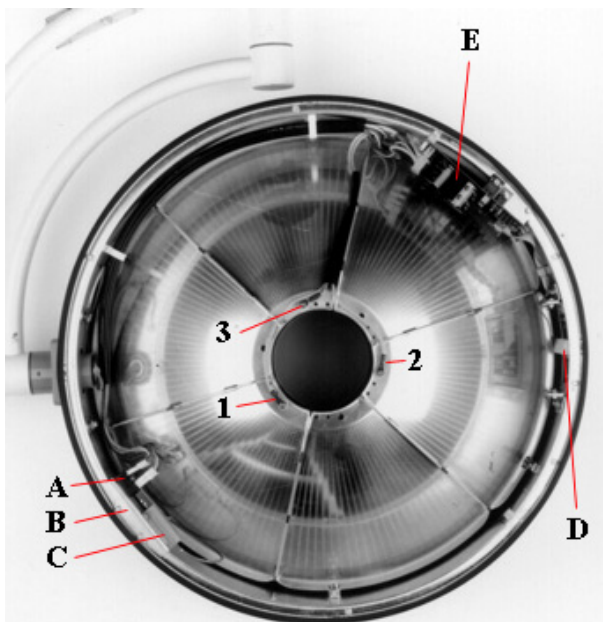
The halogen bulb is operated with a clocked constant voltage (square wave voltage) from the light regulation p.c. board. Therefor a voltmeter which is able to measure a real voltage for AC and DC parts at the same times is required (e. g. Flucke 45).

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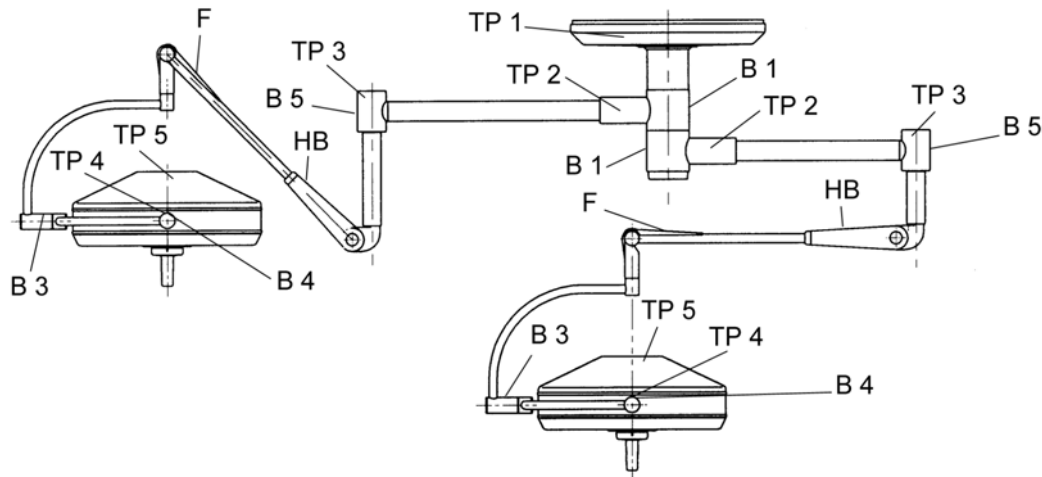B brake screw, see page 23
 TP test points, see page 39 - 40
 F spring adjustment, see page 44
 HB height movement, see page 45

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
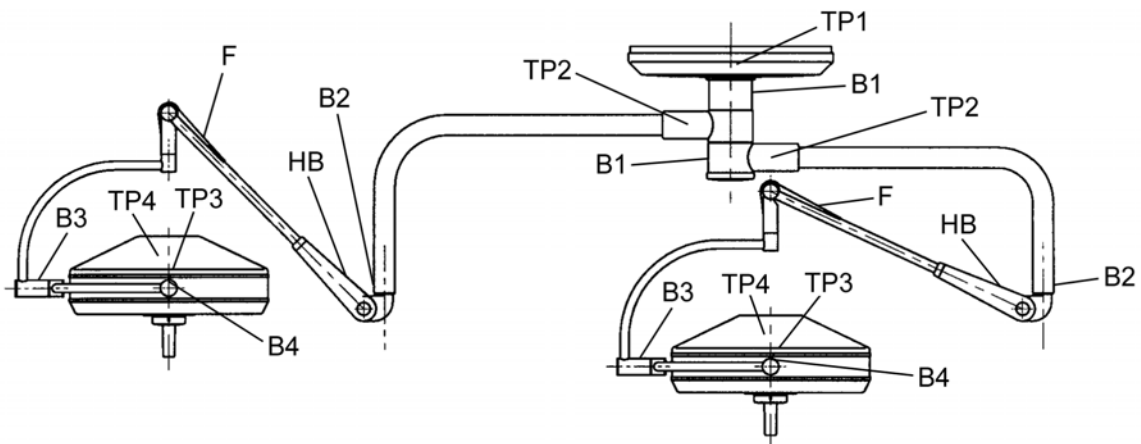
B brake screw, see page 23
TP test points, see page 41
F spring adjustment, see page 44
HB height movement, see page 45

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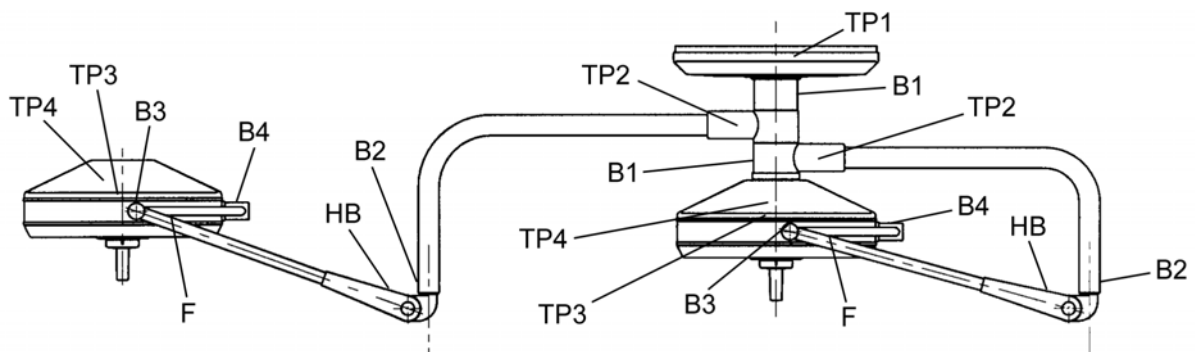B brake screw, see page 23
 TP test points, see page 41
 F spring adjustment, see page 44 - 45
 HB height movement, see page 45

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TP 1 terminal block
 27,5 V A.C. unloaded
 28,5 V D.C. unloaded

TP 2 sliding contact
 27,5 V A.C. unloaded
 28,5 V D.C. unloaded


TP 3 Plug regulation board
 Pin 5 - 6 25,5 V A.C. loaded
 27,0 V D.C. loaded

Pin 1 - 2 27,0 V D.C. loaded

TP 4 main bulb (see page 37)
 between pin 1 and 2
 max. 23,75 V A.C. / D.C. RMS
 Adjustment of the max. bulb voltage see page 38
 by means of resistance R 1

reserve bulb (see page 37)
 between Pin 2 and 3
 max. 23,75 V A.C. / D.C. RMS
 no adjustment

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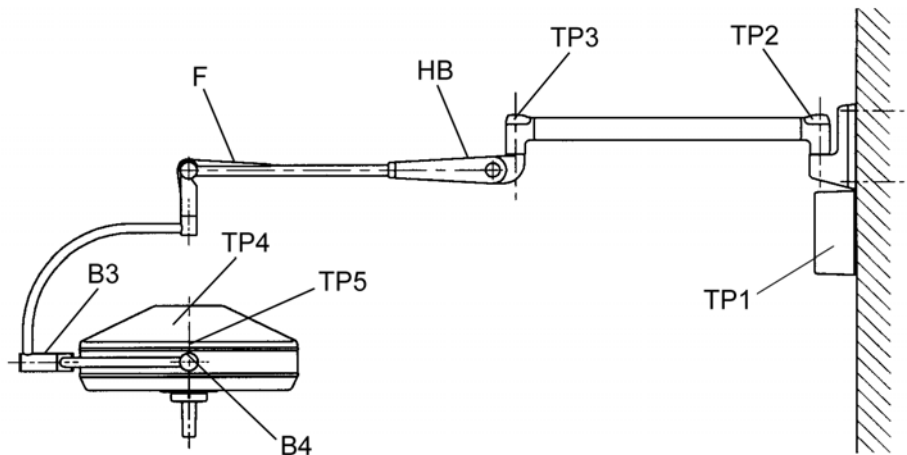
B brake screw
 TP test points, see page 43
 F spring adjustment, see page 44
 HB height movement, see page 45

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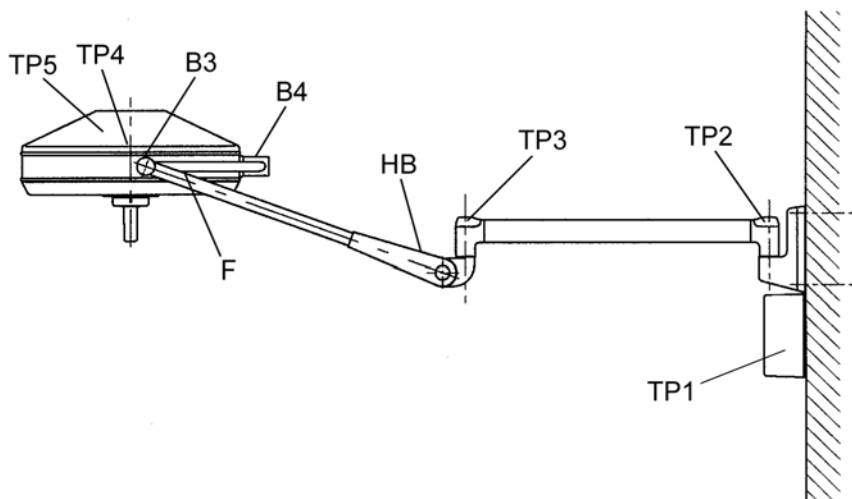
B brake screw
 TP test points, see page 43
 F spring adjustment, see page 44 - 45
 HB height movement, see page 45

Testpunkte CHROMOPHARE® C 570 / C 571 wall
Test points CHROMOPHARE® C 570 / C 571 wall

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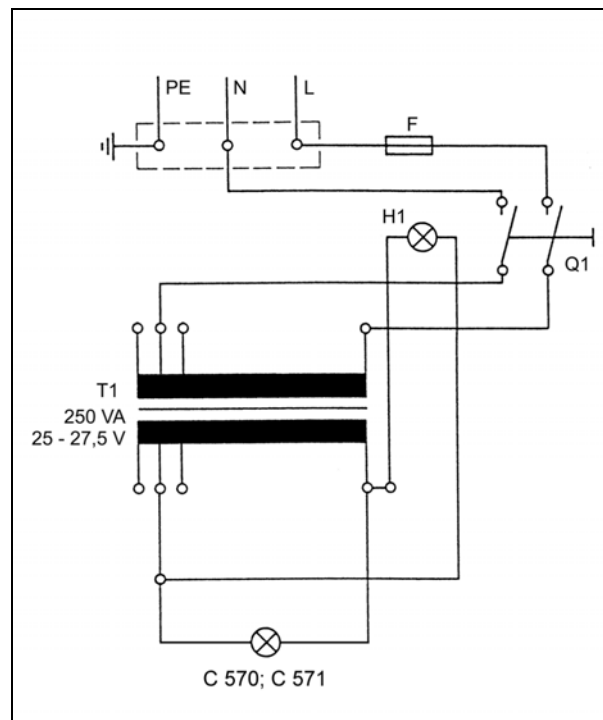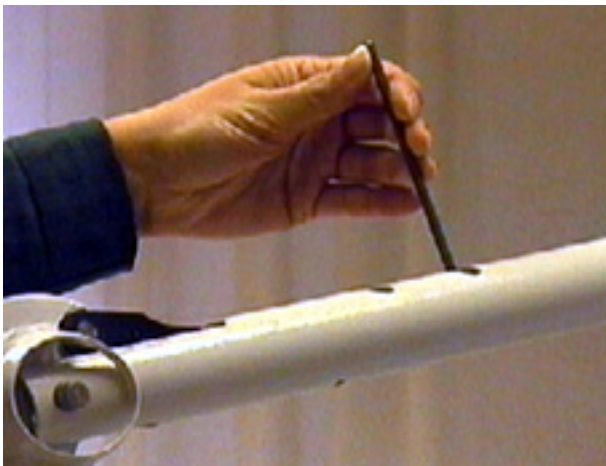
By pulling the round side caps slightly apart the main covering hull can be lifted off of the spring arm.

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Adjustment:

Light body drifts upward too easily:

Move the acrobat arm upward or downward as far as the circular nut becomes visible in one of the two windows. Turn this circular nut with a steel pin, diameter 4 mm, in the direction of the minus symbol.

Light body moves downward:

Turn the circular nut in the direction of the plus symbol.

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Remove the cover plate by using a Phillips screwdriver.

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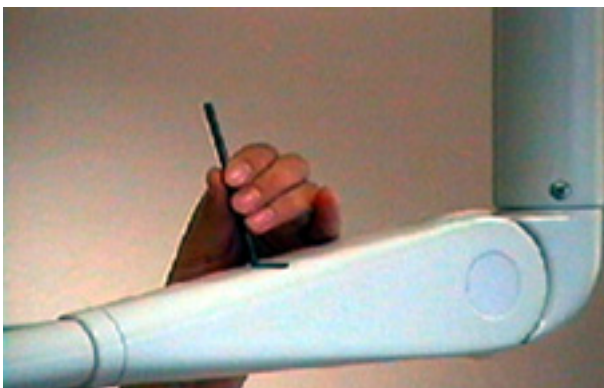
Remove the cover plate by using the steel pin.

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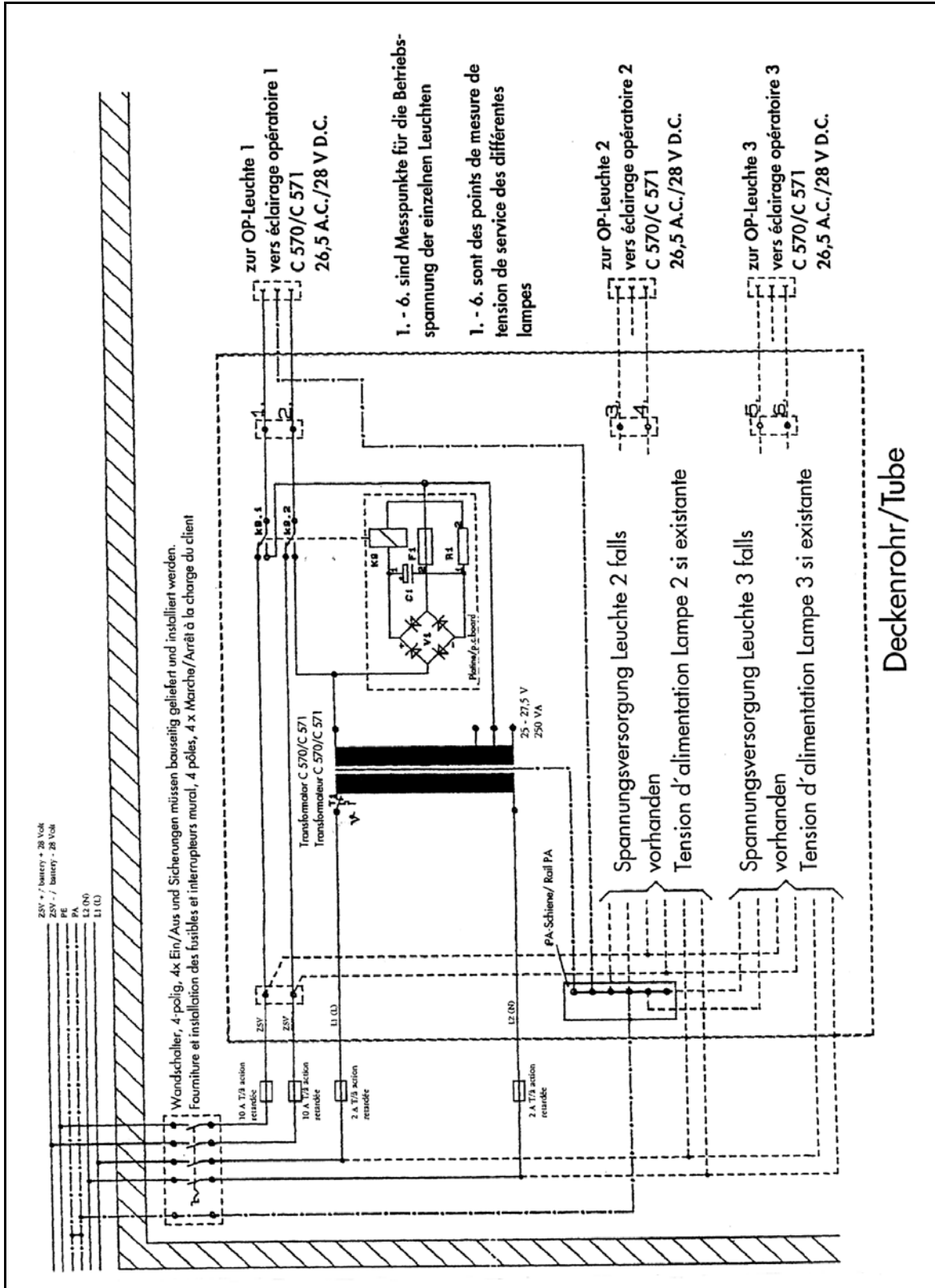
Move the acrobat arm upward or downward as far as the circular nut becomes visible. Turn this circular nut with the steel pin to limit the height movement of the acrobat arm.

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Electrical Diagram for Single or Combination Light CHROMOPHARE® C 570 / C 571

Transformers and energy relays mounted on the ceiling tube.

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
Electrical Diagram for Single or Combination Light CHROMOPHARE® C 570 / C 571

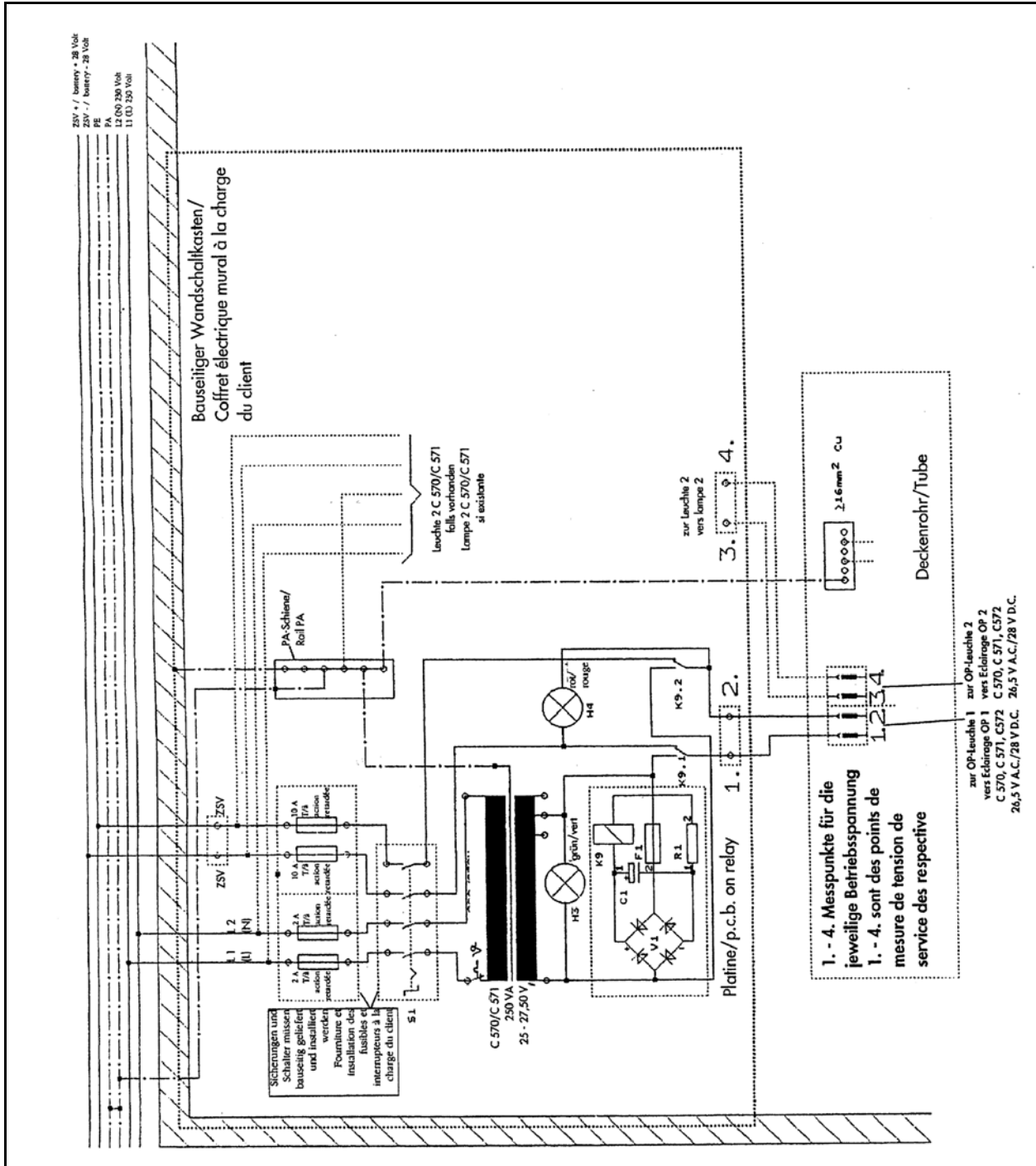Transformers and energy relays mounted in a wall box on side of building.

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Fuse values to be installed on-site [A]

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BERCHTOLD GmbH & Co. KG