

Operator and Service Manual

HOLOGIC SELENIA MAMMO NEVADA HEALTH CENTERS 38.5' L x 8'-6" W x 13'-6" H USA Unit



© 2008 Oshkosh Specialty Vehicles, All Rights Reserved.

This manual contains confidential information of Oshkosh Specialty Vehicles.

You may not copy it or any part of it without the written permission of Oshkosh Specialty Vehicles.

This manual may be used only by you and only for the purposes for which it was intended. You may not disclose this manual or the confidential information it contains outside of your company.

If you wish to copy part of this manual or to use it other than as it has been described above, you must contact Oshkosh Specialty Vehicles seeking permission to do so.

North America

Corporate Headquarters
Oshkosh Specialty Vehicles
16745 South Lathrop Ave.
Harvey, Illinois
60426
USA
(001) 708.596.5066

Europe

Oshkosh Specialty Vehicles, Ltd.
Unit 17, Nelson Way
Tuscum Trading Estate,
Camberley, Surrey
GU15 3DH

United Kingdom
(44) 01276.64490
Buys Ballotstraat 6
3261 LA Oud-Beijerland, Holland
+31 (0) 186-614322
Fax +31 (0) 186-619367
E-mail: smit@smit-mobile.nl



List of Revisions & Warnings

Revisions

00	New Release	February 2008
01	Updated and added new photos	February 2008

Notice

In accordance with our policy of product development, and in compliance with the GEMS VCR program, Oshkosh Specialty Vehicles reserves the right to make changes in the equipment, design, specifications, and materials of the product described herein. If there are any inconsistencies between this manual and the mobile unit that inhibit serviceability, please contact Oshkosh Specialty Vehicles for assistance.

This manual is one of two (2) information documents provided in the mobile unit. The documentation package consists of:

Volume I – Site Guide, Operators Manual, and associated drawings

Volume II – Vendor Information

These volumes should be kept in the mobile unit at all times.

Any problems or questions related to the components or systems covered in this manual may be directed to:

Oshkosh Specialty Vehicles
16745 South Lathrop Avenue
Harvey, Illinois
60426
USA

(001) 800.839.0630 (24 hour service)

(001) 708.596.2208 (fax)

<http://www.oshkoshsv.com/>

Warnings & Safety Alert Conventions

The following terms define the various precautions and notices used in this manual:

NOTE: Whenever information exists that requires additional emphasis beyond the standard textual information, the term “NOTE” is used.

 The term “IMPORTANT” is used whenever information exists that requires special attention to procedures to ensure proper operation of the equipment or to prevent its possible failure.

 The term “CAUTION” is used whenever potential damage to equipment exists, requiring correct procedures / practices for prevention.

 The term “WARNING” is used whenever potential personal injury or death situations exist, requiring correct procedures / practices for prevention.

 The term “DANGER” is used whenever immediate hazards exist that will result in personal injury or death that cannot be eliminated by design safeguards.



This safety alert symbol indicates important safety messages in the manual. When you see this symbol, carefully read the message that follows and be alert to the possibility of personal injury or death.

 Electrical, mechanical, pneumatic, and hydraulic safety devices have been installed on this vehicle to help protect against personal injury and / or damage to equipment. Under no circumstances should any attempt be made to disconnect or in any way render any of these devices inoperative.

If a malfunction of any safety device is discovered to exist, DO NOT operate the vehicle, but immediately notify appropriate maintenance personnel.

Oshkosh Specialty Vehicles shall have no liability with respect to: REPAIRS IMPROPERLY PERFORMED OR REPLACEMENTS IMPROPERLY INSTALLED (or) USE OF REPLACEMENT PARTS OR ACCESSORIES NOT CONFORMING TO Oshkosh SPECIALTY VEHICLE’S SPECIFICATIONS, WHICH ADVERSELY AFFECT PERFORMANCE OR DURABILITY (or) ALTERATIONS OR MODIFICATIONS NOT RECOMMENDED OR APPROVED IN WRITING BY Oshkosh SPECIALTY VEHICLES (or) FOR EQUIPMENT DAMAGE OR PERSONAL INJURY OR DEATH AS A RESULT OF RENDERING ANY SAFETY DEVICE INOPERABLE.

Certain inherent risks are associated with heavy trailers due to the nature of their use. Personnel working in the area of these trailers are subject to certain hazards that cannot be met by mechanical means but only by the exercise of intelligence, care, and common sense. It is therefore essential for the owner of this equipment to have personnel involved in the use and operation of these trailers who are competent, careful, physically and mentally qualified, and trained in the safe operation of this equipment.

Table of Contents

List of Revisions & Warnings	3
Revisions	3
Notice	3
Warnings & Safety Alert Conventions	4
Section 1: Introduction	11
Section 2: Safety Guidelines	13
2.1 Operator's General Safety Precautions	14
2.2 Electrical Safety	15
2.3 Transportation Safety	15
Section 3: Safety Systems	17
3.1 Emergency Lighting	17
3.2 Fire Suppression (manual)	18
3.3 Marker Lights	19
3.4 System Shutdowns	20
Manual Shutdown (Emergency Stop / A1 Stop)	20
Phase / Voltage Shutdown	20
3.5 Warning Lights	20
Section 4: Mobile Unit Overview	21
4.1 Breast Health Suite Overall	21
4.2 Reception / Registration Area Overall	22
4.3 Air Ride Control Valve	23
4.4 Canopies	24
4.5 Exterior Overall	25
4.6 Fuel Compartment	26
4.7 Glad-hand Connections	27
4.8 Levels, Digital	28
4.9 Mobile Unit Controls	29
4.10 Phone & Data Line Connections	31
4.11 Rear Stabilizing Stands	32
4.13 Stair Assembly	33
Section 5: Mobile Unit Setup Procedure	35
5.1 Park the Mobile Unit	35
5.2 Lower the Front Stabilizing Legs	35
5.3 Disconnect the Tractor	35
5.4 Install the Rear Stabilizing Stands	36
5.5 Re-level the Mobile Unit	36
5.6 Disconnect the Tractor Air and Electrical Lines	36
5.7 Lower the Auxiliary Support Legs	36
5.8 Connect to Shore Power	37
5.9 Switch from Generator Power to Shore Power	38
5.10 Connect the Phone and Data Lines	38
5.11 Connect the Water Hose	39
5.12 Install the Stair Assemblies	40
Stair Assembly	40



5.13	Remove Restraining Hardware and Position Equipment for Use	41
5.14	Door Canopy Deployment	41
Section 6: Mobile Unit Transport Procedure.....		43
6.1	Retract Door Canopies.....	43
6.2	Secure all Equipment	44
6.3	Remove and Store the Stair Assemblies	45
	Standard Stair System.....	45
6.4	Disconnect and Switch from Shore Power to Generator Power	45
6.5	Disconnect Phone and Data Lines.....	45
6.6	Disconnect the Water Hose	46
6.7	Raise the Auxiliary Support Legs	46
6.8	Connect the Tractor Air and Electrical Lines.....	46
6.9	Remove the Rear Stabilizing Stands	47
6.10	Connect the Tractor to the Mobile Unit.....	47
6.11	Raise the Front Stabilizing Legs.....	47
6.12	Verify that the Mobile Unit is Ready for Transport.....	48
Section 7: Electrical System.....		49
7.1	120/208 AC Electrical Panels.....	50
7.2	208V AC Phase Power Monitor	51
7.3	Facility Power Connection.....	52
7.4	Power Cable.....	53
Section 8: Generator		55
8.1	Fuel System Priming Procedure:	57
8.2	Generator Start/Stop Switch	57
Section 9: Humidity System		59
9.1	System Operation.....	59
9.2	Water Supply.....	60
9.3	Humidity Controller.....	60
9.4	Humidity Settings	60
9.5	Electrical Connections.....	61
9.6	Instructions	61
Section 10: HVAC System		63
10.1	System Specifications and Descriptions	64
10.2	Exterior HVAC Specifications	64
10.3	Interior HVAC Specifications	65
10.4	Thermostat Temperature Setting.....	65
Section 11: Wheel Chair Lift		67
Section 12: Stabilizing Legs		69
12.1	Stabilizing Stands	70
12.2	Rear Air Suspension System Controls	71
Section 13: Lighting System		73
13.1	Emergency Lighting.....	73
13.2	Exterior Lighting.....	74



Underbody Compartment Lighting.....	74
Entrance Door Lighting / Exterior Service Lighting.....	75
Marker & Running Lights.....	75
13.3 Interior Lighting.....	76
Reception Area.....	76
Breast Health Suite.....	77
13.4 Warning Lights.....	78
AC Power Indicator Light.....	79
Transport Warning Light.....	79
Suspension Transport Warning Light.....	79
Power Loss Transport Warning Strobe Light.....	79
Section 14: General Maintenance	81
14.1 Daily Maintenance.....	81
14.2 Weekly Maintenance.....	81
14.3 Monthly Maintenance.....	82
14.4 Quarterly Maintenance.....	83
Section 15: Specific Maintenance	85
15.1 Door Closer Adjustments.....	85
15.2 Electrical System.....	85
15.3 Generator System.....	86
15.4 Humidity System.....	86
15.5 HVAC System.....	86
15.6 Stabilizing Legs.....	87
Appendix A: Mobile Unit Checklist	89
Mobile Unit Setup Checklist.....	90
Mobile Unit Transport Checklist.....	91
Appendix B: Troubleshooting	93
AC Power Indicator Light is off... ..	93
Transport Warning Light is on.....	93
Suspension Transport Indicator Light is illuminated... ..	93
Temperature is out of specifications... ..	94
Humidity is out of specifications.....	95
Appendix C: HVAC Set Points.....	97
Temperature Controller Settings.....	97
Humidity Settings.....	97
Appendix D: Circuit Malfunction Checklist	99
Category 1.....	99
Category 2.....	99
Appendix E: Lockout/Tagout Procedures	101
Specific Energy Control Procedures.....	101
Machine or Equipment for this Procedure:.....	101
Specialty Vehicle Trailer.....	101
Control of Hazardous Energy:.....	101
Affected Personnel to notify when the Specialty Vehicles Trailer is to be Locked Out:.....	101



Shut down specifications for the Specialty Vehicle Trailers: 102
Methods to dissipate energy: 102
Method of Verifying the Isolation of the Machine or Equipment:..... 102

Appendix F: Quarterly Maintenance Checklist 103

Appendix G: Medical System 107

List of Figures

Figure 1: The Hologic Selenia Mammography Unit.....	11
Figure 2: Emergency Lighting.....	17
Figure 3: Fire Extinguisher.....	18
Figure 4: Marker Lights.....	19
Figure 5: Emergency Shutdown Buttons.....	20
Figure 6: Breast Health Suite Overall.....	21
Figure 7: Registration Area Overall.....	22
Figure 8: Air Ride Control Valve.....	23
Figure 9: Door Canopies.....	24
Figure 10: Exterior Overall.....	25
Figure 11: Fuel Compartment.....	26
Figure 12: Glad Hand Connections.....	27
Figure 13: Digital Levels.....	28
Figure 14 Mobile Unit Controls.....	30
Figure 15: Phone & Data Line Connections.....	31
Figure 16: Rear Stabilizing Stands.....	32
Figure 17: Front Stabilizing Legs and Auxiliary Support Legs.....	32
Figure 18: Stair Assembly.....	33
Figure 19: 208V Power Control Panel.....	37
Figure 20: Humidifier Tank Fill.....	39
Figure 21: Mobile Barrier & Roll Around Sink Restraints.....	44
Figure 22: 120/208V AC Electrical Panel.....	50
Figure 23: 208V AC Phase Power Monitor.....	51
Figure 24: Shore Power Connection.....	52
Figure 25: Generator Compartment.....	56
Figure 26: Advanced Digital Control.....	57
Figure 27: Humidity Controller.....	60
Figure 28: A/C Units.....	63
Figure 29: Thermostat Temperature Control.....	65
Figure 30: Wheel Chair Lift.....	67
Figure 31: Front Stabilizing Leg Assembly.....	69
Figure 32: Rear Stabilizing Stands.....	70
Figure 33: Air Bag Controls.....	71
Figure 34: Emergency Lighting.....	73
Figure 35: Compartment Light.....	74
Figure 36: Entry Door Lighting.....	75
Figure 37: Interior Overall Lighting.....	76
Figure 38: Breast Health Suite Overall Lighting Controls.....	77
Figure 39: Warning Lights.....	78



Section 1: Introduction

WARNING

This manual is intended to instruct and assist personnel already qualified in the proper installation of the mobile unit. This manual is not intended to enable persons unfamiliar with the mobile unit to perform the setup and transport procedures.

IMPORTANT

An outside radiation physicist consultant determines the x-ray shielding based upon unit layout that is provided by Oshkosh Specialty Vehicles and scatter patterns provided the medical equipment manufacturer. It is the user's responsibility to ensure proper maintenance of the x-ray shielding. It is the recommendation of Oshkosh Specialty Vehicles that the end user has the x-ray testing completed on an annual basis to ensure that the mobile unit still meets the minimum requirements.

This manual contains the basic information needed to setup, transport, and service the mobile unit. This mobile unit was designed to operate within certain limitations and specifications. When performing the setup or transport procedures for the mobile unit, follow the proper logical steps that have been outlined in this manual. The drawings in this manual are representative of this product. In accordance with our program of continued product development, designs and specifications are subject to change without notice.



Figure 1: The Hologic Selenia Mammography Unit



As part of Oshkosh Specialty Vehicles' on-going program to improve its products and service, (and their effectiveness in enhancing safety, reliability, performance, productivity, and the useful service life of the equipment) Oshkosh Specialty Vehicles reserves the right to implement product changes and disseminate changes in design and service information without notice or recourse.

For questions regarding the Operation or Service of this unit call Oshkosh Specialty Vehicles at 800-839-0630.



Section 2: Safety Guidelines



Electrical, mechanical, pneumatic, and hydraulic safety devices have been installed on this vehicle to help protect against personal injury and / or damage to equipment. Under no circumstances should any attempt be made to disconnect or in any way render any of these devices inoperative. If a malfunction of any safety device is discovered to exist, **DO NOT** operate the vehicle, but immediately notify appropriate maintenance personnel.



Use and follow the appropriate Lockout/Tagout procedures as required by OSHA Standard 1910.147 when performing maintenance or servicing any electrical, hydraulic or pneumatic systems. See Appendix E for Lockout/Tagout procedures.



It is the operator's responsibility to verify that the shore power receptacle is of the same type and voltage as the connection that is supplied by Oshkosh Specialty Vehicles. Failure to do this can result in injury or death to the operator of the mobile unit as well as irreparable damage to the mobile unit.



Make sure that all electrical parts are serviced only by a certified electrician or qualified personnel. Dangerous voltages are present which could result in injury or death.



Always make sure that eyes are protected while servicing the unit. Wear safety goggles when prying, drilling, grinding, or working with batteries. Wear safety goggles over regular prescription glasses unless the lenses are made of hardened glass and can serve as safety goggles.



Be certain to disconnect the power before working on any of the electrical systems.



When servicing the unit be certain that a first aid kit and fire extinguisher are within reach at all times.

This safety section contains important information in regards to general safety guidelines that should be followed. Before attempting to service the mobile unit, read this safety section as well as all other safety sections found in applicable manufacturers' manuals in the component literature binder.



2.1 Operator's General Safety Precautions

Your safety and the safety of other persons in the area of this vehicle are the result of your correct operation of this vehicle. Know the location, positions, and functions of all the controls. Know the meaning of the various Warning, Caution, Strobe, and Annunciator lights and their associated audible warning sounds.

Read this manual completely and make sure you understand the contents. Make sure you understand, for example, the characteristics of speed, stability, brakes, and steering, etc. of this vehicle. If you have any questions, contact Oshkosh Specialty Vehicles, (800) 839-0630. Always keep a copy of this manual with the vehicle.

The safety information in the manual does not replace any other rules or laws for safety that are used in your area. Know the local rules or laws for safety. Make sure that your vehicle has the correct equipment to operate according to these rules or laws.

All safety hazards that can possibly arise cannot be foreseen and noted in this manual. You must always use common sense and apply the general as well as the specific safety precautions.

Make sure the work area is well ventilated.

Disconnect the electrical power to prevent the possibility of electrical shock when servicing all electrical equipment.

Follow all manufacturers' directions and request material data sheets where applicable.

Always keep tools clean and free of grease.

Do not stand on chairs inside of the mobile unit under any circumstances.

Follow all safety precautions found in the documentation package that is included with the mobile unit.



2.2 Electrical Safety



Use and follow the appropriate Lockout/Tagout procedures as required by OSHA Standard 1910.147 when performing maintenance or servicing any electrical, hydraulic or pneumatic systems. See Appendix E for Lockout/Tagout procedures.



Before connecting or disconnecting from shore power, it is imperative that the shore power connections be moved to the "OFF" position. Failure to do this can result in injury or death to the operator of the mobile unit.



It is the operator's responsibility to verify that the shore power receptacle is of the same type and voltage as the connection that is supplied by Oshkosh Specialty Vehicles. Failure to do this can result in injury or death to the operator of the mobile unit as well as irreparable damage to the mobile unit.



Always inspect the power cable, connectors, and fasteners prior to usage. If during inspection, it is suspected that either internal or external damage has occurred, have a certified electrician inspect and repair the damage before using.

When working with the electrical system for the mobile unit. Follow the warnings and cautions listed above.

2.3 Transportation Safety

Walk around the unit to make certain that all doors are closed and locked.

If any of the warning lights are illuminated, do not move the mobile unit.

Before moving the mobile unit, verify that all marker and running lights are working properly.

Consult with the local motor vehicle authority to determine if there are any travel restrictions or routes.



Section 3: Safety Systems

This safety section contains important information about the safety systems that have been built into the mobile unit to protect all personnel and equipment. Before attempting to service the mobile unit, read this safety section as well as all other safety sections found in applicable manufacturers' manuals in the component literature binder.

3.1 Emergency Lighting

In the event that the main AC power fails, five dual beam emergency lights are provided in the Breast Health Suite, Reception Area, and Main Hallway. These lights will automatically illuminate when the main AC power is lost. The emergency lighting system is wired into a 120V AC electrical system that allows the lights internal circuitry to keep their batteries at 100% charge. The emergency lights will illuminate the exit doors last for approximately 90 minutes.



Figure 2: Emergency Lighting

3.2 Fire Suppression (manual)

Two fire extinguishers are supplied with the mobile unit. Instructions for operation are clearly printed on the canister of the fire extinguisher. The fire extinguisher meets the following standards.



Located on the right hand side of Main entrance door as you enter the unit.



Located to the left of the entrance door of the Breast Health Suite room.

Figure 3: Fire Extinguisher

- It is a class A/B/C 1211 hand held unit.
- It has a charged weight of 2 lbs., 8 oz.
- It is U.L. listed.
- It meets D.O.T. requirements.
- It is in accordance with N.F.P.A. Standard No. 10, "Portable Fire Extinguisher".

3.3 Marker Lights

Extra LED type marker and side turn signal lights are installed on the trailer body to assist the driver with maneuvering the mobile unit.



Figure 4: Marker Lights

3.4 System Shutdowns

There are different types of shutdowns that can take place on the mobile unit. Of the different types, both manual and automatic shutdowns exist. All shutdowns refer only to the medical system and not the HVAC system unless otherwise noted.



Figure 5: Emergency Shutdown Buttons

Manual Shutdown (Emergency Stop / A1 Stop)

Manual shutdowns are those that require the operating personnel to depress "Emergency OFF" buttons in the event of an emergency. The "Emergency OFF" button is located in Breast Health Suite aboard the mobile unit. When this button is depressed, only the medical system will be shutdown. The HVAC system will still be operational.

There is one (1) "Emergency Off" button located in Breast Health Suite on the left side wall.

Phase / Voltage Shutdown

If the incoming power is out of phase or the voltage is out of specification, a power monitor will automatically trip the shunt breaker that feeds the medical system.

3.5 Warning Lights

Please Refer to [Section 13: Lighting System](#) for additional information in regards to these systems.

Section 4: Mobile Unit Overview

The components of the mobile unit have been divided into alphabetical order following a brief description of the unit. With each component a picture and description will be found to better illustrate the components of the mobile unit. Additional components of the mobile unit can be found within the remaining chapters.

4.1 Breast Health Suite Overall

The Breast Health Suite houses the system components that support the mammography medical system. In this room, the operator's console can be found, along with the system controls and cabinets for storage.



Figure 6: Breast Health Suite Overall

4.2 Reception / Registration Area Overall

The Reception / Registration Area is provided to allow our guests to register for a mammography exam in the Breast Health Suite.



Figure 7: Registration Area Overall

4.3 Air Ride Control Valve



The air ride control valve must be in the normal ride position before the mobile unit can be transported. If the air ride control valve is not in the normal ride position, irreparable damage may occur to the mobile unit.

The air ride control valve adjusts the rear air suspension bags. When the mobile unit is being transported, the air ride control valve must be in the normal ride position.



Figure 8: Air Ride Control Valve

4.4 Canopies

The canopies are positioned above the doors to provide shelter from the elements.



Figure 9: Door Canopies

1. Front door Canopy.
2. Wheel Chair Lift Rear door Canopy.

4.5 Exterior Overall

In these illustrations the entry doors and the generator housing can be seen.



Left Side



Right Side

Figure 10: Exterior Overall

4.6 Fuel Compartment

The fuel compartment, located in the lower section of the generator compartment, stores the fuel tank, fuel gauge, fuel pump, and fuel separator.



Figure 11: Fuel Compartment

- Fuel Gauge:** The fuel gauge, located on the right side of the generator compartment, is push button activated and will give an accurate reading of the available fuel supply.
- Fuel Pump (not shown):** The fuel pump pumps the fuel from the fuel tank to the generator.
- Fuel Separator (not shown):** The fuel separator removes debris from the fuel supply.
- Fuel Tank:** The fuel tank stores and supplies fuel to the generator. The capacity of the fuel tank is 70 US gallons. Only use diesel fuel.

4.7 Glad-hand Connections

The glad hands are the connection point between the tractor and the mobile unit. All connections must be made before moving the mobile unit. Failure to make all connections can result in damage to the mobile unit.



Key lock
Box

Figure 12: Glad Hand Connections

Emergency Airline:	Backup airline in the event that the main airline fails.
Key Lock Box:	A combination lock that holds a key to the mobile unit.
Service Airline:	The main airline for the mobile unit.
Standard Electrical Service:	The main electrical connection for the mobile unit.
Generator Access:	The generator housing comes with locking doors to prevent unauthorized access.

4.8 Levels, Digital


The digital levels allow the mobile unit to be leveled both front to back and side to side. It is imperative that the unit be leveled prior to use.



Figure 13: Digital Levels

4.9 Mobile Unit Controls

Located on the walls and panels inside of the mobile unit are the various controls that are used for operating such items as, the interior and exterior lights, emergency stop buttons, fire alarms, and emergency equipment.

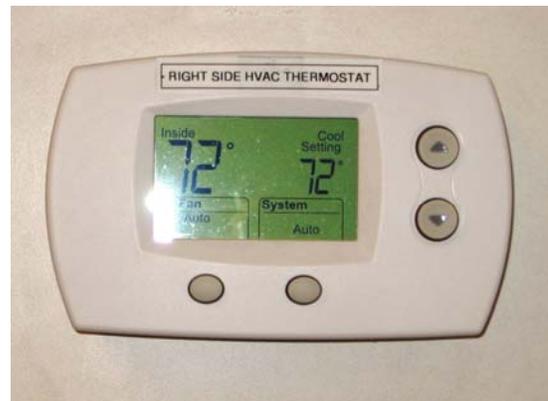

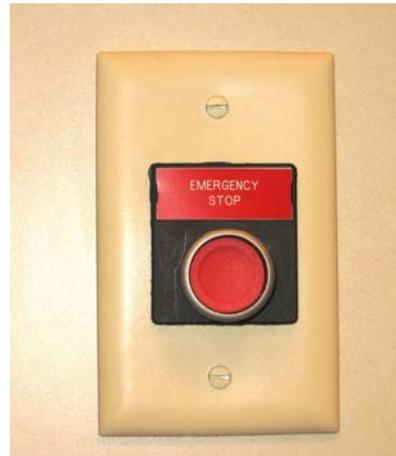


Figure 14 Mobile Unit Controls

- | | |
|---------------------------------------|--|
| X-Ray Room Light Switches: | ON / OFF light switch for X-Ray Room lights. |
| Exterior and Interior Light Switches: | ON / OFF light switch for the exterior lights. |
| Humidistat: | Humidity information of the mobile unit. |
| Thermostat | Temperature controls for the mobile unit. |
| E Stop: | Emergency stop button for the medical system. |

4.10 Phone & Data Line Connections

The phone and data connections are located in the underbody compartments. The connections are used to connect the mobile unit to the shore facility. The telephone connections utilize a Hubbell all weather connection, while the data lines utilize an RJ-45 connection and CAT-5E cabling.



Figure 15: Phone & Data Line Connections

Phone Connections

The phone connections that are utilized are RJ-45. The connections utilize CAT-5E cable and can be connected directly to the facility.

Data Connections

The data connections that are utilized are RJ-45. The connections utilize CAT-5E cable and can be connected directly to the facility.

4.11 Rear Stabilizing Stands

The rear stabilizing stands are placed underneath the rear of the mobile unit when the medical system is in use. These stands help to level the mobile unit and decrease vibration caused by the medical system.



Driver side



Curb side

Figure 16: Rear Stabilizing Stands

4.12 Front Landing / Stabilizing Legs

The Front Landing / Stabilizing legs and auxiliary support legs can be found at front of the mobile unit. They are used in order to level the unit prior to use. Since the landing / stabilizing legs are hydraulically controlled, the manual auxiliary legs must also be used as a backup.



Figure 17: Front Stabilizing Legs and Auxiliary Support Legs



Support Leg

Stabilizing Leg

4.13 Stair Assembly

The stairs allow access to the interior of the mobile unit through the staff door.

The stairs attach directly to the mobile unit, as shown.

Can set up a parallel stair with a platform.



Figure 18: Stair Assembly



Section 5: Mobile Unit Setup Procedure



The Hologic Selenia medical system requires the HVAC system to be supplied power at all times. During transit of the mobile unit via the generator and when the unit is in the parked position via shore power.



The stabilizing legs and rear suspension are not to be used to raise the mobile unit off the ground. The legs are meant only to level the unit and place it in a parked position. If the legs are used in an attempt to raise the mobile unit from the ground, serious damage may occur to the mobile unit.



A checklist can be found in Appendix A that may be used as a guideline for the following procedure.

5.1 Park the Mobile Unit

In order to join the mobile unit to the facility, place the unit on the pad per the site-planning guide. Set the trailer parking brake.

5.2 Lower the Front Stabilizing Legs

After the mobile unit has been parked on the pad per the site-planning guide, the front stabilizing legs must be lowered to stabilize the mobile unit before it can be used. Refer to [Figure 31: Front Stabilizing Leg Assembly](#) for the following procedure.

1. Move and hold the pump switch in the “Pump ON” position.
2. Pull the levers towards you to extend the stabilizing legs to their extended position.
3. Extend the legs far until the front of the unit has been raised high enough to clear the fifth wheel.
4. Release the pump switch. The switch should automatically retract to the “Pump OFF” position.

5.3 Disconnect the Tractor

Once the landing legs have been lowered, the tractor can be disconnected from the mobile unit. Verify that the mobile unit has been raised high enough to clear the tractor fifth wheel, and then disconnect the tractor from the mobile unit. Do not disconnect the air and electrical lines.

5.4 Install the Rear Stabilizing Stands

1. Open the rearmost underbody storage compartments on each side of the mobile unit and remove the rear stabilizing stands.
2. Place that stands underneath the rear of the mobile unit where the structural supports are located.
3. Return to the landing / stabilizing leg controls at the front of the mobile unit.
4. Locate on this control panel a switch that reads “DN” to the far left and “UP” to the far right.
5. Move this switch to the “DN” position.
6. The rear air suspension system will automatically deflate and the mobile unit will lower itself onto the rear stabilizing stands.
7. Check to verify that the structural supports for the mobile unit are centered on the stabilizing stands. If the supports are not centered, then the mobile unit must be raised and the stands must be adjusted accordingly.
8. If shims are needed, only use the shims that have been provided by the mobile unit.
9. Close the underbody compartment doors.

5.5 Re-level the Mobile Unit

After the preceding steps have been completed, the mobile unit may no longer be level. Re-level the unit if necessary using the digital levels that have been provided. Refer to [Figure 13: Digital Levels](#) if needed.

5.6 Disconnect the Tractor Air and Electrical Lines



Failure to completely exhaust the suspension before uncoupling the air lines may result in damage to the suspension of the mobile unit.

After the mobile unit has been re-leveled, the tractor air and electrical lines can safely be removed. Refer to [Figure 12: Glad Hand Connections](#).

5.7 Lower the Auxiliary Support Legs

After the preceding steps have taken place, the auxiliary support legs can now be lowered. Refer to [Figure 31: Front Stabilizing Leg Assembly](#) for the following procedure.

1. Remove the pin that is currently holding the auxiliary leg in the transport position.
2. Lower the auxiliary support leg to within ½” of the sand shoe and insert the pin into the highest available hole to lock the leg in position.

5.8 Connect to Shore Power



Before connecting or disconnecting from shore power, it is imperative that the shore power connections be moved to the “OFF” position. Failure to do this can result in injury or death to the operator of the mobile unit.



It is the operator’s responsibility to verify that the shore power receptacle is of the same type and voltage as the connection that is supplied by Oshkosh Specialty Vehicles. Failure to do this can result in injury or death to the operator of the mobile unit as well as irreparable damage to the mobile unit.



Always inspect the power cable, connectors, and fasteners prior to usage. If during inspection, it is suspected that either internal or external damage has occurred, have a certified electrician inspect and repair the damage before using.

NOTE: The unit must first be switched to shore power before the medical system can be used. Refer to [Figure 24: Shore Power Connection](#) , for the following procedure.

Verify that the shore power disconnect is in the “OFF” position.

Open the underbody compartment door and remove the power cable from the underbody compartments of the mobile unit.

Insert the trailer connector into the shore power receptacle and spin the lock ring clockwise to secure the connection.

Move the shore power disconnect to the “ON” position.

Close the underbody compartment door; making sure that the access flap for the power cable has been released.

Note: The Phase Power Monitor checks the incoming shore power to ensure that it has the correct phase rotation (ABC) and that all three phases are present. If all three phases are present and in the correct rotation, the 208 VAC Light, on the monitor, will illuminate.

If any phase is not present or if the phase rotation is not correct, the 208V AC Fault red Light will illuminate, and a piezo-electric horn will sound. Disconnect shore power immediately and investigate to determine the cause of the fault.



Figure 19: 208V Power Control Panel

5.9 Switch from Generator Power to Shore Power



When switching from generator power to shore power the “480V AC FAULT” may illuminate and flicker. If the “480V AC FAULT” stays illuminated, reconnect to generator power and call a certified electrician before attempting to reconnect to shore power.



When turning the power selector switch for the incoming power, from one position to another, the selector must be paused for a minimum of five seconds, in the “OFF” position between selections. Failure to do so can result in damage to the equipment.

Open the lower compartment door that houses the electrical components to access the Power Supply Control Panel. Refer to [Figure 19: 208V Power Control Panel](#).

Turn the unit power selector on the reversing contactor panel from “Generator1-Lower or 2-Upper to “Line”. Be certain to pause for five seconds in the "OFF" Position between selections.

The two indicators on this panel will illuminate according to the power that is received.

If the supply of power is good, the green indicator labeled “208V AC OKAY” will illuminate.

If the supply of power is experiencing problems, such as being out of phase, the red indicator labeled “208V AC FAULT” will illuminate.

Since minor fluctuations in power are common, the “208V AC FAULT” may briefly flicker. This is to be expected. If the “208V AC FAULT” indicator light stays illuminated, switch back to generator power and contact a qualified electrician before operating the medical system.

If the power supply is good, the selector for the generator can be moved to the “STOP” position. Once this has been done, the generator will enter into a cooling phase and continue running for approximately two minutes.

Turn the Generator selector switch to the “OFF” position, back to "ON" then to "OFF" again to shut down the generator.



The Power Cable could present a trip hazard that could result in personal injury. Care should be taken to ensure that the cable is routed properly to minimize its potential as a trip hazard.

5.10 Connect the Phone and Data Lines

The phone and data lines can be found in the underbody storage compartments. Both the phone and data lines can now be connected from the outlets located in the underbody compartments to the receptacles located at the shore site.

The phone lines make use of Hubbell all weather connections. The data lines make use of CAT-5E cable and RJ-45 connections. Refer to [Figure 15: Phone & Data Line Connections](#).

5.11 Connect the Water Hose

The fresh water connection is located on left side of the unit forward of the Electrical compartment.



Figure 20: Humidifier Tank Fill

Remove the cap that covers the connection.

Attach the supplied water hose to this connection.

Attach the other end of the hose to facility provided faucet.

Turn on the water at the faucet.



5.12 Install the Stair Assemblies

Attach the stairs directly to the mobile unit. The stairs can be setup easier with two people. The instructions are covered below.

Stair Assembly

1. Remove the all stair assemblies from the underbody compartments.
2. Install the clip of the stair assembly into the channel located underneath the entrance door.
3. Adjust the height of the stair legs as necessary to in order to level and secure the stairs.
4. Install the handrail into its operating position and secure in place with the hardware provided.
5. Close the doors to the underbody compartments.

5.13 Remove Restraining Hardware and Position Equipment for Use.

There are two types of restraints that may need to be removed prior to using the medical system. They are as follows.

The first type deals with the restraints that are used by the medical equipment manufacturer. Follow all instructions provided by the medical equipment manufacturer when adding or removing restraints from the medical equipment. These instructions can be found in the system manuals provided by the medical equipment manufacturer.

The second type deals with the restraints that are used by the mobile unit manufacturer. Various items may be secured while the unit is being transported. These items may consist of chairs, monitors, door, cabinets, cameras, and printers. Remove all restraining equipment prior to usage of the medical systems.

5.14 Door Canopy Deployment

Install the canopies above the entry doors to the mobile unit. Please follow the instructions below and refer to [Figure 9: Door Canopies](#). The canopies can be operated by either remote control from the outside of the trailer or by using the wall switches inside the unit.

Retrieve the canopy remote controls from their storage location inside the unit.

Press the remote control switch or the wall switch inside to extend the front and rear canopies.

An anemometer located on the right upper front side of the unit continually senses wind velocity. If the wind speed reaches 20 MPH, the canopies will automatically close. The signal to close the canopies cannot be overridden. The canopies may be redeployed when the wind speed falls below 20 MPH.





Section 6: Mobile Unit Transport Procedure



If the mobile unit is moved without the rear air suspension functioning properly, irreparable damage can occur to the mobile unit.



The Hologic Selenia medical system requires the HVAC system to be supplied power at all times. During transit of the mobile unit via the generator and when the unit is in the parked position via shore power.



The stabilizing legs and rear suspension are not to be used to raise the mobile unit off the ground. The legs are meant only to level the unit and place it in a parked position. If the legs are used in an attempt to raise the mobile unit from the ground, serious damage may occur to the mobile unit.



Before transporting the mobile unit, check to verify all warning lights as well as all exterior marker lights are working correctly.



A checklist can be found in Appendix A that may be used as a guideline for the following procedure.

6.1 Retract Door Canopies

Retract the canopies. Please follow the instructions below and refer to [Figure 9: Door Canopies](#). The canopies can be operated by either remote control from the outside of the trailer or by using the wall switches inside the unit.

Retrieve the canopy remote controls from their storage location inside the unit.

Press the remote control switch or the wall switch inside to retract the front and rear canopies.

6.2 Secure all Equipment



The following procedure must be accomplished prior to transporting this vehicle. If these items are not accomplished, the "Transit Warning Light", located on the left side of the mobile unit will remain illuminated.

Two types of restraints need to be supplied before transporting the mobile unit. They are as follows:

The first type deals with the restraints that are used by the medical equipment manufacturer. Follow all instructions provided by the medical equipment manufacturer when applying restraints to the medical system. These instructions can be found in the system manuals provided by the medical equipment manufacturer.

The second type deals with the restraints that are used by the mobile unit manufacturer. Various items must be secured prior to transporting the mobile unit. Such items may consist of chairs, monitors, doors, cabinets, cameras, and printers. Use the supplied restraining hardware to secure these items before transporting the mobile unit.



Mobile Barrier placed in position



Bungee cord

Mobile Barrier restraint for transport



Restraints in place



Roll Around Sink restraint

Figure 21: Mobile Barrier & Roll Around Sink Restraints

6.3 Remove and Store the Stair Assemblies

Before removing the stair assemblies, check the interior of the unit one last time to verify that all equipment is secure and ready for transport.

The stair assemblies can be taken down easier with two people. The instructions are covered below.

Standard Stair System

1. Close and lock the door with the key that is provided.
2. Open the door to the underbody storage compartment.
3. Loosen the hardware holding the handrail in place. Remove the handrail from the stair assembly.
4. Lift the clip of the stair assembly up and away from the channel that is located underneath the staff door.
5. Place the stair assembly on the ground.
6. Using the sole of your shoe, step on the spring loaded release to retract the adjustable legs on each side of the stair assembly.
7. Place the stair assembly and handrail inside of the underbody storage compartment and close the compartment door.

6.4 Disconnect and Switch from Shore Power to Generator Power



When switching from generator power to shore power the “208V AC FAULT” may illuminate and flicker. If the “208V AC FAULT” stays illuminated, reconnect to generator power and call a certified electrician before attempting to reconnect to shore power.



When turning the power selector switch from one position to another, the selector must be paused for a minimum of five seconds between selections. Failure to do so can result in damage to the equipment.

Please refer to Section 8: Generator, for the following procedure.

Open the lower compartment door that houses the electrical components to access the Power Supply Control Panel. Refer to [Figure 19: 208V Power Control Panel](#).

Move the Generator #1 or #2 Stop / Start selector switch to the “Start” position to start the generator.

Allow the generator to run for approximately two (2) minutes.

Move the Unit Power Selector switch to the “GEN#1-Lower or Gen#2-Upper” position. Be certain to pause for five seconds in the “OFF” position between selections.

Close the Equipment Room door.

6.5 Disconnect Phone and Data Lines

Please refer to [Figure 15: Phone & Data Line Connections](#), for the following procedure.

Disconnect any phone and data lines that are currently attached to shore receptacles.

Open the compartment door and disconnect any phone and data lines that are connected in the underbody storage compartment.

Store the phone and data lines in the underbody storage compartment and close the compartment door.

6.6 Disconnect the Water Hose

On the right side exterior of the mobile unit a water connection can be found. Be sure to fill the fresh water tank prior to disconnecting the fresh water supply. The fresh water tank must be filled on a daily basis. Refer to [Figure 20: Humidifier Tank Fill](#) for procedures.

Verify that the fresh water tank is full.

Turn off the water supply at the facility provided faucet.

Disconnect the hose from the faucet.

Remove the hose from the connection on the mobile unit.

Using the flap provided, cover the connection on the mobile unit.

Coil the hose and store in the underbody compartments.

6.7 Raise the Auxiliary Support Legs

Please refer to [Section 12: Stabilizing Legs](#) for the following procedure.

Remove the pins holding the auxiliary support legs in the locked positions.

Lift the auxiliary support legs high enough for the pin to be inserted into the lowest available hole, thereby holding the leg as high as possible.

Make sure this has been done for both auxiliary support legs. Failure to do this can damage the stabilizing legs when they are retracted.

6.8 Connect the Tractor Air and Electrical Lines

In order to remove the rear stabilizing stands, the air and electrical lines must first be connected from the tractor to the mobile unit. Please refer to [Figure 12: Glad Hand Connections](#) and follow the steps outlined below.

Back up the tractor to the mobile unit, but do not back under it at this time.

Attach the air and electrical lines from the tractor to the mobile unit.

6.9 Remove the Rear Stabilizing Stands

Open the underbody compartment door that stores the rear stabilizing stands.

Open the underbody compartment door that leads to the controls for the Landing / Stabilizing Legs.

Locate on this control panel a switch that reads “DN” to the far left and “UP” to the far right.

Move this switch to the “UP” position.

The rear air suspension system will automatically inflate and the mobile unit will rise to the normal ride height.

Remove the aluminum shims (if applicable). Place the shims in the rear underbody storage compartment.

Remove the rear stabilizing stands. Place the stands in the rear underbody storage compartments.

Close the underbody storage compartment doors.

6.10 Connect the Tractor to the Mobile Unit

Before connecting the tractor to the mobile unit, be certain that enough clearance has been left for the fifth wheel. If the fifth wheel cannot fit underneath the mobile unit, the front end must be raised. If it is necessary to raise the front of the mobile unit to clear the fifth wheel, please follow the steps below. Please refer to [Section 12: Stabilizing Legs](#), for the following procedure.

Move and hold the pump switch in the “ON” position.

Pull the levers towards you to extend the legs. This will lift the front end of the mobile unit.

Extend the legs high enough to clear the fifth wheel only.

Release the pump switch. The pump switch should automatically retract to the “Pump OFF” position.

Check to verify that enough room has been made for fifth wheel clearance and proceed with caution to connect the tractor to the mobile unit.

Verify that the fifth wheel is locked into position.

6.11 Raise the Front Stabilizing Legs

After the tractor has successfully connected to the mobile unit, the stabilizing legs can be raised. Refer to [Section 12: Stabilizing Legs](#) for the following procedure.

Move and hold the pump switch in the “ON” position.

Push the levers away from you to retract the legs. This will retract the legs and lower the front end of the mobile unit.

Retract the legs to their transport positions.

Release the pump switch. The pump switch should automatically retract to the “Pump OFF” position.

Place the Air Suspension Control Switch in the “OFF” / normal ride position to inflate the air bags to normal ride.



6.12 Verify that the Mobile Unit is Ready for Transport

Before the mobile unit can be transported, a final check of all components is necessary. Please refer to the following when checking the mobile unit.

Have the chairs, monitors, doors, cabinets, cameras, and printers been secured? Make sure that all of these items have been secured with the supplied hardware prior to transporting the mobile unit.

Are all exterior doors closed and locked? If not, make sure that all exterior doors are closed and locked.

Are all running & marker lights working correctly? If not, replace any bulb that is not working before transporting the mobile unit.

Are any warning lights illuminated? If so, check to find the cause of the warning. Do not move the mobile unit if any warning lights are flashing. If further assistance is needed, refer to the Oshkosh Specialty Vehicles VOL I Service/Operation Manual binder for a list of local service representatives or call Oshkosh Specialty Vehicles for further assistance.

Is the fuel tank full? Check the fuel gauge, located in the underbody compartment, and fill the fuel tank if necessary.

Verify that the air suspension system is fully inflated and at the proper ride height. The lowest point of the trailer sidewall should be approximately 15" above ground level.

Section 7: Electrical System



Electrical, mechanical, pneumatic, and hydraulic safety devices have been installed on this vehicle to help protect against personal injury and / or damage to equipment. Under no circumstances should any attempt be made to disconnect or in any way render any of these devices inoperative. If a malfunction of any safety device is discovered to exist, **DO NOT** operate the vehicle, but immediately notify appropriate maintenance personnel.



Use and follow the appropriate Lockout/Tagout procedures as required by OSHA Standard 1910.147 when performing maintenance or servicing any electrical, hydraulic or pneumatic systems. See Appendix E for Lockout/Tagout procedures.



Before connecting or disconnecting from shore power, it is imperative that the shore power connections be moved to the "OFF" position. Failure to do this can result in injury or death to the operator of the mobile unit.



It is the operator's responsibility to verify that the shore power receptacle is of the same type and voltage as the connection that is supplied by Oshkosh Specialty Vehicles. Failure to do this can result in injury or death to the operator of the mobile unit as well as irreparable damage to the mobile unit.

The entire electrical system is installed in conformance with the National Electric Code.

The system is completely installed in the factory. Service access is gained through the underbody compartments of the mobile unit with thin wall conduit and/or wire-mold sized to accept the required service entrance conductors used throughout the mobile unit.

All electrical materials, devices, appliances, fittings, and other equipment are approved and listed by Underwriters' Laboratories, Inc. (UL).

All required tags, labels and rating nameplates are permanently installed in their proper locations before the mobile unit leaves the factory.

There are several panels used in the electrical system.

One 120/208V AC distribution panel that is located at the rear of the Scan Room in the mobile unit. This panel is responsible for all AC powered components aboard the mobile.

7.1 120/208 AC Electrical Panels

The 120/208V AC electrical panel is responsible for the power supplied to the equipment aboard the mobile unit. If a problem exists with the equipment, or the power supply to them, a circuit breaker will trip in order to prevent damage. On the inside of the panel access door, a listing of all the circuit breakers can be found.

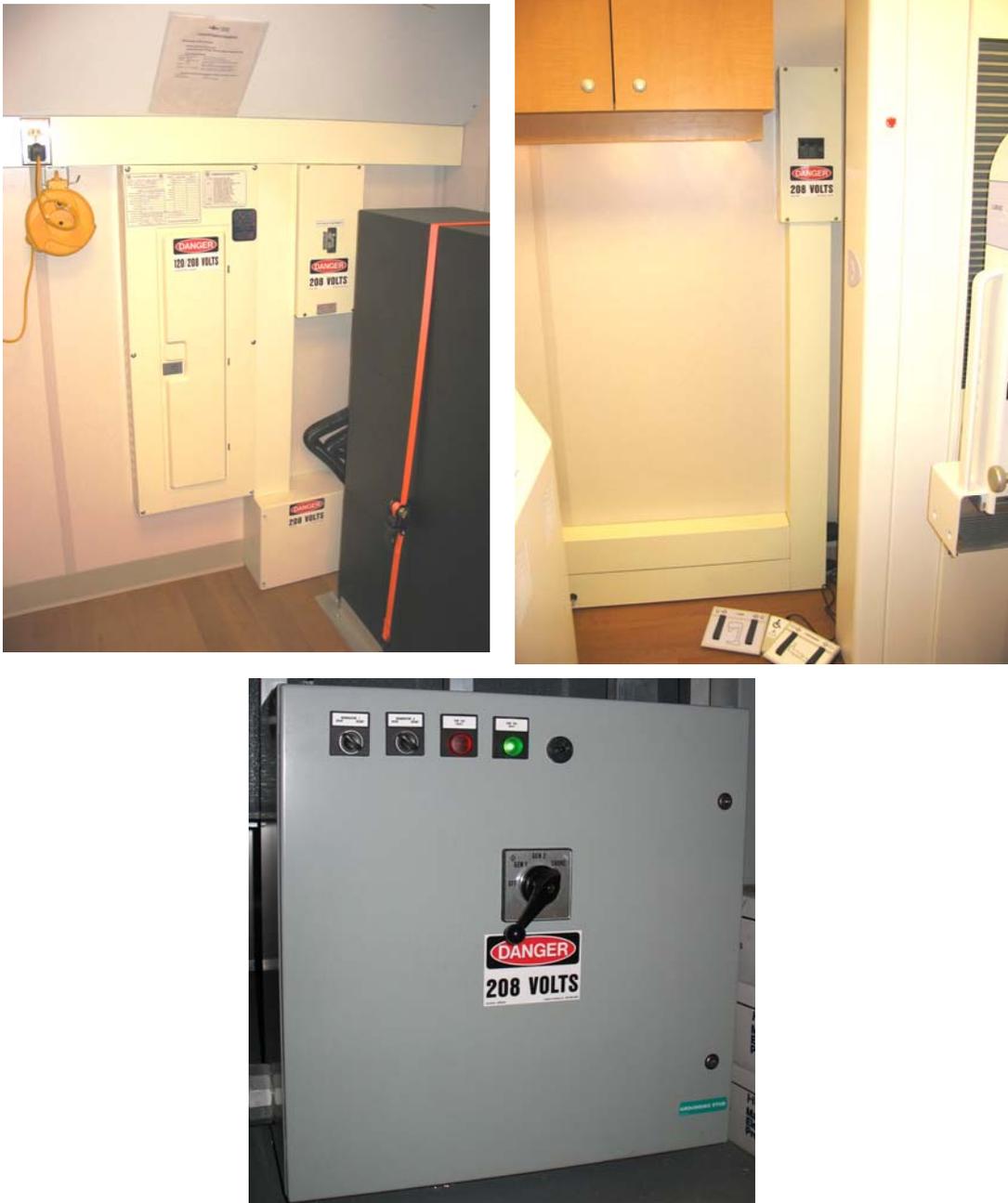


Figure 22: 120/208V AC Electrical Panel

7.2 208V AC Phase Power Monitor



Make sure that all electrical parts are serviced only by a certified electrician or qualified personnel. Dangerous voltages are present which could result in injury or death.

The 208V AC Phase Power Monitor checks the incoming shore power to ensure that it has the correct phase rotation (ABC) and that all three phases are present. If all three phases are present and in the correct rotation, the 208V AC Light, on the monitor, will illuminate.

If any phase is not present or if the phase rotation is not correct, the 208V AC Fault Light will illuminate, a piezo-electric horn will sound. Disconnect shore power immediately and investigate to determine the cause of the fault.



Figure 23: 208V AC Phase Power Monitor

7.3 Facility Power Connection



Figure 24: Shore Power Connection

Although the shore power connection is not an actual physical feature of the mobile unit, it is an integral part of the daily operations.

Oshkosh Specialty Vehicles Connector:

The plug that is provided by Oshkosh Specialty Vehicles for connection to the shore power receptacle.

Power Cable:

The cable that runs between the shore power connections and the 208V AC electrical panel.

Shore Power Disconnect:

The shore power disconnect terminates the power to the receptacle. This must be in the "OFF" position when connecting to the receptacle.

Shore Power Receptacle Outlet:

The receptacle outlet that the shore facility has installed for use with the Oshkosh Specialty Vehicles connector and power cable.

Shore Power Unit:

The complete shore power assembly.



Circuit Breaker	
Manufacturer:	Facility provided
Ampere Rating:	200 A disconnect

Receptacle	
Manufacturer:	Russellstoll
Model:	#DF 2516 FRAB0
Ampere Rating:	200 A

7.4 Power Cable

Descriptions:	Specifications
Service Amps:	200 A
5 Wire:	3 pole plus neutral and ground
Power Plug	Russellstoll # DS2516MP000/DF2034
Cord:	200 A, a #3/0 4 conductor type G, 600V – 2000V, 90° C, 50'-0" (13.71m) long



Section 8: Generator



Use and follow the appropriate Lockout/Tagout procedures as required by OSHA Standard 1910.147 when performing maintenance or servicing any electrical, hydraulic or pneumatic systems. See Appendix E for Lockout/Tagout procedures.



Make sure that all electrical parts are serviced only by a certified electrician or qualified personnel. Dangerous voltages are present which could result in injury or death.



Always make sure that eyes are protected while servicing the unit. Wear safety goggles when prying, drilling, grinding, or working with batteries. Wear safety goggles over regular prescription glasses unless the lenses are made of hardened glass and can serve as safety goggles.



Be certain to disconnect the power before working on any of the electrical systems.



Before connecting or disconnecting from shore power, it is imperative that the shore power connections be moved to the "OFF" position. Failure to do this can result in injury or death to the operator of the mobile unit.



It is the operator's responsibility to verify that the shore power receptacle is of the same type and voltage as the connection that is supplied by Oshkosh Specialty Vehicles. Failure to do this can result in injury or death to the operator of the mobile unit as well as irreparable damage to the mobile unit.



When switching from generator power to shore power the red "208V AC FAULT" may illuminate and flicker. If the red "208V AC FAULT" stays illuminated, reconnect to generator power and call a certified electrician before attempting to reconnect to shore power.



When turning the power selector switch from one position to another, the selector must be paused for a minimum of five seconds between selections. Failure to do so can result in damage to the equipment.



Always inspect the power cable, connectors, and fasteners prior to usage. If during inspection, it is suspected that either internal or external damage has occurred, have a certified electrician inspect and repair the damage before using.



The medical system requires the HVAC system to be supplied power at all times. During transit of the mobile unit via the generator and when the unit is in the parked position via shore power.



When servicing the unit be certain that a first aid kit and fire extinguisher are within reach at all times.

The mobile unit is equipped with two 40 KVA generators that are mounted on the front of the unit. The generator supplies power to the unit during transport. The generators can also be able to power the medical system so the medical procedures can take place when shore power is unavailable.

The generator oil, as well as the oil filter, air filter, and fuel filter must be changed every 250 hours or six months of service, whichever comes first. The number of hours the generator has been in operation can be obtained by checking the microprocessor located on top of the staging unit in the generator compartment.

Once a year, the fuel separator should be checked for contamination and accumulation of debris.

For additional information, refer to the Oshkosh Specialty Vehicles VOL II Vendor Information binder for the product manual.



Figure 25: Generator Compartment

- 120V AC Power Outlet:** An additional outlet has been provided for the operator of the mobile unit to be used if needed.
- Air Filter:** The air filter is responsible for removing all contaminants from the generators air supply.
- Battery:** The battery is used to start the generator.
- Fuel Filter:** The fuel filter is responsible for removing all contaminants from the fuel supply.
- Fuel Pump:** Supplies the generator with fuel from the fuel tank.
- Generator Motor:** The actual motor of the generator.
- Microcomputer:** The microcomputer provides the operator with information that is needed for service purposes.
- Oil Filter:** The oil filter is responsible for removing all contaminants form the oil supply.

8.1 Fuel System Priming Procedure:

Press and hold the Select button (item #2) located on the ADC. See [Figure 26: Advanced Digital Control](#).

Move the generator set master switch (item#3) to the RUN position to enter the programming mode.

Continue to hold the Select button for 10 seconds. The ADC will display “Fuel” and fuel priming pump will start.

Continue to hold the select button for as long as you want the fuel priming pump to function (typically 10 seconds).

Release the Select button and move the Master switch to the off position.

Place the Master switch to the RUN position to start the unit.



Figure 26: Advanced Digital Control

1. ADC (ADVANCE DATA CONTROL)
2. Select button.
3. Generator set master switch.

8.2 Generator Start/Stop Switch

The selector switch that controls the “Stop” and “Start” settings of the generator can be found on the power supply control panel. .

When the generator is to be started, the selector switch must be in the “Start” position. The generator cannot take the full load of the mobile unit until it has been allowed to run briefly. Do not move the Unit Power Selector switch to the “GEN” position until the generator has run for approximately five (5) minutes. When the generator is to be stopped, the selector switch must be in the “Stop” position. Once the selector has been moved to the “Stop” position, the generator will enter into a five (5) minute cooling phase. Do not attempt to stop the generator by repeatedly moving the selector to the “Stop” position.



Section 9: Humidity System



All settings for the humidity system are preset at the factory. Under no circumstances should factory presets be altered.



Proper humidity levels must be maintained to protect sensitive electronic equipment.

The humidifier is responsible for maintaining the humidity levels within the mobile unit. The settings for the humidifier are set to meet the medical system manufacturers' specifications. Under no circumstances should the settings of the humidifier be altered. In order for the humidifier to function properly, the water tank level must be maintained at all times.

Exterior Connection:	The facility must provide a fresh water supply for use with the mobile unit. The incoming supply is then attached to the connection.
Humidifier:	The humidifier provides the required humidity to the mobile unit per the medical manufacturer's requirements.
Humidity Controller:	The humidistat is responsible for the internal humidity of the mobile unit. The setting is preset at the factory to comply with the medical system manufacturers requirements.
Humidity Sensor:	Maintains an accurate reading of the humidity levels inside of the mobile unit.

9.1 System Operation

The humidifier system is capable of producing up to 12 pounds of steam per hour, at 15 amps. A sensor continually monitors the interior of the mobile unit for relative humidity. This sensor is located in the HVAC return duct and is programmed to keep the relative humidity at 40%. If the humidity drops below the set point, the humidifier is signaled to emit more steam. The humidifier creates steam when electrodes in the steam cylinder of the humidifier vaporize the supplied water. The steam then travels through a hose to a distribution pipe located in the return air duct of the HVAC system. Since the steam is injected into the return duct of the HVAC system, both A/C units are supplied with humidified air for distribution throughout the interior of the mobile unit. An air pressure switch is located in the HVAC discharge duct that is interlocked to the humidifier. If for any reason the airflow is disrupted, the humidifier will shut down. When the sensor detects that relative humidity has been reached, a signal is sent to the humidifier to stop it from creating more steam.

9.2 Water Supply

Water is supplied to the humidifier by means of a facility provided water supply. Plumbing connections are as follows:

- One 1" female garden hose coupling for the water supply from the exterior of the mobile unit. (incoming)
- One 0.5" outer diameter copper drain line from the steam cylinder for automatic drain cycles. The drain penetrates the floor of the mobile unit in order to empty to the exterior. (drainage)

9.3 Humidity Controller

CAUTION All settings for the humidity system are preset at the factory. Under no circumstances should factory presets be altered.

The humidity controller is located in Breast Health Suite room above the electrical panel wall next to the air filter access door. The relative humidity setting for the mobile unit is 35%. The humidifier must not be altered from its factory setting.



Figure 27: Humidity Controller

9.4 Humidity Settings

CAUTION All settings for the humidity system are preset at the factory. Under no circumstances should factory presets be altered.

The humidity low set point is 30% RH (relative humidity).

The humidity high set point is 40% RH (relative humidity).

9.5 Electrical Connections

Electrical connections at the humidifier are located on a terminal rail behind the cover of the humidifier.

The distribution panel supplies the required 208V AC power via a 15 amp, three phase breaker.

A humidistat is connected to the humidifier via a controlling transformer cable.

9.6 Instructions

The HVAC system along with the humidifier is set to the required settings per the medical equipment manufacturers' specifications before leaving the factory. Under no circumstances should the settings be altered from their factory specifications.

Please refer to the product manual located in the literature provided by Oshkosh Specialty Vehicles.

1. Attaching the water supply lines:
2. On the passenger side exterior of the mobile unit a water connection can be found. This connection is located on an underbody compartment door.
3. Remove the cap that covers the connection.
4. Attach the supplied water hose to this connection.
5. Attach the other end of the hose to facility provided faucet.
6. Turn on the water at the faucet.



Section 10: HVAC System



The HVAC system is critical to the operation and life of the medical system. The medical system operates within strict specifications regarding temperature and humidity. All aspects of the HVAC system such as damper settings, venting, component set points, and sensor placement are adjusted for optimum operation. Under no circumstances should these settings be altered.



The Hologic Selenia medical system requires the HVAC system to be supplied power at all times. During transit of the mobile unit via the generator and when the unit is in the parked position via shore power.



Figure 28: A/C Units

Two air conditioning units are used to maintain the internal environment of the mobile unit. Both air conditioners come from the factory preset to the specifications required by the medical system manufacturer. Under no circumstances should the factory presets be changed or altered from their factory setting. Irreparable damage can occur to the medical system if this is done.

The HVAC system is designed specifically to maintain only the internal environment of the mobile unit. The HVAC system is not designed to handle areas outside of the mobile unit, such as adjoining corridors or hallways. It is important to keep all exterior doors closed at all times. All interior doors, computer doors, partitions, and damper settings, must be in the intended positions before running the medical equipment. Do not attempt to store any boxes or items in the mobile unit, as this will interrupt the intended airflow requirements.

In order to ensure proper operation of the HVAC system at all times, refer to [Section 14: General Maintenance](#) and [Section 15: Specific Maintenance](#).

10.1 System Specifications and Descriptions

The HVAC system is completely designed and installed in full conformance with all applicable codes.

The HVAC system utilizes forced air.

The HVAC utilizes electricity as the source of power.

Heat producing appliances must be approved by Underwriters Laboratories, Inc. (U.L.) and installed in accordance with the terms on their listings.

The air ducts are constructed of approved materials and installed in conformance with all applicable codes.

Air conditioning and heating registers are installed in accordance with the approved plans.

Return air is provided as required and is in full conformance with all applicable codes.

All warning and identification labels as required are installed at the factory.

All aspects of the HVAC system such as damper settings, venting, component set points, and sensor placement are adjusted for optimum operation. Under no circumstances should these settings be altered.

Two separate and individually controlled units provide air conditioning and heating for the mobile unit.

The air conditioning ductwork is lined with a sound absorbent material for reduced noise and operator and patient comfort.

10.2 Exterior HVAC Specifications

The HVAC system is designed to work within certain limitations. The ambient exterior temperatures must be within the range of -20°F to 110°F.

10.3 Interior HVAC Specifications

Each air conditioner has a cooling capacity of 36,000 BTUH. The temperature in each room is maintained at approximately 70°F with an acceptable range of 68° to 72°F. Both air conditioned and heated air is distributed through an insulated duct which starts at the discharge side of the air conditioner.

A heating controller is provided to help regulate the heat. The controller incorporates a preset sensor, which activates heat strips in the air conditioning unit. There only two setting AC and HC. The controller is located in the in the return air duct vent.

Air is returned to the air conditioner via ceiling vents located throughout the mobile unit. Each duct is strategically placed over the equipment for adequate ventilation. These return air ducts are located in each room and draw air from all rooms.

One 16" x 30" x 1" fiber core air filters are provided at the air return duct of each air conditioning and heating unit. This filter provides dust free air throughout the interior of the mobile unit. The air filter is accessible through an access door on the front of the plenum outside of the unit.

10.4 Thermostat Temperature Setting



The HVAC system is critical to the operation and life of the medical system. The medical system operates within strict specifications regarding temperature and humidity. All aspects of the HVAC system such as damper settings, venting, component set points, and sensor placement are adjusted for optimum operation. Under no circumstances should these settings be altered.

The temperature setting is the controlled by the use of Thermostat Controls. The Thermostat Control must not be set outside of the parameters as defined by the medical system manufacturer.

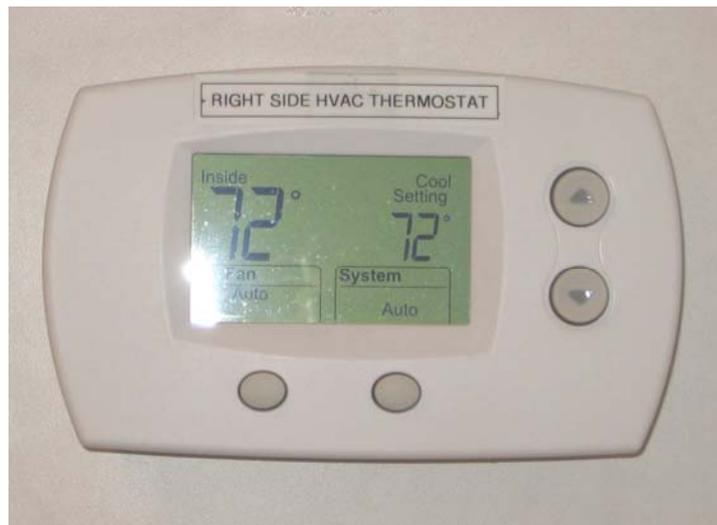


Figure 29: Thermostat Temperature Control



Section 11: Wheel Chair Lift

The mobile unit contains a Ricon Wheel Chair Lift (S5505), which is used to move personnel from the ground level to the floor level of the mobile unit. The wheel chair lift has a maximum design capacity of 800 lbs and a working capacity of 434 lbs. The lifting height is 46”.

Refer to the Service / Operations Manual, Volume II Vendor Information for Installation, Service and Maintenance requirements.



Wheel Chair Lift In the stowed position



Lift Pendant



Switch to turn the power on for the Wheel Chair Lift



Wheel Chair Lift Deployed



Wheel Chair Lift at floor level

Functions of the Lift:

- Deploy
- Down
- Up
- Stow

Wheel Chair Cycle

Figure 30: Wheel Chair Lift



Section 12: Stabilizing Legs



Under no circumstances should the stabilizing legs and the rear air suspension be used to lift the mobile unit from the ground. If any attempt is made to raise the unit from the ground using the only the stabilizing legs and the rear air suspension, serious damage can occur to the suspension system of the mobile unit.

Both the stabilizing legs and the auxiliary support legs can be found at the front and rear of the unit. The stabilizing legs installed on this mobile unit are only for the purpose of parking and stabilizing the mobile unit. Please refer to the product manual located in Volume II of the literature provided by Oshkosh Specialty Vehicles for additional information.



Figure 31: Front Stabilizing Leg Assembly

Stabilizing Leg Controls:	The control box houses the stabilizing leg controls.
Stabilizing Leg:	Allows the mobile unit to be parked without the tractor being attached to the unit.
Digital Levels:	Allows the mobile unit to be leveled both front to back and side to side.
Lever 1:	Right side leg.
Lever 2:	Left side leg.
Air Suspension Control Switch	The "DN" position deflates the air bags. The "UP" position inflates the air bags for normal ride.
Pump ON / OFF Switch	The switch must be held in the ON position when extending or retracting the legs.
Auxiliary Support Legs:	The auxiliary support legs provide a fixed leg for use as a backup in case the stabilizing legs fail.
Sand Shoe:	Helps prevent the stabilizing legs from sinking due to weight.

12.1 Stabilizing Stands

The stabilizing stands are inserted beneath the rear supports of the mobile unit, and allow the mobile unit to be stabilized for all medical procedures. The stands are stored in the rearmost underbody compartments. When in use, the stands must be centered underneath the rear supports of the mobile unit. If shims are needed, use only the aluminum shims provided.



Figure 32: Rear Stabilizing Stands

12.2 Rear Air Suspension System Controls



If the rear air suspension is not functioning properly the mobile unit must not be moved. If the mobile unit is moved, irreparable damage can occur to the medical system and the mobile unit itself.

The rear air suspension controls are located on the landing / stabilizing leg control box. Locate on this control panel a switch that reads “DN” to the far left and “UP” to the far right.

When this selector is in the “DN” position, the rear air suspension will deflate and the mobile unit will lower. Prior to placing the selector in this position, the rear stabilizing stands must be inserted.

When the selector is in the “UP” position, the rear suspension will inflate and the mobile unit will automatically rise to transport height. Failure to turn the selector to the “UP” position prior to transporting the mobile unit can cause irreparable damage to both the mobile unit and medical system.



Figure 33: Air Bag Controls



Section 13: Lighting System

The lighting provided for the mobile unit can be divided into either interior lighting, or exterior lighting. Listed below are explanations concerning the lighting provided.

13.1 Emergency Lighting

In the event that the main AC power fails, three emergency lights are provided. These lights will automatically illuminate when the main AC power is lost. The lights are located in the Reception Area, Main Hallway, and Breast Health Suite. The emergency lighting system is wired into a 120V AC electrical system that allows the lights internal circuitry to keep its batteries at 100% charge. The emergency lights will illuminate the Exit doors and last for approximately 90 minutes.



Breast Health Suite Emergency Light



Reception Area Emergency Light

Figure 34: Emergency Lighting

13.2 Exterior Lighting

IMPORTANT All warning lights are located on the left side of the mobile unit.

The exterior lighting system can be divided as follows. For additional information of the warning lights Please refer to [Appendix B: Troubleshooting](#).

Underbody Compartment Lighting

Located inside of the underbody compartments there are wall mounted halogen lights connected to timers. The timers allow the lights to be set for up to 30 minutes before automatically turning off. There is one light provided on each side of the underbody.



Figure 35: Compartment Light

In addition, since the fuel compartment is sealed off from the others, a push button dome light has been included in this compartment.

Entrance Door Lighting / Exterior Service Lighting

A fixture that is located next to the door provides the entrance door lighting.

The switch for this light is located inside of the mobile unit next to the entry door.



Figure 36: Entry Door Lighting

Marker & Running Lights

When the mobile unit is in transit, federal law requires specific illumination characteristics. The mobile unit meets and exceeds these standards as outlined in Motor Vehicle Safety Standards Guide, Federal Safety Standard No. 108-4.


All lights are 12V DC, and are powered by the tractor. All wiring is run through the underbody wire harnesses. The top marker lights are wired through a 0.5" loom pipe that is run through the sidewalls of the mobile unit. The wires terminate at the glad-hands which are located in the front of the mobile unit for tractor hookup. Two electrical connections are supplied on the glad-hands, one six terminal connection and one seven terminal connection.

13.3 Interior Lighting

The interior lighting system can be divided as follows.

Reception Area

The light controls for the lighting in Reception Area are located just inside the entry door to that leads into the room. Recessed light fixtures are located in the ceiling panels and have been strategically placed for effective illumination of the equipment both during operation and while being service.

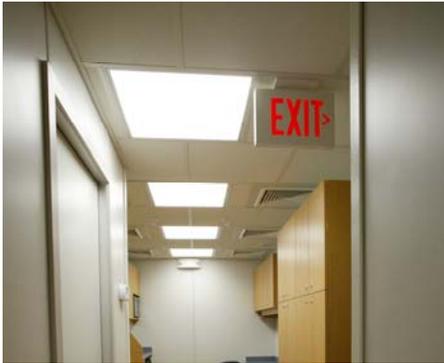

Fluorescent lights Hall Way



Fluorescent lights Breast Health Suite



Halogen lights Reception Room



Halogen lights Breast Health Suite

Figure 37: Interior Overall Lighting

Breast Health Suite

The light controls for the lighting in Breast Health Suite are located just inside the access door that leads into the room and to the left of the wall mounted telephone. Recessed light fixtures are located in the ceiling panels and have been strategically placed for effective illumination of the equipment both during operation and while being service.



Figure 38: Breast Health Suite Overall Lighting Controls

13.4 Warning Lights



Figure 39: Warning Lights

Warning lights have been installed on the exterior left side of the mobile unit in order to provide the operator and technician of the status of the mobile unit at all times during transit or while in the parked position. A description of each of the warning lights and their location can be found below. If any of the warning lights are illuminated, please refer to [Appendix B: Troubleshooting](#) for additional information.

AC Power Indicator Light



The Hologic Selenia medical system requires the HVAC system to be supplied power at all times. During transit of the mobile unit via the generator and when the unit is in the parked position via shore power.

The AC Power Indicator Light is located on the exterior left side of the mobile unit and will illuminate when the mobile unit is receiving power.

The mobile unit must have power at all times.

Transport Warning Light



If the Transport Warning Light is on, the mobile unit must not be moved. If the mobile unit is moved while this light is on, irreparable damage can occur to the mobile unit.

The Transport Warning Light is located on the exterior left side of the mobile unit and will illuminate when the interior doors are not closed and locked in position for transport.

Suspension Transport Warning Light



The air ride control valves must be in the normal ride position before the mobile unit can be transported. If the air ride control valves are not in the normal ride position, irreparable damage may occur to the mobile unit.

The Suspension Transport Warning Light is located on the exterior left side of the unit and will illuminate when the air bags for the rear suspension are either not inflated, or the air pressure is too low. The air bags must be properly inflated prior to transporting the mobile unit. Failure to properly inflate the air bags can result in irreparable damage to the mobile unit.

Power Loss Transport Warning Strobe Light



If the Transport Warning Strobe Light is flashing the mobile unit must not be moved. If the mobile unit is moved while this light is flashing, irreparable damage to the mobile unit, serious personal injury or death can occur.

The Transport Warning Strobe Light is located on the exterior left side of the mobile unit and will illuminate when there is no generator or shore power. It is the Operator's responsibility to ensure that the Transport Warning Light is functioning properly and that the bulb element is in working order. Please refer to the Oshkosh Specialty Vehicles VOL II Vendor Information binder for the product manual, the Oshkosh Specialty Vehicles VOL I Service/Operation Manual binder for a list of local service representatives, or contact Oshkosh Specialty Vehicles for service.



Section 14: General Maintenance



Use and follow the appropriate Lockout/Tagout procedures as required by OSHA Standard 1910.147 when performing maintenance or servicing any electrical, hydraulic or pneumatic systems. See Appendix E for Lockout/Tagout procedures.



Make sure that all electrical parts are serviced only by a certified electrician or qualified personnel. Dangerous voltages are present which could result in injury or death.



Always make sure that eyes are protected while servicing the unit. Wear safety goggles when prying, drilling, grinding, or working with batteries. Wear safety goggles over regular prescription glasses unless the lenses are made of hardened glass and can serve as safety goggles.



Be certain to disconnect the power before working on any of the electrical systems.



When servicing the unit be certain that a first aid kit and fire extinguisher are within reach at all times.

14.1 Daily Maintenance

1. Fuel tank should be checked for proper fuel levels.
2. During cold weather, verify that all underbody heaters are operational.
3. Keep the air intake grills on the computer cabinets for the medical system free and clear of obstructions.
4. Keep the A/C grills clean and free of debris.
5. Check and verify that no warning lights are illuminated.

14.2 Weekly Maintenance

1. Check the A/C filters. Replace if necessary.
2. Check the oil and water levels in the generator and refill if necessary.
3. Check the electrolyte levels in the DC batteries and fill if necessary using only distilled water.
4. Check all running lights, marker lights, brake lights, and turn signals.
5. A qualified technician should check tire pressure and verify that all wheels are at the correct pressure as specified by the tire manufacturer.
6. Check the fluid level in the hydraulic reservoir using the site glass. Add fluid if necessary. Use only (AWF) All Weather Fluid.
7. Check wheel lug nuts with torque wrench and verify that all inner and outer wheels, both the front and rear, are tightened to 450-500 foot-pounds. This must be done after every 500 miles of driving. In accordance with torque procedure, lugs and nuts must be installed dry. Do not use any type of lubricant.

14.3 Monthly Maintenance

1. Put a few drops of 20W oil, or similar graphite oil, on the swivel pin of all door hinges and key openings of all door locks.
2. Check the operation of the smoke detectors and vacuum internally.
3. Check the fire extinguisher gauges for safe charges.
4. Inspect the power cables for any damage.
5. Check the cable tie downs.
6. Check for cut, damaged, or loose wire connections.
7. Check and verify that all connector bolts are tight and secure.
8. A qualified A/C technician must check the A/C condensers every month. Refer to the Air Conditioning Owner's Manual for more information.
9. Lubricate the front stabilizing legs.
10. Check wheel lug nuts with torque wrench and verify that all inner and outer wheels, both the front and rear, are tightened to 450-500 foot pounds. This must be done after every 500 miles of driving. In accordance with torque procedure, lugs and nuts must be installed dry. Do not use any type of lubricant.
11. The generator oil, as well as the oil filter, air filter, and fuel filters must be changed every 250 hours or six months of service, whichever comes first. The number of hours the generator has been in operation can be obtained by checking the microcomputer controller located on top of the staging unit in the generator bay. Refer to [Figure 26: Advanced Digital Control](#).

14.4 Quarterly Maintenance

1. Once a year, check the fuel separator for contamination or debris.
2. Once a year, perform the preventative maintenance on the stabilizing legs and the stabilizing leg controls. Refer to the accompanying manual for the stabilizing gear system.
3. Rotate the tires.
4. Check wheel lug nuts with torque wrench and verify that all inner and outer wheels, both the front and rear, are tightened to 450-500 foot pounds. This must be done after every 500 miles of driving. In accordance with torque procedure, lugs and nuts must be installed dry. Do not use any type of lubricant.



Section 15: Specific Maintenance



Use and follow the appropriate Lockout/Tagout procedures as required by OSHA Standard 1910.147 when performing maintenance or servicing any electrical, hydraulic or pneumatic systems. See Appendix E for Lockout/Tagout procedures.



Make sure that all electrical parts are serviced only by a certified electrician or qualified personnel. Dangerous voltages are present which could result in injury or death.



Always make sure that eyes are protected while servicing the unit. Wear safety goggles when prying, drilling, grinding, or working with batteries. Wear safety goggles over regular prescription glasses unless the lenses are made of hardened glass and can serve as safety goggles.



Be certain to disconnect the power before working on any of the electrical systems.



The HVAC system is critical to the operation and life of the equipment. The medical equipment operates within strict limits regarding temperature and humidity. All aspects of the HVAC system such as baffling, venting, component set points, and sensor placement are adjusted for optimum operation. Under no circumstances should any aspect of the HVAC system be altered from factory specifications.



Image quality can be impaired with improper door closer adjustment.



A power washer should never be used to clean the A/C units. Serious damage to the A/C coils may occur.



When servicing the unit be certain that a first aid kit and fire extinguisher are within reach at all times.

15.1 Door Closer Adjustments

The door closer must be adjusted so that the door does not slam shut. Refer to the door closer component sheet in the component literature manual for proper adjustment. Adjust door closer as required to insure proper non-slamming door action.

15.2 Electrical System

1. Inspect the power cables for any damage.
2. Check the cable tie downs.
3. Check for cut, damaged, or loose wire connections.
4. Check and verify that all connector bolts are tight and secure.

15.3 Generator System

1. The generator oil, as well as the oil filter, air filter, and fuel filter must be changed every 250 hours or six months of service, whichever comes first. The number of hours the generator has been in operation can be obtained by checking the microcomputer controller located on top of the staging unit in the generator bay. Please refer to refer to [Figure 26: Advanced Digital Control](#).
2. Once a year, check the fuel separator for contamination or debris.

15.4 Humidity System



During seasons of low humidity, the humidifier will need to be filled more often.

The 70 gal. fresh water tank supplies the humidifier with water. The water level must be maintained at all times.

1. Check the water tank to determine the water level.
2. Attach one end of a hose to the exterior water tank fill valve and the other end to the shore supply.
3. Turn on the water source to begin filling the tank.
4. After the water tank is full, turn off the water source.
5. Detach the hose at both ends and place in the underbody storage compartments.

15.5 HVAC System



The HVAC system is critical to the operation and life of the equipment. The medical equipment operates within strict limits regarding temperature and humidity. All aspects of the HVAC system such as baffling, venting, component set points, and sensor placement are adjusted for optimum operation. Under no circumstances should any aspect of the HVAC system be altered from factory specifications.

The HVAC system is designed specifically to maintain only the internal environment of the mobile unit. The HVAC system is not designed to handle areas outside of the mobile unit such as adjoining corridors or hallways.

1. It is important to be sure that the doors, partitions, and baffling are in the intended positions before running the medical system.
2. Do not attempt to store boxes, or any other items near computer system air inlets or in the aisles. Such actions will disrupt the intended airflow requirements.
3. A qualified A/C technician must check the A/C condensers every month. Refer to the Air Conditioning Owner's Manual for more information.

15.6 Stabilizing Legs

1. Once a year, perform the preventative maintenance on the stabilizing legs and the stabilizing leg controls. Refer to the accompanying manual for the stabilizing gear system.
2. Change the oil in the stabilizing leg control box and refill with six (6) quarts of PG-AWF or other approved fluid to port level.
3. Add one pint of permanent anti-freeze as necessary.
4. Extend the stabilizing legs and coat lightly with clean grease.
5. Grease the alemite fittings and check the valve on each leg. Use "NGLI" lithium grease with a grade of "00" or "0".
6. Check the fittings and the hydraulic lines for leaks or worn spots. Replace all defective fittings and lines as necessary.
7. Check for loose bolts and nuts. Tighten as necessary.



Appendix A: Mobile Unit Checklist



It is the operator's responsibility to verify that the shore power receptacle is of the same type and voltage as the connection that is supplied by Oshkosh Specialty Vehicles. Failure to do this can result in injury or death to the operator of the mobile unit as well as irreparable damage to the mobile unit.



Before connecting or disconnecting from shore power, it is imperative that the shore power connections be moved to the "OFF" position. Failure to do this can result in injury or death to the operator of the mobile unit.



Make sure that all electrical parts are serviced only by a certified electrician or qualified personnel. Dangerous voltages are present which could result in injury or death.



Always make sure that eyes are protected while servicing the unit. Wear safety goggles when prying, drilling, grinding, or working with batteries. Wear safety goggles over regular prescription glasses unless the lenses are made of hardened glass and can serve as safety goggles.



Be certain to disconnect the power before working on any of the electrical systems.



The Hologic Selenia medical system requires the HVAC system to be supplied power at all times. During transit of the mobile unit via the generator and when the unit is in the parked position via shore power.



Always inspect the power cable, connectors, and fasteners prior to usage. If during inspection, it is suspected that either internal or external damage has occurred, have a certified electrician inspect and repair the damage before using.



The stabilizing legs and rear suspension are not to be used to raise the mobile unit off the ground. The legs are meant only to level the unit and place it in a parked position. If the legs are used in an attempt to raise the mobile unit from the ground, serious damage may occur to the mobile unit.



Failure to completely exhaust the suspension before uncoupling the airlines may result in damage to the suspension of the mobile unit.



The air ride control valves must be in the "OFF" / normal ride position before the mobile unit can be transported. If the air ride control switch is not in the normal ride position, irreparable damage may occur to the mobile unit.



Before transporting the mobile unit, check to verify all warning lights as well as all exterior marker lights are working correctly.



When servicing the unit be certain that a first aid kit and fire extinguisher are within reach at all times.



Mobile Unit Setup Checklist

1. Park the mobile unit on the pad per the site-planning guide. Set the trailer parking brake.
2. Lower the front stabilizing legs.
3. Disconnect the tractor while leaving the air and electrical lines engaged.
4. Install the rear Stabilizing Stands.
5. Exhaust the rear suspension.
6. Re-level the mobile unit as needed.
7. Disconnect the tractor air and electrical lines.
8. Lower the auxiliary support legs.
9. Verify that the shore power disconnect is in the "OFF" position and connect to the power cable to the shore power receptacle.
10. Move the shore power disconnect to the "ON" position. The ATS will automatically switch from generator power to shore power.
11. Connect the phone and data lines.
12. Connect the water supply connection.
13. Install the stair assemblies.
14. Remove restraining hardware and position equipment for use.
15. Prepare all medical equipment for use per the OEM provided instructions.
16. Deploy the canopies for use.

Mobile Unit Transport Checklist

1. Retract the canopies above the entry doors.
2. Secure all equipment, this includes all medical equipment per OEM requirements
3. Secure all moveable objects such as chairs, monitors, doors, cabinets, cameras, and printers.
4. Remove and store the Stair Assemblies.
5. Move the shore power disconnect to the "OFF" position and disconnect to the power cable from the shore power receptacle. The generator will automatically start and supply power to the unit.
6. Disconnect the phone and data lines.
7. Disconnect the water supply connection.
8. Raise the auxiliary support legs.
9. Connect the tractor air and electrical connections.
10. Inflate the rear air suspension and remove the rear stabilizing stands.
11. Connect the tractor to the mobile unit.
12. Raise the front stabilizing legs.
13. Verify that the mobile unit is ready for transport.
14. Are all interior sliding doors open and locked and all interior swinging doors closed and locked for transport?
15. Are all exterior doors closed and locked?
16. Are all running & marker lights working correctly?
17. Are any warning lights illuminated?
18. Is the fuel tank full?
19. Verify that the air suspension system is fully inflated and at the proper ride height. . The lowest point of the trailer sidewall should be approximately 15" above ground level.



Appendix B: Troubleshooting

If any of the following troubleshooting guides do not correct the problem, or if the problem worsens, please refer to the Oshkosh Specialty Vehicles VOL II Vendor Information binder for the product manual, the Oshkosh Specialty Vehicles VOL I Service/Operation Manual binder for a list of local service representatives, or contact Oshkosh Specialty Vehicles for service.

AC Power Indicator Light is off...

If the AC Power Indicator Light is “OFF” then the mobile unit is not receiving AC power. The mobile unit must have power at all times. If the mobile unit is on site, shore power must be connected.

If the mobile unit is being transported, then the vehicle must be stopped and the following items checked. For any additional information, please contact Oshkosh Specialty Vehicles.

Generator power is connected when the A/C power Off strobe light illuminates:

1.	Verify that the fuel tank has fuel.
2.	Verify that the Generator selector has been moved to the “Start” position.

Transport Warning Light is on...



If the Transport Warning Light is on, the mobile unit must not be moved. If the mobile unit is moved while this light is on, irreparable damage can occur to the mobile unit.

If the Transport Warning Light is illuminated, the mobile unit is not ready for transport. Before the mobile unit can be transported, this light must be off. Please refer to the following table:

Problem:		Solution:
Breast Health Suite Door is not in the proper transport position	1.	Make sure the door is open and locked in position for transport.
Other Interior Doors are not in the proper transport position		Make sure the swinging doors are closed and locked in position for transport and that all sliding doors are open and locked in position for transport.

Suspension Transport Indicator Light is illuminated...



If the Suspension Transport Indicator Light is on, the mobile unit must not be moved. If the mobile unit is moved without the rear air suspension functioning properly, irreparable damage can occur to the mobile unit.

If the Suspension Transport Indicator Light is on, the rear air bags / suspension of the mobile unit are not ready for transport. This light must be off to move the mobile unit. Check the rear air bag suspension control valve buttons. The buttons must be in the “In” position. This is the normal ride position.

If the buttons are in the correct position and the light still appears a problem exists within the rear suspension. Please contact Oshkosh Specialty Vehicles.

Temperature is out of specifications...

If the temperature is out of specifications, either too high or too low, refer to the following table.

Problem:		Check for:	Solution:
The temperature inside of the mobile unit is too warm.	1.	Check for exterior doors left open during warm weather conditions.	The HVAC system can only support the environment of the mobile unit. Unless opened for use, all exterior doors should remain closed all of the time.
	2.	Check for blocked or dirty air vents and/or air conditioner filters.	Clean the air vents and/or change the air conditioner filters. After this has been done, verify that cold air is blowing.
	3.	Check to see if the A/C disconnect is in the "OFF" position.	Turn the A/C disconnect to the "ON" position.
	4.	The Thermostat Control settings are correct.	Verify that the Thermostat Control is set at 68°F. Please contact Oshkosh Specialty Vehicles for further assistance.
The temperature inside of the mobile unit is too cold.	1.	Check for open exterior doors left open during cold weather conditions.	The HVAC system can only support the environment of the mobile unit. Unless opened for use, all exterior doors should remain closed all of the time.
	2.	Check for blocked or dirty air vents and/or air conditioner filters.	Clean the air vents and/or change the air conditioner filters. After this has been done, verify that warm air is blowing.
	3.	Check to see if the A/C disconnect is in the "OFF" position.	Turn the A/C disconnect to the "ON" position.
	4.	The Thermostat Control settings are correct.	Verify that the Thermostat Control is set at 72°F. Please contact Oshkosh Specialty Vehicles for further assistance.



Humidity is out of specifications...

The humidity settings for the mobile unit are 30% RH to 60% RH (relative humidity). If the mobile unit is experiencing humidity levels outside of this range, either too low or too high, please refer to the following table.

Problem		Check for:	Solution:
The humidity inside of the mobile unit is too high.	1.	Check for exterior doors that have been left open during humid conditions.	The HVAC system can only support the environment of the mobile unit. Unless opened for use, all exterior doors should remain closed all of the time.
	2.	Check for blocked or dirty air vents and/or air conditioner filters.	Clean the air vents and/or change the air conditioner filters. After this has been done, check for any changes to the humidity levels.
	3.	Check to see if the humidifier is constantly running.	Verify that the humidifier is set between 30% and 40% RH (relative humidity). If the humidifier is still running constantly, contact Oshkosh Specialty Vehicles for service.
The humidity inside of the mobile unit is too low.	1.	Check for open exterior doors left open during arid weather conditions.	The HVAC system can only support the environment of the mobile unit. Unless opened for use, all exterior doors should remain closed all of the time.
	2.	Check for blocked or dirty air vents and/or air conditioner filters.	Clean the air vents and/or change the air conditioner filters. After this has been done, check for any changes to the humidity levels.
	3.	Check to see if the A/C disconnect is in the "OFF" position.	Turn the A/C disconnect to the "ON" position.
	4.	Check to see if the humidifier disconnect is in the "ON" position.	Move the humidity disconnect to the "ON" position and verify that the humidifier is set between 30% and 40% RH (relative humidity). If the humidifier is running and the humidity level does not change, a problem exists within the humidity system.



Appendix C: HVAC Set Points



The HVAC system is critical to the operation and life of the equipment. The medical equipment operates within strict limits regarding temperature and humidity. All aspects of the HVAC system such as baffling, venting, component set points, and sensor placement have been adjusted for optimum operation. Under no circumstances should any aspect of the HVAC system be altered from factory specifications.



Be certain that the HVAC system is operational at all times.

There are two set points for the HVAC system. These points are set at the factory and should not be changed under any circumstances. Altering these points can result in damage to the medical equipment.

Temperature Controller Settings

The Controller has only two settings: AC or HC.

Humidity Settings

The humidistat set point is 35% relative humidity.



Appendix D: Circuit Malfunction Checklist

Category 1

Visual Checks – Check for the most common occurrences.

Is the circuit breaker, in the 208V AC electrical panel, in the “ON” position?

Are the Mammography System UPS 60 amp circuit breakers in the 120/208V AC electrical panel in the “ON” position?

Category 2

Component Checks – (some tools may be required).

Check the emergency off button in the Breast Health Suite, N.O.?

For additional troubleshooting, please contact Oshkosh Specialty Vehicles for assistance.



Appendix E: Lockout/Tagout Procedures

Specific Energy Control Procedures

Machine or Equipment for this Procedure:

Specialty Vehicle Trailer: Hologic Selenia Mammography Unit

Control of Hazardous Energy:

Type of Hazardous Energy	When is it Necessary to Lock Out
Electrical 208V AC	When servicing main electrical power line
Electrical 120/208V AC room circuits	When servicing or performing installation inside specific sections of the trailer
Electrical 12V DC	When servicing the following: Generator, Wheelchair Lift, Hydraulic System, Digital Levels, Lights
Electrical 12V DC From Battery	When servicing the following: Generator, Wheelchair Lift, Hydraulic System, Digital Levels, Lights

Affected Personnel to notify when the Specialty Vehicles Trailer is to be Locked Out:

Name/Department:	Location:
Production employees	In the vicinity of the trailer



Shut down specifications for the Specialty Vehicle Trailers:

Energy Type and Rating:	Type of Energy Isolating Device:	Location of Energy Isolating Device:	Lockout Device Used:
Main power feed Electrical 208V AC	Circuit Breaker or Plug	Normally located above the Facility Power Shore	Lock and tag with or without lockout hasp
Light or outlet circuits Electrical 120/208V AC	Wall switch or circuit breaker	Distribution panel for circuit breaker, wall switch for room circuits	Lock and tag with a Universal Wall Switch Lockout, Universal Circuit Breaker Lockout
Generator Power engaged when main power is lost	Generator Breaker Switch	Inside service panels, on front of Generator control cover.	Lock and tag with a Circuit Breaker Lockout attachment device
Power to lift panels Electrical 12V DC	Breaker Switch	Inside service panels	Lock and tag with a Circuit Breaker Lockout attachment device
Electrical 12V DC From Battery	Remove Battery Cables	On battery	Lock and tag with a Plug Lockout attachment device
Medical System Hologic Selenia Mammography	Circuit Breaker	120/208V AC Panel in equipment area	Lock and tag with or without lockout hasp
Air Conditioning System	Circuit Breaker	208V AC Panel in equipment area	Lock and tag with or without lockout hasp
Heating System	Air Conditioning Circuit Breaker	208V AC Panel in equipment area	Lock and tag with or without lockout hasp

Methods to dissipate energy:

N/A

Method of Verifying the Isolation of the Machine or Equipment:

Voltmeter



Appendix F: Quarterly Maintenance Checklist



PREVENTIVE MAINTENANCE CHECKLIST

Company Performing Preventive Maintenance:

Service Technician:

Trailer ID # :	Date	Date	Date	Date	
HVAC	3M	6M	9M	12M	Comments
Inspect/change filters					
Inspect Thermostats					
Verify heat strip operation					
Inspect/clean evaporator coil					
Clean/inspect condenser coils					
Inspect electrical contactors					
Verify refrigerant pressures					
Inspect refrigeration piping abrasion					
Lubricate fan motors if applicable					
Inspect covers/fasteners					
Verify compressor amp draw					
Verify condensate pans/drains					
Verify Condenser motor operation					

Chiller	3M	6M	9M	12M	Comments
Inspect electrical contactors					
Verify refrigerant pressures					
Inspect refrigeration piping abrasion					
Inspect pump seal					
Lubricate motors					
Clean/replace aluminum filters					
Inspect covers/fasteners					
Verify operating/alarm controls					
Verify CW supply temp 45-75 F					
Inspect/replace glycol filter					
Clean/ inspect condensing coils					
Verify/adjust glycol level					
Verify Condenser motor operation					



Trailer	3M	6M	9M	12M	Comments
Test/inspect lift gate					
Inspect rails/ pins					
Inspect lift fittings/pivot points					
Clean / lubricate slide rails					
Verify lift switches and remote					
Load test van battery (lift)					
Verify hydraulic fluid level					
Verify van battery charger					
Verify roll door controls					
Inspect roll door mounting bolts					
Inspect roll door clutch/hardware					
Inspect roll door side track rails					
Inspect roll door key way					
Inspect awning					
Inspect bay door shocks/hardware					
Verify bay light operation					
Inspect clean and RF door gasket. Verify RF door operation					
Verify RF door lock and the handle operate correctly					
Check RF door for binding and loose hardware.					
Check door hinges/stops/latches for proper operation					
Inspect Slide outs for operation					
Inspect Slide out compressor					
Empty compressor drain and verify Y-strainer is cleaned out					
Check Fire system Last Inspection Date _____					
Inspect stair mounts					
Inspect interior flooring					
Verify bay heater operation					
Inspect cabinet latches and hinges					
Verify phone/communication lines					
Inspect landing gear					
Inspect locking pins					
Inspect air drive or air/hydraulic					
Inspect air tanks					
Verify hub fluid levels					
Inspect undercarriage/frame					
Inspect airbags/airlines/fittings					
Inspect shocks/bushings					
Inspect Tires / Rotate as needed					
Note hub meter mileage _____					



Generator	3M	6M	9M	12M	Comments
Clean fuel/water separator & replace filter					
Lamp test on control panel					
Inspect fuel lines & injectors					
Change oil/filters- 250 hrs					
Check crankcase breather					
Check hoses/belts					
Verify radiator coolant level					
Verify coolant freeze point & pH					
Verify block heater operation					
Inspect housing mounting bolts					
Inspect muffler/brackets					
Verify battery charging voltage					
Load test battery/clean terminals					
Verify voltage & hertz output					
Record hours run since last P.M. (_____) Recorded Generator Hours					

Electrical	3M	6M	9M	12M	Comments
Inspect breakers and panels					
Inspect lighting and bulbs					
Inspect power cord and plug					
Inspect 110volt outlets					

Humidifier	3M	6M	9M	12M	Comments
Inspect/replace steam tank					
Verify humid control set point					
Inspect/fill water reservoir					
Clean fill and drain valves					
Verify 12 volt pump					

Misc.	3M	6M	9M	12M	Comments
Attach and/or fill out Quarterly Service Record for all major components					

Comment :



Signature of Technician: _____

Date: _____

Appendix G: Medical System

Hardware Manual for Operators

Appendix B—Using Selenia in a Mobile Environment
Preparing the System for Transport



4.0 Preparing the System for Transport

Perform these steps before transporting the Mobile System:

1. Rotate the C-arm to 0 degrees (CC position).
2. Lower the C-arm to its lowest position.
3. Shut down the system completely.
4. Remove all system accessories (such as the Face Shield, Compression Paddles, etc.) before transport. Store all accessories appropriately to assure the integrity of the components.

5.0 Testing System Integrity After Transport

Instructions: Photocopy the checklist on the following page for use after the system is relocated.

5.1 Selenia Controls and Functional Tests

Perform the Controls and Functional Tests. Refer to Chapter 2, Selenia Controls and Functional Tests, p. 19.

- C-Arm, Compression, and other functional tests
- Emergency Off Switch tests

5.2 Compression Thickness Test

Required Equipment:

Acrylic block or BR-12 of known thickness (approximately 4-5 cm thick)

7.5 cm Spot Contact compression paddle.

1. Center the attenuator laterally on the image receptor and position it so the chestwall edge of the phantom is aligned with the chest wall side of the image receptor.
2. Install the 7.5 cm spot contact compression paddle in the compression device.
3. Apply Full Automatic Compression of approximately 30 pounds to the attenuation block.
4. Confirm that the system displays the thickness of the attenuator (± 0.5 cm).

5.3 Quality Control Tests

Refer to your Selenia Quality Control Manual for detailed instructions.

- Artifact Evaluation
- Signal-to-Noise Ratio Test
- Phantom Image Evaluation (optional)



Hardware Manual for Operators
Appendix B—Using Selenia in a Mobile Environment
Integrity Checklist

6.0 Integrity Checklist

Instructions: Photocopy this checklist for use after the system is relocated.

Name _____ Date _____

System Serial Number _____

Evaluated	Control Functions	Reference/Sec/Page
Mechanical System Tests		
	Compression Up/Down	Chapter 2, Selenia Controls and Functional Tests, p. 19
	Compression Release	Chapter 2, Selenia Controls and Functional Tests, p. 19
	C-arm Rotation	Chapter 2, Selenia Controls and Functional Tests, p. 19
	C-arm Up/down	Chapter 2, Selenia Controls and Functional Tests, p. 19
	Collimator Override	Chapter 2, Selenia Controls and Functional Tests, p. 19
	Light Field Lamp	Chapter 2, Selenia Controls and Functional Tests, p. 19
	Smart Paddle System	Chapter 2, Selenia Controls and Functional Tests, p. 19
	Emergency Off Switches	Chapter 2, Emergency Off Switches, p. 25
	Compression Thickness	Appendix B, Section 5.2, p. 60
Quality Control Tests		
	Artifact Evaluation	Selenia QC Manual
	Signal-to Noise Ratio Test	Selenia QC Manual
	Phantom Image Evaluation	Selenia QC Manual