

daeg  SPO2 Analyzer  
User and Service Manual

**METRON**

## Metron Offices

**USA**

1345 Monroe NW, Suite 255A  
Grand Rapids, MI 49505  
Phone: (+1) 888 863-8766  
Fax: (+1) 616 454-3350  
E-mail: support@metron.us

**FRANCE**

30, rue Paul Claudel  
F-91000 Evry  
Phone: (+33) 1 6078 8899  
Fax: (+33) 1 6078 6839  
E-mail: info@metron.fr

**GERMANY**

Römerstrasse 22  
D-63741 Aschaffenburg  
Phone: (+49) 6021 447 9807  
Fax: (+49) 6021 447 9808  
E-mail: proteges@t-online.de

**NORWAY**

Vegamot 8  
N-7048 Trondheim  
Phone: (+47) 7395 4700  
Fax: (+47) 7395 4701  
E-mail: support@metron.no

***Disclaimer***

METRON provides this publication as is without warranty of any kind, either express or implied, including but not limited to the implied warranties of merchantability or fitness for any particular purpose. Further, METRON reserves the right to revise this publication and to make changes from time to time to the content hereof, without obligation to METRON or its local representatives to notify any person of such revision or changes. Some jurisdictions do not allow disclaimers of expressed or implied warranties in certain transactions; therefore, this statement may not apply to you.

***Limited Warranty***

METRON warrants that the daeg SPO2 Analyzer will substantially conform to published specifications and to the documentation, provided that it is used for the purpose for which it was designed. METRON will, for a period of twelve (12) months from date of purchase, replace or repair any defective analyzer, if the fault is due to a manufacturing defect. In no event will METRON or its local representatives be liable for direct, indirect, special, incidental, or consequential damages arising out of the use of or inability to use the daeg SPO2 Analyzer, even if advised of the possibility of such damages. METRON or its local representatives are not responsible for any costs, loss of profits, loss of data, or claims by third parties due to use of, or inability to use the daeg SPO2 Analyzer. Neither METRON nor its local representatives will accept, nor be bound by any other form of guarantee concerning the daeg SPO2 Analyzer other than this guarantee. Some jurisdictions do not allow disclaimers of expressed or implied warranties in certain transactions; therefore, this statement may not apply to you.

## Table of Contents

Metron Offices.....	2
Table of Contents .....	3
daeg  SpO2 Analyzer Specifications.....	5
daeg Quick Start .....	7
Unit Setup .....	9
Unpacking.....	9
Battery Installation .....	9
Operation.....	11
Power up Screen.....	11
Main Simulation Screens.....	11
Function Selection Menu.....	12
Alarm Limits .....	12
Remote Control.....	13
Probe Analyzer.....	13
Simulation Setup.....	14
Print Options .....	15
Troubleshooting .....	17
Calibration.....	19
Schematics .....	21

This page was intentionally left blank.

# daeg SpO2 Analyzer Specifications

## SPO2 SIMULATION

### GENERAL

- Factory installed, downloadable and user programmable R-curves, patient conditions and templates.
- Probe signal and LED test x

### ACTIVE PROBE SPO2 SIMULATION

- O2 saturation range: 30 - 100 %
- Simulation resolution: 1% SpO2%
- Simulation accuracy: ± 0.5 SpO2% 65-100% SpO2% ± 1 SpO2% 30-64% SpO2%
- Simulation repeatability: ± 0.2 SpO2%
- Simulation rate: 30 - 300 bpm, res. 5 bpm (sync. to ECG for "C-lock" testing)
- Rate accuracy: ± 0.1%
- Pulse amplitude: 0 - 100% of nominal pleth amplitude.
- Amplitude accuracy: ≥30% amplitude ±1%. 1-29% amplitude ±5%.
- Pigmentation: Low, medium and high
- Light repeater bandwidth: DC - 1MHz
- Optical slew rate: 50 mcd/μs
- Light repeater dynamic range: 1mcd + 13 dB - 60 dB
- Receiver optical 3 dB bandwidth: 600 - 1050 nm.

### INSTALLED R-CURVES

- BCI International
- Criticare
- Masimo
- Nellcor
- Novamatrix
- Philips (Agilent)
- Ohmeda

### INSTALLED PATIENT CONDITIONS

- Normal patient
- Geriatric patient
- Obese patient
- Patient with weak pulse
- Bradycardia
- Tachycardia

### ARTIFACTS (ACTIVE PROBE ONLY)

- Motion: 0 - 4Hz
- Ambient sunlight: Ambient, Sun
- Interfering light: 50Hz, 60Hz

### PROBE LED OPTICAL FREQUENCY

- Frequency range: 550 - 1050 nm
- Resolution: 1nm
- Accuracy: ±2% of range

### PROBE TEST

- Available with release of Probe Test Adaptor Box and firmware update.

## ECG SIMULATION - GENERAL

The data below is related both to factory installed and downloadable waveforms.

- Lead configuration: 5 lead RL, RA, LA, LL, V1-6
- Low level output amplitude: 0.2-2mV, resolution 100 mV
- Low level output impedance: 1000 ohm to RL
- High level output amplitude: 0.5V/mV of low level.
- High level output impedance: 50 ohm
- System amplitude accuracy: ±2% @1mV
- System rate accuracy: ± 0.1%
- System time resolution: 10 μs (100 kHz maximum update rate)

## ECG OUTPUT

The data below is related to factory installed waveform types.

### NORMAL ECG SINUS

- 30 - 300 BPM, resolution 1 BPM

## GENERAL INFORMATION

### POWER SUPPLY

- Power sources: 4 standard AA/LR6/MN-1500 cells. 230VAC/115VAC to 9vDC battery eliminator.
- Power consumption: 200 mA w/o display backlight. 400 mA w/full backlight.
- Battery lifetime: 12 h for alkaline cells without backlight.

## HOUSING

High impact plastic case.

## DIMENSIONS

- Height: 237 mm/9.2"
- Width: 122 mm/4.8"
- Depth: 42 mm/1.6"

## WEIGHT

With battery: 0.6 kg/1.3 lb.

## TEMPERATURE

- Storage temperature: 0/32 to +50/122 ° C/F
- Operating temperature: + 15/59 to +35/95° C/F

## HUMIDITY

Operating humidity: 10% - 80% (non-condensing)

## DISPLAY

Graphical LCD, with backlighting.

## CONNECTORS

- 9V DC power inlet (standard 2.1 mm power jack)
- RS-232/C for PC or printer communication (9-pin D-sub male)
- 1 probe connector (Redel 8-pin).
- 5 low level ECG outputs RA, RL, LA, LL and V1 (AHA color coded safety connector)
- 1 high level ECG output (standard phono jack).

## daeg SpO2 Analyzer Ordering Information

Order no:  
12700: **daeg** SpO2 Analyzer

### Accessories:

- 17021: Battery Eliminator 240V or
- 17027: Battery Eliminator 115V
- 17030: Battery Package. 4-cells
- 12710: User and Service Manual

### Options:

- 11150: Carrying case
- 12711: Cable Test Adapter Box
- 12712: PRO-Soft **daeg**
- 12713: PRO-Soft **daeg**, demo
- 12714: User Manual PRO-Soft daeg
- 17024: Universal Banana Adapter

This page was intentionally left blank.

# daeg Quick Start




## daeg Startup

1. Connect daeg's Finger Probe to the main daeg unit, and switch the daeg unit **on**.

The daeg must be powered by the 9VDC 400mA power supply or by 4 AA alkaline batteries loaded into the unit's battery compartment.

2. The version number of the firmware will be briefly displayed, followed by the first display screen (below). The unit is immediately outputting the simulation as displayed.

3. Apply the oximeter's sensor to the daeg finger probe. The sensor's transmitter must be on the bottom (unlabeled) side of the daeg finger.


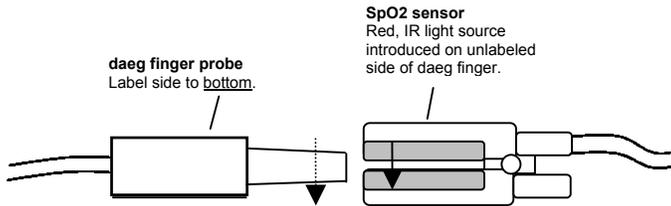


3. To change the simulation setting for oximeter model (Make), Ambient Light, Pigmentation, or to select an automated simulation sequence, select the **F3** function key to go to the second display screen.
4. To access additional test and auxiliary functions, press the **F1** function key while in the first or second screen display.

Additional / Auxiliary functions include:  
Alarm Tests, Print functions, Remote Control Operation, Probe Tests, and Simulation Setup.



## Daeg Operation / Simulation

1. To adjust the simulation settings, use the **F2** function key to select the "active" parameter. The active parameter is marked with an asterisk (\*).

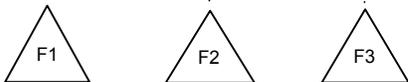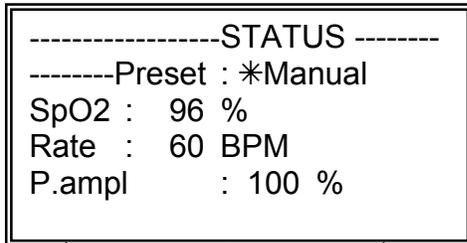

**Note:** Menu items at the bottom of the display are activated/accessed using the F1, F2, and F3 function keys.

2. Adjust the active parameter using the yellow Up and Down arrow keys on the keypad, then press **enter** to confirm the change.

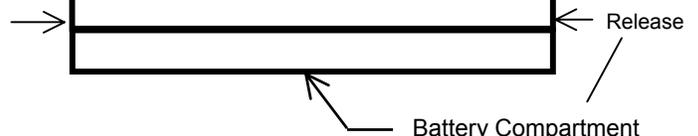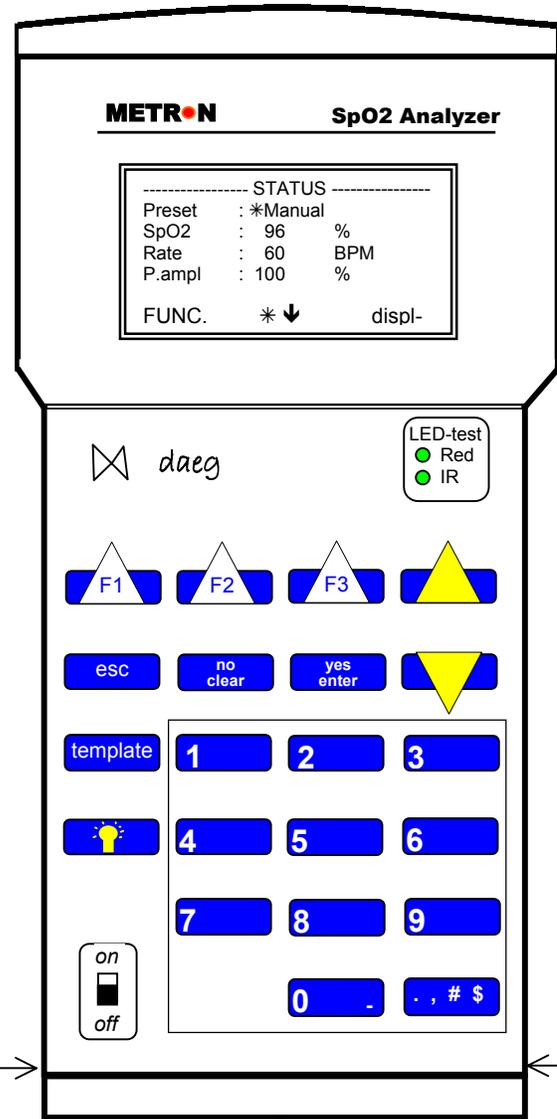

This page was intentionally left blank.

## Unit Setup

### Unpacking


Unpack the unit and confirm the presence of the following components:

- 1 - daeg main unit
- 1 - daeg artificial finger
- 4 - AA batteries
- 1 – battery eliminator

If any of these items are missing or damaged, please contact your local Metron office.

### Battery Installation

Refer to the diagram below when installing the batteries. You will need to depress the tabs shown in the diagram below and remove the bottom dark gray piece of the chassis. This will then expose the battery connector and battery tray. Disconnect the battery clip and remove the battery tray. When inserting the batteries into the tray, be sure to observe the polarity of the batteries. When the tray is loaded back into the unit, reconnect the clip, and then replace the chassis piece.

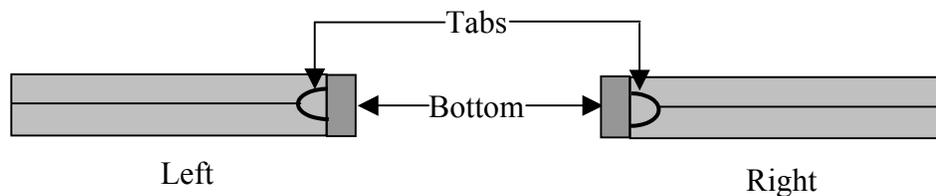

This page was intentionally left blank.

## Operation


### Power up Screen

```

----- METRON daeg ---
SpO2 Meter Analyzer

Revision X.XX
loading sim data...
    
```

This screen is the first one that is visible when the unit is turned on. It indicates which version of the firmware is installed.

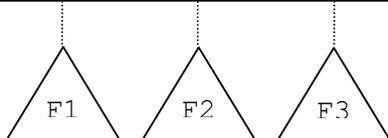


### Main Simulation Screens

```

----- STATUS -----
Preset:*Manual
SpO2 : 96 %
Rate : 60 BPM
P.ampl: 100 %

FUNC. * ↓ displ-1
    
```


These are the two screens that are used when performing SPO2 simulations. You can change between the screens by pressing the (F3) button. The (F2) button will move the cursor, (\*), to the next parameter. Using the Up and Down arrows will let you change the value of the selected parameter. You will then need to press the (yes/enter) button to accept the change. Pressing the (F1) button will take you the **Function Selection Menu**.

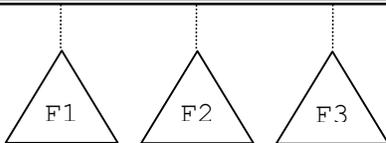





```

----- SETUP -----
Make : *Nellcor
Oper. : Manual
Amb.L.: Manual
Pigm. : Manual

FUNC. * ↓ displ-2
    
```


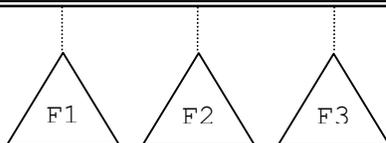


## Function Selection Menu

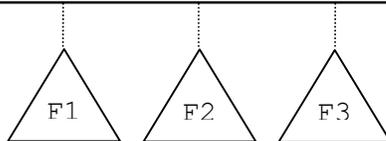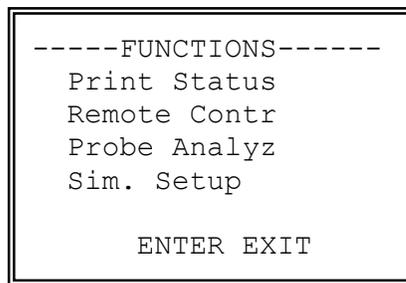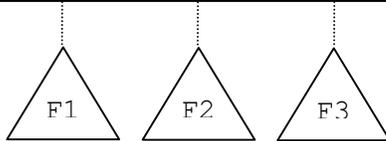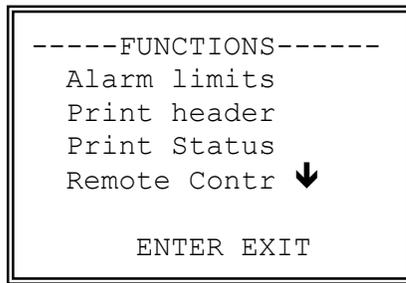

This screen will let you select which testing function of the daeg that you wish to perform. Using the **Up** and **Down** arrows will let you move the cursor (->) next to the function that you wish to use. Once the cursor is placed press the **(F2)** button or the **(yes/enter)** button to enter that function. Pressing the **(F3)** or **(esc)** buttons will take you back to the **Main Simulation Screen**.


## Alarm Limits

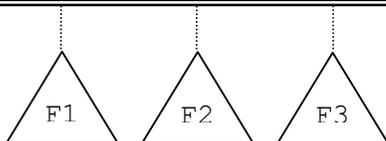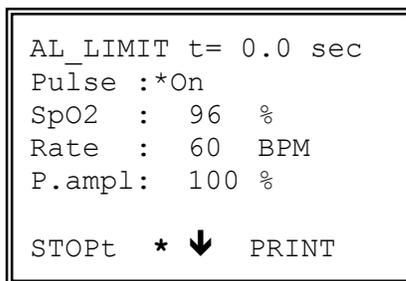

This test is designed to test your alarm system on your monitor. The **(F2)** button will move the cursor, (\*), to the next parameter. Using the Up and Down arrows will let you change the value of the selected parameter. You will then need to press the **(yes/enter)** button to accept the change and start the timer. Press the **(F1)** button to stop the clock when you hear the alarm from your monitor. The **(F3)** button will generate a line of printout via the parallel port. Pressing the **(esc)** button will take you back to the **Main Simulation Screen**.

## Remote Control


```
----- STATUS -----  
Preset: *Manual  
SpO2 : 96 %  
Rate : 60 BPM  
P.ampl: 100 %  
LOCAL  
CONTR. * ↓
```

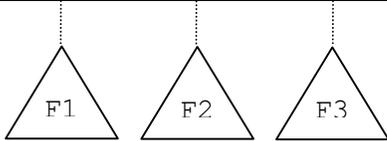

This is the screen that is shown when the daeg is in remote control mode. All currently active parameters are displayed on the screen. Pressing (**F1**) will cancel the remote control mode and take you back to the **Main Simulation Screen**.

## Probe Analyzer

```
-- PROBE ANALYZER --  
  
Choose a test to  
perform on the  
SpO2 probe.  
  
DIODE CONTI SENSI
```

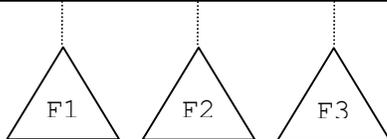


This screen will let you select which test you wish to perform on the finger probe when you have the probe tester adaptor box attached.


## Simulation Setup

```
SIM SETUP:

To change values,
use 'Yes' -button
for up, and
'No' -button for
down!
```

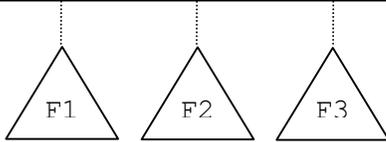

```
Make :Nellcor
RedDC:2048
IRDC :1024
Lcode: 101
Rval :0.48@SpO2:100
          SAVE &
SpO2- SpO2+ QUIT
```

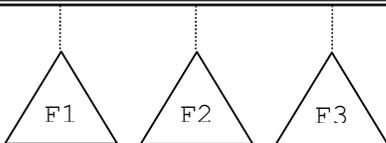

This function lets you manually alter the R-Curves within the daeg unit.


*Be cautious when using this mode as you are altering the analyzer output. If you make any changes that you do not wish to keep you must turn off the daeg for 5 seconds then turn it back on.*

When in this mode the controls will be as follows:

- Up** and **Down** arrows control the cursor position.
- (yes/enter)** increases the indicated value.
- (no/clear)** decreases the selected value.
- (F1)** decreases the selected SPO2 value.
- (F2)** increases the selected SPO2 value.
- (F3)** Saves the changes and takes you back to the **Main Simulation Screen**.

*The saving procedure may take up to 5 minutes. Do not turn off the power during this time.*

## Print Options

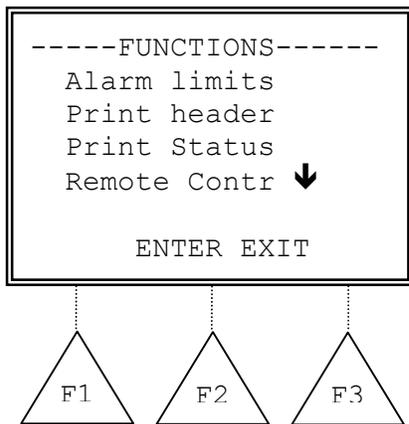

The daeg unit may be attached directly to most printers via the 25-pin parallel printer port located on the side of the unit. Information is sent to the printer in standard ASCII text format. From the (**FUNCTION**) menu, you may select the (**Print Header**) or (**Print Status**) items. Pressing (**F2**) **ENTER** or (**yes/enter**) will initiate the print command.

If you select (**Print Header**) a header will be printed, identifying the daeg unit and version of firmware installed in the analyzer. The header provides spaces for the user to enter information concerning the identity and location of the device being tested, as well as spaces to verify that the test

has passed or failed. There are also spaces to enter the date, sign the form, and add any comments. (See page xx for example of printout.)

Selecting (**Print Status**) will provide a line that identifies the simulation parameter (make, ambient light, pigmentation, SpO2, heart rate, and pulse amplitude. A space is also provided for recording the resulting SpO2 and rate as viewed on the oximeter. (See page 16 for example of printout.)

There is also an option to print from the Alarm Limits Test. At the conclusion of a test, press (**F3**) to print the result of the Alarm Test. The results show the settings for the test and the time/delay until the oximeter alarm was activated. (See page 16 for example of printout.)

### NOTE:

Many printers will hold the header and status/result lines in memory until you prompt the printer to go/advance with the print job. The printer should indicate that it is receiving or has received data when the print commands are made from daeg.

METRON daeg SpO2-meter Analyzer Ver.1.02

DAEG Serial no. : \_\_\_\_\_

Establishment : \_\_\_\_\_

Appliance code: \_\_\_\_\_

Serial no. : \_\_\_\_\_

Status : \_\_\_\_\_

Group : \_\_\_\_\_

Manufacturer : \_\_\_\_\_

Model : \_\_\_\_\_

Type : \_\_\_\_\_

Location : \_\_\_\_\_

Unit passed test: \_\_\_\_\_ Unit failed test: \_\_\_\_\_

Comments : \_\_\_\_\_

Date : \_\_\_\_\_

Signature : \_\_\_\_\_

Test#	Make	Simulation parameters:				Result:		
		AmbL.	Pigm.	Preset	SpO2 Rate	Ampl.	SpO2	Rate
1	Nellcor	Normal	Medium	Normal	98%	60bpm	100%	_____ % _____ bpm
2	Nellcor	Normal	Medium	Weak P	90%	95bpm	10%	_____ % _____ bpm
3	Nellcor	Normal	Medium	Bradyc	88%	45bpm	100%	_____ % _____ bpm
4	Nellcor	Normal	Medium	Tachyc	85%	130bpm	20%	_____ % _____ bpm
5	Nellcor	Normal	Medium	Geriat	92%	95bpm	40%	_____ % _____ bpm
6	Nellcor	Normal	Medium	Motion	96%	75bpm	100%	_____ % _____ bpm
7	Nellcor	Normal	Medium	Motion				83% 75bpm 100%
				SpO2 alarm response time:	11.7	Seconds		
8	Nellcor	Normal	Medium	Normal				98% 50bpm 100%
				Rate alarm response time:	15.1	Seconds		

## Troubleshooting

**Q:** SPO2 values do not match.

**A:** Confirm that the “Make” is properly selected.

**Q:** The monitor is not reading any values.

**A:** Check all connections and verify if the “Red” and “IR” lights on the daeg are illuminated.

This page was intentionally left blank.


## Calibration


\*\*\*<<Coming Soon>>\*\*\*

This page was intentionally left blank.

## **Schematics**


The schematics on the following page are for the **daeg** unit and the **Finger Probe**.

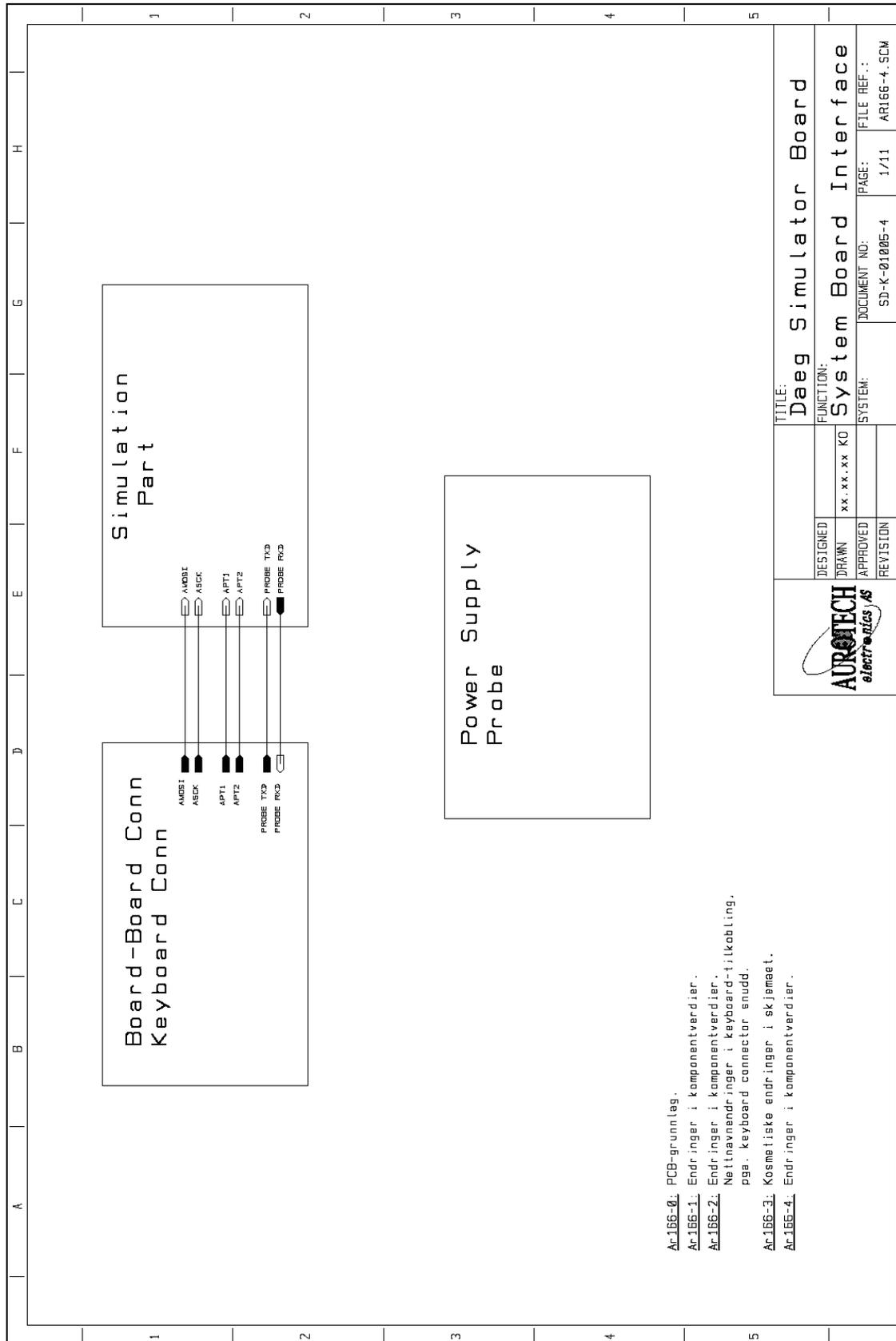

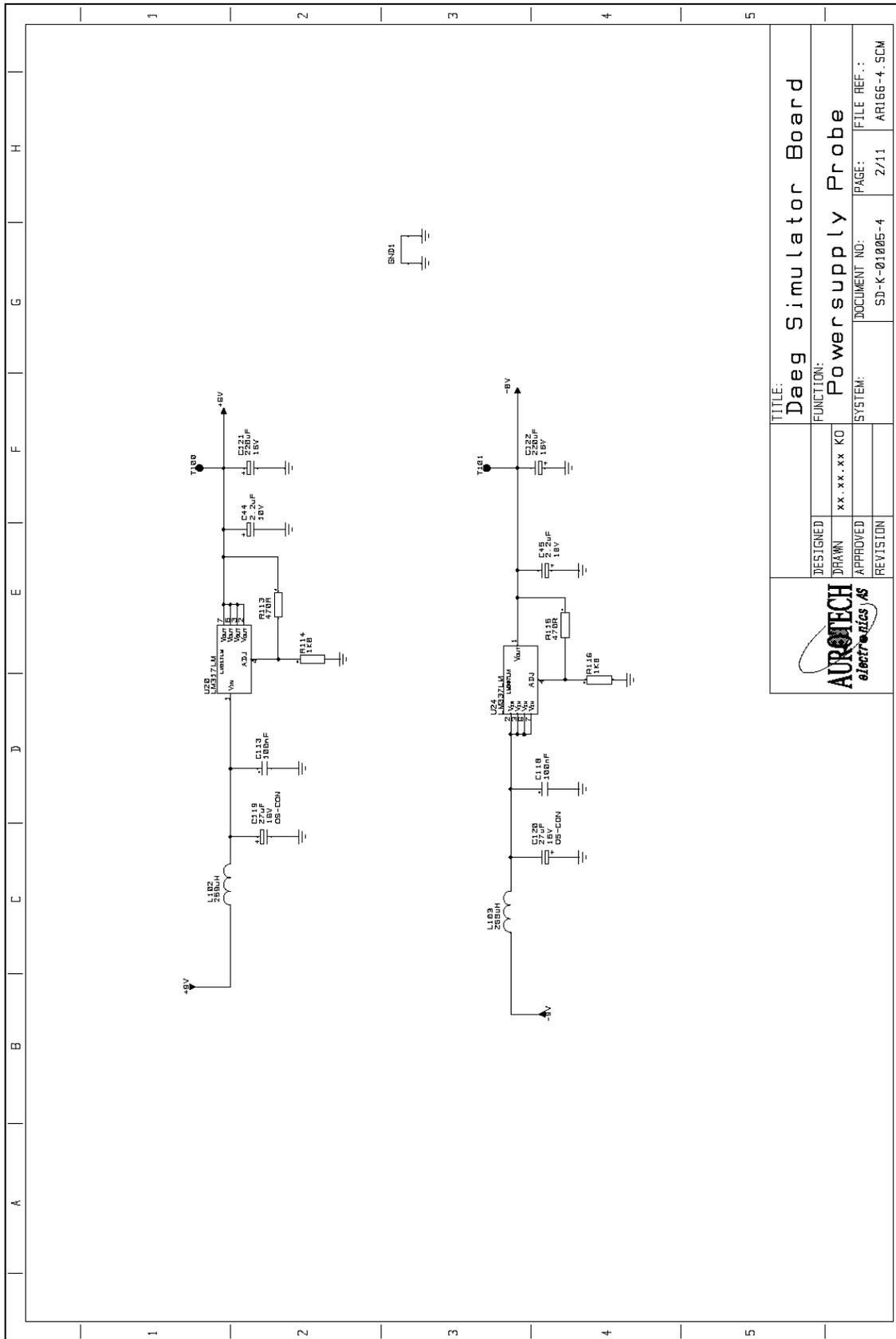


DESIGNED		DRAWN		APPROVED		REVISION	
xx.xx.xx KO		xx.xx.xx KO		xx.xx.xx KO		xx.xx.xx KO	
<b>AURATECH</b> <i>electronics</i> / AS				TITLE: Daeg Simulator Board FUNCTION: Power supply Probe SYSTEM: SD-K-01005-4 PAGE: 2/11 FILE REF.: ARI66-4.SCM			



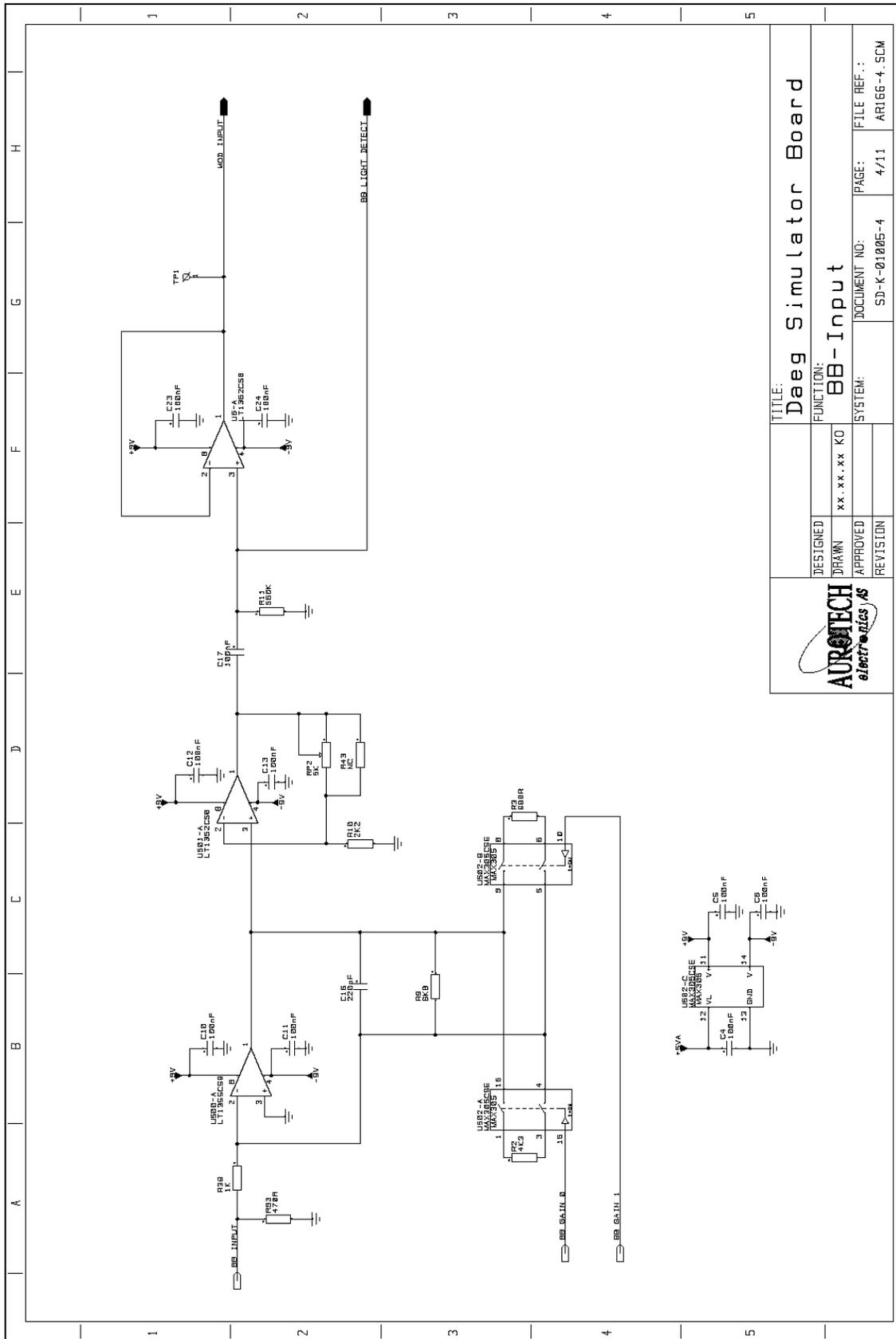


DESIGNED		TITLE: Daeg Simulator Board	
DRAWN		FUNCTION: BB-Input	
APPROVED		DOCUMENT NO: SD-K-01005-4	FILE REF.: ARI66-4.SCM
REVISION		PAGE: 4/11	




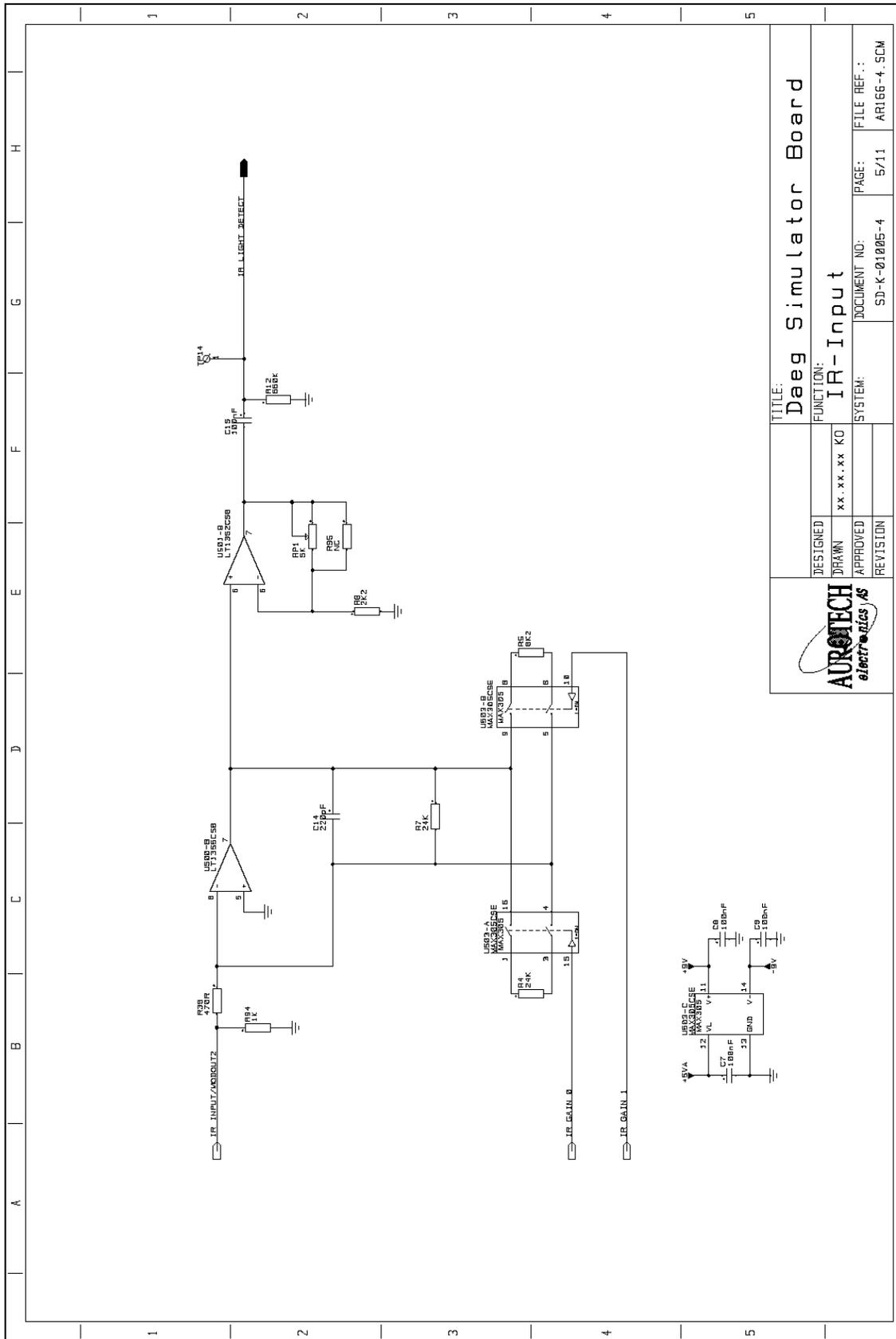


<b>AURATECH</b> <i>electronics</i> / AS		<b>TITLE:</b> Daeg Simulator Board	
<b>DESIGNED</b> xx.xx.xx KO	<b>FUNCTION:</b> IR-Input	<b>DOCUMENT NO.:</b> SD-K-01005-4	<b>PAGE:</b> 5/11
<b>APPROVED</b> REVISION	<b>SYSTEM:</b>	<b>FILE REF.:</b> ARI66-4.SCM	

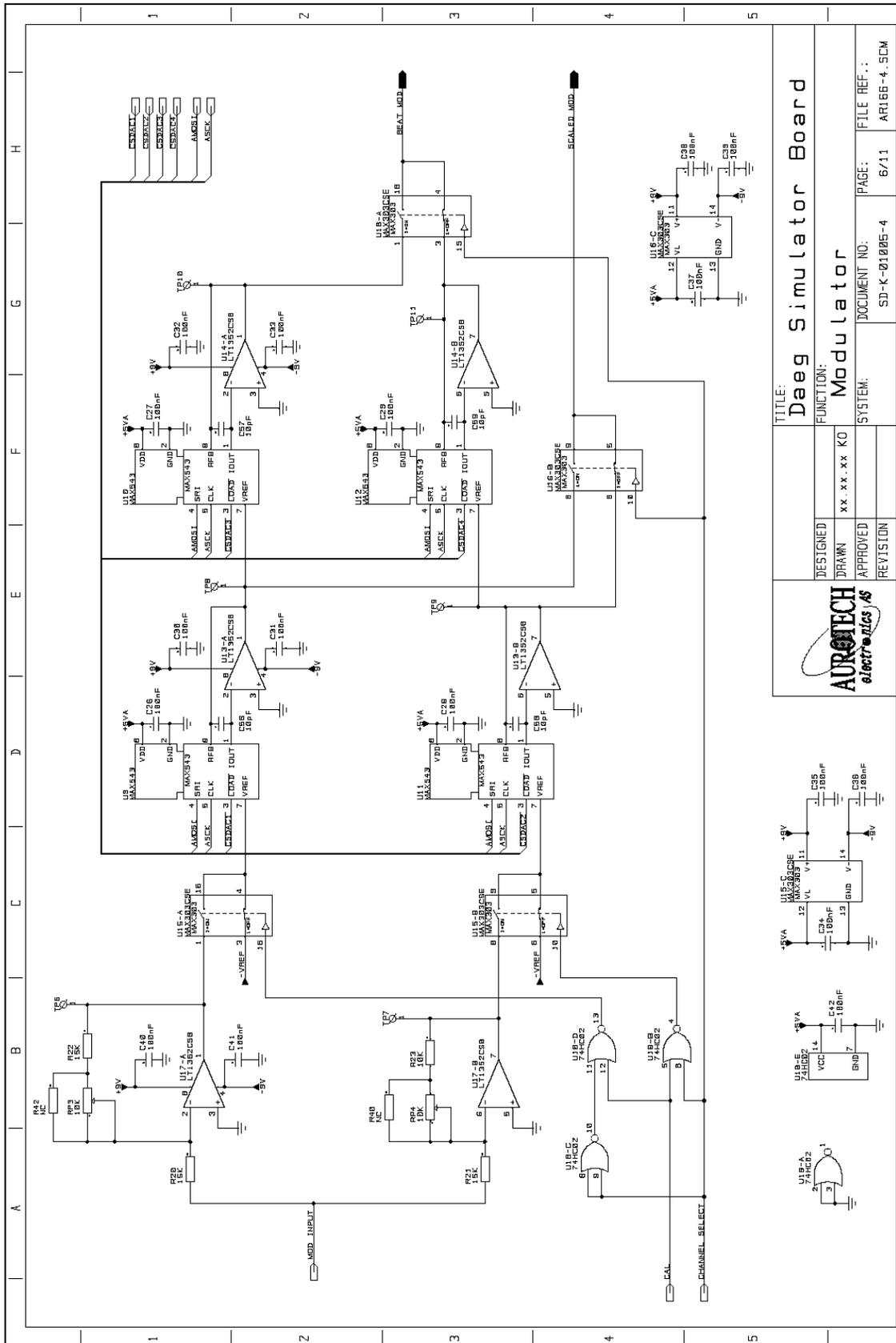


DESIGNED		TITLE: Daeg Simulator Board	
DRAWN		FUNCTION: Modulator	
APPROVED		DOCUMENT NO: SD-K-01005-4	
REVISION		SYSTEM: ARI66-4.5CM	
		PAGE: 6/11	
		FILE REF.:	

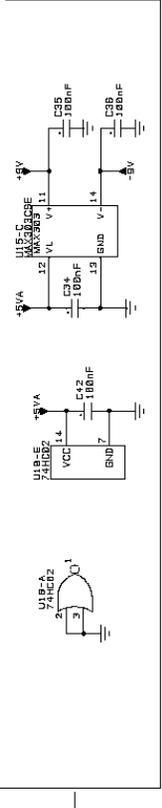

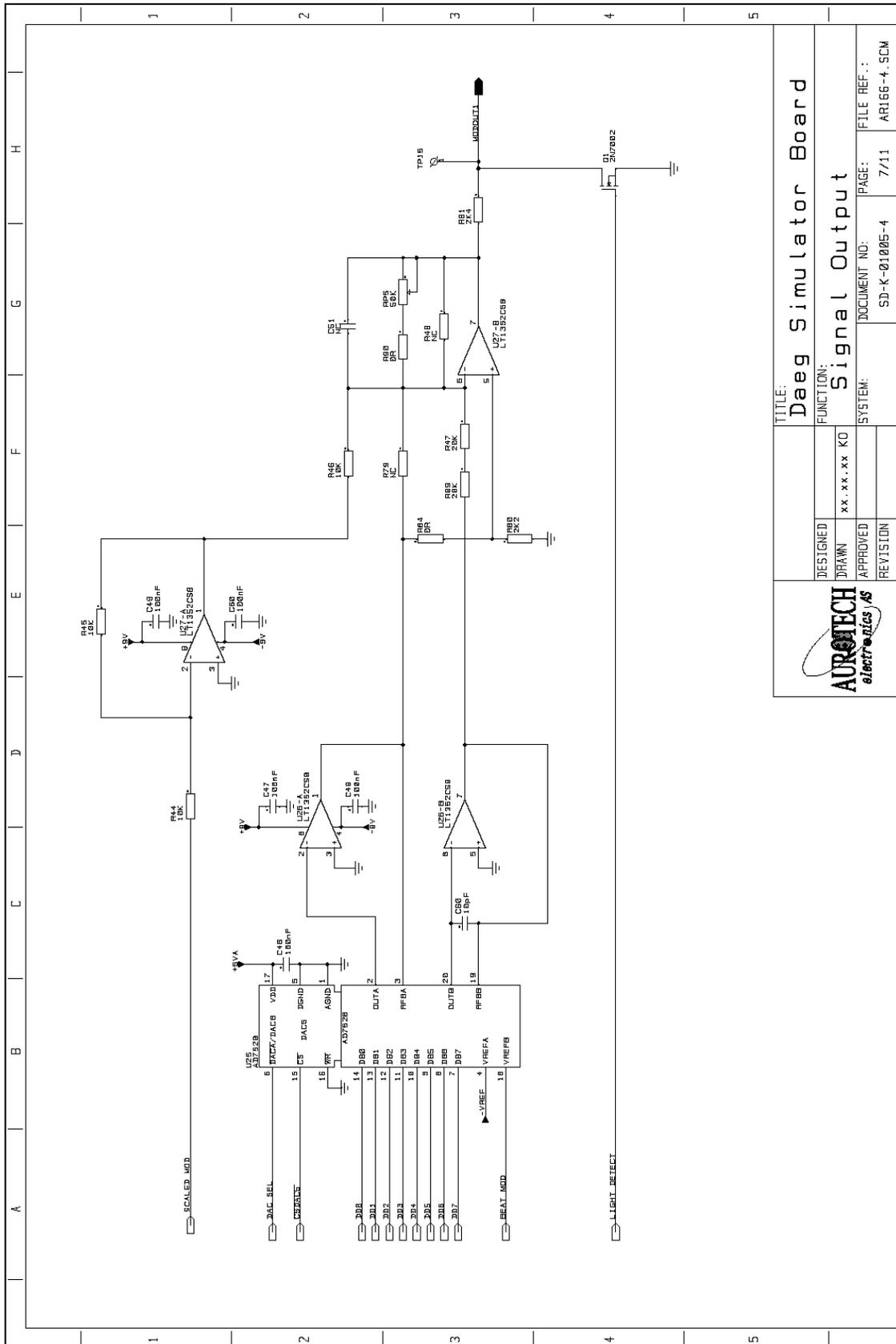


DESIGNED		TITLE: Daeg Simulator Board	
DRAWN xx.xx.xx KO		FUNCTION: Signal Output	
APPROVED		DOCUMENT NO: SD-K-01005-4	
REVISION		PAGE: 7/11	
		FILE REF.: ARI66-4.SCM	



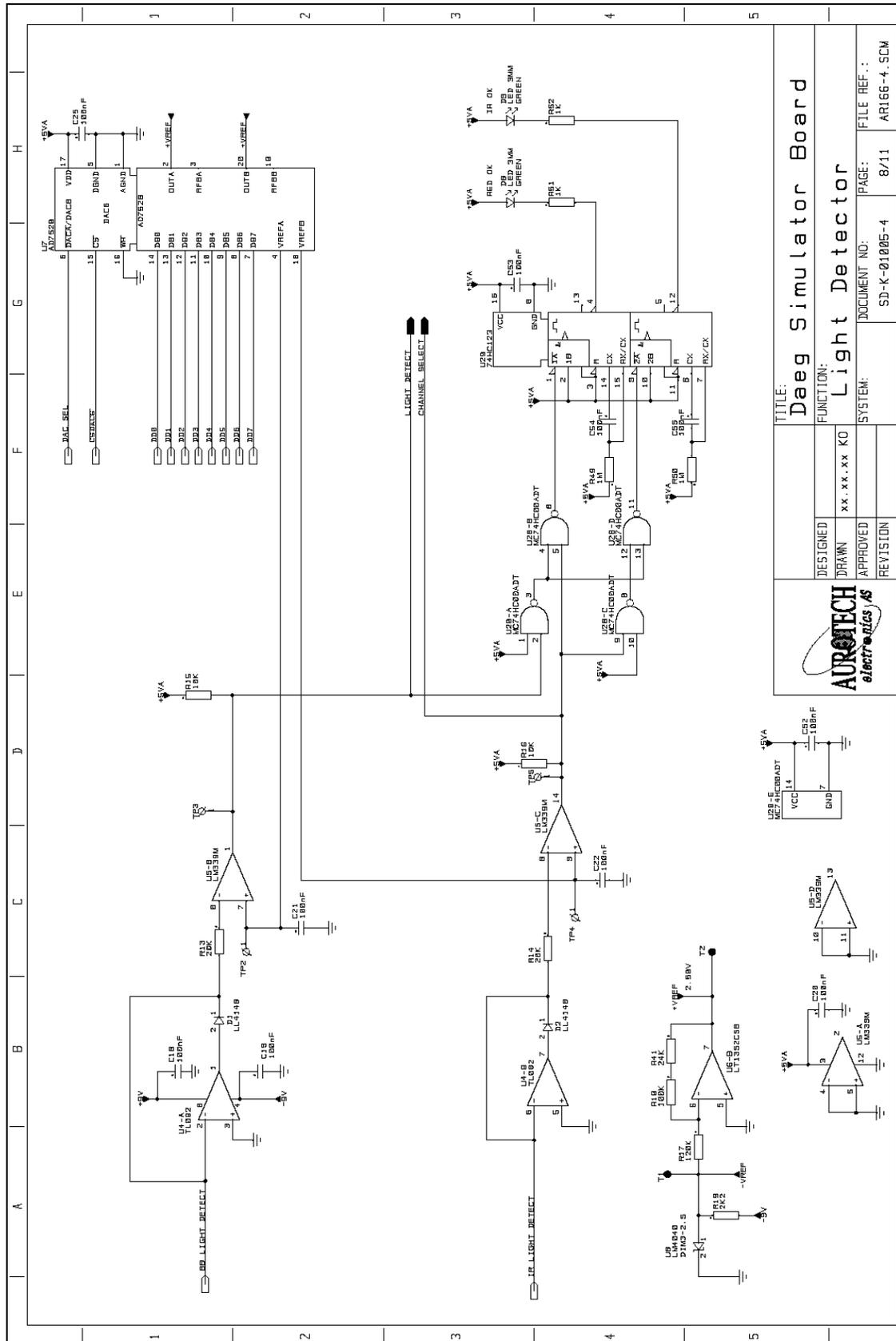


DESIGNED		DRAWN		APPROVED		REVISION	
xx.xx.xx KO		xx.xx.xx KO		xx.xx.xx KO		xx.xx.xx KO	
<b>AURATECH</b> <i>electronics AS</i>				TITLE: Daeg Simulator Board FUNCTION: Light Detector			
SYSTEM:				DOCUMENT NO: SD-K-01005-4			
PAGE: 8/11				FILE REF.: ARI66-4.SCM			

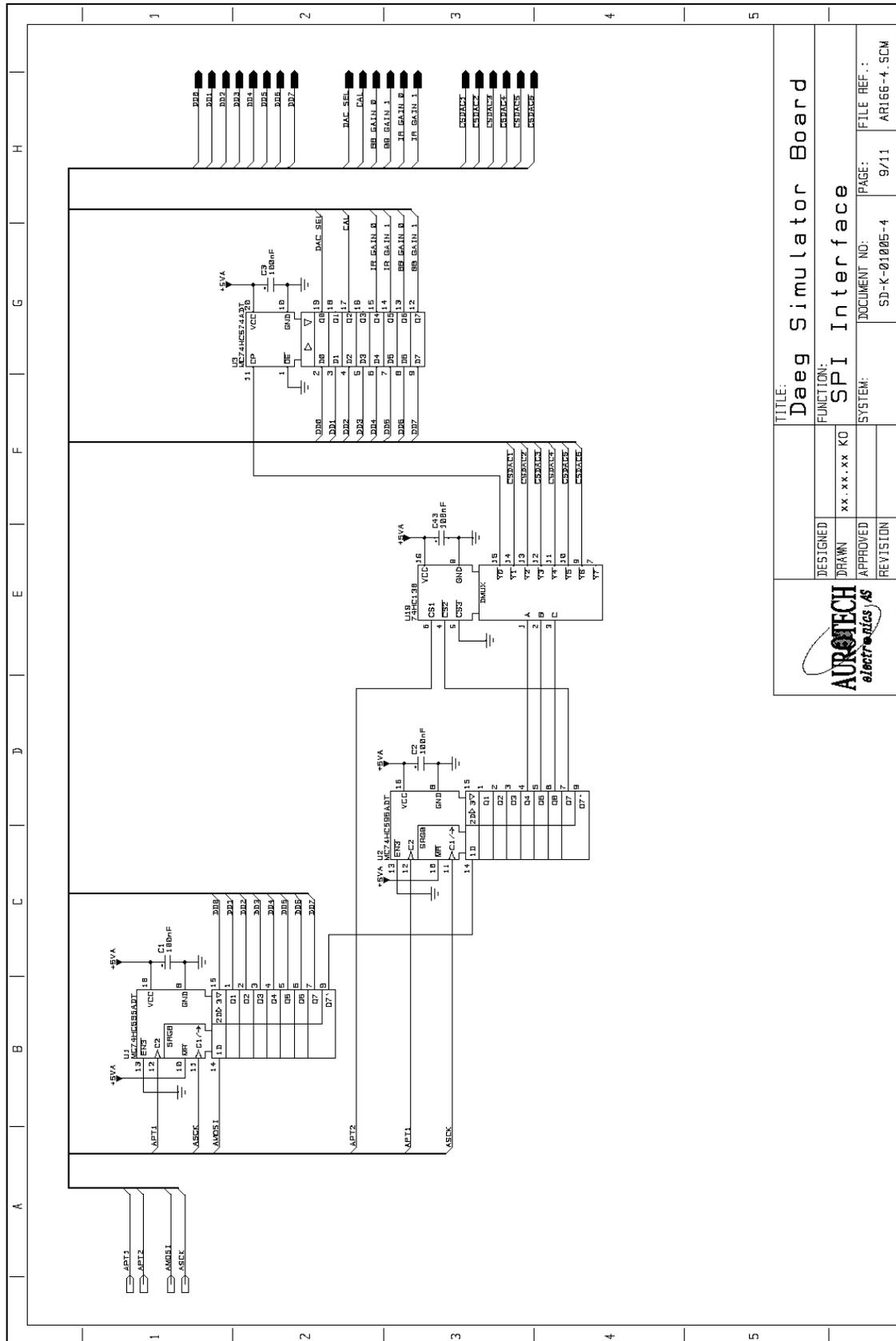

TITLE: Daeg Simulator Board


DESIGNED	xx.xx.xx KO	FUNCTION:	SPI Interface
DRAWN		APPROVED	
REVISION		SYSTEM:	SD-K-01005-4
		DOCUMENT NO:	9/11
		PAGE:	ARI66-4.SCM
		FILE REF.:	



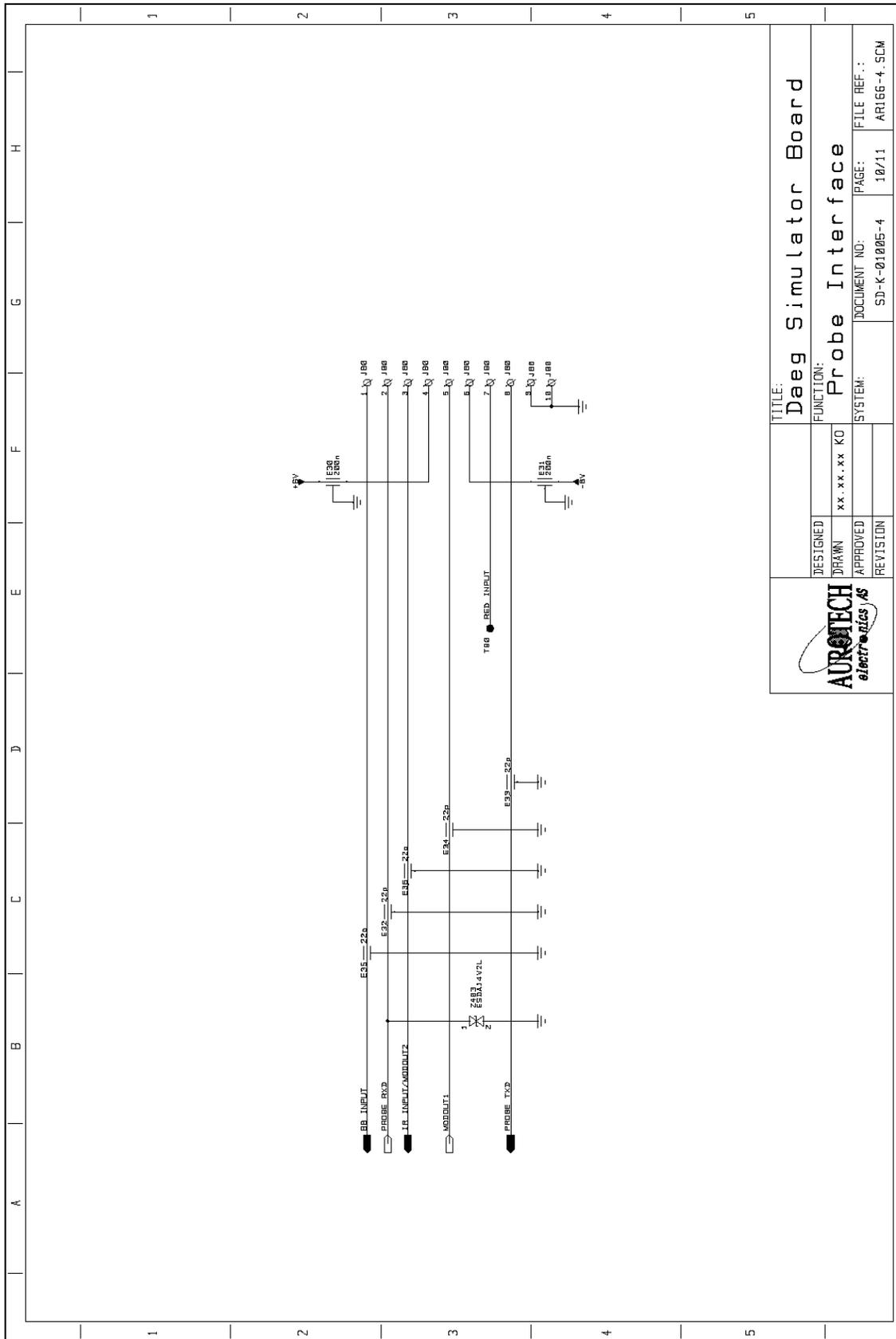

<b>AURATECH</b> <i>electronics</i>		TITLE: Daeg Simulator Board	
DESIGNED	xx.xx.xx KO	FUNCTION: Probe Interface	
DRAWN		DOCUMENT NO: SD-K-01005-4	PAGE: 10/11
APPROVED		SYSTEM:	FILE REF.: ARI66-4.SCM
REVISION			




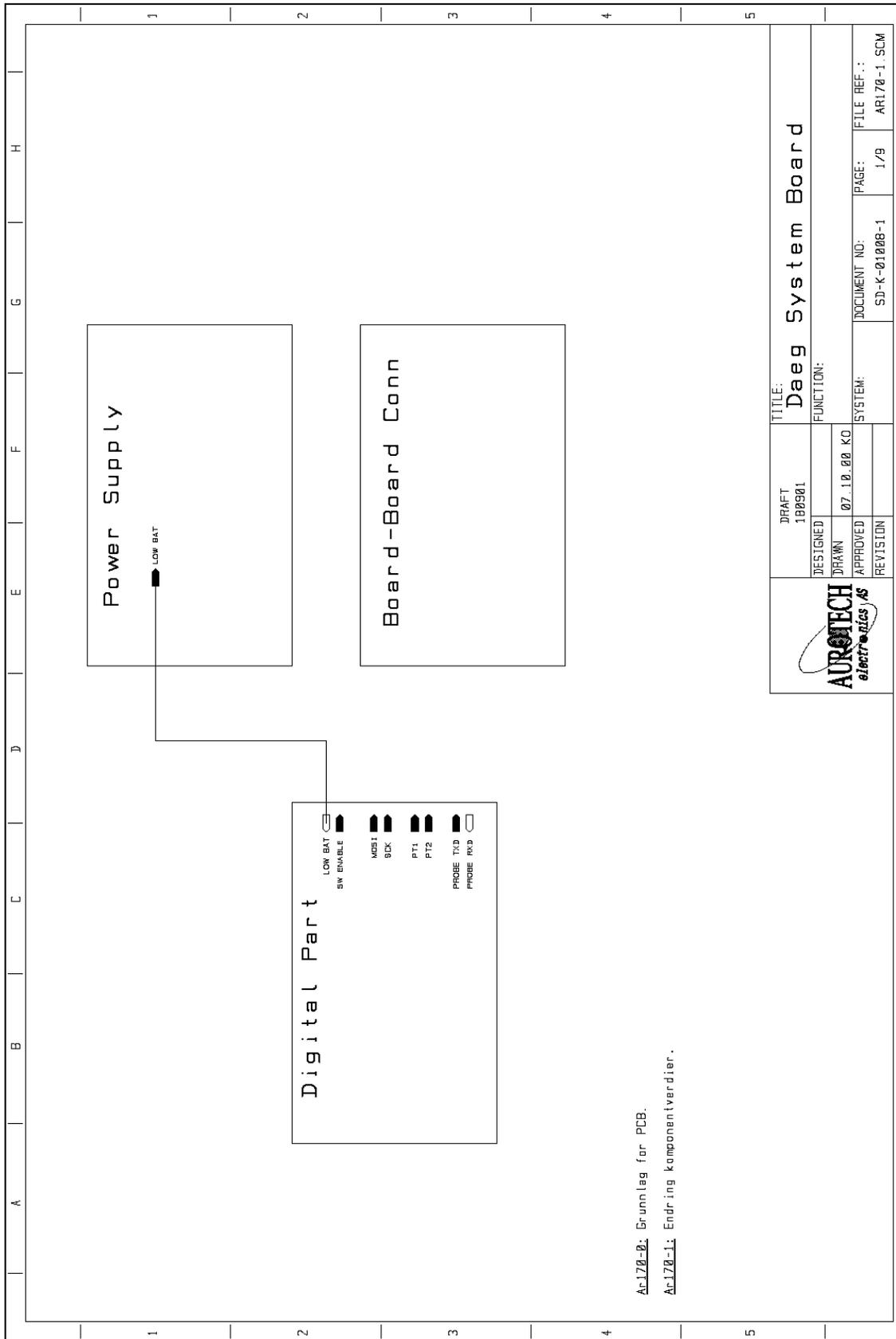

Ac170-0: Grunnlag for PCB.

Ac170-1: Endring komponentverdiert.




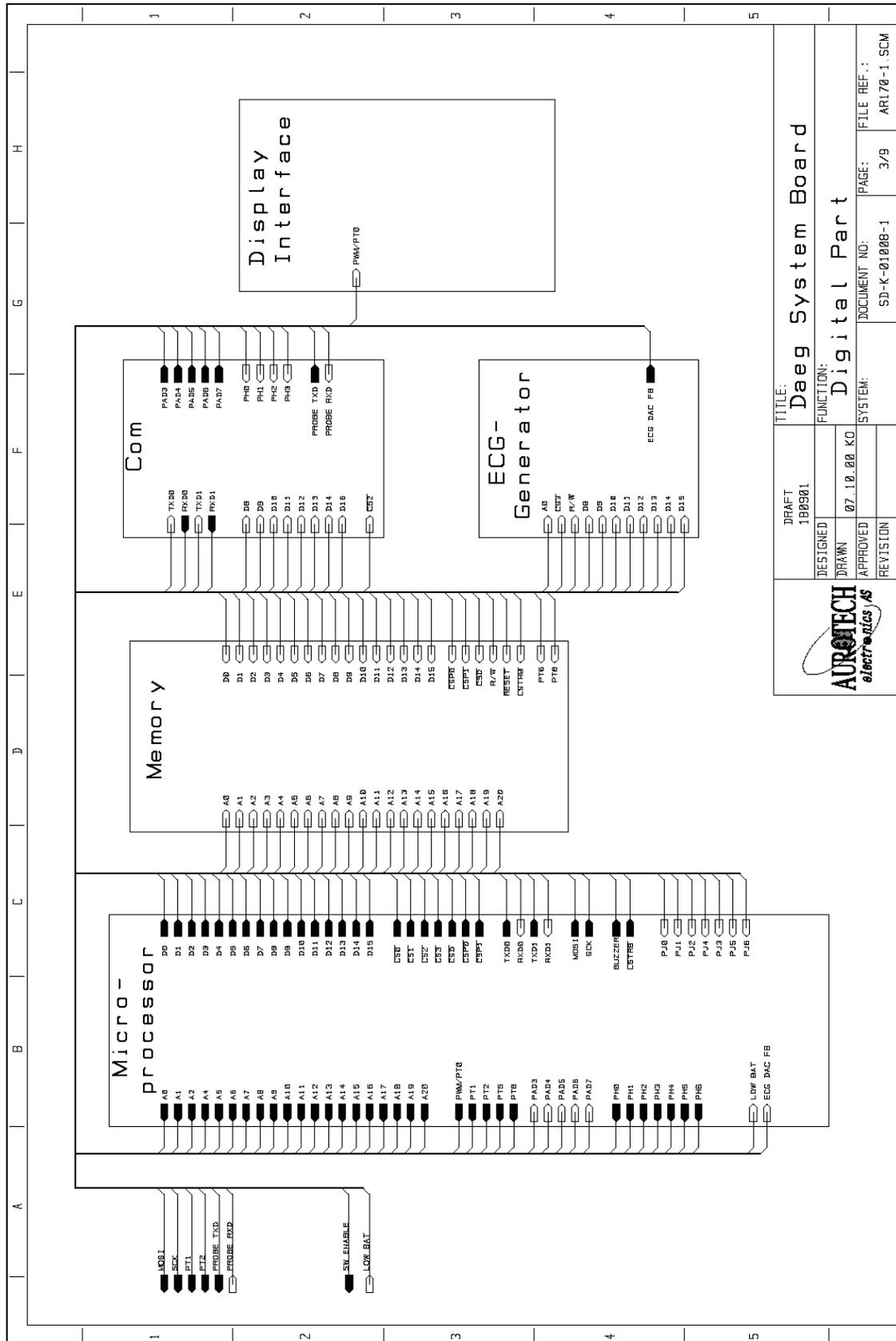

DRAFT 180901		TITLE: Daeg System Board	
DESIGNED	DRAWN	FUNCTION:	Digital Part
APPROVED	REVISION	SYSTEM:	SD-K-01008-1
		DOCUMENT NO:	3/9
		FILE REF.:	ARI70-1, SCM






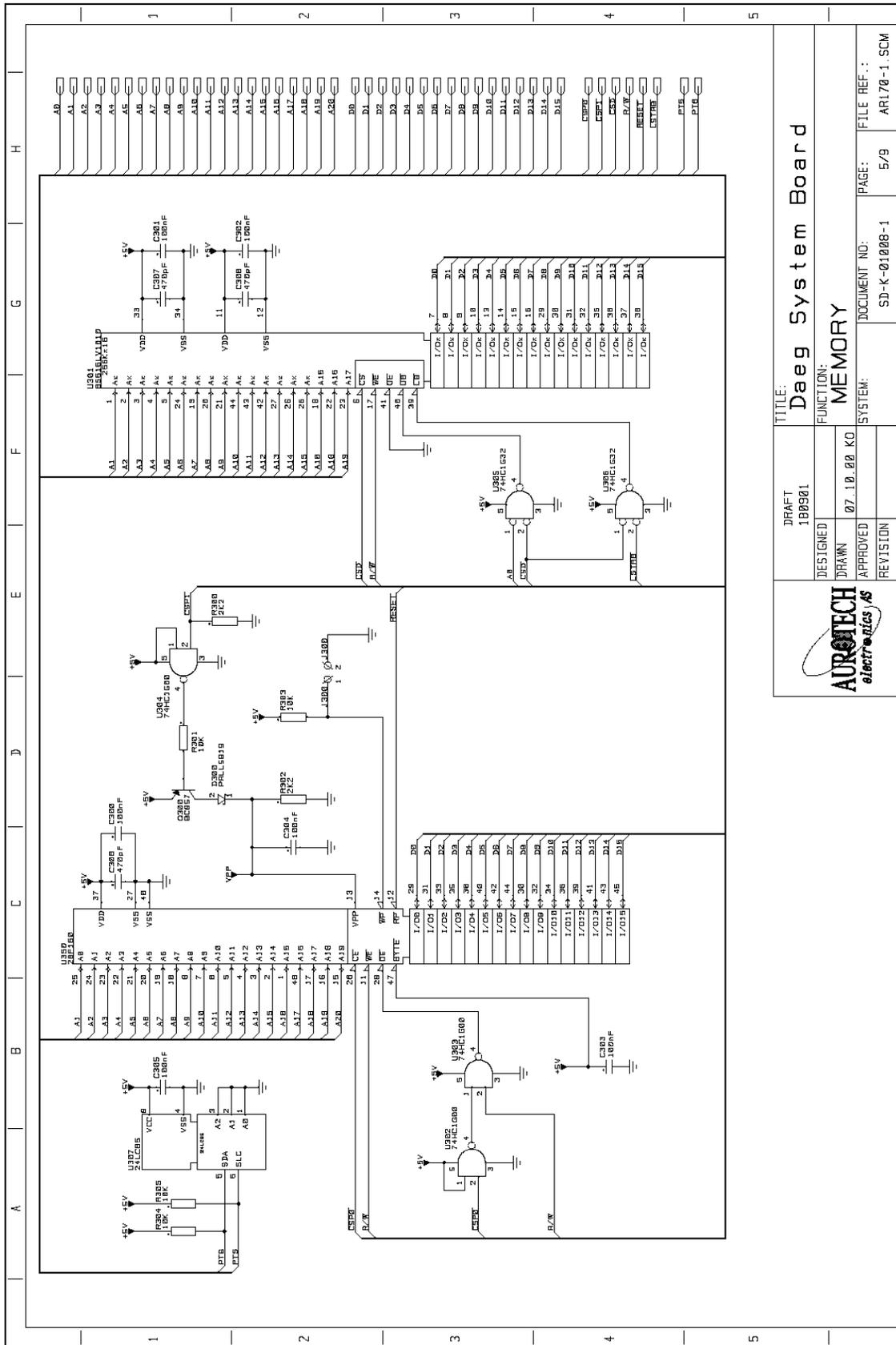


TITLE: Daeg System Board	
DRAFT 100901	FUNCTION: MEMORY
DESIGNED	APPROVED
DRAWN 07.10.00 KO	REVISION
SYSTEM: SD-K-01008-1	DOCUMENT NO: 5/9
FILE REF.: ARI70-1.SCM	






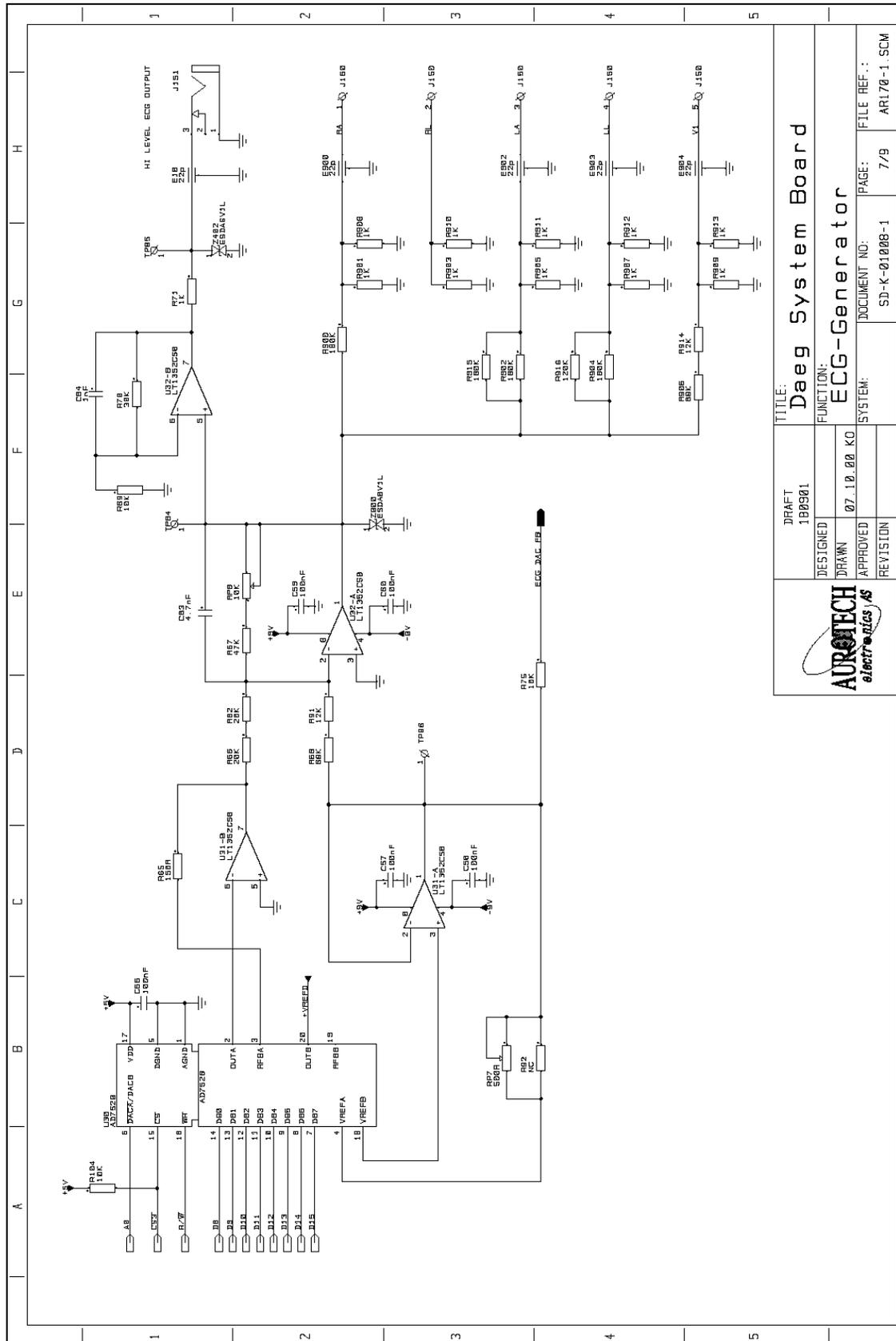

<b>AURATECH</b> <i>electronics</i>		DRAFT 100901	TITLE: <b>Daeg System Board</b>
DESIGNED	07.10.00 KO	FUNCTION:	<b>ECG-Generator</b>
APPROVED		SYSTEM:	SD-K-01008-1
REVISION		DOCUMENT NO:	SD-K-01008-1
		PAGE:	7/9
		FILE REF.:	ARI70-1.SCM

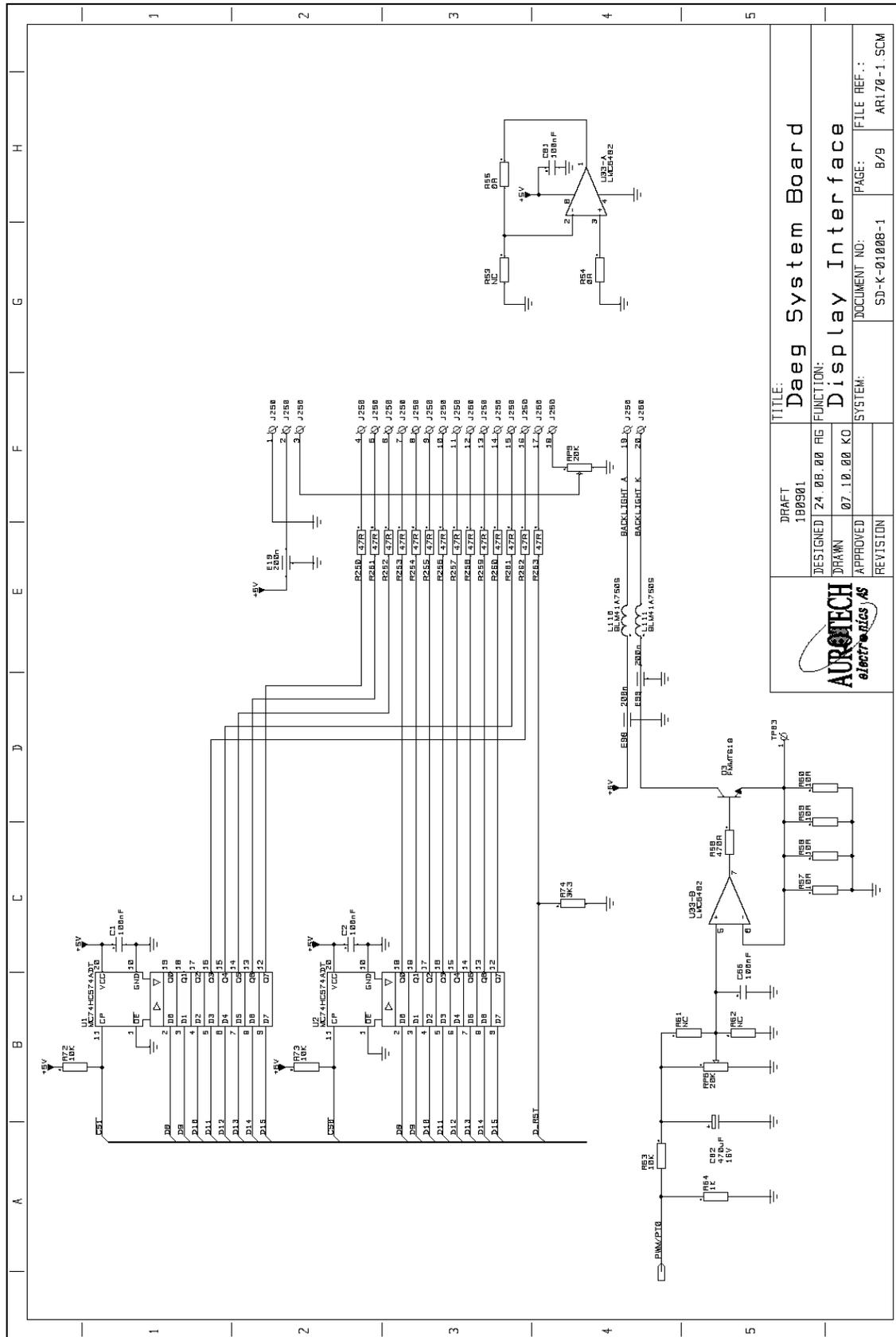

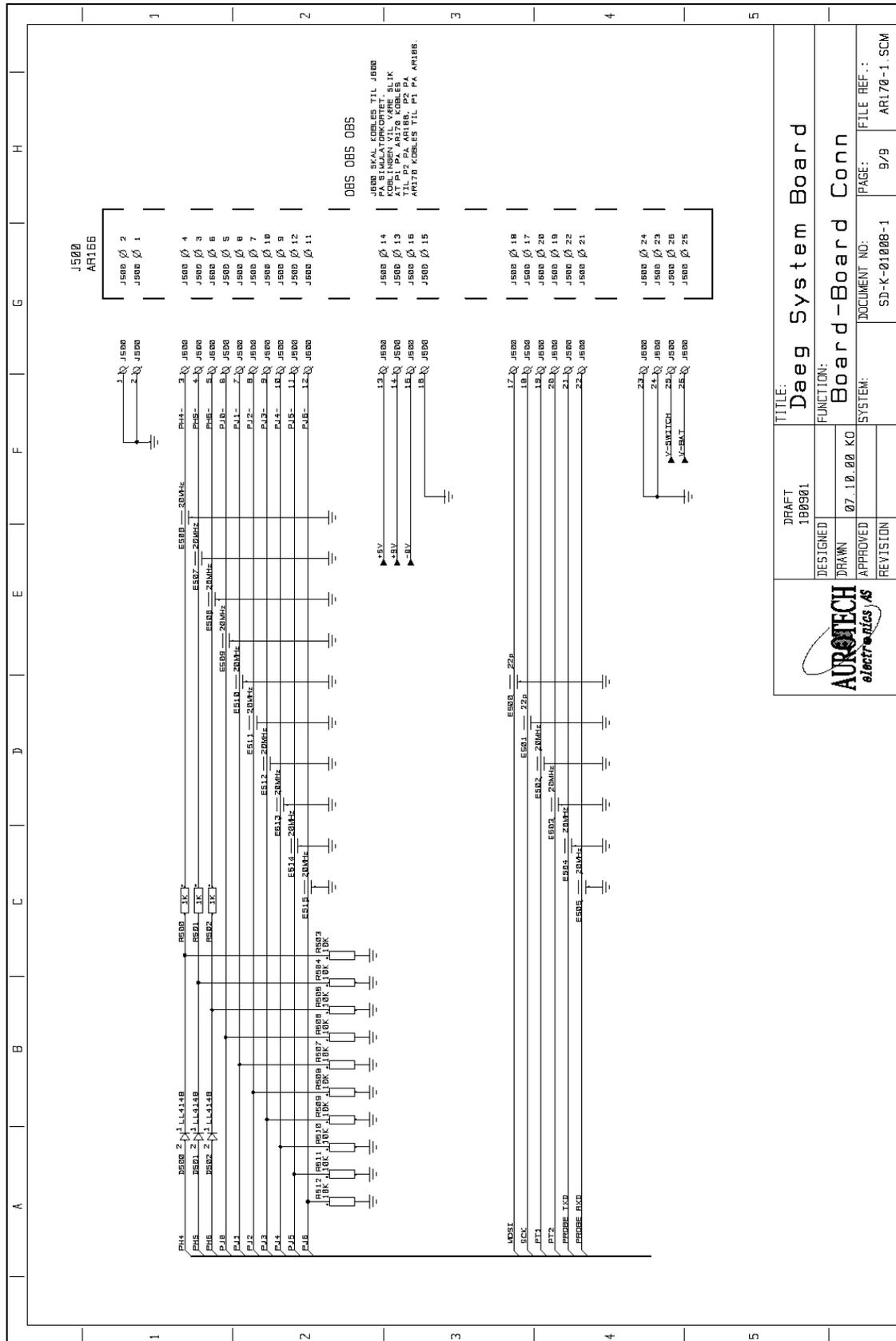

DRAFT 180901		TITLE: Daeg System Board	
DESIGNED		FUNCTION:	
DRAWN		Board-Board Conn	
APPROVED		SYSTEM:	
REVISION		DOCUMENT NO: SD-K-01008-1	
		PAGE: 9/9	
		FILE REF.: ARI70-1.SCM	



