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



**Service Manual**



**Stand and Desktop Systems**



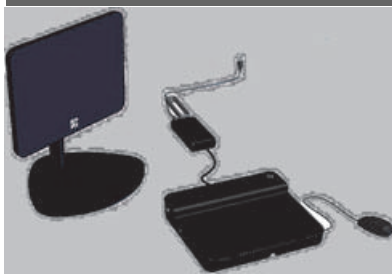

## **Legal Notices**

### **Manufacturer**

#### **FUJIFILM SonoSite, Inc.**

21919 30th Drive SE  
Bothell, WA 98021-3904  
USA

T: (888) 482-9449 or (425) 951-1200

F: (425) 951-1201

### **EC Authorized Representative**

#### **Emergo Europe**

Molenstraat 15  
2513 BH The Hague, The Netherlands

### **Australia Sponsor**

#### **FUJIFILM SonoSite Australasia Pty Ltd**

Suite 9, 13a Narabang Way  
Belrose, New South Wales 2085, Australia

**Caution:** | United States federal law restricts this device to sale by or on the order of a physician.

SonoMB, SonoSite, Steep Needle Profiling, X-Porte, and the SonoSite logo are registered (in some jurisdictions) and unregistered trademarks owned by FUJIFILM SonoSite, Inc. in various jurisdictions.

DICOM is the registered trademark of the National Electrical Manufacturers Association.

All other trademarks are the property of their respective owners.

Patents: US 8,216,146; US 8,213,467; US 8,147,408; US 8,137,278; US 8,088,071; US 8,066,642; US 8,052,606; US 7,819,807; US 7,804,970; US 7,740,586; US 7,686,766; US 7,604,596; US 7,591,786; US 7,588,541; US 7,534,211; US 7,449,640; US 7,169,108; US 6,962,566; US 6,648,826; US 6,575,908; US 6,569,101; US 6,471,651; US 6,416,475; US 6,383,139; US 6,364,839; US 6,203,498; US 6,135,961; US 5,893,363; US 5,817,024; US 5,782,769; US 5,722,412; AU: 730822; AU: 727381; CA: 2,372,152; CA: 2,371,711; CN 98108973.9; CN: 98106133.8; CN: 97113678.5; DE: 69831698.3; DE: 69830539.6; DE: 69730563.5; DE: 602004027882.3; DE: 602004023816.3; DE: 60034670.6; DE: 60029777.2; EP: 1589878; EP: 1552792; EP: 1180971; EP: 0875203; EP: 0815793; EP 1180970; EP 0881492; ES: 2229318; ES: 159878; ES: 1552792; ES: 0881492; FR: 158978; FR: 1552792; FR: 1180970; FR: 0881492; FR: 0875203; FR: 0815793; GB: 158978; GB: 1552792; GB: 1180971; GB: 1180970; GB: 0881492; GB: 0875203; GB: 0815793; IT: 1589878; IT: 1552792; IT: 0881492; IT: 0815793; JP: 4696150; KR: 532359; KR: 528102; NO: 326814; NO: 326202

Copyright © 2013 FUJIFILM SonoSite Inc, Inc. All rights reserved.

P16620-01





# Table of Contents

<b>Introduction .....</b>	<b>1</b>
Audience .....	1
Contact Information .....	1
Terms and symbols .....	2
Labeling symbols .....	2
<b>Specifications .....</b>	<b>3</b>
Specifications .....	3
System Specifications .....	3
Electrical specifications .....	3
Environmental limits .....	4
Battery specifications .....	4
Accessories and peripherals .....	4
<b>Safety .....</b>	<b>5</b>
Electrical safety .....	5
Electrical safety classification .....	7
Equipment safety .....	7
DICOM standard .....	7
HIPAA standard .....	7
Transporting X-Porte Stand Systems .....	7
<b>System Overview .....</b>	<b>9</b>
About the System .....	9
Theory of Operation .....	9
Description of Operating Modes .....	10
Additional System Feature Performances .....	12
Front End Overview .....	13
X-Porte Front End Board .....	14
PW Doppler Processing .....	15
CW Doppler Processing .....	16
Back End Overview .....	17
Control Subsystem .....	18
Power Supply and Control .....	19
ECG Module .....	20
DICOM .....	21
<b>Troubleshooting .....</b>	<b>23</b>
X-Porte Desktop and Stand Versions Major Component Diagrams .....	23
System and Subsystem Diagnosis .....	23
System Repair .....	24
Test Equipment .....	24
Failure (Assert) Codes .....	24
Verifying a System Assert Code .....	24
Clinical Monitor Display .....	25
DICOM .....	26
SPMU .....	27
Touch Control Panel .....	29
X-Porte Power-0 & Power-1 Procedural Steps .....	29

## **Replacement Procedures ..... 33**

Qualifying Service Personnel .....	34
Clinical Monitor Display .....	34
Required Parts .....	34
Required Tools .....	34
Clinical Monitor Display Removal .....	34
Clinical Monitor Display Replacement .....	35
System (“Engine”) .....	36
Required Part .....	36
Required Tools .....	36
System (“Engine”) Removal .....	36
System (“Engine”) Replacement .....	37
Touch Control Panel .....	37
Required Parts .....	37
Required Tools .....	37
Control Panel Removal .....	37
Control Panel Replacement .....	39
Dock .....	39
Required Part .....	39
Required Tools .....	39
Dock Removal .....	39
Dock Replacement .....	40
Stand Power Management Unit (SPMU) .....	40
Required Part .....	40
Required Tools .....	40
Stand Power Management Unit (SPMU) Removal .....	40
Stand Power Management Unit (SPMU) Replacement .....	44
Battery .....	45
Required Part .....	45
Required Tools .....	45
Battery Removal .....	45
Battery Replacement .....	45
Triple Transducer Connect (TTC) .....	45
Required Part .....	45
Required Tools .....	45
Triple Transducer Connect (TTC) Removal .....	45
Triple Transducer Connect (TTC) Replacement .....	47
Optional Equipment - Stand System .....	48
PowerPark .....	48
Required Part .....	48
Required Tools .....	48
PowerPark Stand Module Removal .....	48
PowerPark Stand Module Replacement .....	48
PowerPark Stand Module Removal (Alternate Method-Upright System) .....	48
PowerPark Stand Module Replacement (Alternate Method-Upright System) .....	48
Sony UP-D897MD Printer .....	49
Required Part .....	49
Required Tools .....	49
Sony UP-D897MD Printer Removal .....	49
Sony UP-D897MD Printer Replacement .....	50

## **Maintenance ..... 51**

Periodic Maintenance .....	51
Cleaning and Disinfecting .....	51

## **Performance Testing ..... 53**

Overview .....	53
Recommend Test Equipment .....	53
Setting Up Performance Tests .....	53
Basic Operational Tests .....	54
2D Performance Tests .....	54
2D Performance / Image Quality .....	54
Axial Measurement Accuracy .....	55
Lateral Measurement Accuracy .....	55
Penetration .....	56
Additional Performance Tests .....	56
Color Doppler (Color) .....	56
Color Power Doppler (CPD) .....	57
M Mode Imaging .....	57
Tissue Harmonic Imaging .....	57
Pulsed Wave (PW) Doppler Imaging .....	58
Continuous Wave (CW) Doppler Imaging .....	58
Image Quality Verification Test/Livescan .....	58
Printer .....	59

## **Replacement Parts ..... 61**

Clinical Monitor .....	62
Touch Control Panel .....	63
Dock .....	64
“Ultrasound” Engine .....	65
Stand Power Management Unit (SPMU & Batteries) .....	66
Triple Transducer Connect (TTC) .....	67
Clinical Monitor Hardware .....	68
Control Panel Hardware .....	68
Dock Hardware .....	69
Stand Hardware Parts .....	69
Optional Equipment .....	70
PowerPark .....	70
Sony UP-D897MD B&W Printer .....	71
Desktop System Components .....	72
EPS for Desktop System (Engine) .....	72
EPS for Desktop Clinical Monitor Display .....	72
Other Desktop Components .....	73
Ordering Replacement Parts .....	73

## **Service Event Reporting ..... 75**

Service Event Report Form .....	76
Service Event Report Instructions .....	77
Returning Products to FUJIFILM SonoSite .....	78
Shipping Instructions .....	78





# Chapter 1: Introduction

Before servicing the X-Porte Ultrasound System, please read this manual.

The ultrasound system has multiple configurations and feature sets. All are described in this service manual but not every option may apply to your system. System features depend on your system configuration, transducer, and exam type.

Refer to the *X-Porte Product Information & Safety Guide (P14646)* for additional information regarding safety, system controls, operation, capabilities, and specifications.

## Audience

The intended audience of this manual are properly trained field and in-house service personnel. The ultrasound system operators are expected to be able to replace all the components of the Desktop version and the System "Engine" attached to the Stand version. With technical support guidance, a basic knowledge of hand tools, and the ability to handle about 20 lb (9.1 kg), the system operators are expected to replace the touch Control Panel and Clinical Monitor display. The Stand Power Management Unit (SPMU) and batteries in the Stand version will be serviced by qualified field service personnel. The batteries used are sealed Lithium-ion type which requires no maintenance and are not serviceable. Expected life for these batteries are between 3-6 years depending on system usage and configuration of the Stand system.

## Contact Information

Questions and comments are encouraged. FUJIFILM SonoSite is interested in your feedback regarding the system and user documentation. Please call FUJIFILM SonoSite at 888-482-9449 in the U.S. Outside the U.S., call the nearest FUJIFILM SonoSite representative.

For technical support, please contact FUJIFILM SonoSite as follows:

### FUJIFILM SonoSite Technical Support

Phone (U.S. or Canada):	877-657-8118
Phone (Outside U.S. and Canada):	425-951-1330 Or call your local representative.
Fax:	425-951-6700
E-mail:	service@sonosite.com
Web site:	www.sonosite.com
Europe Service Center:	FUJIFILM SonoSite - Amsterdam Science Park 402 1098 XH Amsterdam The Netherlands Tel (Main): +31 20 751 2020 - English support: +44 14 6234 1151 - French support: +33 1 8288 0702 - German support: +49 69 8088 4030 - Italian support: +39 02 9475 3655 - Spanish support: +34 91 123 8451

## FUJIFILM SonoSite Technical Support (Continued)

Asia  
Service  
Center:

SonoSite Singapore Pte Ltd  
10 New Industrial Road #02-00 Fujifilm  
Building  
Singapore 536501  
Tel: +65 6380-5589

## Terms and symbols

The service manual follows these conventions:

- A **WARNING** describes precautions necessary to prevent injury or loss of life.
- A **Caution** describes precautions necessary to protect the products.
- Numbered steps in procedures must be performed in order.
- Items in bulleted lists do not require performance in sequence.

## Labeling symbols

Please refer to Chapter 4: Labeling Symbols found in the *X-Porte Product Information & Safety Guide (P14646)*

# Chapter 2: Specifications

This chapter contains information regarding system specifications and accessory compatibility. The information applies to the ultrasound system, transducers, accessories, and peripherals.

## Specifications

### System Specifications

#### Stand Dimensions:

- **Length:** 26.4 in. (67.1 cm)
- **Width:** 21.2 in. (53.8 cm)
- **Height (max):** 64 in. (162.6 cm)
- **Height (min):** 42.2 in. (107.2 cm)
- **Height Adjustment:** 9 in. (22.9 cm) travel

#### Touch Panel

- **Capacitive Display Type**
- **Diagonal:** 12.1 in. (30.7 cm)
- **Multi-touch Gestures for System Controls**
- **Tilt Adjustment:** 7.3 in. (18.5 cm)
- **Side to Side Turning:** +/- 90 degrees from center

#### Display (HD Monitor)

- **Diagonal:** 19 in. (48.3 cm)
- **Screen Size:** 1280 x 800
- **Image Size:** 800 x 600
- **Monitor Tilt:** 5° tilting forward from vertical, 20° tilting back from vertical

### Electrical specifications

- **Power Supply Input (Stand Version):** 100-240V~ 6.0A Max, 50-60Hz
  - **Sony B/W Printer:** 100-240V~ 1.5A-0.8A, 50-60Hz
- **Power Supply Output 1 (Stand Version):** 24 VDC, 11.5 A Max (Output not exceeding 275 watts)
  - **Power Supply Output 2 (Stand Version):** 100-240V~ 3.0A - 1.5A, 50-60Hz
- **Power Supply Input (Desktop Version):** 100-240V~ 3.4A - 1.4A, 50-60Hz
- **Power Supply Output (Desktop Version):** 24 VDC, 6.25 A Max (Combined Output not exceeding 150 watts)
  - **Clinical Display Monitor Supply:** 100-240V~ 2.7A, 50-60Hz
  - **Sony B/W Printer:** 100-240V~ 1.5A-0.8A, 50-60Hz
- **Batteries (Stand Version):** 3 Lithium-ion batteries (385Wh total)
- **Battery Use Time:** 1.0 hour, 3 days on idle
- **Battery Charge Time:** 2.5 hours
- **Battery Life:** 3-6 years

## Environmental limits

*Note: The temperature, pressure, and humidity limits apply only to the ultrasound system, transducers, and battery.*

### Operating

#### **System and Transducer**

10–40°C (50–104°F), 15–95% R.H.

700 to 1060hPa (0.7 to 1.05 ATM)

### Shipping and Storage

#### **System and Transducer**

-35–65°C (-31–149°F), 15–95% R.H.

500 to 1060hPa (0.5 to 1.05 ATM)

#### **Battery**

-20–60°C (-4–140°F), 15–95% R.H. (For storage longer than 30 days, store at or below room temperature.)

500 to 1060hPa (0.5 to 1.05 ATM)

## Battery specifications

The X-Porte stand uses three lithium-ion batteries that are charged and controlled by the Stand Power Management Unit (SPMU) that can be switched off via three rocker type switches. These switches can be accessed by lifting up from the stand bottom a black plastic door located under the left rear caster leg.

Run time is up to one hour (up to 3days on idle) depending on imaging mode and display brightness. This chapter contains electrical, and clinical safety information required by regulatory agencies.

## Accessories and peripherals

The system supports various accessories and peripherals. See the *X-Porte Product Information & Safety Guide* (P14646) for a list of compatible products.

# Chapter 3: Safety

This chapter contains electrical and clinical safety information required by regulatory agencies. The information applies to the ultrasound system, transducers, accessories, and peripherals.

## Electrical safety

This system meets EN60601-1, Class I/internally-powered equipment requirements and Type BF (transducers) and Type CF (ECG leads) isolated patient-applied parts safety requirements.

This system complies with the applicable medical equipment requirements published in the Canadian Standards Association (CSA), European Norm Harmonized Standards, and Underwriters Laboratories (UL) safety standards.

For maximum safety observe the following warnings and cautions..

- |                 |  |
|-----------------|--|
| <b>WARNING:</b> | To avoid the risk of injury, do not operate the system in the presence of flammable gasses or anesthetics. Explosion can result.   |
| <b>WARNING:</b> | To avoid the risk of electrical shock or injury, do not open the system enclosures. All internal adjustments and replacements, including battery replacement, must be made by a qualified technician.  |
| <b>WARNING:</b> | <p>To avoid the risk of electrical shock:</p> <ul style="list-style-type: none"><li>• Use only properly grounded equipment. Shock hazards exist if the power supply is not properly grounded. Grounding reliability can be achieved only when equipment is connected to a receptacle marked “Hospital Only” or “Hospital Grade” or equivalent. The grounding wire must not be removed or defeated.</li><li>• This equipment must be connected only to supply mains with protective earth.</li><li>• Do not let any part of the system (including the bar code scanner, external mouse, power supply, or power supply connector), except for the transducer or ECG leads, touch the patient.</li><li>• Do not touch any of the following:<ul style="list-style-type: none"><li>• The ungrounded signal input/output connectors on the back of the ultrasound system</li><li>• The system battery contacts (Stand System Only)</li><li>• The system transducer connector when the transducer or Triple Transducer Connect (TTC) is disconnected</li><li>• Any unused TTC transducer connector when the TTC is connected.</li></ul></li><li>• Do not connect the system's power supply to an MPSO or extension cord.</li><li>• Before using the transducer, inspect the transducer face, housing, and cable. Do not use the transducer if the transducer or cable is damaged.</li><li>• Always disconnect the power supply from the system before cleaning the system.</li><li>• Do not use any transducer that has been immersed beyond the specified cleaning or disinfection level. See “Cleaning and Disinfecting X-Porte Products” P16529</li><li>• Use only accessories and peripherals recommended by FUJIFILM SonoSite, including the power supply. Connection of accessories and peripherals not recommended by FUJIFILM SonoSite could result in electrical shock. Contact FUJIFILM SonoSite or your local representative for a list of accessories and peripherals available from or recommended by FUJIFILM SonoSite.</li></ul> |

- WARNING:** To avoid the risk of electrical shock and fire hazard:
- Inspect the power supply, AC power cords, cables, and plugs on a regular basis. Ensure that they are not damaged.
  - The power cord set that connects the power supply of the ultrasound system or to mains power must only be used with the power supply, and cannot be used to connect other devices to mains power.
- WARNING:** To avoid the risk of electrical shock, burn, or fire, use only AC power supplies specified by FUJIFILM SonoSite.
- WARNING:** To prevent injury to the operator/bystander, the transducer must be removed from patient contact before the application of a high-voltage defibrillation pulse.
- WARNING:** To avoid possible electrical shock or electromagnetic interference, verify proper operation and compliance with relevant safety standards for all equipment before clinical use. Connecting additional equipment to the ultrasound system constitutes configuring a medical system. FUJIFILM SonoSite recommends verifying that the system, all combinations of equipment, and accessories connected to the ultrasound system comply with JACHO installation requirements and/or safety standards such as AAMI-ES1, NFPA 99 OR IEC Standard 60601-1-1 and electromagnetic compatibility standard IEC 60601-1-2 (Electromagnetic compatibility), and are certified according to IEC Standard 60950 (Information Technology Equipment (ITE) or IEC 60601-1.).
- WARNING:** Due to the captive nature of the Stand version system batteries, any shipping transport, service activities, or extended periods of non-operation with the system unplugged from the AC wall source, will require that the battery ON-OFF switch be set to the “•O” (OFF) position. Please refer to [Figure 3.1 on page 6](#).

- Caution:** Do not use the system if an error message appears on the display monitor: note the error code; call FUJIFILM SonoSite or your local representative; turn off the system by pressing and holding the power key until the system powers down.
- Caution:** To avoid increasing the system and transducer connector temperature, do not block the airflow to the rear fans and ventilation holes around the bottom and side perimeter of the Engine.

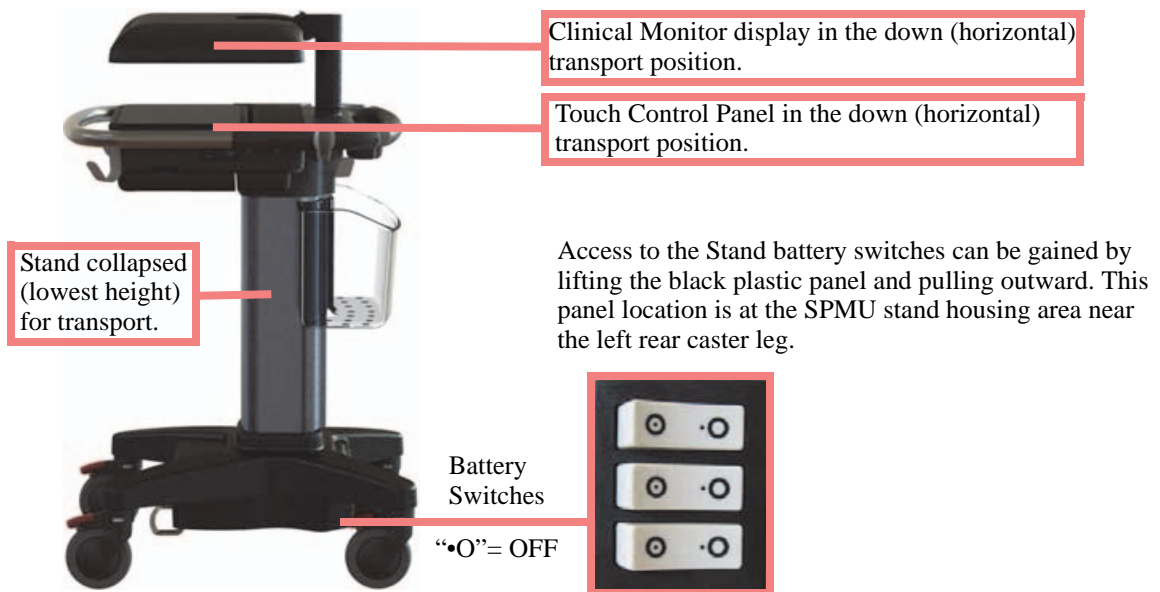


Figure 3.1 Stand System Battery Switches & Transport Positioning

# Electrical safety classification

## Equipment safety

Please refer to the *X-Porte Product Information and Safety Guide* (P14646) for the complete information concerning all safety and standards applicable to the X-Porte Ultrasound System

### DICOM standard

Please refer to the *X-Porte Product Information and Safety Guide* (P14646) for the complete information concerning the DICOM standard applicable to the X-Porte Ultrasound System

### HIPAA standard

Please refer to the *X-Porte Product Information and Safety Guide* (P14646) for the complete information concerning HIPAA standard applicable to the X-Porte Ultrasound System

## Transporting X-Porte Stand Systems

Please observe the prior Warnings and Cautions outlined in this chapter concerning the stand version system for transport and servicing. Lower (collapse) the stand to the lowest height position, fold to the horizontal position the Clinical Monitor display and touch Control Panel, and access the three battery switches and ensure that all switches are in the “•O” (OFF) position. Many government agencies require that no electronic equipment be shipped via air transport with live battery power attached to the equipment. Refer to [Figure 3.1 on page 6](#) as to how to position the system stand and disconnect the battery power for transport and servicing.







The **Transducer** elements convert the pulser voltage to acoustic energy during the transmit portion of the ultrasound acquisition cycle. The elements convert the acoustic echo to voltage in the receive portion of the acquisition. The voltage developed on the transducer elements is sensed by the acquisition subsystem. The system transducers have 64 to 256 elements.

The **Acquisition Subsystem** consists of the beamformer and interface to the transducer. The beamformer controls the timing of the transmit pulses to focus the acoustic beam. The beamformer amplifies the low-level received echos and controls the receive focusing. The system beamformer transmits on up to 128 elements and receives on 64 elements.

The **Processing Subsystem** includes capabilities for interfacing with the beamformer and performing high speed processing. The processing subsystem demodulates, filters, detects, and compresses the signal supplied by the beamformer into display information.

The **Display Subsystem** converts the detected ultrasound data into picture elements (pixels). The software user interface graphics are combined with the ultrasound information and converted to a video stream. The external video port supports NTSC and PAL format.

The **Control Subsystem** consists of the central processing unit, program and video memory, permanent image storage and retrieval memory, external communication interface ports, and connection to the user interface keys. The control software includes the acoustic power and intensity software subsystem, power group monitors, and a beamformer monitor. This software guarantees a level of patient safety by ensuring the system is operating within acoustic power and intensity limits.

The **User Interface Subsystem** represents the software interface and form factor. The software interface is the interaction between the user and the screen layout components. The form factor is the type of physical buttons, location, and grouping of the buttons and the device size, shape, and weight. Dedicated controls are for high usage activities and grouped according to the user workflow.

The **Power Subsystem** provides the system power and protects the hardware from destructive and/or unsafe conditions by detecting failures in the system through hardware and software monitors. Detection of a fault results in disabling of the pulser supply, and signaling of an error to the Control Group. The power subsystem includes the batteries (quantity of three lithium-ion) and battery charging electronics.

## Description of Operating Modes

- 2D Mode** 2D mode is a two dimensional image of the amplitude of the echo signal. It is used for location and measurement of anatomical structures and for spatial orientation during operation of other modes. In 2D, a two-dimensional cross-section of a 3-dimensional soft tissue structure such as the heart is displayed in real time. Ultrasound echoes of different intensities are mapped to different gray scale or color values in the display. The outline of the 2D cross-section may be a rectangle, parallelogram, trapezoid, sector, or a full circle, depending on the particular transducer used. 2D mode can be used in combination with any other modes.
- M Mode** M Mode is also known as "T-M mode" or "time-motion" mode. It is used primarily for cardiac measurements such as valve timing and septal wall thickness when accurate timing information is required.
- Ultrasound echoes of different intensities are mapped to different gray scale values in a scrolling display. M Mode displays time motion information of the ultrasound data derived from a stationary beam. Depth is arranged along the vertical axis with time along the horizontal axis. M Mode can be used alone but is normally used in conjunction with a 2D image for spatial reference. The 2D image has a graphical line (M-line) superimposed on the 2D image indicating where the M Mode beam is located.

**Color Doppler (Color)**

In color Doppler, a real-time, two-dimensional cross-section of blood flow is displayed. The 2D cross-section may be presented as a rectangle, parallelogram, trapezoid, sector, or a full circle, depending on the particular transducer used.

The 2D cross-section is presented as a full color display, with various colors being used to represent the velocity, both positive and negative, of the blood flow echoes. Often, to provide spatial orientation, the full color blood flow cross-section is overlaid on top of the gray scale cross-section of soft tissue structure (2D echo). For each pixel in the overlay, the decision of whether to display VCD, gray scale (echo) information or a blended combination is based on the relative strength of echoes from the soft-tissue structures and from the red blood battery.

A high pass filter (wall filter) is used to remove the signals from stationary or slowly moving structures. Tissue motion is discriminated from blood flow by assuming that blood is moving faster than the surrounding tissue, although additional parameters may also be used to enhance the discrimination. The remaining signal after wall filtering may be averaged over time (persistence) to present a steady state image of blood flow distribution. Variance information may also be displayed to provide information when large variance is observed in the velocity information.

**Color Power Doppler (CPD)**

In CPD, a real-time two-dimensional cross-section of blood flow is displayed. The 2D cross-section may be presented as a rectangle, parallelogram, trapezoid, sector, or a full circle, depending on the particular transducer used.

The 2D cross-section is presented as a full color display, with various colors being used to represent the power in blood flow echoes. Often, to provide spatial orientation, the full color blood flow cross-section is overlaid on top of the gray scale cross-section of soft tissue structure (2D echo). For each pixel in the overlay, the decision of whether to display CPD, gray scale (echo) information or a blended combination is based on the relative strength of echoes from the soft-tissue structures and from the red blood battery.

A high pass filter (wall filter) is used to remove the signals from stationary or slowly moving structures. Tissue motion is discriminated from blood flow by assuming that blood is moving faster than the surrounding tissue, although additional parameters may also be used to enhance the discrimination. The power in the remaining signal after wall filtering may be averaged over time (persistence) to present a steady state image of blood flow distribution.

**Continuous Wave (CW) Doppler**

CW provides a real-time representation of blood flow and is displayed as a velocity-versus-time sweeping output. Velocity (or frequency) is presented as the vertical axis with time along the horizontal axis. The magnitude of the detected signal is represented as different gray scale values.

CW Doppler mode provides the clinician with the ability to obtain blood flow velocities focused about a user specified focal region. A continuous transmit waveform of ultrasound energy with a known frequency is transmitted and focused by the system; on the receive side, the transducer receive echoes are continuously amplified, focused about the focal region and converted to a base band quadrature signal. The signal is analyzed by a quadrature phase detector that establishes two receive channels to allow detection of flow direction. These two channels are then analyzed by a fast complex Fourier transform (FFT) circuit to establish the spectrum of frequencies present in the echoes. The data are displayed as spectrum frequencies with respect to time.

CW can be used alone but is normally used in conjunction with a 2D image for spatial reference. The 2D image has a graphical line (D-line) superimposed on the 2D image indicating where the M-mode beam is located.

**Pulsed Wave (PW) Doppler** PW provides a real-time representation of blood flow and is displayed as a velocity-versus-time sweeping output. Velocity (or frequency) is presented as the vertical axis with time along the horizontal axis. The magnitude of the detected signal is represented as different gray scale values. The ultrasound data is derived from a single area, the sample volume, on a stationary beam.

PW Doppler mode provides the clinician with the ability to obtain blood flow velocities about a spatial sample volume. A burst of ultrasound with a known spectrum is transmitted by the system; on the receive side, the transducer receive echoes are amplified and range gated at the appropriate depth. The signal is analyzed by a quadrature phase detector that establishes two receive channels to allow detection of flow direction. These two channels are then analyzed by a fast complex Fourier transform (FFT) circuit to establish the spectrum of frequencies present in the echoes. The data are displayed as spectrum frequencies with respect to time.

PW can be used alone but is normally used in conjunction with a 2D image for spatial reference. The 2D image has a graphical line (D-line) superimposed on the 2D image indicating where the M-mode beam is located. The sample volume position (depth) and size are also indicated on the D-Line.

## Additional System Feature Performances

**Broadband Imaging** This ultrasound acquisition system uses high resolution broadband technology in the transmit pulsers, transducer, and receivers. The receive path can capture and process signals over a wide spectrum, from below 2.0 MHz to beyond 10 MHz. For each application, the transmit pulse is designed to produce an appropriate bandwidth. For example, in 2D grayscale imaging, a wide band pulse is used to support good axial resolution. For Doppler modes, a narrower band pulse is used, which improves the spectral resolution of the detected Doppler signal.

In addition to transmit pulse control, programmable digital signal processing is used in the receive path to further refine the bandwidth used to produce the final image. Digital filters are applied to the digitized received signal to limit and shape the spectral bandwidth used to generate the displayed output.

**Tissue Specific Imaging** In this feature, parameters for signal and image processing are optimized to maximize the image quality or to obtain the best compromise of resolution and penetration for different specific clinical applications. These parameters include: the order of received filters, the bandwidth, the dynamic range, the compression curve, the gain setting and parameters for compounding frequency band, etc. For example, different system parameter setups are used for abdominal or peritoneal scanning. This feature is for ease of use for the operator by automatically setting up system control parameters rather than manually adjusting settings for best performance.

**Biopsy Guidance** The system can display a pair of biopsy guidelines that represent the anticipated path of the biopsy needle. The image of an anatomical target, biopsy guidelines, a scan plane marker, and a biopsy needle are displayed to assist in guiding the biopsy needle to the target. The system also provides needle guidance for vascular access procedures. For additional information, see the biopsy user guides.

**Measurement and Calculation Capabilities** The system offers a variety of measurements and calculations, specific to exam type and transducer. A list of them, and author references, are in the system user guide. Measurement accuracy is also discussed.

**Continuous Wave Doppler Audio Output**

The system provides for audio output of the CW velocity information. This can be presented as stereo information, with flow moving towards the transducer on one channel and flow away on the other, or as a mono output with the single audio output representing the summation of the flow directions.

**Pulsed Wave Doppler Audio Output**

The system provides for audio output of the PW velocity information. This can be presented as stereo information, with flow moving towards the transducer on one channel and flow away on the other, or as a mono output with the single audio output representing the summation of the flow directions.

**Electrocardiograph (ECG) Display**

ECG is provided to measure the electrical signal generated by the heart. A three lead interface: Right Arm (RA), Left Arm (LA) and Left Leg (LL), is provided on the system.

The ECG signal is displayed as an amplitude-versus-time sweeping output. Amplitude is presented on the vertical axis with time along the horizontal axis.

**Front End Overview**

The Front End is designed to support various imaging modalities such as 2D, M-Mode, Spectral Doppler and Color Doppler. From the Front End's perspective, all modes can be grouped into a few basic types: Single mode, simultaneous modes and triggered modes. All these modes are built from similar, basic transmit and receive sequences controlled within the Front End. A generic top level block diagram of a typical Front End is shown below in [Figure 4.2 on page 13](#)

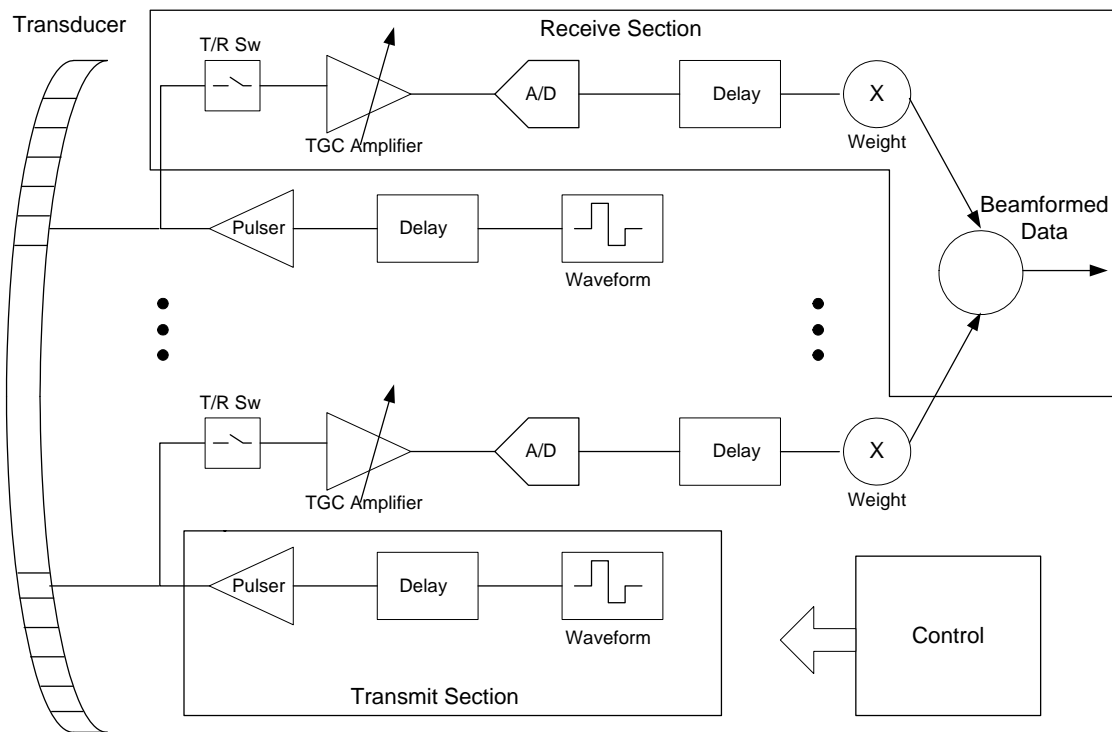


Figure 4.2 Generic Front End Block Diagram

The transmit section consists of a waveform generator, delay block, and high power high voltage driver to excite the transducer element. Multiple elements are driven with delays determined by the time of flight in the medium from the elements to the point in space where the beam is to be focused. The longer the time of flight is to the focal point the smaller the delay is for a given transmit element to allow all to arrive at the focal point at the same time.

The number of elements driven is determined by element sensitivity off axis and depth of field considerations. The waveform is selected to drive the transducer at a certain center frequency, bandwidth, and power and is optimized for the given mode.

The receive section consists of a transmit/receive switch to protect the receiver from the transmit voltage, a variable gain receiver to amplify and condition the return echoes, an A/D to digitize the data, a delay block to focus the return signals and a weight block to scale the return echoes for each channel. All the signals are then summed together to generate the beam-formed receive data. The analog gain varies with depth to compensate for signal attenuation through the medium. The delays and weights are independent for each channel. The delay and weight for the receive channel can typically be changed dynamically to keep the receive beam in continuous focus. The delay is simply set by the time of flight in the medium from the point of interest to the element, which starts at skin-line and proceeds to the deepest depth of interest.

The control section drives the data to the various data path elements on a line by line basis, controls the timing of the transmit and receive sections and controls the tagged information and timing of the data to the rest of the system.

## X-Porte Front End Board

The X-Porte ultrasound system can be divide into two board section groups: Front End and Back End. Below is the block diagram linking the necessary functional groups of an ultrasound system as they apply to the X-Porte system. Please refer to [Figure 4.3 on page 14](#)

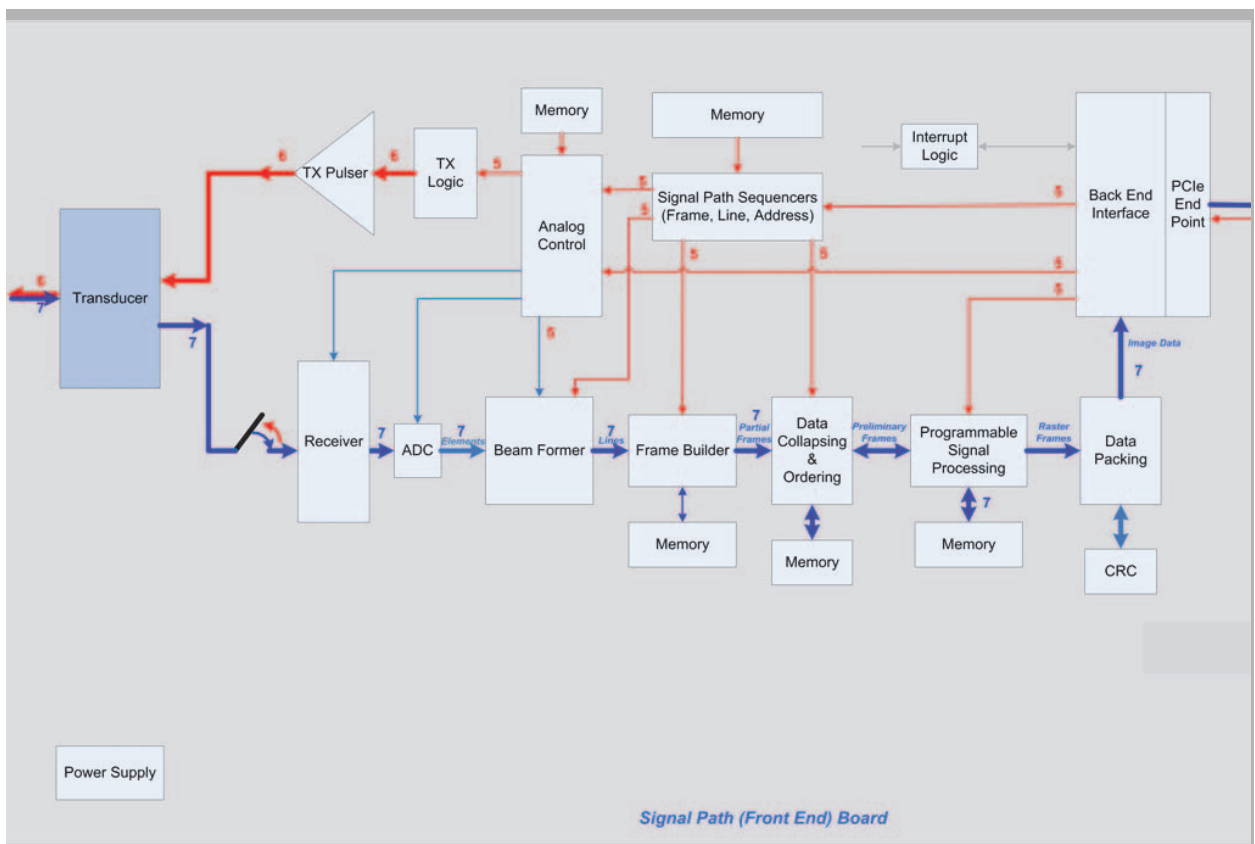


Figure 4.3 X-Porte Front End Board Block Diagram

## PW Doppler Processing

Doppler processing includes both audio processing which presents Doppler signal in the form of stereo audio and spectral processing which generates data for display of Doppler spectrum in the form of a scrolling spectrogram. Doppler power spectrum is estimated performing Discrete Fourier Transforms on short, overlapped segments of wall filtered Doppler signal. Doppler audio data is generated from wall filtered data by phase shifting the in-phase component. Refer to the PW Doppler Processing Function Block Diagram [Figure 4.4 on page 15](#)

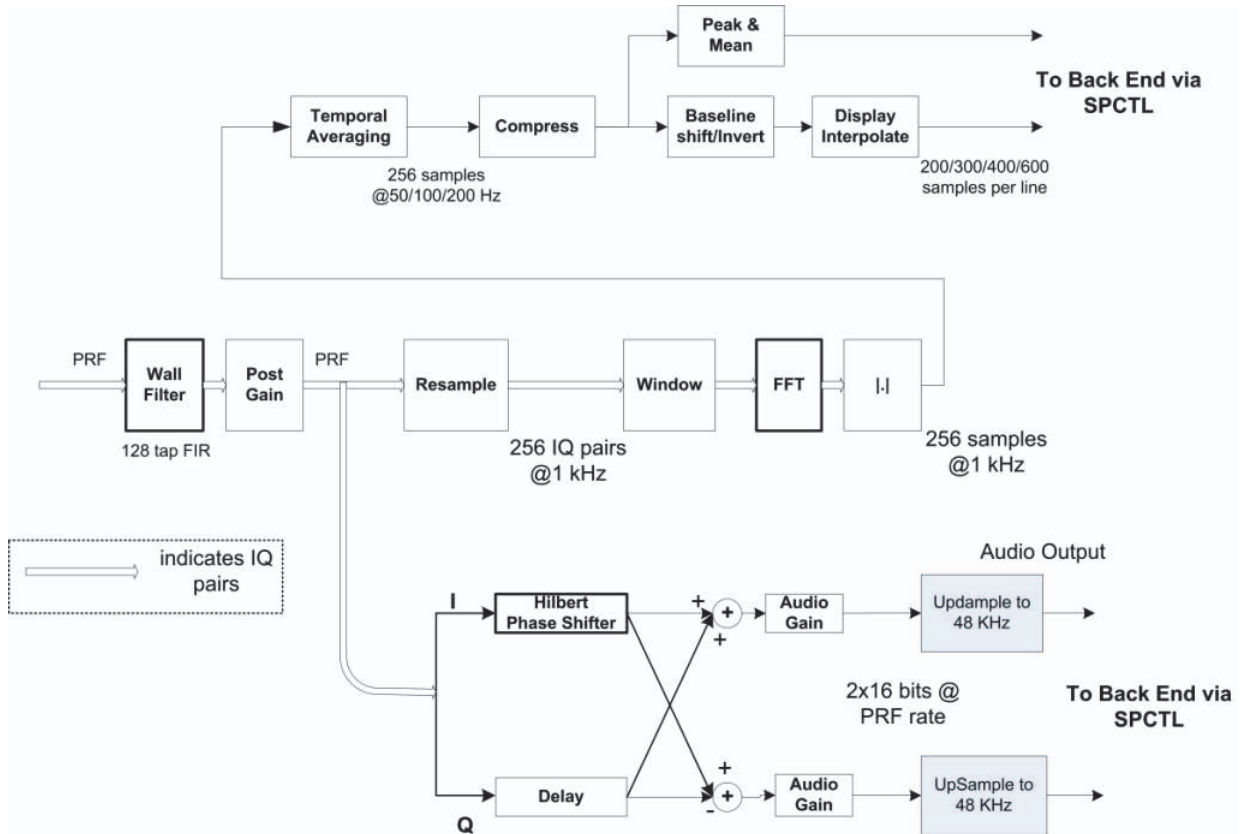


Figure 4.4 PW Doppler Processing Function Block Diagram

## CW Doppler Processing

CW Doppler data will be presented to the signal processor as complex (I/Q) data from the analog front end of the external DSP. The 20-bit data will be presented as consecutive samples at a data rate varying from 1.5 kHz to 64 kHz for the complex pair. Most of CW processing is similar to that of PW except for the QBP function. In place of QBP will be a low pass decimating filter that operates on incoming I/Q data.

The Doppler Processing block must allow storage of 256 undetected I/Q pairs in to allow the system to measure and correct for phase mismatch. Measuring and correcting will need to be accomplished in system software. Refer to the CW Doppler Processing Function Block Diagram [Figure 4.5 on page 16](#)

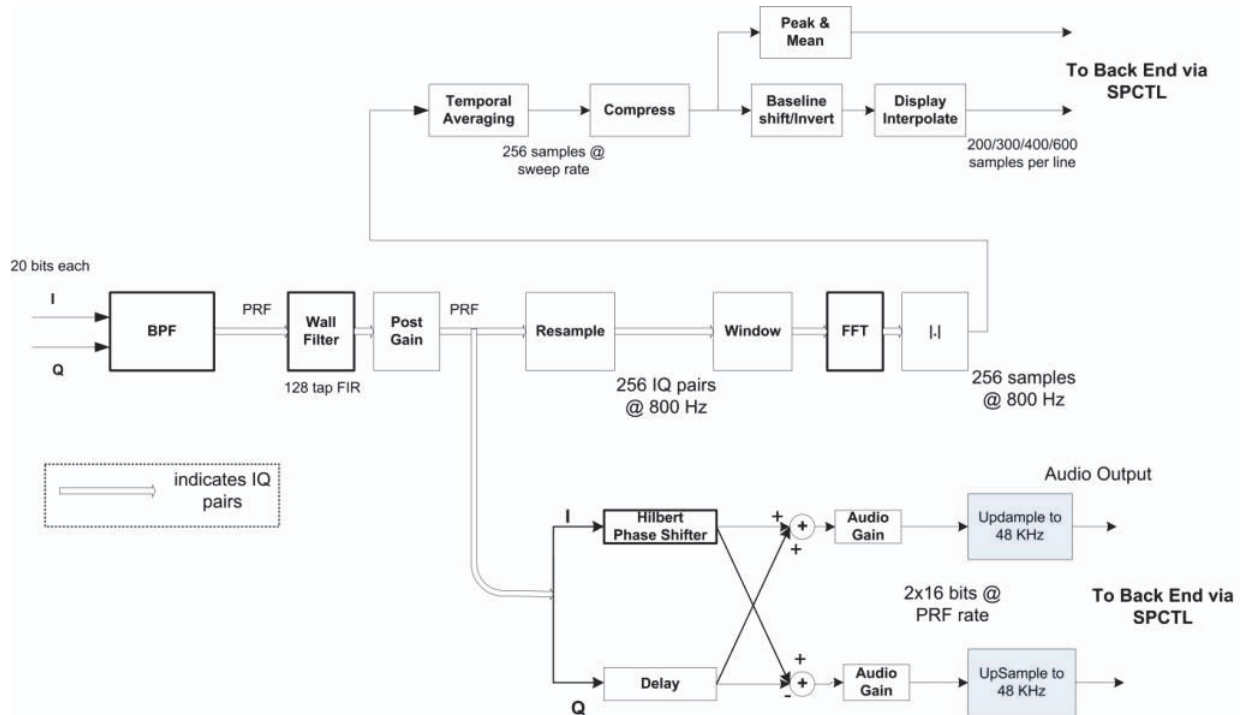


Figure 4.5 CW Doppler Processing Function Block Diagram



## Back End Overview

The Back End subsystem is responsible for the conversion of raw acquisition data into a raster image ready for display. The Back End subsystem also contains the video data path that supports generation of video comprising of the ultrasound image as well as graphics annotation. Video generation of both standard composite interlaced video and progressive scan video is supported. Most functionality is within the ASIC but the memory resources for acquisition memory, and display memory are found in external memory components. The conversion from PC type video to TV type video is also performed externally.

Control is received initially from the CPU to setup each functional block and afterward the hardware is completely data driven. This control takes the form of programming setup registers inside the blocks and setting up scan conversion tables. Each block provides temporary storage as required to buffer data and keep their respective processing pipeline full and operating. Also note that the block diagrams show only the data path, but each block is responsible for generating any necessary memory addresses for their respective input data stream.

The X-Porte Back End subsystem is shown in [Figure 4.6 on page 17](#).

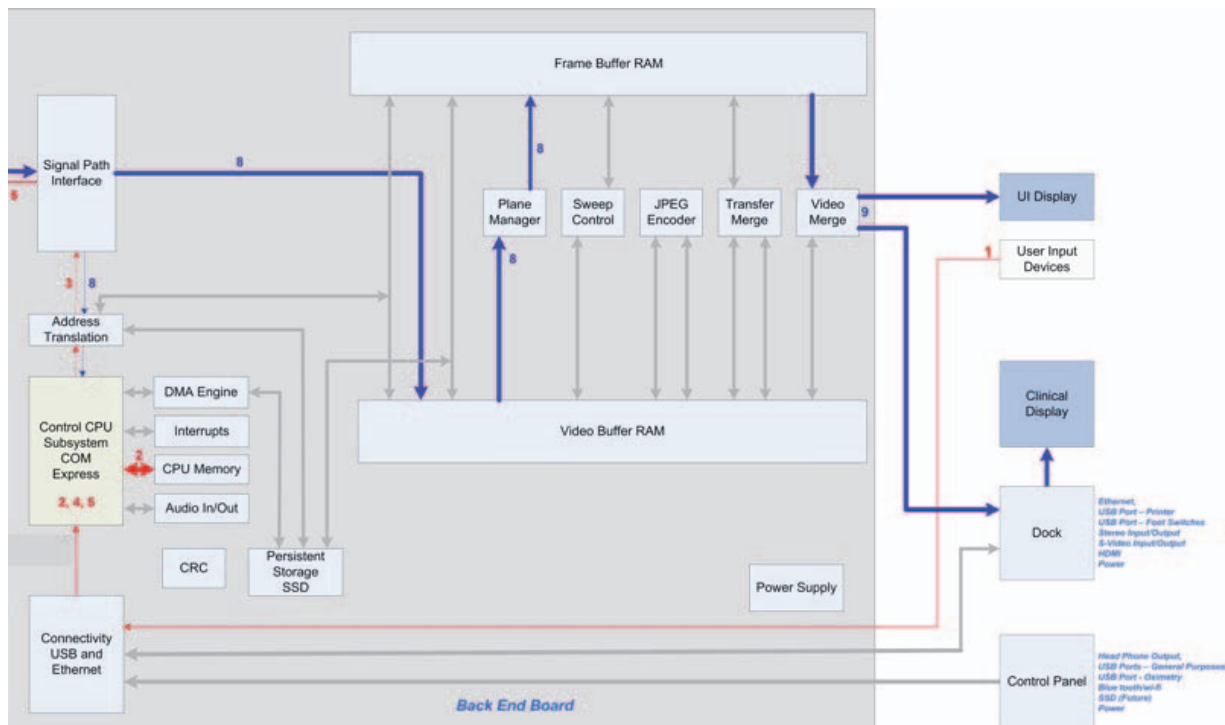


Figure 4.6 X-Porte System Back End Subsystem Block Diagram

The Back End Subsystem performs processing encompassing three main data domains, acquisition data, raster data, and video data. Support for acquisition data includes the input buffer, flash suppression, frame average, and external ACQ memory. Cine buffer management is performed by the acquisition controller. Conversion from acquisition data to raster data is performed by the graphics overlay, scan conversion system, sweeping system, and 3D system. Raster data is stored in an external DISPLAY memory. Also supporting raster operations is the graphics support block that provides acceleration hardware for pixel operations from the CPU and graphics overlay system. The progressive video path includes buffers, priority logic, and LUTs.

# Control Subsystem

The X-Porte Control Subsystem is shown in the figure below.

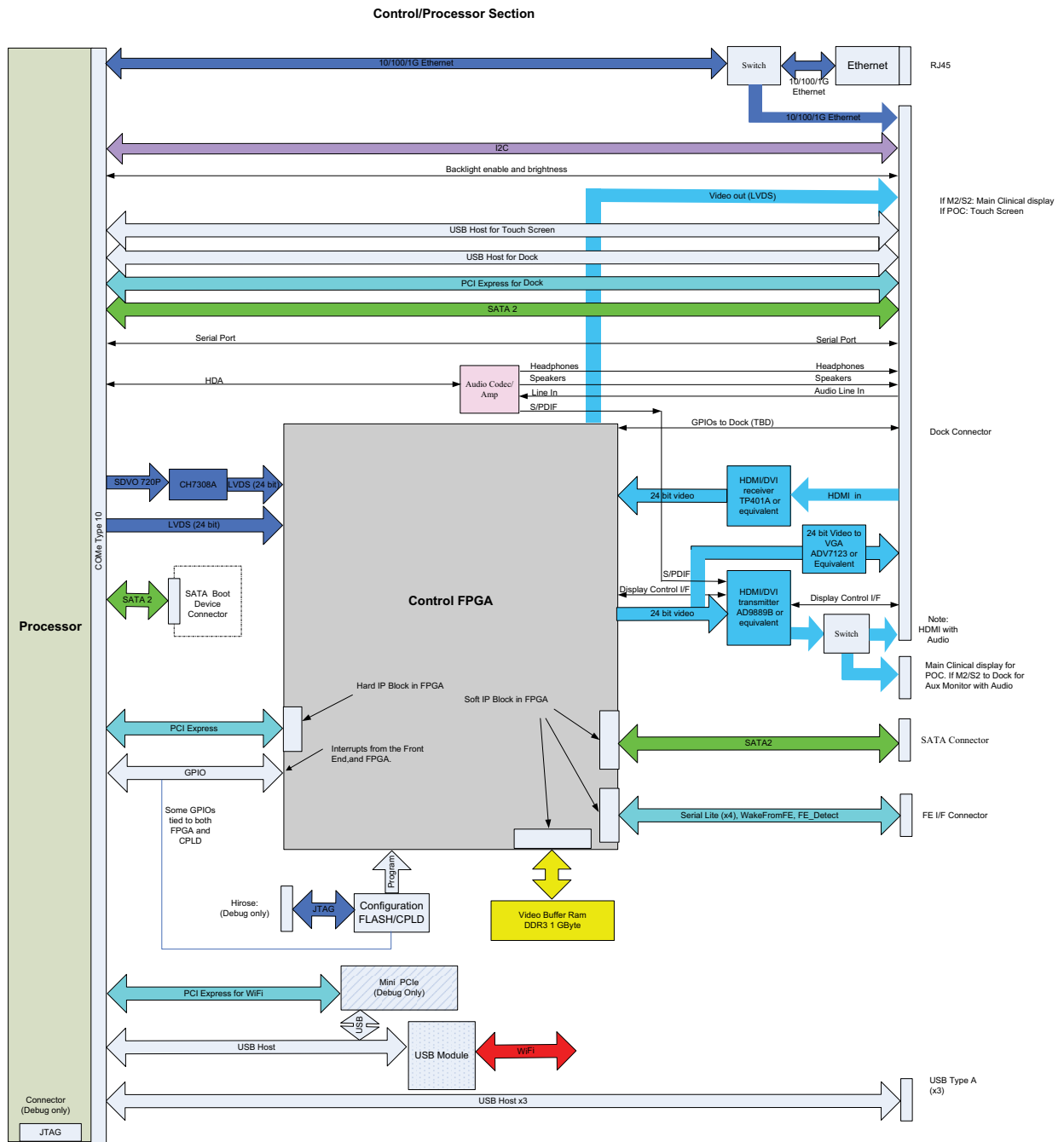


Figure 4.7 Control Subsystem Block Diagram

The core control subsystem contains the processor, the system bus, and the system memory resources of RAM and solid state hard drives. These two solid state hard drive devices are responsible for the system operating system software, ultrasound program software, scanhead operation files, on-board training videos, and storage for patient data (patient demographic information, images, and video clips)

The main operational solid state storage drive is a mSATA form factor device totalling 16 GB of storage capacity. This device is responsible for handling the operational system software, the ultrasound program, and the on-board training videos.

A secondary solid state storage drive of 80 GB with an 1.8 inch form factor is used to handle the scanhead data files, other ultrasound system files, and 66 GB of storage capacity reserved for all patient data. It is with these new solid state devices, the X-Porte architecture is similar to our past products utilizing CF and SD storage cards, but with much improved data storage and throughput without the use of any mechanical hard drive devices.

## Power Supply and Control

The X-Porte ultrasound system utilizes one of two types of power supply arrangements depending on the X-Porte system version, which are Desktop or Stand. The Desktop system uses two separate external power supply (EPS) units. The 150 watt EPS supplies +24VDC to the main Ultrasound System Electronics (Engine) and a separate 65 watt EPS is utilized to supply +24VDC for the Clinical Display Monitor. No batteries can be utilized with this version of system. The electrical block diagram for the 150 watt EPS is shown below in [Figure 4.8 on page 19](#)

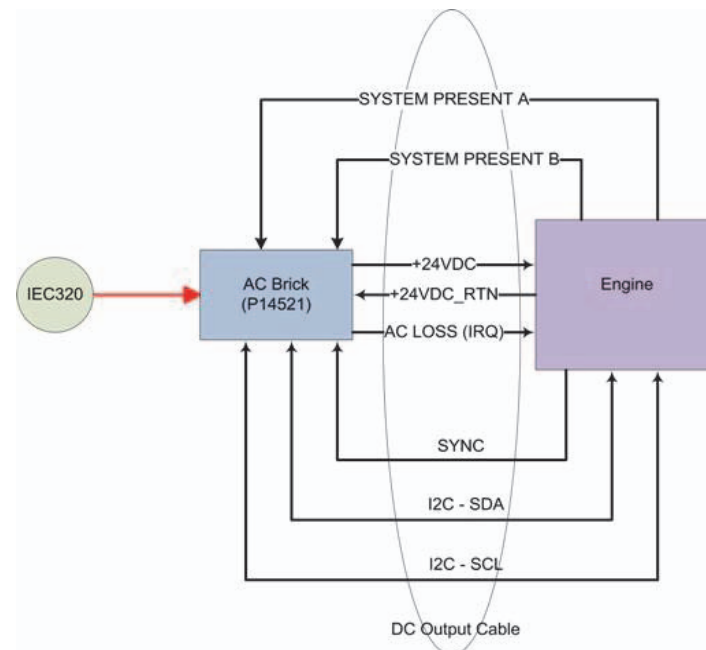


Figure 4.8 Desktop External Power Supply (EPS)

The Stand version X-Porte system utilizes a power supply/power control module mounted under the stand base. This System Power Management Unit (SPMU) supplies the +24VDC power to the Engine and Clinical Display Monitor. This SPMU is also the mounting point for the three lithium-ion batteries, charges the batteries, and is responsible for switching the system from AC power to battery power when needed. Please refer to [Figure 4.9 on page 20](#)

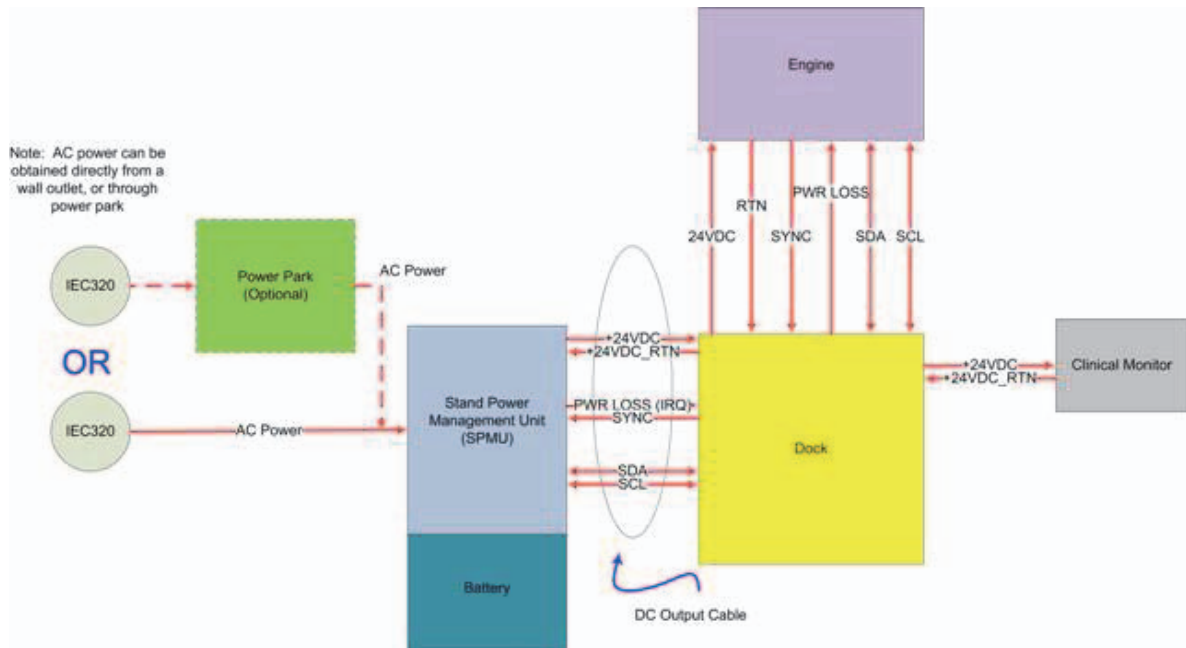


Figure 4.9 Stand Power Management Unit (SPMU)

## ECG Module

The ECG module allows a representation of the heart electrical activity to be displayed in real time with ultrasound images acquired and displayed on the system video display.

The ECG module interfaces to the patient through three (3) ECG leads: Right Arm ECG lead (RA), Left Arm ECG lead (LA), and Left Leg ECG lead (LL). The ECG received signal from the ECG electrodes are isolated, amplified, and filtered by the ECG module before it is sent to the system for further processing and display.

The ECG module and cable are an integrated assembly. The module receives power from the system. Patient isolation is provided by the ECG module, allowing the connection and signals to the system to be system-ground referenced. The isolation between the patient and the system meets the requirements of IEC 601-1 for Type CF equipment.

The ECG function accepts input from an external serial A/D and performs gain, filter, and DC Offset functions. The resultant data is output at either the 200Hz sample rate or decimates the data by 2 or 4 and outputs the data into acquisition memory. The data is assumed to be signed. An interrupt is provided that will interrupt the processor after a set delay from the detected level and slope. A simple block diagram is shown below in [Figure 4.10 on page 21](#)

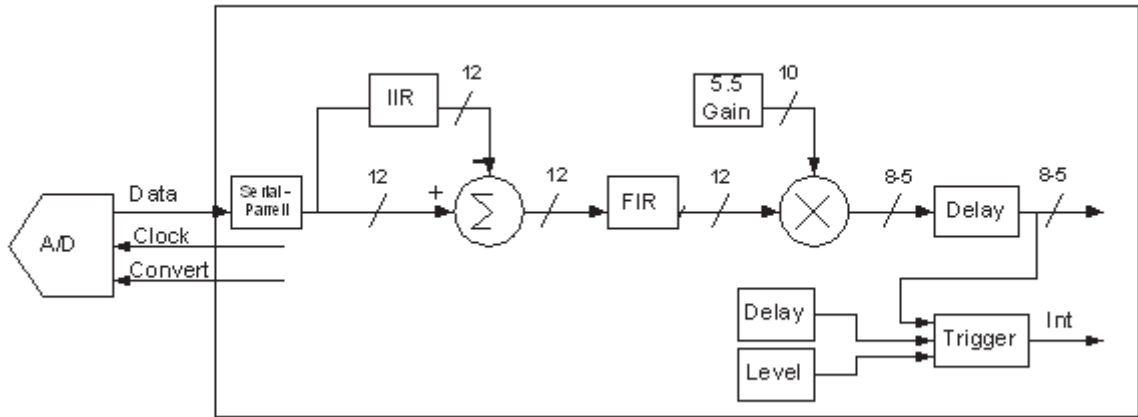


Figure 4.10 ECG Block Diagram

## DICOM

The system features Digital Imaging and Communications (DICOM) capability to provide the acceptance, transfer, display, storage, and digital processing of single ultrasound images as well as loops of ultrasound images.

Please refer to the *X-Porte Product Information & Safety Guide* (P14646) for the DICOM standard and the *X-Porte User Guide* (P14645) for DICOM setup



# Chapter 5: Troubleshooting

## X-Porte Desktop and Stand Versions Major Component Diagrams

Please use [Figure 5.1 on page 23](#) below as a guide to help identify the major component sections of the two X-Porte system types, Desktop and Stand versions.

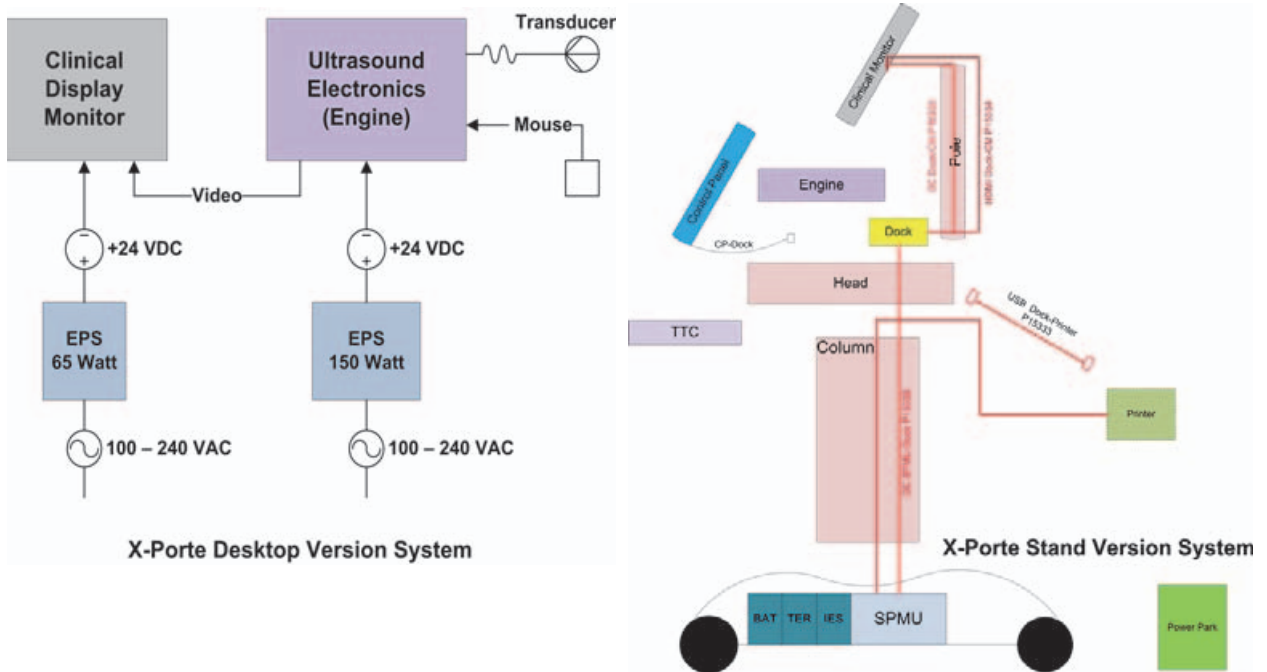


Figure 5.1 X-Porte Major Component Diagrams

## System and Subsystem Diagnosis

This section covers basic diagnostic and troubleshooting procedures you may follow if the system does not operate properly. To diagnose system failures, consult the referenced diagnostic figures that follow or FUJIFILM SonoSite Technical Support.

**Table 5.1: Troubleshooting Subassemblies and Diagnostic Figures**

Subassemblies	Diagnostic Figures or Table
Assert Codes	Figure 5.2; Table 5.2
Battery - Stand System (Linked with SPMU)	Figure 5.3; Table 5.6
Clinical Monitor Display - Stand Version System	Figure 5.1; Figure 5.6; Table 5.3
Clinical Monitor Display - Desktop Version System	Figure 5.1; Table 5.4
DICOM	Table 5.5
SPMU - Stand Version System	Figure 5.3; Figure 5.4; Table 5.6
Touch Control Panel - Stand Version System	Table 5.7
Ultrasound System - Stand Version System	Figure 5.5

## System Repair

The system is repairable through subassembly replacement or through replacement of parts as recommended by FUJIFILM SonoSite. Component level repair of Printed Circuit Board Assemblies is performed only at the FUJIFILM SonoSite repair facility. Replacement of board level components by unauthorized service facilities voids the FUJIFILM SonoSite warranty.

## Test Equipment

Test equipment is not required for this troubleshooting section.

## Failure (Assert) Codes

The X-Porte system is capable in displaying an “assert screen-call for help” instructing the user to contact SonoSite Technical Support for hardware and software issues related to failures. PCB failures typically result in “assert codes” that are output to both the clinical display monitor and the touch panel display. If an assert screen appears, note the assert code information and contact FUJIFILM SonoSite Technical Support to clarify the fault or failure. Refer to [Figure 5.2](#) for how this assert screen will be displayed and the information Technical Support will be asking from this screen.

**Record the following information and report these values to FUJIFILM SonoSite Technical Support.**

P: xxxxx  
C: xxxxx  
D: x

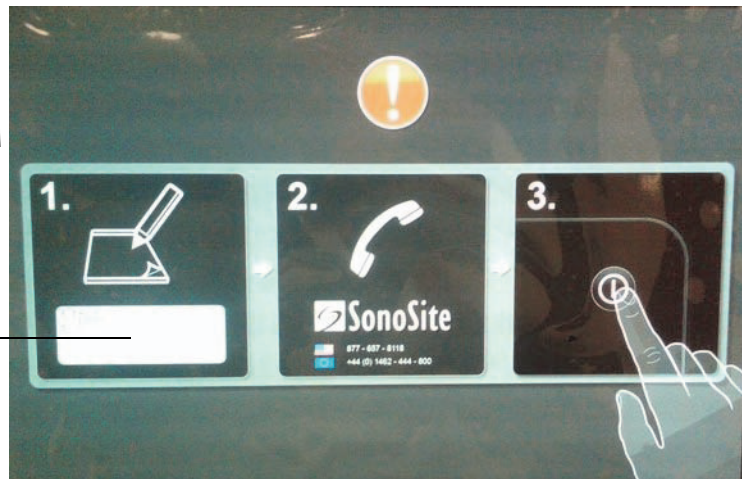


Figure 5.2 Assert Screen-Call For Help

## Verifying a System Assert Code

System asserts are caused by hardware and/or software faults. Hardware asserts typically require a PCB or hardware replacement. Software asserts can be reset and the system may recover. A simple method to identify the cause of the assert is identified here:

- Assert Cause**
- 1 Record the assert code.
  - 2 Press and release the **Power** button to power the system down.
  - 3 Press the **Power** button again to power on the system.
    - If the system powers on normally, it has recovered from the fault (software assert) and you may use the system.
    - If the assert condition remains, corrective action must be taken; usually replacement of a PCB or hardware is required. Contact FUJIFILM SonoSite Technical Support for assistance and to obtain repair parts.

If the **Power** button is not functional, all sources of power must be removed to allow the system to power down. I.e., disconnect AC power and switch the three battery switches to the “•O” (OFF) position. Refer to [Figure 3.1 on page 6](#)



**Table 5.2: Assert Codes**

Code	Code Description Information	Probable Cause	Solution
16203		PS1 PCBA in Engine	Replace Engine
17193		PS1 PCBA in Engine	Replace Engine
18706		BE PCBA In Engine	Replace Engine
19325	istouchinfopresent ()	Touch Control Panel cable connection to Dock Assembly	Reconnect cable connection
19361		BE PCBA in Engine	Replace Engine
19466	Front-end not detected	Internal cables/PCB connections	Replace Engine

Note: Assert Code Table will be updated in future X-Porte Service Manual Revisions

## Clinical Monitor Display

**Table 5.3: Clinical Monitor Display - Stand System**

Symptom	Possible Causes
No displayed video image, blank or black, no screen saver ICON.	Check +24V is present from Dock to Clinical Display Monitor. Replace <i>Stalk Cable Assy.</i> (P16773) or <i>Dock</i> (P15164) to restore +24V. If +24V is present, then replace <i>Clinical Display Monitor</i> (P14627). Note: The Desktop <i>Power Supply</i> (P16776) can be used to test the Clinical Monitor Display operation.
No Ultrasound Image but screen-saver ICON is present.	Check <i>Stalk Cable Assy.</i> connections from the Dock to the Clinical Display Monitor. Replace <i>Stalk Cable Assy.</i> (P16773) and/or <i>Dock</i> (P15164). Try another video source plugged into the HDMI monitor port. Replace <i>Clinical Display Monitor</i> (P14627) if the monitor only displays the screen-saver ICON with another video source. Replace <i>X-Porte Engine</i> (P16055) if monitor functions with other video source.
Brighter background shading (blooming) at any of the image corners.	Replace <i>Clinical Monitor Display</i> (P14627).
Image background is purple, red, or green	Re-seat HDMI cable connection. Replace <i>Stalk Cable Assy.</i> (P16773). Replace <i>Clinical Monitor Display</i> (P14627).
No Audio	Access X-Porte audio controls via "More" on the Touch Control Panel. Check "Mute" is not enabled, adjust volume level higher. Re-seat HDMI cable from DOCK to Clinical Display Monitor. Test with another HDMI cable. Replace <i>Stalk Cable Assy.</i> (P16773), <i>Clinical Display Monitor</i> (P14627), <i>Dock</i> (P15164), then <i>X-Porte Engine</i> (P16055) in this order until audio is restored.

**Table 5.4: Clinical Display Monitor - Desktop System**

Symptom	Possible Causes
No displayed video image, blank or black, no screen saver ICON.	Check +24V is present from external Power Supply to Clinical Display Monitor. Replace Power Supply, 65W, 24V (P16776). Replace <i>Clinical Display Monitor</i> (P14627).
No Ultrasound Image but screen-saver ICON is present.	Check the HDMI cable from the X-Porte Engine HDMI port to the Clinical Monitor Display. Check Monitor Display with another HDMI cable and video source. Replace <i>Cable Assy, HDMI</i> (P15334), replace <i>X-Porte Engine</i> (P16055) if monitor is good, or replace <i>Clinical Display Monitor</i> (P14627) if defective.
Brighter background shading (blooming) at any of the image corners.	Replace <i>Clinical Monitor Display</i> (P14627).
Image background is purple, red, or green	Re-seat HDMI cable. Replace <i>Cable Assy, HDMI</i> (P15334). Replace <i>Clinical Monitor Display</i> (P14627).
No Audio	Access X-Porte audio controls via "More" using the system mouse. Check "Mute" is not enabled, adjust volume level higher. Re-seat HDMI cable from Engine to Clinical Display Monitor. Test with another HDMI cable. Replace <i>Cable Assy, HDMI</i> (P15334), <i>Clinical Display Monitor</i> (P14627), then <i>X-Porte Engine</i> (P16055) in this order until audio is restored.

## DICOM

**Table 5.5: DICOM Troubleshooting**

Error Message	Tiller Error Code	Cause	Troubleshooting
Socket communication failed	TSOCKET_CONNECT_FAILURE	Invalid network configuration. Wrong port number. Application is not running. Printer is offline.	Using Ping, verify that the Printer/Archiver is connected. <ul style="list-style-type: none"> <li>If Ping fails, check the devices IP address, X-Porte IP address, Subnet mask, and Gateway IP address.</li> <li>If Ping is OK, use Verify to check if device is available. If Verify fails: <ol style="list-style-type: none"> <li>Check the Printer/Archiver's Port configuration on the X-Porte.</li> <li>Ensure that the Printer is online and the Archiver's application is running.</li> </ol> </li> </ul>
Archiver transaction failed	TDICARCH_OPEN_FAILURE	Wrong Capture Type selected	Verify that the Archiver supports the selected Capture Type setting, e.g., US Image, SC Image or US-Ret Image.

**Table 5.5: DICOM Troubleshooting (Continued)**

Error Message	Tiller Error Code	Cause	Troubleshooting
Printer transaction failed	TDICPRNT_OPEN_FAILURE	Wrong Image settings	Verify that the printer supports the selected Image settings. E.g., Color (RGB) or Grayscale (Monochrome)
DICOM network communication failed	TDNETWORK_OPEN_FAILURE	Device does not recognize X-Porte, rejects association	Verify that X-Porte AE Title or IP address is correctly configured on the Printer/Archiver. <i>Note: Some devices require that the Imaging modality (X-Porte) be recognized in order to accept images. This requires configuration on the device.</i>
Internal failure detected	TDNETWORK_READ_FAILURE	Invalid DICOM Attribute	Check X-Porte Printer DICOM settings for correctness (e.g., film size, format)

## SPMU

The SPMU module has three LEDs that will help to isolate AC power versus X-Porte hardware failures. On the top-rear portion of the X-Porte stand base is a “Green” LED that is near the equipotential stud. A solid “Green” LED signifies that the X-Porte system is connected to a live AC source. If this LED is flashing at a 1Hz repetitive rate, then the system is charging the system batteries. When these batteries are fully charged, this LED will switch to a solid “Green” LED illumination. Refer to [Figure 5.3](#)

A solid Green LED indicates the system is connected to a live AC source. When batteries are installed, this LED will flash at a 1 Hz rate when the batteries are charging and go to a solid illumination state upon battery charge completion.



Figure 5.3 Upper Stand Base Power LED

Two more LEDs are visible under the stand and can shine onto a piece of white paper placed under the stand base. The solid illuminated “Blue” LED indicates that the SPMU is producing the system’s +24 volts DC necessary to power the system electronics and Clinical Display monitor. The solid illumination faint “Green” LED is indicating that the system is connected to a live AC source. This LED is indicating the same AC as the topside “Green” LED except, this LED does not indicate the battery charge operation and will always be constantly lighted when connected to a live AC source. If the bottom “Green” LED is not visible due to the ambient light conditions, then check for the “Green” topside LED and the bottom “Blue” LED. These indicate that the SPMU is connected to a live AC source and that the SPMU is outputting +24 volts DC. Refer to [Figure 5.4](#)

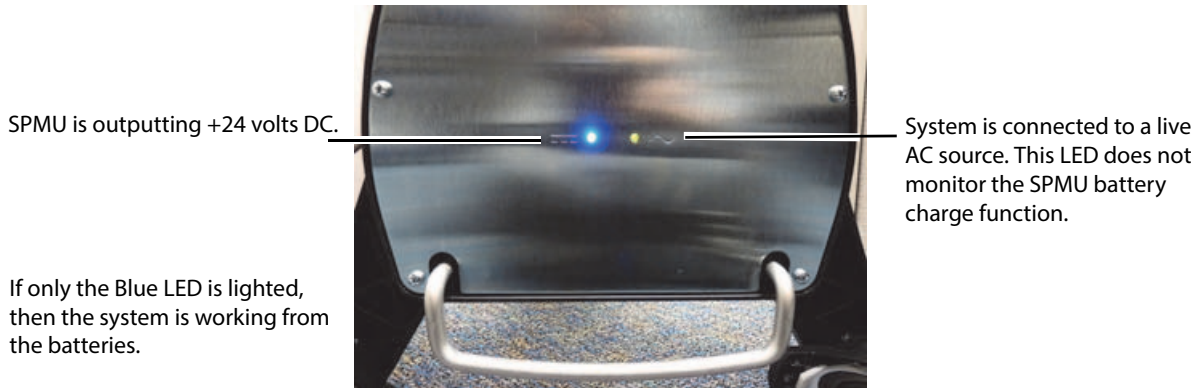


Figure 5.4 Stand Baseplate LEDs

Refer to [Table 5.6](#) below to verify the following normal conditions and configurations that the X-Porte system will operate properly. Any LED conditions other than those outlined in this table that exist could indicate a SPMU problem requiring replacement.

*Note: If all three battery switches have been in the “•O” (OFF) position and all three are set to “O” (ON), AC power has to be applied to the SPMU to allow the system SPMU to develop the +24VDC from battery operation. The lower Blue LED will remain off indicating no +24VDC until performing this procedure.*

**Table 5.6: SPMU LED Conditions - Stand System** [Table 5.7](#)

Normal Operation Conditions	Top Green LED	Lower Green LED	Lower Blue LED
Connected to AC, no batteries	ON	ON	ON (+24Vdc)
Connected to AC, batteries charging	1 Hz flashing rate	ON	ON (+24Vdc)
Connected to AC, batteries fully charged	ON	ON	ON (+24Vdc)
Connected to AC, battery switches “O” (OFF)	ON	ON	ON (+24Vdc)
No AC, operation on battery power	OFF	OFF	* ON (+24Vdc)

\* Requires the AC power to be restored to the SPMU after all three battery switches have been switched to “•O” (OFF) after service or air transport shipping. This Blue LED will not illuminate on battery operation until AC power is first restored.

## Touch Control Panel

**Table 5.7: Touch Control Panel - Stand System**

Symptom	Possible Causes
Assert Code 19325 with description: istouchinfopresent ()	Reconnect Touch Control Panel cable to the Dock port. Replace <i>Touch Control Panel Assy.</i> (P14027) if problems persist. Replace <i>Dock Assy.</i> (P15164) as last option.
Blooming, jumpy, or jittery graphic display	Remove the Engine and check the TTC Assy. has all four roller guides in the Stand channel holes. Re-install TTC if not properly installed to the Stand. Check the Touch Control Panel Assy. cable connection to the Dock Assy. and re-install Engine. Replace the <i>Touch Control Panel Assy.</i> (P14027) if problem persists.
Touch Control Panel is black or not functional	Replace the <i>Touch Control Panel</i> (P14027) first. If problem persists, the replace the <i>Dock Assy.</i> (P15164)

## X-Porte Power-0 & Power-1 Procedural Steps

The Power-0 and Power-1 functionality has been implemented in X-Porte differently than it was in Turbo due to the lack of buttons.

The functionality is simple, but the format must be exact. Simply places a text file called **reset.txt** on a USB stick (P16967) then insert it into a USB slot of the system, remove all other USB sticks, and now power "ON" the system. The text file contains a phrase to tell the system which action to perform. Only one action can be performed at a time. The file will be deleted by the system after it is read to prevent inadvertent usage. Therefore, it is a good idea to save the **reset.txt** file to some other name before inserting into the system and powering up. The actual implementation of the action is performed during system initialization and is complete when the system disk becomes ready.

The **Power-1 Reset** function is specified by placing the phrase "**restore-factory-settings**" in the **reset.txt file**. The command causes the system to:

- "Remove Worklist files
- "Rebuild the patient database
- "Start a new patient
- "Reset the DICOM location
- "Reset the DICOM, user and system logs
- "Reset user preferences
- "Reset the system clock to January 1, 2003
- "Reset the Control Panel brightness and backlight
- "Records the action in the system log

The **Power-0 Reset** function is specified by placing the phrase "**reformat-data-drive**" in the **reset.txt file**. This command causes the system to:

- "Format the patient data drive and deletes all patient data
- "Start a new patient
- "Add a system log message recording the action

Figure 5.5 Troubleshooting Flow Chart - No System Operation


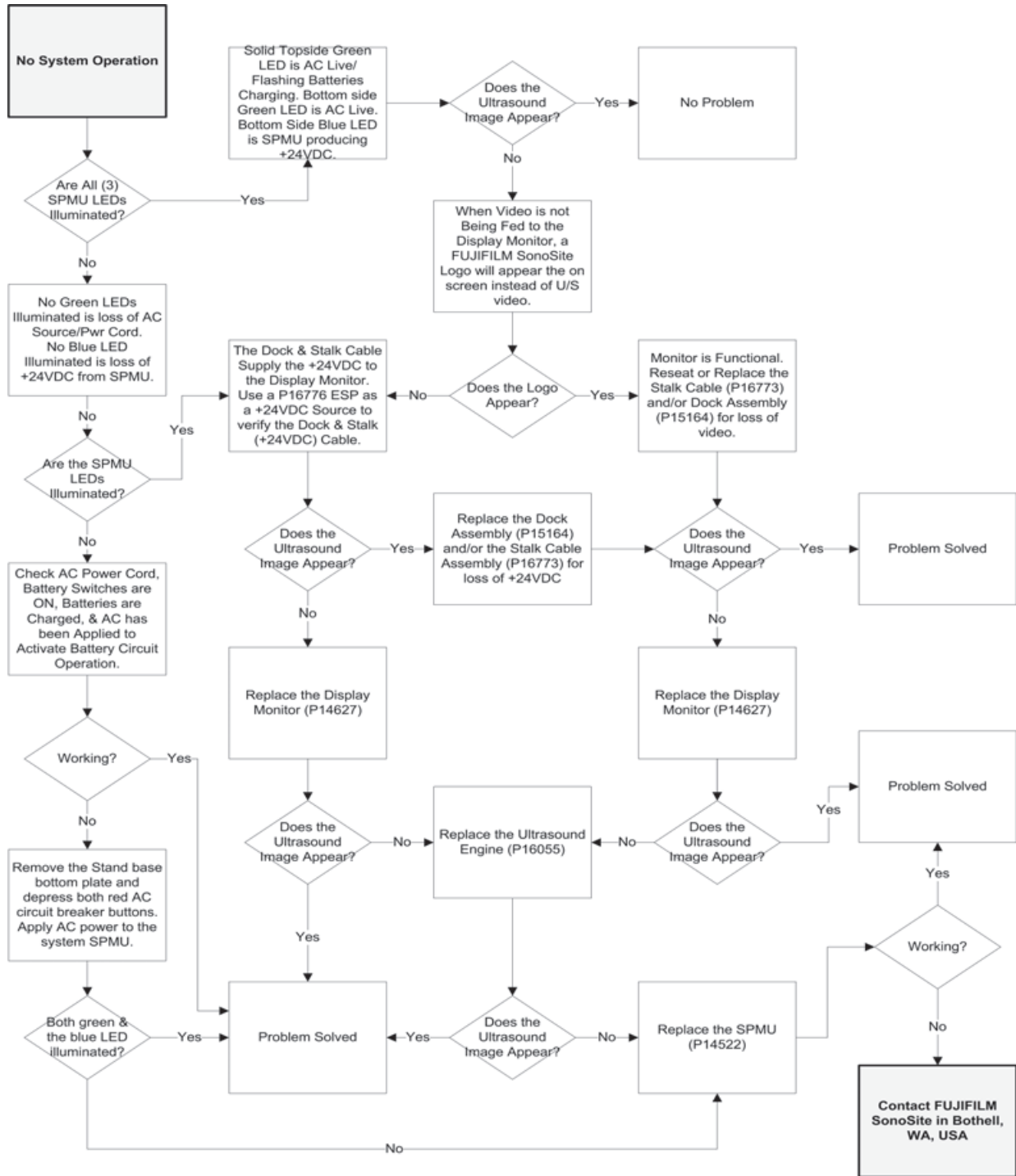


Figure 5.6 Troubleshooting Flow Chart - No Clinical Monitor Display Image





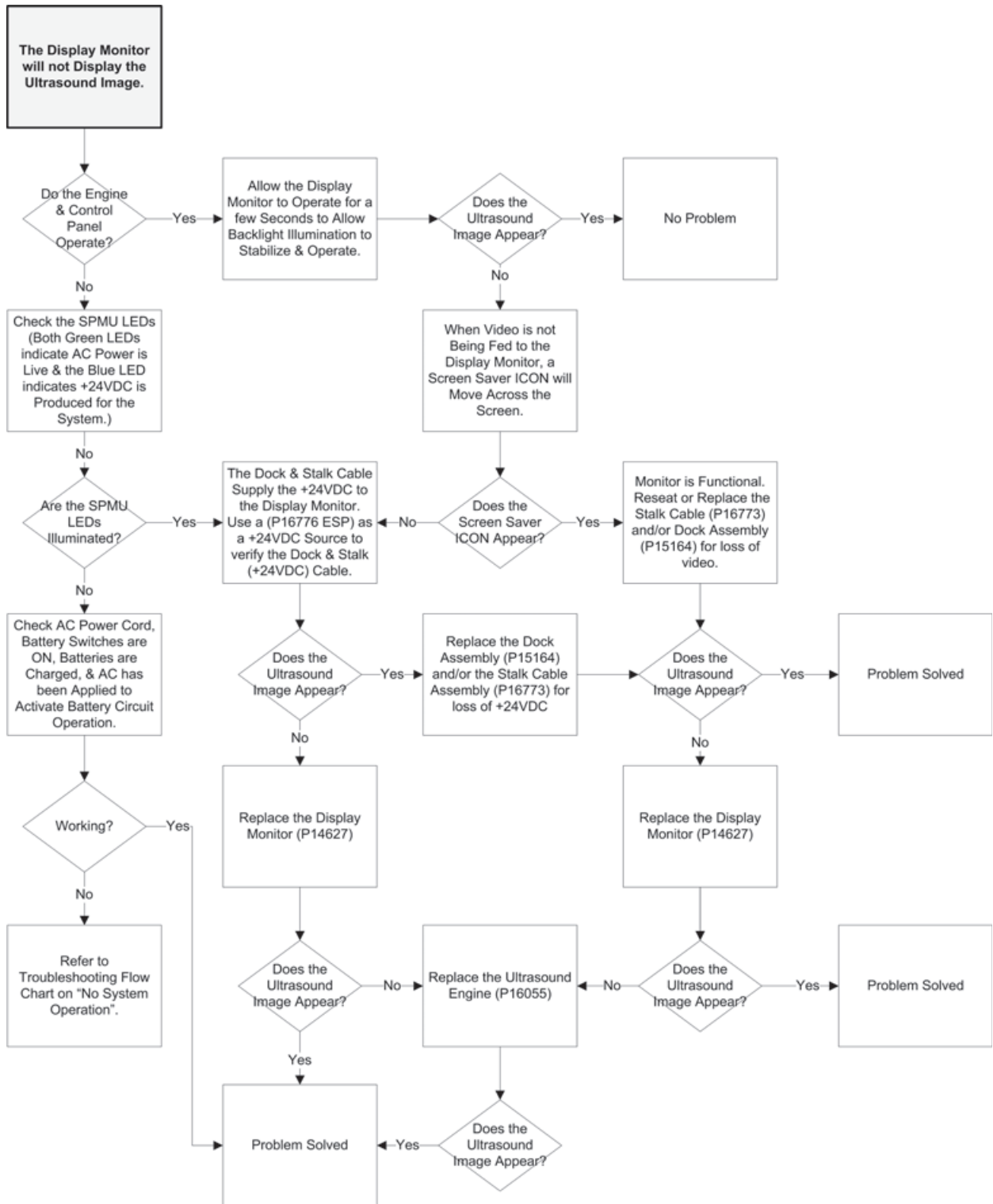

# Chapter 6: Replacement Procedures

FUJIFILM SonoSite has determined several service activities that will be performed only by FUJIFILM SonoSite service personnel and/or their representative qualified to perform the service duties. These are the removal/replacement of the SPMU and the batteries that are in the X-Porte Stand system. Please contact FUJIFILM SonoSite or your appropriate FUJIFILM SonoSite representative when service is required. Refer to [Table 6.1 on page 34](#) for a more complete list of subassemblies and the personnel that should attempt service on the X-Porte systems.

Prior to performing any service to the X-Porte system, remove the A.C. power cord from the wall outlet. This applies to both the X-Porte Stand and X-Porte Desktop system versions.

**WARNING:**

The stand version is equipped with three lithium-ion batteries and require all three battery switches to be switched to their “•O” (OFF) position to cease the +24VDC developed by the SPMU. Access to these switches are by lifting the black plastic panel outward from the bottom of the X-Porte stand SPMU housing area near the left rear caster leg. Refer to [Figure 3.1 on page 6](#) of Chapter 3 under “Safety”.

**WARNING:**

The SPMU removal/replacement as well as the replacement of the batteries in the X-Porte Stand system requires the service personnel to be able to reposition an upright stand system onto its front caster legs (about 65 lb of the system weight) and lay the complete system horizontally on the floor. It is advisable to have a second person to help in repositioning the system if the 65 lb weight requirement is too great for one person. This service should only be performed by trained and qualified FUJIFILM SonoSite service personnel.

**Caution:**

Follow all steps for stand height, Clinical Monitor, and touch Control Panel positioning in the forthcoming service procedures to ensure no damage to the system and/or possible injury to the servicing personnel.

**Caution:**

Always use correct ESD procedures. ESD damage is cumulative and may not be noticeable at first. Initial ESD symptoms may be slightly degraded performance or image quality.

**Caution:**

All fasteners should be torque to 5.5-inch pounds except where noted. Two ranges of torque screwdrivers will be required as torque requirements range from 3-inch pounds to 24-inch pounds. Generally the torque screwdrivers ranges are 1.0-10.0 inch pounds (0.23-1.1 Newton meter) and 10.0-25.0 inch pounds (1.1-2.8 Newton meter)

*Note: If all three battery switches have been in the “•O” (OFF) position and all three are set to “⊙” (ON), AC power has to be applied to the SPMU to allow the system SPMU to develop the +24VDC from battery operation. The lower Blue LED will remain off indicating no +24VDC until performing this procedure.*

# Qualifying Service Personnel

**Table 6.1: Responsible Personnel for Subassemblies**

Subassembly	X-Porte Desktop System Qualified Personnel	X-Porte Stand System Qualified Personnel
Batteries	N/A	FUJIFILM SonoSite Representative
B/W Printer	Customer	Customer
Clinical Monitor Display	Customer	Customer
Touch Control Panel	N/A	Customer
Dock	N/A	Customer
System	Customer	Customer
External Power Supply (EPS)	Customer	N/A
Isolation Transformer	Customer	FUJIFILM SonoSite Representative
Mouse	Customer	Customer
Stand Power Management Unit (SPMU)	N/A	FUJIFILM SonoSite Representative
Transducers	Customer	Customer
Triple Transducer Connect (TTC)	N/A	Customer

## Clinical Monitor Display

The weight of the Clinical Monitor is 16.8 lb (7.62kg) and will require the person removing this assembly to be able to support this weight with one hand while using a screw driver with the other hand.

### Required Parts

- P14627-N Service Assembly, Display Clinical Monitor, X-Porte
- P14627-W Warranty Service Assembly, Display Clinical Monitor, X-Porte
- V14627-N Vet Service Assembly, Display Clinical Monitor, X-Porte
- V14627-W Vet Warranty Service Assembly, Clinical Monitor Display, X-Porte

### Required Tools

- #1 Phillips screwdriver
- Torque screwdriver, 2.0-10.0 inch pounds (0.23-1.1 Newton meter)

## Clinical Monitor Display Removal

### Clinical Monitor Display Removal

- 1 Adjust the stand height to a comfortable level as to allow the best working height to access the mounting screws and cables.
- 2 Adjust the Clinical Monitor to the full vertical position and open the cable access door on the back.
- 3 Loosen the two screws and remove the HDMI video wire clamp and pull the cable connector downward to disconnect from the Clinical Monitor. Refer to [Figure 6.1 on page 35](#).

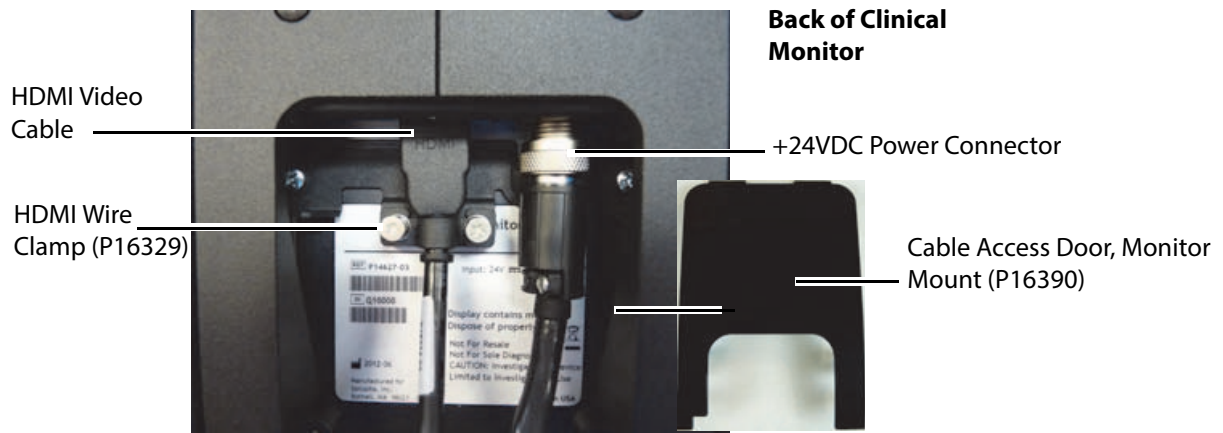


Figure 6.1 Clinical Monitor Cable Connections

- 4 Un-thread the outer ring of the +24VDC power connector and pull downward disconnecting from the Clinical Monitor. Refer to [Figure 6.1 on page 35](#).
- 5 Remove the six black Phillips pan head screws (4 short and 2 long) mounting the Clinical Monitor to the Clinical Monitor Display Mount. Refer to [Figure 6.2 on page 35](#).

*Note: After removing five of the screws, support the monitor against the mount as the indentation pattern of the back of the monitor will help hold the assembly against the mount. Be prepared to support 16.8 lb (7.62 kg) with one hand/arm when the final screw is removed.*

- 6 Remove the Clinical Monitor display with both hands from the mount.

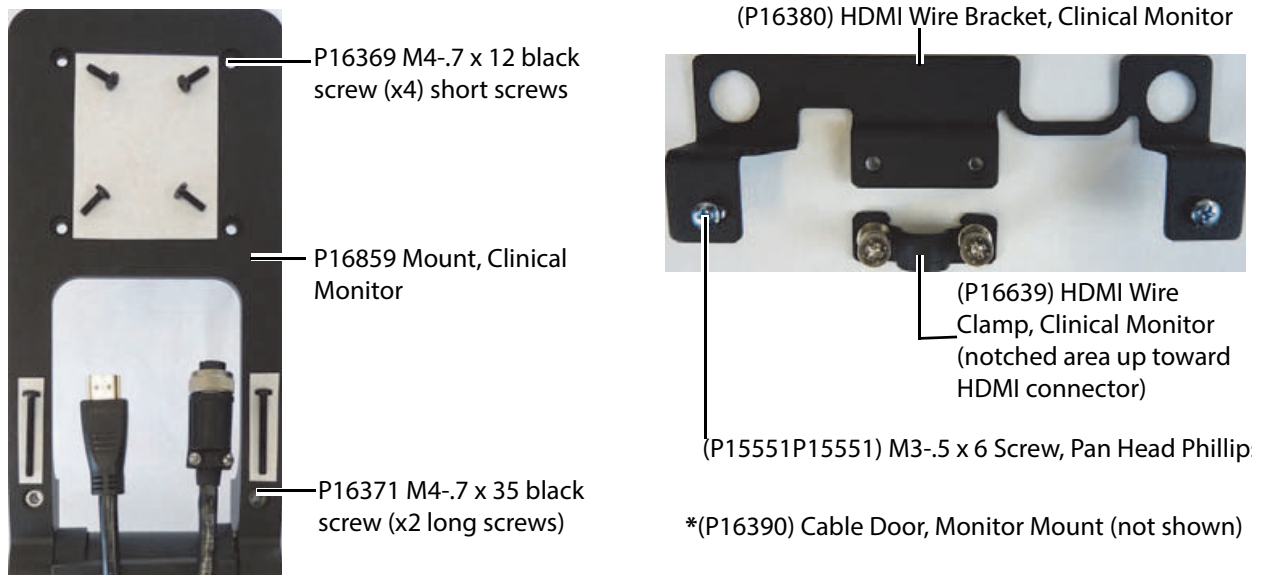


Figure 6.2 Clinical Monitor Mounting Hardware

## Clinical Monitor Display Replacement

### Clinical Monitor Display Replacement

- 1 Perform the Clinical Monitor Display Removal steps 1-6 in the reverse order.
- 2 Tighten all screws to 5.5-inch pounds.

## System (“Engine”)

### Caution:

Prior to unlatching the System from the stand, please check to ensure all USB connections on the right stand side and the ECG cable on the left stand side are removed. Failure to remove any devices plugged into the System ports will inhibit the System from unlatching from the stand as well as possible risk of damage to the connection ports of the System. Check to ensure that the TTC connection has been unlatched and the TTC main connector is pulled downward and away from the base of the System.

### Required Part

- P16055-N Service Assembly, X-Porte Engine, CW
- P16055-W Warranty Service Assembly, X-Porte Engine, CW
- V16055-N Vet Service Assembly, X-Porte Engine, CW
- V16055-W Vet Warranty Service Assembly, X-Porte Engine, CW

### Required Tools

- None

## System (“Engine”) Removal

### System Removal

- 1 Remove any USB devices connected into the right System/Stand side ports as well as the ECG cable connected to the left System/Stand side ports. Refer to [Figure 6.3 on page 36](#).

*Note: Failure to remove connections into the System from either stand side will inhibit the release of the System from the stand as well as cause damage to the System’s connector ports.*

- 2 Unlatch the Triple Transducer Connector (TTC), if installed, or any transducers connected directly to the System (Engine) and pull the TTC connector downward away from the base of the System. Refer to [Figure 6.15 on page 46](#)
- 3 Locate the two gray latch levers under the right and left side stand top platform near where the black metal mates to the shiny metal handle. Refer to [Figure 6.3 on page 36](#).

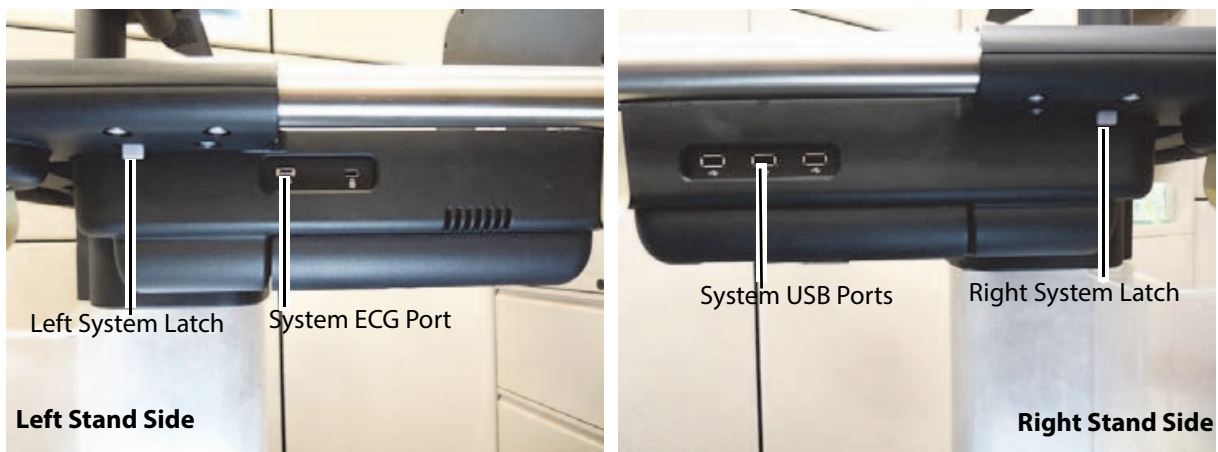


Figure 6.3 System Latch Release & Ports

- 4 Simultaneously push each latch outward away from the System toward the stand outer side edges. If all System connector ports are empty and the TTC main connector properly pulled away from the bottom of the System, the System will eject upward from the stand.
- 5 Lift the System from the back edge and lift forward and up away from the top of the stand.

## System (“Engine”) Replacement

### System Replacement

- 1 Place the replacement X-Porte System (“Engine”) into the stand System head by placing the front edge into the head first just under the two metal retaining tabs at the front edge of the stand’s System head.
- 2 Push down at the top back edge of the System and the stand latches should engage and hold the System in place.
- 3 Reconnect the TTC, if equipped, by pushing upward on the TTC main connector and turning the latch into the locked position.
- 4 Reconnect any USB cables and/or the ECG cable to the right and left sides of the System/Stand area.

## Touch Control Panel

### Required Parts

- P14027-N Service Assembly, Control Panel, X-Porte
- P14027-W Warranty Service Assembly, Control Panel, X-Porte
- V14027-N Vet Service Assembly, Control Panel, X-Porte
- V14027-W Vet Warranty Service Assembly, Control Panel, X-Porte

### Required Tools

- #1 Phillips screwdriver
- 3mm Hex Allen Wrench
- Torque screwdriver, 10.0-25.0 inch pounds (1.1-2.8 Newton meter)

## Control Panel Removal

### Control Panel Removal

- 1 Adjust both the Clinical Monitor display and the Control Panel to their full vertical position.
- 2 Remove the X-Porte System (“Engine”) following the steps in the System (“Engine”) Removal section below and refer to [Figure 6.3 on page 36](#).
- 3 Remove the Control Panel cable connection at the Dock assembly by squeezing both sides of the connector plug and pulling out from the Dock. Refer to [Figure 6.4 on page 38](#)

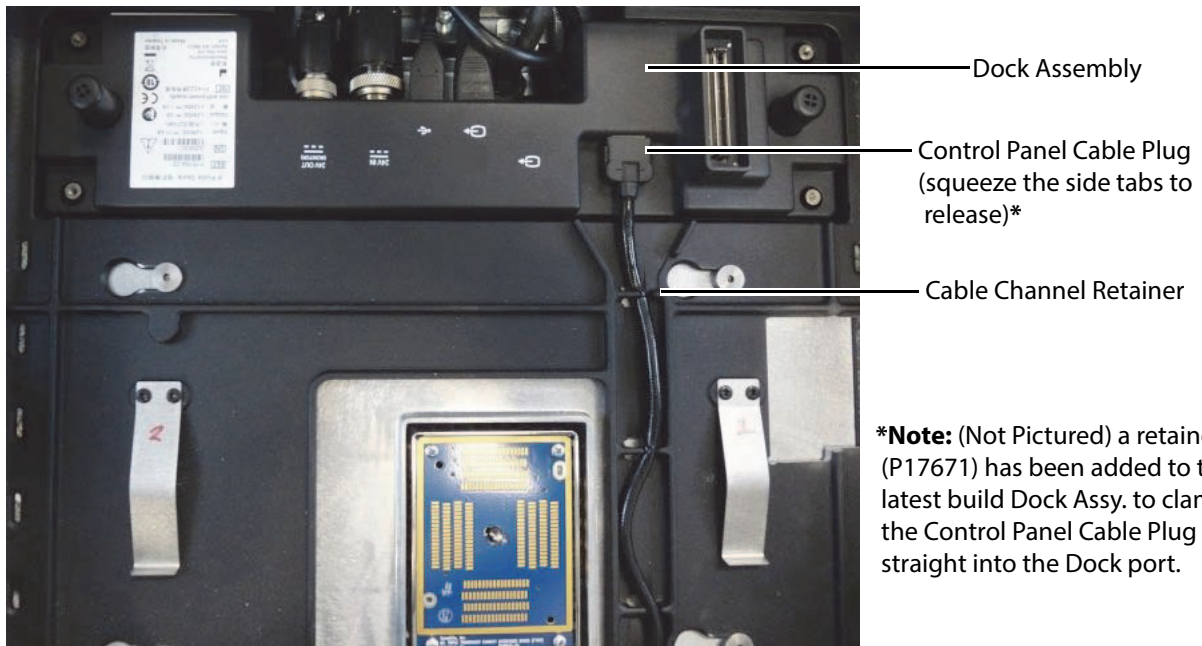


Figure 6.4 Control Panel Cable Plug

- 4 Continue to pull the Control Panel cable out of the stand channel retainers toward the Control Panel

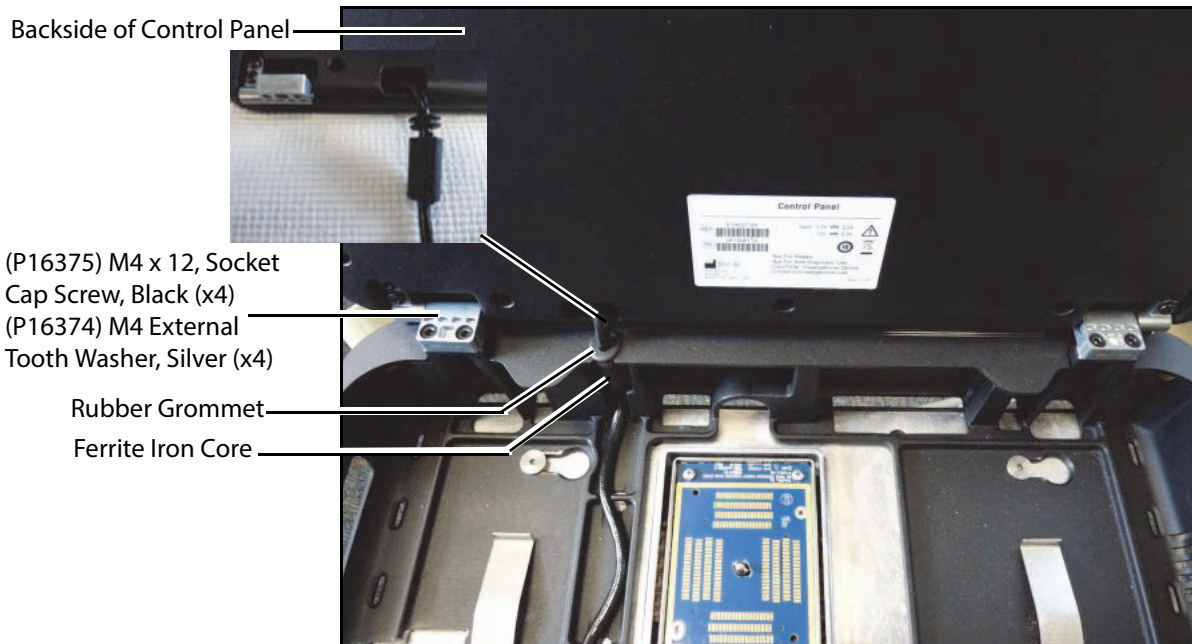


Figure 6.5 Control Panel Mount

- 5 At the vertical cable retention, carefully pry out the rubber grommet and the cylindrical ferrite iron core. It may require some of the cable to be fed up from the bottom to loosen the cable. Refer to [Figure 6.5 on page 38](#).
- 6 Using a 3mm Hex Allen Wrench, remove all four M4 x 12 socket cap (hex) screws in both Control Panel hinges.  
*Note: Use care and do not lose the four M4 external tooth washers on the M4 x 12 socket cap screws.*
- 7 The Control Panel is now unmounted from the stand.

## Control Panel Replacement

### Control Panel Replacement

- 1 Perform steps 1-7 in the reverse order from the Control Panel Removal section of this chapter.
- 2 Refer to [Figure 6.4 on page 38](#) and [Figure 6.5 on page 38](#).
- 3 Torque the four black M4 x 12 socket cap screws with a M4 external tooth washer at each of the two hinges at 24-inch pounds.

## Dock

### Required Part

- P15164-N Service Assembly, Dock with DVR, X-Porte
- P15164-W Warranty Service Assembly, Dock with DVR, X-Porte
- V15164-N Vet Service Assembly, Dock with DVR, X-Porte
- V15164-W Vet Warranty Service Assembly, Dock with DVR, X-Porte

*Note: The Latest version of the Dock Assy. will now incorporate additional hardware to secure the Touch Control Panel cable and the Clinical Display Monitor HDMI cable not shown in this chapter's figures. Two pads will be located on the bottom of the Dock to help maintain the Dock level with the Engine connection port. The extra hardware are as follows: (P17671) Retainer; (P16639) HDMI Wire Clamp; (P16894) HDMI Cable Bracket; and (P18338) PORON Foot (x2). Please refer to [Figure A.4 on page 64](#) for the latest image and hardware comprising the Dock Assembly.*

### Required Tools

- # 2 Phillips Screw Driver

## Dock Removal

### Dock Removal

- 1 Positional the Clinical Monitor display and touch Control Panel to their complete vertical positions.
- 2 Remove the X-Porte System ("Engine") from the Stand using steps 1-5 under the System section of this chapter. Refer to [Figure 6.3 on page 36](#)
- 3 Remove the (P16639) HDMI Wire Clamp from the (P16894) HDMI Cable Bracket. Refer to [Figure A.4 on page 64](#) for the image of the hardware.
- 4 Disconnect the Control Panel cable connector from the Dock. Refer to [Figure 6.4 on page 38](#). The new Dock Assembly will have the (P17671) Retainer requiring it to be flipped upward before removing the Control Panel cable. Please refer to [Figure A.4 on page 64](#)

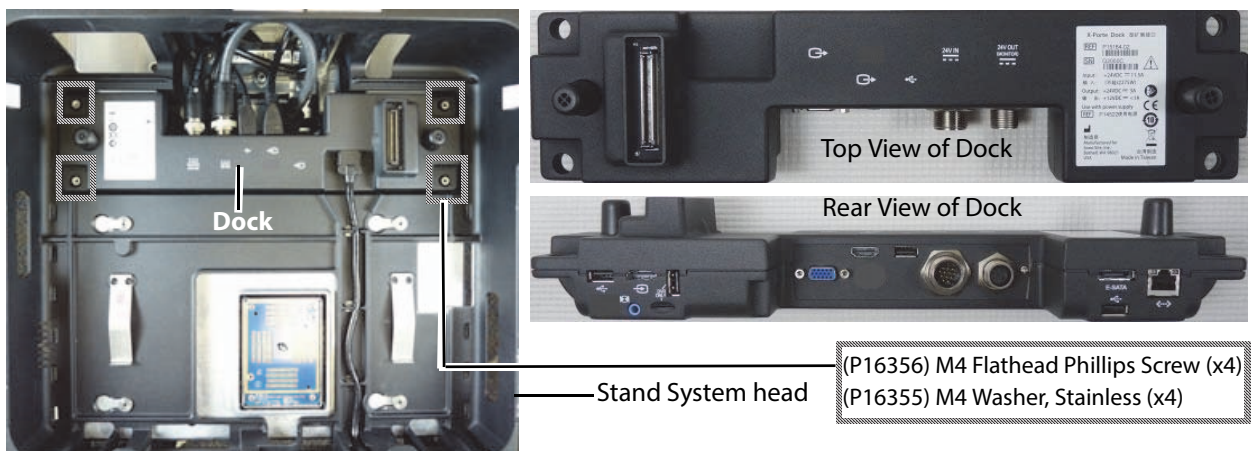


Figure 6.6 Dock Cabling

- 5 Remove the four M4 flathead Phillips screws (P16356) and the four M4 stainless steel countersunk washers (P16355) from the Dock mounting posts located inside the Stand System head area. Refer to [Figure 6.6 on page 39](#)
- 6 Pull the Dock up and away from the four mounting posts.
- 7 Disconnect the remaining cables attached to the rear of the Dock. The Dock can now be moved out of the stand.

## Dock Replacement

### Dock Replacement

- 1 To Replace the Dock, perform steps 1-6 in the reverse order located in the Dock Removal section of this chapter. Refer to [Figure 6.6 on page 39](#) and [Figure 6.4 on page 38](#)

## Stand Power Management Unit (SPMU)

Service for the SPMU and batteries should only be performed by qualified FUJIFILM SonoSite service personnel or their representative. Due to the lifting and positioning nature of performing the following steps, please refer to the Warnings and Cautions printed at the beginning of this chapter.

### Required Part

- P14522-N Service Assembly, Stand Power Management Unit, X-Porte
- P14522-W Warranty Service Assembly, Stand Power Management Unit, X-Porte
- V14522-N Vet Service Assembly, Stand Power Management Unit, X-Porte
- V14522-W Warranty Vet Service Assembly, Stand Power Management Unit, X-Porte

### Required Tools

- #1 Phillips screwdriver
- 5mm Hex Allen Wrench
- Torque screwdriver, 10.0-25.0 inch pounds (1.1-2.8 Newton meter)

## Stand Power Management Unit (SPMU) Removal

### Stand Power Management Unit (SPMU) Removal

- 1 Removal all items from the stand rear storage basket.
- 2 Remove all cables, adapters, and any accessories plugged into either side of the System. Refer to [Figure 6.3 on page 36](#).
- 3 Unlatch the System via the two small gray unlock levers. Refer to [Figure 6.3 on page 36](#).
- 4 Pull up, forward, and away from the stand to remove the System from the stand System head.
- 5 Position the touch Control Panel to the horizontal (shipping position). Refer to [Figure 6.7 on page 41](#).
- 6 Fold the Clinical Monitor display to the horizontal (shipping position). Refer to [Figure 6.7 on page 41](#).
- 7 Adjust the stand to the minimum height (collapsed-shipping position).
- 8 Position the two front casters under the stand caster legs and lock all four casters. Refer to [Figure 6.7 on page 41](#).



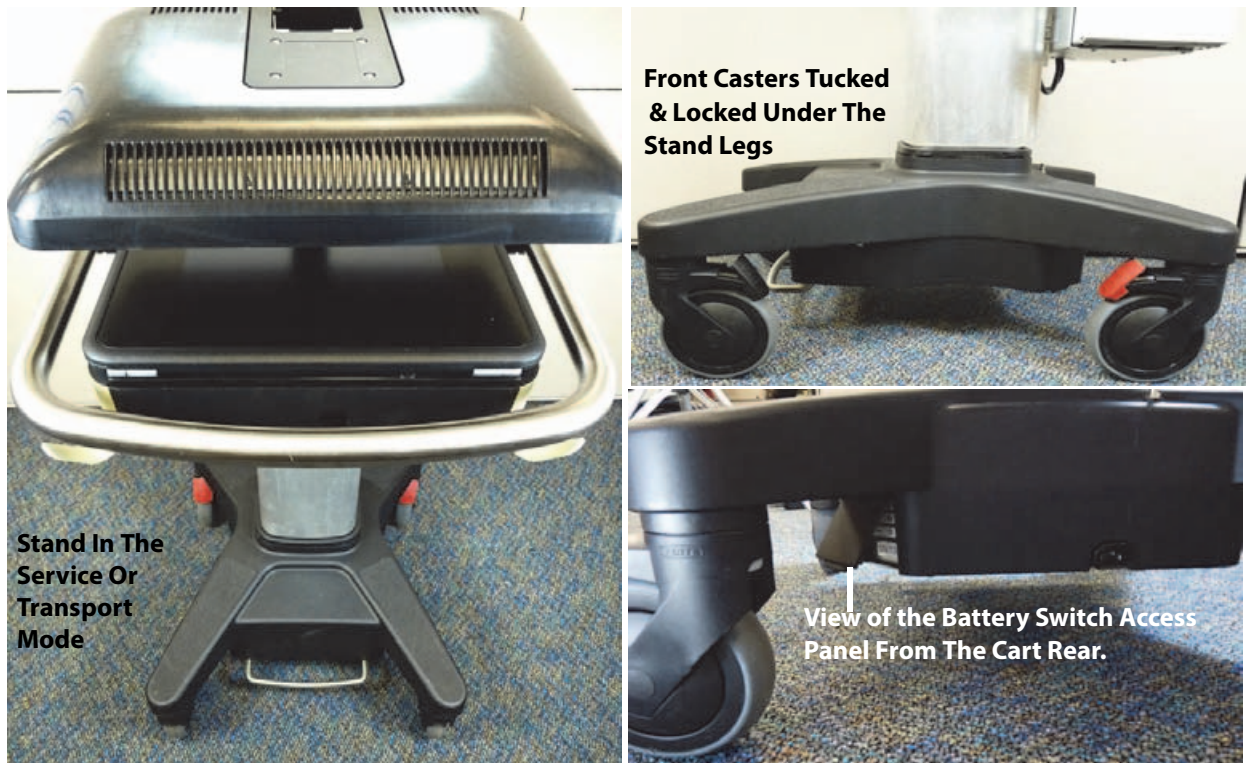


Figure 6.7 Stand Adjusted for Service or Transport

- 9 Access the three battery switches and set all switches to the “•O” (OFF) position. Refer to [Figure 6.7 on page 41](#).
- 10 Tip the stand forward and continue towards the floor so that the stand rests with the front two caster legs and the brushed metal handle rest horizontal on the floor. This will expose the bottom plate of the stand base. Refer to [Figure 6.8 on page 41](#).



Figure 6.8 Stand Tilted Forward-Bottom Plate

- 11 Remove the PowerPark Stand Module two mounting M5 x 20mm (P13629)Phillips screws. This is an optional feature and may not be present on your stand. Refer to [Figure 6.9 on page 42](#)
- 12 Remove the six M5 x 8 (P16382) Phillip screws securing the bottom plate and remove the plate. Refer to [Figure 6.9 on page 42](#)

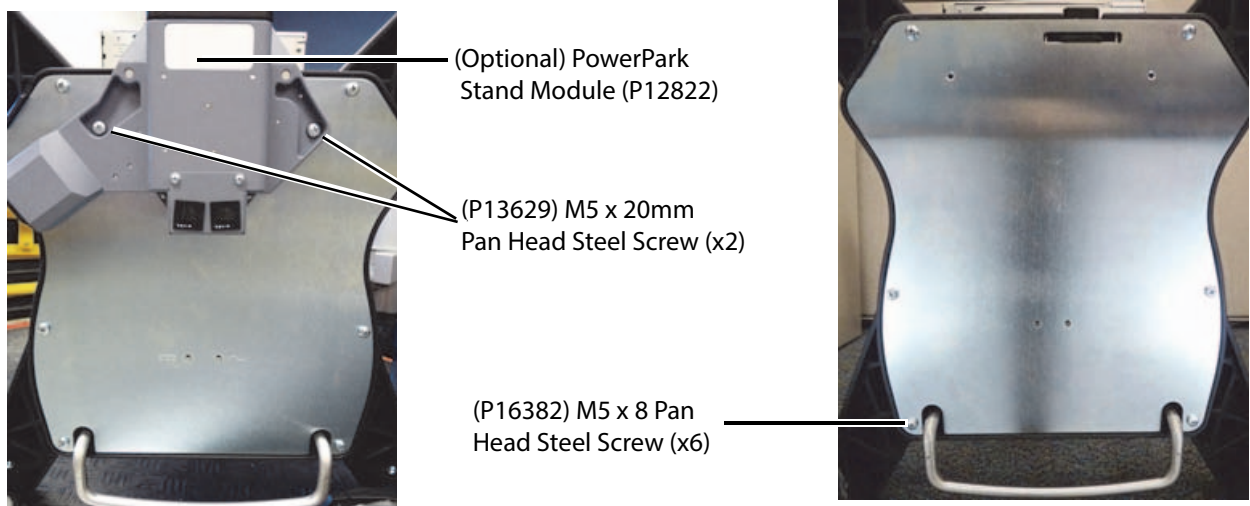


Figure 6.9 Stand Bottom Plate

- 13 Remove the three batteries mounted to the SPMU by loosening the four captive Phillips screws holding each battery and pull each battery off of the SPMU. Refer to [Figure 6.10 on page 42](#).

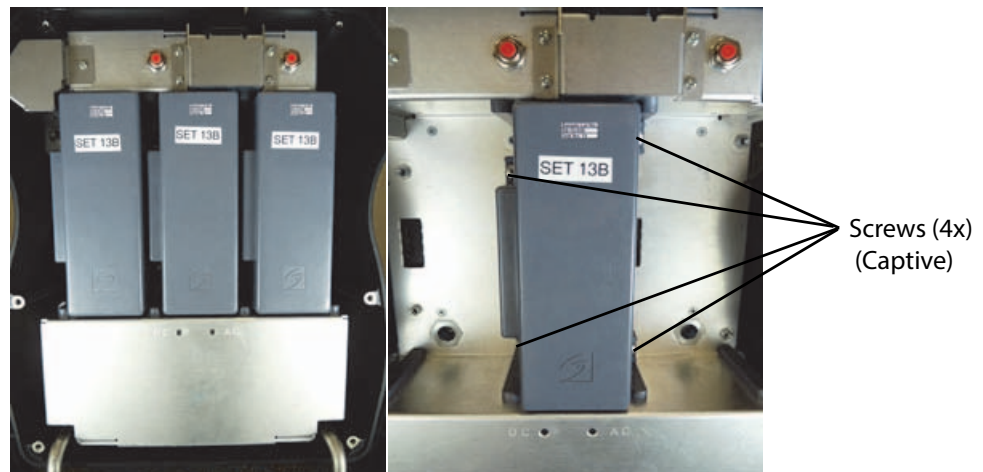


Figure 6.10 Batteries

- 14 Loosen the seven captive 5mm hex head screws securing the SPMU to the bottom of the stand and pull away from the stand. Considerable effort is needed to break the adhesion caused by the gray thermal pad sandwiched between the heat sink of the SPMU and the bottom of the stand. Refer to [Figure 6.11 on page 43](#).

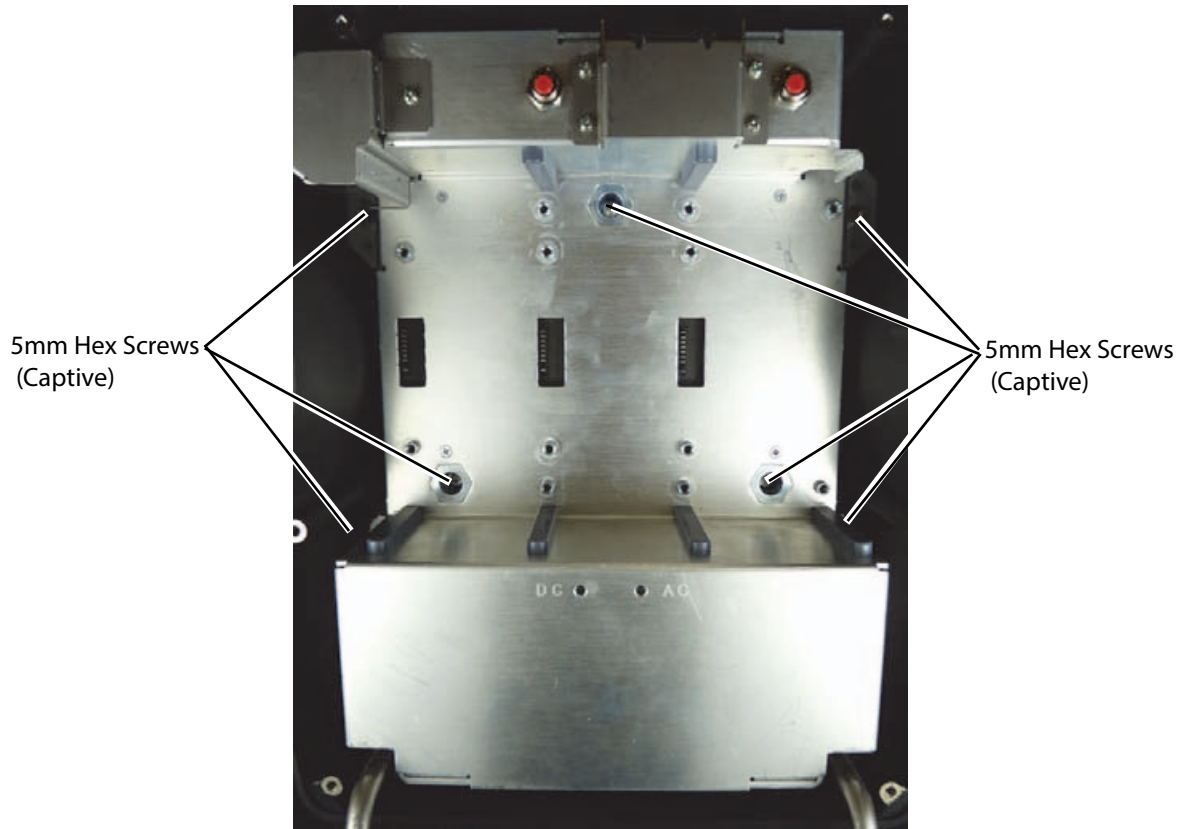


Figure 6.11 SPMU Mount

- 15 Pull the SPMU away from the bottom of the stand and gain access to remove the AC power cord held captive via a metal wire bail. Flip the wire bail to release the AC power cord.
- 16 Un-thread the retaining ring of the DC power cable plug and remove from the SPMU DC output socket. The SPMU can now be removed from the stand. Refer to [Figure 6.12 on page 43](#)

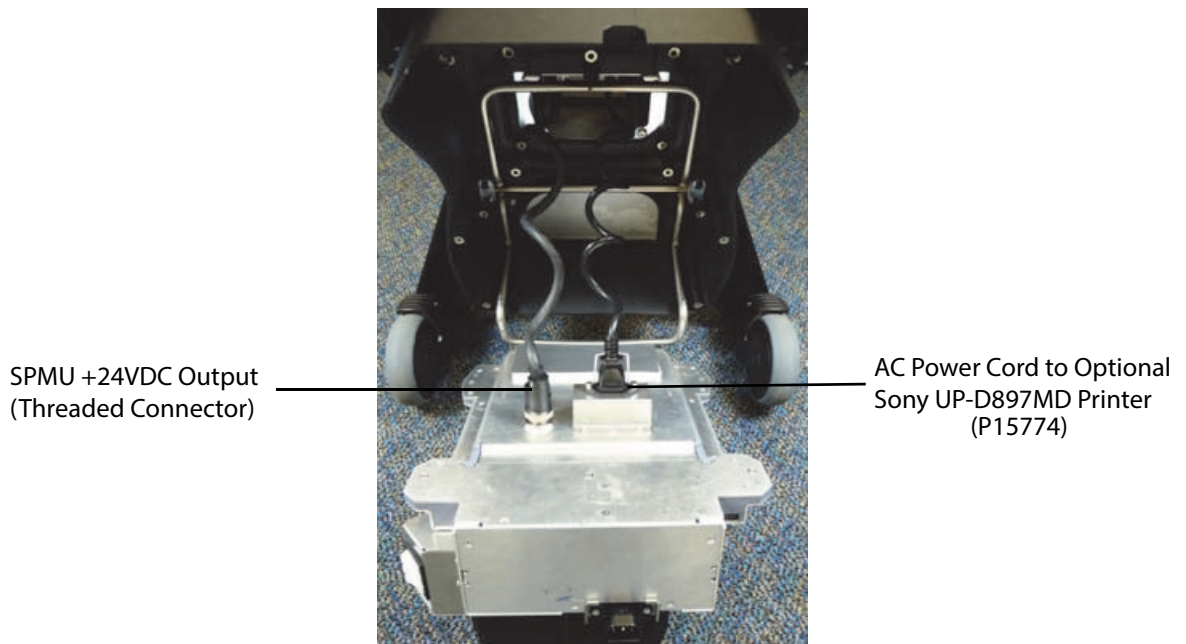


Figure 6.12 SPMU Power Connectors

## Stand Power Management Unit (SPMU) Replacement

### Stand Power Management Unit (SPMU) Replacement

- 1 Remove the clear plastic sheath protecting the gray thermal pad on the replacement SPMU assembly prior to installing in the Stand. Refer to [Figure 6.13 on page 44](#).

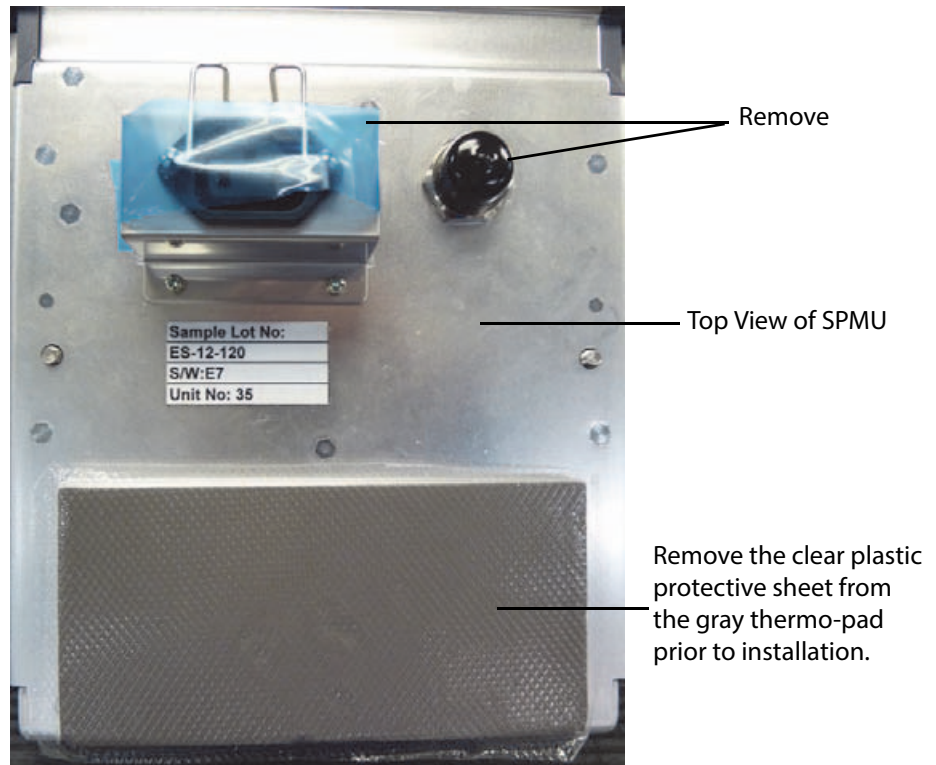


Figure 6.13 SPMU As Shipped

- 2 Follow the above SPMU Removal steps and perform in the reverse order for installation of the replacement SPMU.
- 3 Tighten the seven SPMU 5mm hex head mounting screws, the captive battery screws, and the six M5 x 8 Phillips screws holding the bottom plate to 20-inch pounds.

*Note: The Bottom Plate will mount inside out. When the Bottom Plate is incorrectly mounted, the PowerPark will not connect to the stand. Please ensure that the Bottom Plate has the top opening under the AC power cord IEC socket and that the two captive thread areas are not protruding out of the stand base. Refer to [Figure 6.14 on page 44](#)*

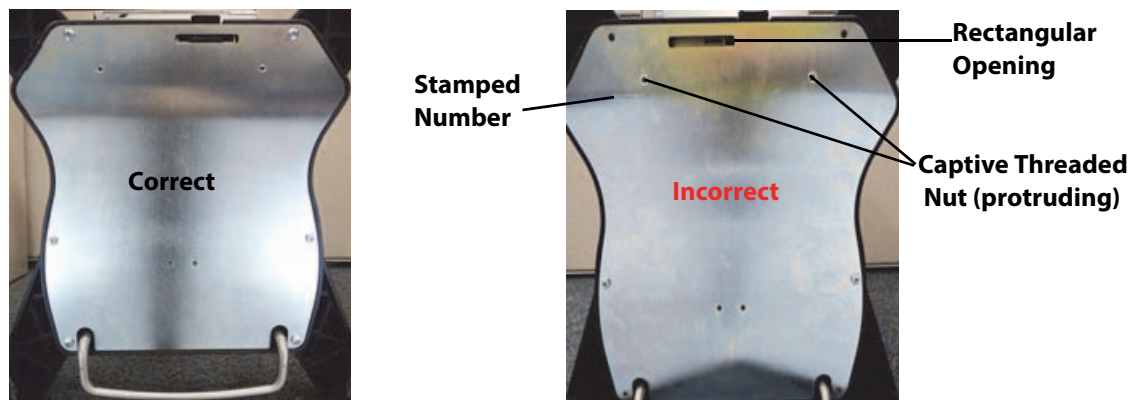


Figure 6.14 Correct Bottom Plate Orientation

- 4 Ensure that the three battery switches on the SPMU are set to "ⓘ" (ON) position after all service has been performed (tested and batteries installed) and the system is back in the upright position.

## Battery

The batteries in the X-Porte Stand are three Lithium-ion batteries and all three must be replaced at the same time. Upon ordering these batteries, they are packaged as one set of three batteries. Since the batteries mount directly to the SPMU module, the procedure for battery removal is explained during the SPMU module removal section.

### Required Part

- P14586 Battery, X-Porte Stand (Set of Qty. Three)

### Required Tools

- #1 Phillips screwdriver
- Torque screwdriver, 10.0-25.0 inch pounds (1.1-2.8 Newton meter)

### Battery Removal

#### Battery Removal

- 1 Please follow steps 1-12 of the “Stand Power Management Unit (SPMU) Removal” section of this chapter.

### Battery Replacement

#### Battery Replacement

- 1 Perform the battery replacement by following in reverse order steps 1-12 of the “Stand Power Management Unit (SPMU) Removal” section of this chapter.
- 2 Tighten the captive battery retention screws and the six M5 x 8 Phillips screws for the bottom stand plate to 20-inch pounds. Refer to [Figure 6.10 on page 42](#) and [Figure 6.14 on page 44](#)

## Triple Transducer Connect (TTC)

### Required Part

- P14644-N Service Assembly, TTC Assembly, X-Porte
- P14644-W Warranty Service Assembly, TTC Assembly, X-Porte
- V14644-N Vet Service Assembly, TTC Assembly, X-Porte
- V14644-W Vet Warranty Service Assembly, TTC Assembly, X-Porte

### Required Tools

- None

### Triple Transducer Connect (TTC) Removal

#### Triple Transducer Connect (TTC) Removal

- 1 Reach under the stand head area and unlatch the main TTC connector from the bottom of the stand head by flipping the latch handle forward and rotate counter clockwise. Refer to [Figure 6.15 on page 46](#)


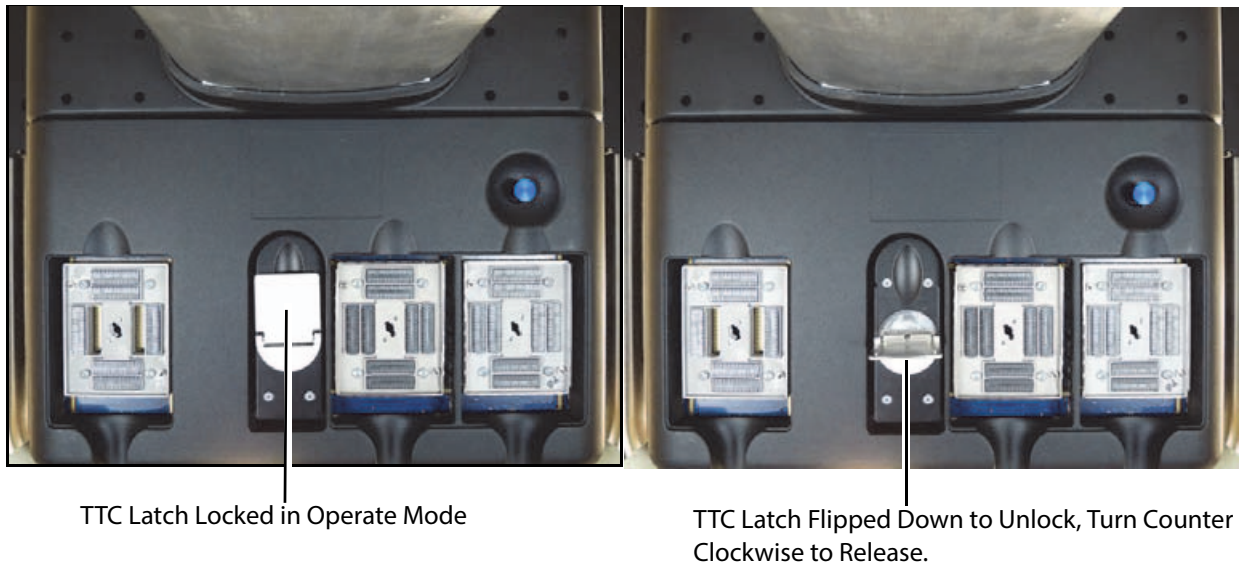


Figure 6.15 TTC Main Latch

- 2 When loose, pull the latch handle down and the connector should drop down several inches. The TTC is free from the System. Refer to [Figure 6.16 on page 46](#).

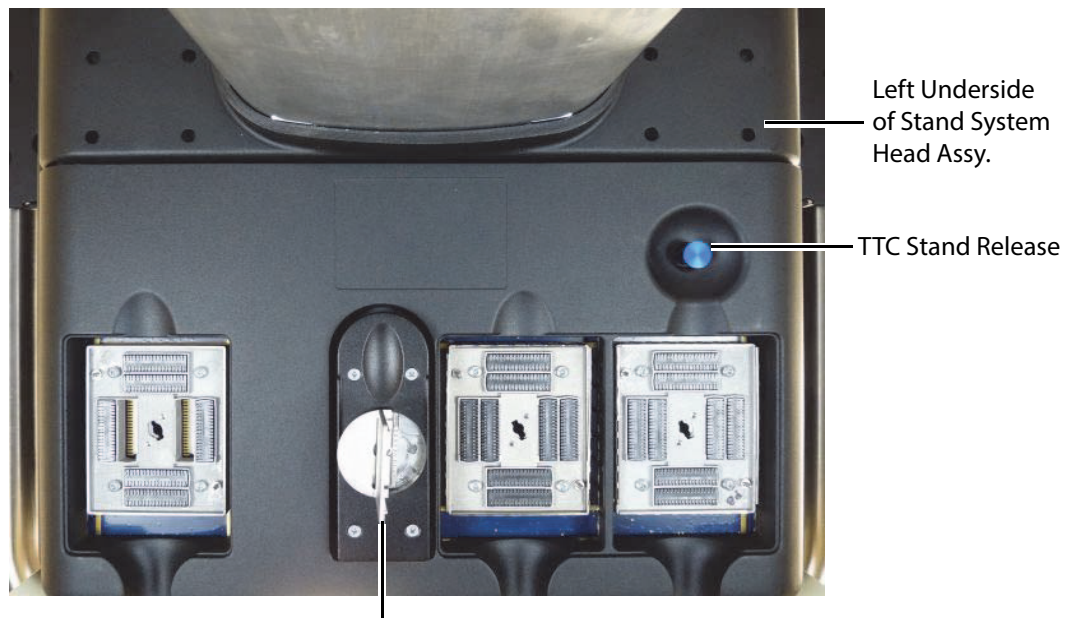

Note that the TTC Connector has Dropped Down & is Loose from the System.

Figure 6.16 TTC Stand Release

*Note: Updated TTC's now incorporate a secondary slide latch on the right side that requires the right hand to activate this latch while the left hand pulls the blue stand release pin while pushing the TTC to the left. This ensures that the TTC will be supported by both hands and not drop to the floor.*

- 3 Place the right hand at the right bottom section of the TTC and the left hand under the TTC next to the blue System Stand release pin. While pulling the blue pin downward and sliding the right secondary latch to release, push the TTC sideways to the left.
- 4 When the TTC protrudes to the left beyond the left stand edge, it will drop free from the System Stand. Be prepared to support the full weight of the TTC.

## Triple Transducer Connect (TTC) Replacement

### Triple Transducer Connect (TTC) Replacement

- 1 To mate the TTC to the stand, start by aligning the four stainless steel guide wheels on the top portion of the TTC assembly which slide into the four TTC guide channels located at the bottom of the stand System head.
- 2 Position the TTC approximately 3/4" farther to the left than the left stand edge to allow the TTC stainless steel guide wheels to enter the stand TTC guide channels. Please refer to [Figure 6.17 on page 47](#) for the location for these attachment points.

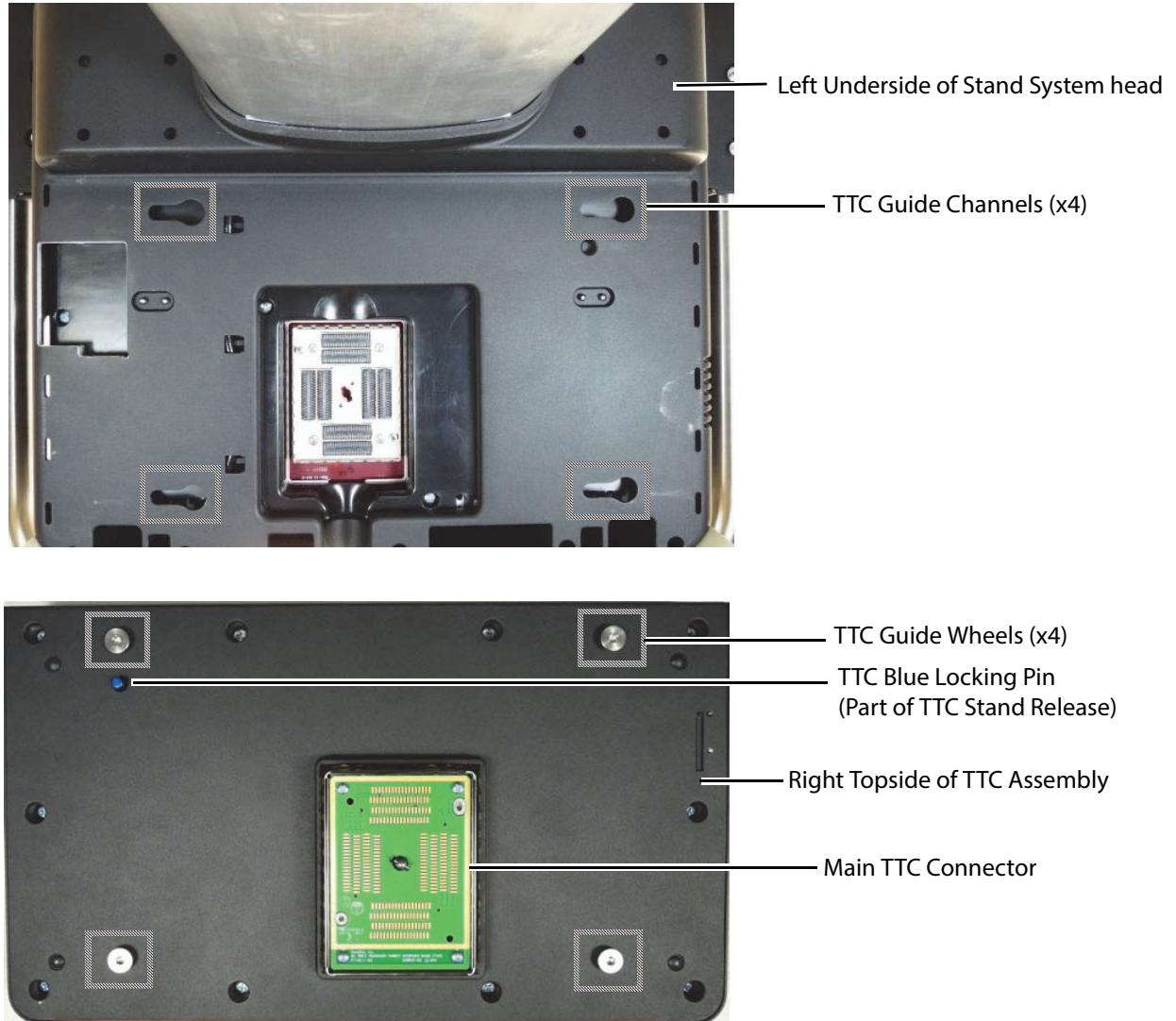


Figure 6.17 TTC Mounting Channels

- 3 Perform the TTC Removal steps 1-4 in the reverse order to complete the replacement operation.

# Optional Equipment - Stand System

## PowerPark

The PowerPark accessory is a cordless charging station for most FUJIFILM SonoSite system stands. It is comprised of two portions with the PowerPark Stand Module (P12822) mounted to the underside of the stand which then connects to the PowerPark Dock Module (P12834) that is connected to the wall AC and sits on the floor. In the event that the stand module needs to be replaced on a X-Porte Stand System, please perform the following procedures.

### Required Part

- P12822 Assembly, PowerPark Stand Module

### Required Tools

- #1 Phillips screwdriver
- #1 Phillips “stubby” screwdriver or ratchet type screwdriver (for repair without laying the stand on its side)

## PowerPark Stand Module Removal

### PowerPark Stand Module Removal

*Note: The following will outline the replacement method by tipping the stand onto its side to gain full view and access to the underside of the stand. If tipping the stand is not possible, then proceed to the “alternate” methods.*

- 1 Please refer to section in this chapter titled “Stand Power Management Unit (SPMU) Removal” and follow steps 1 through 11. Please refer to [Figure 6.7 on page 41](#), [Figure 6.8 on page 41](#), and [Figure 6.9 on page 42](#)
- 2 When the two M5 x 20mm Phillips (P13629) screws have been removed securing the PowerPark Stand Module, slide the module out from stand IEC power cord port.

## PowerPark Stand Module Replacement

### PowerPark Stand Module Replacement

- 1 Perform the PowerPark Stand Module replacement following the prior removal steps 1 through 2 in the reverse order.
- 2 Access the three battery switches and set all switches to the “**⓪**” (ON) position. Refer to [Figure 6.7 on page 41](#)

## PowerPark Stand Module Removal (Alternate Method-Upright System)

### PowerPark Stand Module Removal (Alternate Method)

- 1 Pull the X-Porte Stand System away from the PowerPark Dock Module or remove the AC power cord connected to the PowerPark Stand Module.
- 2 Access the three battery switches and set all switches to the “**•⓪**” (OFF) position. Refer to [Figure 6.7 on page 41](#).
- 3 While lying on the floor to gain access to the underside of the X-Porte Stand, use a “stubby” or ratchet type #1 Phillips screw driver to remove the two M5 x 20mm Phillips (P13629) screws.
- 4 Once the PowerPark Stand Module is loose, slide the module out away from the stand IEC power cord port.

## PowerPark Stand Module Replacement (Alternate Method-Upright System)

### PowerPark Stand Module Replacement (Alternate Method)

- 1 Perform the PowerPark Stand Module replacement following the “alternate” removal steps 1 through 4 in the reverse order.
- 2 Access the three battery switches and set all switches to the “**⓪**” (ON) position. Refer to [Figure 6.7 on page 41](#).



## Sony UP-D897MD Printer

The Sony UP-D897 digital thermal printer is a monochrome black and white printer which uses the USB 2.0 port to integrate to the X-Porte Stand and Desktop systems. This is an optional device and no special installation is necessary for the X-Porte Desktop system. The following instructions are for any replacement needs if the Sony UP-D897MD has to be replaced which is mounted to the X-Porte Stand (Kiosk) system.

### Required Part

- P16193-N Service Assembly, B&W Printer, X-Porte
- P16193-W Warranty Service Assembly, B&W Printer, X-Porte
- V16193-N Vet Service Assembly, B&W Printer, X-Porte
- V16193-W Vet Warranty Service Assembly, B&W Printer, X-Porte

### Required Tools

- #1 Phillips screwdriver
- 3mm hex long T-handle driver (optional)

## Sony UP-D897MD Printer Removal

### Sony UP-D897 Printer Removal


- 1 Raise or lower the Stand to a comfortable height to gain access to the printer shelf hardware.
- 2 From the rear of the Stand, loosen the captive screw that may be tightened to holding the printer cable shield closed. Loosen the screw and glide the screw through the channel to swing open the cover. Refer to [Figure 6.18 on page 49](#)
- 3 Remove the AC power cord and USB digital video cable from the printer.



Figure 6.18 Sony UP-D897MD Cable Cover

- 4 From the underside of the printer mounting shelf, remove the four M3-.5 x 8mm Pan Head Phillips screws (P15551) threaded into the Sony UP-D897MD printer. Refer to [Figure 6.19 on page 50](#)

*Note: Alternate method is to use a 3mm hex T-handle driver to loosen the hex screw in the clear mounting bracket and remove the printer shelf from the stand column. This method allows the four (P15551) M3-.5 x 8 mm screws to be removed and the printer replaced on a table and not from under the mounted shelf.*

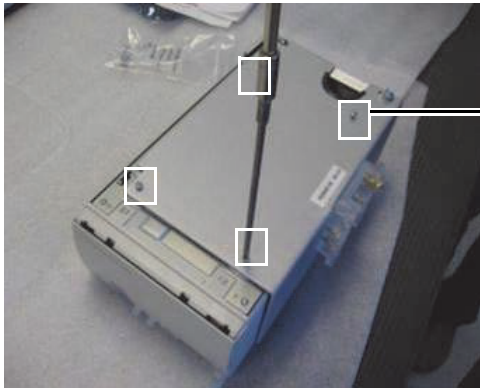

(P15551) M3-.5 x 8mm Pan Head Phillips Mounting Screws (x4)

**Note:** Viewed with the printer shelf removed from the Stand.

Figure 6.19 Stand Printer Shelve & Mounting Screws

- 5 Remove the Sony UP-D897MD printer from the shelf.

## Sony UP-D897MD Printer Replacement

### Sony UP-D897 Printer Replacement

- 1 Perform the Sony printer replacement by following the Sony UP-D897MD removal steps 1 through 5 in the reverse order.

# Chapter 7: Maintenance

This chapter contains information to direct you to where information about care for the system, transducers, and accessories can be obtained.

## Periodic Maintenance

No periodic or preventive maintenance is required for the system, transducers, or accessories other than cleaning and disinfecting the transducer after every use. (See the *“Cleaning and Disinfecting X-Porte Products”* guide (P16529). There are no internal adjustments or alignments required and there are no internal components that require periodic testing, calibration, adjustment, or alignment. Performance tests are described in [Chapter 8, “Performance Testing”](#) of this manual. Performing maintenance procedures not described in this manual may void the product warranty.

Local regulations may require electrical safety testing.

Contact FUJIFILM SonoSite Technical Support for any maintenance questions. (See [“Technical Support \(USA, Canada\)”](#) on page 1.)

## Cleaning and Disinfecting

Please refer to Warnings, Cautions, and Step by Step procedures outlined in the FUJIFILM SonoSite *“Cleaning and Disinfecting X-Porte Products”* guide (P16529)



# Chapter 8: Performance Testing

## Overview

**WARNING:** **Critical Test Function** — A failure of the system functions tested in this section could affect safety or effectiveness of the system adversely. While performing the steps in this section, verify that the images on the system display and on the external monitor are acceptable.

To obtain 2D images, FUJIFILM SonoSite recommends using the Gammex 403GS Soft Tissue Phantom or the Gammex 413A Multipurpose Phantom. A .7db/cm phantom is recommend but not required.

Some features and capabilities are optional and therefore may be unavailable to test.

## Recommend Test Equipment

- FUJIFILM SonoSite ultrasound system under test
- C60xp/5-2 MHz transducer
- P21xp/5-1 MHz transducer
- Gammex 403 GS Multipurpose Phantom, 413A Soft Tissue Phantom, or equivalent.
- Acoustic gel

## Setting Up Performance Tests

---

<b>Set up Performance Tests</b>	1	Attach the C60xp/5-2 MHz transducer to the system.
	2	Select Gen for optimization and OB for exam type.
	3	Couple the transducer to the phantom, adjusting gain settings and transducer for a proper phantom image (e.g., pins are high-level echoes positioned in straight lines; cysts are sonolucent, X-Portes are sharp, and graphite particles of the phantom are mid-grays).

## Basic Operational Tests

---

- |                                     |   |
|-------------------------------------|---|
| <b>Basic System Operation Tests</b> | <ol style="list-style-type: none"><li>1 Verify that the correct transducer name appears in the upper right corner of the system display.</li><li>2 Verify proper date and time.</li><li>3 Verify that the scan plane orientation mark in the image located near the skinline corresponds to element #1 on the transducer. To test, put your finger on the probe and run it across the transducer face. Your finger touching the transducer face should appear at the orientation mark on the display image format.</li><li>4 Verify that all of the keyboard keys are functional. Verify that all controls operate smoothly over their full range and that the system responds properly.</li><li>5 Verify that as the Gain controls are increased and decreased, there is a corresponding increase and decrease in echo intensity.</li><li>6 Capture a Cineloop buffer. Exercise the Cineloop controls and verify proper operation.</li><li>7 Verify the airflow from the fan vents on the rear of the Engine flow outward.</li></ol> |
|-------------------------------------|---|

## 2D Performance Tests

### 2D Performance / Image Quality

---

- |  |   |
|--|---|
| <b>Test 2D Performance and Image Quality</b> | <ol style="list-style-type: none"><li>1 Use a C60xp/5-2 MHz transducer in 2D mode.</li><li>2 Adjust the position of the C60xp/5-2 MHz transducer on the phantom.</li><li>3 With the array pointing down and the orientation mark to the operator's left, element #1 corresponds with the left side of the array.</li><li>4 Use the 2D system controls to obtain a clear image that shows both the horizontal and vertical rows of pins.</li><li>5 Verify that the ultrasound image appears uniform in both the axial and lateral direction, with no dropouts or intensity variations.</li><li>6 Verify that the cystic structure at the focal zone is clearly differentiated from the surrounding tissue and is echo-free, while solid tissue with numerous echo sources, appears solid.</li><li>7 Press the <b>Freeze</b> key and then save the image. Slide the <b>Freeze Unlock</b> key to return to live imaging.</li></ol> |
|--|---|

## Axial Measurement Accuracy

Note: Measurements must be performed while the image is frozen.

---

<b>Set Up Axial Measurement Accuracy</b>	1	Acquire the image.
	2	Press the <b>Freeze</b> key.
	3	Press the <b>Caliper</b> key. The caliper appears on the image display. (See the <i>X-Porte Ultrasound System User Guide</i> , if necessary, for caliper operation.)
	4	Use the touchpad to position one of the calipers.
	5	Press the <b>Select</b> key to fix the caliper and enable the other caliper.
	6	Use the touchpad to move the other caliper. The results update as you move the caliper, and the measurement is complete when you finish moving the calipers. (Press the <b>Select</b> key to alternate the active caliper, and adjust the measurement with the touchpad.)

---

<b>Test Axial Measurement Accuracy</b>	1	Measure the distance, center to center, of any two pins that are 5-12 cm apart vertically.
	2	Verify that the distance measured is within the tolerance listed in <a href="#">Table 8.1</a> .

## Lateral Measurement Accuracy

---

<b>Set Up Lateral Measurement Accuracy</b>	Perform “ <a href="#">Set Up Axial Measurement Accuracy</a> ” on page 55.	
--	---	--

---

<b>Test Lateral Measurement Accuracy</b>	1	Measure the distance, center to center, of any two pins that are 4-10 cm apart horizontally.
	2	Verify that the distance measured is within the tolerance listed in <a href="#">Table 8.1</a> .
	3	Slide the <b>Freeze Unlock</b> to return the system to live 2D mode.

**Table 8.1: System Measurement Accuracy**

---

Measurements	Tolerance
Axial Distance	+/- 2%
Lateral Distance	+/- 2%

---

## Penetration

The penetration measurement is an integral part of the quality assurance program. Penetration is defined as the deepest depth at which an ultrasound system can provide adequate image quality of small anatomical structures.

Penetration measurements should be performed and the results retained for comparison to future measurements. Penetration measurements should remain fairly consistent over time assuming use of the same system settings and scanhead. Degradation of the penetration measurement in excess of 1 cm may indicate a transducer or system electronics issue.

Loss of measured penetration may also be caused by degradation (desiccation) of the ultrasound phantom. Ultrasound phantoms used for penetration measurements must also be part of a quality assurance program to maintain their integrity. Follow all of the phantom manufacturer recommendations for use, storage, and maintenance of the phantom.

---

<b>Test Penetration</b>	<ol style="list-style-type: none"><li>1 Use the same scanhead and system settings as previous measurements if possible.</li><li>2 Adjust the system controls to obtain a clear image that shows the limits of echo penetration.</li><li>3 Press the <b>Freeze</b> key and then save the image.</li><li>4 Measure from the center of the skinline to the deepest vertical position—where the scatter echoes start to break up and tissue definition is lost.</li><li>5 Record and retain the results for future reference. Scanhead type and system settings (exam type, depth, resolution mode, etc.) should also be recorded to ensure proper comparison with future tests.</li><li>6 Slide the <b>Freeze Unlock</b> to return to live imaging.</li></ol>
-------------------------	--

## Additional Performance Tests

### Color Doppler (Color)

---

<b>Test Color</b>	<ol style="list-style-type: none"><li>1 Connect any transducer.</li><li>2 Press the <b>Color</b> key. "Color" should be annotated in the top left corner of the display.</li><li>3 A Region of Interest (ROI) box is displayed on top of the grayscale image. Use the Touch Control Panel or Desktop System mouse to move the Color ROI. Verify that the ROI moves to the new position on the display.</li><li>4 Adjust the <b>Depth</b> control for minimum depth in the image.</li><li>5 Adjust the <b>Gain</b> control so that color speckles just appear inside the ROI box.</li><li>6 Gently tap the face of the transducer and observe that the ROI box fills with color information.</li><li>7 Press the <b>Freeze</b> key and then save the image. Slide the <b>Freeze Unlock</b> to return to live imaging.</li></ol>
-------------------	--



## Color Power Doppler (CPD)

---

- Test CPD**
- 1 Connect any transducer.
  - 2 Press the **Color** key. A Region of Interest (ROI) box is displayed on top of the grayscale image.
  - 3 Press the **Color** softkey to switch to CPD. "CPD" should be annotated in the lower right corner of the display.
  - 4 Adjust the **Depth** control for minimum depth in the image.
  - 5 Adjust the **Gain** control so that color speckles just appear inside the ROI box.
  - 6 Gently tap the face of the transducer and observe that the ROI box fills with color information.

## M Mode Imaging

---

- Test M Mode Imaging**
- 1 Attach a C60xp transducer and acquire an image.
  - 2 Press the **M Mode** key for the M Mode sample line.
  - 3 Position the M Mode sample line over the image using the Touch Control Panel or the Desktop System mouse.
  - 4 Active the **Start** key to the right of the **M Mode** key to turn on M Mode.
  - 5 Select the desired sweep speed from the on-screen menu (slow, med, or fast). The on-screen menu will show the selected sweep speed.
  - 6 Press the **Freeze** key to freeze the image. Save the image. Slide the **Freeze Unlock** to return to live imaging.
  - 7 Press the **2D** key to return to 2D imaging.

## Tissue Harmonic Imaging

---

- Test THI Imaging**
- 1 Attach the C60xp transducer and acquire an image.
  - 2 Set the depth to maximum and note the depth at which echo information is lost.
  - 3 Press the **THI** key on the control panel so it displays THI on the display. Tissue Harmonic Imaging is now active.
  - 4 Observe a decrease in dot size and a significant loss in penetration due to the higher frequency. Image resolution increases.
  - 5 Press the **Freeze** key and then save the image. Slide the **Freeze Unlock** to return to live imaging.
  - 6 Press the **THI** key again to turn off Tissue Harmonic Imaging.

## Pulsed Wave (PW) Doppler Imaging

---

- |  |   |
|--|---|
| <b>Test PW<br/>Doppler<br/>Imaging</b> | <ol style="list-style-type: none"><li>1 Attach the P21xp transducer.</li><li>2 Press the <b>Doppler</b> key for the Doppler sample gate.</li><li>3 Press the <b>Doppler</b> key again for the Doppler spectral trace.</li><li>4 Place a large drop of ultrasound gel on the transducer lens.</li><li>5 Adjust the <b>Gain</b> control as necessary and then gently tap the top of the gel and observe a reflection on the spectral trace and the sound from the speakers.</li><li>6 Press the <b>Freeze</b> key and then save the image. Slide the <b>Freeze Unlock</b> to return to live imaging.</li><li>7 Press the <b>2D</b> key to return to 2D imaging.</li></ol> |
|--|---|

## Continuous Wave (CW) Doppler Imaging

---

- |  |   |
|--|---|
| <b>Test CW<br/>Doppler<br/>Imaging</b> | <ol style="list-style-type: none"><li>1 Attach the P21xp transducer.</li><li>2 Press the <b>Patient</b> key.</li><li>3 Select the <b>Cardiac</b> exam type.</li><li>4 Press the <b>Done</b> softkey.</li><li>5 Press the <b>Doppler</b> key for the Doppler sample gate.</li><li>6 Press the <b>PW</b> softkey to switch to CW Mode.</li><li>7 Press the <b>Doppler</b> key again for the Doppler spectral trace.</li><li>8 Place a large drop of ultrasound gel on the transducer lens.</li><li>9 Adjust the <b>Gain</b> control as necessary and then gently tap the top of the gel and observe a reflection on the spectral trace and the sound from the speakers.</li><li>10 Press the <b>Freeze</b> key and then save the image. Slide the <b>Freeze Unlock</b> to return to live imaging.</li><li>11 Press the <b>2D</b> key to return to 2D imaging.</li></ol> |
|--|---|

## Image Quality Verification Test/Livescan

- Products with replaced subassemblies, or products that have been otherwise disassembled, must undergo an Image Quality Verification Test/Livescan.
- The Image Quality Verification Test/Livescan should be performed after successfully completing all applicable performance tests listed prior in this chapter.
- The test is completed before returning the system to service.
- A certified sonographer must perform the test.
- The Livescan test performed is at the discretion of the Sonographer and will represent their acceptance of a successful service event.
- Review all saved images and verify that the images are displayed properly.

## Printer


The printer test is an optional test that requires the Sony UP-D987MD USB Black & White Printer (P16193) to be connected to the system under test. Skip this test if a printer is not available.

- 
- |                     |   |  |
|---------------------|---|--|
| <b>Test Printer</b> | 1 | Press the print button and verify that the printer begins to print an image. |
| <b>Operation</b>    | 2 | Verify the proper content of the printed image.                              |



# Appendix A: Replacement Parts

The following tables contain all the field-replaceable parts for the X-Porte ultrasound system. Quantities are one unless otherwise noted. Please refer to [Figure A.1 on page 61](#) to identify the major components of the two X-Porte System variants which are the Desktop and Stand systems.

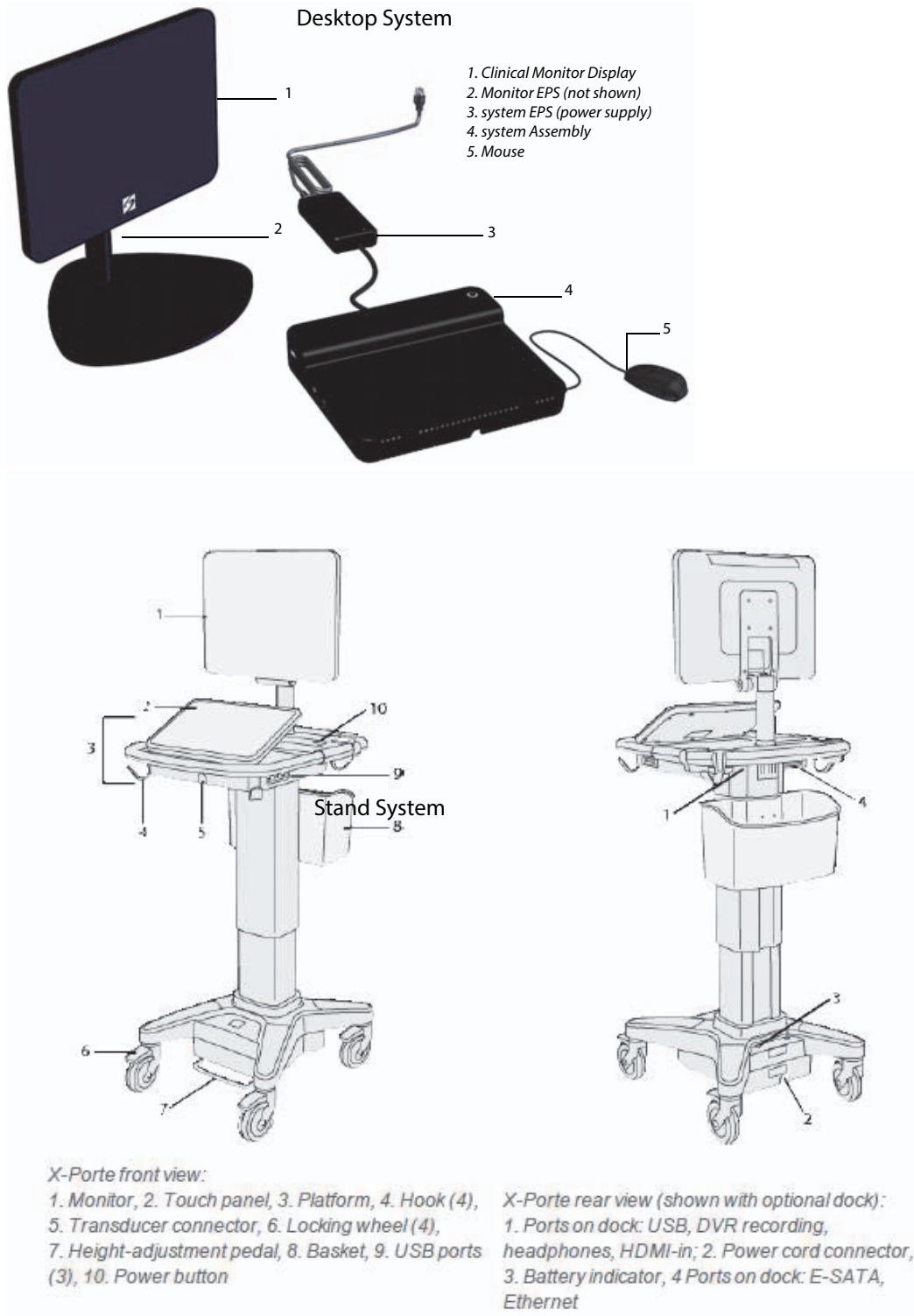

**Figure A.1 X-Porte Major Components**

## Clinical Monitor



**Figure A.2 Clinical Monitor Display**

**Table A.1: Clinical Monitor Display**

Part Number	Description
P14627-N	Service Assembly, Display, Clinical Monitor, X-Porte
P14627-W	Warranty Service Assembly, Display, Clinical Monitor, X-Porte
V14627-N	Vet Service Assembly, Display, Clinical Monitor, X-Porte
V14627-W	Vet Warranty Service Assembly, Display, Clinical Monitor, X-Porte

**Table A.2: Clinical Monitor Display Associated Hardware**

Part Number	Quantity	Description
P16380	1	HDMI Wire Bracket, Clinical Monitor
P16639	1	HDMI Wire Clamp, Clinical Monitor
P15551	2	M3-.5 x 8mm Screw, Pan Head Phillips (used to mount P16380 to P14627)
P16390	1	Cable Door, Monitor Mount
P16859	1	Mount, Clinical Monitor
P16369	4	M4-.7 x 12 Black Screw, Short (used to mount P14627 to P16859)
P16371	2	M4-.7 x 35 Black Screw, Long (used to mount P14627 to P16859)
P16773-N	1	Service Cable Assembly, Clinical Monitor Stalk Cable Set, M2

## Touch Control Panel



Figure A.3 Touch Control Panel


Table A.3: Touch Control Panel

Part Number	Description
P14027-N	Service Assembly, Control Panel, X-Porte
P14027-W	Warranty Service Assembly, Control Panel, X-Porte
V14027-N	Vet Service Assembly, Control Panel, X-Porte
V14027-W	Vet Warranty Service Assembly, Control Panel, X-Porte

Table A.4: Touch Control Panel Associated Hardware

Part Number	Quantity	Description
P16374	4	M4 External Tooth Washer, Silver (used with P16375 to mount P14027 to Stand)
P16375	4	M4 x 12, Socket Cap Screw, Black (used with P16374 to mount P14027 to Stand)

# Dock

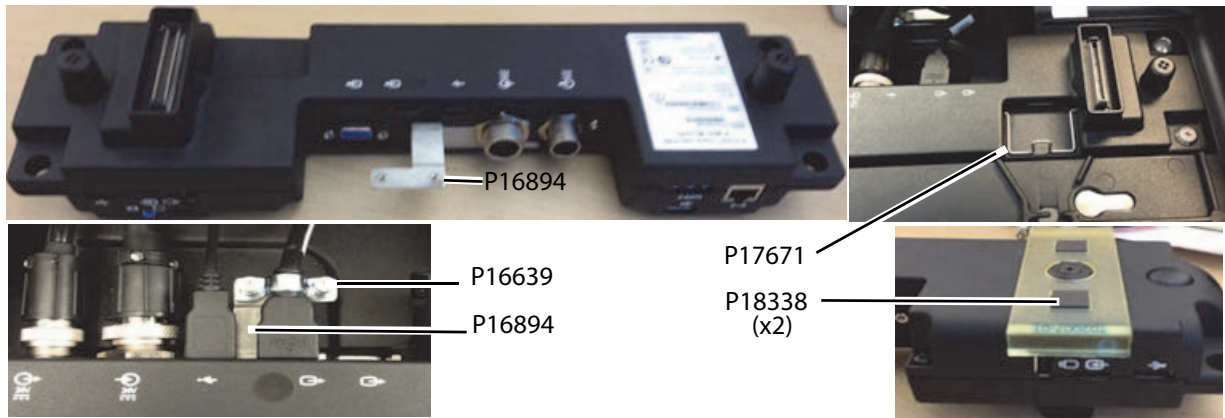

**Figure A.4 Dock**

**Table A.5: Dock**

Part Number	Description
P15164-N	Service Assembly, Dock with DVR, X-Porte
P15164-W	Warranty Service Assembly, Dock with DVR, X-Porte
V15164-N	Vet Service Assembly, Dock with DVR, X-Porte
V15164-W	Vet Warranty Service Assembly, Dock with DVR, X-Porte

**Table A.6: Dock Associated Hardware**

Part Number	Quantity	Description
P16355	4	M4 Washer, Stainless Steel
P16356	4	M4 Flathead Phillips Screw, Stainless Steel
P16639	1	**HDMI Wire Clamp, Clinical Monitor (used with P16894)
P16894	1	**HDMI Cable Bracket, Dock (used with P16639)
P17671	1	**Retainer, Wire, Cable Connector, M2 Control Panel
P18338	2	**Dock Stabilization Foot, PORON, M2

*Note: \*\*Recent production change to the Dock Assembly. Replacement Assembly may require these additional parts to be installed to the Dock Assembly prior to installation in the X-Porte Stand.*



## “Ultrasound” Engine


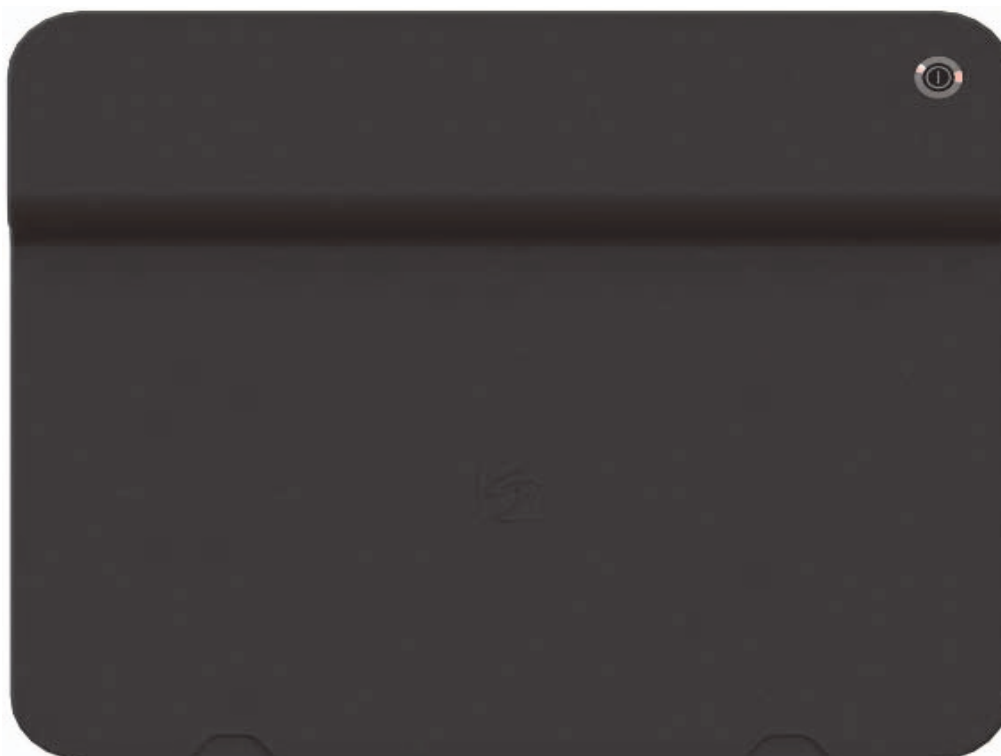


Figure A.5 Engine

Table A.7: “Ultrasound” Engine

Part Number	Description
P16055-N	Service Assembly, X-Porte Engine, CW
P16055-W	Warranty Service Assembly, X-Porte Engine, CW
V16055-N	Vet Service Assembly, X-Porte Engine, CW
V16055-W	Vet Warranty Service Assembly, X-Porte Engine, CW

## Stand Power Management Unit (SPMU & Batteries)

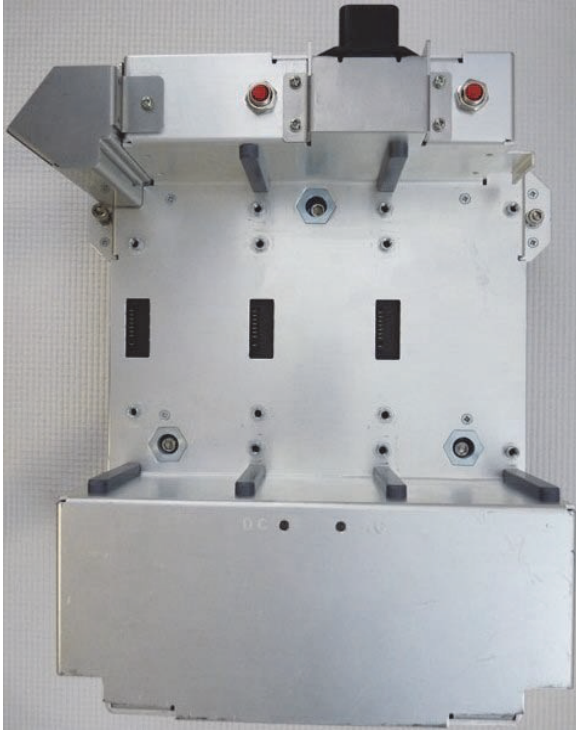

\*Batteries Only Ordered in Sets of 3  
(All 3 Batteries Must Be Replaced as a Set)

**Figure A.6 SPMU & Batteries**

**Table A.8: Stand Power Module Unit (SPMU & Batteries)**

Part Number	Description
P14522-N	Service Assembly, Stand Power Management Unit, X-Porte
P14522-W	Warranty Service Assembly, Stand Power Management Unit, X-Porte
V14522-N	Vet Service Assembly, Stand Power Management Unit, X-Porte
V14522-W	Vet Warranty Service Assembly, Stand Power Management Unit, X-Porte
P14586	Battery, X-Porte Stand (Set of Qty. Three)

## Triple Transducer Connect (TTC)



**Figure A.7 Triple Transducer Connect (TTC)**

**Table A.9: Triple Transducer Connect (TTC)**

<b>Part Number</b>	<b>Description</b>
P14644-N	Service Assembly, TTC Assembly, X-Porte
P14644-W	Warranty Service Assembly, TTC Assembly, X-Porte
V14644-N	Vet Service Assembly, TTC Assembly, X-Porte
V14644-W	Vet Warranty Service Assembly, TTC Assembly, X-Porte

## Clinical Monitor Hardware

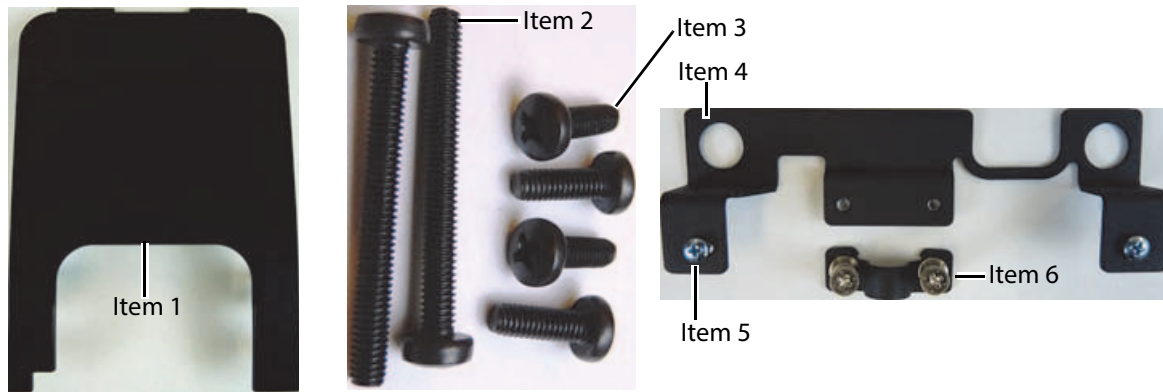


Figure A.8 Clinical Monitor Hardware

Table A.10: Clinical Monitor Hardware

Item	Part Number	Description
1	P16390	Cable Door, Monitor Mount
2	P16371	M4-.7 x 35 Black Screw (x2)
3	P16369	M4-.7 x 12 Black Screw (x4)
4	P16380	HDMI Wire Bracket, Clinical Monitor
5	P15551	M3-.5 x 8mm Screw, Pan Head Phillips (x2)
6	P16639	HDMI Wire Clamp, Clinical Monitor
Figure 6.2	P16859	Mount, Clinical Display

## Control Panel Hardware

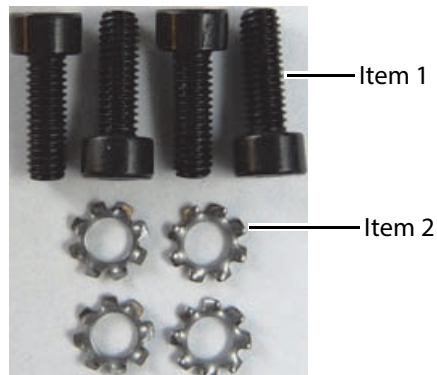


Figure A.9 Control Panel Hardware

Table A.11: Control Panel Hardware

Item	Part Number	Description
1	P16375	M4 x 12, Socket Cap Screw, Black (x4)
2	P16374	M4 External Tooth Washer, Silver (x4)

## Dock Hardware



Figure A.10 Dock Hardware

Table A.12: Dock Hardware

Item	Part Number	Description
1	P16355	M4 Washer, Stainless (x4)
2	P16355	M4 Flathead Phillips Screw (x4)
Figure A.4	P16639	HDMI Wire Clamp, Clinical Monitor
Figure A.4	P16894	HDMI Cable Bracket, Dock
Figure A.4	P17671	Retainer, Wire, Cable Connector, M2 Control Panel
Figure A.4	P18338	Dock Stabilization Foot, PORON, M2

## Stand Hardware Parts

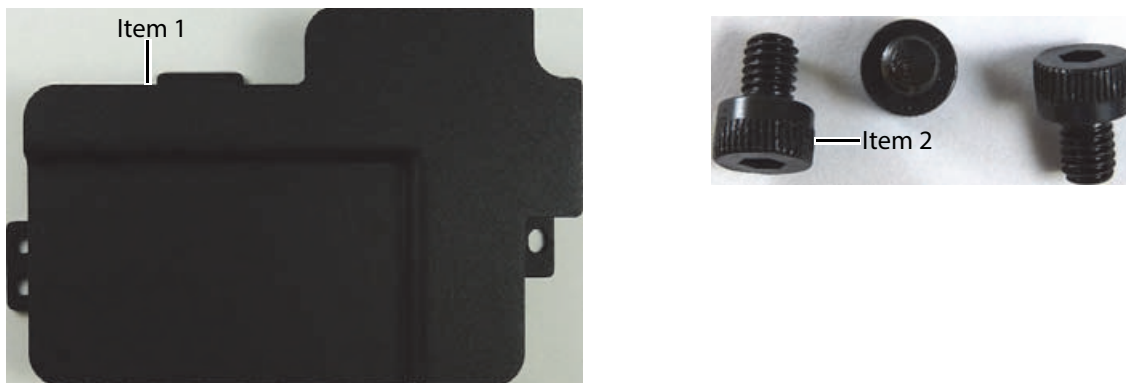


Figure A.11 Stand Hardware Parts

Table A.13: Stand Part

Item	Part Number	Description
1	P16733	X-Porte, Stand, Cable Baffle Plate
2	P16799	X-Porte, Stand, Screw, M4x5 (x3)
Figure 6.9	P16382	Screw, M5 X 8, Pan Head Phillips (x6) - (Bottom Stand Plate)
Figure 6.9	P13629	Screw, Pan Head, M5 X 20mm (x2) - (Power Park Stand Module)

# Optional Equipment

## PowerPark



(P12834) Assembly, PowerPark Dock Module

(P12822) Assembly, PowerPark Stand Module

*\*Note: Shown with the H-Universal Stand*

**Figure A.12 PowerPark**

**Table A.14: PowerPark**

Part Number	Description
P12822	Assembly, PowerPark Stand Module (Mounts to Stand)
P12834	Assembly, PowerPark Dock Module (Floor Portion Connected to AC Source)
P13629	Screw, Pan Head M5 x 20mm, Steel) (Mounts P12822 to Stand Bottom Plate)

# Sony UP-D897MD B&W Printer



**Figure A.13 Sony UP-D897MD B&W Printer**

**Table A.15: Sony UP-D897MD B&W Printer & Hardware**

<b>Part Number</b>	<b>Description</b>
P16193-N	Service Assembly, B&W Printer, X-Porte
P16193-W	Warranty Service Assembly, B&W Printer, X-Porte
V16193-N	Vet Service Assembly, B&W Printer, X-Porte
V16193-W	Vet Warranty Service Assembly, B&W Printer, X-Porte
P15551	Screw, Pan Head, Phillips, M3-.5 x 8mm, Steel (x4)
P15333	Cable Assembly, USB, Dock To Printer, M2
P15774	Cable, M2, AC, SPMU To Printer

# Desktop System Components

## EPS for Desktop System (Engine)

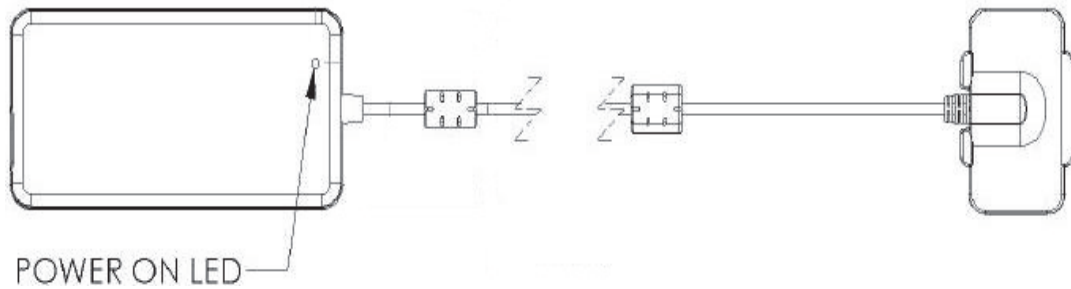


Figure A.14 EPS for Desktop System (Engine)

Table A.16: EPS for Desktop System (Engine)

Part Number	Description
P14521	AC Power Supply, X-Porte, 150 W

## EPS for Desktop Clinical Monitor Display



Figure A.15 EPS for Desktop Clinical Monitor Display

Table A.17: EPS for Desktop Clinical Monitor Display

Part Number	Description
P16776	Power Supply, 65W, 24V, X-Porte, Clinical Monitor



## Other Desktop Components



**Figure A.16 Other Components**

**Table A.18: Other Components**

Item	Part Number	Description
1	P02296	Transformer, Isolation
2	P16786	Pedestal Mount, X-Porte Monitor, 75 mm X 75 mm VESA
3	P14451	Mouse, Optical, 5 Button, Black, Washable, 6 ft. Cable
Not Shown	P15334	Cable Assembly, HDMI, Dock To Clinical Monitor, M2
Not Shown	P16967	USB Memory Stick, 16GB, X-Porte (Used for both Desktop & Stand Systems)

## Ordering Replacement Parts

To order parts, contact FUJIFILM SonoSite Technical Support as indicated in [“Contact Information”](#) on page 1.



## Appendix B: Service Event Reporting

The Service Event Report provides information about product failures to the manufacturer and to authorized service facilities, which provide approved warranty services for FUJIFILM SonoSite products. For all repairs completed, complete the form and email a copy of it to [service@FUJIFILM SonoSite.com](mailto:service@FUJIFILM SonoSite.com) or mail to the following address:

FUJIFILM SonoSite, Inc.  
Technical Support  
21919 30th Drive SE  
Bothell, Washington 98021  
USA

To contact FUJIFILM SonoSite Technical Support, see [“Contact Information”](#) on page 1.

# Service Event Report Form



## Service Event Report

Instructions on reverse

Choose a Service Type

Choose a Part Status

Service Type (check one)
<input type="checkbox"/> Out of Box Failure
<input type="checkbox"/> Warranty Service
<input type="checkbox"/> Out of Warranty Service

Parts Status (check one)
<input type="checkbox"/> No parts necessary for this repair. Service Event Report for your information.
<input type="checkbox"/> I need parts for this repair (list the parts below and attach Purchase Order)
<input type="checkbox"/> I need parts to replenish my stock (list the parts used below and attach Purchase Order)
<input type="checkbox"/> Will not replenish stock. Please give me a RMA for the return of the faulty parts.
<input type="checkbox"/> No parts necessary. Please issue a RMA for repair at SonoSite.

For SonoSite Use Only	
Service Request	
Order Number	
RMA Number	
Work Order	

Enter complete contact information

### Service Provider

Name:		Provider Reference:	
Company:		Date Reported:	
Address:			
Phone Number:		Fax Number:	
E-mail address:			

Enter product information for the system being repaired

### Device Description

Ref Number:		Serial Number:	
Name:		Lot Number:	
ARM/SHDB Version:		Configuration:	

### Problem Found

--

Put the system configuration here. You can find this on the System Information Page

Enter DETAILED problem description here.

### Service Performed

Performed By:		Date:	

### Parts Removed

Part Name	Part Number	Serial Number	Lot Number	Rev	Replace

Enter details for parts being removed from the system.

Enter details for parts being installed into the system.

### Parts Installed

Part Name	Part Number	Serial Number	Lot Number	Rev	Replace

### Tests Performed (attach test data)

Test:		Test:	
-------	--	-------	--

# Service Event Report Instructions

## Instructions for completing the Service Event Report

Sections highlighted in yellow must be completed for SonoSite to accept the Service Event Report. If additional information is required for certain circumstances you will be advised.

### Forward the completed form to:

Email: [service@sonosite.com](mailto:service@sonosite.com)

Fax: +1-425-951-6700

### Service Type

- **Out of Box Failure:** the item has arrived from SonoSite with failures.
- **Warranty Service:** the item has failed after arrival and is covered by either the included warranty or a valid extended warranty.
- **Out of Warranty Service:** the item has failed and it is no longer covered by a warranty.

### Parts Status

- Check One.

### Service Provider

- **Name:** the name of the technician performing the work.
- **Provider Reference:** a unique number used by the Provider to track Service Event Reports. Any format is acceptable.
- **Company:** the name of the Distributor or authorized repair facility.
- **Address:** the address replacement parts will be shipped to.
- **Date Reported:** the date the failure was reported to SonoSite.
- **Phone Number:** the phone number to contact the service technician.
- **Fax Number:** the fax number to contact the service technician.
- **Email Address:** the email address to contact the service technician.

### Device Description:

- **Name:** the description of the failed product.
- **Ref Number:** the reference number from the part number label of the failed product.
- **Serial Number:** the serial number from the part number label of the failed product.
- **Lot Number:** if applicable, the Lot Number from the device identification label.
- **ARM/SHDB Version:** the software level of the failed device. Typically found on the system information screen.
- **Configuration:** for configurable devices, the optional features enabled.

### Event Description

- A description of the problem in the words of the user. Typically what the user reports to the repair facility.

### Diagnosis

- A description of what the repair technician found. Include a list of the suspect parts.

### Service Performed

- A description of the work performed to repair the system. Typically only completed if it is repaired from stock repair parts.

### Parts Removed

- **Part Name:** the name of the failed/suspect part to be replaced.
- **Part Number:** the part number of the failed/suspect part.
- **Serial Number:** the serial number from the failed/suspect part.
- **Lot Number:** the lot number if applicable.
- **Rev:** the revision of the failed/suspect part if available.
- **Replaced By:** the person replacing the part.

### Parts Installed

- The same information as the Parts Removed except from the parts installed if work has already been performed. If you are waiting for parts to be ordered, leave this section blank.

### Tests Performed

- The results of any testing performed, if testing has already been performed.

## Returning Products to FUJIFILM SonoSite

You will be asked to provide the following information:

- Contact name and phone number
- Product name
- Serial number
- Description of the problem

### Shipping Instructions

Please contact FUJIFILM SonoSite to get a return material authorization number (RMA). Contact FUJIFILM SonoSite before returning any product.

The shipping address for all returned products is:

FUJIFILM SonoSite, Inc.  
Attn: Technical Support RMA \_\_\_\_\_  
21919 30th Drive SE  
Bothell, Washington 98021  
USA

# INDEX

## Numerics

- 2D performance tests
  - axial measurement accuracy 55
  - image quality 55
  - lateral measurement accuracy 55
  - penetration 56

## A

- assert code 24
- assistance, customer 1

## B

- battery
  - safety 7
  - specifications 4

## C

- cautions, definition 2

## E

- electrical
  - safety 5
  - specifications 3
- equipment safety 7
- error message 6

## F

- Failure Reporting 75

## H

- humidity limits 4

## I

- image
  - quality verification test 58
  - review 59

## M

- main PCBA
  - failures 24

## P

- performance tests
  - 2D 54
  - CPD 57
  - CW 58
  - M-Mode 57
  - overview 53
  - printer 59
  - PW 58
  - THI 57
  - Velocity Color 56
- periodic maintenance 51
- pressure limits 4
- printer
  - test 59
- Product Failures 75

## R

replacement parts

list 61

ordering 73

Return Material Authorization number (RMA) 78

Returning Products 78

## S

safety

battery 7

electrical 5

equipment 7

Service Event Report 75

Shipping Instructions 78

shipping specifications 4

storage specifications

equipment 4

subassembly replacement 23

system

measurement accuracy 55

overview 9

## T

temperature limits 4

theory of operation 9

## U

user guide, conventions used 2

## W

warnings, definition 2





P16620-01

