Aviso Urgente de Segurança

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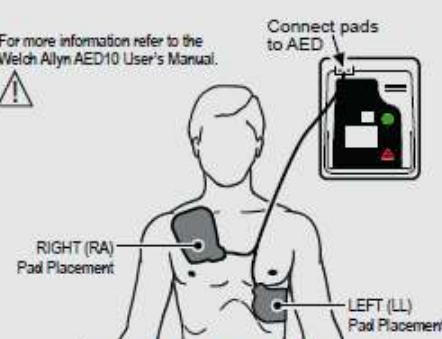
A

Número de série

Power Input: 12V DC, Internally Powered
 Use only Welch Allyn battery: 00185-2 (Non-Rechargeable)
 Enclosure Protection: IP24
 Manufacturer: Welch Allyn Protocol Inc.
 Beaverton Oregon USA
 Patents Pending
 Made in USA

INSTRUCTIONS FOR USING THE WELCH ALLYN AED10

For more information refer to the Welch Allyn AED10 User's Manual.

PLACE PADS ON BARE CHEST

If the patient is unconscious, not breathing, and unresponsive:

- 1
▶
Press the Green "On/Off" Button to turn system on.
- 2
▶
Follow voice and screen prompts.
 - Apply pads to patient's bare chest as shown in diagram.
 - Connect pads to AED.
- 3
▶
If prompted, press Flashing Red "Shock" Button.

DANGER

Explosion Hazard. DO NOT use in the presence of flammable anesthetics, concentrated oxygen, or other flammable gases.

CAUTION

HAZARDOUS ELECTRICAL OUTPUT. This equipment is for use only by qualified personnel.

ELECTRIC SHOCK HAZARD. DO NOT remove cover. Refer servicing to qualified service personnel.

DO NOT sterilize except as directed in the operating instructions. Use only the accessories supplied with this equipment.

WARNING

Federal law restricts this device to sale by or on the order of a physician. Be sure no one is touching patient before pressing the "Shock" button.

Installation Date _____

MEDICAL EQUIPMENT

www.welchallyn.com/waose

TESTED & CERTIFIED BY CSA INTERNATIONAL TO COMPLY WITH APPLICABLE U.S. & CANADIAN MEDICAL SAFETY STANDARDS.

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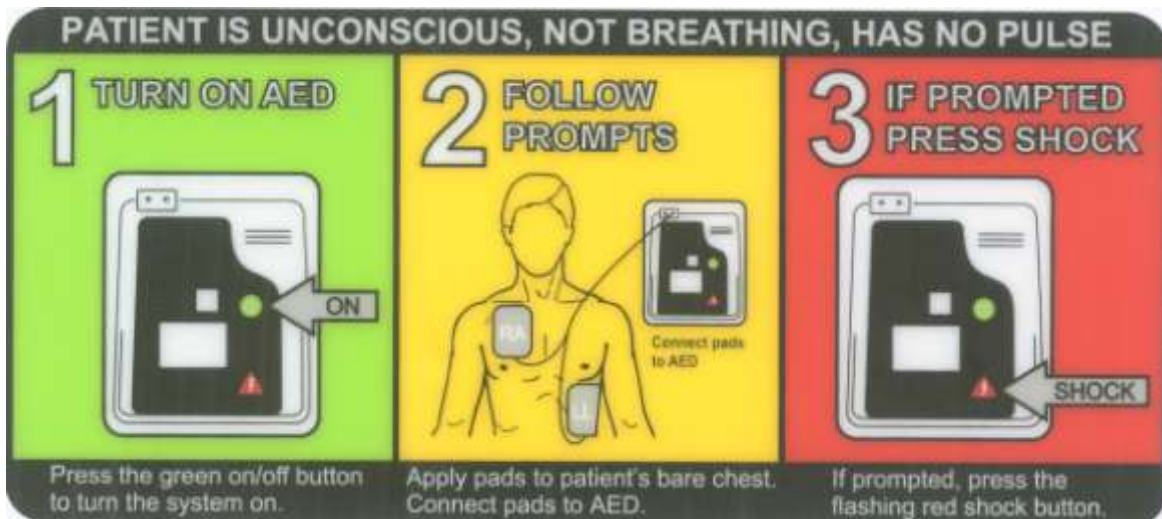
FAX 00353 4690 67755

fieldsafetynotice@welchallyn.com

**Regulatory Department,
Welch Allyn Ltd.
IDA Business Park,
Dublin Road,
Navan,
Co. Meath
República da Irlanda.**