**GEBRAUCHSANLEITUNG  
DIRECTIONS FOR USE  
NOTICE D'UTILISATION  
GEBRUIKSHANDLEIDING  
MANUALE PER L'USO  
MANUAL DO UTILIZADOR  
MANUAL DE INSTRUCCIONES**

**BREEZY 200**

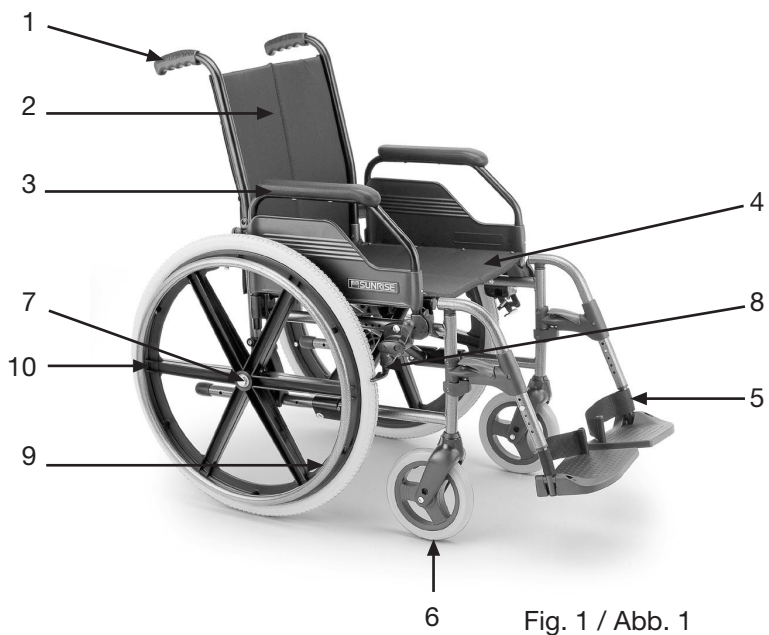
Fig. 2 / Abb. 2

Fig. 3 / Abb. 3

Fig. 4 / Abb. 4

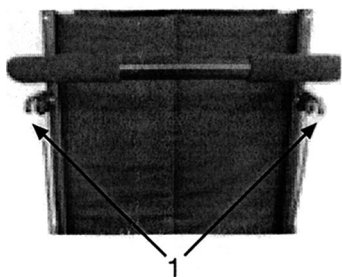
Fig. 5 / Abb. 5

Fig. 6 / Abb. 6

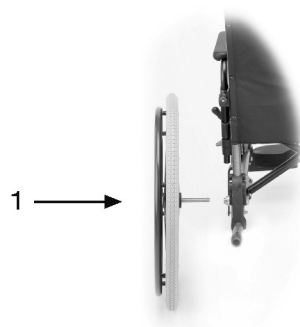
Fig. 7 / Abb. 7

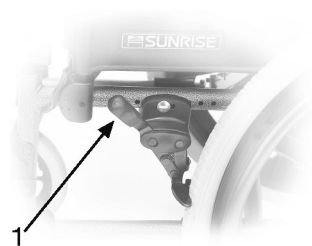
Fig. 8 / Abb. 8

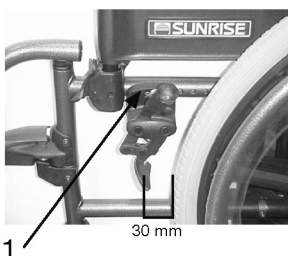
Fig. 9 / Abb. 9

Fig. 10 / Abb. 10

Fig. 11 / Abb. 11

Fig. 12 / Abb. 12

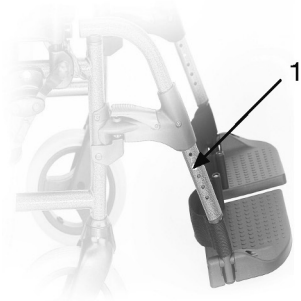
Fig. 13 / Abb. 13

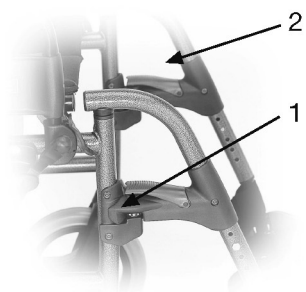
Fig. 14 / Abb. 14

Fig. 15 / Abb. 15

Fig. 16 / Abb. 16

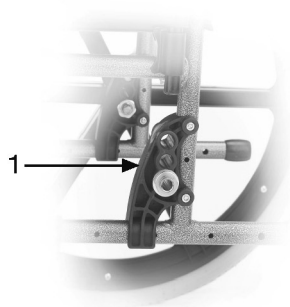
Fig. 17 / Abb. 17

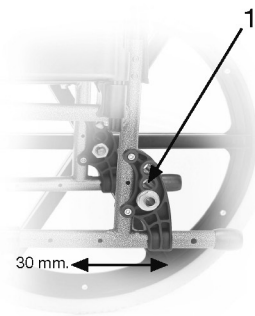
Fig. 18 / Abb. 18

Fig. 19 / Abb. 19

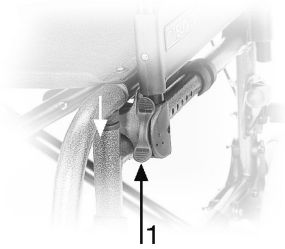
Fig. 20 / Abb. 20

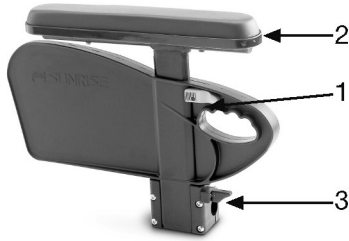
Fig. 21 / Abb. 21

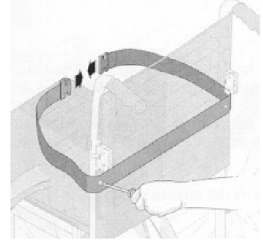
Fig. 22 / Abb. 22

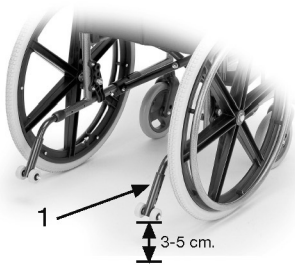
Fig. 23 / Abb. 23

Fig. 24 / Abb. 24

Fig. 25 / Abb. 25

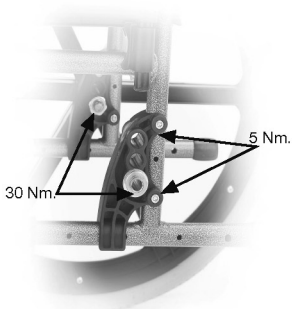
Fig. 26 / Abb. 26

Fig. 27 / Abb. 27

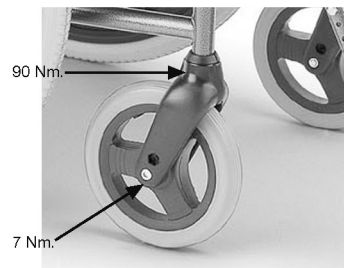
Fig. 28 / Abb. 28

## VORWORT

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## FOREWORD

Dear Customer,

We are very happy that you have decided in favour of a high-quality product from SUNRISE MEDICAL. This user's manual will provide numerous tips and ideas so that your new wheelchair can become a trustworthy and reliable partner in your life.

Maintaining close links with our customers is of great importance to us at Sunrise Medical. We therefore like to keep you up-to-date with our new and current developments. Keeping close to our customers also means fast service when you need replacement parts or accessories, or just have a question about your wheelchair -and with as little red tape as possible. We want you to be satisfied with our products and service. Sunrise Medical therefore constantly works at continuous development of its products. For this reason, changes can occur in our range of products with regard to shape, technology, and fittings. Consequently, no claims can be construed from the data or pictures contained in this user's manual.

SUNRISE MEDICAL has been awarded the ISO 9001 Certificate, which affirms the quality of our products at every stage, from R & D to production.

Please contact your local, authorised SUNRISE MEDICAL dealer if you have any questions concerning the use, maintenance, or safety of your wheelchair.

In the case that there is no authorised dealer in your area or you have any questions, you can contact Sunrise Medical either in writing or by telephone at:

Sunrise Medical Ltd.

Sunrise Business Park

High Street, Wollaston

West Midlands DY8 4PS

England

Telephone: +44/1384-446688

Fax: +44/1384-446699

## TABLE OF CONTENTS

Foreword for Wheelchairs .....	17	Seat belt .....	24
Safety Tips for Wheelchairs.....	18	Anti-tip Tubes .....	25
Transportation in vehicles.....	18	Crutch Holder .....	26
Wheelchair Components .....	22	Tray .....	26
Handling .....	22	Tires and Mounting .....	26
Folding up and Unfolding .....	22	Possible Problems .....	26
<b>OPTIONS</b>			
Tipping Tubes .....	22	Maintenance.....	26
Brakes .....	23	Technical data .....	26
Footplates .....	23	Nameplates/Guarantee .....	27
Amputee Axle Plate .....	23	Torque .....	27
Sideguard .....	24		

This manual gives information on all features sold across different countries, without stating whether they are actually available in your country or are optional or standard features. For this information please refer to the order-form/prescription form or your prescriber/supplier.

## SAFETY TIPS

The engineering and construction of this wheelchair have been designed to provide maximum safety. International safety standards currently in force have either been fulfilled or exceeded. Nevertheless, users may put themselves at risk by improperly using their wheelchairs. For your own safety, the following rules must absolutely be observed.

Unprofessional or incorrect adaptation or adjustment work could increase the risk of accidents.

As a wheelchair user, you are also part of the daily traffic on streets and pavements, just like anyone else.

We would like to remind you that you are therefore also subject to any and all traffic laws.

For this reason, you should always wear clothing which is light-coloured when it is dark, so that you can be seen more easily. Also, make sure that the reflectors are not covered.

Be careful during your first ride in this wheelchair. Get to know your wheelchair.

Test what effect changing the centre of gravity would have on your wheelchair.

Practice using your wheelchair on level ground as well as going uphill and downhill.

## TRANSPORTATION IN VEHICLES

This product is suitable to use as a seat in vehicles as per certificate *ISO/FDIS 7176-19 :2001 (E)* and using the restraint system **Q strain Q-5001-T2**.

A wheelchair secured in a vehicle will not provide the equivalent level of safety and security of a vehicle seating system. It is always recommended that the user transfers to the vehicle seating. It is recognised that this is not always practical for the user to be transferred and in these circumstances where the user must be transported whilst in the wheelchair the following advice should be followed.

1. Confirm that the vehicle is suitably equipped to transport a passenger in a wheelchair, and ensure the method of access/egress is suitable for your wheelchair type. The vehicle should have the floor strength to take the combined weight of the user, the wheel chair and accessories.
2. Sufficient space should be available around the wheelchair to enable clear access to attach, tighten and release the wheelchair and occupant tie down restraints and safety belts.
3. The occupied wheelchair must be located in a forward facing position and secured by the wheelchair tie down and occupant restraint straps (WTORS tie downs) meeting the requirements of ISO 10542 or SAE J2249, in accordance with the WTORS manufacturers' instructions.
4. The wheelchair's use in other positions within a vehicle has not been tested e.g. transportation in a side facing position must not be carried out under any circumstances. (Fig. 10)

 **Warning** There is a risk of serious injury or death if this if this advice is ignored.

5. The wheelchair should be secured by a Tie Down Restraint system, conforming to ISO 10542 or SAE J2249 with non-adjustable front straps and adjustable rear straps, which typically use Karabiner clips/S hooks and tongue and buckle fittings. These restraints generally comprise of 4 individual straps that are attached to each corner of the wheelchair.
6. The tie-down restraints should be fitted to the main frame of the wheelchair as indicated in the diagram on the following page, and not to any attachments or accessories, e.g. not around the spokes of wheels, brakes or footrests.
7. The tie-down restraints should be attached as close as possible at an angle of 45 degrees and tightened securely in accordance with the manufacturer's instructions.
8. Alterations or substitutions must not be made to the wheelchair securement points or to structural and frame or components without consulting the manufacturer. Failure to do so will invalidate the ability of a Sunrise Medical wheelchair to be transported within a vehicle.

- Both pelvic and upper torso restraint belts must be used to restrain the occupant to reduce the possibility of head and chest impacts with the vehicle components and serious risk of injury to the user and other vehicle occupants. (Fig. 11) The upper torso restraint belt should be mounted to the vehicle "B" pillar - failure to do so will increase the risk of serious abdominal injuries to the user.

Fig. 10

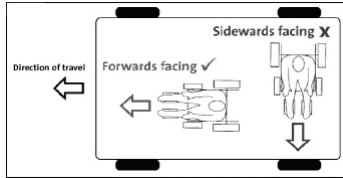
Fig. 11


- A head restraint suitable for transportation (see label on headrest) must be fitted and suitably positioned at all times during transportation.
- Postural supports (lap straps, lap belts) should not be used or relied on for occupant restraint in a moving vehicle unless they are labelled as meeting the requirements specified in ISO 7176-19:2001 or SAE J2249.
- The safety of the user during transportation depends upon the diligence of the person securing the tie-down restraints and they should have received appropriate instructions and/or training in their use.
- Wherever possible remove and stow safely away from the wheelchair all auxiliary equipment, for example:
  - Crutches
  - Loose cushions
  - Tray Tables
- Articulating/elevating leg rest should not be used in the elevated position when the wheelchair and user are being transported and the wheelchair is restrained using Wheelchair Transport and Occupant Restraints.
- Reclining backrests should be returned to an upright position.
- The manual brakes must be firmly applied.

**Occupant Restraint Instruction**

- The pelvic restraint belt must be worn low across the front of the pelvis so that the angle of the pelvic belt is within the preferred zone of 30 to 75 degrees to the horizontal.

A steeper (greater) angle within the preferred zone is desirable i.e. closer to, but never exceeding 75degrees.(Fig 12)

Fig. 12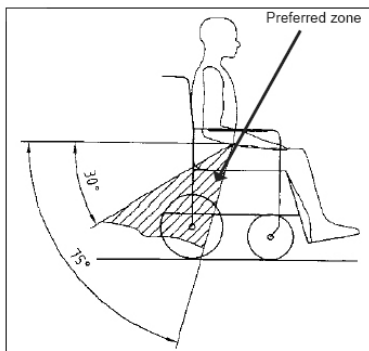

2. The upper torso restraint belt must fit over the shoulder and across the chest as illustrated Fig 13 and Fig 14.  
 Restraint belts must be adjusted as tightly as possible consistent with user comfort.  
 Restraint belt webbing must not be twisted when in use.  
 The upper torso restraint belt must fit over the shoulder and across the shoulder as illustrated in Fig 13 and Fig 14.
3. Restraints should be mounted to the vehicle "B" pillar and should not be held away from the body by wheelchair components such as armrest or wheels.

Fig. 13


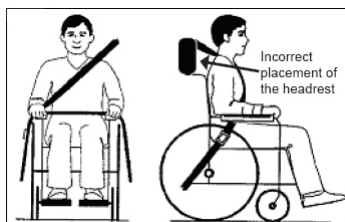
Fig. 14

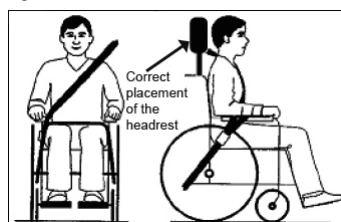4. The attachment points to the chair are the inner front side frame just above the castor and the rear side frame. The straps are fitted around the side frames at the intersection of the horizontal and vertical frame tubes. (See Figs 15-20)
5. The tie down symbol (Fig 16) on the wheelchair frame indicates the position of the wheelchair restraint straps. The straps are then tensioned after the front straps have been fitted to secure the wheelchair.

**Transportability – positioning of wheelchair tie down restraints on wheelchair**

Fig. 15

Fig. 16 Tie down label

The wheelchair secured with front and rear wheelchair tie down restraints. Positioning of the restraint straps is shown in more detail below.

Fig. 17

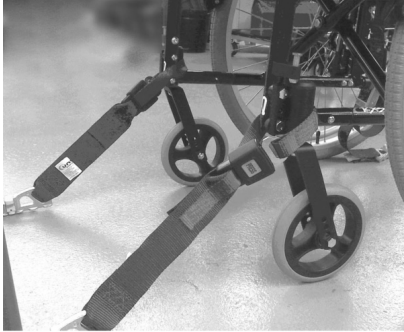
Fig. 18

Position of the front wheelchair tie down restraint and the tie down label.

Fig. 19

Fig. 20

Position of the rear wheelchair tie down restraint and the tie down label on the wheelchair.

**CAUTION:** The wheel locks are not intended to brake your wheelchair. They are only there to ensure that your wheelchair does not begin rolling unintentionally. When you stop on uneven ground, you should always use the wheellocks to prevent such rolling. Always use both wheellocks; otherwise, your wheelchair could tip over. The wheel locks have not been designed to be used as brakes for a moving wheelchair.

Always make use of elevators and ramps. If these are not available, you can reach your destination with the aid of two attendants. They should grip the wheelchair at securely mounted parts.

If your wheelchair is equipped with anti-tip tubes, these should be folded away or removed. A wheelchair should never be lifted with an occupant; it should only be pushed.

Before each use, the following should be checked:

- Quick-release axles (if fitted) on the rear wheels .
- Tyres, tyre pressure, and wheellocks.

Wheellocks will not function properly if tyre pressure is too low or the interval between tyres and wheellocks is too large.

The maximum user weight is 120 kilograms.

This wheelchair should be used exclusively to convey one person at a time in the seat. Any other use beyond this limit is in violation of safety regulations.

Before changing any of the adjustments of this wheelchair, it is important to read the corresponding section of the user's manual.

It is possible that potholes or uneven ground could cause this wheelchair to tip over, especially when riding uphill or downhill.

When riding over a step or up an incline frontally, the body should be leaning forward.

## WHEELCHAIR COMPONENTS

We at SUNRISE MEDICAL have been awarded the ISO 9001 Certificate, which affirms the quality of our products at every stage, from R & D to production.

This product complies with the standards set forth in EU directives.

Optional equipment and accessories are available at extra charge.

Wheelchairs: (Fig. 1)

- |                   |                        |
|-------------------|------------------------|
| 1. Push handles   | 6. Casters             |
| 2. Backrest sling | 7. Quick-release axles |
| 3. Armrest        | 8. Wheel locks         |
| 4. Seat sling     | 9. Handrim             |
| 5. Footrest       | 10. Rear wheel         |

## HANDLING

**Folding up:** First remove the seat cushion from the wheelchair and flip up the (platform) footplate or the individual footplates. (Fig. 2) Take hold of the sling from underneath in the middle and pull upwards. The wheelchair will then fold up. To fold your wheelchair until it is as small as possible, e.g., to put into a car, you can remove the footrests (depending on the model). For this purpose, open the latch from the out-side and swing the footrest to the side. Then the footrest only has to be drawn out of the frame tubing. (Fig.4)

Caution: Before folding the wheelchair, make sure that the stabilizer bar is removed.

**Unfolding:** Open the strap on folded frame prior to unfolding. (Fig.3)

Press down on the seat tubing (see picture). Your wheelchair is then unfolded. Now snap the seat tubing into position in the seat saddle. This can be facilitated by slightly tipping your wheelchair. In this way, the weight can be taken off of one rear wheel so that the wheelchair can be more easily unfolded.

Be careful not to get your fingers caught in the cross-tube assembly.

**RECLINING BACK (4° - 55°) (FIG. 5):** By pulling the 2 levers (1) at the same time the backrest is released and can be set to the required position.

Releasing the 2 levers (1) will lock the backrest automatically in the new position. To bring the backrest back to the upright position, just move it (no release necessary) to the required position.

Caution: The reclining back is only to be used in combination with the active anti-tips setting

Caution: The reclining back is only to be used in combination with a stabilizer bar. Please always make sure that the stabilizer bar is locked in place on both sides.

Caution: In order to avoid any reduction in the cable function, you must make sure that the cables are not completely taut.

Caution: When adjusting the back angle, be careful not to get your fingers caught.

Caution: Before adjusting the back angle, put the anti tip tubes in to the active position

### Quick-Release Axles for Rear Wheels

The rear wheels are equipped with quick-release axles. The wheels can thus be installed or removed without using tools. To remove a wheel, simply depress the quick-release button on the axle (1) and pull it out. (Fig.6,7)

CAUTION: Hold the quick-release button on the axle depressed when inserting the axle into the frame to mount the rear wheels. Release the button to lock the wheel in place. The quick-release button should snap back to its original position.

## OPTIONS

### Tipping Tubes

Tipping tubes are used by attendants to tip a wheelchair over an obstacle. Simply step on the tube to push a wheelchair, for example, over a curb or step.

## Wheel Locks

Your wheelchair is equipped with two wheel locks. They are applied directly against the tires. To engage, press both wheel-lock levers forward against the stops (Fig. 8). To release the wheel, pull the levers back to their original positions.

Braking power will decrease with: Worn tire tread / Tire pressure that is too low / Wet tires / Improperly adjusted wheel locks.

The wheel locks have not been designed to be used as brakes for a moving wheelchair. The wheel locks should therefore never be used to brake a moving wheelchair. Always use the handrims for braking.

Make sure that the interval between the tires and wheel locks complies with given specifications (Fig. 9). To readjust, loosen screw (1) and set the appropriate interval. Tighten screw (see the page on torque)

CAUTION: After each adjustment of the rear wheels, check the interval to the wheel locks and readjust if necessary.

### Extension for the Brake Lever (Fig.10)

The extension for the brake lever can be removed or folded down. The longer lever helps to minimize the effort needed to set the wheel locks.

Caution: Mounting the wheel lock too close toward the wheel will result in a higher effort to operate. This might cause the brake lever extension to break!. Leaning onto the brake lever extension while transferring will cause the lever to break! .Splashing water from tires might cause the wheel lock to malfunction.

### Drum Brakes (Fig. 11, 12)

Drum brakes permit safe and convenient braking for an attendant. They can also be set with the aid of a locking lever (1) to prevent rolling. This lever must snap audibly into place. Drum brakes do not independent on the aid of air pressure inside the tires.

Your wheelchair cannot be moved when the drum brakes are set.

CAUTION: Drum brakes should only be adjusted by authorized dealers.

## Footplates

The footplates can be flipped up to facilitate getting in and getting out of your wheelchair.

### Lower Leg Length (Fig 13)

By removing the screws (1), the legrest can be adjusted to fit any lower leg length. Loosen the screws, set the tubing with legrest in the desired position, and then tighten the screws (see the page on torque).

A minimum interval of 2.5 centimetres from the ground should always be maintained.

### Legrests and Latch (Fig 14)

The legrest can be swung in and outwards.

Fit your legrest by inserting it with the footplate(s) pointing toward the outside. Then rotate it toward the inside until it locks into place (1).

To remove the legrest, depress the lever (2), swing the footrest towards the outside and lift to remove it.

Make sure that the legrest is properly locked into place.

CAUTION: The legrests should not be used to lift or to carry the wheelchair with an occupant.

### Elevating legrest (Fig 15,16)

To elevate: Lift your lower leg and press the release button. Pull the legrest upwards and adjust it to the desired height. As soon as you release the button, the legrest will click back into place.

To lower: Lift your lower leg and press the release button to unlock the legrest to lower it. As soon as you release the button, the legrest will click back into place.

CAUTION: The elevating legrest is not allowed to be adjusted into the lowest position to avoid any interference with the casters.

CAUTION: Keep hands clear of the adjustment mechanism between the frame and the movable parts of the legrest while elevating or lowering the legrest.

CAUTION: Legrests are not to be used for lifting or carrying the wheelchair with an Occupant.

### Amputee Axle Plate

The longer the wheel base, the greater the stability of the wheelchair. Turn the axle bracket to the rear (1) in order to extend the wheel base. (Fig.17,18)

With the amputee support (accessory) (Fig. 19) the wheel base can be extended by 7 cm. When you have completed the adjustment, make sure that all screws have been properly tightened (see the page on torque). CAUTION: Wheel locks must be adjusted for the new position.

**Sideguard**

Standard Armrests

Standard armrest, detachable with short or long armpads. The front of the short armrest is angled downward, which allows you to pull up close to a desk or table top. To detach the armrest, push the lever (1) (Fig 20) downwards to unlock. After reinserting the armrest, push the lever (1) upwards. Always make sure that the armrest is locked in place.

CAUTION: Neither the sideguards nor the armrests are to be used for lifting or carrying the wheelchair.


Height-adjustable armrest (Fig. 21)

Adjust the height of the armrest as follows: Slide the knob (1) forward until it stops. Adjust the armrest (2) to the desired height. Slide the knob toward the back and push the armrest (2) down until you hear it click into place. Always make sure that the sideguards have locked into place properly. To detach the armrest, press the lever (3) downward until it stops. CAUTION: Neither the sideguards nor the armrests are to be used for lifting or carrying the wheelchair.

**Safety belt (optional)**

**Before using your wheelchair ensure the seat belt (when fitted) is worn and correctly adjusted before use.**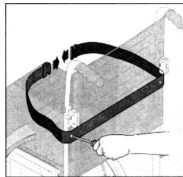

The lap belt is fitted to the wheelchair as shown in the illustrations. Fit the safety belt by passing it around the backrest tubes, from the outside. Secure the safety belt to the backrest tubes using the same bolt that secures the upholstery to the backrest tube. You must remove this bolt beforehand. Using the same hole, push the bolt through the belt and upholstery, and then retighten it to the backrest tube.

Adjust the belt position so that the buckles are in the centre of the seat.

Adjust the lap belt to suit the user's needs as follows:

**To increase the belt length:**


Feed the belt through slide adjusters and male buckle to provide more belt length.

**To reduce the belt length:**

Feed the belt back through male buckle and slide adjusters.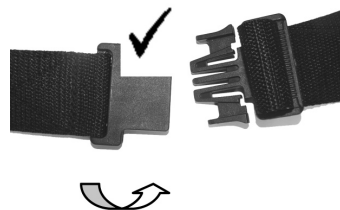

Ensure the belt is not looped at the male buckle


When fastened check space between the lap belt and the user, when correctly adjusted it should be possible to insert the flat of the hand between the lap belt and the user.


Generally the Lap Belt should be fixed so that the straps sit at an angle of approximately 45°, and when correctly adjusted should not allow user to slip down in the seat.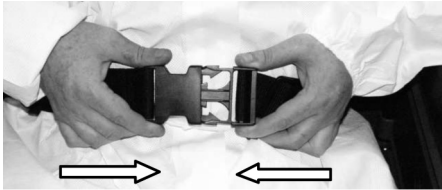

**To fasten buckle:**

Firmly push the male buckle into female buckle.

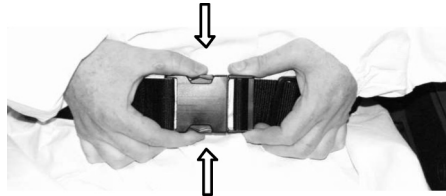**To release belt:**

Press the exposed sides of the male buckle and push towards the centre whilst gently pulling apart.

**Do not rely on the lap belt only when the wheelchair and occupant are transported in a vehicle, use the separate occupant lap and diagonal restraints provided in the vehicle.**

**Advice to Client**

The lap belt must be checked on a daily basis to ensure it is adjusted correctly and it is free from any obstruction or adverse wear.

**Failure to make sure that the lap belt is secure and adjusted prior to use could cause serious injury to the user. E.g. too loose a strap may allow the user to slip down in the chair and risk suffocation.**

**Maintenance**

Check lap belt, and securing components; at regular intervals for any sign of fraying or damage. Replace if necessary.

Clean the lap belt with warm soapy water and allow to dry.

**Note**

**The lap belt should be adjusted to suit the end user as detailed above**

**Sunrise Medical also recommend that the length and fit of the belt is checked on a regular basis to reduce the risk of the end user inadvertently re-adjusting the belt to an excessive length**

**If in doubt about the use and operation of the seat belt then ask your healthcare professional, wheelchair dealer, carer or attendant for assistance.**

**Anti- Tip Tubes (Fig. 23):**

Anti-tip tubes (1) provide additional safety for inexperienced users when they are still learning how to operate their wheelchairs. Anti-tip tubes (1) prevent a wheelchair from tipping over backwards. Pressing the release button allows the anti-tip tubes (1) to be moved upwards or removed. An interval of 3 cm to 5 cm should be maintained between them and the ground.

You must swing the anti-tip tubes upwards when going up and down large obstacles (such as a curb) to prevent them from touching the ground.

**Crutch Holder**

This device permits crutches to be transported directly on a wheelchair. It has a Velcro loop (1) to fasten crutches or other aids. (Fig.24) CAUTION: Never try to use or even remove the crutches or other aids while moving.

**Tray (Fig.25)**

The tray provides a flat surface for most activities. Before using a tray, it first has to be adjusted once to the width of the seat by an authorized dealer. The user must be sitting in the wheelchair during the trial run.

**TIRES AND MOUNTING:**

Always make sure you that you maintain the correct tire pressure, as this can have an effect on wheelchair performance. If the tire pressure is too low, rolling resistance will increase, requiring more effort to move the chair forward; low tire pressure also has a negative impact on manoeuvrability. If the tire pressure is too high, the tire could burst. The correct pressure for a given tire is printed on the surface of the tire itself.

Tires can be mounted the same way as an ordinary bicycle tire. Before installing a new inner tube, you should always make sure that the base of the rim and the interior of the tire are free of foreign objects.

Check the pressure after mounting or repairing a tire. It is critical to your safety and to the wheelchair's performance that regulation air pressure be maintained and that tires be in good condition.

**POSSIBLE PROBLEMS:**

**Wheelchair pulls to one side:** Check tyre pressure. Check to make sure wheel turns easily (bearings, axle). Check to make sure both casters are making proper contact with ground

**Casters begin to wobble:** Check to make sure bolts are secure; tighten if necessary (see the page on torque). Check to make sure both casters are making proper contact with the ground

**Wheelchair/Cross-tube assembly does not snap into position in the seat saddle:** Chair is still new, i.e., the seat or backrest upholstery is still very stiff. This will improve with continued use

**Wheelchair is difficult to fold up:** Screws of the cross-tube brackets are too tight

**Wheelchair squeaks and rattles :** Check to make sure bolts are secure; tighten if necessary (see the page on torque). Apply small amount of lubrication to spots where movable parts come in contact with one another

**Wheelchair begins to wobble:** Check tyre pressure. Check to see if rear wheels are adjusted differently.

**MAINTENANCE**

- Check the tire pressure every 4 weeks. Check all of the tires for wear or damage.
- Check the brakes approximately every 4 weeks to make sure that they are working properly and easy to use.
- Change tires as you would an ordinary bicycle tire.
- All of the joints that are critical to safe using your wheelchair safely are self-locking. Please check every three months to make sure that all bolts are secure (See the page on torque). Safety nuts should only be used once and should be replaced after multiple uses.
- Please use only mild household cleansers when your wheelchair is dirty. Use only soap and water when cleaning the seat upholstery. If your wheelchair should ever get wet, please dry it after use.
- A small amount of sewing-machine oil should be applied to quick-release axles approximately every 8 weeks.
- Depending on the frequency and type of use, we recommend taking your wheelchair to your authorized dealer every 6 months to have it inspected by trained personnel.

CAUTION: Sand and sea water (or salt in the winter) can damage the bearings of the front and rear wheels. Clean the wheelchair thoroughly after exposure.

**TECHNICAL DATA**

Total Width:

- |   |              |
|---|--------------|
| • With standard wheels, incl. hand rims, close-mounted: | SW + 21 cm . |
| • With 12" rear wheels:                                 | SW + 15 cm . |
| • Wheels with drum brakes, hand rims, close-mounted:    | SW + 24 cm   |

**Dimensions, folded:**

- With rear wheels: approx. 35 cm
- Without rear wheels: approx. 17 cm
- With 12" rear wheels: approx. 26 cm

**Weight in kilograms:**

- Wheelchair without footrests (24 ") 15.1 kg
- Wheelchair without footrests (12") 13.7 kg
- Footrests (Pair) 1.5 kg
- Standard sideguards 1 kg
- Rear wheels 24" (solid) 5 kg

**Maximum weight limit:** Approved for user weights up to 120 kg /265 lbs.

**The nameplate** is located on either the cross-tube assembly or the transverse frame tube. The nameplate indicates the exact model designation and other technical specifications. Please provide the following information whenever you have to order replacement parts or to file a claim:

Serial number / Order number / Month/year

**GUARANTEE**

You have purchased a high-quality SUNRISE MEDICAL product. SUNRISE MEDICAL guarantees the wheelchair against defect in materials or manufacture for the period established by the current guarantee law in each country. We are not responsible for any damage resulting from inappropriate or unprofessional installation and/or repairs, through neglect and wear, or from changes in any wheelchair components caused either by the user or by third parties. In such cases, this guarantee shall be considered null and void. Custom wheelchairs cannot be exchanged.

**TORQUE** (Fig.26, 27, 28) After opening always secure by LOCTITE 243

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