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Congratulations on having purchased the Joycare Alcohol Test an accurate and reliable product.

For the limits of alcoholic rate of various countries, refer to the internal laws.

WARNINGS

The tester provides you with information to determine your blood alcohol level at the same time you use the test. Blood alcohol concentration changes as time passes and measurement may be affected by environmental conditions.

It must be used according to the instructions, contained in this manual and it is important that the user reads and understands the use and maintenance information of the device.

Keep away from dust and humidity; do not expose to sun or to excessive temperatures.

This appliance may be used by children of more than 8 years of age, persons with physical, sensory or mental capabilities or inexperienced persons only if they have been previously trained in using it

in a safe manner, and only if they have been informed of the dangers the product can cause.

Children may not play with the appliance. Children may not clean or carry out any maintenance on the appliance without adult supervision.

This product is designed to be used as reference only in order to promote a responsible and safe driving. It doesn't replace or exclude possible tests conducted by competent authorities, which can have different readings. The tester readings have no legal value and do not relieve any individual responsibility. You should not rely upon it as the sole basis to determine intoxication or whether it is safe to drive a vehicle, operate equipment, or engage in dangerous activities.

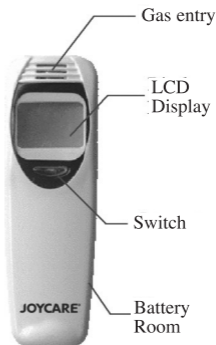
Joycare is not liable for the improper use of this product for any reason.

IMPORTAT SAFETY INSTRUCTIONS

- Before using the product, please read carefully this instruction manual.
- Use the device only for its intended purpose
- In case of doubts, do not use the product but address to specialised centres
- Store the product away from heated surfaces, from direct sunlight, dampness and sharp objects.
- In case the product is not used for a long period, it is advisable to remove batteries.
- Never try to repair the product by yourself but always address to specialised centres

FEATURES

- Advanced NM semiconductor alcohol sensor
- Quick response
- SMD assembling, stable performance
- Smart MCU control
- Direct Testing process
LCD Indication
- Pocket size design
- No mouthpiece is necessary
- Audio warning when the pre-set limit has seen exceeded
- Low battery indicator
- Auto-switch off after 8 seconds of non-use

TECHNICAL INFORMATIONS

- Advanced NM semiconductor alcohol sensor
- Pre-set limit: 0,5 g/L
- Sensor warm-up Time < 10 sec
- Testing Time 8 secs
- Display 3 digital LCD indication with red backup light
- Power supply 3VDC 2 AAA batteries (test batteries are included) The supplied batteries are for trial use only and they may have a shorter life.
- Dimension: 90mm X 30mm X 16mm
- Weight 45g (including batteries)

Operation use conditions:

Room temperature: from 10°C to 40°C

Relative humidity: from 30% to 75% without condensation.

Pressure: from 700 – 1060 hPa

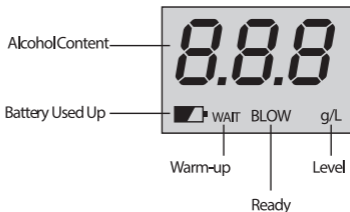
Environmental transportation and storage conditions:

Temperature: from 10 to 40°C

Humidity: from 5 to 95%

Pressure: from 700 – 1060 hPa

INDICATION

- 0.00 g/L** = Breathed alcohol content
WAIT = Warming, please wait
BLOW = Ready to test
 = Battery used up, please replace it.

Alcohol Test can detect quickly the alcohol level through your breath. If you are in excessive drink, the tester would give you audio warning beyond pre-set limit.

BEFORE STARTING USING THE PRODUCT

Slide down to open the battery compartment. Insert batteries and close it carefully; please note the correct polarity direction in the battery compartment, otherwise, you may damage the unit.

To ensure an accurate and not altered testing result it is advisable not to smoke, eat menthol candies, chocolate and take medicines. Moreover, please avoid using the product if there is too much wind or in crowded environments, where generally there is a high smoke concentration.

Keep the unit away from all liquids and store in a safe place. Do not wipe it with any cleaning agent.

WHEN IT IS ADVISABLE TO USE THE PRODUCT

In order to get the more reliable record as possible, it is advisable to wait about 45 minutes, from the last alcohol consumption, before measuring. As matter of fact, blood needs about 45 minutes to absorb alcohol.

On the contrary, if the measuring is carried out before these 45 minutes, values can be altered in the following ways: if the measuring is carried within the first 7 minutes from the last alcohol consumption, records are very high; because month

and oesophagus mucosa absorb alcohol partially and they need about 7 minutes to eliminate it.

On the contrary, if the measuring is carried out between 30-45 minutes after the last alcohol consumption, values can be lower since blood may need some minutes before absorbing alcohol totally.

INSTRUCTIONS FOR A CORRECT USE OF THE APPLIANCE

Press the button; the backup lamp would switch on with a buzzer brief ringing. The writing "WAIT" flashes on the display and the 18 seconds initial warm-up begins, there is a countdown on the LCD from 18-0; then the writing "WAIT" disappears and the "BLOW" guide you to have a testing immediately.

When the "BLOW" appears, blow into the "Gas Entry". Please have a deep breath and breathe out directly to gas entry within 3 cm from the sensor. The measured value will be displayed for 8 seconds and at the same time you hear a buzzer ringing; then the tester will switch itself automatically off.

For a new testing, please press the switch button again and repeat the above steps.

For a more accurate reading, it is advisable to wait 60 sec. between different measurements.

MAINTENANCE

If necessary, clean with a damp cloth moistened with water: avoid any water infiltration, especially in the sensor. Never use chemical detergents. Do not immerse the product in water. As it is a precision instrument, the tester must be treated with care at all times. Do not drop it. Do not open or tamper

with the appliance: repairs must be performed only by authorised personnel; otherwise the warranty will expire.

If the product is not used for a long period of time, it is advisable to remove the battery, as a possible loss of liquid may damage the appliance.

Moreover, as battery quality can vary from model to model, the product warranty does not cover any damage caused by a possible loss of liquid.

In case of malfunction:

Check that the batteries are inserted properly

If problems persist, contact the Service Centre JOYCARE

IMPORTANT NOTES

- Avoid any strong shock and drop to keep the tester in careful package or deposit.
- Store it in a dry place and away from heated surfaces.
- Do not wipe it with any cleaning agents; in case of some gas spreading, the normal testing would be disturbed.
- Store the product out of children reach Never immerse the product into water or into any other liquids.
- In case the measuring is carrying out in smoking environment, records can be altered.
- Pre-set limit: 0,5 g/L
- After long time storage, if the initial alcohol level of the LCD is on high level, please switch the product on and off different times to stabilize the level at 0.00.

 This item is made in conformity with all the applicable European directives

**SMALTIMENTO**

The device (including its removable parts and accessories) must not be disposed of together with municipal waste at the end of its life, but in compliance with European Directive. Since it must be handled separately from household waste, either carry it to a separately collected waste disposal centre for electrical and electronic appliances or give it back to the retailer on purchasing a new device with the same purpose. Any infringement will be severely prosecuted.

The batteries used in this device must be disposed of in the special bins at the end of their life.

Specifications and designs are based on the latest information available at the time of printing and subject to change without notice.

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