0197*

*für C137A, C137G, C137IA, C137IG, C137IM, C1372A, C1372G, C1372M, C109G, C129G, C129M, C110G, C110M

Hersteller / Manufacturer / Fabricant / Fabricante / Fabrikant / Tillverkare / Fabrikant / Περιουμπούρος / Producent / Výrobca / Proizvajalec / Výrobce / Produsent / Производитель

Institut Virion\Serion GmbH
Friedrich-Bergius-Ring 19 | D - 97076 Würzburg, Germany
Tel. +49 (0)931 30 45 0 | Fax +49 (0)931 30 45 100
E-Mail: dialog@virion-serion.de | Internet: www.virion-serion.de

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Instructions - English

Composition

Human serum in protein containing phosphate buffer with Tween 20; liquid, 5 x 3 ml; negative for anti-HIV-Ab, HBs-Ag (Hepatitis B-Virus surface antigen) and anti-HCV-Ab; preservative: < 0.1 % sodium azide; colouring: 0.01 g/l Bromphenol blue.

Intended Use

SERION ELISA controls are positive control sera for monitoring qualitative and quantitative antibody determination when using immunoassays from the SERION ELISA product line. They are to be regarded as additional controls to the reagents supplied with the test kits. They are used to determine validity of SERION ELISA test runs as well as precision and reliability of the method. SERION ELISA controls are particularly recommended as an aid for internal quality management in accredited laboratories.

Stability and Storage

The ready-to-use SERION ELISA controls are stable for 18 months from date of production when stored at 2-8 °C; expiry date see label. Do not freeze.

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Precautions

Althought all SERION ELISA controls have been found to be negative for HBsAg, HCV and HIV-antibodies, they should be considered potentially infectious. The product contains sodium azide as a preservative. Swallowing and contact with skin or mucous membranes must be avoided!

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