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Owner's Manual

Manual del Propietario

Seladora BIO Pack
Sealer BIO Pack
Sellador BIO Pack

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INDEX

English

PRESENTATION OF MANUAL	20
IDENTIFICATION OF EQUIPMENT	21
MODULES, ACCESSORIES, OPTIONS AND MATERIALS OF CONSUMPTION	23
TECHNICAL SPECIFICATIONS	24
- Technical characteristics.....	24
- Dimension	25
- Symbologies of packaging.....	26
- Symbologies of product.....	26
- Contents of the accessible and inaccessible markings.....	27
INSTALLATION OF EQUIPMENT	27
OPERATION OF EQUIPMENT	28
PRECAUTIONS, RESTRICTIONS AND WARNINGS	29
- Conditions of transport and storage	29
- Sensitiveness to environmental conditions foreseeable in normal situations of use.....	29
- Recommendations for preserving the equipment.....	29
- Precautions and warnings “during the installation” of equipment.....	30
- Precautions and warnings “during the use” of equipment.....	30
- Precautions and warnings “after” the use of equipment.....	30
- Precauções e advertencias durante a “limpeza e desinfecção” do equipamento	30
- Precautions and warnings during the “cleaning and disinfection” of equipment.....	31
- Precautions to be adopted against foreseeable or uncommon risks, related to the deactivation and abandoning of equipment	31
- Additional procedures for reuse.....	31
CORRECTIVE AND PREVENTIVE MAINTENANCE AND PRESERVATION	31
- Cleaning	31
- Disinfection.....	32
- Preventive maintenance	32
- Corrective maintenance.....	32
UNFORESEEN EVENTS – SOLUTION OF PROBLEMS	33
WARRANTY OF EQUIPMENT	34
FINAL CONSIDERATIONS	34

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ATTENTION

For greater safety:

Read and understand all the instructions contained in these Instructions for Use before installing or operating this Equipment.

Note: These Instructions for Use must be read by all the operators of this Equipment.

IDENTIFICATION OF THE EQUIPMENT

Dear Customer

Congratulations. You have made a good choice when you decided to buy a GNATUS QUALITY product comparable to the best products available in the World. This manual is a general presentation of your product and it will give you important details to help you to solve possible problems.

Please, read it and keep this with you.

Identification

Technical name: Sealer

Brand: Gnatus

Model: BIO Pack

IDENTIFICATION OF THE EQUIPMENT

Description of the Equipment

BIO Pack Sealer

BIO Pack Sealer is designed for a hermetic sealing of surgical grade papers of all sizes on the market, ensuring the sterility of items since sterilization until the moment of use.

Simple and safe operation because:

It has isolated resistance, which inhibits the risk of burning the paper, even when the paper is not removed from the sealing region and does not transmit heat to the surface of product.

It allows temperature adjustment for fast and safe sealing for each type of paper.

It is provided with a trail type paper cutter, which cuts both ways, allowing cutting the paper in the desired size with no need to pass the knife twice.

The sealer's closing is performed by hand lever with lock, which enables better control of sealing time and allows sealing the surgical grade paper without using both hands of the operator.

Its body is made of stainless steel, which prevents corrosion, has high mechanical resistance and it is easy to clean and disinfect, besides providing a pleasing appearance.

It has a removable front support plate that makes the product more compact and easy to transport.

Physical principle used by the sealing equipment

Equipment for sealing surgical grade paper through bars at high temperature that are pressed on it with the help of a lever, which provides a hermetic sealing, it is also equipped with a double-edged blade that allows an easy way to cut both ways, after sealing.

Indication of Equipment

This equipment is for dental use only. It must be operated and utilized by specialized professional (certified professional, according to the legislation of the country) and following the instructions of the manual. The operation of the equipment required, for the professional, the utilization of correct instruments and it should to be in perfect conditions of the use, and to protect the professional, the patients and others, in the eventual danger situation.

Purpose of the equipment

Every article to be sterilized must be carefully packed and sealed for a safe process, the BIO Pack Sealer has the purpose of sealing packaging in an efficient and hermetic way to a perfect packaging.


MODULES, ACCESSORIES, OPTIONS AND MATERIALS OF CONSUMPTION

The content of this page is for information purposes, and the device may be presented different from that illustrated. Therefore, when purchasing the product check the technical compatibility between the device, coupling and accessories.

The use of any part, accessory or material not specified or referred to in these instructions for use is the user's responsibility.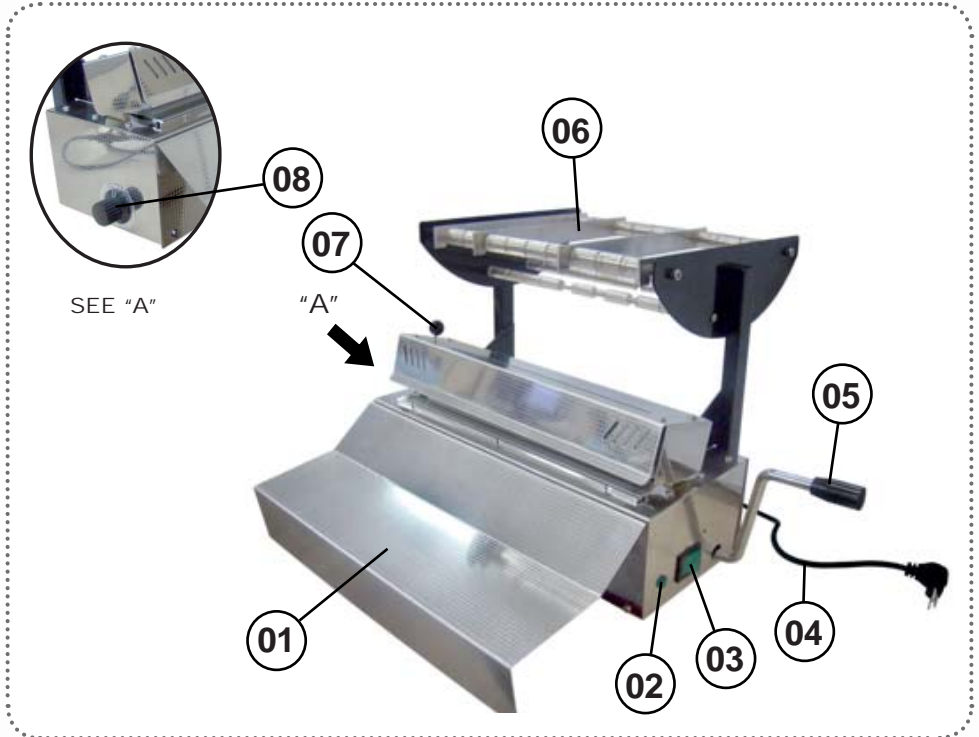

- 01 - Front Support plate;
- 02 - LED indicator;
- 03 - Power on/Power off switch;
- 04 - Power input cable;
- 05 - Operating lever;
- 06 - Lock;
- 07 - Knife handle;
- 08 - Temperature adjustment switch.

The materials used to produce the equipment are Biocompatible.

TECHNICAL SPECIFICATIONS

Technical features

Power Supply

Operation mode:

Continuous operation.

Power supply voltage:

127V~ / 110V~ - 140V~ / 220V~: 198V~ - 240V~

Frequency:

50/60 Hz

Potency for resistance:

127V~ / 220V~ = 250W

Power:

570 VA (127 V) / 460 VA (220 V)

Rated current:

2,1 A (220 V)

4,5 A (127 V)

Fuses:

2,5 A (220 V)

5 A (127 V)

Other Specifications

Sealing area:

260mm

Width sealing

10mm

Support rollers (maximum length):

275mm

External housing:

Stainless steel

Temperature (heating system)

The heating system can reach to 300Deg C maximum.

But we have installed temp. protector to limit the heating temp. at max. 220Deg C.

Net weight:

6,40 Kg

Gross weight:

8,26 Kg

TECHNICAL SPECIFICATIONS

Dimensions (mm)

TECHNICAL SPECIFICATIONS

Packing symbols

Maximum stacking:
It determines the maximum quantity of boxes which can be stacked during transportation and storage "as per packaging".

Packing to be transported and / or stored avoiding humidity, rains and wet floor.


Packing to be transported and / or stored with the harrows up.

The packing must be stored and transported away from direct sun light exposure.

Packing to be transported and / or stored with care (should not suffer drop and neither receive impact).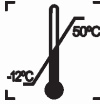

Temperature limit for the packing to be stored or transported.

Product symbols

Warning - Consult the manual

General Warning

Landing (in many parts of the equipment) indicates the condition of being landed.

Important: It indicates an instruction of safety for operation of the product. Not following it, can lead to serious danger to the patient.

Turned off

Note: It indicates useful information for operation of the product.

Turned on

TECHNICAL SPECIFICATIONS

Content of accessible and non-accessible demarcations

INSTALLATION OF EQUIPMENT

The materials used to produce the equipment are Biocompatible.

The installation of this equipment requires specialized technical assistance (Gnatus).

- This equipment shall only be able to be unpacked and installed by a Gnatus authorized technician, under penalty of losing the warranty, as only (s)he has the information, suitable tools and training required to execute this task.


- Gnatus bears no responsibility for damages or accidents caused by poor installation executed by a technician not authorized by Gnatus.

- Only after the equipment has been installed and duly tested by the authorized technician representing Gnatus, will it be ready to start work operations.

INSTALLATION OF EQUIPMENT

Moving parts installation

- Put the front support plate (01) into the Sealer, fitting the rip to the supports (A).
- Fit the operating lever in the vertical position (05), on the side of the Sealer (B).
- Connect the Power input cable (04), to the connection behind the Sealer (C).

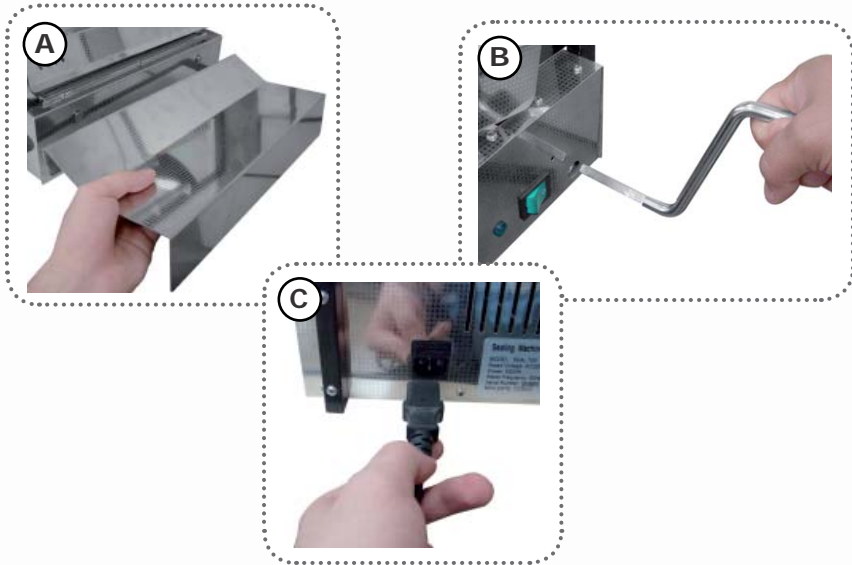EQUIPMENT'S OPERATION

Press the key (03) to power the equipment on, the LED indicator will light up (02).

Set the temperature adjustment switch (08) between ranges 3 and 4.

Put the surgical grade paper on the scroll bars, by positioning the locks (06) so as to better accommodate it according to its size.

Wait until the LED indicator turns off, indicating that the Sealer has reached the selected temperature.

Pass the surgical grade paper behind and between the pressure and sealing bars, so that the paper siding is upside down and the plastic siding is upwards.

Pull the surgical grade paper forward sufficiently to encapsulate the instruments.

Operate the lever (05) until it locks and hold it for approximately 4 seconds. Using the handle (07) quickly pass the knife in the direction of the paper (the knife cuts in both sides), deactivate the lever (05) and remove the surgical grade paper cut and sealed.

EQUIPMENT'S OPERATION

Place the instruments to be autoclaved inside the paper, which is now with a sealed side.

Place the other end (open) of surgical grade paper between the sealing bars and operate the lever again (08) until it locks and hold for approximately 4 seconds, deactivate the lever.

ATTENTION

The sealing bar operation temperature bar is very high. Watch out not to touch it directly.

In the first sealing, check the quality of it, if necessary, adjust the temperature by using the adjustment switch (08). After the adjustment, the same material can be sealed without new adjustments.

To start the operation it is necessary to wait for the LED indicator to turn off, the LED will turn off and on again during the operation, these signs can be ignored.

PRECAUTIONS, RESTRICTIONS AND WARNINGS

Transportation and storage

This equipment must be transported and stored observing the following directions:

- Avoid falls and impacts;
- Keep it dry, do not expose it to rain, water drops or wet floor;
- Keep it away from water and direct sunlight, and in its original wrapping;
- Don't move it over irregular surfaces, protect it from rain and observe the maximum stack quantity specified in the packaging;

The Equipment maintains its condition of safety and efficacy, provided that it is maintained (stored) as mentioned in this instruction of use. Thus, the equipment will not lose or alter its physical and dimensional features.

Sensitivity to environmental conditions in normal situations of use

- The equipment has been planned not to be sensitive to interference such as magnetic fields, external electrical factors, electrostatic discharge, pressure or variance of pressure, provided that the equipment is installed, maintained, clean, preserved, transported and operated as per this instruction for use.

Recommendations for the dental equipment maintenance.

Your Gnatus equipment has been designed and developed according to the standards of modern technology. Similarly to other kinds of equipment, it requires special care, which is many times neglected due to several reasons and circumstances. Therefore, here are some important reminders for your daily routine. Try to follow these simple rules, which will save you a lot of time and will avoid unnecessary expenses once they start making part of your working procedure.

PRECAUTIONS, RESTRICTIONS AND WARNINGS

Precautions and warnings “during the installation” of equipment

- The equipment should only be installed by Gnatus authorized technical assistance or technicians.
- Check that the socket in which the device will be connected has a ground connection. According to the ABNT standard, this is essential for the safe operation of the system;
- Position the equipment in a place where it will not get wet.
- Install the equipment in a place where it will not be damaged by pressure, temperature, humidity, direct sunlight, dust, salts, or sulfur compounds.
- This equipment was not planned for use in an environment where vapors, anesthetic mixtures inflammable with air, or oxygen and nitrous oxide can be detected.
- Check the voltage of the equipment at the moment of executing the electrical installation.
- The equipment must be grounded correctly.
- This equipment must be installed so that it is level.
- Before the first use and/or after long interruptions from work such as vacations, clean and disinfect the equipment.

Precautions and warnings “during the use” of equipment

- The equipment should only be operated by duly enabled and trained technicians (Dental Surgeons, Capacitated Professionals)
- If any maintenance should be required, only use services of the Gnatus Authorized Technical Assistance.

Gnatus shall not be responsible for:

- Use of the equipment differing from that for which it is intended.
- Damages caused to the equipment, the professional and/or the patient by the incorrect installation and erroneous procedures of maintenance, differing from those described in these. Instructions for use which come with the equipment or by the incorrect operation of it.

The sealing bar operation temperature bar is very high. Watch out not to touch it directly.

Precautions and warnings “after” the use of equipment

- Turn off the main switch of the dental set when it is not in use for an extended period of time.
- Always maintain the equipment clean for the next operation.
- Do not modify any part of the equipment. Do not disconnect the cable or other connections without need.
- After using the equipment, clean and disinfect all the parts which may be in contact with the patient.

Precautions and warnings during the “cleaning and disinfection” of equipment

- Before cleaning the equipment, turn off the main switch.
- Avoid spilling water, or other liquids inside the equipment, which could cause short circuits.
- Do not use microabrasive material or steel wool when cleaning, or employ organic solvents or detergents which contain solvents such as ether, stain remover, gasoline etc.

PRECAUTIONS, RESTRICTIONS AND WARNINGS

Precautions in case of alteration in the functioning of equipment

- If the equipment has any abnormality, check if the problem is related to any item listed in the topic of unforeseen events (failures, causes and solutions). If it is not possible to resolve the problem, turn off the equipment, remove the power supply cable from the socket and contact your representative (Gnatus).

Precautions to be adopted against foreseeable or uncommon risks, related to the deactivation and abandoning of equipment

In order to avoid environmental contamination or undue use of the Equipment after it has become useless, it should be discarded in the suitable place (as per the local legislation of the country).

- Pay attention to the local legislation of the country for the conditions of installation and disposal of residue.

Additional procedures for reuse

The equipment can be reused in undetermined, i.e. unlimited, quantities, only needing to be cleaned and disinfected.

CORRECTIVE AND PREVENTIVE MAINTENANCE AND PRESERVATION

Clean

Important: In order to execute cleaning or any type of maintenance, ensure that the equipment is disconnected from the electrical network.

To clean the equipment, we recommend the use of "BactSpray" (Reg n° MS: 3.2079.0041.001-5) or any other similar product: Active component:

Benzalkonium chloride (tri-quaternary ammonium)

Solution 50%..... 0.329%

Chemical composition: Butyl Glycol, Decyl polyglucose, Sodium Benzoate,

Sodium Nitrate, Essence, Deodorized Propane / Butane, demineralized Water.

For more information concerning cleaning procedures, see manufacturer's instructions.

NOTE: The registration at the Ministry of Health of the "BactSpray" is executed separately from the product described in this manual, as the "BactSpray" is not manufactured by Gnatus.

WARNING:

- In order to prevent risks and damages to equipment, make sure that the liquid does not enter into the unit.

- The application of other solvent-based cleaning products or sodium hypochloride isn't recommended, because they may damage the equipment.

CORRECTIVE AND PREVENTIVE MAINTENANCE AND PRESERVATION

Disinfection

- Use clean and soft cloth dampened in alcohol 70% to disinfection of the equipment.
- Never use corrosive disinfectants or solvents.

Note: Use gloves and other systems of protection, during the disinfection.

Preventive Maintenance

The equipment should be calibrated routinely, as per the legislation in force in the country.

But never with a period exceeding 3 years. In order to protect your equipment, seek Gnatus technical assistance for periodic revisions of preventive maintenance.

Corrective Maintenance


If the equipment has any abnormality, check if the problem is related to any of the items listed in the item Unforeseen Events (situation, cause and solution). If it is not possible to solve the problem, turn off the equipment, and request Gnatus technical assistance.

Replacement of fuses

Remove the power input cable (04) (D).

With the help of a screwdriver, release the fuseholder's cover (E), afterwards replace the fuse.

WARNING: After replacing the fuses for the first time, we recommend to reserve some fuses for future replacements.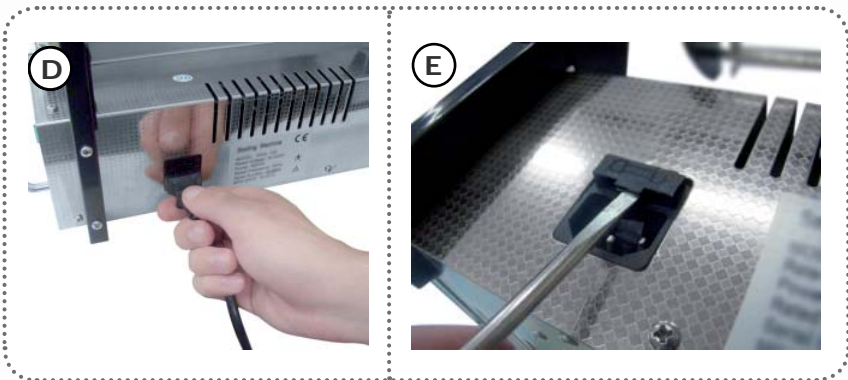

UNFORESEEN EVENTS – SOLUTION OF PROBLEMS

 Upon coming across any problem in operation, follow the instructions below to check and repair the problem, and/or get in touch with your representative.

Problem	Probable cause	Solution
<ul style="list-style-type: none"> - The equipment is not sealing. 	<ul style="list-style-type: none"> - Power input cable disconnected; - Switch turned off; - Burned fuses; - Bad adjustment of temperature; - Lack of pressure to the lever; - Before the first sealing, time was not enough to reach the adjusted temperature; - Burned resistor. 	<ul style="list-style-type: none"> - Connect the cable to the plug - Check the voltage; - Turn on the switch (03); - Replace the fuses; - Adjust the temperature on the switch(08); <ul style="list-style-type: none"> - When operating the lever, put more pressure and hold it for four seconds; - When the Sealer is turned on, before the first sealing, wait until the LED indicator turns off; - Turn the equipment off and contact the technical assistance.
<ul style="list-style-type: none"> - Inappropriate cut. 	<ul style="list-style-type: none"> - Low speed while cutting; - Knife does not cut anymore. 	<ul style="list-style-type: none"> - Pull the knife's handle quickly; - Turn the equipment off and contact the technical assistance.

WARRANTY OF EQUIPMENT

This equipment is covered by the warranty terms counting from the date of installation, as specified below; provided that the defect has occurred in normal conditions of use and that the equipment has not remained stored for more than 06 months counting from the issue date of the sales document until the date of the actual installation.

- WARRANTY TERMS: TWELVE MONTHS;

Obs.: It is not covered by the warranty the following items: Fuses.

- LOSS OF THE WARRANTY:

A) Attempt to repair using an inadequate tool or by unauthorized technicians;

B) Installation of the equipment by an unauthorized technician;

C) Damage arising from inappropriate storage or signs of infringement;

D) Incorrect use of the equipment;

E) Use of a cleaning product not indicated by the factory;

F) Falls or blows which the equipment may undergo or lack of observation of a compliance with the guidelines of the Owner's Manual, which was delivered with the present document, together with the equipment. Repair or replacement of parts during the warranty period shall not extend the validity term of their warranty.

- This warranty does not exempt the customer from paying the service charge for the visit and the travel expenses of the technician, except when the customer sends the equipment to execute the maintenance inside the establishment of the technical assistance.

"Consumer Defense Code - art. 50, unique paragraph".

- The Warranty Certificate comes with the product and must be filled in upon the date of installation by the Gnatius Authorized Technician.

- Queries and information: GNATUS Help Desk (+55) 16 2102-5000.

FINAL CONSIDERATIONS

The most important aspect related to equipment care is that concerning spare parts.

To guarantee the life span of your equipment, use only original Gnatius spare parts. They are sure to follow the technical specifications and standards required by Gnatius.

We must also point out to you our chain of authorized dealers. Only dealers that make part of this chain will be able to keep your equipment constantly new for they count on technical assistants who have been trained and on specific tools for the correct maintenance of your equipment.

Doubts and information: GNATUS Call center (55-16) 2102-5000.

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