TCW 2043 SDD

SOLAR DIRECT DRIVEN
REFRIGERATOR / FREEZER

PQS CODE E003/043

INSTALLATION, OPERATION and MAINTENANCE

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CONTENTS

	page
GENERAL INFORMATION.....	4
SAFETY.....	4
TRANSPORT.....	4
ENVIRONMENTAL PROTECTION.....	4
INSTALLATION.....	5
CLEANING / DISINFECTING.....	5
ELECTRICAL CONNECTION.....	6
CONNECTING TO THE DOMETIC SOLAR GENERATOR SYSTEM.....	6
OPERATION.....	7
TEMPERATURE-DISPLAY.....	7
TEMPERATURE-SETPOINTS.....	7
STORAGE OF VACCINES IN THE REFRIGERATOR.....	7
FREEZER COMPARTMENT.....	8
USING THE TCW2043SDD.....	8
CONDENSATION WATER.....	9
CLEANING.....	9
PERIODIC MAINTENANCE.....	10
DAILY.....	10
WEEKLY.....	10
MONTHLY.....	10
TROUBLE SHOOTING.....	10
SPARE PARTS - OUTSIDE.....	35
SPARE PARTS - INSIDE.....	36
SPARE PARTS - ELECTRONIC.....	37
SPARE PARTS - COMPRESSOR.....	38
WIRING DIAGRAM.....	39

GENERAL INFORMATION

- The TCW2043SDD consists of 2 separate units : a refrigerator and a freezer.
- Before using the TCW2043SDD, read these operating instructions carefully, including all the information on operating safety, use and maintenance.
- Keep these operating instructions ready at hand and leave them with the unit, so that all users can find out about the functions and safety regulations.
- Ensure that there is sufficient room around the unit for air circulation. Please refer to the installation information regarding this point.
- The refrigerator as well as the freezer compartment has a bank of waterpacks filled with water and distributed inside the cabinet. During operation the waterpacks freeze and act as cold storage for protection of the vaccines as well as the icepacks stored in the TCW2043SDD during periods without energy supply (night time).

SAFETY

- The unit must only be used by adults. This appliance is not intended for use by persons (including children) with reduced physical, sensory or mental capabilities, or lack of experience and knowledge, unless they have been given supervision or instruction concerning use of the appliance by a person responsible for their safety. Children should be supervised to ensure that they do not play with the appliance.
- All installation work and adjustments to the unit must only be carried out by qualified personal. Work performed by persons with insufficient technical knowledge may adversely affect the performance of the unit or cause physical injury or damage to the equipment.
- All servicing and repairs must only be carried out by a qualified customer service engineer. Only genuine spare parts must be used.
- Ensure that the connecting cable is not squeezed or bent when the unit is being installed or moved.
- Before cleaning or carrying out maintenance work, always switch the unit off and disconnect from power supply.

WARNING : The unit's cooling system contains flammable refrigerant. Due to this fact, the following information is particularly important.

- Do not damage the cooling system! Both the unit and the products stored inside can be severely damaged if this cooling system starts to leak. Make sure, therefore, that no sharp or pointed objects come into contact with the cooling system.
- No electrical equipment may be operated inside the appliance.
- The refrigerant in the appliance is an environment-friendly. Before disposing of the appliance, the refrigeration circuit should be opened outside so that the refrigerant can escape.
Attention: The refrigerant is flammable!

TRANSPORT

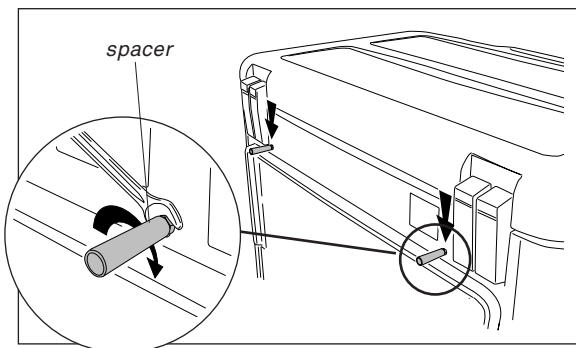
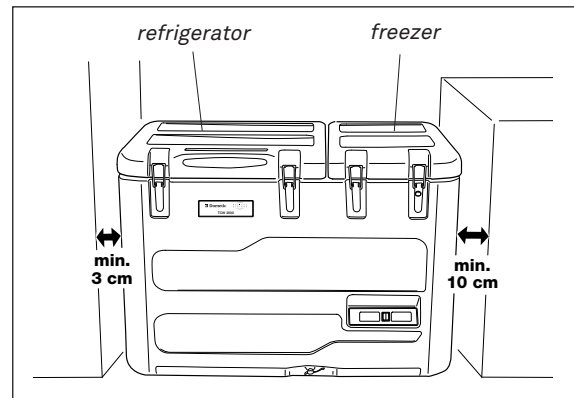
- Check whether the unit has been delivered undamaged. If you find that damage has occurred in transit, immediately contact the delivery service or relevant sales outlet, submitting the delivery note or proof of purchase. Do not operate a unit that has been damaged in transit! If you are unsure, contact your sales outlet and ask them.
- The unit must be transported in an upright position only (maximum inclination 45°).

ENVIRONMENTAL PROTECTION

- Make your contribution to saving the environment. Bear in mind that orderly and proper disposal is required. Packaging materials and devices are always recyclable and should be taken for recycling.
- Before scrapping an old unit, remove the lids so that children cannot lock themselves inside while playing.
- When disposing of the unit, make sure that it does not get too hot, as combustible gas would cause the insulating foam to froth up.


INSTALLATION

- The TCW2043SDD must be set up in a dry well-ventilated place. Avoid direct sunlight or locating it close to a heat source. The place must be protected against bearing rain and dust.
- Ensure that the unit is steady and perfectly straight and does not touch any of the units next to it.
- A good air circulation especially around the compressor is essential for trouble-free operation. Keep the ventilation openings of the compressor cover always free from any obstructions. Make sure that a minimum distance is provided between the TCW2043SDD and any wall or other device located next to it.

- Two spacers are included with the TCW2043SDD. These must be attached to the rear of the unit as shown in the figure, before positioning the unit against the wall.
- Do not locate the unit below a ceiling fan or right next to air-conditioning equipment.
- To ensure proper function, the device must be standing horizontally. Use a water level to check and to correct if necessary.

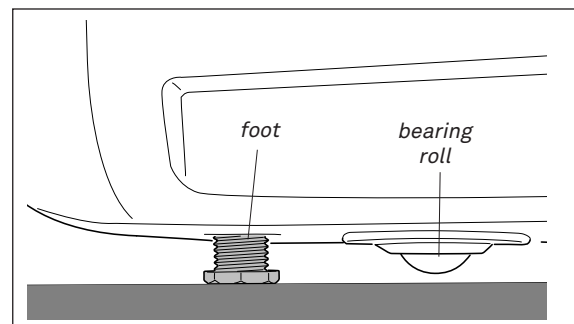- The TCW2043SDD is equipped with bearing rolls. Once it is in its final position, the feet must be unscrewed to immobilise it and stop it rolling away. Screw the feet with the provided key until the rolls do not touch the floor anymore.
- Make sure that during operation the drainage plugs are closed properly.

CLEANING / DISINFECTING

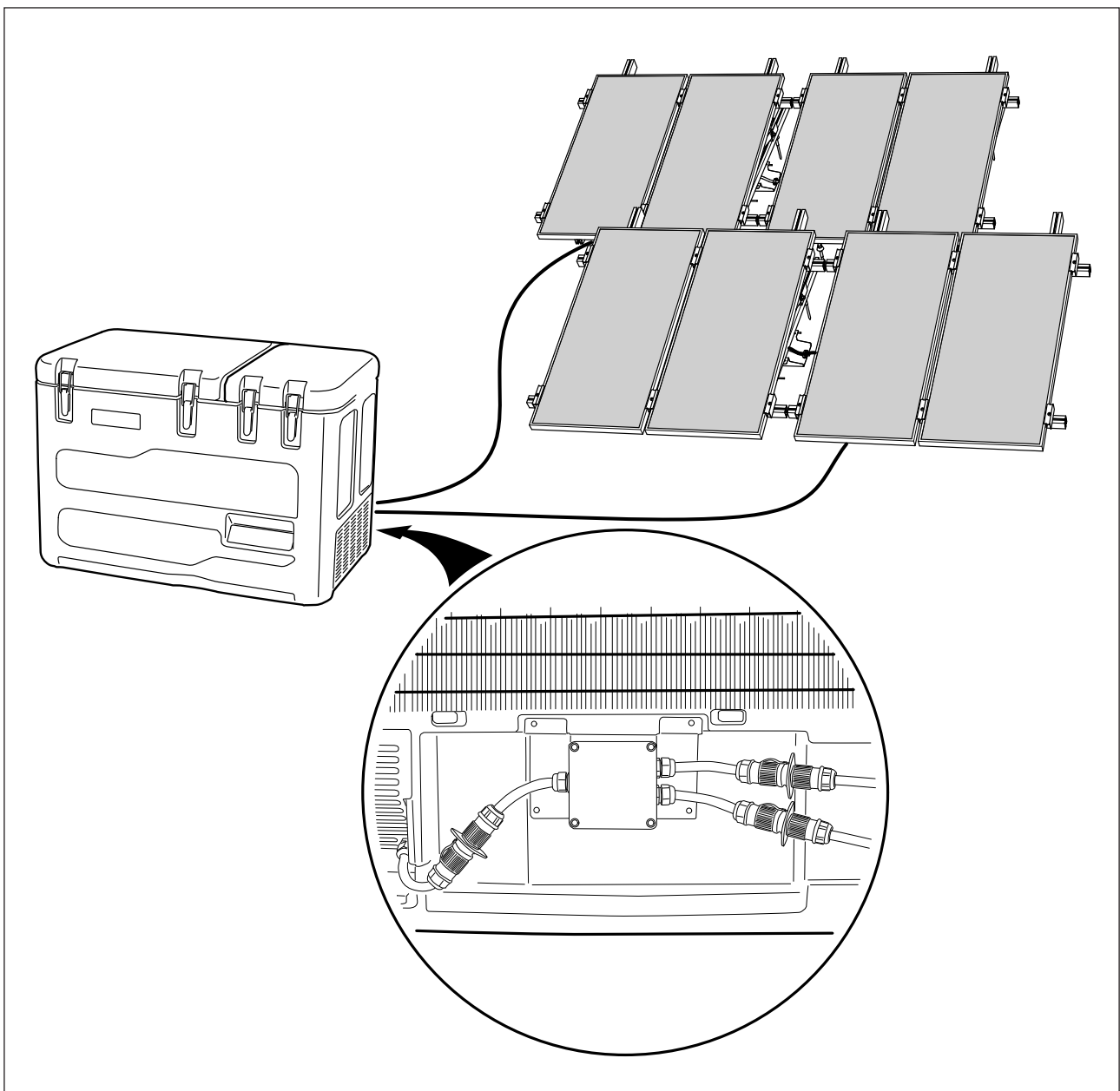
- Clean or disinfect the unit inside and outside before using it. Use only gentle cleaning agents. Never use aggressive or caustic cleaning agents, scouring powder, steel wool, abrasive sponges or chemical solvents. When cleaning, make sure that no fluids of any kind run into the ventilation housing.
- For disinfecting, we recommend all the surface disinfecting agents commonly used by the customer, provided they are recommended by the German Association for Hygiene and Microbiology (DGHM) or other national organisations. For disinfecting small areas, we recommend using a concentrated alcoholic agent.

ELECTRICAL CONNECTION

- The TCW2043SDD is intended to be connected to a double Dometic Solar generator system.
- Each unit (refrigerator and freezer) is designed to work with a minimum solar radiation reference period of 3.5 kWh/m²/day according to WHO-specification.
- Each generator must provide a voltage range of 12V to 20V DC with a minimum power of 320W at STC.
- Before connecting the TCW2043SDD and switching it on for the first time, allow it to stand for 30 minutes.

CONNECTING TO THE DOMETIC SOLAR GENERATOR SYSTEM


- For installation of the solar power generator refer to the Installation instructions.

- Connect the TCW2043SDD to the generators with the provided plug system.

OPERATION

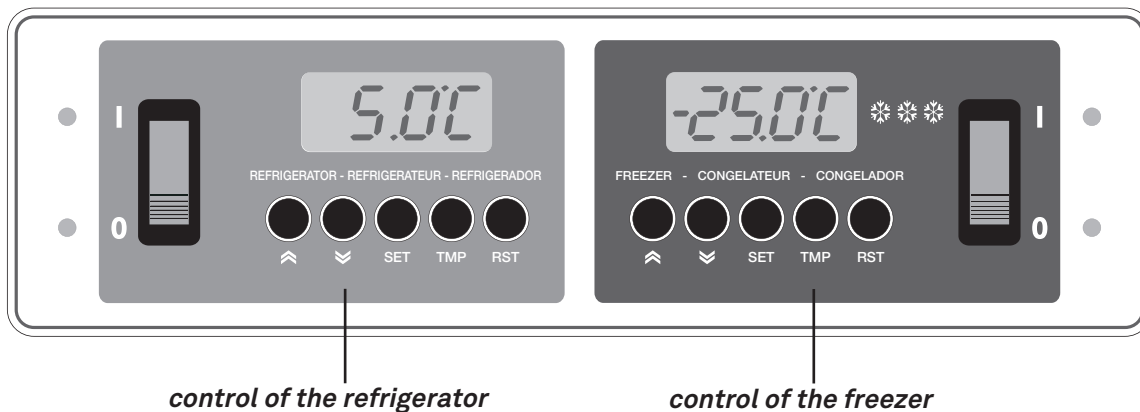TEMPERATURE-DISPLAY

- The TCW2043SDD and its control panel is composed of 2 separate units : the refrigerator part on the left side, and the freezer part on the right side of the control panel.
- In normal use, the display shows the inside temperature of the appliance. This temperature is displayed in °C with one decimal place.
- During times without solar energy supply or in case of power failure the display is switched off. The inside temperature can be checked by pressing the “TMP” button.

TEMPERATURE-SETPOINTS

- The temperature setpoints of the refrigerator- and the freezer unit are preset at the factory at following values: +5°C for the refrigerator, -25°C for the freezer.
- These temperature setpoints cannot be changed by the user.
- Press the SET-key to view the actual temperature setpoint.

STORAGE OF VACCINES IN THE REFRIGERATOR

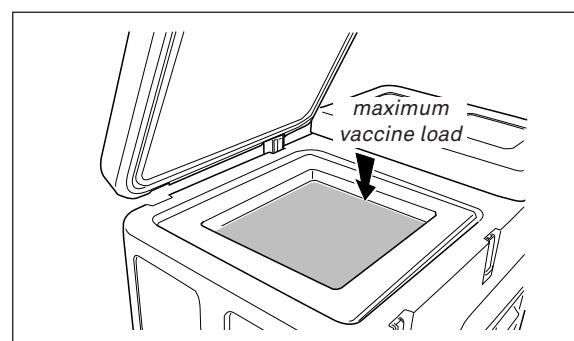
IMPORTANT :

When using the TCW2043SDD refrigerator, it is of utmost importance that the icelining waterpacks are in place. Make sure that all waterpacks are in place before starting the TCW2043SDD in refrigerator mode. Never remove the waterpacks from the unit.

Special care should be taken to preserve the vaccines that get damaged by freezing. Never store them touching the side surfaces in the cabinet of a TCW2043SDD. It is advisable to preserve the vaccines in the provided baskets.

For storing vaccines :


- Keep the packets containing the vaccines in neat rows.
- Different vaccines should be kept separately to facilitate easy identification.
- Keep about 2 cm space between rows for circulation of air.
- Do not load vaccines higher than the indicated maximum level.

FREEZER COMPARTMENT

IMPORTANT :

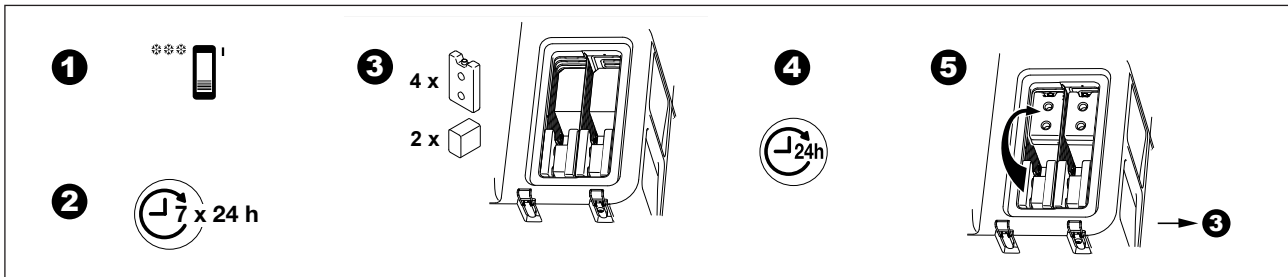
The freezing capacity of the freezer compartment is highly dependent on the intensity and duration of solar radiation, the ambient temperature and the usage of the refrigerator compartment. For optimum performance it is of utmost importance to follow the steps described below.

- Switch on the freezer unit ❶ and allow it to run for one week until the icelining waterpacks are completely frozen ❷ .
- After one week you can insert 4 icepacks into the freezing area of the compartment ❸. Use the provided separation part to press the icepacks against the sidewalls of the freezer compartment. These 4 icepacks will freeze within 24 hours ❹ .
- After these 24 hours take the frozen icepacks and put them into the storage area of the freezer compartment ❺. Insert 4 new unfrozen icepacks into the freezing area ❸.



USING THE TCW2043SDD

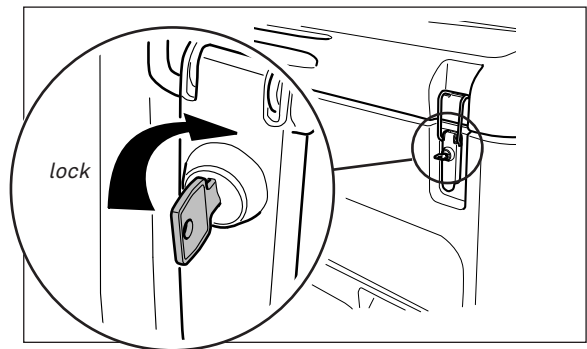- Install the TCW2043SDD as detailed under “Installation”.
- Shift the ON/OFF-switch to position “I”. The temperature display lights up to show that the unit is ready for operation.
- After switching on, the compressor only starts after a delay of 3 minutes. This also applies, if the equipment is switched off and on again.
- Close the lid and allow the unit to run. The temperature inside the cabinet is indicated on the display. Observe the temperature from time to time.
- Initially the temperature inside the TCW2043SDD will be observed to be similar to or slightly below the room temperature. It will decrease slowly and remain steady after several days.
- In order to assure a complete freezing of the icelining waterpacks it is advisable to start loading the TCW2043SDD with the vaccines after 1 week of operation.
- When the temperature inside is observed to be steady for several hours within the required range, only then the vaccines to be preserved should be transferred to the TCW2043SDD.
- The vaccines are advised to be kept in the baskets provided.
- After storing the vaccines keep the lid properly closed and locked.
- When fresh vaccines are stored in the TCW2043SDD, the cabinet temperature may go up initially but should fall and come to the required range after few hours.
- The cabinet temperature may vary depending upon various factors, such as,
 - the ambient temperature
 - the intensity and duration of solar radiation
 - amount of vaccine preserved in the TCW2043SDD
 - frequency of lid opening
 - circulation of air around the TCW2043SDD

CONDENSATION WATER

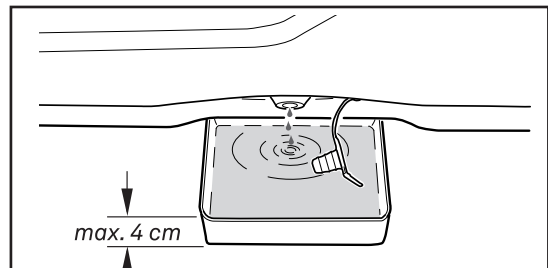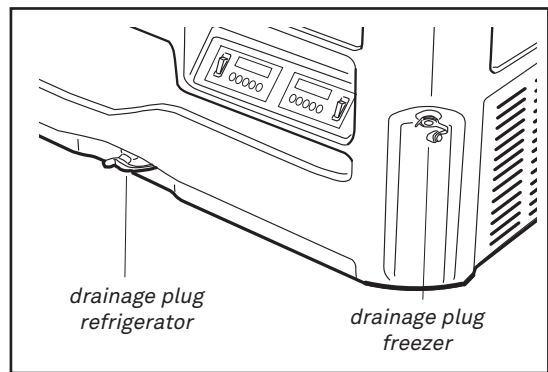- The moisture in the air, which enters the cabinet due to lid opening is attracted by the cold surfaces inside. The moisture will condensate on these cold surfaces. The condensation water is collected at the bottom of the inner cabinet and must be evacuated on a regular basis (at least once a week) using the drainage plugs on the front side of the cabinet.
- Open the drainage plug at the bottom front side of the cabinet for the refrigerator or at the right corner for the freezer compartment. Keep a suitable container under the drain hose to collect the water.
- Close the drainage plug after all water is evacuated.

CLEANING

IMPORTANT :

Before cleaning the TCW2043SDD, the vaccines preserved in it will have to be removed and stored temporarily in other working refrigerator. If a second refrigerator is not available, the vaccines will have to be preserved in Cold-Box or Vaccine carrier.

- Switch off the desired unit. The second unit may continue working.

Refrigerator unit :

- Prepare the temporary storage and transfer the vaccines and preserve them properly with care.
- Gently remove the cabinet frame and the complete inner liner by holding it on its two opposite sides and lifting it uniformly.
- Clean all parts inside and outside the cabinet, the lid and the lid gaskets with warm water and mild detergent. Allow the cleaned parts to dry completely.
- Examine the waterpacks for any leakage. A defective waterpack will have to be replaced.
- After the evaporator surface is free of condensation water, put back carefully the inner liner into its position.
- Replace the cabinet frame.

Freezer unit :

- Take the frozen ice packs out. Keep them wrapped together with a thick blanket or similar insulating material. This will keep the coldness of the icepacks during cleaning of the unit.
- Clean all parts inside and outside the cabinet, the lid and the lid gaskets with warm water and mild detergent. Allow the cleaned parts to dry completely.
- Examine the waterpacks for any leakage. A defective waterpack will have to be replaced.
- Close the lid and switch on the unit.
- Allow it to run and observe the cabinet temperature on the thermometer.
- Put back the vaccines into the refrigerator, only when it has attained safe recommended temperature range for storage of the vaccines.

PERIODIC MAINTENANCE

DAILY

- Evacuate the condensation water as described before.
- Take temperature readings and note down the temperature and the time of reading. Keep the temperature records systematically. It is suggested that minimum 2 readings should be taken (in the morning and afternoon) preferably at the same hours of each day.

WEEKLY

- If it is observed that quantity of condensation water is increased :
 - Examine the gaskets on the lid, if they sit properly. Any gap between the cabinet and gaskets will allow outside air to go in and form condensation. In such cases the hinges should be adjusted or the gaskets should be changed.
 - Restrict and reduce frequency of opening the TCW2043SDD. Open only when absolutely necessary.

MONTHLY

- Clean the unit from any dust :
 - Switch off the TCW2043SDD and disconnect it from power supply.
 - Clean the condenser on the back side of the unit with a soft brush or a vacuum cleaner.
- Clean the lid seal.

TROUBLE SHOOTING

IMPORTANT :

If a TCW2043SDD is found to be not working at all or not working properly see that the vaccine temperature is within the recommended limits. Do not open the lid unless very essential.

For any abnormal sound, smoke, smell etc. in the TCW2043SDD disconnect the unit from power supply and notify refrigerator technician.

The electronic displays an error message :

F1/ F2 : Probe failure. The unit will continue to work in “ON/OFF-mode” to keep the temperature in the desired range - notify Refrigerator Technician.

E1 : Voltage protection cut-out. The voltage is outside the cut-out setting - check power supply.

E3 : Compressor start error. The compressor cannot start. Keep the unit switched on. This problem might occur if the solar generator doesn't deliver enough power (cloudy day, shadow on solar panels, bad positioning of solar panels etc.). Check the actuals situation of the panels, and if the problem persists after necessary corrections, notify Refrigerator Technician.

E4 : Minimum compressor speed error. Keep the unit switched on. This problem might occur if the solar generator doesn't deliver enough power (cloudy day, shadow on solar panels, bad positioning of solar panels etc.). Check the actuals situation of the panels, and if the problem persists after necessary corrections, notify Refrigerator Technician.

E5 : Thermal cut-out of electronic unit. The ambient temperature is too high, the electronic unit will run too hot. Switch the unit off. If after one hour the problem still persists, notify Refrigerator Technician.

HIGH : The measured temperature is high than +40°C.

LOW : The measured temperature is lower than -40°C.

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