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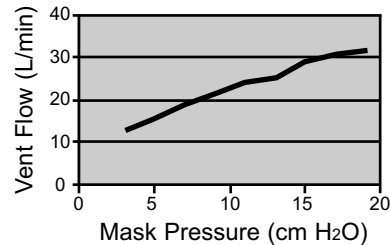
### VENT HOLE PRESSURE/FLOW CHARACTERISTICS (Approx.)

CARACTERÍSTICAS DE PRESIÓN/FLUJO DEL ORIFICIO DE VENTILACIÓN (APROX.)  
ENTLÜFTUNGSLOCHDRUCK / FLIESSVERHALTEN (ungefähr)  
CARACTÉRISTIQUES DE PRESSION/DÉBIT DES ORIFICES D'ÉVACUATION (approx.)  
VENTILATIEOPENINGEN DRUK-/STROMINGSEIGENSCHAPPEN (bij benadering)  
CARATTERISTICHE PRESSIONE/FLUSSO FORO DI SFOGO (Appross.)  
ΠΙΕΣΗ ΟΠΗΣ ΑΕΡΑ / ΧΑΡΑΚΤΗΡΙΣΤΙΚΑ ΡΟΗΣ (Περίπου)  
LUFTHULSTRYK/GENNEMSTRØMNINGSKENDETEGN (ca.)  
TRYCK/FLÖDESKARAKTÄRISTIK, VENTILATIONSHÅL (approximativt)  
CARACTERÍSTICAS DA PRESSÃO/FLUXO DOS ORIFÍCIOS DE VENTILAÇÃO (aprox.)  
WŁAŚCIWOŚCI CIŚNIENIA OTWORU ODPOWIETRZAJĄCEGO/PRZEPLYWU (W przybliżeniu)  
TUULETUSAUKON PAINE/VIRTAUSOMINAISUUDET (likimääräiset)  
HAVA DELİĞİ BASINCI/AKIŞ ÖZELLİKLERİ (Yaklaşık)

Vent Flow (L/min)	Mask Pressure (cm H <sub>2</sub> O)
Flujo de ventilación	Presión de la máscara
Entlüftungsrate	Maskendruck
Débit d'évacuation	Pression du masque
Ventilatiestroming	Maskerdruk
Flusso foro di sfogo	Pressione maschera
Ροή Αεραγωγού (Λίτρα/ λεπτό)	Πίεση Μάσκας (εκ. H <sub>2</sub> O)
Lufthulsgennemstrømning	Masketryk
Ventilationsflöde	Masktryck
Fluxo dos Orifícios	Pressão da Máscara
Przepływ odpowietrzający	Ciśnienie maski
Virtaus tuuletusaukoissa	Maskin paine
Delik Akışı (l/dk)	Maske Basıncı (cm H <sub>2</sub> O)

### IQ<sup>®</sup> Nasal Mask

CONTENTS: IQ<sup>®</sup> Nasal Mask, Tubing Assembly, and Headgear

Rx ONLY LATEX FREE MADE IN U.S.A.

The SleepNet Corporation IQ<sup>®</sup> Nasal Mask is intended to be used with positive airway pressure devices, such as CPAP, operating at or above 3 cm H<sub>2</sub>O for the treatment of adult obstructive sleep apnea.

#### GENERAL INFORMATION - WARNINGS AND CAUTIONS

**CAUTION:** U.S. Federal Law restricts this device for sale by or on the order of a physician.

**CAUTION:** Do not expose the IQ<sup>®</sup> Nasal Mask to temperatures above 170°F (75°C).

**CAUTION:** This mask may be cleaned and used repeatedly on the same person. Do not sterilize. USA ONLY: For multiple patients use, high level disinfection protocol must be followed.

**WARNING:** At low baseline pressure, the flow through the vent holes may be inadequate to clear all exhaled gas from the tubing. Some re-breathing may occur.

**WARNING:** This mask should be used with the positive airway pressure device recommended by the manufacturer, your physician, or a respiratory therapist. The mask should not be used unless the positive airway pressure device is turned on and operating properly. The mask's vent holes should never be blocked. The vent holes allow a continuous flow of air out of the mask. When the positive airway pressure device is turned on and functioning properly, new air from the device flushes the exhaled air out through the mask's vent holes. When the positive airway pressure device is not operating, exhaled air may be re-breathed. Re-breathing of exhaled air can, in some circumstances, lead to suffocation. This warning applies to most models of nasal positive airway pressure devices and masks.

**WARNING:** Do not use the IQ<sup>®</sup> Nasal Mask with baseline pressure less than 3 cm H<sub>2</sub>O or greater than 20 cm H<sub>2</sub>O.

**WARNING:** Consult manufacturer for proper O<sub>2</sub> enrichment tubing.

#### GENERAL INFORMATION

The IQ<sup>®</sup> Nasal Mask has a 3 month warranty for manufacturing defects. If a manufacturing defect is suspected, contact your supplier. The mask has an expected service life of 6 to 12 months. Please dispose of the mask and accessories in accordance with local and regional guidelines. The IQ<sup>®</sup> Nasal Mask and its accessories are Latex Free.

#### CLEANING AND MAINTENANCE - SINGLE PATIENT USE

Cleaning is recommended on a daily basis. Wash the mask and components in warm water using a mild detergent. Rinse the mask thoroughly after washing. Visually verify that the vent holes are clear and allow the mask to air dry. Washing your face prior to fitting the mask will remove excess facial oils and help extend the life of the mask.

**NOTE:** Inspect mask prior to each use. Replace the mask if parts are damaged or gel is exposed due to tear or punctures.

**NOTE:** The gel cushion, although silky to touch, will not tear, leak or detach from the mask under normal use and following recommended cleaning and maintenance protocols. However, the gel cushion is not indestructible. It will tear, cut and rip if subjected to mishandling or abuse. Do not clean in a dishwashing machine. The warranty does not cover these types of product failures. Therefore, please handle your new IQ<sup>®</sup> Nasal Mask with care.

#### TECHNICAL INFORMATION

Resistance to air flow at 50 L/min: 1.10 cm H<sub>2</sub>O; at 100 L/min: 2.52 cm H<sub>2</sub>O

A-weighted Sound Pressure Level at 1 m distance: 29.9 db (A); A-weighted Sound Power Level: 37.9 db (A)

### User Instructions for IQ<sup>®</sup> 3 Point Headgear

1. Attach tubing to mask.
2. Thread lower straps through slots in the mask headgear connector, and thread center strap through top slot on headgear connector.
3. Place headgear on head with blue fabric facing outward. Pull straps outward to center the mask over nose. Affix Velcro tabs to straps and adjust as necessary.
4. Adjust center strap. Attach tubing to headgear with loop strap, if preferred.

**NOTE:** The mask should rest comfortably against the face and still provide an effective air seal. Pulling the straps too tight can cause skin irritation. The mask should not constrict your nose; you should be able to breathe freely and comfortably. If the mask and headgear are adjusted correctly and a problem persists, contact your healthcare professional.

5. Attach mask tubing to CPAP tubing; turn CPAP device on and verify airflow through mask.
6. Gently bend pliable ring embedded inside mask shell to change mask shape, create a comfortable fit, and reduce leaks.

#### REMOVING THE MASK AND HEADGEAR

7. Disconnect the mask tubing assembly from the CPAP tubing by pressing the sides of the quick connect. Part of the quick connect attachment will remain on the CPAP tubing.
8. Open one of the horizontal straps and detach from headgear connector on the mask.

**CE** 0123 IQ<sup>®</sup> is a trademark of SleepNet Corporation. Patent Pending. CE Mark and identification number of Notified Body. Product conforms to the essential requirements of the Medical Devices Directive 93/42/EEC.

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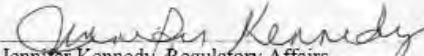
#### DECLARATION OF CONFORMITY

MANUFACTURER:	SLEEPNET CORPORATION 5 MERRILL INDUSTRIAL DRIVE HAMPTON, NH USA 03842
AUTHORIZED REPRESENTATIVE:	MEDICAL PRODUCT SERVICES GmbH BORNGASSE 20 35619 BRAUNFELS GERMANY
PRODUCTS:	PHANTOM NASAL MASK, IQ NASAL MASK, MiniMe NASAL MASK & MOJO FULL FACE MASK
PRODUCT TYPE:	POSITIVE PRESSURE BREATHING ATTACHMENT
PRODUCT OPTIONS:	ACCESSORIES: PHANTOM NASAL MASK: HEADGEAR  IQ NASAL MASK: HEADGEAR  MiniMe NASAL MASK: HEADGEAR  MOJO FULL FACE MASK: HEADGEAR
VARIANTS:	N/A
PRODUCT CLASSIFICATION:	IIa, ANNEX IX RULE 2
WE HEREWITH DECLARE THAT THE ABOVE MENTIONED PRODUCTS MEET THE PROVISIONS OF THE COUNCIL DIRECTIVE 93/42/EEC FOR MEDICAL DEVICES (MDD 93/42/EEC).	
STANDARDS APPLIED:	MDD ESSENTIAL REQUIREMENTS
GMDN CODES:	NASAL MASK: P36033 FULL FACE MASK: P44521
UMDNS CODE:	12447
NOTIFIED BODY:	TUV PRODUCT SERVICE, GmbH

THIS DECLARATION APPLIES TO ALL CE MARKED DEVICES MANUFACTURED FROM THE DATE OF ISSUANCE UNTIL IT IS EITHER SUPERCEDED BY ANOTHER DECLARATION OR WITHDRAWN.

HAMPTON, NH 03842 USA. 2008

  
Jennifer Kennedy, Regulatory Affairs  
Date: 8/13/08