

Baha[®] Compact

Patient user manual

Baha[®] Compact, multilingual

Hear now. And always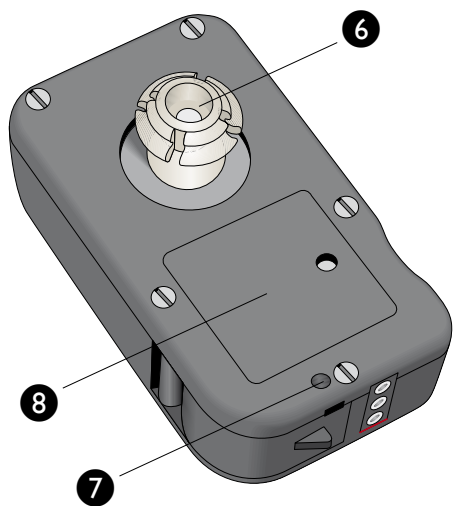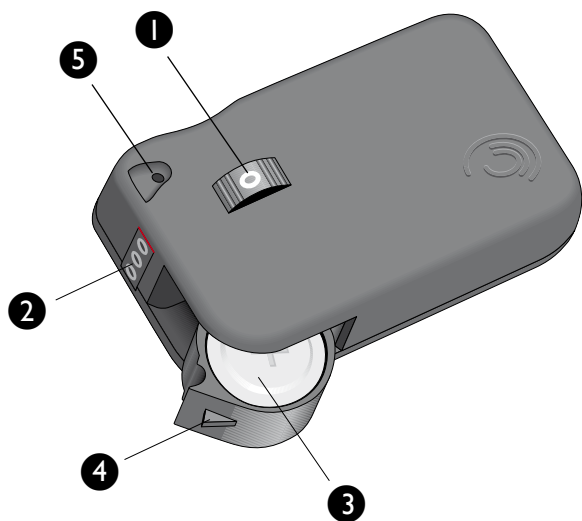

Contents	Dansk	4
	Deutsch	10
	English	16
	Español	22
	Français	28
	Ελληνικά	34
	Italiano	40
	Nederlands	46
	Polski	52
	Português	58
	Suomi	64
	Svenska	69
	Türkçe	74
	Instructions for use in the USA	83

Brugsanvisning for Baha® Compact

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User's Manual for Baha® Compact

Baha® Compact is a bone anchored sound processor which allows you to hear via direct bone conduction. Direct bone conduction means that sound is transmitted directly through the skull bone via a titanium implant, bypassing the middle ear. The sound processor is held in place by an abutment which is directly connected to the implant. The titanium implant is firmly anchored in the bone by the process of osseointegration.

Please read this manual carefully to learn about your sound processor. Be sure to discuss any questions or concerns that you may have regarding hearing or use of this system with your hearing health care professional.

Illustration of the sound processor

This list refers to the illustration located on the inside of the front page.


- ❶ Volume control, on / off
- ❷ Electrical input
- ❸ Battery compartment
- ❹ Battery door
- ❺ Microphone
- ❻ Plastic snap
- ❼ Attachment for safety line
- ❽ Serial number

How to connect and disconnect the sound processor

A plastic snap is mounted to the sound processor. It is designed to snap into the abutment and hold the sound processor securely in place.

Connecting

Hold the sound processor so that the plastic snap is accessible and move any hair away from the abutment. Bend and push the unit in carefully and it will attach to the abutment.

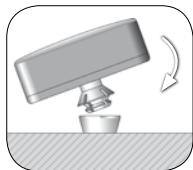Disconnecting

Bending the sound processor slightly in any direction will release it from the abutment.

Note: *Do not push the sound processor straight in and do not pull it straight out.*

Volume control

If you turn the volume control upwards the volume increases and if you turn it downwards it will decrease. If you want to switch off the sound processor, turn the volume control all the way down until it clicks off.

Electrical input

The sound processor has a three-pin electrical input designed for connecting external equipment. Note that the red marking on the plug must match up with the corresponding marking on the sound processor.

Safety instructions when connecting external equipment

Only connection cables supplied by Cochlear should be used. Non-standard connecting cables can cause damage to the sound processor and injury to the patient.

Cochlear is not liable for any damage arising as a result of using incorrect equipment or incorrect use.

WARNING! It is extremely dangerous and potentially fatal to connect any equipment that is plugged into a 110 and / or 220 V power supply without using the audio adapter.

Maintenance

The sound processor can be worn during waking hours apart from when you take a shower, swim or engage in physical activity that might damage it. Ensure that the sound processor is switched off when not in use to save the battery.

At night, and in a clean environment, the box and battery compartment may be left open to allow any trapped moisture to evaporate.

Battery replacement

Zinc-air batteries type 13 should be used. Please note that these batteries are not rechargeable.

Open the battery compartment by releasing the catch on the 'door', which allows it to swing outwards. Turn the sound processor over and shake the device gently or pull the battery out. Place the new battery in the door with the + mark facing upwards and close the door.

WARNING! The Baha sound processor contains small parts that may be a risk for small children. Batteries can be harmful if swallowed. Be sure to always keep your batteries out of reach of small children or people of diminished mental capacity.

In the event a battery is accidentally swallowed, seek prompt medical attention at the nearest emergency centre.

Cleaning

You must clean the skin around the abutment daily. Skin debris may build up around the abutment and it is important that you remove it. Use a soft cleaning brush, soap and water.

Contact your doctor if you have persistent problems such as soreness, formation of pus or inflammation around the abutment.


For children who have been fitted with a Baha it is extremely important that the parent or caregiver of the child inspect and aid in the proper cleaning and maintenance, as detailed in the information above.

Telephones

To use a telephone with your sound processor, hold the receiver in the normal way, but close to the microphone inlet of the sound processor instead of close to your ear. It is important that you do not touch the sound processor since this may cause feedback ("whistling").

Attachment of the safety line

When you engage in physical activities we advise you to attach a safety line to your sound processor so that it is not lost if it disconnects (7). The clip can then be fastened to your shirt or jacket collar.

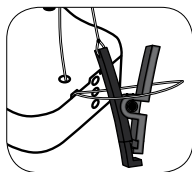
Abutment cover

An abutment cover is included in the kit. It can be placed over the abutment when the sound processor is not in place. Simply press the cover on to the abutment.

Share the Experience™

The test rod provided can be used by others to listen to your Baha sound processor themselves. Snap the sound processor on to the test rod and hold the rod against the skull bone behind the ear (take care not to allow the sound processor itself to touch anything as this may cause feedback).

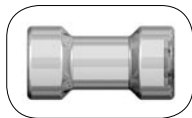Accessories

There are several accessories available for your sound processor. Please contact your hearing aid centre for more information.

General advice

- The sound processor is not waterproof! Never wear it in the bath or shower! If it gets wet by accident open the battery door immediately and take the battery out. Leave the battery door open until it is dry.
- Do not expose the sound processor to extreme heat etc.
- If you are to undergo an MRI (Magnetic Resonance Imaging) procedure, the sound processor should be removed beforehand. The implant and the abutment remain in place.
- Your sound processor should be protected from dust and dirt so always keep it in its box when not being worn.

Troubleshooting

- If pain, redness or soreness occurs, clean the skin around the abutment and apply a mild antibiotic cream / ointment provided by the clinic / your health professional.
- If pain, redness or soreness persists, you should contact your clinic.
- If an implant loss occurs you should contact your clinic or surgeon. Implant loss can be caused by trauma, infection or be simply spontaneous (without any specific reason). A new implant can easily be surgically inserted in the same place.
- If the sound processor makes a crackling or buzzing sound try changing the battery.
- If the sound processor does not work (is silent) or is working intermittently, change the battery and check that the position of the volume control is correct.
- If either of the sound processor problems above persists you should contact your hearing aid centre.
- If the sound processor is in contact with the surrounding skin and this causes feedback problems (whistling sound), the skin may have to be reduced around the implant site. Please contact your clinic.

Service and repairs

If there is a fault with your Baha sound processor, contact your ear clinic or hearing aid centre and they will help you to get it serviced.

Telephone number of hearing aid centre: _____

Contact person: _____

Telephone number of ear clinic: _____

Contact person: _____

Manufacturer's Warranty

Cochlear will repair or replace your Baha sound processor without charge during the warranty time if defective materials or workmanship cause the system to be inoperable. The warranty does not cover loss or damage to the sound processor. Batteries and accessories are not covered under the manufacturer's warranty. Please consult your hearing healthcare professional for exact duration, further terms and extent of this warranty.

Your opinion is important. If you have any suggestions for product improvements, please write to the address on the back of this manual.

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Instructions for use in the USA

WARNING to hearing aid dispensers

A prospective user of Baha® must always have a medical evaluation by a licensed physician (preferably an ear specialist) and have a fixture placed in the skull bone before a Baha sound processor can be dispensed.

Important notice for prospective user's of a Baha® sound processor

- A prospective user of Baha must always have a thorough medical evaluation by a licensed physician (preferably a physician who specialises in diseases of the ear, otolaryngologist, otologist or otorhinolaryngologist) and have a fixture placed in the skull bone before a Baha sound processor is purchased.
- The purpose of the medical evaluation is to ensure that all medically treatable conditions that may affect hearing are identified and treated before the sound processor is purchased. Following the medical evaluation, the physician will give you a written statement that states that your hearing loss has been medically evaluated and that you may be considered a candidate for Baha. The physician will refer you to an audiologist or a hearing aid dispenser, as appropriate, for an audiological evaluation.
- Federal law restricts the sale of Baha sound processors to those individuals who have obtained a medical evaluation from a licensed physician. Federal law permits a fully informed adult to sign a waiver statement declining the medical evaluation for religious or personal beliefs that preclude consultation with a physician. The exercise of such a waiver is not in your best health interest and its use is strongly discouraged.

A word of caution about batteries

Batteries can be harmful if swallowed. Be sure to always keep your batteries out of reach of small children or people of diminished mental capacity. In the event a battery is accidentally swallowed, seek prompt medical attention at the nearest emergency centre, or call the **National Button Battery Hotline** collect at (202) 625-3333.

Cochlear™

Cochlear Bone Anchored Solutions AB Konstruktionsvägen 14, SE - 435 33 Mölnlycke,
Sweden Tel: +46 31 792 44 00 Fax: +46 31 792 46 95

Cochlear Americas

400 Inverness Parkway, Suite 400 Englewood CO 80112,
USA Tel: +1 303 790 9010 Fax: +1 303 792 9025

Cochlear AG

European Headquarters, Margarethenstrasse 47, CH - 4053 Basel,
Switzerland
Tel: +41 61 205 0404 Fax: +41 61 205 0405

Cochlear Ltd

(ABN 96 002 618 073) 14 Mars Road, Lane Cove NSW 2066,
Australia Tel: +61 2 9428 6555 Fax: +61 2 9428 6352

Cochlear Benelux NV

Schaliënhoefedreef 20 I, B - 2800 Mechelen,
Belgium Tel: +32 15 36 28 77 Fax: +32 15 36 28 70

Cochlear Canada Inc

2500-120 Adelaide Street West, Toronto, ON M5H 1T1
Canada Tel: +1 416 972 5082 Fax: +1 416 972 5083

Cochlear Bone Anchored Solutions France S.A.S.

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Cochlear Europe Ltd

9 Weybridge Business Park, Addlestone Road, Addlestone, Surrey KT15 2UF,
United Kingdom Tel: +44 1932 87 1500 Fax: +44 1932 87 1526

www.cochlear.com

Baha is a registered trademark of Cochlear Bone Anchored Solutions AB.
Cochlear and the elliptical logo are trademarks of Cochlear Limited.

E80566 D NOV07. Salomonsson's ord.nr: 200708. Printed in Sweden.

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Cochlear GmbH
Cochlear (HK) Ltd
Nihon Cochlear Co Ltd
Cochlear Nordic AB