Basic Stand User Guide

Basic Stand User Guide

Basic Stand-Benutzerhandbuch

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T: +44-1462-444800

F: +44-1462-444801

Caution: | Federal (United States) law restricts this device to sale by or on the order of a physician.

"PowerMap," "SiteCharge," "SitePack," "SonoHeart," "SonoSite," and "SiteStand" are trademarks of SonoSite, Inc.

Non-SonoSite product names may be trademarks or registered trademarks of their respective owners.

SonoSite products may be covered by one or more of the following U.S. patents: 4454884, 4462408, 4469106, 4474184, 4475376, 4515017, 4534357, 4542653, 4543960, 4552607, 4561807, 4566035, 4567895, 4581636, 4591355, 4603702, 4607642, 4644795, 4670339, 4773140, 4817618, 4883059, 4887306, 5016641, 5050610, 5095910, 5099847, 5123415, 5158088, 5197477, 5207225, 5215094, 5226420, 5226422, 5233994, 5255682, 5275167, 5287753, 5305756, 5353354, 5365929, 5381795, 5386830, 5390674, 5402793, 5,423,220, 5438994, 5450851, 5456257, 5471989, 5471990, 5474073, 5476097, 5479930, 5482045, 5482047, 5485842, 5492134, 5517994, 5529070, 5546946, 5555887, 5603323, 5606972, 5617863, 5634465, 5634466, 5636631, 5645066, 5648942, 5669385, 5706819, 5715823, 5718229, 5720291, 5722412, 5752517, 5762067, 5782769, 5800356, 5817024, 5833613, 5846200, 5860924, 5893363, 5916168, 5951478, 6036643, 6102863, 6104126, 6113547, 6117085, 6142946, 6203498 B1, 6371918, 6135961, 6364839, 6383139, 6416475, 6447451, 6471651, 6569101, 6575908, 6604630, 6648826, 6835177, D0280762, D0285484, D0286325, D0300241, D0306343, D0328095, D0369307, D0379231, D456509, D461895. Other patents pending.

P01710-03 10/2006

Copyright 2006 by SonoSite, Inc.

All rights reserved.

Basic Stand User Guide

Contents

Overview	1
Contact Information	3
Assemble Basic Stand	4
Attaching Column to Base	4
Adjusting Basket	5
Attaching Printer Bracket	6
Installing Printer on Printer Bracket	7
Connect Printer	9
Install System	10
Clean and Disinfect Basic Stand	12

Overview

Please read the information in the user guide before using the Basic Stand. It applies to the Basic Stand, accessories, and peripherals. See the manufacturer's instruction for specific information about peripherals.

Basic Stand

The Basic Stand does the following:

- Provides a mobile, ergonomic work surface with a basket.
- Holds the SonoSite system, two transducers, a black and white printer, and accessories.

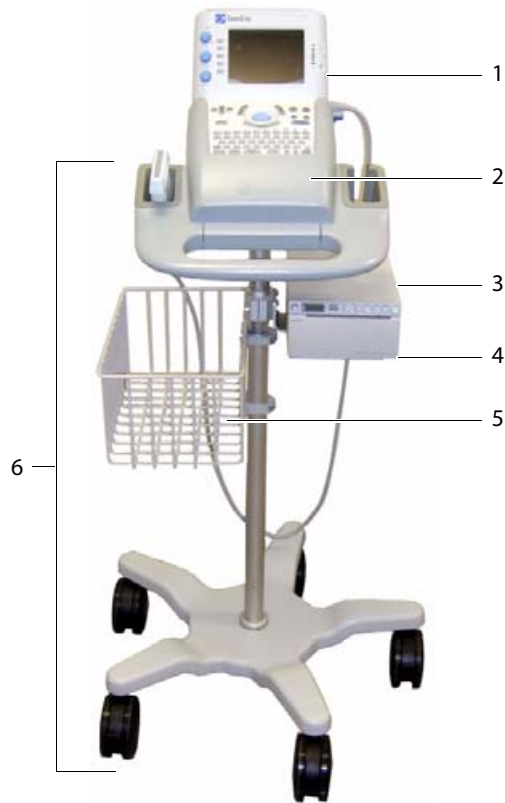
Figure 1 Basic Stand

Table 1: Basic Stand

Number	Description
1	SonoSite system
2	Sleeve
3	Printer
4	Printer tray
5	Basket
6	Basic Stand

Conventions

These conventions are used in this user guide:

- A **WARNING** describes precautions necessary to avoid injury or loss of life.
- A **Caution** describes precautions necessary to protect the products.
- Numbered steps in the operating instructions must be performed in a specific order.
- Bulleted lists present information in list format but do not imply a sequence.

Symbols and Terms

Symbols and terms used on the system and transducers are explained in the *SonoSite Ultrasound System User Guide*.

Contact Information

If you encounter difficulty, contact SonoSite technical support at the following numbers or addresses:

Technical Support (USA, Canada):	1-877-657-8118
Technical Support fax:	1-425-951-6700
Technical Support e-mail:	service@sonosite.com
SonoSite website:	www.sonosite.com and select Support
International Technical Support:	Contact your local representative or call (USA) +425-951-1330
Europe Service Center:	+44-(0)1462-444-800 e-mail: uk.service@sonosite.com

Assemble Basic Stand

The Basic Stand requires the following assembly:

- Attaching column to base
- Adjusting basket
- Attaching printer bracket
- Installing printer on printer bracket

Attaching Column to Base


Provided Tools and Equipment

- Basic Stand
- Printer bracket
- Allen wrench (5/32")
- Phillips screws (4) (10 mm)
- Bolt with washers

Required Tools and Equipment

- 1/2" wrench
- #1 Phillips screwdriver
- Sony black and white printers: UP-897MD, UP-895MD, UP-890MD (optional)

Attach Column to Base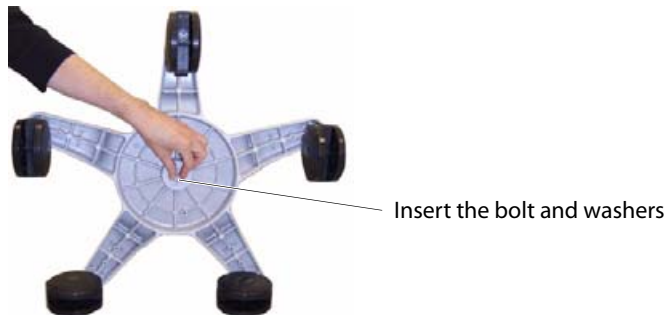


- 1 Remove bolt, flat washer, and lock washer from the base.
- 2 Place the column into the base.
- 3 Slide the lock washer and then the flat washer onto the bolt.
- 4 Insert the bolt into the bottom of the base.
- 5 Securely fasten the bolt using a 1/2 " wrench.

Adjusting Basket

Adjust the basket low enough to allow room for the printer bracket above it.

Adjust Basket

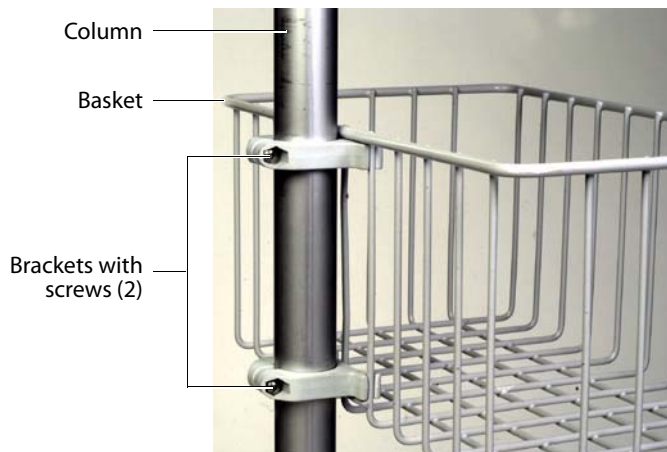- 1 Loosen the two screws on the two brackets using a #1 Phillips screwdriver.
- 2 Rotate the basket 90 degrees to the left as viewed from the front of the stand.
- 3 Position the basket approximately 8 inches (20 cm) below the system sleeve.
- 4 Tighten the two screws on the two brackets to secure the basket to the column.

Attaching Printer Bracket

Attach the printer bracket close enough to the system for cables to reach from the system to the printer.

Attach Printer Bracket

- 1 Remove the two screws from the bracket using the 5/32" Allen wrench.
- 2 Position the printer bracket on the right side of the stand just above the basket.
Note: Position the printer bracket close enough to the sleeve so that the 1 foot (30 cm) cables will reach when connected.
- 3 Insert the two screws into the printer bracket.
- 4 Tighten the two screws that secure the printer bracket to the column.

Installing Printer on Printer Bracket

Locate the appropriate model number to determine the correct screw holes.

Attach Printer to Bracket

Note: The printer is an optional feature.


Note: The printer installation procedure is dependent on the printer model number.

Locate the appropriate printer model number and procedure.

UP-897MD

- 1 Place the printer on the printer bracket aligning the four holes on the printer with the four, inside holes on the bracket.
- 2 Insert the four, 10 mm screws through the printer bracket and into the printer.
- 3 Securely fasten the screws using a #1 Phillips screwdriver.
- 4 See the manufacturer's operating instructions for information on operating and maintaining the printer.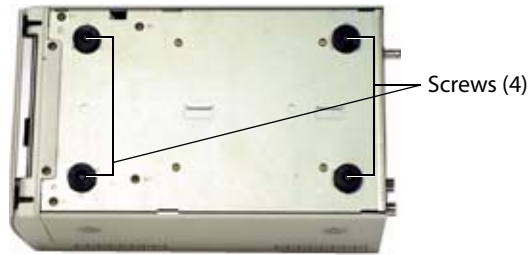

UP-890MD or UP-895MD

- 1 Remove the 4 screws from the bottom of the printer using a #1 Phillips screwdriver.
- 2 Discard these screws. Leave the feet on.
- 3 Place the printer on the printer bracket aligning the four holes on the printer with the four, inside holes on the bracket.
- 4 Insert the four, 16 mm screws through the printer bracket and into the printer.
- 5 Securely fasten the screws.
- 6 See the manufacturer's operating instructions for information on operating and maintaining the printer.

Connect Printer

Follow procedure to ensure correct cable connections between printer and system.

Provided Equipment

- Basic Stand
- Video cable (BNC/BNC) (1.0 ft/ 0.3 m) with BNC female to RCA male adapter or video cable (BNC/RCA) (1.0 ft/ 0.3 m)
- Printer control cable (1.0 ft / 0.3 m)

Required Equipment

- SonoSite 180® PLUS hand-carried ultrasound system or SonoHeart® ELITE
- System power supply
- Sony black and white printer: UP-897MD, UP-895MD, UP-890MD
- Printer AC power cord (ships with Sony printer)
- System power cord (ships with ultrasound system)

Connect Printer to System

Left side of system

Back of printer

- 1 Attach the cables:
 - Video cable (BNC/BNC) (**A**) from the system Video Out to the printer VIDEO IN
 - Printer control cable (**B**) from the system printer control out to the printer REMOTE
 - Power supply (**C**) from power supply output to the system power in
 - Printer AC power cord (**D**) from printer AC IN to a hospital grade outlet
 - System power cord from power supply AC input to hospital grade outlet
- 2 Ensure printer is set up on ultrasound system. See *SonoSite Ultrasound System User Guide*.

Install System

Slide the SonoSite system into the sleeve of the Basic Stand.

Insert System

- 1 Press on the release lever to open the sleeve.

- 2 Insert the transducer into the holder on the Basic Stand. Slide the system into the sleeve.

- 3 Latch the Basic Stand by using both hands to press on both sides of the sleeve; press until an audible click is heard.

Remove System

- 1 Hold the system by the handle.
- 2 Press on the release lever to open the sleeve.
- 3 Carefully remove the system by pulling it up and back.
- 4 Remove the transducer.

Lock and Release Wheels

- 1 Lock the wheels by pushing the locking levers down on the front wheels.
- 2 Release the wheels by pulling the locking levers up on the front wheels.

Clean and Disinfect Basic Stand

The exterior surface of the Basic Stand can be cleaned and disinfected using a recommended cleaner or disinfectant.

WARNING: To avoid infection always use protective eyewear and gloves when performing cleaning and disinfecting procedures.

To avoid infection, if a pre-mixed disinfection solution is used, observe the solution expiration date, and ensure that the date has not passed.

To avoid infection, the level of disinfection required for a product is dictated by the type of tissue it contacts during use. Ensure the solution strength and duration of contact are appropriate for the equipment. For information, see the disinfectant label instructions and the recommendations of the Association for Professionals in Infection Control and Epidemiology (APIC) and FDA.

Caution: Use only recommended cleaners or disinfectants on Basic Stand surfaces. Immersion-type disinfectants are not tested for use on Basic Stand surfaces.

-
- | | |
|----------------------------|---|
| Clean and Disinfect | <ol style="list-style-type: none">1 Remove the system and all contents from the Basic Stand.2 Use a soft cloth lightly dampened in a mild soap or detergent cleaning solution to clean the exterior surfaces.3 Mix the disinfection solution compatible with your system according to the label instructions for the solution strength. Theracide disinfectant is recommended.4 Wipe the surfaces with the disinfection solution, following disinfectant label instructions for solution strengths and disinfectant contact duration. Ensure the solution strength and duration of contact are appropriate for the intended clinical application.5 Air dry or towel dry with a clean cloth according to the instructions on the disinfectant label. |
|----------------------------|---|

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