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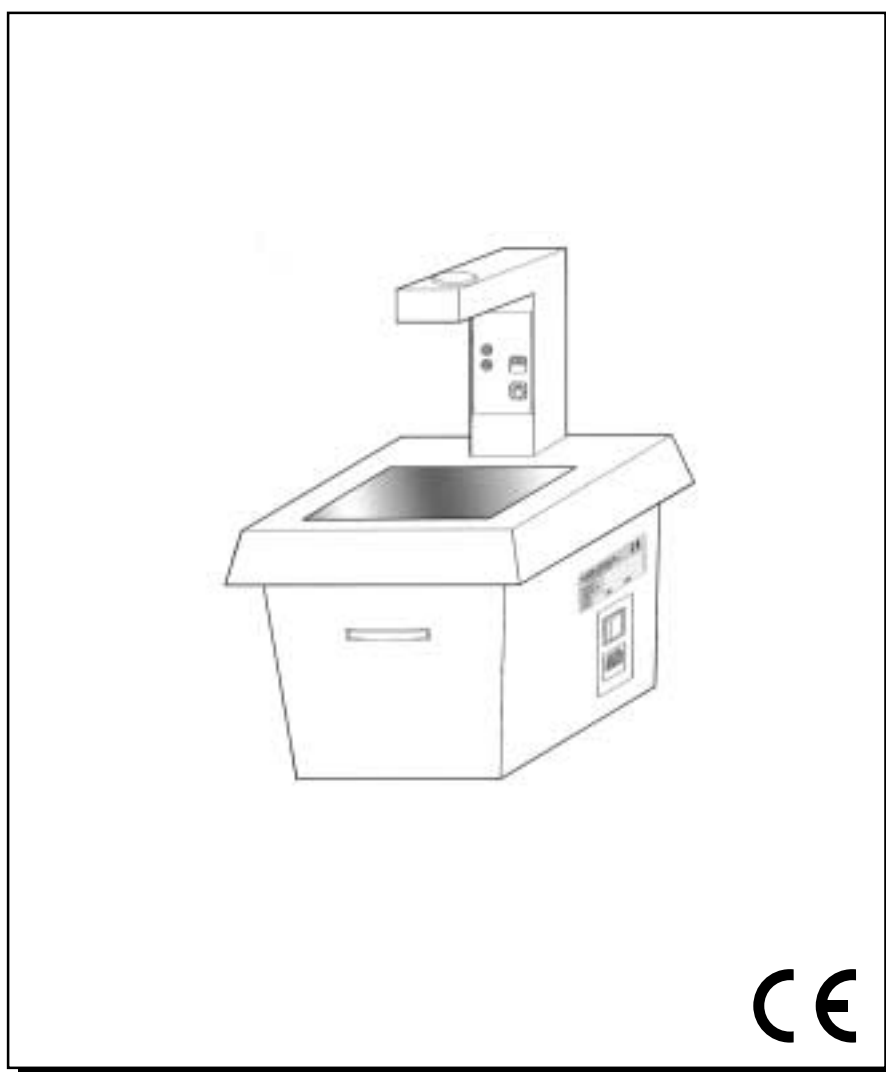
DAKO 6230

This manual has been written by the manufacturer and it is an integral part of the machine. Its information are addressed to the user and contain all Safety Recommendations. Before using this oven, it is recommended that you read

this manual carefully, especially the first time, to ensure you know the control devices, their function and position. It is also advisable to make use tests. This manual must be kept for future references.

GENERAL INFORMATION

Identification of the machine manufacturer

On the machine find a label as shown on Pic. 1.

pic. 1

Use

The **DAKO 6230** Drilling machine is capable of drilling plaster models or Plexiglass and PVC bases or other

plastic materials in dental laboratories and in conformity with the safety requirements. The machine is supplied with a Laser pointer system that allows a fast and safe alignment of the perforation..

Safety devices

The machine is protected by fixed guards to avoid access to the electrical and mechanical parts by the operator. Only specialized and authorized personell must have access to these parts for extraordinary maintainance and repairs.

Technical Data

Dimensions	cm25x26x40(LxPxH)
Weight	Kg 15
Power Supply	220 - 230 V
Power	150 W
Rotating Speed	2800 giri/min
Tip Diameter	3 mm
Pointer	Laser
Fuses	5A

MOVING AND INSTALLING

Packaging and unpackaging

The packaging is as follows:

- Polystyrene plates K10 protecting the machine on sides, top and bottom.
- Hard cardboard cover

Loading and unloading

This machine can be moved either by carts or manually.

While moving the machine avoid absolutely any kind of bumping, dropping or tilting: they could seriously damage it. Obviously, the manufacturer is not responsible for damages caused by droppings, improper use and maintenance which are not in strict accordance with the manufacturer's instructions contained in this manual.

Packaging waste disposal must be done with respect

to the environment and the law in force.

Installation instructions

The machine must be located on a safe place and on a horizontal position. Make sure there is sufficient air where you position the machine.

It is duty of the user to ensure that the electric plant is in accordance with the safety laws in force. It is particular important to make sure that the grounding is well efficient.

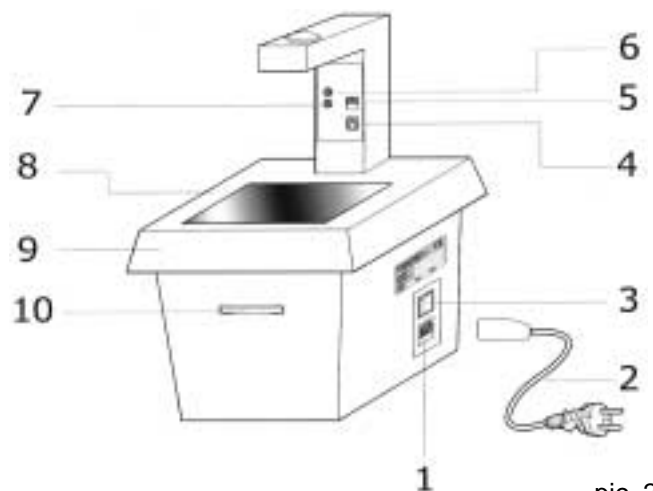
Furthermore, it is important to verify the network voltage: in case the voltage is too low (lower than 21 0V), the engine will start too slow and it might be necessary to install a voltage stabilizer. After having placed the machine and gone through the above due verifications, follow these instructions:

- The general switch must be in the "OFF" (0) position.
- Connect the power supply plug to a 220-230 V AC outlet, using the provided cable.

USE INSTRUCTIONS

Operating description (Pic.2)

- 1 - Cylinder holder with 5A fuses.
- 2 - Feeder cable 220 V AC
- 3 - General Switch: by switching it the machine will turn ON and the Display 8 will light up.
- 4 - Laser ON switch
- 5 - Button START/STOP: motor ignition.
- 6 - Ignition led: is lit when the general switch is on.
- 7 - Engine functioning led: it flashes when the engine is working.
- 8 - Holding-model tray.
- 9 - Moving working area: the drilling of the model is done by lowering the working surface with both hands.
- 10 - Regulating Nut: the perforation depth can be regulated by rotating the nut clearly located on the front of the machine.

pic. 2

ENGLISH

Performance and Use

- Turn ON the machine by using the general switch 3.
- Lay the model or the Plexiglass or the PVC base on the working surface next to the pointer delivery side.
- Push and let the surface 9 go all the way down and check the depth of the milling hole: regulate such depth by rotating the regulating nut 10. By rotating it to the right you'll increase the depth; by rotating it to the left you'll decrease it. Once you have performed this regulation, let the surface free and it will go back to its initial and high position.
- Turn On the motor by using the START!STOP 5 Switch.
- Turn On the Laser pointer by using the Switch 4 .
- Move the model on the surface until the Laser pointer identifies exactly the position of the perforation.
- By holding firmly the model with both hands, push the surface and let it go all the way down: the perforation is obtained in this way.
- Release the pressure on the surface and let it go back to its initial and high position. Proceed then with another perforation.

Useful recommendations

AWARE!!

When the motor is ON, absolutely avoid putting your hands and pushing with them on the working surface: it could cause injuries to your hands!!!

- Absolutely avoid regulating the perforation depth when the motor is moving.
- When you want to substitute the mill, do as follows: take out the holding-model tray 8, keep in place the spindle with a curved n. 10 key, loosen the bush with a n. 14 key, take out the old mill and replace it with a new one.
- To get rid of chips and dust due to the working process do as follows: take out the holding-model tray 8, suck in with a vacuum cleaner the chips and dust from the inner chamber.

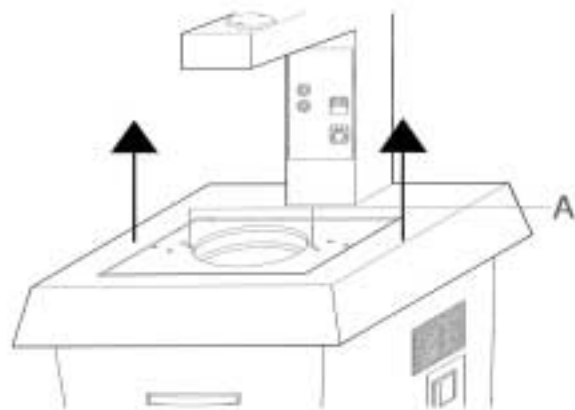
MAINTENANCE INFORMATION

Ordinary maintenance

It is advisable to keep the machine free from chips and dust: this easy maintenance operation allows better results and a better machine performance.

Never use any solvents for the cleaning off the machine, it is enough to vacuum the dust.

In order to better clean and suck the dust from the inside of the machine, please do as follows: take the moving working surface 9 apart. To do this (Pic.3) you need to: take the holding-model tray out, loosen of 3 complete turns the screws A, shown in the picture using the supplied key, lift the surface and let it come off its slides, move the surface letting it rotate around the neck of the machine. To put it back in place follow the same instructions backwards.

pic. 3

Extraordinary maintenance

To repair the machine or replace some of its parts, the user should consult directly the manufacturer or only specialized technicians.

Absolutely avoid opening any of the fixed guards without proper precautions.

Disconnect the power supply plug from the outlet before any extraordinary maintenance operations.

Precautions

Inside the machine there are components subjects to 220V network voltage.

Before performing any operation inside the machine

make sure you have turned off the general switch 3 and unplugged the power supply 2 (same precautions must be taken when replacing the fuses).

It is our most concern to point out what has been said in "Useful recommendation" the possibility of injuries to the hands exists due to the fact that the engine is always functioning during the working drilling phase. It is fundamental, therefore from the user's end, the maximum attention and the complete observations of the safety regulations.

If the user doesn't strictly follow the rules contained in this manual the manufacturer is lifted from any guarantee and responsibility.

WARRANTY

Certificate of guarantee (approved by UNIDI)

1. Through the present document the manufacturer certifies a correct construction of the product and assures that first materials have been used; all necessary approvals and trials have been made so that the product is conform to the regulations in force. The guarantee on this product is valid for a period of 12 months starting on the day of delivery to the purchaser. The purchaser approves the guarantee regulations by filling in and signing his part and returning it to the manufacturer. The guarantee only covers those parts to be substituted or replaced because of construction faults and doesn't cover labour expenses, travelling indemnity of technicians, transport expenses, packing etc. The guarantee excludes damages or failures caused through a bad maintenance, a wrong feeding supply, negligences, unskillfulness or other causes not imputable to the manufacturer. The guarantee excludes as well failures caused through ommised maintenance due to the negligences of the user. The guarantee doesn't include any kind of refund versus persons or objects for damages, direct or indirect, due to eventual inefficiency of the equipment.
2. The guarantee forfeits automatically in case that the equipment has been repaired, modified or tampered by the purchaser himself or by unauthorized third parties.
3. For interventions under guarantee, the purchaser must address himself to the selling-agent, or the by him indicated technical-service or direct to the manufacturer. The guarantee gives the right to substitution of the defective part, but excludes the substitution of the entire equipment.
4. In case of contestation concerning the application of the guarantee regulations about the quality or conditions of the delivered equipments, the purchaser will in no way suspend or delay the payment of the price or the rates of price.
5. No refund can be requested by the purchases for the temporary stop of the equipment.
6. The guarantee forfeits if:
 - a) the equipment presents damages of dropping; exposure to flames; overturning of liquid; natural disasters; or any other causes not imputable to fabrication defaults;
 - b) the installation wasn't correct;
 - c) the connection to the main supply was wrong rated feeding voltage;
 - d) the serial no has been taken of, cancelled or altered.
7. The components, to be substituted under guarantee must be sent back to the manufacturer for reimbursement following carefully his shipment instructions. For no reason we will accept the returning of equipments or spare parts of them without having given written authorization before. In case of non observation of these conditions the manufacturer reserves himself the right to send the goods back to the sender.
8. **Due to tax reasons the spare parts will be granted under guarantee only if we have received back the guarantee coupon properly filled in with all customer's data.**

GUARANTEE REGULATIONS

1. The non restitution of the guarantee certificate involves the immediate withdrawal of the certificate itself.

The guarantee doesn't cover labour expenses and travelling indemnity which will always be on charge of the purchaser.


2. The payment of the labour expenses, travelling indemnity, and right of appeal bill will be have to fullfilled immediate at the presentation of the bill itself. Not fullfilling the payment of the bill involves immediate withdrawl of the guarantee itself.
3. Neither the manufacturer or the dental deposit are obliged to provide substitutive equipment during the repair period.
4. For any other cases not foreseen in the present guarantee certificate and regulations reference is made to the italian civil code.

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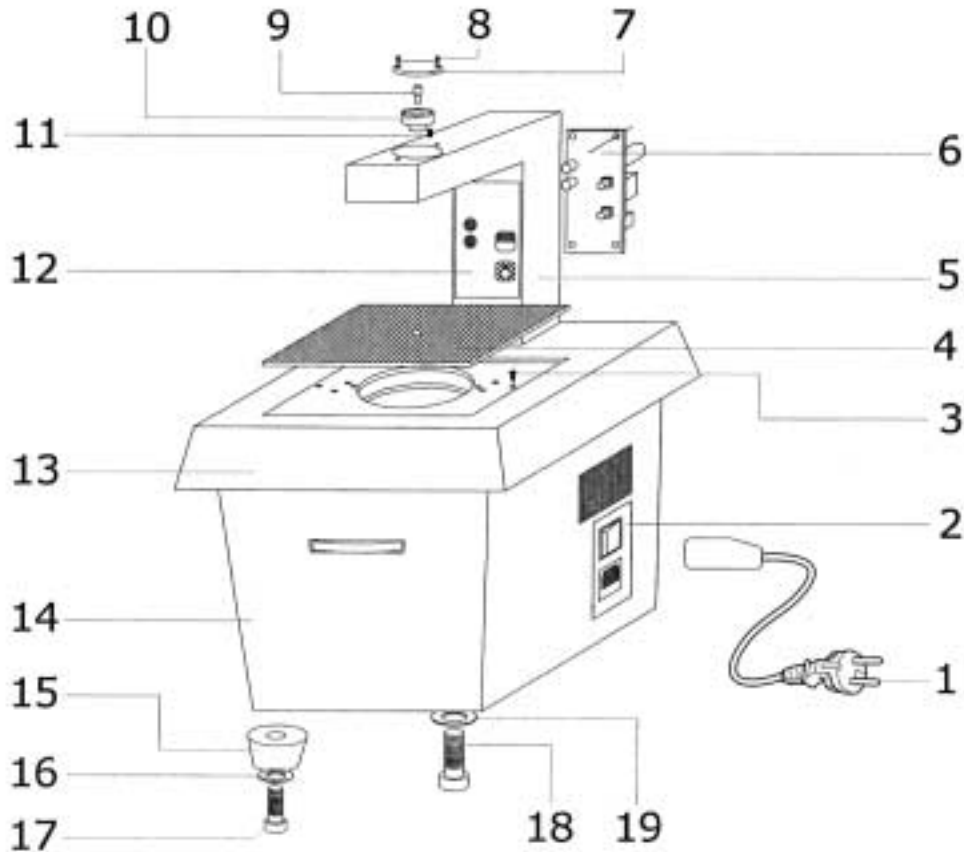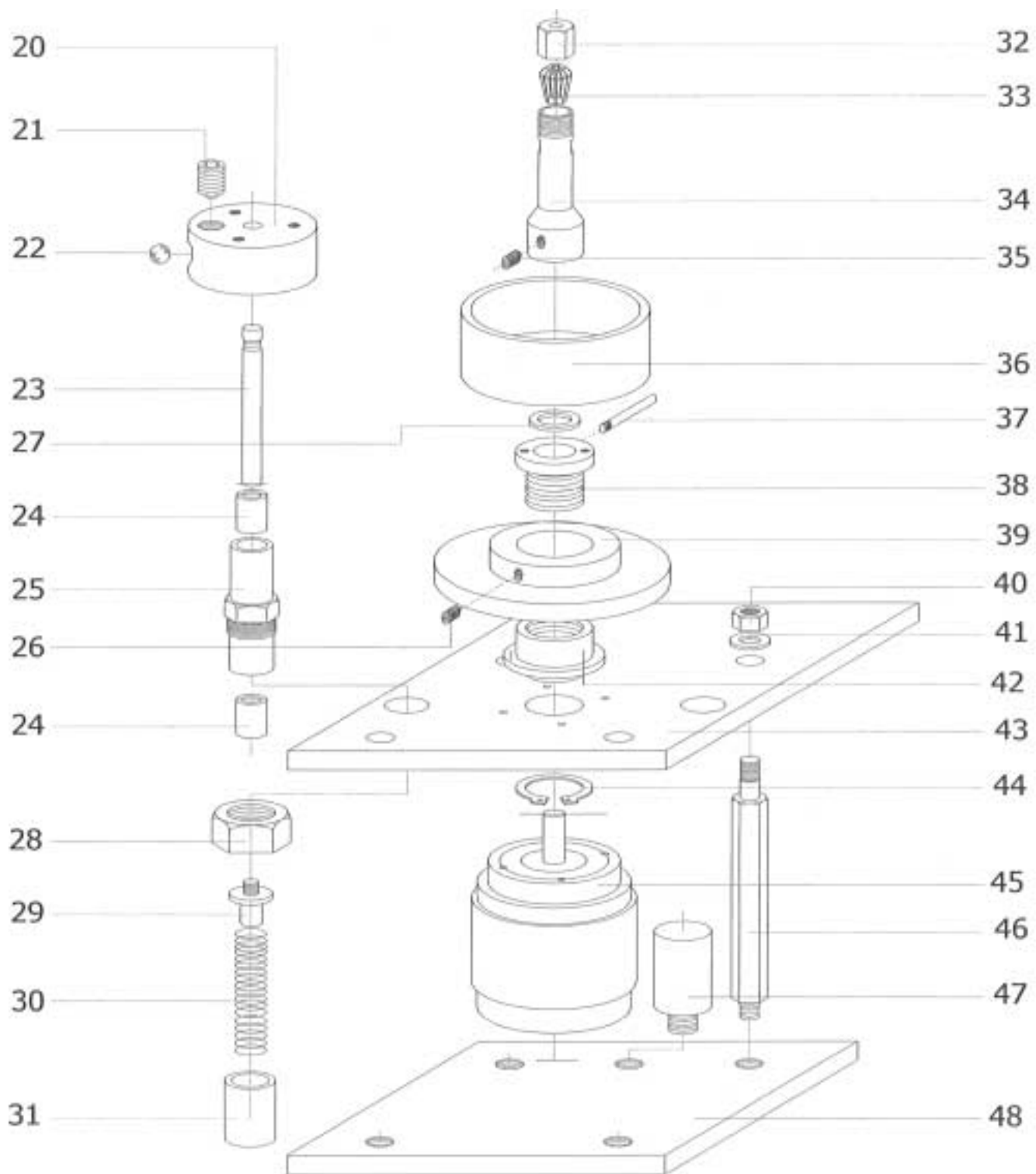
Direttive: **89/392/CEE - 89/336/CEE - 73/23/CEE**
Directives: **E.E.C.89/392 – E.E.C.89/336 – E.E.C.73/23**

20/05/02

Addi - Date

Bruno Bortolus

Amministratore Delegato – Managing Director

SARATOGA S.P.A.

Manufacturing of dental surgeries and laboratories equipments

via Malignani, 14 - 33107 Pordenone - ITALY
Tel. +39.0434.572600 r.a. Fax +39.0434.572477
[http: www.saratoga-on-line.com](http://www.saratoga-on-line.com) e-mail: info@saratoga-on-line.com

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Nome del fabbricante: **SARATOGA S.p.A.**