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Torino - Turin, 28/04/2011

Amm. Delegato - Managing Director
Alberto CATELLA

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SOLAR LED 3x is a precision instrument which has been manufactured in compliance with the highest optical and mechanical features in order to ideally satisfy the daily requirements of the Dental Laboratory.

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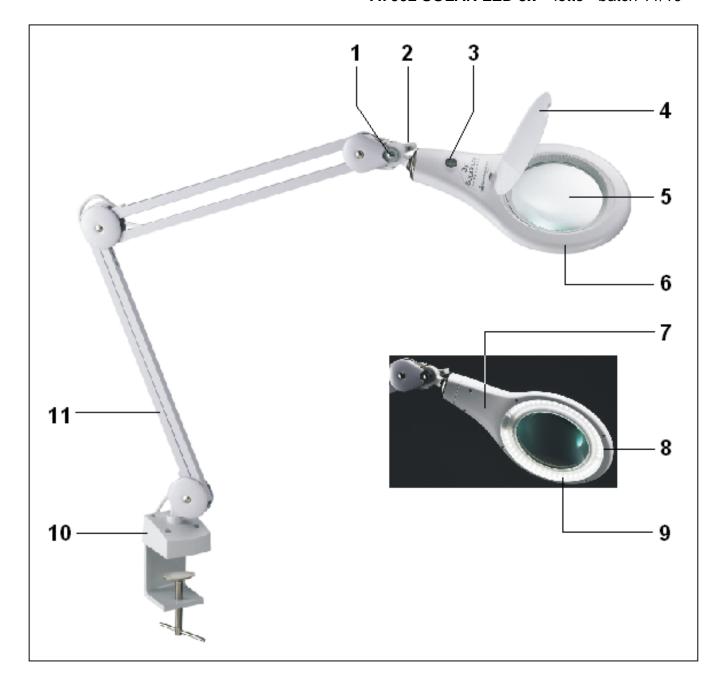
#### EXPLODED DRAWING AND SPARE PART LIST SOLAR 3x LED LAMP

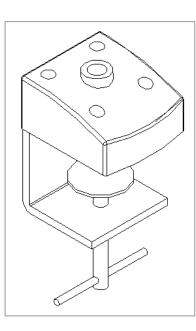
POS.	CODE	DESCRIPTION
1	7002024	ARTICULATION PIVOT
2	7002014	ARTICULATION KNOB
3	7002023	SWITCH
4	7002013	LENS PROTECTION CAP
5	7002019	3X MAGNIFYING LENS
6	7002011	UPPER PLASTIC BODY
7	7002012	LOWER PLASTIC BODY
8	7002015	LAMP PROTECTION SHIELD
9	7002021	COMPLETE LED RING
10	7002005	BENCH TOP – SUPPORT
11	7002010	EXTENSIBLE ARM
12	7002022	ELECTRONIC BALLAST
13	7001007	WALL-MOUNTED SUPPORT (OPTIONAL)
14	7001006	BASE (OPTIONAL)

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
Gently remove the lamp from its packaging and take out the accessories. The appliance is supplied with the support for bench fixing.

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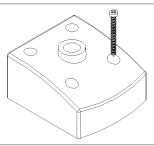Bench-top: whenever it is possible, place the clamp beside the bench, tighten firmly, then insert the rotating pin assembled on the lower end of the arm. Should you prefer to fix the unit on the work bench, unloose the 4 screws connecting the upper part to the metallic clamp, mark the positions for the holes and clamp the plastic block.

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As ideal place for the installation, choose a well ventilated, clean and dry location, with temperature ranging between 0°C and 40°C and relative humidity not exceeding 90% (indoor use only).

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Before connecting the electrical cable, be sure that the rated voltage stated on the label corresponds to the mains voltage and that the socket is duly supplied with earthed wire. The section of the feeding cable supplied with the unit suits to the highest absorbed current (specified in the technical features) and it is one meter long. Should you use any extension, be sure that the section of the cable matches to its length and that it is at least of the same size as the original cable.

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The roof lamp is fitted with a protective cover ensuring that the lens is kept clean; the cover must be closed in idle mode.

Switch on and true the lamp on the object to be enlarged, then adjust the focusing by gently lifting or lowering softly the lamp. The extensible arm of the lamp is fitted with articulated joints which can be locked by means of knobs in order to fix in place a certain position. Never force the mechanisms and do not try to force the lamp in extreme positions. Do not forget to switch off the lamp at the end of the work.

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Use a soft cloth to clean the lens, eventually use a liquid, antistatic and non-abrasive cleansing agent. The other parts of the lamp can be cleaned with any non-aggressive detergent or disinfectant agent specific for the plastic and the enameled metallic surfaces. Operations to be avoided: to clean the lens surface with your hands, to misuse and apply unnecessary loads to the extensible arm, to clean the appliance with volatile solvents or abrasive cleansing agents, to try to recondition the lamp by yourselves.

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**Replacement of the lens** – gently rotate the knurled ring which is fixing the lens on the upper plastic body and push up from the lower side.

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**Replacement of the complete LED ring** – before any intervention inside the roof lamp, disconnect the unit. Remove the 6 screws fixing the two parts of the plastic body, remove the transparent protection and disconnect the LED circuit from the electronic ballast.

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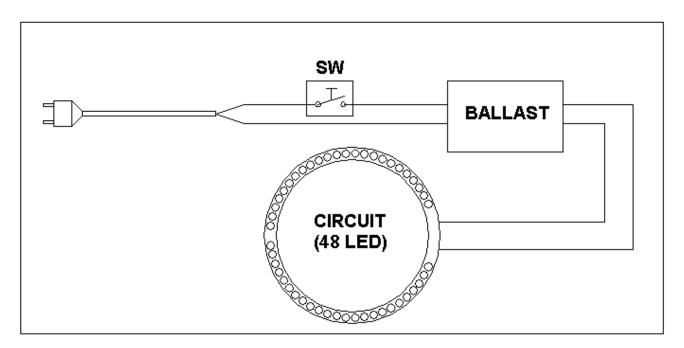
Never use the unit for any use differing from the operating purposes it has been designed for and which are clearly illustrated in the present instruction manual.

In case of demolition, comply scrupulously with the regulations in force in the Country where the operation takes place, always considering that you are handling an electrical device.

C.I.E. Dentalfarm Srl reserves the right to modify at any time and without prior notice any features of its products.

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