# GIMA

## GIMA-QUICK 240

**MANUALE D'USO E MANUTENZIONE**  
**USE AND MAINTENANCE BOOK**  
**INSTRUCTIONS DE FONCTIONNEMENT ET ENTRETIEN**  
**MANUAL DE USO Y MANTENIMIENTO**  
**MANUAL DE USO E MANUTENÇÃO**  
**ΕΓΧΕΙΡΙΔΙΟ ΧΡΗΣΗΣ ΚΑΙ ΣΥΝΤΗΡΗΣΗΣ**

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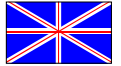
## CLASSIFICATION

Sterilizer: “Gima”  
 Model: **Gima Quick**  
 Manufacturing year:

## SYMBOLS EXPLANATION

I = main power switch on  
 0 = main power switch off

Dangerous temperature surface indication

Grounding: 

## TECHNICAL DATA

Absorbed power	150 Watt
Voltage / Frequency	230 Volt - 50 Hertz
Fuse	1 A
Operating temperature	235° ~
Use	Files, needles, probes, mirrors, paper tips, phial tubes, curettes, burs, clamps, handpieces, contra-angles, etc.
Main Switch	Green light
Overall size (w.d.h.)	cm. 13 x 19,5 x 14 approx.
Weight	Kg. 2,8 approx.
Glassbeads packet	Gr. 220 (Each sterilizer is provided with a glassbeads packet)

## DESCRIPTION

The sterilizer is designed for a quickly and safely sterilization of the surgical tools' tip parts. The operating temperature 230°C ensures the elimination of any micro-organisms in few seconds. The sterilizer is equipped with:

- armoured resistance 150 Watt
- main switch with green light
- lamps to control the correct working of the equipment
- automatic thermostat to maintain a constant temperature
- protection fuse 1A with removable box
- removable power cable with safety plug.

The safety of the equipment is granted by a planning and manufacturing in accordance with the rules in force (i.e.: double thermic insulation, a very accurate electric insulation, protection fuse, power cable with safety plug, insulation externally protected by a stainless steel plate, armoured resistance, air circulation around five sides, switch and lamps with safety marks, etc.).

The quartz balls container is extractable and is made with nickel copper. It has an edge to avoid loss of glassbeads. Quickly cleaning and quartz balls recharged.

## INSTRUCTIONS

### Carriage

The sterilizer is shipped in an adequate package that requires the common handling cautions for fragile materials, as shown on the package itself. Use only the original package if the equipment must be returned to the manufacturer. To carry the sterilizer without package, keep it in a horizontal position and use the base as holding point.

### Installation - start up

Remove the equipment from the package handling with care. Check the power supply system is correctly grounded. Place the sterilizer on a level surface so that air can freely circulate under the equipment bottom side. Keep the back side of the sterilizer at least 10cm. away from walls or others appliances to ensure an adequate ventilation. Do not place the sterilizer near heat sources that could influence negatively the thermostat. Full the container with glassbeads, and pay attention to pull them 5-6mm. below the edge, avoiding in this way their discharge. Insert the container into its housing.

Insert the safety plug of the power supply cable, firstly into the equipment socket and then into the electric system. Press switch position "I". Check if the lamp switch and orange lamp are light up.

Note: a typical smell of heated ceramic fibre may be present at the first power on and remain for a short period of time.

### Instructions to use

Press the power switch on "I".

Keep the upper cover closed to avoid any loss of heat, till the equipment is working. The equipment will take up 30 minutes to reach its operating temperature (230°C in the container).

When the sterilizer is operating, the green and orange lamps alternatively light up: that means temperature is between 230°C and 235°C. approx.

When the working temperature is reached, the surgical tools can be sterilized by introducing their tips, in accordance with the required depth, into the quartz balls.

On the front panel you can find the sterilization time (see Table "A" too). Tools of small dimensions, particular clamps have to be used to avoid possible burns.

Before the sterilization, surgical tools must be cleaned and dried.

Thanks to the special materials used, the design characteristics and the air circulation system on five sides of the equipment, allow the sterilizer to operate all working day. However for energy saving, it is recommended to keep the cover closed when the equipment is ready but it is not working. A periodical control of the sterilization process is also recommended, using the sterilization indicator strips or similar products on the market.

**TABLE “A”**

**Recommended sterilization times**

Files, probes, mirrors, phial tubes	5 sec.
Cotton, paper tips, surgical tools	5-10 sec.
Curettes, clamps, burs, handpieces, contra-angles	10-15 sec.

**Maintenance and repair**

Always check the power is unplugged before maintenance is carried out. The glassbeads must be periodically replaced, when they lost their transparency. To replace the glassbeads, the container must be pulled out when the sterilizer is completely cold. The sterilizer needs no particular maintenance, provided the adequate environment are satisfied, except for a regular cleaning of its external parts. It is recommended to clean the equipment with a soft cloth and wate or mild detergents. Regularly check if the power cable is in good conditions. It is possible to replace the safety fuse pulling out the fuse block at the rear. Please always follow the characteristics stated in the technical data sheet. If the sterilizer doesn't work properly, contact the manufacturer or the authorized service. If any repairs are needed, please return the equipment directly to the manufacturer or to the local distributor using only the original package. However it is possible to contact specialized technicians, who must work in accordance to general safety and accident prevention rules of the country in which the equipment is installed.

**Out of order**

No particular cautions.

**SAFETY CAUTIONS**

This equipment must be used only by authorized personnel thoroughly familiar with the correct use.

The sterilizer must be used only for its designed purpose:

*to sterilizer surgical instruments or tools in the medical and paramedical field. The sterilizer must not be used for all treatments concerning contact lenses (Rule 15 - All.9 - Comma 1).*

It is necessary to take all precautions to avoid unauthorized use. Do not modify, change or put out of order any electrical or thermal safety protection of the equipment.

When the sterilizer is operating, pay attention to the people near the equipment.

The user must not attempt to carry out any repair unless authorized.

Do not clean or repair the sterilizer while it is connected to an electrical system and be sure the container is at room temperature. Regularly check if the power cable is damaged.

When authorized technicians decide to replace spare parts, they must use original replacements and moreover they must adhere to the manufacturer's information and technical characteristics. In case of any doubt about safety rules, please contact the manufacturer or local distributor before using the equipment.

However use the sterilizer in accordance with the general safety and accident prevention rules of the country in which the equipment is installed.

**ATTENTION !**

*Use only glassbeads to fill the container. Avoid any accidental contacts with fluids. Avoid to use the sterilizer in an environment where the explosive gases may be present.*

**CAUTIONS TO THE USER**

*Place the sterilizer on a level surface so that air can freely circulate under the equipment bottom side.*

*Keep the sterilizer at least 10cm. away from walls or others appliances to ensure an adequate ventilation. Do not place the sterilizer near heat sources that could influence negatively the thermostat.*

*Keep the door closed when the sterilization process is running.*

*Do not place any object on the upper part of the equipment.*

*Use adequate clamps for small tools to sterilize.*

*Do not clean, move or repair the sterilizer while it is connected to an electrical source and when it is still hot.*

**ATTENTION !**

*When the sterilizer is operating do not put hands near it : glassbeads' temperature is 230°C.*

**MANUFACTURER'S RESPONSIBILITY**

The sterilizer must be used only for its designed purpose.

Improper use of the equipment is to the user's own risk and sole responsibility.

**GUARANTEE**

Thank you for having acquired our product.

This product meets the most stringent requirements regarding the selection of quality manufacturing materials and also the final control. The product has a 12-month warranty, valid from the date it is delivered by GIMA.

During the guarantee period free repair and/or replacement of any defective parts due to faulty manufacture will be given, labour, postal, transport and packaging charges etc. are not included. The guarantee therefore excludes components subject to wear and tear such as parts in rubber or PVC, doppler probes, SpO<sub>2</sub> sensors, lamps, batteries, electrodes, handpieces, washers and packings, resistances, etc.

No compensation will be given for loss of use of the product.

Furthermore replacement or repairs effected during the guarantee period do not lengthen the duration of the guarantee.

*This guarantee is void in the event of: repairs effected by unauthorised persons or with spare parts not approved by GIMA, damage or defects caused by negligence, blows, abnormal use of the appliance or faulty installation.*

The guarantee will be invalidated if the registration number has been removed, cancelled or altered.

Faulty appliances must be returned only to the retailer where the item was bought. Any item dispatched directly to us will be refused.

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