**Zentrales Saugsystem**

**EXCOM hybrid 1s**

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DE

**Central Suction System**

**EXCOM hybrid 1s**

Equipment Logbook

Assembly, operation and maintenance

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## Information on guarantee claims

EN

METASYS delivers exclusively on the basis of the present terms and conditions, insofar as contractual partners have not struck a deviating agreement in writing.

1. Guarantee claims on our products are valid for a period from 12 to 36 months (depending on the product).
2. This guarantee includes all claims regarding material failures that affect the function of the device. Damages caused by wrong or improper handling or normal abrasion and the exchange of the collection amalgam container are excluded from the guarantee claim. Compulsory maintenance rates have to be met precisely. The producer reserves to himself the right to request the corresponding product documents to ascertain service rates.
3. To assert the validity of the guarantee claim it is necessary to return the attached installation proof to the producer right after the installation. Installation and return of installation proof must be completed within 24 months after date of sale from METASYS.
4. Both the installation and the required inspections have to be carried out by a technician trained by METASYS and have to be registered in the METASYS equipment logbook.
5. We point out that guarantee claims for amalgam separators, air-water separators and central suction systems can only be asserted if the device has been treated with the suction system disinfectant METASYS GREEN&CLEAN M2 according to the directions of use. In case of water decontamination system WEK, the unexceptional use of the water decontamination preparation GREEN&CLEAN WK is essential for the validity of a guarantee claim. This regulation does also apply to the hygiene system H1, which must exclusively be operated with GREEN&CLEAN H1. Implied warranty remains unaffected of this.

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## 1. Index

| Chapter                                    | Page |
|--------------------------------------------|------|
| 1. Index                                   | 10   |
| 2. Explanation of the pictograms           | 10   |
| 3. General information                     | 10   |
| 4. Application                             | 10   |
| 5. Construction                            | 10   |
| 6. Explanation of the type plate           | 10   |
| 7. Technical data                          | 11   |
| 8. Functional description                  | 11   |
| 9. Installation guidelines                 | 11   |
| 10. Hose connections                       | 11   |
| 11. Electrical connections                 | 12   |
| 12. Maintenance, cleaning and disinfection | 12   |
| 13. Commissioning                          | 12   |
| 14. Servicing                              | 13   |
| 15. Disposal                               | 13   |

## 2. Explanation of the pictograms

**Caution!**

## 3. General information

**The safety, reliability and performance of the appliance is only guaranteed by METASYS if the following instructions are observed:**

- Assembly, alterations or repairs may exclusively be carried out by authorized service personnel in compliance with EN Standard 60601-1 (International Standard for Medical Electrical Apparatus, in particular Part 1: General Rules for Safety).
- The electrical installation must comply with the regulations of the IEC (International Commission for Electrical Engineering).
- The apparatus must exclusively be used in conformity with the instructions for installation, operation and maintenance.
- Only original parts may be used for repairs or replacements.
- All the guidelines provided by the manufacturer on the correct use of the apparatus are followed.
- Following the commissioning of the apparatus, the Installation Proof must be completed and returned to METASYS to establish the guarantee period.
- When requested by an authorized engineer, METASYS agrees to make all documents available for the use of technically qualified service personnel.

- METASYS accepts no responsibility for damages caused due to external factors, such as wrong installation, improper use of the apparatus or unauthorized technical intervention.
- Users must study equipment and assure themselves of its good condition before every use.

**The equipment is not suitable for use in an explosive or combustible environment.**

## 4. Application

**1** EXCOM hybrid 1s is a central suction system with integrated air/water separation for simultaneous operation of 1 treatment unit or for up to 2 treatment units with 60% efficiency. It is used for centralised vacuum production, as well as the separation of air and liquids in dentistry.

EXCOM hybrid 1s is a vacuum engine with integrated separation. It can be operated both as semi-wet or dry suction system.

## 5. Construction **2**

### Picture **2.1**: Suction engine with dynamic separation unit

The suction engine is a powerful dry vacuum engine operating according to the principle of the side channel vacuum pump.

The dynamic separation unit centrally separates liquids and solids from the air stream without interruption of the suction's output. This eliminates the need for a separator in the treatment unit.

### Picture **2.2**: Control unit (optional)

The control unit contains all electrical components necessary to control and monitor the entire device.

### Picture **2.3**: Prefilter

Coarse solid particles are held back in the prefilter.

### Picture **2.4**: Hose connections

### Picture **2.5**: Water collector

The water collector protects the suction pump from water backflow and transports the water to the outlet, if necessary.

## 6. Explanation of the type plate **3**

**3.1** Equipment type

**3.2** Mains supply data

**3.3** Serial number

- 3.4** Address of the manufacturer
- 3.5** CE mark of conformity
- 3.6** Separate collection electrical/electronic equipment
- 3.7** Protection class I

## 7. Technical data

|                             |                 |
|-----------------------------|-----------------|
| Power supply                | 230 V AC        |
| Frequency                   | 50/60 Hz        |
| Max. current consumption    | 3,3 / 4,1 A     |
| Max. electrical shaft power | 0,75 / 0,9 kW   |
| Max. ambient temperature    | 40° C           |
| Air flow rate               | 1100 l/min      |
| Negative pressure regulated | 120 mbar        |
| Operating time              | 100%            |
| Weight                      | 15 kg           |
| Noise level                 | 63 dB(A)        |
| Dimensions (L x W x H) mm   | 530 x 320 x 350 |

## 8. Functional description

**Picture 4**: On removing a suction hose from its rest in the treatment unit, the dynamic separation **4.1** and the EXCOM hybrid 1s central system suction engine **4.2** start.

The mixture of liquids, solids and air sucked from the treatment units flows through the suction connection **4.3** and prefilter **4.4** into the separation chamber **4.1**. The mixture is accelerated into a circular movement by the rapidly rotating impeller blades. The liquids and solids are tangentially centrifuged, whilst the air flows through the blade shafts into the hose connection and into the suction engine **4.2**. The dry air is discharged via the optional bio filter into the atmosphere through the exhaust air connection. The factory-made follow-up time of the dynamic air / water separation and of the suction motor is approx. 60 seconds. This can be extended according to the installation situation.

The centrifuged liquids and solid particles smaller than the mesh of the prefilters **4.4** are either led into the normal sewage system via the water outlet **4.5** and the drainage connection.

## 9. Installation guidelines

The EXCOM hybrid 1s suction system is designed to be installed only in dry, adequately ventilated rooms.

**Its use in areas subject to explosive and fire hazards is not permitted!**

The permitted room temperature ranges from between +10° C and +40° C. The relative humidity must not exceed 70%. In case of a room temperature of more than +35° C, a fan must be installed for additional ventilation.

Installation can be on the same level as the treatment units, in a side room or one floor lower.

In order to avoid vibrations, the suction system must be installed on a firm base.

When the EXCOM hybrid 1s suction system is installed, the connection side must be placed at least 15 cm from the wall so that the hoses can be connected.

The front of the device must be easily accessible. If the EXCOM hybrid 1s is installed with the covering hood, nothing must be placed on top of it. To allow the removal of the covering hood, a free space above equal to the equipment's height equal to half its width at the sides is required. There must be clear space of approx. 5 cm around the device to guarantee adequate air circulation.

**Do not lift the device at separation!**  
**While the suction is used, the device must not be switched off at the main switch!**

## Picture 5: Pipe and hose installation

**Any pipe or hose used must be vacuum tight and resistant to all chemicals normally used in a dental practice (e.g. HT discharge pipes made from PP, PVC-C, PVC-U, PE-HD).**

Connections to the EXCOM hybrid 1s central suction system must be made by flexible hoses and be as short as possible.

To avoid a loss of suction power, a pipe diameter of 40 mm is recommended.

Avoid right-angle bends in order not to lose suction power (recommendation: 2 x 45° degree bends).

Discharge pipes must meet applicable local legislation or DIN 1986, Parts 1 and 2.

Waste water must be allowed to drain off freely without any backup. Waste water pipes must have a hydraulic gradient of at least 2%.

The air must be discharged out-of-doors. For reasons of hygiene and in order to avoid noise pollution we recommend that the outgoing air connection is fitted with a bio filter.

The diameter of the discharged air connection must be equal to or bigger than the diameter of the suction connection.

## 10. Hose connections

### Picture 6

- 6.1** Connection for the suction hose (from the treatment units):  
Ø 40 mm

- 6.2** Connection for exhaust air: für Abluft: Ø 40 mm
- 6.3** Connection for waste water (clean water discharge): Ø 15 mm

All hose connections must be secured with hose clamps!

For the exhaust air connections only heat-resistant ( $\geq 130^{\circ}\text{C}$ ) hose and pipe material must be used.

In case of water discharge at the water collector all connections, especially the water discharge pipe, must be checked.

## 11. Electrical connections

### Mains connection

The mains connection must only be carried out by a trained electrician. The electrical installation must be carried out in accordance with applicable local regulations.

Before connecting with the mains, the nominal voltage stated on the type plate on the equipment must be compared with the mains voltage. The EXCOM hybrid 1s suction system must only be connected to the power supply with the supplied power cable. Extension cables must not be used.

### Main switch

Connection to the mains must be established after the practise's main switch.

### Picture 7: Wiring diagram

- A1** control board
- K1** motor contactor
- M1** motor for vac
- Q1** protection switch

### Hose rest signal

The control cable for the hose rest signal light is already connected internally by a 2 pole cable, 3 metres long.

The suction system starts when the two contacts are connected. The control cable is to be properly fixed into a junction box.

### Picture 8: Control box

- A1** control board
- Q1** protection switch
- W1** power supply
- W2** control cable - motor for vac
- W3** control cable - support signal

## 12. Maintenance, cleaning and disinfection

**Picture 9:** Shortly flush spittoon bowl after each treachment!

**Picture 10:** Suck some water through all suction hoses after every treatment.

**Picture 11:** Twice a day, after having sucked off some water, use either the prescribed disinfectant for suction systems GREEN&CLEAN M2 for cleaning and disinfecting, or the prescribed cleanser for suction systems GREEN&CLEAN CL.

**Picture 12:** The spittoon bowl should also be rinsed with GREEN&CLEAN M2 or GREEN&CLEAN CL twice a day.

**Cleaning the prefilter:** The prefilter must be emptied and cleaned at least once a week. This may also be carried out daily, depending on workload.

**Picture 13:** The amalgam residue from the prefilter box must be collected in the METASYS ECOCENTER and disposed of properly with ECOTRANSFORM.

Visit  
[www.metasys.com](http://www.metasys.com)  
or scan this QR code  
to receive more information  
about ECOTRANSFORM!

Please go to  
[www.metasys.com](http://www.metasys.com)  
to locate your national collection point.

## 13. Commissioning

**Picture 14:** Switch on practice and the equipment main switch.

**Picture 15:** Remove the suction hose from its holder. Check that all hose connections and other connections in the suction pipe are airtight.

**Picture 16:** Suck 600 ml of water and check that the EXCOM hybrid 1s central suction system is operating correctly.

Perform electrical safety checks as required by local legislation, and record that the checks have been made.

## 14. Servicing

**Picture 17.1:** The following filters must be checked and cleaned every week:

- filter in the hose rest or the suction hose (not shown)
- filter in spittoon bowl outlet and the spittoon valve
- base filter **17.1**

### Cleaning the prefilter:

The prefilters must be cleaned at least once a week. However, depending on the method of working, this may be necessary every day.

A clogged prefilter is perceivable by a reduction of suction power.

### Exhaust air filter:

The optional exhaust air bio filter must be replaced at least once a year.

## 15. Disposal

The devices may be contaminated. Please inform the disposal company of this so that the necessary precautions can be taken. Disposal of amalgam separator components, such as sieves, filters, hoses etc., must also comply with local regulations.

Uncontaminated plastic components of the suction system may be recycled.

The built-in control unit, electronic circuit boards and components may be disposed of as electro-technical scrap. Other metal components may be disposed of as ordinary metal scrap.

If the device is returned, for example to the dealer or METASYS Medizintechnik GmbH, all connections must be sealed so that they are watertight.

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