
seca 676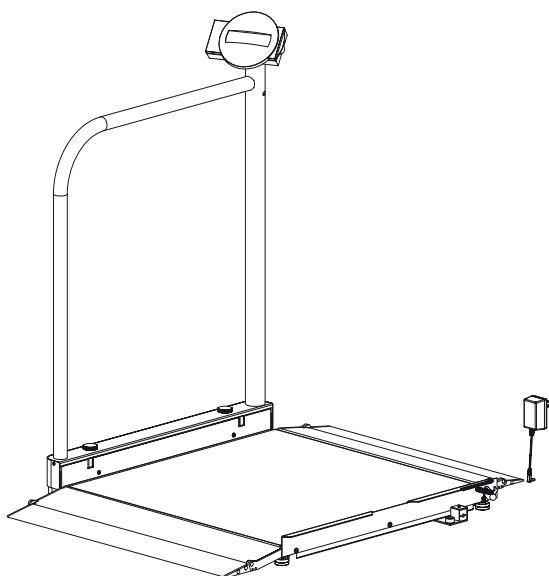

D	<i>Bedienungsanleitung und Garantieerklärung</i>	2
GB	<i>Instruction manual and guarantee</i>	16
F	<i>Mode d'emploi et garantie</i>	30
I	<i>Manuale di istruzioni e garanzia</i>	44
E	<i>Manual de instrucciones y garantía</i>	58
P	<i>Instruções de utilização e declaração de garantia</i>	72

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1. Congratulations!

By purchasing the **seca 676** electronic platform scale you have acquired a highly accurate and sturdy piece of equipment.

seca has been putting its experience at the service of health for over 150 years now, and as market leader in many countries of the world, is always setting new standards with its innovative developments for weighing and measuring.

In accordance with national regulations, the **seca 676** electronic platform scale is used primarily in hospitals, doctors' surgeries and in-patient nursing facilities.

The weight display can be switched

between kilogrammes (kg), pounds (lbs) and stones (sts). Weight is determined within a few seconds.

You can use the pre-Tare function to record the weights of, e.g. three types of wheelchairs. This weight is automatically deducted in the weighing operation.

The two fold-away ramps allow the wheelchair-user to get on and off in comfort.

The scale is of a very sturdy construction and will give you long and loyal service. It is easy to operate and the large display is easy to read.

The **seca 676** can be moved on castors.

2. Safety

Before using the new scale, please take a little time to read the following safety instructions.

- Follow the safety instructions in the instruction manual.
- Keep the operating manual and the declaration of conformity contained therein in a safe place.
- Do not drop the scale or subject it to violent shocks.
- Place the scale carefully in its position for use.
- Avoid pulling hard on the connecting cable to the display unit.
- Secure the scale so that it does not fall over when stored in the upright position.
- When moving the scale in its upright position, take care that it does not fall over.
- Check that the screwed connection of the handrail is firmly tightened before releasing the scale for use by patients.
- Be aware of tripping and falling hazards for patients with disabilities and wheelchair-users and do your best to prevent them.
- Be sure to use only the seca mains unit supplied. Before plugging the mains unit into a socket, check that the mains voltage indicated on the mains unit matches local mains voltage.
- When using the scale with a mains unit, ensure that the supply cable is routed in such a way as to exclude any type of tripping hazard.
- Have scale serviced on a regular basis (see "Servicing" on page 27).
- Have repairs carried out exclusively by authorised persons.


3. Before you start...

Unpacking

Remove the packaging and place the scale on a firm, level surface.


The scope of supply includes:

- Weighing platform with folding handrail
- Display head plus connecting cable
- 3 Screws
- 1 seca mains unit
- 1 set of operating instructions plus declaration of conformity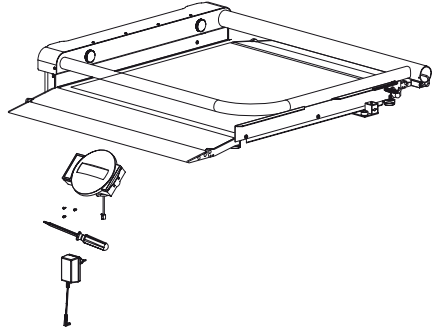


Fitting the display

- Carefully put down the scale.
- Release the latch and lift up the handrail. Retain it in this position so that it cannot drop back down.
- Screw up the two locking bolts. Ensure that the handrail is rigid and has no play.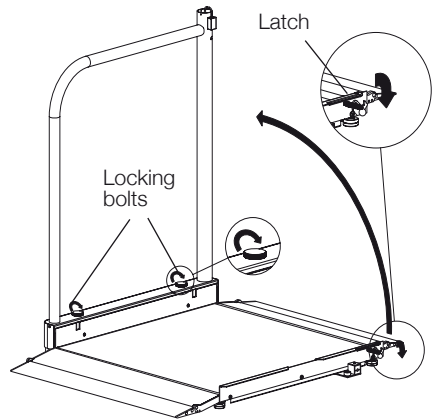


- Undo the mounting for the display cable on the handrail and connect it to the cable for the display.
- Push the cable carefully into the handrail and push the display onto the handrail. Ensure that the positions of the adhesive labels on the handrail and the display match.
- Check that the holes on the bottom of the display match the holes on the handrail and then screw the display to the handrail using the three screws.

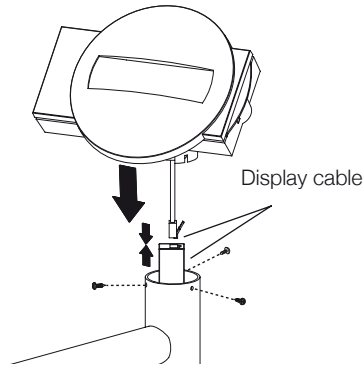Power supply


The scale is powered by a mains unit.

Please use only the mains unit supplied or mains units from the seca range of accessories. Before plugging the unit into the socket, check whether the mains voltage marked on the mains unit matches local mains voltage.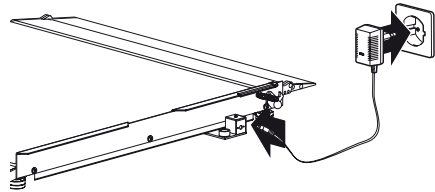

If your scale is equipped with seca mains power unit 68-32-10-263, please proceed as follows:


- The mains power unit can be adapted to a wide variety of socket systems using the adapters. Change the adapter of the mains power unit if required: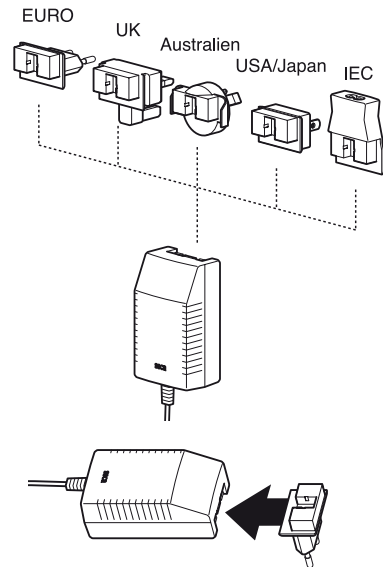

pull the adapter off the mains power unit from the side.


Then plug the correct adapter onto the mains power unit. Ensure that the adapter is properly engaged and securely located.

- Connect the mains power unit to the scale.
- Connect the mains power unit to the socket.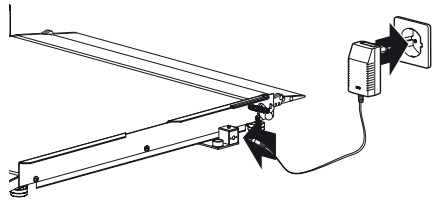


4. Setting up and aligning the scale

- Place the scale on a level surface.
- Set up the handrail as described in the previous section.
- Undo the locknuts using the four foot screws (1).
- Align the scale by undoing the foot screws. The air bubble in the spirit level must be right in the centre of the circle (2).
- Then re-tighten the locknuts (3).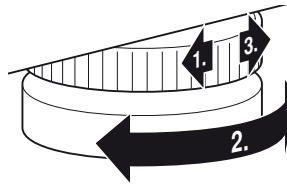

- Fold down the ramps required.
- Ensure that only the feet of the scale are in contact with the floor. The scale may not be in contact at any other point.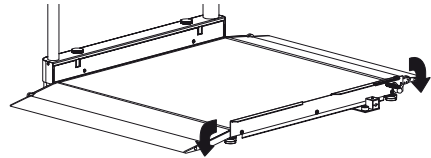


Important!

The alignment of the base of the scale must be checked and corrected if necessary every time the location of the scale is changed.

5. Operation

Controls and displays

The controls and display are in the display part.

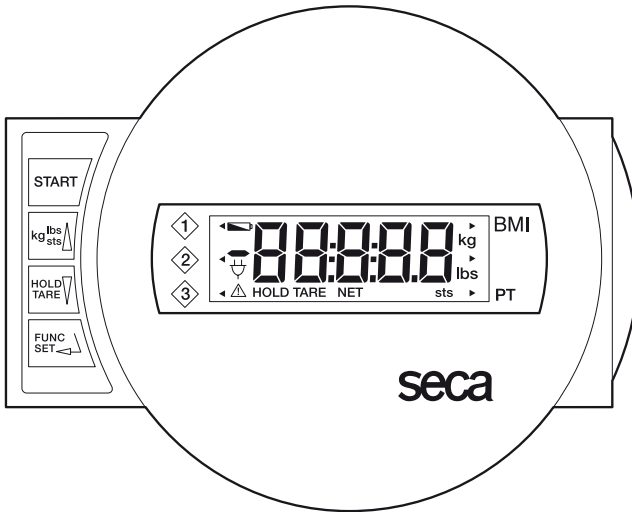Switch scale on and off

press briefly → switch between kg and lbs
hold down → switch between kg and sts
in function menu → increase value, scroll

press briefly → activate hold function
hold down → set display to zero, activate/deactivate tare function
in function menu → decrease value, scroll


Menu selection, Confirmation, Parameter adjustment

Memories

In order to ensure that the display can be read from different positions, the display head can be rotated 180°.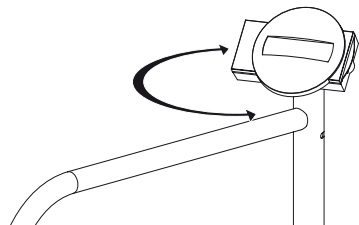

Correct weighing

- Press the Start key with no load on the scale.
SECR, **88888** and **00** appear consecutively in the display. The scale is then automatically set to zero and ready to use.
- If necessary, switch the weight display (see page 21).
- Stand on the scale and keep still.
- Read the weighing result off the digital display.
- If a load greater than 360 kg or 800 lbs is placed on the scale, **STOP** appears in the display.
- To switch off the scale, press the Start key again.

Switching the weight display

The weight display can be switched between kilogrammes (kg) and pounds (lbs) or between kilogrammes (kg) and stones (sts).

- Switch on the scale using the green Start key. Weight is initially displayed in the last setting selected.
- To switch the weight display between kilogrammes and pounds, press the **kg/lbs/sts** key.
- The **kg** or **lbs** display lights up.
- To switch weight display between pounds and stones, hold down the **kg/lbs/sts** key for approx. 2 seconds. The **lbs** or **sts** display lights up.

Switching off the scale

- To switch off the scale, press the **Start** key again.

Taring weight display (TARE)

The Tare function is very useful if an additional weight, e.g. that of a wheelchair, is to be ignored for weighing purposes.

Proceed as follows:

- Press the Start key with no load on the scale.

- First place the additional weight (e.g. the wheelchair) on the scale and activate the Tare function by pressing and holding the **HOLD/TARE** key.

The display is reset to zero, **0.0** flashes in the display until weight is detected to be stable. Only then is the Tare function fully activated. The note "TARE" appears.

- Now move the patient and the wheelchair previously weighed onto the scale. The scale determines the patient's weight.

You can now perform as many weighing operations as you like.

- Hold the key down again to deactivate the Tare function; the scale is then back in normal weighing mode and the tare value has been erased.
- The tare value is likewise erased by the scale being switched off.

Store weight value (HOLD)

You can also continue displaying the weight determined once the load has been removed. This makes it possible to look after a patient first, before having to note the weight:

- Press the **HOLD/TARE** key whilst the load is on the scale.

"HOLD" appears in the digital display. The value is stored until the HOLD function switches off automatically.

- The HOLD function can be switched off by pressing the **HOLD** key again.

0.0 appears in the display if the scale has no load on it, otherwise the current weight. The "HOLD" display vanishes.

Note:

The HOLD and TARE functions can also be used simultaneously.

Body-Mass-Index (BMI)

Body Mass Index puts height and weight in relationship to one another, providing more accurate information than ideal Broca weight, for example. A tolerance range is given which is considered ideal for health.

- Press the Start key with no load on the scale.
SECR, **88888** and **00** appear consecutively in the display. The scale is then automatically set to zero and ready for operation.
- Press the **FUNC** key.
The last function used appears in the display.
- Use the arrow keys to select the BMI function, **FU 1**. Activate the function by pressing the **FUNC** key again.

An arrow on the left-hand side of the display indicates which memory is set.

- Use the arrow keys to select the memory position under which you wish to record – or under which you have already recorded – the patient's height.
- Press the **FUNC** key again. The patient's recorded height will then appear on the display.
- You can use the arrow keys to adjust the value in increments according to the graduations displayed.
- Once the correct value is set, press the **FUNC** key again. The BMI function is now activated.
- Ask the patient to mount the scale. Once on the scale, the patient should stand still.
- Read off the BMI on the digital display and compare it with the categories on page 24.
- To return to normal weighing mode, press the **FUNC** key.
- To switch off the scale, press the Start key again.

Evaluating Body Mass Index for adults:

Compare the value determined with the categories quoted below which correspond to those used by WHO, 2000.

- **BMI below 18.5:**

Warning, this patient weighs too little. There could be a tendency to anorexia. An increase in weight is recommended to improve well-being and performance. If in doubt, consult a specialist.

- **BMI between 18.5 and 24.9:**

The patient is of a normal weight. He can stay as he is.

- **BMI between 25 and 30 (pre-obese):**

The patient is slightly to moderately overweight. He should cut his weight if he is already ill (e.g. diabetes, hypertension, gout, fat metabolism disorders).

- **BMI over 30:**

Weight reduction is essential. Metabolism, cardiovascular system and bones are all under strain. A consistent diet, plenty of movement and behavioural training are all recommended. If in doubt, consult a specialist.

Weighing with pre-Tare

If the pre-Tare function is activated, a stored weight value is deducted from the currently measured weight. This function is useful, for example, if the weight of a wheelchair is not to be displayed.

Three different tare weights (wheelchair weights) can be stored.

- Press the Start key with no load on the scale.

SECA, 88888 and 0.0 appear consecutively in the display. The scale is then automatically set to zero and ready for operation.

- Press the **FUNC** key.

The last function used appears in the display.

- Use the arrow keys to select the pre-Tare function, **FU 3**. Activate the function by pressing the **FUNC** key again.

The last memory used appears in the display.

An arrow on the left-hand side of the display also indicates which memory is set.


- Use the arrow keys to select the desired memory for storing the additional weight or where a weight may already be stored.

- Press the **FUNC** key again. In the display you will see the tare value from the memory.

- You can use the arrow keys to adjust the value in increments according to the graduations displayed.
- Once the correct value is set, press the **FUNC** key again. The pre-Tare function is now activated.
- Move onto the scale in the wheelchair associated with that memory. Read the weighing result off the digital display.
- To return to normal weighing mode, press the **FUNC** key.
- To switch off the scale, press the Start key again.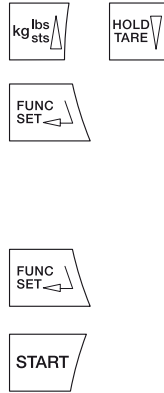

Tip: note weight and the memory in which the weight is stored on your wheelchairs.

Setting damping

Damping is useful for reducing weight determination malfunctions (due to patient movements, for example). The stronger the damping, the more slowly the weight display reacts. You can set damping to suit requirements.

- Press the **FUNC** key and keep it depressed for 2 seconds.

FIL and the current setting appear in the display:

- 0 = slight damping
 - fast determination of weight
- 1 = average damping
 - normal determination of weight
- 2 = strong damping
 - slow determination of weight

- Press an arrow key if you want to change the set value.

- Press the **FUNC** key again to store the displayed value. The scale is then back in normal weighing mode.

The settings are retained even once the scale is switched off.


6. Storing the scale

The scale can be stored upright to save space. Secure the scale so that it does not fall over when stored in the upright position.

- Remove the mains unit plug from the mains socket.
- Remove the mains unit plug from the connecting socket on the scale.
- Undo the locking bolts and fold the handrail down carefully until it is in full contact.
- Push up the latch, ensuring that it engages securely in the handrail.
- Fold up the ramps.
- Carefully pick up the scale by the handle and move it using the castors on the other side of the scale.

Note:

Please avoid knocks, especially to the display part, when transporting the scale.

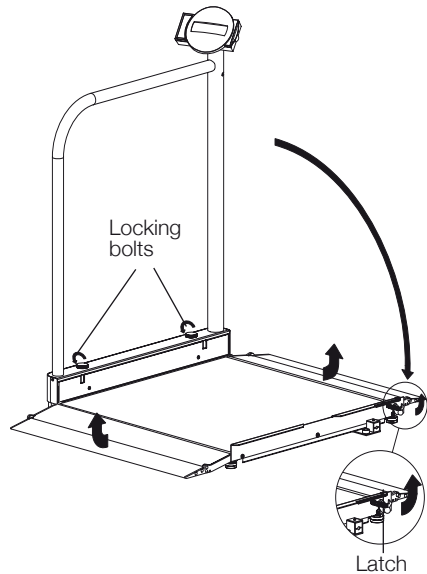7. Cleaning

Clean the surface and the housing as required using a domestic cleaning agent or commercially-available disinfectant. Follow the manufacturer's instructions.

Under no circumstances use abrasive or acid cleaners, white spirit, benzene or the like for cleaning. Such substances can damage the high-quality surfaces.

8. What do I do if...

... no weight display comes on?

- Is the scale switched on?
- Is the plug contact of the mains unit properly connected to the scale?
- Is the plug contact for connecting the display to the scale correctly connected?

... 0.0 does not appear before weighing?

- Press the Start key again – there must not be any load on the scale – and only its feet should be in contact with the floor.

... one segment is illuminated constantly or not at all?

- The relevant segment has a fault. Inform Servicing.

... 5tOP appears in the display?

- Maximum load has been exceeded.

... the display flashes?

- If you have just activated the Hold or Tare function, please wait until the scale has detected a steady weight value. The display will then stop flashing.
- If there has been no activation of a function beforehand, remove the load from the scale and wait until 0.0 is displayed, then weigh again.

... the display tENP appears?

- The ambient temperature of the scale is too high or too low. Place the scale in an ambient temperature between +10 °C and +40 °C. Wait about 15 minutes for the scale to adapt to the ambient temperature and then weigh again.

... the display E and a number appear?

- Switch off the scale with the Start key and start the scale again. The scale will then work normally again.

If this is not the case, disconnect the power supply by briefly removing the mains unit. If this measure is equally unsuccessful, inform the Service department.

9. Servicing

Your seca scale leaves the factory with an accuracy greater than $\pm 0.15\%$. To ensure that this accuracy is maintained, the product must be carefully set up and regularly maintained. Depending on the frequency of use, we recommend servicing every 3 to 5 years. Please ensure that a qualified service agent is used. If in doubt, contact seca Customer Service for further information.

10. Technical data

Dimensions scale

Folded up:

width: 1089 mm / 42.9 ins
height: 151 mm / 6 ins
depth: 965 mm / 38 ins

Set up:

width: 951 mm / 37.4 ins
height: 1107⁺¹⁰ mm / 43.6^{+0.4} ins
depth: 965 mm / 38 ins

Dimensions weighing platform

width: 800 mm / 31.5 ins
height: 51⁺¹⁰ mm / 2^{+0.4} ins
depth: 965 mm / 38 ins

Height of figures

20 mm

Weight

approx. 34.7 kg

Temperature range

+10 °C to +40 °C

EN 60 601-1:

Insulated equipment, Protection Class II:

medical electrical equipment type B:

Colour

silver grey/black

Maximum load

360 kg / 800 lbs / 57 sts

Graduations

100 g / 0.2 lbs (USA: 0.1 lbs)

Accuracy

greater than $\pm 0.15\%$ / ± 100 g

Power supply

mains unit

11. Spare parts and accessories

Model dependent mains unit

Description: plug-in mains unit, Euro, 230V~ / 50Hz / 12V= / 130mA

seca no.: 68-32-10-252

Description: plug-in mains unit, USA, 120V~ / 60Hz / 12V= / 150mA

seca no.: 68-32-10-259

Description: desk-top mains unit, switchable, Euro,

115V~ / 230V~ / 50-60Hz / 9V= / 350mA

seca no.: 68-32-10-243

Plug-in mains power unit, world-wide: 100–240V~ / 50–60Hz / 12V= / 1,25A

seca no. 68-32-10-263

12. Disposal

Disposing of the device

Do not dispose of the device in domestic waste. The device must be disposed of properly as electronic waste.

Follow the national regulations which apply in your case. For further information, contact our service department at:

service@seca.com

13. Warranty

A two-year warranty from date of delivery applies to defects attributable to poor materials or workmanship. All moveable parts - batteries, cables, mains units, rechargeable batteries etc. - are excluded. Defects which come under warranty will be made good for the customer at no charge on production of the receipt. No further claims can be entertained. The costs of transport in both directions will be borne by the customer should the equipment be located anywhere other than the customer's premises. In the event of transport damage, claims under warranty can be

honoured only if the complete original packaging was used for any transport and the scale secured and attached in that packaging just as it was when originally packed. All the packaging should therefore be retained.

A claim under warranty will not be honoured if the equipment is opened by persons not expressly authorised by seca to do so.

We would ask our customers abroad to contact their local sales agent in the event of a warranty matter.

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GB ...largely fulfils the electromagnetic compatibility guideline 89/336/EEC.

! If temporary interference and/or inaccurate weighing should nevertheless occur when the scale is used in the vicinity of intensive electromagnetic radiation sources such as microwave appliances, mobile telephones, walkie-talkies and the like, please move the scale away from the interference source or switch off the interference source and repeat the weighing operation.

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