GIMA

PROFESSIONAL MEDICAL PRODUCTS

Gima S.p.A. - Via Marconi, 1 - 20060 Gessate (MI) Italy
Italia: tel. 199 400 401 - fax 199 400 403
Export: tel. +39 02 953854209/221/225 fax +39 02 95380056
gima@gimaitaly.com - export@gimaitaly.com
www.gimaitaly.com

Aerosol ad Ultrasuoni Professionale Professional Ultrasonic Nebulizer

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CHARACTERISTICS

- Easy to use: it uses water as a conductor of ultrasounds
- Simply functions: touch controls for operation of the aerosol
- Silent: thanks to the technology of ultrasound vibrations
- 3 speed aerosol: it allows for excellent treatment for all types of medicines of different densities
- Multifunction: it acts as a dehumidifier if filled with water and as an aerosol if fitted with medicinal products
- Special design: with an adjustable air nozzle for all types of treatment
- Easy to clean: air nozzle, cover and container for the medicines are easy to remove and clean

PREPARATION

Warnings for preparation

- Only use the device for the purposes indicated in this manual. Do not use any other components not recommended by the manufacturer.
- Only use the device with medicines if recommended by your doctor.
- Do not use the device if damages have been caused or if the product is dropped in water. Take the device to the assistance centre for control and repairs.
- Always remove the plug before filling it and after use.
- When used for the first time or if a long time has passed from the first use, make sure that all parts are cleaned and sterilized, as indicated in the cleaning instructions.
- The medicine container, the mouthpiece and the air filter should be checked before any use. Replace them with new ones if they are dirty or worn.
- The device should not be used close to flammable gas, spray products or oxygen.
- When the container for medicine or water is empty, do not try to turn the device back on
- In the case of faults, interrupt use until the device has been checked and repaired
- Do not leave the device unprotected when connected to electricity
- Do not use solutions other than water in the tank, they may cause damage to the transducer
- Only use the device on solid and flat surfaces
- Opening of the cover during operation may cause serious damage

OPERATION

Warnings for operation

- If used by children or the disabled, the supervision of an adult is recommended
- Keep eyes far from the emission of medicinal products
- In the case of defects or inconvenient situations, interrupt use of the device immediately
- Comply with the general safety norms while using the UT-12L ultrasound aerosol
- Do not use during driving

STORAGE

Warnings for storage

- Keep out of direct sunlight, far from high temperatures and humidity
- Keep out of reach of children
- Always remove the plug when not in use

CLEANING

Warnings for cleaning

- Before every use check the medicine container, the air nozzle, the masks, the extended tube and the air filter. The dirty or worn areas must be replaced.
- Do not plunge the device in water, it may get damaged
- Before carrying out the cleaning operations of the device, remove the plug
- Clean every part after use as indicated in the manual

TECHNICAL CHARACTERISTICS

Name: 28085 ultrasound aerosol

Power: 230 VAC 50 Hz

Energy consumption: 30 VA

Ultrasound frequency: 2.5 MHz

Fuse characteristics: 250V, 315 mA

Nebulisation capacity: 3 adjustable speeds: 0.3~0.6mL/min, 0.6~0.9mL/min, 0.9~1.2mL/min

Diameter of the particle: 3-5 μ m typical

Capacity of the medicine container: 10ml

Environmental conditions

Temperature: 10°C – 40°C (50-104°F)

Humidity: 30% - 75% RH no condensation

Atmospheric pressure: 700 hPa 1 060 hPa

Storage temperature

Temperature: -10°C – 50°C (14-122°F)


Relative humidity: 10% - 95% RH no condensation

Atmospheric pressure: 700 hPa 1 060 hPa


Dimensions: 220 x 110 x 205mm

Weight: approximately 1300 g

DESCRIPTION OF THE PARTS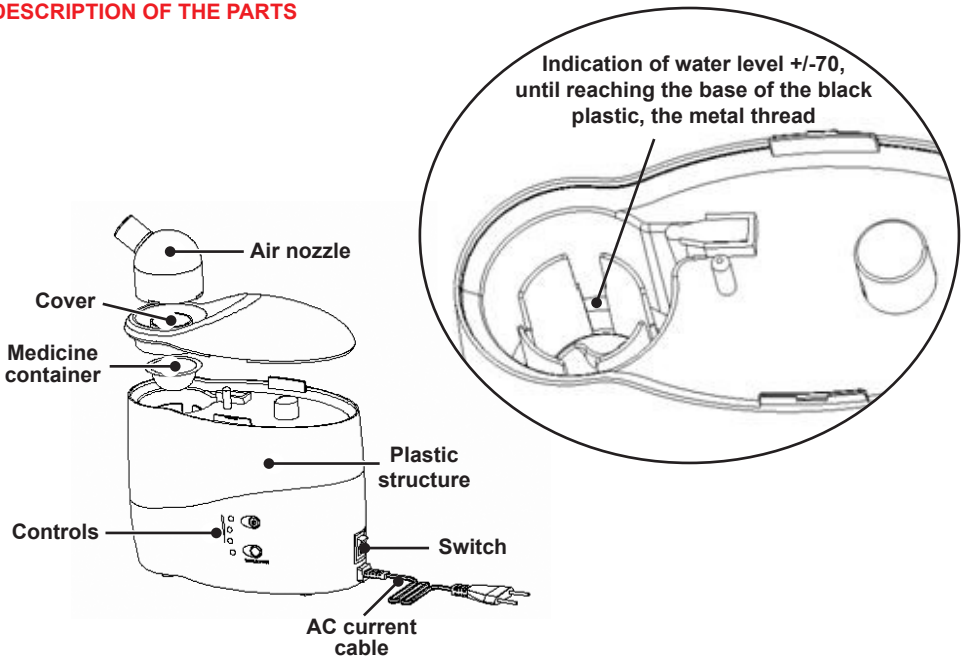

NB: The water level indicator indicates the water level recommended for the ultrasound aerosol. When the water level is below this point, the water may not come into contact with the medicine container and nebulisation may be poor. However, when the water level is above this mark, the water may overflow when the medicine container is inserted.

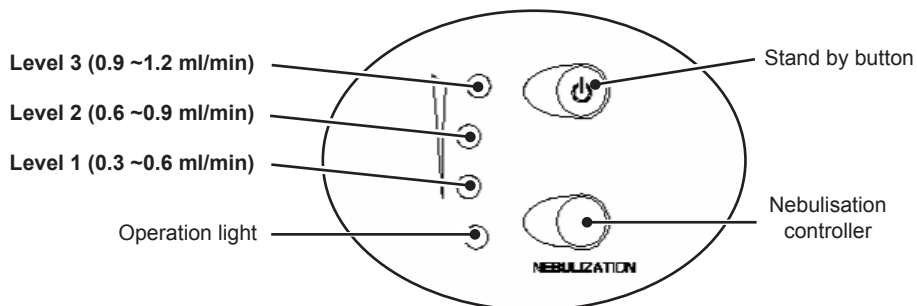DESCRIPTION OF COMMANDS

Operation light: it indicates the state of the **28085** ultrasound aerosol

- No light means that the device is not on
- A green light indicates that the device is in a stand-by mode

Level 1 - Level 3: it indicates the level of nebulisation

- Level 1: low nebulisation
- Level 2: average nebulisation
- Level 3: high nebulisation

Stand-by button: nebulisation begins

Nebulisation control: it regulates the quantity of nebulisation. By pressing these buttons Level 1, Level 2 or Level 3 may be selected.

ACCESSORIES

Before using the **28085** ultrasound aerosol, check that the packaging contains the following:


User manual: 1 piece

28085 ultrasound aerosol: 1 piece

Medicine container: 5 pieces

Air nozzle: 1 piece

Optional: adult and children's mask, extension tube, container for transportation

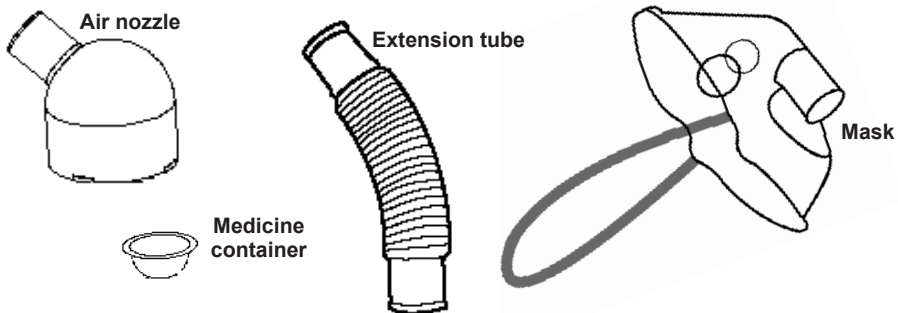Only use the specific parts indicated in the list above, the use of any other parts may be dangerous.

The medicine container may only be used once. After each use, the medicine container should be disposed of according to legal requirements.

USE

The **28085** ultrasound aerosol is used to treat asthma, chronic obstructive pulmonary disorders and other breathing problems in which a therapy with aerosol is required. It is suitable for use in one single patient.

CLEANING AND ORDINARY MAINTENANCE

It is important to clean the Aerosol before use and after every use, drying it well in every part.

Remember: remove the plug and do not plunge in water

1. Remove the cover
2. Remove any medicine left in the container
3. Remove the water from the tank
4. Rinse the air nozzle, the extension tube and the mask in warm running water for 30 seconds
5. Clean the main element, the cover and the lower air cover as well as the water tank using a damp cloth and a normal detergent
6. Dry using a clean cloth

Once or twice a week

1. Clean the air nozzle, the extension tube and the mask in a much more in-depth manner. Plunge them in a solution containing 1 glass of white vinegar and 2 glasses of warm water for 30 minutes.
2. Remove and rinse them.
3. Clean the main unit, the cover, the small air cover and the water tank using a soft cloth with water and a normal detergent.
4. Dry using a clean cloth.

OPERATION OF THE AEROSOL

Following the instructions provided above for cleaning before using the ultrasound Aerosol for the first time or after a long period in which the device is not used.

Remember: Always remove the plug before carrying out cleaning operations and filling with every use.

1. The operator should sit in a comfortable and erect position.
2. Connect the aerosol to the plug
3. Press the ignition button at the back of the device in order to activate the stand-by mode. The green light will turn on.

First step: fill the water tank

1. Remove the cover by lifting it, pull out the medicine container if available
2. Use water at a temperature of between 20°C and 30°C (68°F and 86°F), otherwise nebulisation will be affected. The use of distilled water or a physiological solution is preferred as these liquids prevent the formation of limestone inside the device that may be ruined if limestone is formed at the base of the device, a base that allows for the activation of ultrasounds.
3. Fill the tank with 70 ml of water as described above, until reaching the black plastic base.

NB: -The use of cold water (below 20°C or 68°F) would generate a low level of nebulisation
-The use of hot water (more than 30°C or 86°F) would generate greater nebulisation

Second step: Fill the medicine container

1. Place the medicine container inside the water tank
2. Pour the medicine directly into the relative container
3. Put the cover back on as well as the air cover and press the stand-by button to start nebulisation

Remember:

- *The maximum capacity of the medicine container is 10 ml and this level should not be exceeded*
- *Only use medicinal products recommended by your doctor*

The aerosol is programmed initially for level 1 (low level of nebulisation).

According to indications given by a therapist, the operator may select level 2 or level 3 (the maximum level of nebulisation) by pressing the respective nebulisation commands. By pressing the button repeatedly, level 2 or 3 may be selected or level 1.

After use press the stand-by button in order to stop nebulisation. The switch on the side is used to turn the aerosol off.

Warning: *Do not put the air nozzle directly in your mouth. Use the accessories recommended with the air nozzle.*

Third step: after each use

1. Remove the plug of the aerosol
2. Carefully remove the medicine container and dispose of the medicine left over
3. Empty the water tank
4. Dry the product in the best possible way

PROBLEM SOLVING

In the event of a fault, check and correct as follows:

Conditions	Solutions
When I press the ignition button of the device it does not turn on	<ul style="list-style-type: none"> - Make sure that the plug has been correctly inserted into the wall - Check that the water level in the tank is as indicated by the relative marker
The device is on but it does not nebulise or nebulisation is poor	<ul style="list-style-type: none"> - Check that the medicinal product is in the container. - Check that the medicine container is not damaged. - Check that the transducer is clean
The device is on but the fan does not work correctly	Check that the fan has been correctly fitted and fixed to the motor shaft

Disposal: *The product must not be disposed of along with other domestic waste. The users must dispose of this equipment by bringing it to a specific recycling point for electric and electronic equipment.*

For further information on recycling points contact the local authorities, the local recycling center or the shop where the product was purchased. If the equipment is not disposed of correctly, fines or penalties may be applied in accordance with the national legislation and regulations.

GIMA WARRANTY CONDITIONS

Congratulations for purchasing a GIMA product.

This product meets high qualitative standards both as regards the material and the production. The warranty is valid for 12 months from the date of supply of GIMA.

During the period of validity of the warranty, GIMA will repair and/or replace free of charge all the defected parts due to production reasons. Labor costs and personnel traveling expenses and packaging not included. All components subject to wear are not included in the warranty.

The repair or replacement performed during the warranty period shall not extend the warranty.

The warranty is void in the following cases: repairs performed by unauthorized personnel or with non-original spare parts, defects caused by negligence or incorrect use.

GIMA cannot be held responsible for malfunctioning on electronic devices or software due to outside agents such as: voltage changes, electro-magnetic fields, radio interferences, etc.

The warranty is void if the above regulations are not observed and if the serial code (if available) has been removed, cancelled or changed. The defected products must be returned only to the dealer the product was purchased from. Products sent to GIMA will be rejected.