B

Exhaust vent / Luftauslassöffnung /
Orifice de ventilation / Presa
d'aerazione / Ventilación / Orificio de
ventilación / Utloppsventil

Mask cushion / Maskeneinsatz / Bulle
du masque / Cuscinetto della
maschera / Almohadilla de la
mascarilla / Almofada da máscara /
Maskkudde

Luer lock port cap/s /
Anschlusskappen für Luer-
Verbindungsstück / Bouchon(s)
d'entrée de raccord Luer / Tappi
porte con connessione Luer /
Tapas de puerto Luer lock /
Tampas das portas com fecho
do tipo Luer / Luerportlock

Mask port/s / Anschlussöffnung /
Port(s) du masque / Porta della
maschera / Puerto(s) de la
mascarilla / Orificio da máscara /
Maskport

Mask frame / Maskenhalter / Entourage rigide /
Telaio della maschera / Armazón de la
mascarilla / Armação da máscara / Maskram

Cushion clip / Clip des
Maskeneinsatzes / Clip de la
bulle / Fermaglio del cuscinetto /
Traba de la almohadilla /
Dispositivo de fixação da
almofada / Kuddklämma

Valve membrane / Ventilmembran /
Membrane de la valve / Membrana della
valvola / Membrana de la válvula /
Membrana da válvula / Klaffhinna

Valve elbow / Ventilknüststück / Coude
de la valve / Gomito della valvola /
Codo de válvula / Cotovelo da válvula /
Klaffbäge

Elbow retainer / Knüststückhalter /
Anneau de retenue du coude /
Fermaglio del gomito / Retenedor
del codo / Retentor do cotovelo /
Båghållare

Swivel clip / Drehgelenkclips /
Clip de la pièce pivotante / Clip
della parte girevole / Clip de la
pieza giratoria / Dispositivo de
fixação da peça giratória /
Svängtappsklämma

Valve frame / Ventilrahmen / Corps
de la valve / Telaio della valvola /
Armazón de la válvula / Armação
da válvula / Klafffram

Swivel / Drehgelenk / Pièce
pivotante / Parte girevole /
Pieza giratoria / Peça giratória /
Svängtapp

Side upper Velcro straps / Die oberen
Seitenbänder (Velcro) / Bandes Velcro
latérales supérieures / Cinghie laterali
superiori (Velcro) / Correas laterales
superiores (Velcro) / Correas Velcro
laterais superiores / Övre sidorem


Top Velcro strap / Das obere Band (Velcro) /
Bande Velcro supérieure / Cinghia superiore
(Velcro) / Correa superior (Velcro) /
Correia Velcro superior / Övre kardborrerem

Quick release tab / Schnellöffnungsgriff /
Languette de retrait rapide / Languetta di
rilascio rapido / Tira de apertura rápida /
Patilha de desengate rápido /
Snabblösningssflik


Side lower Velcro straps / Die
unteren Seitenbänder (Velcro) /
Bandes Velcro latérales inférieures /
Cinghie laterali inferiori (Velcro) /
Correas laterales inferiores (Velcro) /
Correas Velcro laterais inferiores /
Lågre sidans kardborreremmar

C


C-1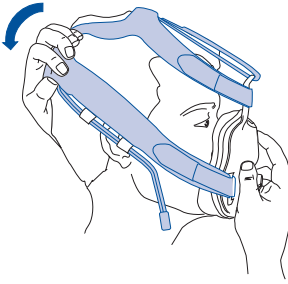


C-2

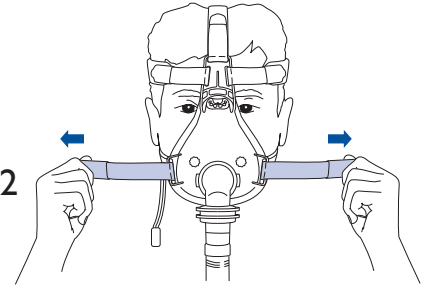
C-3

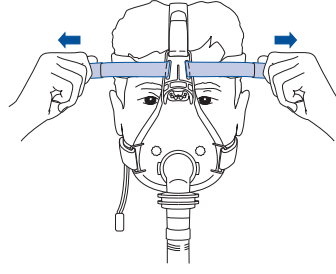
C-4

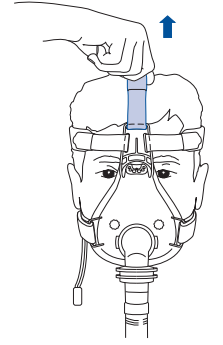
C-5

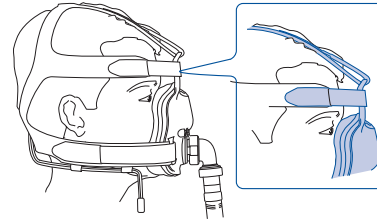C-7

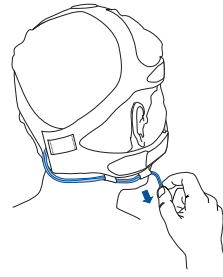C-6

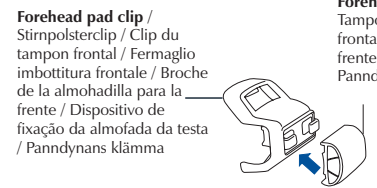Forehead pad clip /
Stimpolsterclip / Clip du
tampon frontal / Fermaglio
imbottitura frontale / Broche
de la almohadilla para la
frente / Dispositivo de
fixação da almofada da testa
/ Panndynans klämma

Forehead pad / Stimpolster /
Tampon frontal / Imbottitura
frontale / Almohadilla para la
frente / Almofada da testa /
Panndyna


D

L-shaped ridge / L-förmige Leiste /
Bord en L / Costa sinistra a L / Borde
en forma de L / Saliência em forma
de L / Högra L-formade kanten


LIFT / HEBEN / SOULEVER /
SOLLEVARE / LEVANTE /
LEVANTE / LYFT

PUSH / DRÜCKEN / POUSSER /
SPINGERE / EMPUJE / EMPURRE /
TRYCK


D-1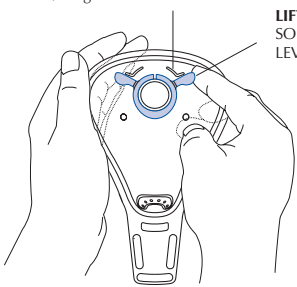


D-2

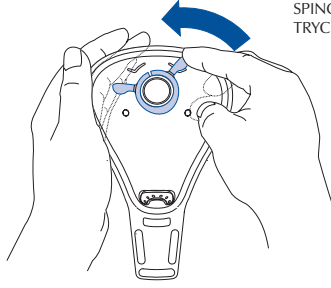D-3

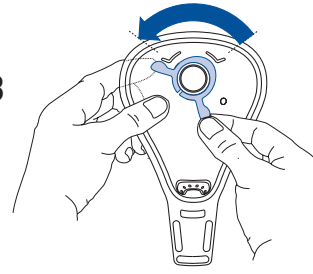D-4

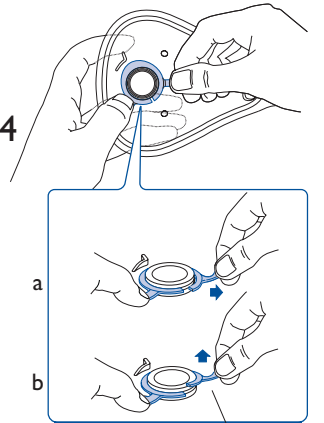
Swivel clip / Drehgelenkclips /
Clip de la pièce pivotante / Clip
della parte girevole / Clip de la
pieza giratoria / Dispositivo de
fixação da peça giratória /
Svängtappsklämma

D-5


D-6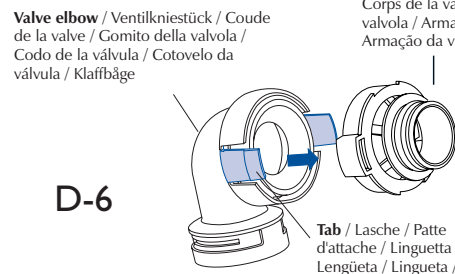

Valve elbow / Ventilknüststück / Coude
de la valve / Gomito della valvola /
Codo de la válvula / Cotovelo da
válvula / Klaffbäge

Valve frame / Ventilrahmen / Corps
de la valve / Telaio della valvola /
Armazón de la válvula / Armação
da válvula / Klafffram

D-7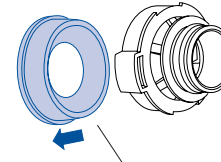

Valve membrane / Ventilmembran /
Membrane de la valve / Membrana della
valvola / Membrana de la válvula /
Membrana da válvula / Klaffhinna


Elbow retainer /
Knüststückhalter / Anneau de
retenue du coude / Fermaglio
del gomito / Retenedor del
codo / Retentor do cotovelo /
Båghållare

Swivel / Drehgelenk / Pièce
pivotante / Parte girevole /
Pieza giratoria / Peça giratória /
Svängtapp


Tab / Lasche / Patte
d'attache / Languetta /
Lengüeta / Lingueta / Flik

E


E-1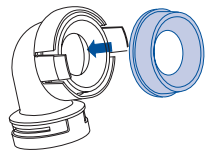

E-2

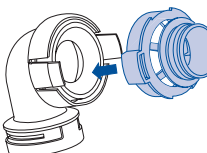
E-3

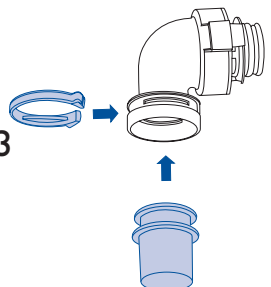
F

F-1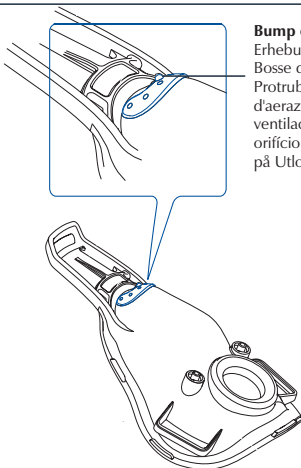


Bump on the exhaust vent /
Erhebung an der Luftauslassöffnung /
Bosse de l'orifice de ventilation /
Protruberanza sulla presa
d'aerazione / Protuberancia de la
ventilación / Protuberância no
orificio de ventilação / Utbuktning
på Utloppsventil

F-2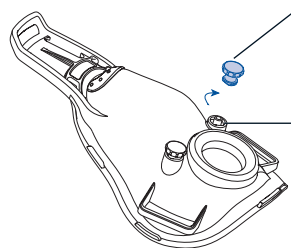


Luer lock port cap/s /
Anschlusskappen für Luer-
Verbindungsstück / Bouchon(s)
d'entrée de raccord Luer / Tappi
porte con connessione Luer /
Tapas de puerto Luer lock /
Tampas das portas com fecho
do tipo Luer / Luerportlock


F-3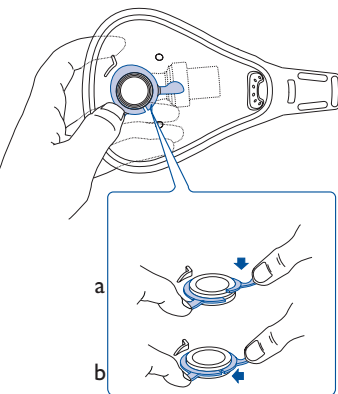

F-4

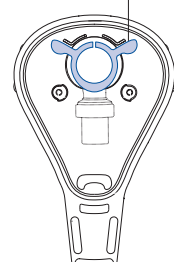
F-5

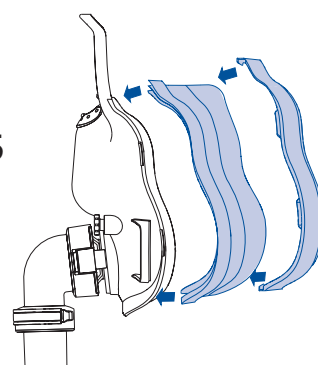
Elbow retainer / Knüststückhalter /
Anneau de retenue du coude /
Fermaglio del gomito / Retenedor
del codo / Retentor do cotovelo /
Båghållare

G

G-1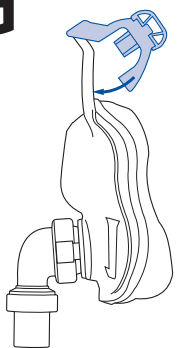

G-2

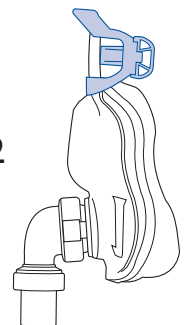USER'S GUIDE

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MODE D'EMPLOI

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GUIA DO UTILIZADOR

BRUKSANVISNING

www.resmed.com

MANUFACTURED BY:

ResMed Ltd 97 Waterloo Road North Ryde NSW 2113 Australia
Tel: +61 (2) 9886 5000 or 1 800 658 189 (toll free) Fax: +61 (2) 9878 0120 Email: reception@resmed.com.au

DISTRIBUTED BY:

ResMed Corp 14040 Danielson Street Poway CA 92064-6857 USA
Tel: +1 (858) 746-2400 or 1-800-424-0737 (toll free) Fax: +1 (858) 746-2900 Email: reception@resmed.com

ResMed (UK) Ltd 65 Milton Park Abingdon Oxfordshire OX14 4RX UK
Tel: +44 (1235) 862 997 Fax: +44 (1235) 831 336 Email: reception@resmed.co.uk

ResMed Asia Pacific Ltd 97 Waterloo Road North Ryde NSW 2113 Australia
Tel: +61 (2) 9886 5000 or 1 800 991 900 (toll free) Fax: +61 (2) 9889 1471 Email: reception@resmed.com.au

ResMed NZ Ltd PO Box 51-048 Pakuranga Auckland New Zealand
Mob: +64 25 737 633 Fax: +64 9 239 0193 Email: reception@resmed.co.nz

ResMed GmbH & Co. KG Rudolfstraße 10 D-41068 Mönchengladbach Deutschland
Tel: +49 (0) 2161-3521-0 (Reception), +49 (0) 180 22 22 668 (Service-Telefon; 0,06 €/Anruf),
+49 (0) 180 22 66 888 (Wartungstelefon; 0,06 €/Anruf) Fax: +49 (0) 2161-3521-299
Email: reception@resmed.de

ResMed SA Parc de la Bandonnière 2, rue Maurice Audibert 69800 Saint-Priest France
Tél: +33 (0) 4 37 251 251 Fax: +33 (0) 4 37 251 260 Email: reception@resmed.fr

ResMed Sweden AB Industrigatan 2 S-461 37 Trollhättan Sverige
Tel: +46 520 420 110 Fax: +46 520 397 15 Email: reception@resmed.se

ResMed Finland Nittykatu 6 Fin 02200 Espoo Suomi
Puh: +358 9 8676820 Fax: +358 9 86768222 Email: Sähköposti:jussiv@resmed.de

Labhardt AG Thannerstrasse 57 CH-4054 Basel Schweiz
Tel: +41 (061) 307 9711 Fax: +41 (061z) 307 9722 Email: info@labhardt.ch

ResMed Singapore Pte Ltd 57 Ubi Avenue 1 #07-09 Ubi Centre Singapore 408936
Tel: +65 284 7177 Fax: +65 284 7787 Email: reception@resmed.com.sg

ResMed Malaysia Sdn Bhd Suite E-10-20, Plaza Mon't Kiara No. 2, Jalan 1/70C, Mon't Kiara 50480 Kuala Lumpur, Malaysia Tel: +60 3 6201 7177 Fax: +60 3 6201 2177 Email: reception@resmed.com.my

ResMed Japan Nihonbashi Hisamatsu Bldg. 4F, 2-28-1 Nihonbashi-Hamacho, Chuo-Ku, Tokyo 103-0007, Japan
Tel: +81 (3) 3662 5056 Fax: +81 (3) 3662 5040

US DESIGNATED AGENT: ResMed Corp

EU AUTHORISED REPRESENTATIVE: ResMed (UK) Ltd

Internet

<http://www.resmed.com>

Mirage Full Face Mask

Protected by patents: AU 710733, AU 712236, AU 724360, AU 744593, AU 753244, DE 29724224, SE 75238, US 6112746, US 6357441, US 6422238, US6491034. Other patents pending. Designs registered: AU 147929, DE 40110080.4, FR 017044.

Mirage is a trademark of ResMed Ltd.

Velcro is a registered trademark of Velcro Industries. CIDEX and STERRAD are registered trademarks of Johnson & Johnson.


© 2003 ResMed Ltd.

Re-order number: 16889/3 03 02

A

Vent flow rate / Durchflussrate an den Luftauslassöffnungen / Débit à l'orifice de ventilation / Coefficiente di flusso della presa d'aerazione / Caudal de flujo de ventilación / Taxa de fluxo de ventilação / Ventilflödestakt (L/min)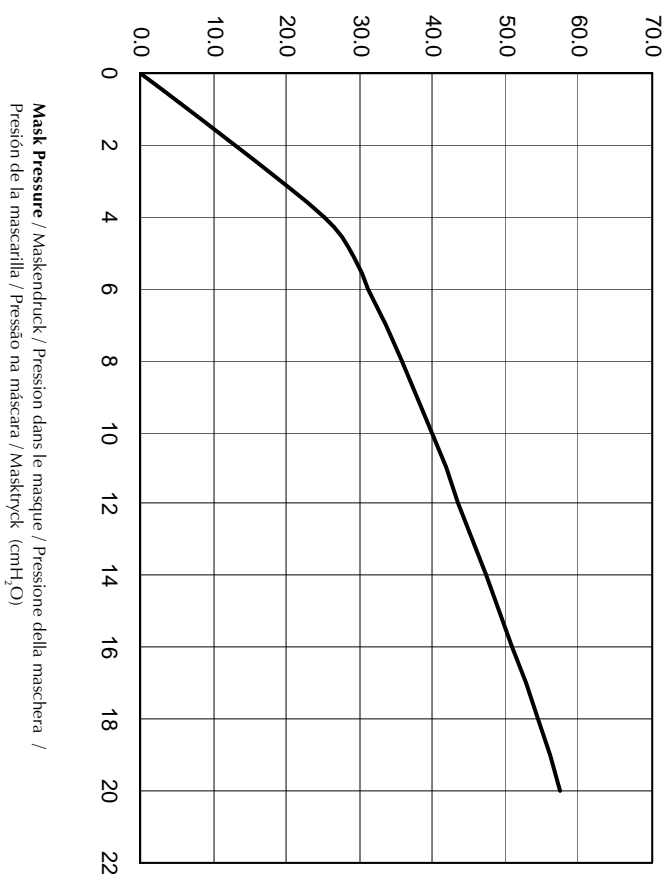

RESMED

MIRAGE® FULL FACE MASK SERIES 2

The MIRAGE® FULL FACE MASK SERIES 2 is indicated for adult patients using the following:

- continuous positive airway pressure (CPAP) systems
- bilevel systems
- other pressure support ventilators intended to work with vented masks.

MEDICAL INFORMATION

CONTRAINDICATION

The MIRAGE FULL FACE MASK SERIES 2 must be used under qualified supervision for patients who are unable to remove the mask.

WARNINGS

- The mask should not be used unless the CPAP or bilevel system is turned on and operating properly.
- When the CPAP or bilevel system is functioning properly, fresh air flushes exhaled air out through the exhaust vents of the mask. Rebreathing exhaled air may be harmful if continued for prolonged periods. Never block the exhaust vents.
- The exhaust vent flow is lower at lower mask pressures (see Pressure/Flow Curve - **Figure A**). If the mask pressure during expiration is too low, there may not be enough exhaust vent flow to completely flush the air delivery hose of exhaled air before the start of the next inspiration. The possibility of rebreathing should be assessed in patients prescribed expiratory pressures below 4 cmH₂O. However, when the system is not operating, exhaled air may be rebreathed. Rebreathing of exhaled air can in some circumstances, lead to suffocation.
- The mask should not be worn if the anti-asphyxia valve membrane is damaged or missing. If the valve membrane is damaged, distorted or torn, it should be replaced with a new valve membrane.
- The anti-asphyxia valve membrane should be replaced at least every six months.
- The mask may not be suitable for those at increased risk of gastric regurgitation.
- If supplemental oxygen is used, inhaled oxygen concentration will vary, depending on the pressure settings, patient breathing pattern, mask selection and the leak rate.
- When using a humidifier with the system, always place the humidifier below the level of your body. If it is placed at or above the level of your body, heated water may flow down the air tubing and into the mask.

PARTS OF THE MASK

See **Figure B**.

FITTING THE MASK

Note: Before fitting the mask, refer to the section on “Cleaning the Mask in the Home”.

The **MIRAGE FULL FACE MASK SERIES 2** is supplied fully assembled.

- 1 Connect the main air tubing to the swivel.
- 2 Adjust the headgear Velcro® straps so that the headgear can be pulled freely over your head.
- 3 Seat the mask cushion over your nose and mouth. The cushion should rest on your chin and between your eyebrows.
- 4 Pull the headgear over your head (**Fig. C-1**). Make sure the side lower straps pass below your ears, and that the side upper straps pass over your forehead.
- 5 Tighten the side lower (**Fig. C-2**) and then the side upper (**Fig. C-3**) straps to achieve a secure and comfortable fit.
- 6 Adjust the top strap (**Fig. C-4**) with very slight tension to add stability to the fit of the mask. The mask should be secure but not overly tight.
- 7 The fitted mask should look like this (**Fig. C-5**).

Note: *The top of the mask frame should not touch your forehead. In most cases, the top of the mask frame should be some distance away from your forehead.*

- 8 Connect the free end of the air tubing to your flow generator and turn the flow generator on. If you feel air leaks around the cushion, reposition the mask on your face to achieve a better seal. You may also need to readjust the Velcro straps.

Take time to adjust the Velcro straps and the position of the mask to achieve a comfortable and secure fit.

FOREHEAD PAD CLIP ASSEMBLY

WARNING

The forehead pad clip assembly is for single-patient use only.

The **MIRAGE FULL FACE MASK SERIES 2** is supplied with a forehead pad clip assembly kit (**Fig. C-6**). This kit is an optional accessory for patients who may require additional support and stability. For assembly instructions, see section "Reassembling the Mask" (step 8).

Notes:

- The forehead pad clip is supplied with the following sizes of forehead pads: standard (assembled with the forehead pad clip), shallow, and deep. Choose the appropriate size to achieve a comfortable and secure fit.
- When the mask is fitted with a forehead pad clip assembly attached, the forehead pad should lightly touch the patient's forehead without the mask putting pressure on the bridge of the nose.

REMOVING THE MASK

To remove the **MIRAGE FULL FACE MASK SERIES 2**, release the side lower Velcro straps and pull the mask and headgear up over your head.

Note: *The Quick Release Tab should be used to remove the mask only in the event of an emergency.*

QUICK RELEASE TAB

A feature of the **MIRAGE FULL FACE MASK SERIES 2** is the Quick Release Tab that allows the mask to come away from the face quickly in the event of an emergency.

Simply pull the Quick Release Tab (**Fig. C-7**) and the rear Velcro opening of the headgear separates, allowing you to remove the mask from your face.

Note: *The Quick Release Tab should only be used to remove the mask in the event of an emergency. Overuse of the Quick Release Tab will result in the deterioration of the Velcro opening.*

DISASSEMBLING FOR CLEANING

- 1 Detach and separate the air tubing, headgear, exhaust vent, luer lock port caps, cushion clip, mask cushion and forehead pad clip assembly (if attached).
- 2 From the inside of the mask frame, lift (**Fig. D-1**) and push (**Fig. D-2**) the right wing of the elbow retainer over the right L-shaped ridge on the mask frame.
- 3 Rotate the elbow retainer to the position shown in **Figure D-3**. The elbow retainer is now in the correct position for removal.

- 4 To remove the elbow retainer, hold one wing of the elbow retainer down, and firmly pull the other wing out (Fig. D-4a) and then up (Fig. D-4b) over the top of the valve. Remove the elbow retainer and valve from the mask frame.
- 5 Use your thumb to remove the swivel clip from the valve elbow. This releases the swivel from the valve (Fig. D-5).
- 6 Squeeze the tabs of the valve elbow to remove the valve frame from the valve elbow (Fig. D-6).
- 7 Remove the valve membrane from the valve frame carefully (Fig. D-7).

CLEANING THE MASK IN THE HOME

Note: Wash the headgear before first time use, as the dye may run. It is important to always handwash the headgear.

- 1 Handwash the headgear in warm, soapy water. Use a mild detergent such as pure soap. Rinse well and allow it to air dry out of direct sunlight. Do this at least once a week.
- 2 Handwash the mask components in warm, soapy water. Use a mild detergent such as pure soap. Rinse the components well and allow them to air dry out of direct sunlight. Do this after each use.

To clean the valve membrane, immerse it in soapy water and gently rub the membrane to remove any foreign matter. Rinse the membrane and allow it to air dry out of direct sunlight.

Inspect the valve membrane for wear and deterioration. If the valve membrane is damaged, distorted or torn, replace it with a new valve membrane. Be sure that the membrane is clean and dry before re-inserting it into the valve.

CAUTION

- Do not use bleach-, chlorine-, alcohol-, or aromatic-based solutions, moisturising or antibacterial soaps, or scented oils to clean the cushion, mask frame, headgear or air

tubing. These solutions may cause damage and reduce the life of the product.

- Do not expose any part of the mask system or tubing to direct sunlight as it may cause the product to deteriorate.

CLEANING AND DISINFECTING/ STERILISING THE MASK BETWEEN PATIENTS

The following are the recommended and validated procedures for cleaning, disinfection and sterilisation of the MIRAGE FULL FACE MASK SERIES 2. We advise that each healthcare facility consult its own procedures before carrying out the following instructions.

WARNINGS

- ResMed cannot give any assurance that deviations from the procedures listed below, and their effect on the performance of the product, will be acceptable.
- MIRAGE FULL FACE MASK SERIES 2 components should not be subjected to autoclave or ethylene-oxide gas sterilisation.

After disassembly, thoroughly clean the mask components according to the following cleaning procedure.

CLEANING PRIOR TO DISINFECTION OR STERILISATION

- 1 Handwash the headgear in warm water using a mild detergent, such as pure soap. Rinse well and allow it to air dry out of direct sunlight.
- 2 Clean the individual parts of the mask with a soft bristle brush for one minute while soaking them in detergent. Pay particular attention to all crevices and cavities.
- 3 Rinse the components twice by agitating them vigorously in water and allow them to air dry out of direct sunlight.

CAUTION

- Do not disinfect or sterilise the headgear and forehead pad clip assembly. These parts cannot be disinfected or sterilised.
- The valve membrane must be replaced with a new valve membrane when the mask is reused on another patient. This part cannot be disinfected or sterilised.

After disassembly and cleaning, disinfect the reusable mask components according to the following procedure for chemical or thermal disinfection OR

Sterilise the mask according to the following procedure for the STERRAD® Sterilisation Process.

Note: The mask must be disassembled and cleaned prior to disinfection or sterilisation. The mask must also be inspected after disinfection/sterilisation. See section "Inspection after Disinfection/Sterilisation" for inspection details.

OPTION 1: CHEMICAL DISINFECTION

- 1 Soak the reusable mask components in a commercially available solution of glutaraldehyde 3.4% (e.g. CIDEX Plus®), peracetic acid 0.08% (e.g. CIDEX PA) or orthophthalaldehyde 0.55% (CIDEX® OPA) as described in the manufacturer's instructions.
- 2 Rinse the mask components as described in the manufacturer's instructions and allow them to air dry out of direct sunlight.

Testing by ResMed has shown that the mask components can withstand 15 cycles using solutions of glutaraldehyde 3.4% (CIDEX Plus®), peracetic acid 0.08% (CIDEX PA) or orthophthalaldehyde 0.55% (CIDEX® OPA) when following the manufacturer's instructions.

OPTION 2: THERMAL DISINFECTION

Using a certified hot water disinfection system, soak the mask components for one of the following times:

- 70°C for 100 minutes or
- 75°C for 30 minutes or

- 80°C for 10 minutes or
- 90°C for 1 minute.

Testing by ResMed has shown that the mask components can withstand 15 cycles of disinfection using the above time-temperature combinations.

OPTION 3: STERRAD® 100S STERILISATION PROCESS

Sterilise the mask components as described in the manufacturer's instructions for the STERRAD 100S Sterilisation System.

Testing by ResMed has shown that the mask components can withstand 15 cycles of the STERRAD 100S Sterilisation System.

INSPECTION AFTER DISINFECTION/ STERILISATION

Perform a visual inspection of each mask component. If any visible deterioration of a mask component is apparent (cracking, tears etc), the mask component should be discarded and replaced. Yellow discolouration of the cushion may occur. This discolouration is acceptable.

Note: If a healthcare facility requires an additional disinfection or sterilisation process after reassembly, the number of validated cycles is halved.

REASSEMBLING THE VALVE

Make sure all parts of the mask system, especially the valve membrane, are clean and dry before reassembling. If the valve membrane is damaged, distorted or torn, replace it with a new valve membrane.

CAUTION

The valve membrane must be replaced with a new valve membrane when the mask is reused on another patient. This part cannot be disinfected or sterilised.

- 1 Insert the valve membrane into the valve elbow and gently push it into place (**Fig. E-1**). Make sure it is seated evenly and flatly around the rim of the valve elbow, as the membrane may produce a noise while you are breathing out if it is fitted incorrectly.

- 2 Push the valve frame over the tabs of the valve elbow until they lock together (**Fig. E-2**). Make sure the valve frame and valve elbow are securely connected to each other.
- 3 Insert the swivel into the end of the valve elbow and secure it with the swivel clip (**Fig. E-3**).

REASSEMBLING THE MASK

- 1 Check that the holes in the exhaust vent are not obstructed. If they are obstructed, replace the exhaust vent. Push the exhaust vent onto the mask frame so the small round bump on the exhaust vent is on the outside of the mask frame (**Fig. F-1**).
- 2 Twist the luer lock port caps onto the mask ports (**Fig. F-2**).
- 3 Insert the valve into the mask frame. Hold the mask frame and the inserted valve firmly together.
- 4 To attach the elbow retainer, fit one side of the elbow retainer against the groove of the valve. While holding one wing of the elbow retainer down (**Fig. F-3a**), press the other wing down until the elbow retainer clicks into place (**Fig. F-3b**).
- 5 Rotate the elbow retainer to its original position (**Fig. F-4**).
- 6 Check that you have fitted the valve and elbow retainer correctly by firmly pulling the valve away from the frame. If the valve is fitted correctly, it should not separate from the frame.
If the valve comes away from the frame, try refitting the elbow retainer as described in Steps 3 and 4. If the valve still comes away from the frame when you pull on it, you may need to replace the elbow retainer with a new one.
- 7 Fit the grooved edge of the mask cushion to the mask frame. Fit the top section first, then work your way around the mask, pushing the cushion onto the frame, ensuring a good seal has been achieved.
Then push the cushion clip over the cushion onto the mask frame, ensuring it clicks into place in all locations (**Fig. F-5**).

- 8 **Optional:** Attach the forehead pad clip assembly to the top of the mask frame (**Figs. G-1 and G-2**).
- 9 Thread the straps of the headgear through the slots on the mask frame and fasten the Velcro straps. The **MIRAGE FULL FACE MASK SERIES 2** headgear label should face outwards and be near your neck when fitted.

Note: Make sure the cord of the Quick Release Tab is not worn or damaged. If it is damaged, replace the entire headgear.

ACCESSORIES/REPLACEMENT PARTS

Part No.	Item
I6659	Mirage Full Face Mask Series 2 System, <i>Small, Standard</i>
I6660	Mirage Full Face Mask Series 2 System, <i>Small, Shallow</i>
I6661	Mirage Full Face Mask Series 2 System, <i>Medium, Standard</i>
I6662	Mirage Full Face Mask Series 2 System, <i>Medium, Shallow</i>
I6663	Mirage Full Face Mask Series 2 System, <i>Large, Standard</i>
I6664	Mirage Full Face Mask Series 2 System, <i>Large, Shallow</i>
I6613	Anti-asphyxia valve (1 pk.)
I6604	Cushion, <i>Small, Standard</i> (1 pk.)
I6671	Cushion, <i>Small, Shallow</i> (1 pk.)
I6605	Cushion, <i>Medium, Standard</i> (1 pk.)
I6672	Cushion, <i>Medium, Shallow</i> (1 pk.)
I6606	Cushion, <i>Large, Standard</i> (1 pk.)
I6673	Cushion, <i>Large, Shallow</i> (1 pk.)
I6674	Cushion clip, <i>Small</i> (1 pk.)
I6675	Cushion clip, <i>Medium</i> (1 pk.)
I6676	Cushion clip, <i>Large</i> (1 pk.)
I6509	Exhaust vent (1 pk.)
I6510	Exhaust vent (10 pk.)
I6680	Headgear (1 pk.)
I6681	Luer lock port caps (2 pk.)
I6682	Luer lock port caps (10 pk.)

Part No.	Item
16677	Mask frame, <i>Small</i> (1 pk.)
16678	Mask frame, <i>Medium</i> (1 pk.)
16679	Mask frame, <i>Large</i> (1 pk.)
16637	Swivel clip (5 pk.)
16624	Valve elbow (1 pk.)
16617	Valve membrane (1 pk.)
16618	Valve membrane (10 pk.)
16972	Forehead pad clip accessory kit (1 pk.)
60900	Forehead pad, <i>Shallow</i> (2 pk.)
16561	Forehead pad, <i>Standard</i> (2 pk.)
60901	Forehead pad, <i>Deep</i> (2 pk.)

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