MEDISANA®

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1.1 Thank you very much

Thank you for your confidence! You have purchased the blood pressure monitor **MTA**, a quality product by **MEDISANA**.

We recommend that you carefully read the following information on use and care of the product so that you have the expected results and enjoy using your **MEDISANA MTA** blood pressure monitor for many years.

1.2 Tips for your well being

WARNING

Do not take any therapeutic measures based on your own measurements! Never change the quantity of medication prescribed by your doctor!

- Irregularity of pulse or arrhythmia can lead to difficulties in recording a correct reading when measurements are taken using oscillometric blood pressure devices.
- If you should suffer from conditions such as arterial occlusive disease, please consult your doctor before using the unit.
- The unit may not be used to check the heart rate of patients with a pacemaker.
- Pregnant patients should take their own personal state of health into consideration and take all necessary precautions before using the unit. If in doubt, consult your doctor.
- Should you experience discomfort or other complaints during measurement, for example pain in the upper arm, please take the following action: Press the **ON/OFF/START** button **4** to deflate the cuff immediately. Loosen the cuff and remove it from your arm. Please contact either your dealer or **MEDISANA** directly.
- Patients with a weak pulse wave should raise their arm and open and close their hand around 10 times before taking a reading. This exercise optimizes the pulse wave and the measuring process.

1.3 Important considerations

- The unit is only to be used for the specific purpose described in these operating instructions.
- Any misuse will void the warranty.
- On the rare occasion of a fault causing the cuff to remain fully inflated during measurement, open the cuff immediately.
- Do not use the unit near to equipment emitting a strong electric or magnetic field, such as radios. This may impair the correct functioning of the unit.
- Do not let children operate the unit. Medical products are not toys!
- If a fault occurs, do not try to repair the unit yourself. Attempts to do so will void the warranty. Refer all servicing to authorized service personnel.
- Protect the unit from moisture. Should moisture enter the unit, remove the batteries and stop using it immediately. In this case, please contact either your dealer or **MEDISANA** directly. You can find details of how to contact us on the address page.

**1.4
Items supplied
and packaging**

Please ensure that the package contents are complete. These are as follows:

- 1 **MEDISANA** Blood pressure monitor **MTA**
- 1 Cuff with air hose
- 4 Batteries (AA type, LR 6) 1,5V
- 1 Storage bag
- 1 Operating instructions

The packaging can be reused or recycled. Please dispose properly of any packaging material no longer required. If you notice any transport damage during unpacking, please contact your dealer without delay.

WARNING

To avoid risk of suffocation, keep packaging film away from children!

**2.1
What is blood
pressure?**

Blood pressure is the pressure created in the blood vessels by each heart beat. When the heart contracts (= systole) and pumps blood into the arteries, it brings about a rise in pressure. The highest value is known as the systolic pressure and is the first value to be measured.

When the cardiac muscle relaxes to take in fresh blood, the pressure in the arteries also falls. Once the arteries are relaxed, a second reading is taken – the diastolic pressure.

**2.2
How is blood
pressure
measured?**

The **MTA** is a blood pressure computer which is used to measure blood pressure at the upper arm. The measurement is carried out by a microprocessor, which, via a pressure sensor, measures vibrations resulting from the inflation and deflation of the cuff over the artery.

**2.3
Why should
you take your
blood pressure
at home?**

MEDISANA has had many years of experience in the field of blood pressure measurement. The high precision of **MEDISANA** devices has been confirmed by extensive clinical tests, carried out according to strict international standards. One strong argument in favour of taking blood pressure at home is the fact that the readings are taken in familiar surroundings under relaxed conditions. The most important reading is the so-called „baseline value“, the one taken before breakfast, directly after getting up in the morning. As far as it is possible, you should always try to take your blood pressure at the same time and under the same conditions each day.

This ensures comparable results and helps in detecting the first signs of high blood pressure. If high blood pressure remains undetected, it increases the risk of further cardiovascular diseases.

**THEREFORE WE RECOMMEND:**

Take a blood pressure reading daily and on a regular basis, even if you are experiencing no apparent complaints.

2.4 WHO blood pressure classification

The following table shows the classification of high and low blood pressure, without taking age into consideration. These values were determined by the World Health Organization (WHO).

Low blood pressure	Normal blood pressure
systolic below 100	systolic 100–140
diastolic below 60	diastolic 60–90

Types of high blood pressure

Stage 1	Stage 2	Stage 3
mild hypertension	medium hypertension	serious hypertension
systolic 140–159	systolic 160–179	systolic über 180
diastolic über 90	diastolic über 95	diastolic über 110


2.5 Fluctuations in blood pressure

There are many factors that can influence blood pressure. Readings can be affected by stress, fear, heavy physical exertion or the time of day at which they are taken. Your personal blood pressure values are subject to considerable fluctuation during the course of the day and the year. These fluctuations are particularly pronounced in the case of patients suffering from hypertension. Blood pressure is usually at its highest during physical exertion and at its lowest at night when asleep.

2.6 Influencing and evaluating readings

- Measure your blood pressure several times, then record and compare the results. Do not draw any conclusions from a single reading.
- Your blood pressure readings should always be evaluated by a doctor who is also familiar with your personal medical history. When using the unit regularly and recording the values for your doctor, you should visit the doctor from time to time to keep him updated.
- When taking readings, remember that the daily values are influenced by several factors. Smoking, consumption of alcohol, drugs and physical exertion influence the measured values in various ways.
- Measure your blood pressure before meals.
- Before taking readings, allow yourself at least five minutes rest.
- If the systolic and diastolic readings seem unusual (too high or too low) on several occasions, despite correct use of the unit, please inform your doctor. This also applies to the rare occasions when an irregular or very weak pulse prevents you from taking readings.

3.1 Inserting / removing batteries

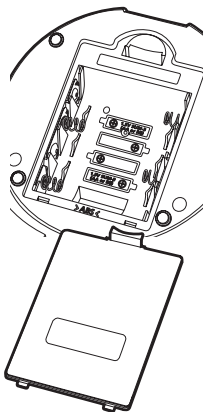Insertion: Open the battery compartment **5** underneath the unit by lightly pressing against the bar and insert the four batteries supplied (batteries, type AA LR 6). Remember to insert the batteries as indicated in the diagram in the battery compartment. Replace the lid, pressing it until it snaps into place.

Removal: When the battery symbol appears in the display, please replace the batteries with new ones. Should the used batteries be jammed in the compartment, use a pointed instrument, such as a ballpoint pen, to carefully prise them out.

After replacing batteries, remember that the time and date have to be reentered. Previously stored results will be lost. Alternatively, you can use the unit with a mains adaptor (MEDISANA Art.–No. 51008) This connects **8** to the socket provided at the back of the unit.


BATTERY SAFETY INFORMATION

- Keep away from children!
- Do not recharge!
- Do not short-circuit!
- Do not throw into fire!
- Do not dispose of old batteries with your household waste, but dispose of them at a battery collection station at a recycling site or in a shop!


3.2 Setting time and date

1. Press the **TIME/SET** button **2** to enter the setting mode for date and time. First of all the entry point for the month flashes (1, 2, 3... 12).
2. Press the **MEMORY** button **3** repeatedly until the desired number for the month appears.
3. Press the **TIME/SET** button **2** again to accept the setting. Now the entry point for the date flashes.
4. Proceed in exactly the same manner as described in point 2 when entering the date, hours and minutes.
5. In each case, press the **MEMORY** button **3** to select and the **TIME/SET** button **2** to accept the entry.
6. On completing the entry, a **0** appears on the display **1** and the unit is ready for a measurement.

4.1 Fitting the cuff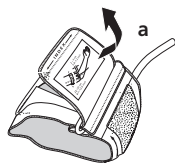


1. Push the end piece of the air tube into the hole on the unit **7** prior to use. The blood pressure monitor is intended for use as a table-top unit.
2. Thread the end of the cuff **6** marked "INDEX" through the metal buckle so that the cuff takes on a cylindrical shape **(a)**. Ensure that the cuff itself is not twisted in any way.
3. Place the cuff around your left upper arm **(b)**. Position the red arterial area in the centre of your arm as an extension of the middle finger **(c)**. Please ensure that the cuff buckle is not positioned directly over the artery, as this can result in an incorrect blood pressure reading. Tighten the cuff to create a snug fit.
4. Place your arm on the table with the palm uppermost. If wearing tight clothing, ensure that the circulation in the arm is not impaired when rolling up the sleeve. The lower edge of the cuff should be 2 -3 cm above the crease of the elbow **(d)**.

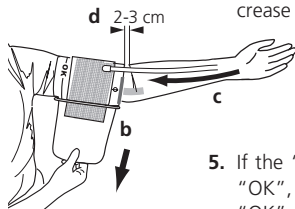5. If the "INDEX" arrow lies within the area on the edge of the cuff marked "OK", the cuff is ready for use. If the "INDEX" arrow is not within the "OK" area, you need a special cuff (MEDISANA Art.-Nr. 51007). Avoid creating a space between arm and cuff. It may falsify the measured result.
6. Only position the cuff on the right arm if it cannot be used on the left arm. Always carry out measurements on the same arm.

4.2 Correct position for taking readings

- Carry out the measurement whilst seated.
- Relax your arm and place it comfortably, on a table for instance. Keep still during the measurement: do not move and do not talk as this may alter the measurements.

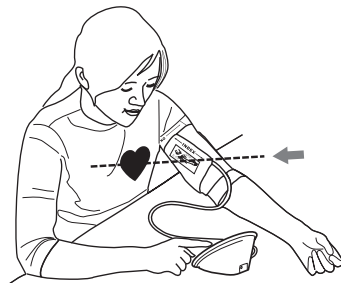4.3 Taking blood pressure readings

1. Switch the unit on by pressing the **ON/OFF/START** button **4**.
2. All the symbols on the display light up when the **ON/OFF/START** button **4** is pressed. This test can be used to check that the display is indicating properly and in full. The display test disappears after approximately 1 second. The figure **0** appears briefly.
3. The unit is ready for measurement. It automatically inflates the cuff **6** slowly in order to measure your blood pressure. The pressure on the display changes continuously during the time it continues to inflate.
4. The unit inflates the cuff until sufficient pressure has built up for a measurement. Then the unit slowly releases air from the cuff and carries out the measurement.
5. When the measurement is finished, the cuff is deflated completely. The systolic and diastolic blood pressure and the pulse value with the pulse symbol appear on the display **1**. The measured values are stored automatically. The memory holds 60 measured values.

4.4 Switching off the unit

The unit has an automatic switch-off function. The unit switches off automatically after a minute if no buttons are pressed.
The unit may also be switched off using the **ON/OFF/START** button **4**.

5.1 Displaying stored results

You can store up to 60 measurements (blood pressure and pulse) in the memory of your blood pressure monitor. The measured values are stored automatically. Press the **MEMORY** button **3** to call up the measured values stored. The stored data appears on the display. The most recently measured value is displayed first. Repeatedly pressing the **MEMORY** button **3** displays the respective values measured previously.
Should all 60 memory cells be occupied, the oldest value is deleted with a new reading.

PLEASE NOTE

The saved measured values are retained even when the unit is switched off.

5.2 Deleting memory

Press and hold the **MEMORY** button **3** for approximately 5 seconds and all measured data stored will be deleted automatically.

6.1 Guide to Display Symbols

Battery spent.
Please insert new battery.

Pulse detection

Indicates the memory slot number

Faulty measurement

Fault in air circuit

Pressure exceeds 300 mmHg

Memory error

6.2 Troubleshooting

The unit has not been able to determine the blood pressure value properly if one of the following error messages appears on the display.

Error Message

Possible Reasons and Solutions

The display fails to show any symbols.

There are no batteries inserted, or they have been wrongly inserted, or they are completely spent. Insert batteries, check their position or replace them with new ones.

The batteries are spent.

Replace all four batteries with new 1.5 V AA type batteries.

- The cuff is applied incorrectly.
Apply the cuff correctly.
- The unit displays extremely low or high values.
Keep still for the duration of the measurement.
- You have spoken or moved during the measurement.
Do not talk.
- The connection to the air tube may have been interrupted or no pulse has been detected.
Check the connection between the upper arm cuff and the unit.

The upper arm cuff is not positioned correctly or the tube connection is not sealed tight.
Check the connections and repeat the measurement.

The value of the pressure measured exceeds 300 mmHg.
Switch the unit off and repeat the measurement.

The batteries are weak.

Replace all four batteries with new 1.5 V AA type batteries.

Please contact the service centre if the error message appears again when the measurement is repeated.

6.3 Care and maintenance

- Remove the batteries before cleaning the unit.
- Never use strong detergents or hard brushes.
- Clean the unit with a soft cloth, moistened with mild soapy water.
- Do not let water enter the unit. After cleaning, only use the unit when completely dry.
- Remove the batteries from the unit if you do not wish to use it for an extended period of time, otherwise there is a risk of battery leakage.
- Do not expose the unit to direct sunlight, and protect it from dust and moisture.
- Only inflate the cuff when it is in position around the upper arm.
- You are advised to store the unit in the storage case acquired upon purchase and to keep it in a clean and dry place.
- **Servicing and calibration**

The unit has been calibrated by the manufacturer for a period of two years' use. In the case of commercial use, the unit must be serviced and recalibrated at least once every two years. This calibration will be charged for and can be carried out by an appropriate authority or an authorised service centre - in accordance with the regulations for users of medical equipment.

6.4 Disposal

Dispose of your old electrical unit in an environmentally acceptable manner when it has reached the end of its service life!

Packaging is re-usable or may be re-cycled; metal parts must be disposed of as scrap metal; plastics, electrical and electronic components must be disposed of as electro-scrap.

Please remove the batteries before disposing of the equipment. Do not dispose of old batteries with your household waste, but dispose of them at a battery collection station at a recycling site or in a shop.

Consult your local council or your dealer for recycling information.

6.5 Directives / Norms

This blood pressure monitor meets the requirements of the EU standard for non-invasive blood pressure monitors. It is certified in accordance with EC Guidelines and carries the CE symbol (conformity symbol) "CE 0297".

The blood pressure monitor corresponds to European standards EN 1060-1/1995 and EN 1060-3/1997.

The specifications of EU Guideline "93/42/EEC of the Council Directive dated 14 June 1993 concerning medical devices" are met and likewise those of EU Guideline 89/336/EEC.

Device classification: type BF 

User group:

The unit is suitable for non-invasive blood pressure measurements on adults (this means that it is suitable for external use).

6.6 Specifications

Name and model	: MEDISANA blood pressure monitor MTA
Display system	: Digital display
Memory slots	: 60
Measuring technique	: Oscillometric
Power supply	: 6 V=, 4 x 1,5 V battery AA LR 6
Blood pressure measuring range	: 40 – 250 mmHg
Pulse measuring range	: 40 – 199 beats / Min.
Maximum error tolerance for static pressure	: ± 3 mmHg
Maximum error tolerance for pulse rate	: ± 5 % of the reading
Pressure generation	: Automatic, using pump
Deflation	: Automatic
Autom. switch-off	: After 1 minute
Operating conditions	: +10 °C to + 40 °C, 40 % – 85 % RH
Storage conditions	: - 10 °C to + 60 °C 10 % – 95 % RH
Dimensions	: 150 mm x 135 mm x 74,45 mm
Cuff size	: 24 – 36 cm Adult cuff for average upper arm circumference
Weight	: Approx. 395 g without batteries
Article number	: 51005
EAN number	: 4015588510052
Accessories	: Mains adapter Art. No. 51008 Cuff, large 32 - 42 cm for adults with upper arm of large circumference Art.-No. 51007

CE 0297

In accordance with our policy of continual product improvement, we reserve the right to technical and design changes without notice.

7.1 Warranty and repair terms

Please contact your dealer or the service centre in case of a claim under the warranty. If you have to send in the unit, please enclose a copy of your receipt and state what the defect is.

The following warranty terms apply:

1. The warranty period for **MEDISANA** products is three years from date of purchase. In case of a warranty claim, the date of purchase has to be proven by means of the sales receipt or invoice.
2. Defects in material or workmanship will be removed free of charge within the warranty period.
3. Repairs under warranty do not extend the warranty period either for the unit or for the replacement parts.
4. The following is excluded under the warranty:
 - a. All damage which has arisen due to improper treatment, e.g. non-observance of the user instructions.
 - b. All damage which is due to repairs or tampering by the customer or unauthorised third parties.
 - c. Damage which has arisen during transport from the manufacturer to the consumer or during transport to the service centre.
 - d. Accessories which are subject to normal wear and tear.
5. Liability for direct or indirect consequential losses caused by the unit are excluded even if the damage to the unit is accepted as a warranty claim.

MEDISANA AG

Itterpark 7-9
D-40724 Hilden
Tel.: +49 (0) 2103 / 2007-60
Fax: +49 (0) 2103 / 2007-626
eMail: info@medisana.de
Internet: www.medisana.de

The service centre address is shown on the last page.

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D&L Marketing Oy
ALERE Electronics AB