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Italiano

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ATMOS S 61 Servant vision

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ATMOS S 61 Servant
vision:
531.0000.D

2014-08 Index: 15

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Manual disinfection of instruments

Disinfettante	Ingredienti	(in 100 g)	Produttore
Korsolex basic (Application concentrate)	Glutaral (ethylene-dioxy)dimethanol tensides, salts, corrosion inhibitors	15.2 g 19.7 g	Bode Chemie, Hamburg
Korsolex extra (Application concentrate)	(ethylene-dioxy)dimethanol Glutaral Benzyl-C12-C18-alkyldimethyl- ammonium chloride, tensides, didecylmethylammonium chloride corrosion inhibitors	15.3 g 7.5 g 1.0 g	Bode Chemie, Hamburg
neodisher Septo MED (Application concentrate)	dodecylbispropylentriamine didecyl dimethyl ammonium chloride non-ionic tensides, fragrances	9.2 g 13.0 g	Dr. Weigert, Hamburg
Sekusept aktiv (Application concentrate)	sodiumpercarbonate, phosphonates non-ionic tensides		Ecolab, Düsseldorf
Gigasept FF new (Application concentrate)	succinic acid dialdehyde dimethoxy tetrahydrofurane corrosion inhibitors	11.0 g 3.0 g	Schülke & Mayr, Norderstedt

Machine disinfection of instruments

Dismoclean 24 Vario (Application concentrate)	tensides, micro-encapsulated enzymes corrosion inhibitors complexing agent		Bode Chemie, Hamburg
Dismoclean twin basic/twin zyme Dismoclean twin basic	alkali dispenser, complexing agent corrosion inhibitors		Bode Chemie, Hamburg
Dismoclean twin zyme	surface active materials, enzymes, stabilisers, corrosion inhibitors		
neodisher MediClean forte (Application concentrate)	non-ionic tensides NTA (nitrilotriacetic acid) enzymes, preservative agent	<5 g 5-15 g	Dr. Weigert, Hamburg
Sekumatic MultiClean (Application concentrate)	phosphates, silicates, potassium hydroxide corrosion inhibitors		Ecolab, Düsseldorf
Thermosept alka clean forte (Application concentrate)	non-ionic tensides anionic tensides, enzymes polycarboxyl, corrosion inhibitors	<5 g <5 g	Schülke & Mayr, Norderstedt

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Disinfettante	Ingredienti	(in 100 g)	Produttore
Dismozon pur (Application concentrate)	magnesium monoperoxyphthalate hexahydrate	80 g	Bode Chemie, Hamburg
Green & Clean SK (Application concentrate)	alkyl-dimethyl-benzyl-ammonium chloride dialkyl-dimethyl-ammonium chloride-	<1 g	Metasys, Rum (Austria)
Perform	Pentakalium bis(peroxymonosulfate)- bis(sulfate)	45.0 g	Schülke & Mayr, Norderstedt

Cleaning and disinfection plan ATMOS S 61 Servant

	What	How			Recommendations	When				Who
	Reusable parts	C Cleaning	D Disinfection	S Sterilisation		After each procedure	Daily	Weekly	Monthly	Qualified and trained staff who are familiar with reprocessing. (Please fill in the responsible person -> use a water-based overhead marker)
Secretion canister										
	Hose connection (nozzle)	X	X ^{2,4,5}		Cleaning and disinfection (automatic or manual)		X			
	Suction lid	X	X ^{2,4,5}		Cleaning and disinfection (automatic or manual)		X			
	Gasket	X	X ^{2,4,5}		Cleaning and disinfection (automatic or manual)		X			
	Bacterial filter				Daily exchange or when filter is blocked		X			
	Splash protection	X	X ^{2,4,5}		Cleaning and disinfection (automatic or manual)		X			
	Float ball	X	X		Cleaning and disinfection (automatic or manual)		X			
	Suction hose in the canister	X	X ^{2,4,5}		Cleaning and disinfection (automatic or manual)		X			
	Secretion collection canister	X	X		Empty when the canister is full; at least daily Cleaning and disinfection (automatic or manual)		X			
	Disposable canister system				Exchange and disposal of full canister		X			
Hose irrigation system										
	Suction nozzle for hose irrigation	X	X ³		Wipe cleaning and disinfection		X			
	Silicone attachment piece	X	X ^{2,4,5,6}		Cleaning and disinfection (automatic or manual)		X			
					Exchange of the Silicone attachment				X	
	Suction nipple	X			Manual cleaning after each patient	X				
				X ^{2,4,5,6}	Manual or automatic cleaning and disinfection		X			
	Secretion suction hose	X			Rinse the secretion hose with the hose irrigation system after each procedure;	X				
				X ^{2,4,5,6}	Exchange or disinfection of the hose				X	
	Storage container for hose irrigation system	X	X ^{2,4,5,6}		Cleaning with a brush; Manual or automatic cleaning and disinfection			X		
Ear irrigation / Thermal nystagmus stimulation										
	Ear irrigation bowl	X	X ^{2,4,5}		Manual or automatic cleaning and disinfection	X				
	Handle	X	X ³		Wipe cleaning and wipe disinfection		X			
	Jet connection	X	X ^{2,4,5,6}		Manual or automatic cleaning and disinfection		X			
	Splash protection	X	X ^{2,4,5}		Cleaning and disinfection (automatic or manual)		X			
	Hose tip (disposable)				Exchange after each application	X				
	Rinsing attachment	X	X ^{2,4,5}		Manual or automatic cleaning and disinfection	X				
	Hygiene filter				See operating instructions for hygiene filter				X	
	Rinsing lid with rinsing hose	X	X ^{2,4,5}		Manual or automatic cleaning and disinfection		X			
	Rinsing bottle	X	X ^{2,4,5,6}		Cleaning and disinfection; cleaning in the dishwasher with the glass care programme		X			
Medication nebulisation / Politzer										
	Handle for compressed air	X	X		Manual cleaning and disinfection		X			
	Sprayer jet	X			Cleaning after each application	X				
				X ^{2,4,5,6}	Manual or automatic cleaning and disinfection		X			
	Sprayer head	X	X ^{2,4,5}		Multiple rinsing of the sprayer head with water			X		
	Hose at sprayer head	X	X		Weekly exchange of the hose or when changing the medication			X		
	Sprayer bottle	X	X ^{2,4,5,6}		Cleaning in a cleaning and disinfection device; weekly or when changing the medication			X		
	Poltizer olive	X	X ^{2,4,5,6}		Exchange after each application. Cleaning and disinfection	X				
	Poltizer connection	X	X ^{2,4,5,6}		Exchange after each application. Cleaning and disinfection	X				
Endoscope management										
	Plastic quiver	X	X ^{2,4,5}		Cleaning with a brush; disinfection		X			
	Metal quiver	X	X ^{2,4,5,6}		Cleaning with a brush; disinfection (automatic or manual)		X			
	Fixation adapter for plastic quiver	X	X ^{2,4,5}		Manual or automatic cleaning and disinfection		X			
	Protective sleeve (teflon element for metal quiver)	X	X ^{2,4,5}		Manual or automatic cleaning and disinfection		X			

	What	How			Recommendations	When				Who
	Reusable parts	C Cleaning	D Disinfection	S Sterilisation		After each procedure	Daily	Weekly	Monthly	Qualified and trained staff who are familiar with reprocessing. (Please fill in the responsible person -> use a water-based overhead marker)

Instrument management

ENT instruments	X	X ^{2,4,5}	X	Immerse instruments into solution immediately after use, complete wetting is required, air must be removed from any cavities, after the contact time instruments must be rinsed with water, have to be dried and sterilised afterwards. Please also observe the ATMOS operating instructions for ENT instruments.	X				
Instrument bowl	X	X ^{2,4,5}		Manual or automatic cleaning and disinfection		X			
Instrument bowl with cover	X	X ^{2,4,5}		Cleaning with a brush; afterwards disinfection Manual or automatic cleaning and disinfection		X			

Visualisation

ATMOS Cam 21 / 31	X	X ³		Wipe cleaning and wipe disinfection	X				
ATMOS Strobe 21 LED	X	X ³		Wipe cleaning and wipe disinfection		X			
Flexible scope	X	X ¹⁾	X ¹⁾	Immediate pre-cleaning after the procedure	X				
Rigid scope	X	X ¹⁾	X ¹⁾	Immediate pre-cleaning after the procedure	X				
Laryngoscope	X	X ¹⁾	X ¹⁾	Immediate pre-cleaning after the procedure	X				
Light conductor	X	X ³		Wipe cleaning and wipe disinfection		X			
Light source	X	X ³		Wipe cleaning and wipe disinfection		X			
Microscope	X	X ³		Wipe cleaning and wipe disinfection		X			
Headlight	X	X ³		Wipe cleaning and wipe disinfection		X			

Radiofrequency surgery

ATMOS RS 221 (surface)	X	X ³		Wipe cleaning and wipe disinfection		X			
Ergonomic plastic handles	X	X ^{1,2,4,5}	X ¹⁾	Wipe cleaning and wipe disinfection	X				
Bipolar tweezers	X	X ^{1,2,4,5}	X ¹⁾	Immediate pre-cleaning after the procedure cleaning and disinfection (automatic or manual); use of enzymatic detergents	X				
Bipolar electrode	X	X ^{1,2,4,5}	X ¹⁾		X				
Bipolar electrode cable	X	X ^{1,2,4,5}	X ¹⁾	Immediate pre-cleaning after the procedure cleaning and disinfection (automatic or manual); use of enzymatic detergents	X				
Neutral electrode	X	X ^{1,2,4,5}	X ¹⁾	Immediate pre-cleaning after the procedure cleaning and disinfection (automatic or manual); use of enzymatic detergents	X				
Neutral electrode cable	X	X ^{1,2,4,5}	X ¹⁾		X				
ENT electrodes	X	X ^{1,2,4,5}	X ¹⁾	Immediate pre-cleaning after the procedure cleaning and disinfection (automatic or manual); use of enzymatic detergents	X				

Surfaces

Housing	X	X ³		Wipe cleaning and wipe disinfection		X			
Roller shutter	X	X ³		Wipe cleaning and wipe disinfection		X			
System frame	X	X ³		Wipe cleaning and wipe disinfection		X			
Drawers	X	X ³		Wipe cleaning and wipe disinfection		X			
Writing surface	X	X ³		Wipe cleaning and wipe disinfection		X			
Instrument deposit	X	X ³		Wipe cleaning and wipe disinfection		X			
Mirror pre-heater	X	X ³		Wipe cleaning and wipe disinfection		X			
Tongue patches and swab dispenser	X	X ³		Wipe cleaning and wipe disinfection, every day or when refilling		X			
Waste disposal	X	X ³		Wipe cleaning and wipe disinfection, every day or when emptying the container		X			
Instrument tray	X	X ³		Wipe cleaning and wipe disinfection, daily when replacing with new instruments		X			

Recommended disinfectants

³⁾ Surface disinfection for coated surfaces:

- Green & Clean SK (ATMOS)
- Dismozon[®] pur (Bode Chemie)
- Kohrsolin[®] FF (Bode Chemie)
- Perform[®] (Schülke & Mayr)
- Terralin[®] Protect (Schülke & Mayr)

Other surfaces:

- Dismozon[®] pur (Bode Chemie)
- Kohrsolin[®] FF (Bode Chemie)
- Bacillocid[®] rasant (Bode Chemie)
- Mikrobac[®] forte (Bode Chemie)
- Perform[®] (Schülke & Mayr)
- Terralin[®] Protect (Schülke & Mayr)
- Surface disinfection FD 312 (Durr Dental)
- Quick disinfection B 30 (Orrochemie)

⁴⁾ Instruments - manual disinfection:

- Korsolex[®] AF (Bode Chemie)
- Korsolex[®] basic (Bode Chemie)
- Korsolex[®] plus (Bode Chemie)
- Korsolex[®] extra (Bode Chemie)
- neodisher[®] Septo MED (Dr. Weigert)
- neodisher[®] Septo 3000 (Dr. Weigert)
- Sekusept[®] PLUS (Ecolab)
- Sekusept[®] aktiv (Ecolab)
- Gigasept[®] Instru AF (Schülke & Mayr)
- Gigazyme[®] (Schülke & Mayr)
- Gigasept FF neu (Schülke & Mayr)

⁵⁾ Instruments - automatic disinfection:

- Dismoclean[®] 21 clean (Bode Chemie)
- Dismoclean[®] 24 Vario (Bode Chemie)
- Dismoclean[®] 28 alka one (Bode Chemie)
- Dismoclean[®] twin basic/twin zyme (Bode Chemie)
- neodisher[®] FA (Dr. Weigert)
- neodisher[®] FA forte (Dr. Weigert)
- neodisher[®] MedClean forte (Dr. Weigert)
- Thermosept[®] alka clean forte (Schülke & Mayr)
- Thermosept[®] RKN-zym (Schülke & Mayr)

Please see the manufacturer's instructions for concentration, contact time, temperature and the compatibility of materials.

Important information

Wipe cleaning and wipe disinfection: All surfaces have to be wiped with a clean (disposable) wipe which is damped with disinfectant solution; the entire surface has to be wiped thoroughly and may not be dried afterwards.

¹⁾ Please observe the manufacturer's operating instructions.

²⁾ Alternative to manual cleaning and disinfection: Wash-Disinfector 78°C / 172°F

⁶⁾ Material dimensionally stable at 134°C

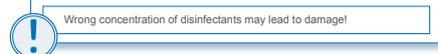
The above stated hygiene requirements are based on the regulations according to the Medical Devices Act, the Medical Devices Operator Ordinance, §18 IfSG and the recommendations of the Robert Koch Institute. Definition of the required reprocessing steps result from the recommendations of the Robert Koch Institute: „Requirements for the reprocessing of medical products“. The medical products were categorised in the risk groups uncritical, semicritical and critical. The reprocessing measures mentioned in this cleaning and disinfection plan are a recommendation of ATMOS MedizinTechnik. Any additional reprocessing measures are at the operator's discretion.

All the recommended disinfectants which are stated herein are listed disinfectants (VAH/RKI) and have been tested on their suitability of use on the ATMOS S 61 Servant. ATMOS MedizinTechnik cannot be held liable for any damage caused by wrong concentration of the disinfectants or by the application of any other disinfectants.

Patients with suspicion of a clinical disease or who developed a transmissible spongiform encephalopathy (CJK, vCJK, etc.) have to be treated at facilities which are able to provide for the necessary preventive measures against infection. The reprocessing of the reusable instruments and material may only be performed at facilities which have an externally certified QM Management acc. to DIN EN ISO 13485.

The Medical Devices Act, IfSG, the RKI directives, BGR 250 and TRBA 250 always have to be considered.

ATMOS MedizinTechnik GmbH & Co. KG
Ludwig-Kegel-Str. 16 ■ 79853 Lenzkirch/Germany
Phone +49 7653 689-0 ■ Fax +49 7653 689-190
atmos@atmosmed.de ■ www.atmosmed.de

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1. General:

Our General Standard Terms and Conditions apply exclusively. Client's terms and conditions which are contrary to or deviate from our General Standard Terms and Conditions are not recognised unless their validity is explicitly confirmed in writing. Our General Standard Terms and Conditions also apply even if we deliver to clients without reservation, in the knowledge of the client's contrary terms and conditions. Our General Standard Terms and Conditions also apply to all future business with that client.

2. Proposal - Order Confirmation

Our proposals are subject to change without notice unless otherwise stated in our order confirmation. Each order is only accepted by us following our written order confirmation.

3. Orders

Every order requires an exact description of all of our product's details. We assume no liability for errors and damage caused by inaccurate or incomplete ordering details.

4. Prices

Unless otherwise stated in the order confirmation, our prices in the order confirmation are ex factory prices and exclude packaging and value added tax. Packaging is charged separately at cost price in the invoice. Value added tax is charged separately in the invoice according to the legal rate on the invoice date. We reserve the right to change prices appropriately should price reductions or increases, especially due to wage settlements, changes in the price of materials or currency fluctuations, be incurred. Proof of such changes will be provided for the client on request.

5. Payment Conditions - Balancing

Unless otherwise stated in the order confirmation, our invoices are payable with a 3% discount within 10 days (except for repair and assembly services) or within 21 days from the invoice date net cash; money receipts is decisive for complying with this term. We are entitled to charge interest after the due date at a rate 2% above the relevant basic interest rate of the German Federal Bank. Should the client have payment arrears, we are entitled to charge interest on arrears at a rate 5% above the relevant basic interest rate of the German Federal Bank. Should we be able to prove higher damages due to arrears, we are also entitled to claim these. The client only has the right to balance invoices against its own claims should such claims be confirmed in a court of law or recognised by us. The client does not have the right of retention due to disputed counterclaims.

6. Delivery Periods

Fulfillment of our delivery duties requires the punctual and proper fulfillment of the client's duties. The right to defense on the grounds of an unfulfilled contract is reserved. Should the client default in accepting the goods delivery or breach other cooperation duties, we are entitled either to withdraw from the contract or claim compensation for any increased costs incurred up to that time without setting a further deadline. The right to make further claims is reserved. Furthermore, in such cases, the risk of coin-cidental destruction or a coincidental deterioration in the quality of the delivered goods is transferred to the client in the case of default in accepting such goods or payment arrears. Acts of God or stoppages (due to insufficient supplies of material, industrial disputes etc.) entitle us either to demand an appropriate extension of delivery periods or to partly or entirely dissolve the delivery contract. This does not give the client the right to claim damages. We have fulfilled delivery periods if the delivery goods have left our factory or the client has been informed of the goods' readiness for delivery within such delivery periods. Delivery periods stipulated by the client are not recognised by us unless they form part of our order confirmation. We adhere to legal terms and conditions in cases where, as a result of an undue delay in the delivery for which we are liable, the client is entitled to claim that his interests in a continued fulfillment of the contract have ceased. We also adhere to legal terms and conditions should a delay in delivery be caused by deliberate or grossly negligent action by us or our representatives for which we are responsible. We are also responsible for such actions by our representatives or agents. Should the delivery delay not be caused by our deliberate infringement of contractual duties for which we are responsible, our liability is limited to damage which is regarded as typical for that case. We are liable according to the legal terms and conditions if and in so far as the delivery delay for which we are responsible is caused by an infringement of a substantial contractual duty. In such cases, our liability is also limited to damage which is regarded as typical for that

case. Should the delivery delay be caused by a culpable infringement of non-substantial contractual duties, our client is also entitled to claim a one-off damage compensation worth 3 percentage points of the delivery value of the goods for each week's delay, up to a maximum which is no higher than 15 percentage points of the delivery value of the goods

7. Delivery - Familiarisation

In the case of the delivery of devices for the medico-technical industry which require assembly and/or familiarisation for the final customer using specialist trade personnel (such as Ear, Nose and Throat Apparatus and Suction Units), we reserve the right to deliver the goods exclusively to the relevant specialist traders. Should the trader not carry out assembly and/or familiarisation for the final customer, this is carried out by us. In such cases, we reserve the right to charge the client for the additionally created costs. Our specialist traders operate a recording system so that, if necessary, our products can be traced to the final customer. The specialist trader undertakes to immediately report to us all events and risks which must be reported in connection with our products.

8. Passage of Risk - Packaging

Unless otherwise stated in our order confirmation, delivery is agreed ex factory. The risk of the goods' damage or loss is therefore transferred to the client as soon as the goods leave the factory or the client is in default of acceptance of the goods. This also applies to cases where we confirm prepaid carriage. Transport packaging and all other packaging according to the packaging regulations is not returnable. Our client is responsible for disposing the packaging at its own cost. Our deliveries are insured by us at the client's expense unless explicitly otherwise agreed. No insurance is arranged in the case of goods which are collected by our clients. In the case of transport damage, claims are only handled if the client receives confirmation of any damage, reduced weight or loss by the shipping company before accepting the delivery.

9. Warranty

The client is responsible for examining the delivered goods immediately after receiving them to determine any eventual deficiencies or delivery errors, and to report these immediately. Should the client fulfil this examining and reporting responsibility, and should payment conditions be fulfilled, we shall be liable to the client within the scope of legal regulations. Our period of warranty shall in all cases be two years. Our client can make use of the warranty as follows, so long as he can provide first buyer proof (in the form of an invoice or delivery note) and provided that the product still has the original, unchanged serial number:

- We choose whether to fulfil our guarantee by providing repair services free of charge - either on the client's premises or in our factory - or replacing the product. We can also provide these guarantee services through an authorised company;
- Should a product be returned to us, the client agrees to send the product in its original or similar packaging, offering the same protection as the original packaging, to our address or any address notified by us.
- Our guarantee ceases to apply if changes of any kind have been made to our product, unless such changes have been made by us or a company authorised by us, or have been previously agreed upon in writing by us. Our guarantee also ceases to apply if third parties have carried out repairs to our products or replaced parts thereof. This applies regardless of the fact whether these measures individually or collectively led to a deficiency of the product;
- We accept no responsibility for damage defects caused by
 - operational wear and tear;
 - incorrect installation or incorrect or insufficient maintenance;
 - incorrect operation of the product (in contradiction to the operating instructions delivered with the product);
 - improper use or operating faults;
 - inappropriate or negligent handling and care, especially with respect to dirt, lime, suction of fluids, inappropriate cleaning and sterilisation;
 - using accessories and/or replacement parts which are not explicitly approved;
 - incorrect assembly and/or initial operation by the client or third parties;
 - the client's negligence in handling the product;
 - unacceptable operating conditions, such as humidity, temperatures, the power supply, vibrations.
 - accidents, acts of God, especially lightning, water, fire, public unrest and insufficient ventilation. We are not liable for damage to other objects apart from our product itself, except in the case of any deliberate or grossly negligent actions by us or our representatives or agents. Should no deliberate breach of contract be claimed, our liability

is limited to damage which is regarded as typical for that case. This also applies in the case of our culpable infringement of substantial contractual duties. The indispensable conditions of German Liability Law remain unaffected thereby.

- For second-hand equipment, the period of warranty shall be reduced to a period of twelve months.

10. Reservation of Ownership

We retain ownership of our goods until the receipt of all payments arising from the business relationship, including all demands arising from installation orders, subsequent orders, repairs, accessory deliveries and replacement orders. Should we have agreed upon payment on the basis of cheque and bill transactions, the ownership reservation applies until the cheque received by us has been paid in, and does not expire through our credit upon receiving the client's cheque. In the case of a breach of contract by the client, especially payment arrears, we are entitled to repossess our goods. Repossession of our goods represents a withdrawal from the contract, unless explicitly declared in writing by us. We have the right to utilise the product after its repossession, whilst the income from such use is balanced against the client's arrears, after deducting appropriate utilisation costs. The client is responsible for handling the goods with care. Should maintenance and inspection work be necessary, the client must carry these out punctually at his own cost. Our client is entitled to sell the goods he has bought from us in a proper sale transaction. However, he must immediately assign all outstanding claims to the value of the final invoice sum (including value added tax) of our claims to his customers or third parties. The client is entitled to collect this claim even after such assignment. Our right to collect the claim ourselves remains unaffected thereby. We undertake to release the securities to which we are entitled if requested to do so by the client should the realisable value of the our securities be more than 10 percentage points higher than the outstanding claims. We reserve the right to choose the securities to be released.

11. Plans and Illustrations

We retain ownership of and copyrights to all plans, illustrations, calculations and other documents which are attached to our proposals. The client must receive explicit written permission before passing these on to third parties. Imitating our legally patented products is forbidden and will be prosecuted.

12. Jurisdiction and Place of Performance

Our central office is the place of performance for all disputes in connection with these General Standard Terms and Conditions and the contracts closed with clients under them. This jurisdiction excludes other jurisdiction relating to persons or subject-matter. Furthermore, our client is not entitled to bring charges against us in another court should he file counter-charges, carry out counterbalancing or declare retention. We, however, are entitled to bring charges against our client at their general place of jurisdiction or at another relevant court recognised by German or foreign law. Unless otherwise stated in the order confirmation, our central office is the place of performance.

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