317L20

DE Gebrauchsanweisung	2
EN Instructions for Use	5
FR Instructions d'utilisation	8
IT Istruzioni per l'uso	11
ES Instrucciones de uso	14
PT Manual de utilização	17
NL Gebruiksaanwijzing	20
SV Bruksanvisning	23
DA Brugsanvisning	26
NO Bruksanvisning	29
PL Instrukcja użytkowania	31
HU Használati utasítás	34
CS Návod k použití	37
TR Kullanma talimatı	40
EL Οδηγίες χρήσης	43
RU Руководство по применению	46

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Date of the last update: 2013-03-12

- Please read this document carefully.
- Follow the safety instructions.

Before using the pulse charger, please read these instructions as well as those for the 317B20 Interchangeable Battery. No liability will be assumed if the pulse charger is operated incorrectly or used for purposes other than as originally intended.

1 Intended Use

Use the **317L20** Pulse Charger **exclusively** for charging **317B20** Ottobock Interchangeable Batteries.

2 Safety Instructions

Attention!

Risk of short circuit

- Do not use the charger if there is any visible damage. Check the pulse charger for any visible damage before every use. In case of damage immediately send the charger to an Ottobock Service facility.
- Never open the pulse charger. The pulse charger as well as individual components of it may only be repaired by an authorized Ottobock Service technician.
- The charging station may be operated only together with the 757L16-2 AC-Adapter.
- Use the pulse charger only under dry conditions.
- Do not let foreign particles or fluids enter the pulse charger.
- Do not expose the pulse charger to excessive smoke, dust, mechanical vibrations, shocks or high temperatures.

3 Scope of Delivery

- 1 pc. Charging Station
- 1 pc. 757L16-2 AC-Adapter with EC and US plug
- 1 pc. Instructions for Use

4 Description and Function

With the 317L20 Pulse Charger you can charge one or two interchangeable batteries simultaneously. The battery charger consists of the charging station and the 757L16-2 AC-Adapter.

The interchangeable batteries are charged with a pulse-shaped current that is automatically reduced to a trickle charging current once the voltage reaches a certain upper limit. The standby current is reduced to a minimum. This improves efficiency and lengthens life expectancy of the interchangeable batteries. Two LEDs give information regarding the charging state of the batteries.

5 Putting into Operation

Attention!

Continuous charging over several days can damage the interchangeable battery.

Avoid charging processes of more than 12 hours and remove the interchangeable battery from the battery charger after every charging process, if possible.

1. Connect the AC-adapter to the charging station and plug the AC-adapter into an electrical outlet.
2. Place one or two interchangeable batteries into the charger.

The charger is ready to use and the charging process will begin when the LEDs corresponding to each battery light up (refer to sections 6 and 7).

Charging time for one empty interchangeable battery is between 8 to 10 hours (refer to section 11). When the interchangeable battery is completely charged the current is reduced to a trickle charging current. The interchangeable batteries may therefore remain in the charger for several hours. Charging ceases when the interchangeable batteries are removed from the charger.

6 Light Emitting Diodes

The LEDs show charging and operating states of the batteries.

Signal	Signification	Recommendation
<i>Permanent light</i>	Interchangeable battery is charging	Leave the interchangeable battery in the charging station
<i>Regular flashing (On/Off → equal interval)</i>	Fast flashing	Charging has not yet been completed and must be continued
	Slow flashing	Interchangeable battery is fully charged and can be removed

7 Disruptions of the Charging Function and Possible Causes

If your interchangeable battery does not start charging, please verify the following possible causes.

Signal	Cause	Recommendation
<i>No LED is lit when putting the charger into operation</i>	Battery charger/ AC-adapter not connected	Connect AC-adapter to battery charger/ battery charger to electrical outlet
	Defective battery charger	Return the battery charger to an Ottobock Service facility
<i>Two interchangeable batteries inserted, but only one LED is lit</i>	Change interchangeable battery positions!	Return the battery charger to an Ottobock Service facility
	a) The same LED lights up: defective battery charger b) The other LED lights up: defective interchangeable battery	Return the interchangeable battery to an Ottobock Service facility

Signal	Cause	Recommendation
<i>Flashing begins immediately after a completely discharged interchangeable battery is inserted in the charger</i>	Defective interchangeable battery	Return the interchangeable battery to an Ottobock Service facility
<i>No flashing after approximately 8 hours of charging</i>	Defective interchangeable battery	Return the interchangeable battery to an Ottobock Service facility
<i>LED is lit although the battery has been removed</i>	Defective battery charger	Return the battery charger to an Ottobock Service facility

8 Cleaning Instructions

- Keep the electrical contacts of interchangeable batteries and battery charger clean and free of grease. Clean with dry cotton swabs.
- For cleaning the surface use a *slightly* moistened cloth.

9 Liability

Otto Bock Healthcare GmbH, hereafter referred to as manufacturer, assumes liability only if the user complies with the processing, operating and maintenance instructions as well as the service intervals. The manufacturer explicitly states that this device may only be used in combination with components that were authorised by the manufacturer (see instructions for use and catalogues). The manufacturer does not assume liability for damage caused by component combinations which it did not authorise.

The device may only be opened and repaired by authorised Ottobock technicians.

10 CE Conformity

This device meets the requirements of the 93/42/EEC guidelines for medical devices. This device has been classified as a class I device according to the classification criteria outlined in appendix IX of the guidelines. The declaration of conformity was therefore created by Ottobock with sole responsibility according to appendix VII of the guidelines.

11 Technical Data

Housing:	impact and break resistant plastic
Dimensions:	100 × 74 × 46 mm (length × width × height)
Design:	in conformity with relevant prescriptions
Operating voltage:	100 - 240 V
Charging current for 317L20:	pulse-shaped, average DC current approx. 250 mA, reduction to trickle charge current
Weight:	190 g
Charging time 317B20:	approx. 10 hours for completely discharged batteries
Operating temperature:	0 – 40 °C

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Otto Bock HealthCare GmbH