
ottobock.

3R38, 3R39

DE	Gebrauchsanweisung	5
EN	Instructions for Use	8
FR	Instructions d'utilisation	12
IT	Istruzioni per l'uso	15
ES	Instrucciones de uso	19
PT	Manual de utilização	22
NL	Gebruiksaanwijzing	26
SV	Bruksanvisning	29
DA	Brugsanvisning	33
HU	Használati utasítás	36
CS	Návod k použití	39
RO	Instrucțiuni de utilizare	43
HR	Upute za uporabu	46
BS	Uputstvo za upotrebu	50

3R38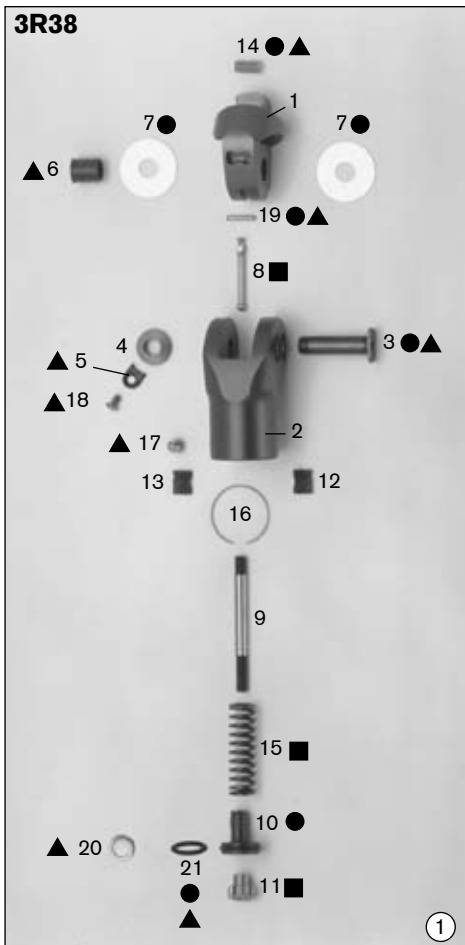

● **4D15**

Einzelteile-Pack
Single-Component Pack
Pack de pièces détachées
Kit componenti
Kit de componentes
Conjunto de componentes
Onderdelenpakket
Service-Set
Komponentpakke
Komponentpakke
Zestaw naprawczy
alkatrészsomag
Set de piese de schimb
Σετ μεμονωμένων εξαρτημάτων
Комплект деталей
交換パーツセット
維修組件


Mindestmenge
Minimum order quantity
Quantité minimum
Quantità minima per l'ordine
Cantidad mínima
Quantidade mínima
Minimikvantitet
Minimum aantal
Mindste mængde
Mindste mængde
Ilość minimalna
Minimális mennyiség
Cantitate minimă la o comandă
Ελάχιστη ποσότητα
Минимальное количество
最低注文個数あり
最低起订量

Einzelteile
Single components
Pièces détachées
Singoli componenti
Componentes
Componentes
Onderdelen
Separata delar
Komponenter
Komponenter
Podzespolny
alkatrészek
Componente
Μεμονωμένα εξαρτήματα
Отдельные детали
個別販売
单个部件

3R39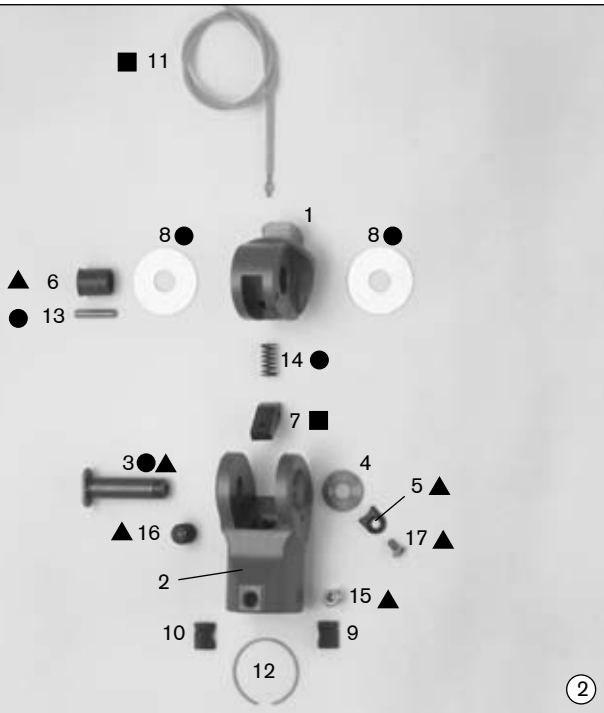

● 4D20


Einzelteile-Pack
Single-Component Pack
Pack de pièces détachées
Kit componenti
Kit de componentes
Conjunto de componentes
Onderdelenpakket
Service-Set
Komponentpakke
Komponentpakke
Zestaw naprawczy
alkatrészcsomag
Set de piese de schimb
Σετ μεμονωμένων εξαρτημάτων
Комплект деталей
交換パーツセット
维修组件

Mindestmenge
Minimum order quantity
Quantité minimum
Quantità minima per l'ordine
Cantidad mínima
Quantidade mínima
Minimikvantitet
Minimum aantal
Mindste mængde
Mindste mængde
Ilość minimalna
Minimális mennyiség
Cantitate minimă la o comandă
Ελάχιστη ποσότητα
Минимальное количество
最低注文個数あり
最低起订量

Einzelteile
Single components
Pièces détachées
Singoli componenti
Componentes
Componentes
Onderdelen
Separata delar
Komponenter
Komponenter
Podzespolny
alkatrészek
Componente
Μεμονωμένα εξαρτήματα
Отдельные детали
個別販売
单个部件

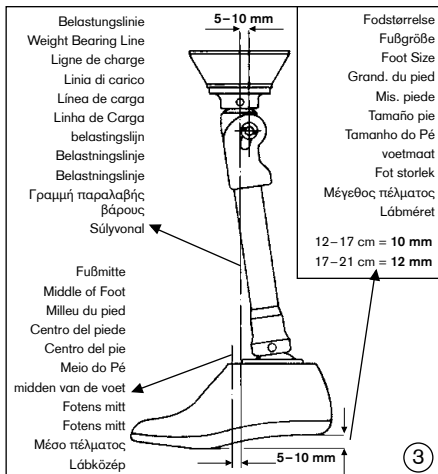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English

These instructions for use were last updated on 2013/06/13.

Please read these Instructions for Use carefully. Pay special attention to safety instructions!

1 Components – Overview

Components 3R38 (fig. 1)

- | | |
|-------------------------------------|---|
| (1) 4G131 Upper Joint Section | (12) 4Z56=L Clamping Wedge |
| (2) 4G132 Lower Joint Section | (13) 4Z56=R Clamping Wedge |
| (3) 4A59 Joint Axis | (14) 4Z59=5x10 Knee stop |
| (4) 4A60 Nut for Joint Axis | (15) 513D48 Spring for Extension Assist |
| (5) 4A61 Safety Plate | (16) 513S2 Lock Ring |
| (6) 4B90=12-K Bearing Bushing | (17) 506G1=M6x8 Set crew |
| (7) 4Z49=25 Bearing Washers | (18) 501S27=M3x6 Oval Counter-sunk Head Screw |
| (8) 4Z52=1 Pin for Extension Assist | (19) 506A8=2x12 Straight Pin |
| (9) 4Z53 Guide Pin | (20) 507U23 Washer |
| (10) 4Z54 Guide Sleeve | (21) 627F1=9.5x2.5 O-Ring |
| (11) 4Z55 Adjustment Nut | |

Components 3R39 (fig. 2)

- | | |
|-------------------------------|---|
| (1) 4G137 Upper Joint Section | (10) 4Z56=R Clamping Wedge |
| (2) 4G138 Lower Joint Section | (11) 10Y22=1 Cable, complete |
| (3) 4A59 Joint Axis | (12) 513S2 Lock Ring |
| (4) 4A60 Nut for Joint Axis | (13) 506A16 Straight Pin |
| (5) 4A61 Safety Plate | (14) 513D50 Compression Spring |
| (6) 4B90=16 Bearing Bushing | (15) 506G1=M6x8 Set Crew |
| (7) 4G139 Lock Cam | (16) 506G3=M8x10 Set Crew |
| (8) 4Z49=25 Bearing Washers | (17) 501S27=M3x6 Oval Counter-sunk Head Screw |
| (9) 4Z56=L Clamping Wedge | |

1.1 Single-Component Packs

4D15 Single-Component Pack for 3R38 (fig. 1, ●)

Consisting of: 2 bearing washers, 1 joint axis, 1 O-ring, 1 guide sleeve, 1 straight pin, 2 knee stops.

4D20 Single-Component Pack for 3R39 (fig. 2, ●)

Consisting of: 2 bearing washers, 1 joint axis, 1 straight pin, 1 compression spring.

2 Application and Description

The 3R38 Modular Single Axis Knee Joint and the 3R39 with Lock are to be used **exclusively** for children's prosthetic fitting of amputations of the lower limb (fig. 1).

The use of these components is limited to patients up to 145 cm (4'9") tall with foot lengths up to 21 cm and weighing up to 45 kg (99 lbs).

The knee axis (4) is placed 8 mm posterior to the weight bearing line and provides connection between the upper joint section (1) and the lower joint section (2) in which the extension mechanism is accommodated.

The integrated lock mechanism secures the 3R39 Joint in extended position. By means of the lock cord, coming out from the upper joint section, the lock is released and flexion may take place.

Connection to proximal (to the socket) is provided by means of the adjustment pyramid (3), to distal it takes place by means of the clamp attachment in the lower joint section where the modular tube is inserted.

3 Alignment (fig. 3)

4 Adjustment of the Joints

4.1 Adjusting tension of the extension assist

Spring tension is adjusted by turning the adjustment nut (5) (fig. 4).

Turning clockwise = increased tension

Turning counterclockwise = decreased tension

4.2 Adjusting the lock mechanism in 3R39

The 3R39 Modular Knee Joint has been factory inspected and set for immediate use. The lock mechanism is adjustable with the screw. For adjustment a 4 mm Allen wrench should be used (fig. 5).

Turn stop screw clockwise for more play.

Turn stop screw counterclockwise for less play.

Attention!

After adjustment by means of the stop screw has taken place, make sure that the lock catches and that the joint is reliably locked. Slight clearance does not result in functional disadvantages.

Note

Use of the 710D1 Torque Wrench is strongly recommended.

Torque of clamping screw: 6 Nm/4.4 ft. lbf./53 lbf. in. (fig. 6)

Torque of adjustment screws 506G3=M6

- in the Tube Adapter 2R41=2: 7 Nm/5.2 ft.lbf./62 lbf.in.

- in the Tube Adapter 3R41=1: 9 Nm/6.6 ft.lbf./80 lbf.in.

- in the Tube Adapter 2R48: 7 Nm/5.2 ft.lbf./62 lbf.in.

Secure the screws by means of 636K13 Loctite.

Attention!

To eliminate noise in the cosmetic foam cover, use 519L5 Silicone Spray. **Do not use talcum powder!** Talcum powder reduces the lubrication of the mechanical parts, which may lead to a malfunction and thus increase the risk of falling. Using this medical product after application of talcum powder will render all claims against Otto Bock HealthCare GmbH null and void.

Apply Silicone Spray 519L5 directly on the friction surfaces of the cosmetic foam cover!

5 Servicing

Due to the extreme stresses that children place on the components, regular inspections and servicing by the prosthetist are necessary at pre-determined intervals. **We recommend that the components be inspected every 3 months.** Function should be tested and the tube adapters examined for any evidence of deformities or damage.

6 Liability

The manufacturer recommends using the product only under the specified conditions and for the intended purposes. The prosthesis must only be operated with tested modular components and maintained according to the Instructions for Use. The manufacturer is not liable for damage caused by component combinations that were not authorised by the manufacturer.

7 CE conformity

This product meets the requirements of the 93/42/EWG guidelines for medical products. This product has been classified as a class I product according to the classification criteria outlined in appendix IX of the guidelines. The declaration of conformity was therefore created by Ottobock with sole responsibility according to appendix VII of the guidelines.

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