# movita by elmi

< therapy and relax

movita child

movita junior

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Dear Customer,

We would like to thank you for your trust and for the purchase of our products.

**movita** is outstanding for its quality, its safety, its simple use and its modern design. In order to use this item properly, please read these user's instructions very carefully.

**Dear Customer,  
please read this user's manual carefully before using movita for the first time!**

## **INDEX**

Safety instructions . . . . .	27
Warranty and product liability . . . . .	28
Intended use . . . . .	28
Components . . . . .	29
Adjustment and operation . . . . .	30
Assembly and mounting of accessories . . . . .	34
Technical data . . . . .	35
Recommendations for cleaning . . . . .	36
Maintenance . . . . .	36
Spare parts . . . . .	36

**SAFETY INSTRUCTIONS**

**movita** is provided with the CE-marking and the TÜV and conforms to the requirements as laid down by the Council Directive 93/42/EEC. The standing-and-walking support **movita** has been developed following the latest regulations. The manufacturing process emphasizes continuous quality control.

**Weight limit:**

<b>movita child I</b>	60 kg max user weight
<b>movita child II</b>	60 kg max user weight
<b>movita junior</b>	130 kg max user weight
<b>movita adult medium</b>	130 kg max user weight
<b>movita adult large</b>	130 kg max user weight

**Precautions!**

- **movita** should only be used if a qualified person or a family member is present.
- **movita** should only be used by people who are able to control the movements of the standing and walking support.
- **movita** is suitable for indoor and outdoor use.
- Please remove any objects from furniture that can be knocked over or fall down while using **movita**.
- **movita** can only be used by people who are familiar with the instructions and applications of this user manual.
- Please check that in the user's area there are no obstacles, such as stairs, steps, or items of any kind lying on the floor that could cause the **movita** to overturn.
- Do not carry out any movements or interference, if you are not absolutely sure of their effects and consequences; in case of the slightest doubt please contact the nearest service centre or the manufacturer directly.
- The application of the use of **movita** must be carried out with the support of qualified personnel.
- Pay attention to possible points of danger! The patient can move very quickly.
- The use of **movita** and its accessories should be done under medical supervision
- Do not use **movita** on uneven ground, on slopes or on stairs.
- The use of **movita** by unauthorized people -even if it is only for playing -is forbidden.
- When stopping always block the brakes, even on level surfaces.
- Before each regulation always block the brakes.
- Before every use, check that the brakes function correctly; always check the correct functioning of the brakes, the casters have not become worn, and that all the accessories and components that could get loose are adjusted and properly tightened.
- Never leave the patient unattended with the standing-and-walking support **movita**.
- Do not systemize or leave **movita** on steep and uneven ground.
- Do not use the standing-and-walking support if parts are faulty or missing.
- Before use, check that all the fasteners are tightened.
- **movita** should be cleaned as described in the "cleaning" chapter.
- In case of an accident or defects, contact your dealer who delivered the walking support.
- Please keep these user's instructions for further consultation.
- Service should be performed annually on the product to assure long-term use by avoiding damage to the product and the individual.
- An authorized person should carry out the servicing.

## **WARRANTY AND PRODUCT LIABILITY**

Elmi GmbH/srl grants a two years' warranty on **movita**.

Only original spare parts and accessories can be used. The guarantee becomes void if repairs or changes to the **movita** or on its components are undertaken by unauthorized persons. Every product responsibility of the manufacturer is in this case excluded beforehand.

**The manufacturer is not responsible for damages to persons, animals or things in the following cases:**

- improper use of the standing-and walking support
- use of the support by unauthorized persons
- inappropriate applications of parts or accessories
- changes or interventions not previously authorized by the supplier
- usage of non-original spare parts
- non-observance of the instructions in this user's manual
- exceptional incidents

## **USE**

**movita** is suitable for physically handicapped children and adults.

**movita has been designed and built:**

- to allow self initiated movement , if independent walking is not possible
- to develop and train independent walking. The standing-and walking support can
- therefore be used to train the walking function and as a means of transfer.

**Remember that walking consists of moving the body in an erect position from one point to another in space; the standing and walking support makes it possible to train:**

- the gait pattern ( flexion of one leg and contemporaneous extension of the other leg)
- the standing tolerance and the balance of the trunk and pelvis, first on both legs and then only on one leg while moving forward
- space perception and distances estimation

**In summary the movita is used to:**

- evoke and stimulate the gait pattern
- train the standing tolerance and the balance of the pelvis and trunk
- transmit the patient a better and wider perception of space, distances and of the surroundings in an erect position
- structure "intentions" and "activities", offering for example the possibility to see an object, think of its use, organise his/her movements, reach it and finally use it
- develop the autonomy of the body and psychological independence, as the patient analyses the possibility to act alone without external help

**movita** can only be used for the applications mentioned above. The manufacturer assumes no responsibility for other applications and for damages caused by improper use.

**COMPONENTS**

**movita** is a standing-and walking support, designed to support and facilitate self-initiated movement of the user.

- A upper frame
- B base
- C thoracic ring
- D interior
- E seat
- F seat support
- G support handle
- H anti-tipping safety system
- I footrest
- J direction lock
- K lateral protection
- L foot divider

fig 1

fig. 2

**L**

fig. 3

**K**

fig. 4

**H**

fig. 10

**J**

fig. 11

**I**

## ADJUSTMENT AND OPERATION

**movita** functions in a very easy and secure way: the lack of balance is compensated with a large surface of support; this gives the patient the possibility to move on casters/wheels using his own force.

Even though the use of **movita** is simple and secure, a good introduction by the dealer/therapist is necessary.

There are instructions to be considered for the optimal use of this support.

### Start-up:

The standing- and walking support **movita** is delivered in two parts: base frame with wheels and brakes, and the upper frame with saddle and thoracic ring. Under the seat you find the allen key.

### Assembly of the upper frame and the base:

The upper frame is placed on the base, with the desired height adjusted via the lever clamps.

### Regulation of the standing-and walking support:

When the patient is put into the standing-and walking support, close the thoracic ring (figure 5), and then adjust the height of the following components:

1. thoracic ring ( lever clamps figure 6B)
2. seat (lever clamp figure 6A)

fig. 5

fig. 6

### Height of thoracic ring and seat: (figure 6)

To adjust the thoracic ring height, loosen the two lower release handles (figure 6B). After reaching the desired height, tighten the handles again; both sides must have the same height; the height can

be checked with the help of the marks on both posts. The seat height is adjusted by loosening the two upper release handles (figure 6A). After reaching the height desired the two release handles must be tightened again. Both sides have to be at the same height.

The thoracic ring should be adjusted to the approx. armpit height. Then the seat should be adjusted; the right position is about 2-3 cm under the inseam.

**Thoracic ring:** (figure 5 page 30)

the thoracic ring can be opened and closed by slightly rotating the upholstered tube.

**Springs:**

The two springs in the lower tubes can be exchanged if a harder or softer spring tension is necessary. The springs are available in various strengths in relation to the following bodyweights. For identification the springs have different colours:

**For standing-and walking support movita child I and child II:**

green = min. 10 kg  
light blue = min. 15 kg  
orange = min. 20 kg (standard)  
white = min. 25 kg

**For standing-and walking support movita junior, adult medium and adult large:**

white = min. 20 kg  
yellow = min. 40 kg (standard)  
blue = min. 70 kg  
red = min. 100 kg

Each package with spare springs contains labels with the correspondent min. loading capacity. If the springs are substituted these labels have to be stuck on the outer side of the frame under the spring coverage.

To change the springs the upper frame has to be removed.

**Seat:**

The seat can be slid forward and backward by loosening the big screw under the tube of the seat. After reaching the desired position tighten the screw again. The seat angle can also be adjusted by loosening the two hexagonal screws underneath the seat. Tilt the seat to the desired angle and tighten the screws to secure.

**Pelvis support:**

The pelvis support can be tipped up or down with a hexagonal screw. After loosening the screw the support can be tipped down. After adjustment the screw is tightened to secure.

**Support handle:**

The support handle is fixed to the frame by an aluminium clamp assembly. Loosen the two hexagonal screws to adjust height and depth of the handle. After reaching the desired position tighten the

screw again.

- To adjust the height of the handle (figure 7) loosen the hexagonal screw A, adjust the desired height and retighten the hexagonal screw A.
- To adjust the depth of the handle loosen screw B, adjust to the desired depth and then retighten screw B.

fig. 7

If the patients tend to cross their legs and feet it may be necessary to use the foot divider L, to counteract this pathological behaviour.

After all these preparations it is time to start with the introduction to the use of **movita**.

fig. 8

### No pre-regulations made!

The height of the support for the armpit depends on the capacity to control the trunk. (If the individual's trunk control is good, the support should be lowered).

Also in this case it is suitable to start with the support right under the armpit and then to lower it as the patient's skills improve.

If there is a good trunk control, the support for the armpit can be removed.

It is therapeutically advisable to arrive at a position where shoulders, trunk and hips are on the same line.

It is important to get the patient to know the standing and walking support and its functions. Normally neither children nor adults are afraid of using it. On the contrary: very often they react with enthusiasm and feel like moving in all directions.

The vigour of these “adventurous” people should not be slowed down, but stimulated. We advise that the **movita** should be used quite freely, so that the patient gets familiar with all of its functions.

For persons who tend to have vehement and uncontrollable movements, the anti tippers should be mounted for safety reasons. The doctor who prescribes the support makes that decision (figure 9).

At the beginning it is always advisable to make use of the direction lock (figure 10), to avoid uncontrolled changes in direction and the danger of bumping against objects.

fig. 9

fig. 10

For “anxious” people, who are afraid of using **movita**, (probably because of problems with perception which make it difficult to comprehend space and distances), another procedure is advisable:  
- When the patient is trying to make his first steps he should not be left alone, but should be accompanied by qualified personnel or the therapist and a family member. They should place themselves in front, next to or behind the patient and move the **movita** slowly and cautiously.

These cautionary measures give the patient a great sense of security, especially, if at the same time he gets an explanation of what is happening, and what they are doing with him/her, e.g. “ Now we are going to the table, we arrive at the television, we are going to get the ball on the table, we are going back....”.

Initially it is necessary to try to reach a movement of walking with constant speed.

The progressive therapy requires then a faster or slower walking movement, with an increase or decrease of frequency of steps.

Limited distances, whether for “daring” or for “anxious patients, are advisable, as they produce a useful effect of orientation and progress.

**Please observe the indications for security!**

## **MOUNTING OF THE ACCESSORIES**

Together with movita it is possible to use the following accessories:

- foot divider (figure 12 L)
- lateral protection (figure 13 K)
- anti-tipping safety system (figure 14 H)

### **Mounting of the foot divider:**

- Insert the support of the foot divider into the adjustment tube underneath the seat and tighten the screw **A**!
- Adjust the height of the foot. Tighten screw **B** again.

**A** **B** **L**

fig. 12

### **Mounting of the lateral protection:**

- Fix both clamps on the base frame as shown on figure 13.

**K**

fig. 13

### **Mounting of the anti-tipping safety system:**

- As shown on figure 14, the antitipping system is fastened on the vertical tube. Between the floor and the anti-tipping system there should be a distance of 5 cm to prevent the anti tippers from disruption of walking.

**H**

fig. 14

**TECHNICAL DATA**

movita child I and movita child II		
Technical data	movita child I	movita child II
Article number	201000	202000
Length	72 cm	72 cm
Width	64,5 cm	64,5 cm
Seat height	from 27 to 41,5 cm	from 27 to 61 cm
Thoracic ring height	from 54 to 70,5 cm	from 69,5 to 84,5 cm
Thoracic ring perimeter	70 cm / 90 cm	70 cm / 90 cm
Wheel dimensions front	140 x 30 mm	140 x 30 mm
Wheel dimensions rear	250 x 50 mm	250 x 50 mm
Load capacity of each spring	min. 10 kg / min. 15 kg / min. 20 kg (standard) / min. 25 kg	min. 10 kg / min. 15 kg / min. 20 kg (standard) / min. 25 kg
Weight	17 kg	18 kg
Weight limit	60 kg	60 kg

movita junior	
Technical data	movita junior
Article number	207000
Length	86 cm
Width	69 cm
Seat height	from 46 to 67 cm
Thoracic ring height	from 76 to 97 cm
Thoracic ring perimeter	70 cm / 90 cm / 105 cm
Wheel dimensions front	140 x 30 mm
Wheel dimensions rear	400 x 40 mm
Load capacity of each spring	min. 20 kg / min. 40 kg (standard) / min. 70 kg / min. 100 kg
Weight	28 kg
Weight limit	130 kg

movita adult medium and movita adult large		
Technical data	movita adult medium	movita adult large
Article number	204000	205000
Length	96 cm	96 cm
Width	69 cm	69 cm
Seat height	from 62 to 96 cm	from 77,0 to 111,0 cm
Thoracic ring height	from 95,0 to 121,0 cm	from 112,0 to 145,5 cm
Thoracic ring perimeter	70 cm / 90 cm / 105 cm / 120 cm	70 cm / 90 cm / 105 cm / 120 cm
Wheel dimensions front	200 x 50 mm	200 x 50 mm
Wheel dimensions rear	400 x 40 mm	400 x 40 mm
Load capacity of each spring	min. 20 kg / min. 40 kg (standard) / min. 70 kg / min. 100 kg	min. 20 kg / min. 40 kg (standard) / min. 70 kg / min. 100 kg
Weight	28 kg	29 kg
Weight limit	130 kg	130 kg

## **CLEANING INSTRUCTIONS**

Never make any tilt adjustment or any substitutions of parts or accessories with the patient in the standing and walking support.

### **Cleaning of the metal parts:**

Clean the enamel and chrome parts of the standing and walking support with a mild alcohol based product of commercial quality and wipe them with a soft cloth.

### **Cleaning of the casters/wheels:**

Please keep the casters/wheels always clean; eliminate dust, hairs or other materials regularly.

## **MAINTENANCE**

Never make any substitutions of parts or accessories with the patient in the standing and walking support.

### **Accessories:**

For any substitutions of accessories please refer to the instructions in chapter "Assembly and mounting of accessories"; remember that for dismounting follow the instructions in reverse order.

### **Maintenance:**

Maintenance has to be performed once a year by the dealer or by authorized personnel.

## **SPARE PARTS**

Only original spare parts and accessories should be used. The guarantee becomes void if repairs or changes are undertaken on **movita** or its components by unauthorized persons.

Any product liability from the side of the manufacturer (Elmi GmbH/srl, Handwerkerzone 25/ Zona Artigianale 25, 39052 Kaltern/ Caldaro, BZ, Italy) is excluded from the beginning in such cases.

For questions on spare parts and accessories please contact your authorized dealer or the manufacturer directly.

Environmental factors:

All packing materials are recyclable.

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Elmi GmbH-Srl