Manufacturer:
Cochlear Bone Anchored Solutions AB Konstruktionsvägen 14, SE - 435 33 Mölnlycke, Sweden Tel: +46 31 792 44 00 Fax: +46 31 792 46 95

Regional offices:
Cochlear Ltd (ABN 96 002 618 073) 1 University Avenue, Macquarie University NSW 2109, Australia Tel: +61 2 9428 6555 Fax: +61 2 9428 6352
Cochlear Americas 13059 E Peakview Avenue, Centennial, CO 80111, USA Tel: +1 303 790 9010 Fax: +1 303 792 9025
Cochlear AG EMEA Headquarters, Peter Merian-Weg 4, 4052 Basel, Switzerland Tel: +41 61 205 0404 Fax: +41 61 205 0405

Local offices:
Cochlear Deutschland GmbH & Co. KG Karl-Wiechert-Allee 76A, D-30625 Hannover, Germany Tel: +49 511 542 770 Fax: +49 511 542 7770
Cochlear Europe Ltd 6 Dashwood Lang Road, Bourne Business Park, Addlestone, Surrey KT15 2HJ, United Kingdom Tel: +44 1932 87 1500 Fax: +44 1932 87 1526
Nihon Cochlear Co Ltd Ochanomizu-Motomachi Bldg, 2-3-7 Hongo, Bunkyo-Ku, Tokyo 113-0033, Japan Tel: +81 3 3817 0241 Fax: +81 3 3817 0245
Cochlear (HK) Limited Unit 1810, Hopewell Centre, 183 Queens Road East, Wan Chai, Hong Kong SAR Tel: +852 2530 5773 Fax: +852 2530 5183
Cochlear Medical Device (Beijing) Co LtdUnit 2208-2212, Tower B, No.9 Germdale Building, No.91 Jianguo Road, Chaoyang District, Beijing, PR China 100022
P.R. CHINA Tel: +86 10 5909 7800 Fax: (+86 10) 5909 7900
Cochlear Limited (Singapore Branch) 6 Sin Ming Road, #01-16 Sin Ming Plaza Tower 2, Singapore 575585 Tel: +65 6553 3814 Fax: +65 6451 4105
Cochlear Korea Ltd 1st floor, Cheongwon building, 828-5, Yuksam dong, Kangnam gu, Seoul, Korea Tel: +82 2 533 4663 Fax: +82 2 533 8408
Cochlear Benelux NV Schaliënhoeverdreef 20 1, B - 2800 Mechelen, Belgium Tel: +32 1579 5511 Fax: +32 1579 5500
Cochlear Medical Device Company India (P) Ltd Platina Building, Ground Floor, Plot No. C-59, G-Block, BKC, Bandra (E), Mumbai – 400 0051, India
Tel: +91 22 6112 1111 Fax: +91 22 61121100
Cochlear Italia S.r.l. Via Larga n°33, 40138 Bologna, Italia Tel: +39 051 601 53 11 Fax: +39 051 39 20 62
Cochlear France S.A.S. Route de l'Orme aux Merisiers, Z.I. Les Algorithmes - Bât. Homère, 91190 Saint Aubin, France Tel: +33 811 111 993 Fax: +33 160 196 499
Cochlear Nordic AB Konstruktionsvägen 14, SE - 435 33 Mölnlycke, Sweden Tel: +46 31 335 14 61 Fax: +46 31 335 14 60
Cochlear Tıbbi Cihazlar ve Sağlık Hizmetleri Ltd. Şti. Cubuklu Mah. Bogazici Cad., Bogazici Plaza No: 6/1, Kavacik,TR - 34805 Beykoz-Istanbul, Turkey
Tel: +90 216 538 5900 Fax: +90 216 538 5919
Cochlear Canada Inc 2500-120 Adelaide Street West, Toronto, ON M5H 1T1 Canada Tel: +1 416 972 5082 Fax: +1 416 972 5083

www.cochlear.com

Cochlear™

Baha® BP110 Power

User manual part A

- 3
GB
- 10
FR
- 17
DE
- 24
NL
- 31
IT

English Français Deutsch Nederlands Italiano **ZONE 1**

Welcome

Congratulations on your Cochlear™ Baha® BP110 Power Sound Processor.

You are now ready to use Cochlear's highly advanced bone conduction sound processor.

This manual is full of tips and advice on how to best use and care for your Baha sound processor. By reading this manual and then keeping it handy for future reference, you'll ensure that you get the most benefit out of your Baha sound processor.

Key to device

See figure 

1. Microphones
2. Battery compartment door
3. Direct Audio Input
4. Visual status indicators
5. Volume up button
6. Plastic snap connector
7. Program button, on/off button
8. Volume down button
9. Attachment point for the safety line

Note on figures: The figures included on the front cover correspond to information specific to this model of sound processor. Please reference the appropriate figure when reading. The images shown are not to scale.

Introduction

The Cochlear™ Baha® BP110 Power Sound Processor uses direct bone conduction to transmit sounds to the cochlea (inner ear). It works by combining a sound processor and a small titanium implant that is placed in the skull behind the ear. The skull bone integrates with the titanium implant through a process called osseointegration. This allows sound to be conducted via the skull bone directly to the cochlea, which improves hearing performance.

You should have received your Baha sound processor from your hearing care professional along with the contents of this kit. To ensure optimal performance, your hearing care professional will fit the processor to suit your needs based on your connection configuration. Be sure to discuss any questions or concerns that you may have regarding your hearing or use of this system with your hearing care professional.

Be sure to follow all applicable instructions in this manual based on your specific connection configuration.

Warranty

The warranty does not cover defects or damage arising from, associated with, or related to the use of this product with any non-Cochlear processing unit and/or any non-Cochlear implant. See "Cochlear Baha Global Limited Warranty card" for more details.

Contacting customer service

We strive to provide the best possible products and services to our clients. Your views and experiences with our products and services are important to us. If you have any comments you would like to share, please contact us.

Customer Service – Cochlear Americas

13059 E. Peakview Avenue,
Centennial, CO 80111, USA
Toll free (North America) 1800 523 5798

Tel: +1 303 790 9010, Fax: +1 303 792 9025
E-mail: customer@cochlear.com

Customer Service – Cochlear Europe

6 Dashwood Lang Road
Bourne Business Park, Addlestone,
Surrey KT15 2HJ, United Kingdom

Tel: +44 1932 87 1500, Fax: +44 1932 87 1526
E-mail: info@cochlear.co.uk

Customer Service – Cochlear Asia Pacific

1 University Avenue, Macquarie University
NSW 2109, Australia

Toll free (Australia) 1800 620 929
Toll free (New Zealand) 0800 444 819

Tel: +61 2 9428 6555, Fax: +61 2 9428 6352
or Toll free 1800 005 215
E-mail: customerservice@cochlear.com.au

Key to symbols

Throughout these documents you will see small symbols. Please refer to the list below for explanations:

	"Consult instructions for use" or "Consult operating instructions"
	"Caution, consult accompanying documents" or "Attention, see instructions for use"
	Audible signal
	CE-mark
	Temperature limitation
	Interference risk
	1 Product name 2 Date of manufacture 3 Serial number
	Batch Code
	Serial Number
	Refer to instructions/booklet

Using your sound processor

The sound processor is equipped with buttons for turning the sound processor on or off, selecting different listening programs and adjusting the volume level. Audio and visual indicators can be activated to alert you to the different settings and processor status.

Your sound processor is programmed to be used specifically on either the right side or the left side of your head.

On/off and indicators

See figure 2

Turn on your sound processor by pressing and holding down the middle button until the visual indicator flashes **3 times**. To turn off your sound processor, press the middle button and hold it down for 2 seconds.

If the sound processor is turned off and then turned on again within 15 minutes, it returns to the last used program and volume level.

If the sound processor is turned off for longer than 15 minutes, or if the battery is changed or removed, it will return to the default program and volume (as set by your hearing care professional) when it is turned on again.

Audible and visual status indicators

See figure 3

Your sound processor is equipped with audible and visual status indicators ① (beeps and LEDs). These provide a simple and effective means of monitoring the status of the sound processor, and they are especially useful for caregivers to ensure that their child's sound processor is working at all times.

When you turn on your processor, the visible indicators flash. If they flash orange, ② your sound processor is configured for the right side of your head. If they flash blue, ③ your sound processor is for the left side. If you are a bilateral user, make sure your processor is used on the correct side.

Note: Your hearing care professional has the ability to disable the flashing indicators and beeps, if desired. However, the orange and blue indicators at startup cannot be disabled.

If your sound processor flashes	Then it is configured for use on the
Orange	Right side
Blue	Left side

Changing the program See figure 4

Your hearing care professional has, in discussion with you, selected up to three programs for your Baha sound processor:

Program 1:

Program 2:

Program 3:


(These programs are suitable for different listening environments. Ask your audiologist to fill in your specific programs).

To switch programs, quickly press and release the middle button. Audio and visual signals let you know which program you have switched to:

Program ①: 1 beep/flash

Program ②: 2 beeps/flashes

Program ③: 3 beeps/flashes

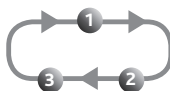Adjusting volume and setting the Keylock *See figure 5*

Adjusting volume

Your hearing care professional has set the volume level for your sound processor. You are also able to manually adjust the volume control.

To control your listening comfort in different listening environments, you can adjust the volume:

- 1 For sound processor on right side, press the front button to increase volume. Press the back button to decrease volume.
- 2 For sound processor on left side, press the front button to decrease volume. Press the back button to increase volume.

A beep sounds each time a volume button is pressed. When you reach your default volume, 2 short flashes/beeps occur. When you reach the minimum or maximum volume level, a long flash/beep occurs.

Keylock *See figure 6*

Keylock protects program and volume level settings and ensures they cannot be changed inadvertently.

- To activate Keylock, press and hold both volume buttons for 5 seconds 1. The blue visual indicator flashes to show that Keylock is activated 2.
- To disable Keylock, press and hold both volume buttons for 5 seconds 1. The orange visual indicator flashes to show that Keylock is disabled 3.

Note: Keylock remains in effect even when the battery is removed. When Keylock is enabled, the visual indicator will flash 6 times when the processor is turned on.

Note: Do not press and hold all 3 buttons at the same time. This will put the sound processor into a temporary programming mode with flashing indicators, and is not suitable for listening.

Note: If both indicators flash - stop. Take out and reinsert the battery. Retry and make sure to press only the two volume buttons.

Using the telephone *See figure 7*

For best results when talking on the telephone, place the receiver near the microphone inlet of the sound processor instead of near your ear canal. Make sure the receiver does not touch the sound processor as feedback may result.

Attaching the safety line

See figure 8

Before connecting the sound processor, attach the safety line to the sound processor and clip the other end of the safety line to your shirt or jacket.

Cochlear recommends connecting the safety line when engaging in physical activities. Children should use the safety line at all times.

Direct Audio Input

Your sound processor has a a DAI (Direct Audio Input) port that is used to connect to external audio sources, such as MP3 players (requires the use of the optional Cochlear Baha Audio adapter), Bluetooth devices, or the Cochlear Baha Telecoil. Consult your hearing care professional for more information regarding the available accessories for your sound processor.

Changing batteries

When there is approximately 1 hour of remaining battery power, 2 beeps/ashes occur 3 times to let you know that soon the battery will need replacing. At this time you may experience low amplification. If the battery runs down completely, the sound processor will beep/flash and then stop working.

Use one of the included batteries (zinc-air, non-rechargeable) as a replacement. The batteries included in the sound processor kit reflect Cochlear's latest recommendations. Contact your hearing care professional for additional batteries.

Battery tips

- Battery life decreases as soon as the battery is exposed to air (plastic strip is removed).
- Battery life depends on daily usage, volume setting, sound environment, program setting, and battery strength.
- To maximize battery life, switch off the sound processor when not in use.
- Keep extra batteries available if using an FM receiver as they have high energy consumption rates.
- If a battery leaks, immediately replace it.

Note: The inside of the battery door is lined with a moisture barrier that prevents dust and water from entering the sound processor. To ensure an effective seal, Cochlear recommends replacing the battery door every 6 months. In humid environments, replace the battery door more frequently.

Contact your hearing care professional for additional battery doors.

To replace the battery in BP110 Power:

See figure **9**

- 1 2 Gently slide the battery door downwards until you feel a "click". You can now remove the door.
- 3 Remove the old battery. Use the included magnetic removal tool if needed. Dispose of battery according to local regulations.
- 4 Remove the sticker on the + side of the new battery. Insert the new battery with the + sign facing the + marked on the processor's battery compartment.
- 5 Attach the battery door and slide it gently back into its locked position.

Optional tamper-proof battery door

To prevent the user from accidentally opening the battery door and exposing the battery, an optional tamper-proof battery door is available. This is particularly useful for children. The tamperproof battery door should only be locked and unlocked using the specially designed tool included in the tamperproof battery door kit to avoid accidental damage to the door. Contact your hearing care professional for tamper-proof battery doors.

To use the tamper-proof battery door:

See figure **10**

1. Make sure the screw slot is positioned vertically.
2. Attach the battery door and slide it to its closed position.
3. Turn the screw a quarter-turn clockwise to lock the battery door in the horizontal position.
4. To unlock the battery door, turn the screw a quarter-turn counter-clockwise to vertical. The battery door can now be removed.

Caution: Batteries can be harmful if swallowed. Be sure to keep your batteries out of reach of small children, those with mental challenges, and household pets. In the event a battery is accidentally swallowed, seek immediate medical attention at the nearest emergency center.

General processor care

Your Baha sound processor is a delicate electrical instrument. Follow these guidelines to maintain its proper working order:

- If you will not be using your sound processor for any length of time, remove the battery door to let the sound processor ventilate.
- If you will not be using the sound processor for a long period, remove the battery.
- When not in use, turn off your sound processor and store it free from dust and dirt.
- During physical activities, secure your sound processor using the safety line. If the physical activity involves contact, Cochlear recommends removing your sound processor and protecting the implant site with a sweat band or similar.
- Remove your sound processor before applying hair spray or gel. If using hair dye, remove the sound processor.

Using Share the Experience

Family members and friends can “share the experience” of bone conduction hearing. The test rod is used by others to simulate hearing with the Baha sound processor.

To use the test rod:

See figure 11

- 1 Snap the sound processor onto the test rod using the tilt technique. Hold the rod against the skull bone behind an ear.
- 2 Plug both ears and introduce sound.

To avoid feedback (whistling), the sound processor should not touch anything other than the test rod.

Warnings and precautions

General advice


A sound processor will not restore normal hearing and will not prevent or improve a hearing impairment resulting from organic conditions.

- Infrequent use of a sound processor may not permit a user to attain full benefit from it.
- The use of a sound processor is only part of hearing rehabilitation and may need to be supplemented by auditory training and instructions in lip-reading.

Warnings

- The sound processor is a digital, electrical, medical instrument designed for specific use. As such, due care and attention must be exercised by the user at all times.
- The sound processor is not waterproof! Never wear it in heavy rain, in the bath or shower!

- Do not expose the sound processor to extreme temperatures. The sound processor is designed to operate within the temperature ranges +5°C (+41°F) to +40°C (+104°F). In particular, battery performance deteriorates in temperatures under +5°C. The processor should not be subject, at any time, to temperatures below -20°C (-4°F) or above +50°C (+122°F).

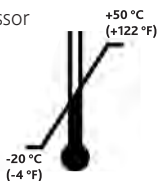- This product is not suitable for use in flammable and/or explosive environments.
- If you are to undergo an MRI (Magnetic Resonance Imaging) procedure, refer to the MRI Reference Card included in the document pack.
- Always use the Audio adapter when connecting an external audio source to your sound processor.
- Portable and mobile RF (radio frequency) communications equipment can affect the performance of your sound processor.
- The sound processor is suitable for use in electromagnetic environments with mains power of typical commercial or hospital quality, and power frequency magnetic fields of typical commercial or hospital levels. It is intended for use in an electromagnetic environment in which radiated RF (radio frequency) disturbances are controlled.
- Interference may occur in the vicinity of equipment with the symbol to the right.

- Dispose of batteries and electronic items in accordance with your local regulations.
- Do not dispose of your sound processor in an incinerator or into fire.

If the sound processor becomes soaked with water:

1. Immediately open the battery door and remove the battery.
2. Put the sound processor in a container with drying capsules such as a Dri-aid kit or similar. Leave the processor to dry out overnight. Drying kits are available from most hearing care professionals.

Feedback (whistling) problems

See figure 12

- ➊ Check that items such as hats and glasses do not come in contact with the sound processor. Also, ensure that neither the head nor the ear come in contact with the sound processor.
- ➋ Decrease the volume.
- ➌ Check that the battery door is closed.

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