# TURBINES ALTO®

400 LC / 500 LC

## COUPLEUR ALTO® CL

Ref. 60300483-C

*Your Endo Specialist™*

**MICRO-MEGA®**

5-12, rue du Tunnel  
F-25006 BESANCON Cedex

**DISTRIBUTION :**

**MICRO-MEGA®  
France Distribution**

Tél : +33 (0)3 81 54 42 36  
Fax : +33 (0)3 81 54 42 39  
[commercial@micro-mega.com](mailto:commercial@micro-mega.com)

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## I. WARNINGS

### 1.1 General warnings

- The device is meant for professional dental use only.
- Perform regular maintenance.
- Use the turbine only for dental treatment. Any other use or alteration of the device is banned and may be dangerous. Intended for withdrawing caries matter, reducing hard dental structure, preparing cavities, finishing tooth preparation, tooth restoration and polishing.
- Verify that only rotary instruments in good condition are used.
- Eliminate diamond drills where grains are partially missing.
- Verify that the instrument is properly and firmly held before starting the turbine.
- Avoid working with a turbine which heats up abnormally.
- It is recommended to wear appropriate protection gloves and eye-protection while handling the turbine and the rotary instruments.
- Use a dental dam during intervention on a patient.
- During long treatments, repeat lubrication every ten minutes.
- Never dismantle the turbine. Any dismantling of the turbine may only be carried out by an approved dealer or the manufacturer.
- Respect the input pressure for optimal functioning.
- Contact the MICRO-MEGA® after sales service as soon as a problem with the turbine occurs.

- Stop using the turbine in case of damage, overheating, irregular operating sound, excessive vibration, excessive run-out of the instrument or inadequate instrument fixation. In order to prevent further damage, contact the MICRO-MEGA® after sales service or your approved MICRO-MEGA® dealer.

## 1.2 Warnings for reprocessing (cleaning - disinfection - lubrication - sterilization)

- Sterilize and lubricate the turbine before first use and after every intervention on a patient.
- Remove the rotary instruments (drills, etc.) before reprocessing.
- Passing the turbine in a basin or an ultrasound system is banned.
- Soaking the turbine in a detergent product provided to this effect must not exceed 15 minutes.
- Only use a detergent-disinfectant product specifically intended to the pre-disinfection of dental handpieces, contra-angles and turbines which is compatible with aluminium and does not contain soda based products or aldehyde. Do not use any alkaline detergents or solutions (pH > 7).
- Comply with the concentration recommended by the pre-disinfectant manufacturer.
- Respect the immersion time recommended by the pre-disinfectant manufacturer.
- Open the sterilization bag just before use.
- In case the turbine is not used for an extended period of time, clean and lubricate it in accordance with the instructions hereafter and store it in a dry place.

FR

EN

DE

IT

ES

## II. ADVICE FOR SET UP AND USE

A supply of dry, clean and uncontaminated compressed air in accordance with the standard EN ISO 7494-2 must be provided. Contaminated and humid air can lead to early wear of the ball bearings.

### 2.1 Connection to the unit


Screw the ALTO® CL coupler on to the unit's Midwest tubing by tightly screwing the turbine's coupling to the flexible tube, then fit the turbine in an axial position to the coupler and push until a click can be heard. Set the input pressure to 2.2 bars (32 PSI) while running the turbine.

### 2.2 Use of the push-buttons

Do not run the turbine without a canal instrument inserted in the chuck.

Insertion of the instrument ( Ø 1.6 mm):

- Make sure that the turbine has stopped.
- Present the canal instrument in its lodging and push to the end while pressing the push-button.

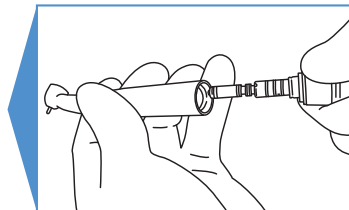- Release the push-button.
- Make sure that the instrument is properly connected and firmly held.

Removal of the instrument:

- Make sure that the turbine has stopped.
- Press the push-button and pull the instrument.
- Release the push-button.

### III. REPROCESSING OF THE TURBINE

#### 3.1 General information

**(Follow instructions after every use.)**

- The sterilization process does not affect the performance of dental handpieces, contra-angles and turbines. The end of the devices' service life is usually determined by damages due to use.
- Immediately after every use, withdraw excessive residues with a disinfecting wipe.
- Do not use products containing acetone, chlorine or bleach as disinfectants! If using disinfecting wipes or jets, follow the instructions of use for these products.
- It is recommended to reprocess the turbine directly after every use. If this is not possible, immediately rinse the turbine with water to prevent any impurities, blood or saliva from drying or coagulating inside the device.

FR

EN

DE

IT

ES

## 3.2 Manual cleaning prior to sterilization

**(Follow instructions after every use.)**

The immersion of the turbine during pre-disinfecting stage (immersion time must not exceed 15 minutes) is authorized in accordance with the following instructions:

- Use a detergent-disinfectant product specifically intended to the pre-disinfection of dental handpieces, contra-angles and turbines which does not contain soda based products or aldehyde.
- Comply with the concentration recommended by the pre-disinfectant manufacturer.
- Respect the immersion time recommended by the pre-disinfectant manufacturer.
- After immersion, rinse abundantly to eliminate the liquid/residues mixture contained in the turbine.
- Immediately after rinsing, thoroughly dry the inside of the turbine using compressed air.
- Before sterilization, lubricate the turbine using an oil spray ("500I" by MICRO-MEGA®).

The last three stages of this cleaning process (rinsing, drying and lubrication) may be performed by automatic devices available on the dental market, following the instructions of use of these devices.

**MICRO-MEGA® recommends using X-Cidol® 2 as pre-disinfectant product.**

### 3.3 Automatic cleaning prior to sterilization

**(Follow instructions after every use.)**


In order to eliminate all handling, after each use on a patient place the turbine in a device of the product line X-Cid®. These devices clean, disinfect and lubricate your handpieces, contra-angles and turbines without previous dismantling and guarantee a rapid bactericide, fungicide and virucide effect, ensuring the destruction of microbial agents such as HIV, HBV and Tuberculosis. The regular use of a device of the product line X-Cid® significantly extends the life span of your handpieces, contra-angles and turbines.

### 3.4 Maintenance

**(Follow instructions after every use.)**

Changing the spray distributor (ALTO® 400 LC):

- Unscrew the distributor using the wrench (Ref 59800601).
- Clean the surfaces. Do not leave the O ring inside the head.
- When refitting, position the O ring in the distributor then screw tight.
- Do not over tighten.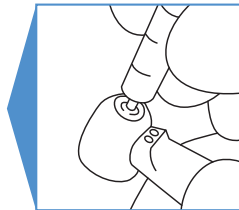

FR

EN





DE

IT

ES

Changing the chuck (ALTO® 400 LC and ALTO® 500 LC):

- Remove the bur from the chuck.
- Unscrew the push-button using the wrench (Ref 50900148).
- Remove the chuck from its housing.
- Insert the new chuck.
- Reinsert the push-button, screw it tight and lock it.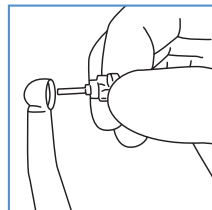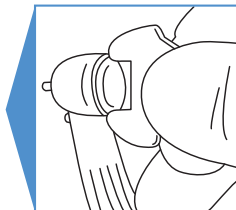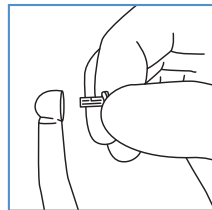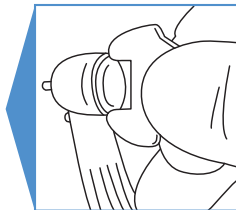


Changing the cartridge (ALTO® 400 LC and ALTO® 500 LC):

- Remove the instrument from the chuck.
- Unscrew the push-button using the wrench (Ref 50900148).
- Remove the cartridge from its housing.
- Insert the new cartridge.
- Reinsert the push-button, screw it tight and lock it.


#### Changing the O rings (ALTO® CL):

- Remove the O rings.
- Clean the coupler.
- Position the new O rings in their grooves.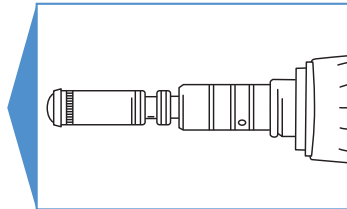

#### Changing the bulb (ALTO® CL):

- Unscrew the bulb protection sleeve.
- Remove the bulb and insert a new one.
- Screw on the bulb protection sleeve.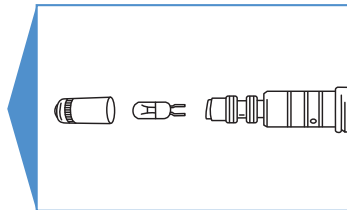

FR

EN

DE


IT

ES

### 3.5 Lubrication

**(Follow instructions after every use.)**

- Remove the root canal instrument.
- Introduce the “500I” spray into the rear end of the turbine.
- Press the spray a number of times lasting several seconds until the liquid coming out of the turbine is clean.
- Make the turbine turn in order to evacuate excess oil before use.
- During long treatments, repeat the lubrication process every ten minutes.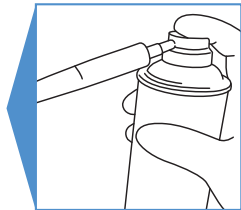

### 3.6 Sterilization

**(Follow instructions after every use.)**

- MICRO-MEGA® handpieces, contra-angles and turbines can be sterilized (wrapped in a sterilization bag) in a steam autoclave at 135°C (275°F) to 138°C (280.4°F) and 2.2 bar during 18 minutes.
- Following manual or automatic preparation for sterilization and lubrication, place the turbine in a hermetic sterilization bag:

- Individual packaging: The sterilization bag must be of appropriate size for the turbine and not be distended.
- Batch processing: The turbines may be placed in an instrument box provided to this effect or in a tray generally used for sterilization. Wrap up the trays using appropriate methods.
- Perform sterilization in a sterilizer compliant with the standard EN 13060. Contact corrosion due to humidity may be possible.
- In case you want to sterilize several handpieces, contra-angles or turbines in one autoclave cycle, make sure that the autoclave's maximum load is not exceeded (cf. the autoclave's instructions of use).
- Immediately remove the device from the autoclave once the sterilization cycle is accomplished.
- Respect the sterilization bags' duration of storage indicated by the manufacturer.
- Quality and use of the wrapping material must be appropriate for sterilization and comply with the applicable standards.
- Wrap the turbine separately in a retractable sterilization bag.
- Open the sterilization bag just before use.

FDA recommends that turbines, contra-angles and handpieces are to be heat sterilized in between patients.

## IV. REPAIRS

For service or repairs, send the complete turbine.

Inspections and repairs may only be carried out by an approved dealer or by the manufacturer.

**MICRO-MEGA® recommends annual service for all turbines, contra-angles and handpieces.**

Caution - U.S. Federal law restricts this device to sale by or in the order of a dentist.

## V. GUARANTEE AND AFTER SALES SERVICE

Your MICRO-MEGA® turbine is an accurate medical device, produced under maximum quality and test conditions. To guarantee personal follow-up and traceability of its medical devices, MICRO-MEGA® would like you to return the guarantee card, duly filled in, within 10 days of your purchase. This return ensures that the procedure for after sales care of your turbine will be instigated so that MICRO-MEGA® can give you full satisfaction with the service you expect and manage your guarantee correctly.

**You can also register the guarantee for your turbine on our website: [www.micro-mega.com](http://www.micro-mega.com)**

MICRO-MEGA® guarantees your turbine for one year against any hidden manufacturing defect, excluding faults due to:

- Improper use.
- Inappropriate maintenance or application, not in accordance with our recommendations and instructions for use.
- Accidental damage (fall, impact, etc.).
- Attempted intervention or repair by personnel not approved by MICRO-MEGA®.
- Attempted product modification.

## VI. SPECIFICATIONS

|                                                                       | ALTO® 400 LC                                                             | ALTO® 500 LC       |
|-----------------------------------------------------------------------|--------------------------------------------------------------------------|--------------------|
| Class in accordance with Directive 93/42 CEE                          | IIA                                                                      | IIA                |
| Maxi motor speed                                                      | 340 000 rpm                                                              | 440 000 rpm        |
| Motor connection standard (coupling)                                  | Multiflex®*                                                              |                    |
| Instrument fixation                                                   | Push-button                                                              |                    |
| Maximum power                                                         | 14W at 180 000 rpm                                                       | 12W at 200 000 rpm |
| Light power                                                           | Transmission of 80% of input light power (i.e. 17 000 Lux with ALTO® CL) |                    |
| Instrument type in accordance with NF EN ISO 1797-1 (Instrument type) | Type 3                                                                   |                    |
| Instrument diameter (mm)<br>in accordance with NF EN ISO 1797-1       | 1.6                                                                      |                    |

FR

EN

DE

IT

ES

FR

EN

DE

IT

ES

|                                                         | ALTO® 400 LC    | ALTO® 500 LC |
|---------------------------------------------------------|-----------------|--------------|
| Length                                                  | 116 mm          | 119 mm       |
| Head dimensions                                         | 14 x 13 mm      | 12 x 10 mm   |
| Level of noise                                          | Less than 80 dB |              |
| Max. air consumption at a pressure of 2.2 bars (32 PSI) | 56.6 NI/mn      |              |


|                                              | ALTO® CL Coupler   |
|----------------------------------------------|--------------------|
| Class in accordance with Directive 93/42 CEE | I                  |
| Light power                                  | 50 000 Lux at 8 mm |
| Unit coupling                                | ISO 9168 type C    |
| Turbine coupling                             | Multiflex®*        |
| Dimensions                                   | 71 x 18.2 mm       |

\* Registered KaVo® trademark.


## VII. ACCESSORIES

### 7.1 References of spare parts for ALTO® LC Turbines

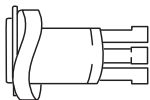
Chuck for ALTO® 400 LC  
or 50 LC:  
**Ref 50900219**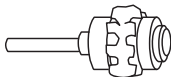

Cartridge for ALTO® 400 LC:  
**Ref 50900218**


Cartridge for ALTO® 500 LC:  
**Ref 50900217**

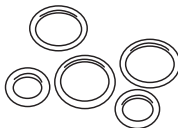O ring kit:  
**Ref 50900131**

Bulb:  
**Ref 50900137**

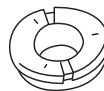Distributor:  
**Ref 50900101**

FR

EN

DE

IT

ES

## 7.2 References of accessories recommended by MICRO-MEGA®

**Ref 50700008**

500 l Spray supplied without nozzle

**Ref 50700029**

Spray for X-Cid®: 500 l CO2 Spray

**Ref 50700017**

500 l Spray supplied with 3 nozzles

**Ref 50704005**

X-Cidol® 2

**Ref** *Please contact us for further information.*  
X-Cid® 2

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