
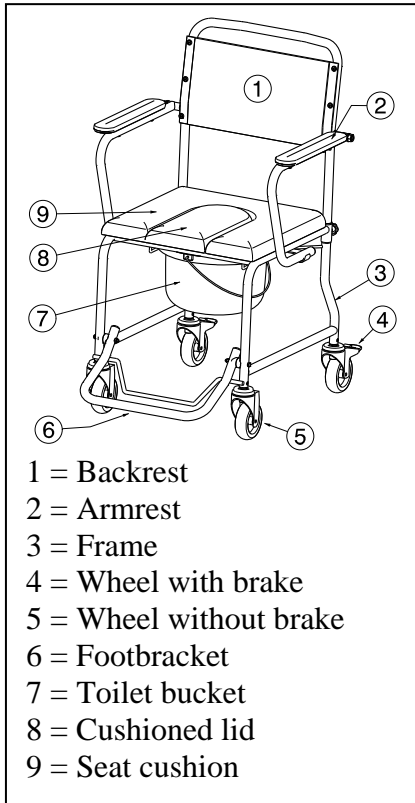
INSTRUCTION MANUAL

Toilet chair Model 9139

9139SP

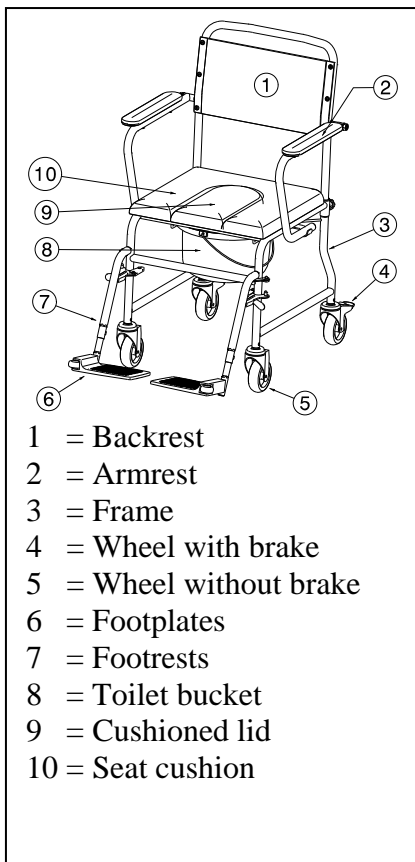9139SPAV

First of all we wish to thank you for the trust you placed in us by selecting a **VERMEIREN** product.

Before using the toiletchair, read the instruction manual carefully: it will familiarise you with this product.

You will also be given hints about the care and service of your toiletchair. Please remember that observance of our hints would keep your toiletchair in an excellent condition of preservation and it will function perfectly even after years of use.

If you have any further questions, please consult your specialist dealer. **To find a service facility or specialist dealer near you, contact the nearest Vermeiren facility.**

TECHNICAL DETAILS

9139SP

Total height:	935 mm
Height of seat:	520 mm
Total width:	575 mm
Width of seat:	438 mm
Height of backrest:	425 mm
Height armrest:	223 mm
Depth:	795 mm
Weight:	ca. 13,8 kg
Max. load:	120 kg

9139SPAV

Total height:	935 mm
Height of seat:	520 mm
Total width:	575 mm
Width of seat:	438 mm
Height of backrest:	425 mm
Height armrest:	223 mm
Length footrest:	475 mm - 535 mm
Depth:	960 mm
Weight:	ca. 13,8 kg
Max. load:	120 kg

We reserve the right to introduce technical changes. Measurement tolerance +/- 15 mm / 1,5 kg

EXPLANATION OF SYMBOLS

Maximum mass

CE conformity

Type designation

CONTENTS OF CONSIGNMENT

Unpack your product and verify if the delivery is complete. Following items should be included:

9139SP

- Toilet chair + seat cushion
- Cushioned lid
- Backrest
- Toilet bucket with cover
- Instruction manual

9139SPAV

- Toilet chair + seat cushion
- Cushioned lid
- Backrest
- Footrests
- Toilet bucket with cover
- Instruction manual

Verify your product for transport damage. If you find any damages after delivery proceed as follows:

- contact the transporter
- have a list made of any problems
- contact your supplier

APPLICABILITY

Mobile, re-usable toilet commode. Should not be used for taking a shower, for storage, as a ladder, or over built-in toilets.

ASSEMBLY INSTRUCTIONS

Place the toilet chair on level surfaces only and make sure that all four wheels stand solidly on the floor. Activate the brakes.

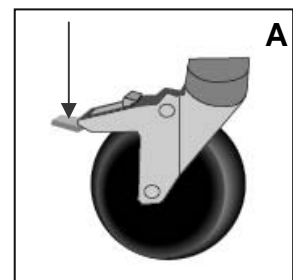
Operating the brakes:

WARNING: Good operation of the brakes is influenced by wear and contamination of the wheels (water, oil, ...) – Check the condition of the wheels before each use.


WARNING: The brakes can wear – Check the operation of the brakes before each use.

The rear wheels can be locked to prevent involuntary rolling.

- **Securing (A)**
Push the surface of the tilting lever downwards by foot until it brakes the wheel.

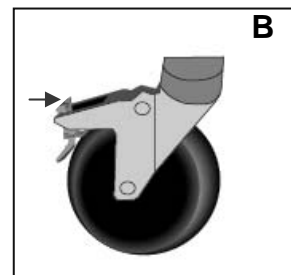- **Releasing (B)**
Push the surface of the tilting lever forwards by foot until the wheel is released.

Mounting / removing backrest + armrest:

⚠ WARNING: Risk of injury – Make sure that both star knobs ① are engaged.

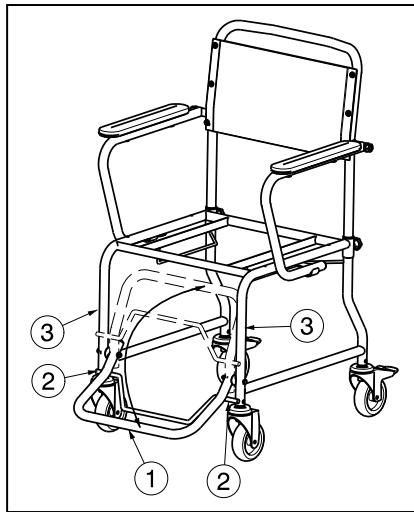
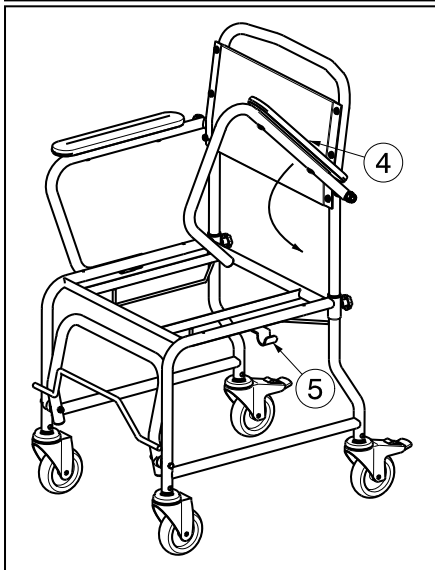
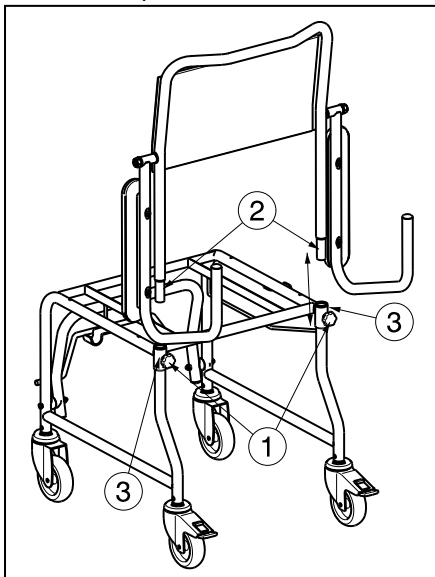
For transport the backrest of the toiletchair can be completely removed and put on the seat.

To mount the backrest on the toiletchair:

1. Mount the backrest tubes ② in the tubes of the frame ③.
2. Retighten the star knobs ① properly.
3. Swing the armrest ④ until they are blocked in the armrest support ⑤.

To remove the backrest from the toiletchair:

1. Loosen the star knobs ①.
2. Remove the backrest tubes ② out the tubes of the frame ③.
3. Put the backrest on the seat for transportation.

For transport the footbracket ① can be completely folded away under the toiletchair.

1. Grasp the footbracket ①.
2. Rotate the footbracket ① upwards until the tubes ② touch the frame tube ③.

Mounting / removing footrests (only for 9139SPAV):

⚠ WARNING: Risk of tipping over the toiletchair – Do not stand on the footplates to get in and out the toiletchair. Fold the footplates upwards to prevent standing on them.

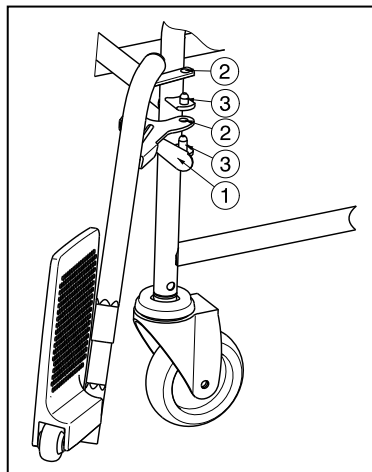
⚠ WARNING: Risk of injury – Check that the footrests are securely locked in place before use.

The mounting of the footrests is done as follows:


1. Hold the footrest sideways at the outside of the wheelchairs frame.
2. Mount the holes ② of the footrest on the pins ③ located on the frame.
3. Turn the footrest inwards till it clicks in position. If this lock is a little stiff, press the lever gently back.
4. Turn the footplate downwards.

To take off the footrests:


1. Pull back the lever ① near the footrest attachment.
2. Turn the footrest to the outside of the wheelchair.
3. Lift the holes ② of the footrest out of the pins located on the frame ③.

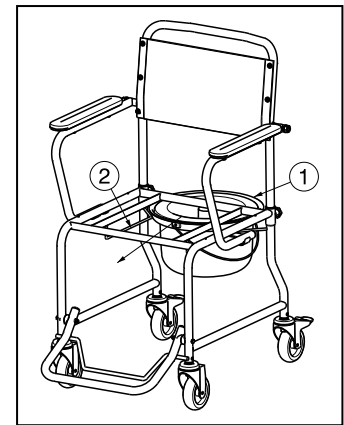The toiletbucket ① can now be placed onto the guide rails ② below the seat (on the backside of the toiletchair).

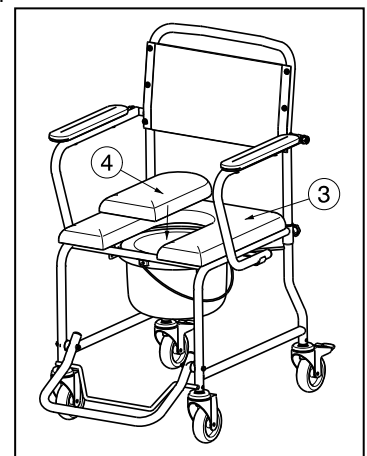Now the seat cushion ③ can be placed on the seat frame. When not in use the seat can be covered by the cushioned lid ④ supplied.

➔ Remove the bucket before transporting the chair.

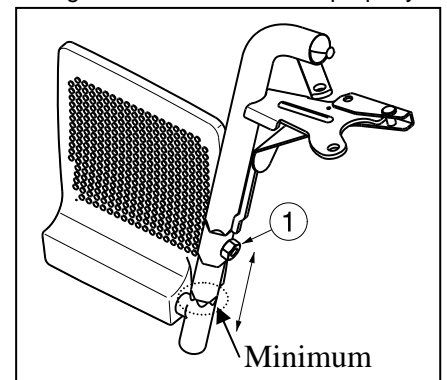
➔ Remove and empty the bucket every time after use.

HINT: Pour some water in the upper edge of the bucket. The bucket is then sealed airtight against bad odours when the lid is closed.

ADJUSTING INSTRUCTIONS

Adjust the length of the footrests as follow:

1. Remove the Allen screw ① on the lower end of the upper part of the footrest.
2. Adjust the length of the footrest to a comfortable length by moving the hexagonal inner tube out or in.
3. Tighten the Allen screw ① properly.

Footbracket (only for 9139SP):

⚠ WARNING: Risk of tipping over the toiletchair – Do not stand on the footbracket ① to get in and out the toiletchair.

The **minimum** length is when the footrest tube changes from hexagonal in round.

The **maximum** length is until the mark in the tube (These mark is a straight line).

SAFETY

- ➔ When setting up the chair and when it is transported, carefully guard against bruising and shearing - danger of injury!
- ➔ Do not grasp sideways further than the length of your arm allows, otherwise the chair can tip over.
- ➔ Before using, check whether all wheels stand evenly on the floor – risk of tipping over!
- ➔ Do not lean too far forwards -- risk of tipping over!
- ➔ Remove the bucket before transporting the chair.
- ➔ Do not exceed the max load (see technical details).
- ➔ Repairs and disinfection should only be done by authorised persons !
- ➔ Empty the bucket every time after use.
- ➔ Risk of burns – Be careful when using in hot or cold environments (sunshine, extreme cold, saunas, etc.) for a sufficient amount of time and when touching.
- ➔ Take note of the instructions for care and service. The manufacturer is not liable for damage caused by improper servicing/care.

CARE

Cleaning shall be according following instructions:

- **Upholstery / frame**
 - Clean upholstery / frame with a cloth moistened with hot water. Be aware that you do not soak the upholstery / frame.
 - Use a mild commercial detergent for removing stubborn dirt.
 - Stains can be removed by using a sponge or a soft brush.
 - Do not use strong cleaning liquids like solvents, nor use hard brushes.
 - Never clean with steam and/or pressure washers.
 - The manufacturer is not liable for damage caused by improper cleaning agents.
- **Plastic parts**

Clean plastic parts of your toilet chair with commercial plastic cleaners. Only use a soft brush or soft sponge.
Examples of plastic parts are the bucket,
- **Surface layer**

The high quality of the surface layer guarantees optimal

protection against corrosion. If the outer coat is damaged by scratching or in some other way, get your specialist dealer to repair the affected surface.

When cleaning, only use warm water and normal household detergents and soft brushes and cloths. Ensure that no wetness gets into the tubes.

Initially the zinc parts only require rubbing with a dry cloth. Stubborn dirt can best be removed by using a suitable commercial zinc polish.

SERVICE

Lifetime of the toiletchair is influenced by its use, storage, regular maintenance, servicing and cleaning.

The toilet chair must be checked at least every second year by authorised persons/specialist dealers, and at least every time before resuming its use. The following parameters must be checked:

- Check the frame for plastic deformation and reduced functioning.
- Visual inspection of damage to the paintwork (danger of corrosion)
- Check the condition of the cushions (tears, brittleness, soaking)
- Bucket sealing (not in the case of re-using)
- Completeness of consignment ?
- Instruction manual available?
- Cleaning

The service should only be confirmed in the "Service plan" if at least the above aspects have been checked. If your specialist dealer does not undertake the services which you require, kindly contact the manufacturer. We will gladly refer you to authorised specialist dealers in your vicinity.

DISINFECTION

All parts of the chair can be disinfected by scrubbing. In principle all surfaces of a system or a product have to be disinfected before handing it over to another user (re-use).

- ➔ Only authorised technicians trained in the functioning and application of disinfectants should apply disinfectants.
- ➔ Wear suitable protective clothing to prevent skin irritations which might be caused by the disinfectant. Also note the product information of the solutions used.
- ➔ The manufacturer does not accept liability for damage and injury caused by improper implementation of the disinfection.

Use only disinfectants included in the list of the Robert Koch Institute and the verified procedures of disinfection by scrubbing or wiping.

The current situation in regard to the disinfectants included in the RKI list, can

be obtained from the Robert Koch Institute (RKI, home page: www.rki.de).

Toilet buckets can be treated hygienically. For use at home: toilet buckets and cushioned seats are not meant to be treated hygienically. **Replacement parts can be obtained from your specialist dealer.**

SHIPPING AND STORAGE

The shipping and storage of the toiletchair shall be according following instructions:

- Store in a dry place (between +5°C and +41°C).
- The relative humidity of the air should be between 30% and 70%.
- Provide sufficient covering or packaging to protect the toilet chair from rust and foreign bodies (e.g. salt water, sea air, sand, dust).

WARRANTY

Excerpt from the "General Business Conditions".

(...)

5. The terms of the guarantee may differ from country to country. Consult your specialist dealer for the guarantee period for warranty claims.

(...)

The warranty excludes damage arising from structural changes to our products, insufficient maintenance, defective or improper handling or storage or the use of non-original parts.

DISPOSAL

When disposing of the toiletchair, contact your local disposal centre or return the product to your specialist dealer who, after submitting it to a hygienic procedure, will be able to send it back to the manufacturer who will dispose of and recycle it correctly, separating it into its component materials.

Packaging materials can be taken to disposal or recycling centers or to your specialist dealer.

DECLARATION OF CONFORMITY

The manufacturer or his authorized representative :

N.V. VERMEIREN N.V.

Address :

Vermeirenplein 1/15

2920 Kalmthout

Belgium

declares under his sole responsibility that the CE marked medical devices :

Productgroup: Commode chairs

Brand: Vermeiren

Type: 9042, 9052, 9060, 9062, 9062 XXL, 9063

have been classified as class I, according to annex IX MDD 93/42/EEC, rule 1,

and is manufactured in full conformity with the European instructions below - including the latest modifications - and with the national law, that organizes this directions :

Medical devices directive MDD 93/42/EEC

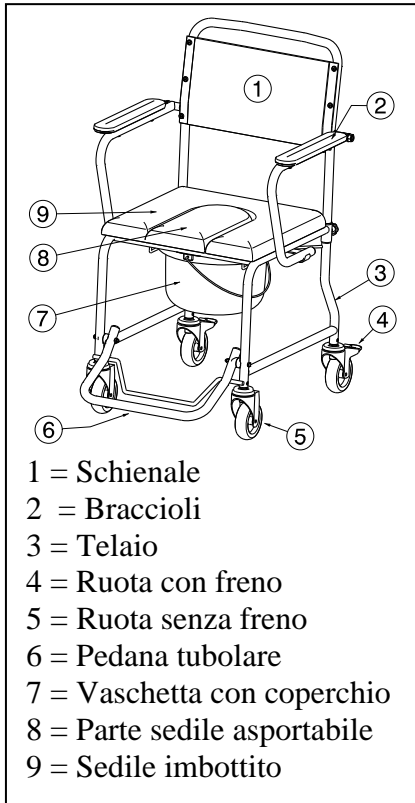
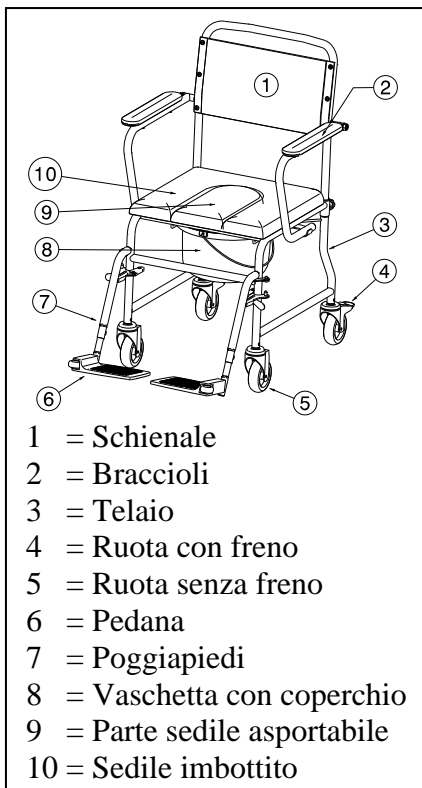
and is in conformity with the relevant European harmonized standards:

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Vermeiren Suisse S.A.