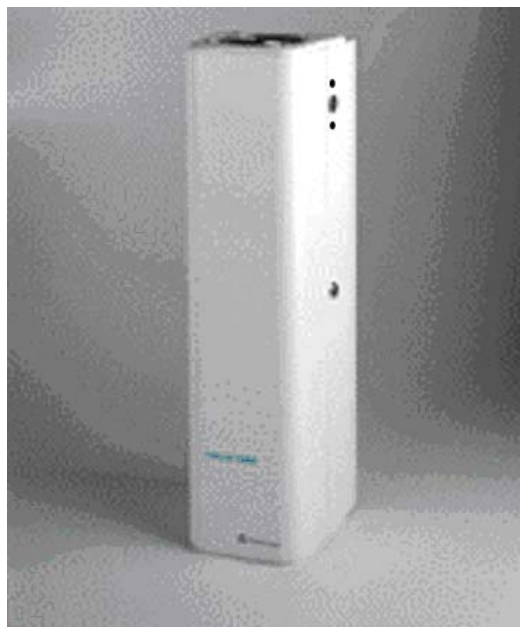
Istruzioni per l'uso
Instructions for use
Mode d'emploi
Bedienungsanleitung
Instrucciones para el uso

STERILAIR 3000

INDICE - INDEX

I	INFORMAZIONI ALL'UTILIZZATORE	
GB	INFORMATION TO THE USER	
F	INFORMATIONS POUR L'UTILISATEUR	2-5
D	ANWENDER INFORMATIONEN	
E	INFORMACIONES AL USUARIO	
I	ITALIANO	6-19
GB	ENGLISH	20-33
F	FRANÇAIS	34-47
D	DEUTSCH	
E	ESPAÑOL	

PAR 01

**INFORMAZIONI ALL'UTILIZZATORE / INFORMATION TO THE USER /
INFORMATIONS POUR L'UTILISATEUR / ANWENDER INFORMATIONEN /
INFORMACIONES AL USUARIO**

**1.1 DICHIARAZIONE DI CONFORMITÀ CE / CE DECLARATION OF CONFORMITÄ
/ DECLARATION CE DE CONFORMITE / KONFORMITÄTSEKTLÄRUNG /
DECLARACIÓN DE CONFORMIDAD CE**

- La sottoscritta - The undersigned – La soussignée - Die unterzeichnende Firma - La que suscribe:
TECNO-GAZ S.p.A., Strada Cavalli n. 4, 43038 Sala Baganza, Parma, Italia.
- Dichiaro sotto la propria responsabilità che / Declares under its own responsibility that Déclare sous sa propre responsabilité que / Erklärt auf eigene Verantwortung / Declara bajo su propia responsabilidad que

Modello – Model - Modale – Modell - Modelo: STERILAIR 3000

Articolo – Artiche - Artiche – Artikel - Artículo : 235/S

Dati macchina /
Machine data /
Données de la machine
Maschinendaten/
Datos de la máquina

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WARRANTY

The device is under warranty for five years, unless the electric parts which are under warranty for twelve months. (N.B. Only for the Italian Market). The warranty starts from the delivery date, confirmed by the return of the coupon: installation minutes filled in, stamped and signed by the dealer. In case of dispute, the date on the sale bill is the valid one. The repair or the replacement, under warranty, of a part, is done after unquestionable TECNO-GAZ decision and does not include the travelling expenses of the staff, the packaging costs and the shipment freights. The lamps, the fuses and the damages caused by a bad maintenance, negligence, inaccuracy and other causes are excluded. The objects and accessories subject to wear and tear (ex. buttons) are not included in the warranty. The replacement of the whole device is not considered. The warranty does not include any compensation for direct or indirect damages of every kind to people and things, caused by the inefficiency of the device. TECNO-GAZ is not answerable for damages caused by the improper use of the device, by the lack of maintenance or by negligence. There is no compensation for the impossibility of use of the device. The warranty falls as soon as the device is tampered, repaired or changed by customer or by non authorized staff. For every action, the customer must address to the dealer or to the customer services indicated by TECNO-GAZ. The parts replaced under warranty must be returned to TECNO-GAZ free port. The non return involves the debit of the part's cost to the applicant. TECNO-GAZ does not accept returns from the end users. The return in TECNO-GAZ for repair must be arranged by the dealer or by the customer service chosen by the end user, complying with the commercial procedures CM-P-003 called "MANAGEMENT OF THE DEMANDS OF RETURN, REPAIR, REPLACEMENT OF TECNO-GAZ PRODUCTS AND CUSTOMER'S COMPLAINTS". The return in TECNO-GAZ must be documented and authorized by inner procedures. The products returning in TECNO-GAZ, must enclose the documentation of authorization for the return and a paper describing the damage. All the products under repair must be delivered to TECNO-GAZ in free port and well packed (by original packaging).

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E-Mail : info@tecnogaz.com

<http://www.tecnogaz.com>

For technical service please apply to the dealer that invoiced the device, or directly to:

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INDEX

PAR	DESCRIPTION	PAGE
01	INFORMATION TO THE USER	2-5
02	PRODUCT 'S INTRODUCTION	21
03	TECHNICAL FEATURES	21
04	PACKING - TRANSPORT - STORAGE – INSTALLATION – DEMOLITION - DISPOSAL	22-24
05	WORKING AND USE	25-26
06	INSTRUCTIONS FOR USER	27
07	ORDINARY AND EXTRAORDINARY MAINTENANCE	28-32
08	COMMERCIAL FEATURES, SPARE PARTS AND DOCUMENTATION	32
09	DIAGRAMS	33

PAR 02

PRODUCT 'S INTRODUCTION

Sterilair 3000 is an innovative air sterilizer which employs ultraviolet light with germicidal function.

It prevents the spreading of air born illnesses.

PAR 03

TECHNICAL FEATURES

SIZE		
Height	694	mm
Breadth	202	mm
Depth	202	mm
Weight	8	Kg
ELECTRIC ENERGY		
Consumption	90	Watt
Neon	4 x 25	Watt
Ozone	NO	
Certificate IEC	NEMKO	N° P02102198
Supply		
Voltage	220-240	Volt
Frequency	50	Hz
OPERATIONS		
Air speed	0.3	m / sec
Volume air passage	25	m ³ / h
Exposure (energy for decontamination room)	23000	μW sec / cm ²
Noisiness at 1 metre distance	35	dB

PAR 04

PACKING - TRANSPORT - STORAGE – INSTALLATION – DEMOLITION - DISPOSAL

**4.1
PACKING - TRANSPORT - STORAGE**

The equipment is contained in a cardboard box with shockproof recyclable material.

! Verify that the device is intact before performing the installation, inform the distributor in case it is transport- damaged. ATTENTION! Do not activate the equipment.

Keep the package for at least the whole warranty period.

The product is for exclusive INTERNAL use, it is not fit for DAMP rooms.
In order to prevent risks of fire or electric shock do not use the equipment in places where the equipment could come into contact with water.


! Do not use the equipment in rooms with risk of explosion.

The equipment can be wall-mounted or “free standing” on a base.

- It is advisable to perform the installation at an height of 1.10 metre from the floor.
- One single equipment sterilizes 25 m³ of air/hour.

NUMBER OF MACHINES REQUIRED FOR THE AIR TREATMENT IN RELATION TO THE ROOM (m³) VOLUME.

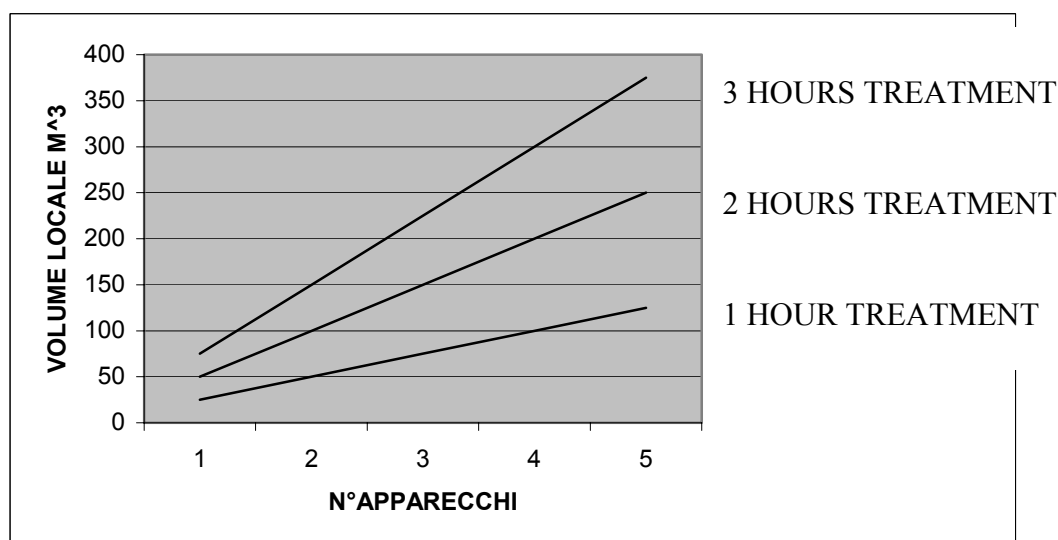SCHEDULE 1

**TYPICAL ROOMS AND LEVEL OF PROTECTION REQUIRED.
THIS CAN HELP TO FIX THE NUMBER OF MACHINES WHICH ARE NECESSARY.**

OFFICES	LOW
CHANGING ROOMS	LOW
KITCHENS AND FOOD PROCESSING	MEDIUM
CANNING INDUSTRIES	MEDIUM
FIRST-AID STATIONS	MEDIUM
LAUNDRIES	MEDIUM
ENTERTAINMENT PLACES	MEDIUM
GYMS AND FITNESS CENTRES	MEDIUM
LABORATORIES	HIGH
SURGERIES	HIGH
PRACTICES	HIGH
OPERATING THEATRES	HIGH

4.2 INSTALLATION

Read carefully this instruction manual before performing the installation and for the wall-installation stick to the instructions below.

The scheme attached to the manual is full scale for an easy installation.

BEFORE STARTING UP THE EQUIPMENT MAKE SURE THAT:

- the kind of supply indicated on the plate next to the cable (Volt-Hz) corresponds to your supply from the net.
- the outlet is equipped with a 3 ampere safety fuse to assure safety.

ITALY

220-240 Volt 50Hz

VERIFY THAT THE LABEL BEARS THE CORRECT VALUE


- Make sure that the outlet's earthing is connected.
- The supplied cable fits the equipment's needs, use only original spare parts.

Connect the cable to the equipment and then to the outlet, **perform the activation** :

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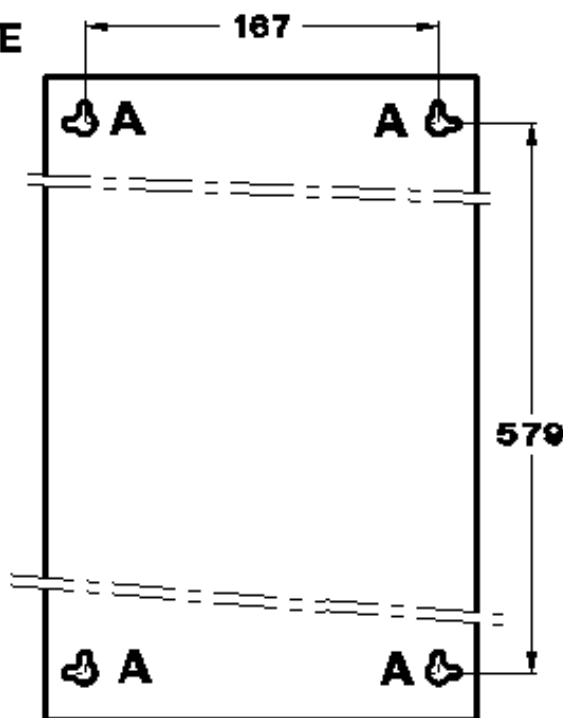STERILAIR 3000 : WALL – INSTALLATION

- 1) Read carefully the manual to verify the equipment's ideal position before starting to use the drill
 - 2) NB: the wall-installation, possibly in vertical position at about 110 cm from the ground, assures the ideal local conditions.
 - 3) Place the installation template in the position you have chosen on the wall : the middle holes must be parallel to the floor.
 - 4) For the vertical wall-installation pierce the template in the points corresponding to the holes marked with "A" to fix their position on the wall and then remove the template.
 - 5) Drill holes in the points you have marked and then insert the supplied -----into the holes.
 - 6) Insert the supplied screw leaving an opening of 3/4mm from the screw's head to the wall.
 - 7) Now place STERILAIR 3000 on the screws, hang it up on the screws' head and let it slide until it reaches the right position. Make sure that STERILAIR 3000 is held up tightly by the screws.
 - 8) Once STERILAIR 3000 is in the right position, tighten the screws.
- THANK YOU FOR YOUR CHOICE

4.3 DEMOLITION – DISPOSAL

Do not dump the machine in unauthorized dumping ground, consign it to disposal companies. Disposal must be carried out in accordance with the laws of the country where the machine is used or by means of specialized companies. The equipment consists of plastic, metallic, electromechanical and electronic components and of U.V. discharge tubes.

PAR 05**WORKING AND USE**

PRESS THE SWITCH WHICH IS ON THE MACHINE'S RIGHT SIDE (N.7, FIG.05) TO ACTIVATE IT.

GREEN LED ON (N.9 FIG.05) – POSITION I – RUNNING EQUIPMENT

GREEN LED OFF (N.9 FIG.05) - POSITION O - EQUIPMENT OFF

MICRO-ORGANISMS

The equipment has been tested in laboratories for a range of micro-organisms and their derivatives.

SALMONELLA TYPHIMURIUM

E.COLI 0157 H7

STAPHYLOCOCCUS AUREUS; NCTC 11939; EPIDEMIC METHICILLIN RESISTANT STRAIN

STAPHYLOCOCCUS AUREUS; NCTC 11940; EPIDEMIC METHICILLIN RESISTANT STRAIN

STAPHYLOCOCCUS AUREUS; NCTC 11962; ASSOCIATED WITH POST-OPERATIVE TOXIC SHOCK

BACILLUS GOLBIGII

BACILLUS SUBTILIS

BACILLUS MEGATERIUM

BACILLUS CEREUS

ENERGETIC LEVELS

THE EQUIPMENT'S EXPOSURE LEVEL (ENERGY FOR AREA UNIT) IS EQUAL TO 23000 $\mu\text{W SEC/CM}^2$.

BELOW YOU CAN FIND THE UVC LEVELS WHICH ARE NECESSARY TO ELIMINATE SPECIFIC

MICRO-ORGANISMS.

MICRO-ORGANISMS REQUIRING A PARTICULAR HIGH DOSAGE CAN BE ELIMINATED THROUGH DIFFERENT PASSAGES INSIDE THE DECONTAMINATION CHAMBER, SINCE THE IRRADIATION LEVELS COMBINE.

GERMICIDICAL ENERGY WHICH IS NECESSARY TO ELIMINATE THE MOST COMMON MICRO-ORGANISMS	ENERGY μW SEC/CM²
BACTERIA	
BACILLUS ANTHRACIS	4520
BACILLUS MEGATERIUM	1300
BACILLUS MEGATERIUM SPORES	2730
BACILLUS SUBTILIS	7100
BACILLUS SUBTILIS SPORES	12000
CORYNEBACTERIUM DIPHTHERIAE	3370
ESCHERICHIA COLI	3000
MICROCOCCUS LUTEA	19700
MICROCOCCUS SPHEROIDS	10000
NEISSERIA CATARRHALIS	4400
PROTEUS VULGARIS	2640
PSEUDOMONAS AERUGINOSA	3500
PSEUDOMONAS FLUORESCENS	8000
SALMONELLA ENTERITIDIS	4000
SALMONELLA TYPHIMURIUM	8000
SERRATIA MARCESCENS	2420
SHIGELLS PARADYSENTERIAE	1680
SPIRILLUM RUBRUM	4400
STAPHYLOCOCCUS ALBUS	1840
STAPHYLOCOCCUS AUREUS	2600
STAPHYLOCOCCUS HEMOLYTICUS	2610
STAPHYLOCOCCUS LACTIS	6150
STAPHYLOCOCCUS VIRIDANS	2000
YEASTS	
SACCHAROMYCES CEREVISIAE	6000
SACCHAROMYCES ELLIPSOIDS	6000
BREWERS YEAST	3300
BAKERS YEAST	3900
MOULDS' SPORES	
ASPERGILLUS FLEVUS	60000
ASPERGILLUS GLAUCUS	44000
MUCOR RACEMOSUS	17000
OOSPORA LACTIS	6000
PENICILLUM DIGITATUM	44000
PENICILLUM EXPANSUM	13000
PENICILLUM ROQUEFORTI	13000

PAR 06**INSTRUCTIONS FOR USER****WARNING**

- Maintenance, apart from cleaning and the replacement of UVC tubes, must be performed by skilled personnel.
- Disconnect the cable from the net before cleaning and maintenance.
- Do not stretch the cable and keep it away from sources of heat, oil and sharp parts.
- In case the cable is damaged do not use the equipment and replace the cable with an original spare part.
- Do not introduce extraneous matters inside the equipment.
- The equipment can be connected only to outlets with grounding.
- Do not use extensions to connect the equipment to the net.

STERILAIR employs short waves ultraviolet radiations at 253.7 nanometres to obtain a germicidal/antimicrobial effect.

This radiation is known as UVC.

It is invisible to the human eye and it is dangerous for the skin and particularly for the eyes.

The machine is equipped with an internal stopping device to prevent accidental expositions.

It has also been tested to prevent the dispersion of the chamber's radiations.

DO NOT TRY TO REMOVE THE SAFETY PROTECTION OR TO PERFORM AN UNAUTHORIZED MAINTENANCE, IF YOU HAVE DOUBTS CONSULT THE ASSISTANCE CENTRE.

A well-done maintenance is essential to preserve the equipment's efficiency.

Only spare parts: ORIGINAL UVC NEONS and FILTERS must be used to preserve the equipment's efficiency and safety.

Neon have a 12 months maximum efficiency and must be systematically replaced within this time interval.

PAR 07**ORDINARY AND EXTRAORDINARY MAINTENANCE****ORDINARY MAINTENANCE**

For a well-done maintenance it is necessary to clean the equipment's inside and outside at least once in a month or more frequently if the room is dusty.

Turn off the equipment and disconnect the cable before performing the maintenance.

Always use protective gloves and clean the equipment carefully with a soft antistatic duster which does not shed nap.

Make sure that the entry filter and the extractor's propellers are not covered by dust or blocked by extraneous matters.

Open the cover gently by means of the supplied key POS 1

Do not leave the key in the lock while the machine is working

Take the germicidal chamber's hood/cover off by loosening the two screws with $\frac{1}{4}$ of screw POS 2.

**FIG01**

Remove the neon one by one holding them tightly and turning them with both hands for two semi-snaps in one direction POS 3. Now you can take the neon off their base and get them out of middle of the chamber.

Take all the 4 neon off and lean them on a flat surface.

Clean gently the germicidal chamber's internal walls POS 5 with a soft antistatic duster to get them rid of dust and dirt.

Clean the UVC neon with a soft duster before the installation.

Replace the neon when they are no more efficient or in case the red led lightens (FIG.05,Pos.8)

Perform the installation with the same passages of the old neon's removal.

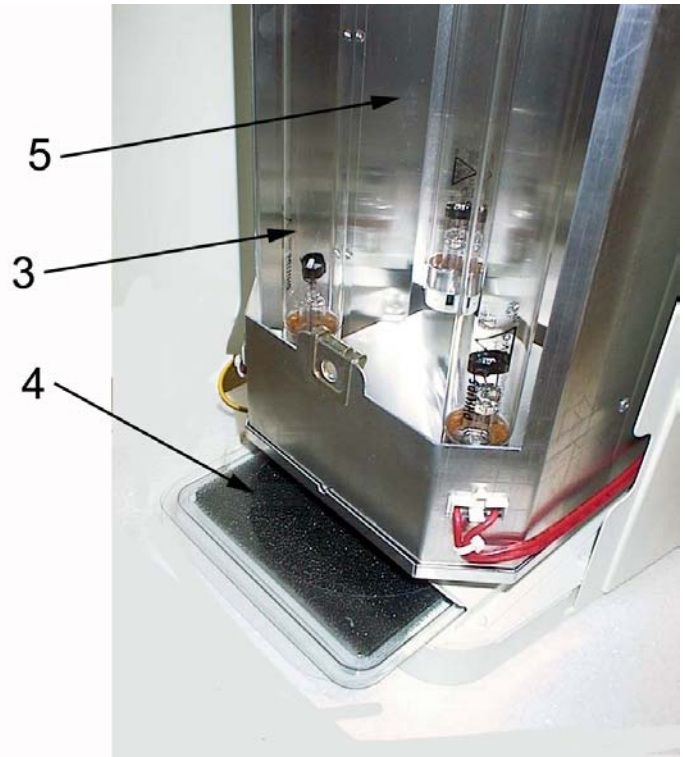
FIG02

Install the chamber's cover and tighten the 2 screws on it.

In order to clean or replace the internal filter POS 4 pull the tab and take it off completely , then clean or replace the cartridge filter.

Lock and take the key off the lock.

Keep a record of the time the neon have lasted. They should be replaced at least every 12 months. (cfr schedule 2)

EXTRAORDINARY MAINTENANCE

Verify that the blades of the extractor fan are not covered by dust or that some foreign bodies are not present.

If these problems should verify, proceed at the cleaning as follows:

Switch off and disconnect the device from the electricity before opening the cover.

Unscrew the two screws POS 6 on the cap and remove the cap.

Insert the pipe of the Hoover and clean without knocking the blades of the extractor.

Replace the cap fixing it with its screws and close the device.

If you should find some difficulties please contact the after sales service.


FIG03

FIG04

TIME - WEEKS	EFFICIENCY LEVEL %	DECAY %
0	100	0
6	93	7
12	90	10
18	88,9	11,1
24	88	12
30	87,3	12,7
36	86,6	13,4
42	86	14
48	85,5	14,5
54	85	15

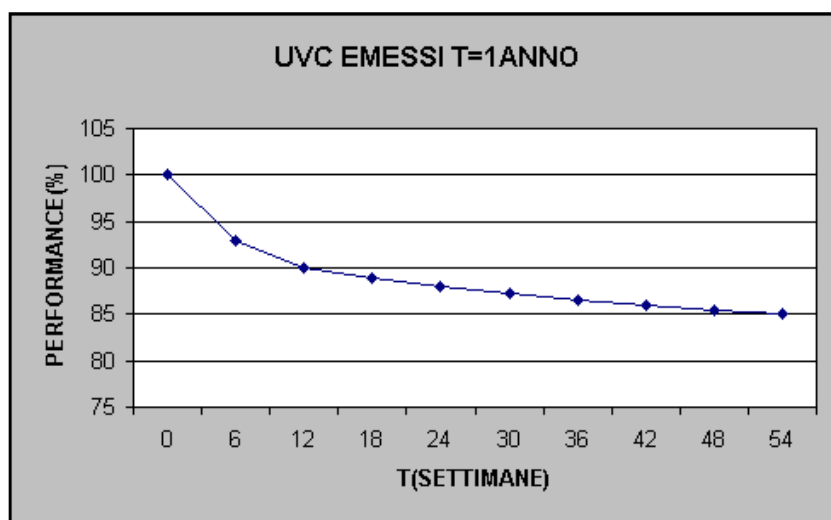TAB2

TIME IS CALCULATED
CONSIDERING 24 H OF
WORKING EVERY DAY

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A defective functioning of one UVC neon is signaled by a RED LED POS 8 (light indicator), when it lights you're recommended to replace the 4 NEON. If after having replaced the 4 NEON the RED LED is still lightened, it's necessary to contact a qualified technician.

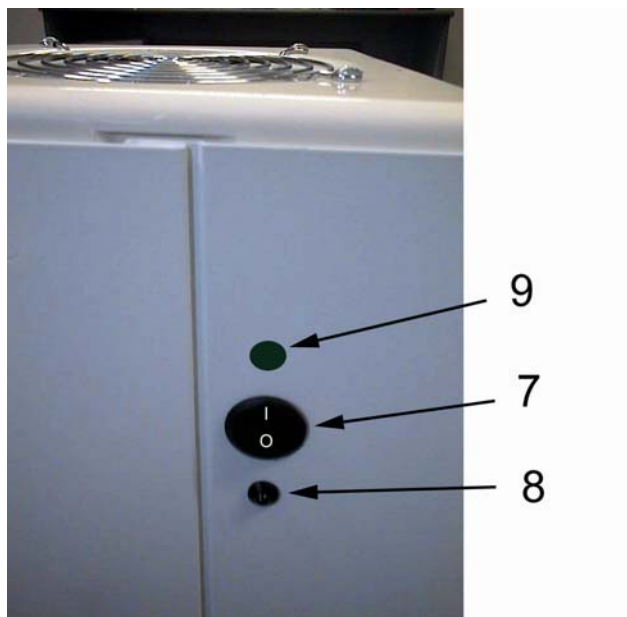
FIG05

PAR 08 COMMERCIAL FEATURES, SPARE PARTS AND DOCUMENTATION


COMPONENTS SUBJECT TO WEAR THAT CAN BE REPLACED :

DESCRIPTION	POSITION	CODE
UVC LAMPS	POS 3 FIG02	SSAA085
CARTRIDGE FILTER	POS 4 FIG02	SSAA104

PAR 9

DIAGRAMS

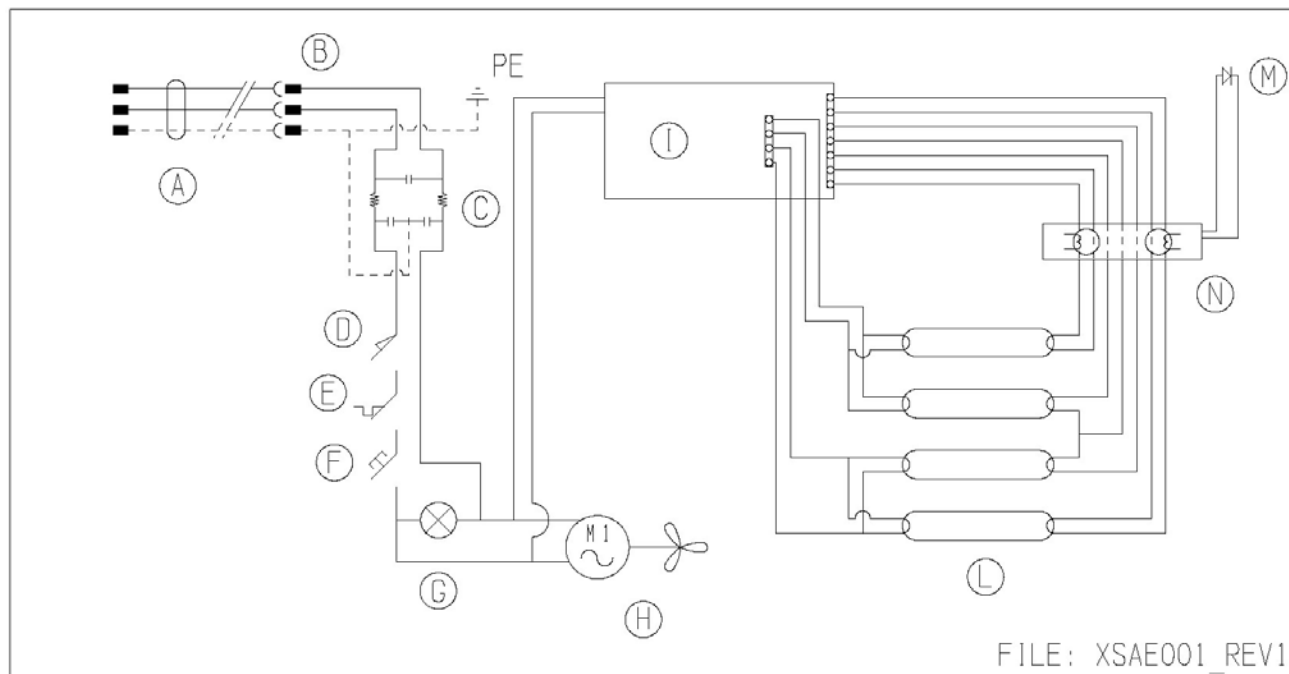ELECTRIC SCHEME

- A) CABLE
- B) OUTLET
- C) FILTER
- D) MECHANICAL SAFETY DEVICE
- E) THERMIC SAFETY DEVICE
- F) SWITCH
- G) GREEN LED
- H) EXTRACTOR
- I) LAMP SUPPLY BOARD
- L) LAMPS
- M) RED LED
- N) LED SUPPLY BOARD

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PAR 04

EMBALLAGE – TRANSPORT – STOCKAGE – INSTALLATION – DESTRUCTION

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