Safety measures

- The instrument is delivered in not sterile condition. Clean, disinfect, and sterilize the Instrument before first application and immediately after each use!
- Do not clean the instrument with compressed air!
- The Micro Compass saw can be used with a maximum motor speed of 15,000 rpm!
- The Micro Compass saw may only be attached to the motor when the motor is standing still!
- Never manipulate Instrument with running motor to prevent of risk of injuries!
- The instrument may be operated by qualified and trained personnel only!
- When storing the instrument without a saw blade attached, the chuck must be loosened!
- Improper use or repair, or failure to observe these instructions, relieve us from any obligation arising from warranty provisions or other claims.

Symbols

Conform to EU standards

Autoclavable at 135°C

Suitable for thermal disinfection

Technical data

Micro Compass Saw MSS 5000, stainless steel

Transmission ratio	3,3 : 1
Coupling to motor	ISO 3964
Shank clamping depth/width	10/0.4 mm
Max. motor speed	15,000 rpm
Working stroke	4 mm
Weight	130 g

Scope of delivery/Accessories


Micro Compass Saw MSS 5000 *	Art. No. 5040
Wrench *	Art. No. 5092
Cleaning filament *	Art. No. 1940
Irrigation tube *	Art. No. 5072
NouClean-Spray	Art. No. 1984
Nozzle for NouClean-Spray	Art. No. 1958
Oil bottle	Art. No. 1882

* included in delivery

Operation

- The Micro Compass saw can be used with a maximum motor speed of 15,000 rpm!
- Never remove fastening nut while motor is running!

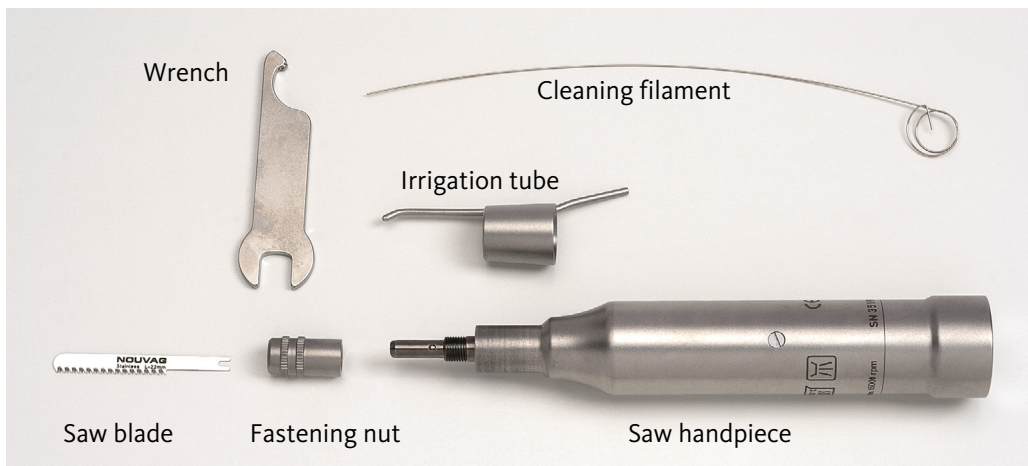Exchanging the saw blade

1. Remove irrigation tube.
2. Loosen fastening nut with the wrench by a half turn counterclockwise.
3. Attach new saw blade (not included in delivery) in the clamping mechanism and push it in all the way it goes.
4. Tighten fastening nut by turning the wrench clockwise.
5. Clip irrigation tube back on the saw.

Preparation instructions

Restrictions of treatment	Frequent reprocessing hardly affects the Micro Compass Saw. The products life end usually is determined by damage, wear and tear caused by usage.
----------------------------------	---

INSTRUCTIONS

Location of use	Wipe dirty surfaces with an expendable cloth/ tissue paper.
Storage and transport	No special requirements. Due to the risk of drying and corrosion, reprocessing must be performed without undue delay.
Cleaning preparations	<ol style="list-style-type: none"> 1. Remove tightening nut and irrigation tube under running water. 2. Dispose of saw blade expertly. 3. Remove soiling from the Micro Compass Saw by flushing it under running water. Do not put Micro Compass Saw into ultrasonic bath.
Automatic cleaning and disinfection	<p>Equipment: Washers and disinfectors with a special loading rack, which ensures handpiece connection to the washer/disinfector and channel irrigation. Use a neutral or alkaline cleaning agent in the recommended concentration.</p> <ol style="list-style-type: none"> 1. Load Micro Compass Saw in the load carrier (irrigation of the channels must be ensured). 2. Place tightening nut and irrigation tube into fine-meshed basket and close it. 3. Set cleaning cycle with effectual rinsing and cleaning. Carry out rinse cycle with deionized water. 4. For thermal disinfection perform rinse cycle with deionized water at 93°C for 10 minutes. 5. When removing the Micro Compass Saw, tightening nut and the irrigation tube, check the parts to verify whether soiling is still visible in gaps and grooves. If necessary, repeat cycle or clean manually.
Manual cleaning	<p>Equipment: Neutral cleaning agent, soft brush, cleaning filament, running water.</p> <p><i>Procedure:</i></p> <ol style="list-style-type: none"> 1. Rinse and brush off surface dirt from the Micro Compass Saw, the tightening nut and the irrigation tube under running water. 2. Apply cleaning agent with the brush on all surfaces and gaps. 3. Remove deposits in the irrigation tube by using the provided cleaning filament. 4. Thoroughly rinse Micro Compass Saw, tightening nut and irrigation tube under running water. 
Manual disinfection	<p>For manual disinfection, place Micro Compass Saw, tightening nut and irrigation tube in a RKI* verified disinfectant. (effective duration according to disinfectant instructions)</p> <p>* Consult disinfectant list of RKI (Robert Koch institute).</p>
Drying	If the drying/disinfection equipment does not provide any drying program dry Micro Compass Saw and small parts manually or in a drying chamber.
Inspection, assembly and maintenance	<p>Check for visual damage, corrosion, and wear.</p> <ol style="list-style-type: none"> 1. Screw tightening nut back on the clamping mechanism. 2. Mount spray nozzle onto the NouClean-Spray can. 3. Insert the spray nozzle at the coupling connection of the Micro Compass Saw all the way it goes. 4. Perform a spray puff of about 3 seconds duration and clean spilling with a moistened cloth. 
Art. no.: 1958	
Packaging	<p>Single: Pack the Micro Compass Saw in an individual sterile supply package. The bag must be large enough that the seal is not under tension.</p> <p>Sets: Arrange Micro Compass Saws into the foreseen trays or put them onto an all-purpose sterilization tray.</p>
Sterilization	<p>Autoclave in vacuum autoclaves at 135°C for at least 5 minutes*. Do not exceed the maximum load of the sterilizer, if several instruments are sterilized during one sterilization cycle. Observe a drying phase for autoclaves without post-vacuum. Allow the Micro Compass Saw to dry in the bag with the paper side up.</p> <p>*The temperature hold times must comply with the local guidelines and standards.</p>
Storage	No special requirements. In case the sterilized Micro Compass Saw is not used immediately after sterilization the sterilization packaging must be marked with the sterilization date. Adding a sterility indicator is recommended.

The above-mentioned instructions were validated as suitable from NOUVAG AG. The person accomplishing the treatment is responsible that the treatment that actually took place leads to the desired results in the treatment facility with the available equipment, materials, and personnel. To ensure this it is normally necessary to validate and supervise the process. Each deviation from the instructions provided should be evaluated carefully with regard to effects and negative consequences.

Troubleshooting

Problem	Cause	Solution
Motor is running, but Saw does not move	Contra angle is not connected properly to the Motor	Press Micro Compass Saw firmly on Motor, until it snaps in. Check fitting.
Saw blade functions irregularly	Saw blade is not clamped in or tighten properly	Fix tool and tighten chuck
Contra angle is noisy	Ball bearings are not lubricated or dirty	Spray the contra angle interior with NouClean-Spray

Spare parts: Saw blades/rasp

article	Art. No.	length	thickness	PU*
	5041	11 mm	0.4 mm	6 x
	5042	14 mm	0.4 mm	6 x
	5043	18 mm	0.4 mm	6 x
	5044	22 mm	0.4 mm	6 x
	5045	26 mm	0.4 mm	6 x
	5105	17 mm	0.4 mm	6 x
	5106	21 mm	0.4 mm	6 x
	5046	hardened steel		1 x

* PU = Packaging unit

Service centers

Switzerland

Nouvag AG • St.Gallerstr. 23-25 • CH-9403 Goldach
Phone +41 (0)71 846 66 00 • Fax +41 (0)71 845 35 36
info@nouvag.com • www.nouvag.com

Germany

Nouvag GmbH • Schulthaißstrasse 15 • D-78462 Konstanz
Phone +49 (0)7531 1290-0 • Fax +49 (0)7531 1290-12
info-de@nouvag.com • www.nouvag.com

Nouvag Service centers worldwide: www.nouvag.com

USA

Nouvag USA Inc. • 18058 Albyn Court • Lake Hughes, CA 93532 • USA
Phone +1 (661) 724 0217 • Fax +1 (661) 724 1590 • Toll free (800) 673 7427
paul@nouvagusa.com • www.nouvag.com

For service, maintenance, spare parts and general inquiries please contact your local dealer or representative.

Disposal:

Disposal of device, components and accessories must strictly conform to local laws and regulations as set out by the relevant authorities.

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