[**comfort zone**]

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Guarantee conditions:

- 1) **DURATION:** the product is covered by a 12-month (twelve) guarantee.
- 2) **EXCLUSION:** the guarantee excludes:
 - a) faults originating from non-performance of scheduled maintenance and due to negligence or improper use of the product by the User;
 - b) periodic checks and maintenance;
 - c) repair or replacement of parts subject to wear that are fragile or have an unpredictable life, unless they prove faulty at the time of their delivery;
 - d) faults caused by personnel or originating from technical personnel transfer and transportation;
 - e) faults or damage originating from improper use or operating errors;
 - f) faults or damage originating from contamination in the water and air infeed systems, extraordinary chemical agents or electrical events;
 - g) faults or damage originating from the use of detergents, disinfecting agents, sterilizing fluids or processes that are not described explicitly in the instruction manual for operation and maintenance;
 - h) natural color change of plastic components.
- 3) **INSTALLATION AND TESTING REPORT:** an essential prerequisite for guarantee validity is the return of the installation and testing report after all its sections have been filled in by the User and the Fitter. The report must be returned within 15 days after installation, otherwise the guarantee becomes null and void.
- 4) **LIMITS:** the guarantee entitles its holder to repairs or replacement of faulty components free of charge. The right to replace the entire machine is excluded. As for components either applied or added to TECNO-GAZ products or by third parties, having their own guarantee certificate, the conditions, limits and exclusions indicated in the relevant certificates apply.
Under no circumstances shall carrying out of one or more than one repair during the guarantee term change the guarantee expiry date.
- 5) **CLAIMS:** in the event of the Purchaser making a claim relating to the application of the guarantee and the quality or conditions of the delivered equipment, the said Purchaser shall in no way suspend and/or delay payments.
- 6) **DISPUTES:** if a dispute arises on the application and interpretation of this Certificate of guarantee, the Court of PARMA (Italy) shall be the court of jurisdiction, regardless of the location in which the purchase agreement was stipulated.
- 7) **DEROGATIONS:** any special derogations from the guarantee terms described in this section shall not imply the granting to the Purchaser of any right and shall be considered as granted for a specific case only.
- 8) **MISCELLANEOUS:** for any other issue, which is not covered by this Certificate of Guarantee, the rules described in the Civil Code and existing relevant Laws in the Republic of Italy apply.

Loss of the guarantee

The guarantee becomes null and void in the event that:

- a) the equipment presents damage caused by a fall, exposure to flames, liquid spillage, natural events, bad weather conditions or by any other cause that is not due to manufacturing defects;
- b) installation has not been performed in line with TECNO-GAZ instructions and by authorized personnel;
- c) the equipment is repaired, modified or tampered with by the Purchaser or by non-authorized third parties;
- d) when asking for an intervention under guarantee, the product serial number has been removed, deleted, counterfeited, etc.;
- e) the installation and testing report is not returned suitably filled in and signed within 15 days from installation;
- f) the Purchaser suspends and/or delays payment of amounts due, for any reason whatsoever, relating to the purchase and/or maintenance of the equipment;
- g) scheduled periodic maintenance or any other instruction envisaged in the manual for operation and maintenance is not carried out.

Guarantee conditions:

- 1) **DURATION:** the product is covered by a 12-month (twelve) guarantee.
- 2) **EXCLUSION:** the guarantee excludes:
 - a) faults originating from non-performance of scheduled maintenance and due to negligence or improper use of the product by the User;
 - b) periodic checks and maintenance;
 - c) repair or replacement of parts subject to wear that are fragile or have an unpredictable life, unless they prove faulty at the time of their delivery;
 - d) faults caused by personnel or originating from technical personnel transfer and transportation;
 - e) faults or damage originating from improper use or operating errors;
 - f) faults or damage originating from contamination in the water and air infeed systems, extraordinary chemical agents or electrical events;
 - g) faults or damage originating from the use of detergents, disinfecting agents, sterilizing fluids or processes that are not described explicitly in the instruction manual for operation and maintenance;
 - h) natural color change of plastic components.
- 3) **INSTALLATION AND TESTING REPORT:** an essential prerequisite for guarantee validity is the return of the installation and testing report after all its sections have been filled in by the User and the Fitter. The report must be returned within 15 days after installation, otherwise the guarantee becomes null and void.
- 4) **LIMITS:** the guarantee entitles its holder to repairs or replacement of faulty components free of charge. The right to replace the entire machine is excluded. As for components either applied or added to TECNO-GAZ products or by third parties, having their own guarantee certificate, the conditions, limits and exclusions indicated in the relevant certificates apply.
Under no circumstances shall carrying out of one or more than one repair during the guarantee term change the guarantee expiry date.
- 5) **CLAIMS:** in the event of the Purchaser making a claim relating to the application of the guarantee and the quality or conditions of the delivered equipment, the said Purchaser shall in no way suspend and/or delay payments.
- 6) **DISPUTES:** if a dispute arises on the application and interpretation of this Certificate of guarantee, the Court of PARMA (Italy) shall be the court of jurisdiction, regardless of the location in which the purchase agreement was stipulated.
- 7) **DEROGATIONS:** any special derogations from the guarantee terms described in this section shall not imply the granting to the Purchaser of any right and shall be considered as granted for a specific case only.
- 8) **MISCELLANEOUS:** for any other issue, which is not covered by this Certificate of Guarantee, the rules described in the Civil Code and existing relevant Laws in the Republic of Italy apply.

Loss of the guarantee

The guarantee becomes null and void in the event that:

- a) the equipment presents damage caused by a fall, exposure to flames, liquid spillage, natural events, bad weather conditions or by any other cause that is not due to manufacturing defects;
- b) installation has not been performed in line with TECNO-GAZ instructions and by authorized personnel;
- c) the equipment is repaired, modified or tampered with by the Purchaser or by non-authorized third parties;
- d) when asking for an intervention under guarantee, the product serial number has been removed, deleted, counterfeited, etc.;
- e) the installation and testing report is not returned suitably filled in and signed within 15 days from installation;
- f) the Purchaser suspends and/or delays payment of amounts due, for any reason whatsoever, relating to the purchase and/or maintenance of the equipment;
- g) scheduled periodic maintenance or any other instruction envisaged in the manual for operation and maintenance is not carried out.

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Delivery terms

1. All goods delivered must be packaged in their original packaging. If the goods are delivered in a non-genuine packaging, a charge will be applied to restore the original packaging.
2. All goods must be delivered CARRIAGE PAID.
3. The client shall ask a written authorization to TECNO-GAZ sales dept. before returning the goods by means of the "Form for Client's Returned Goods Management" (CM-M-002) in compliance with the requirements of TECNO-GAZ Quality Management System (ref. CM-P-003, CM-I-009).
4. The document accompanying the returned goods shall show:
 - a. the TECNO-GAZ product ode/item no.;
 - b. the number of the TECNO-GAZ purchase invoice or shipment document;
 - c. the product serial number and batch number;
 - d. a description of the goods returned;
 - e. the number of the Form for Client's Returned Goods Management" (CM-M-002) or a copy of the same attached.
5. Returned goods are considered as accepted only after they have been examined. If they prove non-conforming, they may be sent back to the consignor at TECNO-GAZ full discretion.
6. Charges will be applied to restore and test the returned product.
7. Parts replaced under guarantee must be returned, carriage paid, to TECNO-GAZ.
8. If the parts are not returned, their cost will automatically be debited to the recipient.
9. TECNO-GAZ does not accept goods returns from end users.
10. Goods sent to TECNO-GAZ for repair are managed as returned goods.

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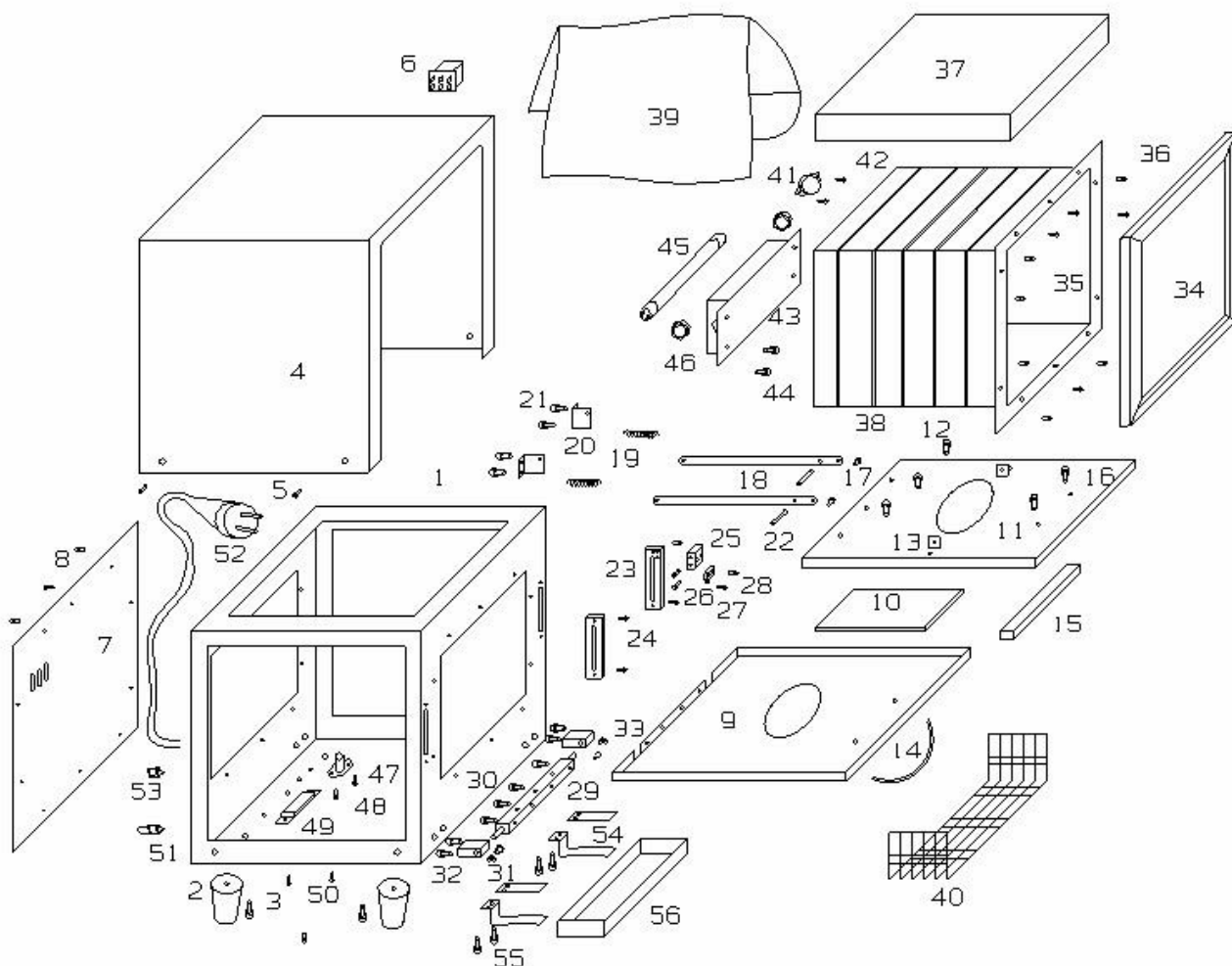
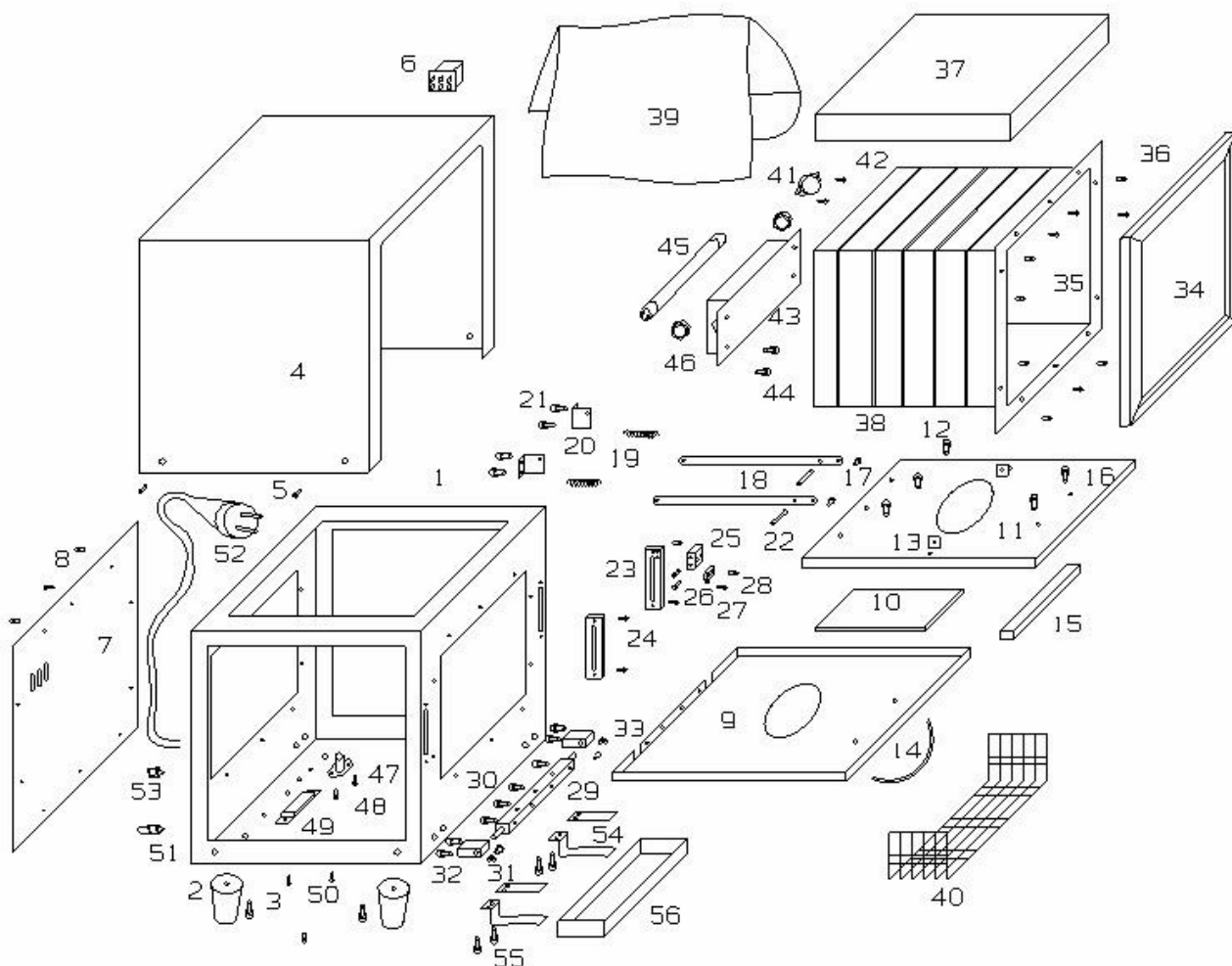
10. Schema

10. Schema

INDEX		
Pr.	CONTENTS	Page
3.	Product presentation	17
3.1	Scope of the product	17
4.	Technical specifications	17
4.1	Mechanical specifications	17
4.2	Electrical specifications	17
5.	Installation	18
5.1	Positioning	18
5.2	Installation (foundation)	18
5.3	Connection and putting into service	18
5.4	Machine cleaning and disinfection	18
5.5	Machine cleaning and disinfection	19
6.	Machine operation and use	19
6.1	Description of machine operation	19
6.2	Operating and environmental limits	20
7.	Routine and scheduled maintenance and extraordinary maintenance	20
8.	Diagnostics and troubleshooting	20
9.	Commercial components, spare parts and relevant documentation	21
10.	Scheme	22

INDEX		
Pr.	CONTENTS	Page
3.	Product presentation	17
3.1	Scope of the product	17
4.	Technical specifications	17
4.1	Mechanical specifications	17
4.2	Electrical specifications	17
5.	Installation	18
5.1	Positioning	18
5.2	Installation (foundation)	18
5.3	Connection and putting into service	18
5.4	Machine cleaning and disinfection	18
5.5	Machine cleaning and disinfection	19
6.	Machine operation and use	19
6.1	Description of machine operation	19
6.2	Operating and environmental limits	20
7.	Routine and scheduled maintenance and extraordinary maintenance	20
8.	Diagnostics and troubleshooting	20
9.	Commercial components, spare parts and relevant documentation	21
10.	Scheme	22

3. Product presentation

3.1 Scope of the product

The product has been designed for use in beauty parlours.

The purposes of the product are:

- towel heating
- bactericidal action (UV rays)

	<p>Read the instructions for use carefully before installing and using the device.</p> <p>Illegitimate use of the device and operator negligence can cause damage to persons, the environment and property; TECNO-GAZ cannot be held responsible for said events.</p> <p>The operator who uses the device must have been correctly trained.</p>
---	---

4. Technical specifications

4.1 Mechanical specifications

OVEN SIZE		
Width	470	mm
Depth	240	mm
Height	350	mm
Weight	12	kg
Colour	RAL 8019	

4.2 Electrical specifications

Single-phase supply	230 ~	V
Frequency	50	Hz
Heating power	100	W
UV lamp power (germicide)	8	W
Maximum absorbed power	120	W

3. Product presentation

3.1 Scope of the product

The product has been designed for use in beauty parlours.

The purposes of the product are:

- towel heating
- bactericidal action (UV rays)

	<p>Read the instructions for use carefully before installing and using the device.</p> <p>Illegitimate use of the device and operator negligence can cause damage to persons, the environment and property; TECNO-GAZ cannot be held responsible for said events.</p> <p>The operator who uses the device must have been correctly trained.</p>
---	---

4. Technical specifications

4.1 Mechanical specifications

OVEN SIZE		
Width	470	mm
Depth	240	mm
Height	350	mm
Weight	12	kg
Colour	RAL 8019	

4.2 Electrical specifications

Single-phase supply	230 ~	V
Frequency	50	Hz
Heating power	100	W
UV lamp power (germicide)	8	W
Maximum absorbed power	120	W

5. Installation

5.1 Positioning

“HOT CAB” is put in a cellophane bag and then arranged in a corrugated cardboard box and prevented from moving about with polystyrene foam.

It must be transported following the rules for the transportation of fragile goods.

The original packaging should be kept for the duration of the guarantee period, since only in this container is it possible to avoid damage in transit.

Storage Temperature	- 20 ÷ 45 °C
Humidity	10 ÷ 100 %
Pressure	500 ÷ 1060 hpa

Failure to comply with the above will terminate the guarantee.

Disposal of the packaging components must be carried out in compliance with the current regulations in the country where the device is used.

5.2 Installation (foundation)

Before installing, check the integrity of the device; if there are signs of transit damage, please notify the dealer.

WARNING! Do not switch on the device.

Position the oven on a level surface resting on its feet.

5.3 Connection and putting into service

The electrical connection must be made in accordance with current technical regulations.

Make sure the measured mains voltage value corresponds to that written on the machine's identification plate.

Make sure the electrical systems are efficiently earthed.

5.4 Machine cleaning and disinfection

For cleaning, please refer to the health and hygiene regulations.

- Switch off the machine and pull the plug out of the electrical socket.
- Use a sponge dampened with soapy water and dry with a clean dry cloth.
- Do not use harsh detergents, which could irretrievably alter the appearance of the plastic parts of the machine.
- Do not spray directly onto the machine, as this would risk damaging the electrical part inside it.

5. Installation

5.1 Positioning

“HOT CAB” is put in a cellophane bag and then arranged in a corrugated cardboard box and prevented from moving about with polystyrene foam.

It must be transported following the rules for the transportation of fragile goods.

The original packaging should be kept for the duration of the guarantee period, since only in this container is it possible to avoid damage in transit.

Storage Temperature	- 20 ÷ 45 °C
Humidity	10 ÷ 100 %
Pressure	500 ÷ 1060 hpa

Failure to comply with the above will terminate the guarantee.

Disposal of the packaging components must be carried out in compliance with the current regulations in the country where the device is used.

5.2 Installation (foundation)

Before installing, check the integrity of the device; if there are signs of transit damage, please notify the dealer.

WARNING! Do not switch on the device.

Position the oven on a level surface resting on its feet.

5.3 Connection and putting into service

The electrical connection must be made in accordance with current technical regulations.

Make sure the measured mains voltage value corresponds to that written on the machine's identification plate.

Make sure the electrical systems are efficiently earthed.

5.4 Machine cleaning and disinfection

For cleaning, please refer to the health and hygiene regulations.

- Switch off the machine and pull the plug out of the electrical socket.
- Use a sponge dampened with soapy water and dry with a clean dry cloth.
- Do not use harsh detergents, which could irretrievably alter the appearance of the plastic parts of the machine.
- Do not spray directly onto the machine, as this would risk damaging the electrical part inside it.

5.5 Machine cleaning and disinfection

Directive on waste from electrical and electronic equipment (WEEE).

Pursuant to DIRECTIVE 2002/96/EC, this symbol shows that the product must not be disposed of as urban waste at the end of its operating life. The product must be taken to centers specialized in separate collection of electric and electronic equipment or to a dealer, when a similar product is purchased. The machine holder shall be responsible for bringing it to collection centers. For further information about collection systems, address the local waste disposal service.

Correct disposal of the equipment, which is no longer used, prevents negative consequences for the environment and human health.

Abuses will be prosecuted by law.

6. Machine operation and use

6.1 Description of machine operation

Use 100% cotton towels (approx size 18x18 cm).

Place them in the basket and put them inside the oven.

With the two switches in position O, plug the device into the mains socket.

When you set the first switch to position I, the switch will light up and the oven heating will come on.

About 15 minutes are required for the oven to reach optimum working temperature, and the machine will then stay at the same temperature, through a control device, until the switch is switched off.

The second switch switches on the UV germicide lamp, and you can see that it has come on by looking through the window. This is used to keep towels clean and avoid unpleasant smells.

The lamp switches off automatically when the door is opened and comes on again when the door is closed.

The water collection tray underneath can be removed for emptying and then it should be put back in position.

Do not put items in the oven other than the towels indicated above.

Do not clean the internal container with chemical detergents. Use anti-scale solutions but not those that are aggressive to aluminium.

5.5 Machine cleaning and disinfection

Directive on waste from electrical and electronic equipment (WEEE).

Pursuant to DIRECTIVE 2002/96/EC, this symbol shows that the product must not be disposed of as urban waste at the end of its operating life. The product must be taken to centers specialized in separate collection of electric and electronic equipment or to a dealer, when a similar product is purchased. The machine holder shall be responsible for bringing it to collection centers. For further information about collection systems, address the local waste disposal service.

Correct disposal of the equipment, which is no longer used, prevents negative consequences for the environment and human health.

Abuses will be prosecuted by law.

6. Machine operation and use

6.1 Description of machine operation

Use 100% cotton towels (approx size 18x18 cm).

Place them in the basket and put them inside the oven.

With the two switches in position O, plug the device into the mains socket.

When you set the first switch to position I, the switch will light up and the oven heating will come on.

About 15 minutes are required for the oven to reach optimum working temperature, and the machine will then stay at the same temperature, through a control device, until the switch is switched off.

The second switch switches on the UV germicide lamp, and you can see that it has come on by looking through the window. This is used to keep towels clean and avoid unpleasant smells.

The lamp switches off automatically when the door is opened and comes on again when the door is closed.

The water collection tray underneath can be removed for emptying and then it should be put back in position.

Do not put items in the oven other than the towels indicated above.

Do not clean the internal container with chemical detergents. Use anti-scale solutions but not those that are aggressive to aluminium.

6.2 Operating and environmental limits

Storage Temperature	- 20 ÷ 45 °C
Humidity	10 ÷ 100 %
Pressure	500 ÷ 1060 hpa

7. Routine and scheduled maintenance and extraordinary maintenance

To replace the heating element, the machine should be returned to the manufacturer.

To replace the UV germicide lamp, the machine should be returned to the manufacturer.

Switch off the machine and pull the plug out of the socket.

To replace the thermostat: unscrew the 10 self-tapping screws and remove the back plate, pull out the connectors on the thermostat, remove the screws joining the thermostat to the aluminium box, and replace the thermostat by repeating the previous the operations in reverse order to reassemble.

To replace the starter or reactor of the lamp: unscrew the 10 screws and remove the back plate, remove the wire connections, remove the screws joining the starter or the reactor to the bottom of the oven and replace them by repeating the previous operations in reverse order.

8. Diagnostics and troubleshooting

Problem:	Causes / Remedies:
The machine does not heat	Check that there is voltage and the power cable
	Check the fuse
	Check the switch
	Check the heating element
The UV germicide lamp does not come on	Check the switch
	Check the starter and the reactor
	Check the closed-door microswitch
	Check the lamp

If the appliance works abnormally or not at all, please contact an authorized service centre. It is not possible to work directly on the appliance.

6.2 Operating and environmental limits

Storage Temperature	- 20 ÷ 45 °C
Humidity	10 ÷ 100 %
Pressure	500 ÷ 1060 hpa

7. Routine and scheduled maintenance and extraordinary maintenance

To replace the heating element, the machine should be returned to the manufacturer.

To replace the UV germicide lamp, the machine should be returned to the manufacturer.

Switch off the machine and pull the plug out of the socket.

To replace the thermostat: unscrew the 10 self-tapping screws and remove the back plate, pull out the connectors on the thermostat, remove the screws joining the thermostat to the aluminium box, and replace the thermostat by repeating the previous the operations in reverse order to reassemble.

To replace the starter or reactor of the lamp: unscrew the 10 screws and remove the back plate, remove the wire connections, remove the screws joining the starter or the reactor to the bottom of the oven and replace them by repeating the previous operations in reverse order.

8. Diagnostics and troubleshooting

Problem:	Causes / Remedies:
The machine does not heat	Check that there is voltage and the power cable
	Check the fuse
	Check the switch
	Check the heating element
The UV germicide lamp does not come on	Check the switch
	Check the starter and the reactor
	Check the closed-door microswitch
	Check the lamp

If the appliance works abnormally or not at all, please contact an authorized service centre. It is not possible to work directly on the appliance.

9. Diagnostics and troubleshooting

List of parts:

1	Painted box
2	Feet
3	Foot screws
4	Painted lid
5	Lid screws
6	Double switch
7	Painted back plate
8	Back plate screws
9	Painted door
10	Transparent anti-UV polycarbonate
11	Door closing plate
12	Door closing plate screws
13	Door cam brackets
14	Handle
15	Handle spacers
16	Handle screws
17	Door cam pins
18	Door cams
19	Door springs
20	Spring hooking bracket
21	Spring hooking bracket screws
22	Door cam stop pin
23	Door cam track
24	Cam track screws
25	Door microswitch holder
26	Microswitch holder screws
27	Door microswitch
28	Microswitch screws

29	Door fulcrum
30	Fulcrum screws
31	Door hinges
32	Hinge screws
33	Hinge bushes
34	Gasket
35	Heating box
36	Heating box screws
37	Insulator
38	Heating element
39	Heating element cover
40	Grill mesh
41	Thermostat
42	Thermostat screws
43	Lamp box
44	Lamp box screws
45	UV germicide lamp
46	Lamp rubber
47	Starter
48	Starter screws
49	Reactor
50	Reactor screws
51	Fuse holder and fuse
52	Power cable with plug
53	Cable gland
54	Vessel holding plates
55	Vessel holding plate screws
56	Door fulcrum

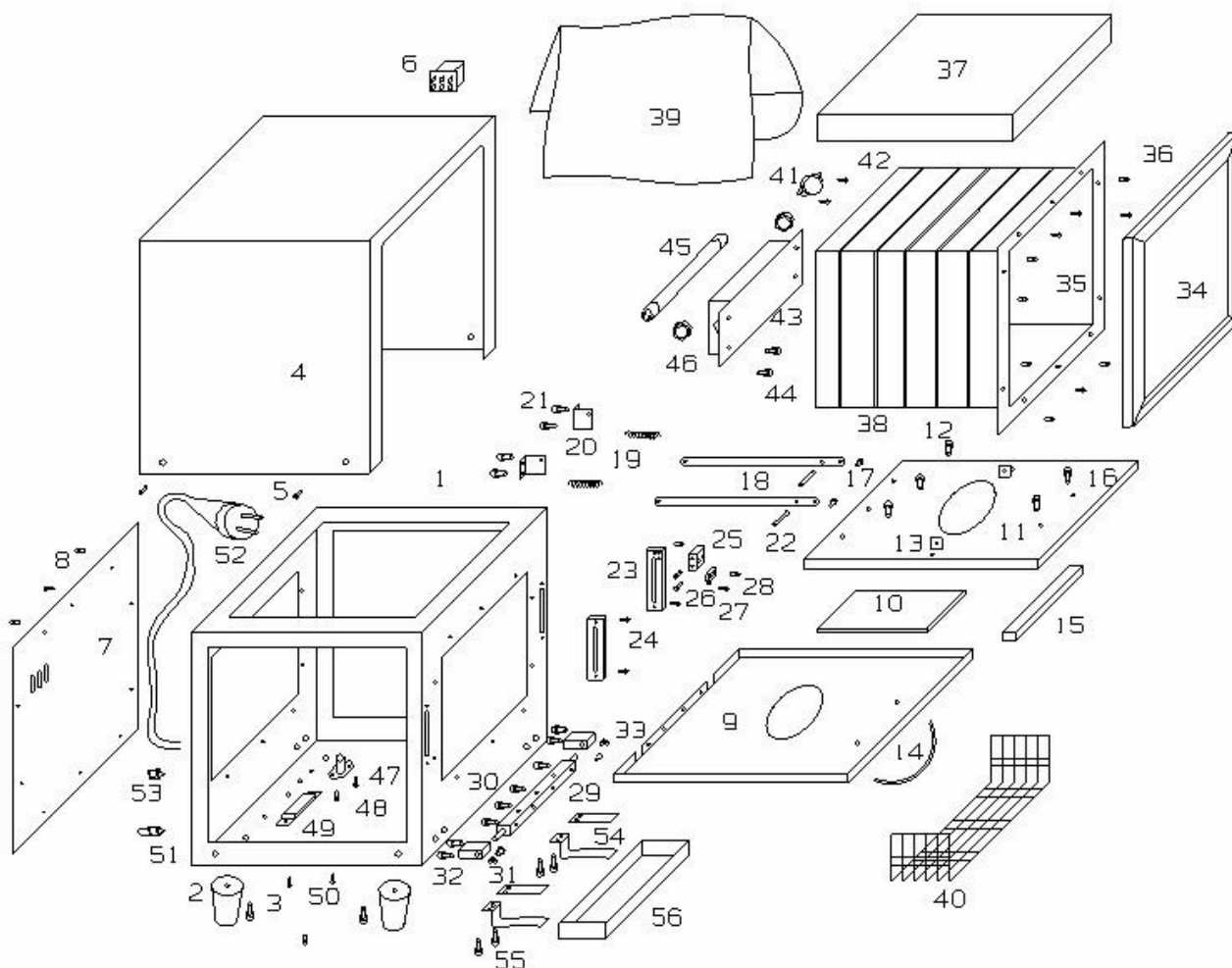
9. Diagnostics and troubleshooting

List of parts:

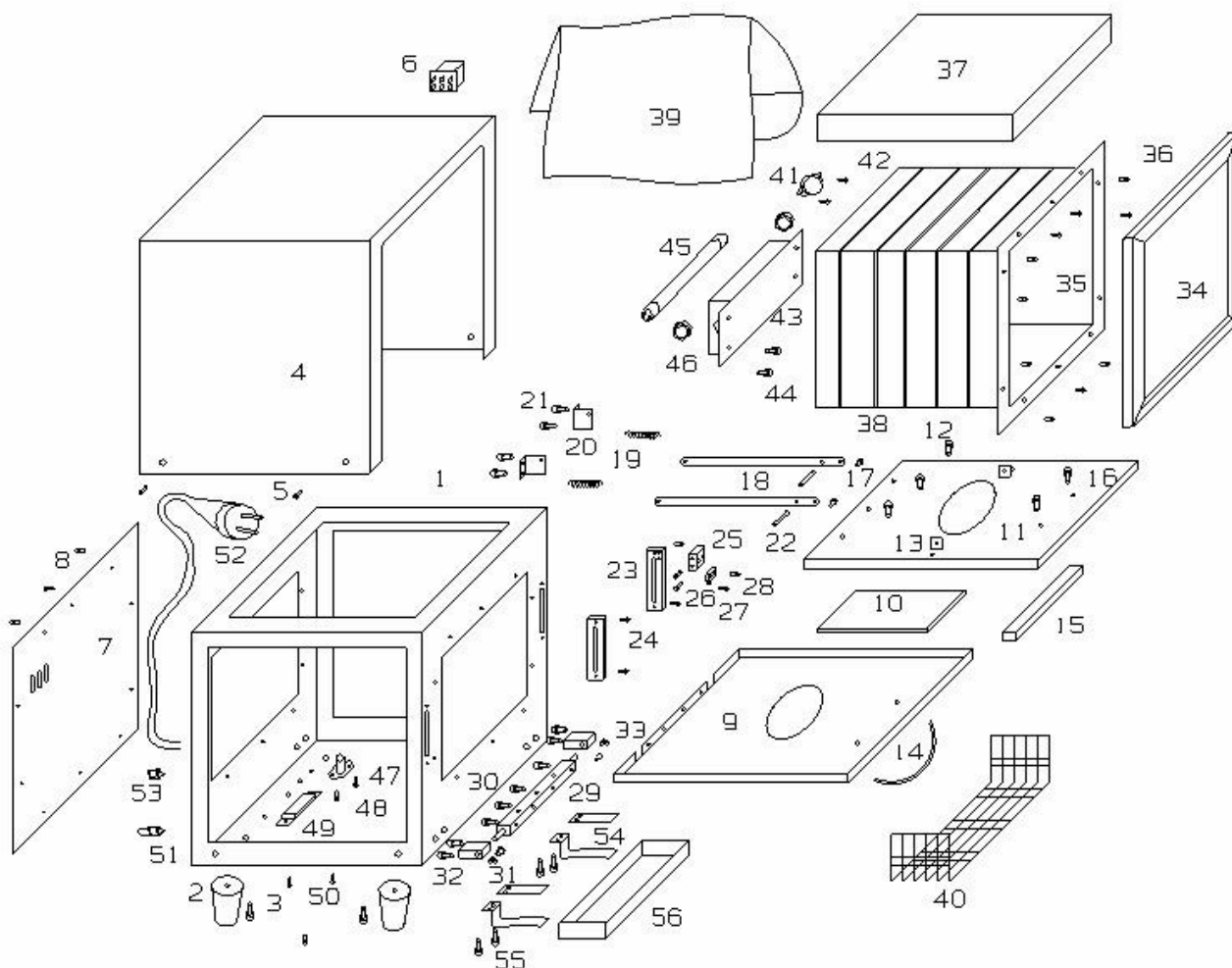
1	Painted box
2	Feet
3	Foot screws
4	Painted lid
5	Lid screws
6	Double switch
7	Painted back plate
8	Back plate screws
9	Painted door
10	Transparent anti-UV polycarbonate
11	Door closing plate
12	Door closing plate screws
13	Door cam brackets
14	Handle
15	Handle spacers
16	Handle screws
17	Door cam pins
18	Door cams
19	Door springs
20	Spring hooking bracket
21	Spring hooking bracket screws
22	Door cam stop pin
23	Door cam track
24	Cam track screws
25	Door microswitch holder
26	Microswitch holder screws
27	Door microswitch
28	Microswitch screws

29	Door fulcrum
30	Fulcrum screws
31	Door hinges
32	Hinge screws
33	Hinge bushes
34	Gasket
35	Heating box
36	Heating box screws
37	Insulator
38	Heating element
39	Heating element cover
40	Grill mesh
41	Thermostat
42	Thermostat screws
43	Lamp box
44	Lamp box screws
45	UV germicide lamp
46	Lamp rubber
47	Starter
48	Starter screws
49	Reactor
50	Reactor screws
51	Fuse holder and fuse
52	Power cable with plug
53	Cable gland
54	Vessel holding plates
55	Vessel holding plate screws
56	Door fulcrum

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GB

This manual must always be kept with the product, in complying with the Directives of European Community.

DAVINES S.p.A reserves the right to modify the enclosed document without notice.

DAVINES S.p.A reserves the property of the document and forbids others to use it or spread it without its approval.

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DAVINES S.p.A. si riserva il diritto di apporre modifiche al presente documento senza dare alcun pre-avviso.