Adhesive Resin Cement

ESTECM

CE 0120
REF BIS02

ENGLISH

Read all information, precautions and notes before using.

■ PRODUCT DESCRIPTION AND GENERAL INFORMATION

1. ESTECM is a dual-cure (light and/or self-cure), radiopaque, adhesive resin cement system with excellent handling, aesthetic and adhesive properties to teeth and all prosthetic materials. The main components of the ESTECM KIT are the PASTE (A/B) and primers. The PASTE is dispensed in two auto-mix syringes. The primer component in the KIT consists of (Universal, Clear, and White) which are used to bond to ceramic and composite materials. (Universal, Clear, and White) are the prime components in the KIT of ESTECM. The PASTE is dispensed in two auto-mix syringes. The primer component in the KIT of ESTECM is BOND A and BOND B and TOKUYAMA UNIVERSAL PRIMER (A/B). ESTELINK enhances the adhesion of PASTE to tooth structure. TOKUYAMA UNIVERSAL PRIMER enhances the adhesion of PASTE to prosthetic materials.

2. The PASTE contains Bisphenol A diglycidylmethacrylate (Bis-GMA), triethyleneglycol dimethacrylate (Bis-MPEPP), Peroxide, Camphorquinone and Silica-Zirconia Filler (Filler Load 74 w%).

3. The ESTELINK contains Phosphoric acid monomer, Bisphenol A di(2-hydroxy propoxy) dimethacrylate (Bis-GMA), triethyleneglycol dimethacrylate (TEGDMA), 1,6-bis (methacryloyloxy) hexane (UDMA), ethanol and acetone.

■ CEMENTATION

- Cementation of crowns, bridges, inlays, and onlays made of glass/oxide ceramics (porcelain, zirconia, alumina), metal/alloys (precious and non-precious) and resin materials including inorganic filler (composite materials).

- Repair of fractured porcelain fixed to metal crowns and all ceramic restorations.

- Cementation of veneers.

- Cementation of adhesion bridges.

- Cementation of metal or resin cores, metal or glass-fiber posts.

■ CEMENTATION OF CROWNS, BRIDGES, INLAYS, AND ONLAYS MADE OF GLASS/OXIDE CERAMICS (PORCELAIN, ZIRCONIA, ALUMINA), METAL/ALLOYS (PRECIOUS AND NON-PRECIOUS) AND RESIN MATERIALS INCLUDING INORGANIC FILLER (COMPOSITE MATERIALS)

- Cementation of crowns, bridges, inlays, and onlays made of glass/oxide ceramics (porcelain, zirconia, alumina), metal/alloys (precious and non-precious) and resin materials including inorganic filler (composite materials).

- Repair of fractured porcelain fixed to metal crowns and all ceramic restorations.

- Cementation of veneers.

- Cementation of adhesion bridges.

- Cementation of metal or resin cores, metal or glass-fiber posts.

■ PRECAUTIONS

1) Use ESTELINK and/or TOKUYAMA UNIVERSAL PRIMER as primers for the PASTE. DO NOT use other primers or adhesives as primer for the adhesive surfaces.

2) DO NOT use ESTECM for any purpose other than those listed in these instructions. Use ESTECM only as dental keratin.

3) ESTECM is designed for sale and use by licensed dental care professionals only. It is not designed for sale nor is it suitable for use by non-dental care professionals.

4) DO NOT use the ESTECM KIT, PASTE, ESTELINK or TOKUYAMA UNIVERSAL PRIMER if the safety seals are broken or appear to have been tampered with.

5) If ESTECM causes an allergic reaction or oversensitivity, discontinued its use immediately.

6) Use examination gloves (plastic, vinyl or latex) at all times when handling ESTECM to avoid the possibility of allergic reactions from methacrylic monomers. Note: Certain substances/materials may penetrate through examination gloves. If the PASTE, ESTELINK or TOKUYAMA UNIVERSAL PRIMER comes in contact with the examination gloves, remove and dispose of the gloves, wash hands thoroughly with water as soon as possible.

7) Avoid contact of the PASTE, ESTELINK or TOKUYAMA UNIVERSAL PRIMER with eyes, mucous membranes, skin and clothing.

- If the PASTE, ESTELINK or TOKUYAMA UNIVERSAL PRIMER comes in contact with the eyes, flush eyes with water and immediately contact an ophthalmologist.

- If the PASTE, ESTELINK or TOKUYAMA UNIVERSAL PRIMER comes in contact with the mucosal membrane, wipe the affected area immediately, and thoroughly flush with water. Use a clean aspirator to prevent spatter of the primer.

- If saliva, blood, or other fluids contaminating the applied ESTELINK, thoroughly rinse with water, dry and re-apply fresh ESTELINK.

- DO NOT wait too long after the application of ESTELINK; this will lead to the possibility of the restoration not seating properly with cementation. Air dry within 30 seconds after application, to ensure the appropriate film thickness.

8) Cementation and Final Bonding

1) Mix and Applying PASTE A and B

- Before dispensing, remove the cap from auto-mix syringe, and squeeze out a small quantity of paste A and B, making sure equal amounts are being dispersed through the two outlets of the syringe.

- How to remove the cap: turn the cap 1/4 of a turn counter-clockwise to align the projections on the cap with the two outlets of the syringe. Holding the base of the cap, turn the cap 1/4 of a turn clockwise to press down air.

- DO NOT allow PASTE A to come into contact with PASTE B, there is a possibility of hardening of the PASTE at the tip of contact. Wipe the excess PASTE from the tip of syringe with a clean cloth or tissue before using a new mixing tip.

Attack a new tip to the syringe, squeeze PASTE (A and B) from the syringe and apply the PASTE to the tooth.

- If equal amounts of PASTE A and B are not dispersed, there is a possibility of poor polymerization. Ensure equal amounts are being dispersed through the two outlets of the syringe prior to attaching a mixing tip.

- How to attach the mixing tip: align the projections on the adapter with the grooves in the syringe as closely as possible, and turn the mixing tip 1/4 of a turn clockwise to lock it in place.

- Be sure to attach a mixing tip to the syringe when applying the PASTE to the surface of the restoration.

- After use, remove the mixing tip from auto-mix syringe and replace the cap of the auto-mix syringe.

2) Placement of the Restoration

Place the restoration on the tooth with firm pressure. Any excess PASTE at the margin can be removed with a brush tip.

- DO NOT apply the PASTE to the adherent surface treated with ESTELINK to avoid re-setting of the PASTE. Especially when the use of cementing posts, never place the PASTE on the post canal. Be sure to apply the PASTE to the post and then place into the root canal. ESTELINK will accelerate the setting of the PASTE.

- To keep the appropriate film thickness, seat the restoration as soon as possible after the application of the PASTE (within 1 minute after dispensing), if not there is a possibility that the restoration does not seat properly with cementation.

3) Mixing and Mixing Paste

There are two methods for removal of excess PASTE:

a) When using the light-cure method

Once the restoration is seated, light-cure the excess PASTE with a dental curing light for 2 to 4 seconds. The gel state of the excess PASTE can be easily removed.

- Excess light curing will lead to difficult removal of the excess cement.

b) By using the self-cure method

The gel state of the excess cement can be easily removed within 1 to 3.5 minutes after seating.

- Waiting more than 3.5 minutes after seating will lead to difficult removal of the excess cement.

c) The setting time of the PASTE may change (approximately 20 seconds at the maximum) depending on the oral temperature in each clinical case.

4) Final Adjustment

a) Final PASTE hardening

- In case of translucent restoration materials such as ceramics or composite materials.

- Remove the PASTE on the seated restoration for 20 seconds or more. When the adhesive area is too large to be completed by a single light-cure, multiple cures are recommended.

- Confirm that the light-curing unit has sufficient intensity (>500mW/cm²) before using.

- Note that using a cracked light guide will lower the intensity.

b) Removal of translucent restoration materials such as metal.

- Light-cure along margins for 20 seconds or more, then allow the PASTE to set for 8 minutes with the patient applying normal occlusal pressure to the restoration.

2) Finishing

Cheat patient's bite, if necessary. Re-polish the restoration with material of choice.

IMPORTANT NOTE: The manufacturer is not responsible for damage or injury caused by improper use of this product. It is the personal responsibility of the user to ensure that this product is suitable for an appropriate application before use.

3) To avoid unexpected hardening of the PASTE, after each use remove the mixing-tip from the auto-mix syringe. Using a clean cloth or tissue wipe the excess paste from the tip of syringe prior to recapping the syringe using the cap provided with the syringe at the time of purchase.

■ PRECAUTIONS FOR PASTE

1) Use the mixing-tip included in this KIT when extruding the mixed PASTE.

2) Use a clean instrument or explorer for the application or removal of the PASTE.

3) To avoid unexpected hardening of the PASTE, after each use remove the mixing-tip from the auto-mix syringe. Using a clean cloth or tissue wipe the excess paste from the tip of syringe prior to recapping the syringe using the cap provided with the syringe at the time of purchase.

■ PRECAUTIONS FOR ESTELINK

1) To avoid splashing the liquid, the bottles of ESTELINK should not be opened immediately after taking out of the refrigerator.

2) Use the rubber dispensing well provided when dispensing and mixing of ESTELINK. If the liquid is not dispensed into the rubber dispensing well it may be difficult to remove the cured ESTELINK from dispensing well. The cured ESTELINK can be removed by inverting the rubber.

3) Some materials and medicaments (hemostatic material) inhibit setting/adhesion of ESTELINK for an extended period even after careful meticulous cleansing with water. DO NOT USE products which contain:

- aluminum chloride;

- ferric sulfate;

- calcium sulfate;

- aluminum sulfite;

- aluminum fluoride [molecular formula: Ag(NH₃)₂F].

4) ESTELINK does not adhere to tooth structure immediately after applications of materials which contain:

- hydrogen peroxide (oxido).

- sodium hypochlorite.

WARNING: Avoid the use of these materials on the day of cementation, however these materials can be used for post canal treatment as the inhibition caused by these materials usually evaporates within five days.

5) Make sure to hold the bottles of ESTELINK BOND A and B vertically when dispensing. Do not hold the bottle diagonally or horizontally while dispensing; this could cause the primer to flow back into the nozzle and become contaminated.

■ PRECAUTIONS FOR UNIVERSAL PRIMER

1) To avoid contamination, the bottles of TOKUYAMA UNIVERSAL PRIMER should not be opened immediately after taking out of the refrigerator.

2) Be sure to hold the bottles of TOKUYAMA UNIVERSAL PRIMER A and B vertically when dispensing. Do not hold the bottle diagonally or horizontally while dispensing; this could cause the primer to flow back into the nozzle and become contaminated.

■ STORAGE

1) Store under refrigeration at temperatures between 0 to 10°C (32 to 50°F).

2) Keep away from heat, direct sunlight, sparks and open flames.

3) DO NOT use after indicated expiration date on the syringe/bottle/package.

■ DISPOSAL

- To dispose of the PASTE, extrude unused paste from the auto-mix syringe, mix and allow paste to harden before disposal.

- Use of ESTELINK and TOKUYAMA UNIVERSAL PRIMER is to be disposed into an inert absorbent material such as gauze or cotton, and disposed of in accordance with local regulations.

■ CLINICAL PROCEDURES

1. **Tooth Preparation**

Remove contamination from the tooth surface such as plaque, temporary cement residue, oil or debris to test crown fit, or from a hand-piece, saliva, blood and exudate fluids by brush and/or water spray, ultrasound scaling or cleaning the surface of the tooth with alcohol. Thoroughly rinse and dry the surface.

- If there is an adherent surface including uncet enamel, apply etching agent to the uncet enamel.

- DO NOT desicate the vital tooth. Desication can lead to post-operative sensitivity.

2. **Restoration Preparation**

Follow the procedure shown below, depending on the material to be restored.

1) Ceramics other than Porcelain, and Composite materials

Roughen the interior of the restoration by sandblasting (0.1 to 0.2MPa), air abrasion or grinding with a diamond bur using a slow speed to prepare the surface for adhesion. Thoroughly rinse and dry the surface.

2) Metals

DO NOT roughen the surface: Prepare the interior of the restoration, providing a clean and fresh surface for adhesion. Thoroughly rinse and dry the surface.

3) Metal restorations

Roughen the area by sandblasting (0.1 to 0.5MPa), air abrasion or grinding with a diamond bur using a slow speed to prepare the surface for adhesion. Thoroughly rinse and dry the surface.

4) When using the sandblaster or air abrasion, use at least 30 to 50 micron grit of alumina powder.

- If unable to roughen, then thoroughly rinse and dry.

- The substances listed below inhibit adhesion and must be removed from the adhesive surface by mechanical cleaning or alcohol soaking. Clean the restoration after adjustment is complete.

1) Gypsum and investment

- Etch the area with 10% hydrochloric acid (10w/v).

2) Oil/grease

- Remove the oil/grease with acetone.

3) Plaque, tartar, salvia, blood and exudate fluids

- If necessary, protect the surface not being bonded with metal separator, paraffin film, Vaseline, etc.

4) Auto-clave, sterilization

- If autoclaved, soak the surface with 10% hydrochloric acid (10w/v).

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