# MEDISANA®

## Infrarot-Fieberthermometer FTO

Infrared Ear Thermometer FTO

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## 1.1 Thank you very much!

Congratulations on your purchase and thank you for your confidence!

By purchasing the **FTO** infra-red thermometer, you have gained a high-quality product made by **MEDISANA**.

To ensure the best results and long-term satisfaction with your **MEDISANA FTO** Infra-red Ear Thermometer, we recommend that you read the following operating and maintenance instructions carefully.

Ihre MEDISANA AG

## 1.2 Important

### 1.2.1 When taking body temperature

- To ensure an accurate reading, it is crucial that the sensor is angled towards the ear drum and inserted sufficiently deeply into the auditory canal.
- Some people produce different results in the left and right ear. In order to accurately measure changes in temperature, always take readings in the same ear with any given person.
- The position in which you sleep can influence readings. If someone has been lying on one ear for a long period of time, the temperature in that ear will be higher than normal. In this case, either use the other ear or wait a few minutes before taking a reading.
- Earwax can influence the accuracy of a reading. If necessary, clean the ear and the sensor tip of the thermometer before taking a reading in order to prevent inaccurate results.

### 1.2.2 General tips

- Only use the unit for the specific purpose described in the operating instructions. Any misuse will void the warranty.
- The unit is not designed for commercial use.
- Do not let children use the unit. Medical products are not toys!
- If a fault occurs or the unit is damaged, do not attempt to repair the unit yourself, as this will void the warranty. Contact your dealer and only have repairs carried out by authorized service partners.
- Do not immerse the thermometer in water or other liquids. However, should moisture somehow get into the unit, remove the battery and do not take any further readings with the thermometer. Before any further use, contact your specialist dealer or **MEDISANA** directly.
- Do not shake or tap the thermometer.

### 1.3 Scope of supply

Please check first of all that the unit is complete.

The following parts are included as standard:

- 1 **MEDISANA FTO** Infrared Ear Thermometer
- 1 Battery CR-2032 (Already inserted!)
- 1 Operating instructions
- 5 Alcohol wipes
- 1 Storage bag

Please contact your dealer if you notice any transport damage on unpacking the unit.

### 1.4 Useful information on taking temperature with the MEDISANA FTO infra-red ear thermometer

#### 1.4.1 What is fever?

Our normal body temperature is between 36 °C and 37,5 °C. Fluctuations are quite normal, depending on time of day and the body's condition, for example in the female menstruation cycle. However, when body temperature rises above 38 °C, the phenomenon is known as fever.

Fever often occurs in the case of infections, and is a sign that the body's immune system is working overtime. This perfectly normal reaction to foreign bodies, germs and microorganisms should be checked at regular intervals. If the fever should exceed 39° C and not fall again within 1 to 2 days, you should definitely contact your doctor.

#### 1.4.2 Taking a temperature reading – quickly and accurately!

A fever is often associated with fatigue, pains in the limbs, shivering and chills. The course of an illness can be best monitored by taking a temperature reading 2 to 3 times a day at 4 to 6 hour intervals. Readings should always be taken in the same ear and at approximately the same time so that the results can be compared.

Using the **MEDISANA FTO** ear thermometer to take an infra-red temperature reading in the ear is ideal for this task, as the eardrum shares a common blood supply with the nearby temperature-regulating centre of the brain. Because of this, variations in body temperature are shown very accurately and without any significant delay.

#### 1.4.3 Taking other temperature readings

As well as the important function of taking body temperature, the **MEDISANA FTO** infra-red ear thermometer can also be used to measure the temperature of objects (e.g. babies' bottles) and liquids (e.g. bath water). It is even possible to use the unit to measure ambient temperature.

## 2.1 Getting started

### Removing / inserting batteries

A battery providing power for approx. 1000 readings is already inserted in the **MEDISANA FTO** infra-red ear thermometer. Should the battery symbol in the LCD display start to flash after an extended period of use, the battery (CR-2032 type) must be replaced.

#### Removal:

Gently pull off the cover of the battery compartment from the infra-red thermometer. Remove the battery by carefully levering it out of the mounting with a pointed object.

#### Insertion:

Insert the new battery (CR 2032 type) with the positive pole pointing upwards. Replace the cover and push it up until you hear it snap into place.

## 2.2 Operation

Remove the protective cap from the **MEDISANA FTO** infra-red ear thermometer.

There is no need to attach a special probe cover to the sensor area, as the sensor is completely sealed (see also: cleaning and maintenance tips).

### 2.2.1 Switching on

Press the **ON/OFF** button **3**, to be found directly below the LCD display **2**. All display symbols and the result of the last reading taken will appear for about 2 seconds, after which the backlight for the display will switch off. As soon as the unit is ready to use, you will hear a short tone. The LCD display now shows a thermometer symbol and the chosen unit of measurement. "°C" indicates that the results will be displayed in degrees Centigrade, while "°F" means that body temperature will be displayed in degrees Fahrenheit.

The measuring range of the **MEDISANA FTO** infra-red thermometer is from 0 °C to 100 °C (32 °F to 212 °F), displayed in steps of 0,1 °C or 0,1 °F.

### 2.2.2 Taking a body temperature reading

Tilt the patient's head slightly in the opposite direction and support the head (especially in the case of children). **Pull on the ear slightly in order to straighten the auditory canal. Carefully insert the sensor into the auditory canal.** If you are taking your own temperature, use one hand to pull on the ear and the other to operate the thermometer.

Once the sensor is correctly positioned in the auditory canal, use your index finger to press the **START** button **5** on the back of the unit. When the reading is successfully completed, you will hear a short tone and can remove the unit from the ear. The display will immediately show the temperature measured and automatically store the result. In the case of a raised body temperature (above 37,5 °C / 99,5 °F) you will hear 10 short tones after the reading has been taken.

After approx. 5 seconds, the unit will be ready to take another reading when the unit of measurement symbol (°C / °F) starts to flash. After taking 3 – 5 consecutive readings, wait for approx. 30 seconds before taking a further reading.

In each of the following cases, you should take three consecutive readings in the same ear, using the highest reading as a reference:

- With newly born children under 3 months old.
- With children under 3 years of age who suffer from diseases of the immune system, and with persons for whom deviations from normal body temperature can be dangerous.
- If you are not familiar with taking temperature and there are large discrepancies between consecutive readings.

### 2.2.3 Other temperature readings

The infra-red thermometer can also be used to measure the temperature of surfaces, such as the skin or of babies' bottles. To do this, switch on the thermometer and hold the sensor gently against the object to be measured. After you hear a short tone, the unit is ready to use. Give the **START** button **5** a short press and keep the thermometer in place until a long tone sounds, signalling the end of the reading.

The result is shown on the backlit LCD display and stored automatically. If the result is not within the range 32 °C to 42,2 °C, a crossed-out ear symbol will appear in the display.

When measuring the temperature of liquids, e.g. drinks, bath water etc., you must only insert the light-grey sensor tip. Make sure that no liquid can get into the unit.

### 2.2.4 Switching off

The thermometer features an energy-saving function and will switch off automatically if no buttons are pressed for 45 seconds. You may also switch off the unit with a short press of the **ON/OFF** button **3**.

### 3.1 Recalling stored data

The **MEDISANA FTO** infra-red ear thermometer can store up to 12 results.

You can call up your stored data with a short press of the **START** button **5**, without having previously switched on the thermometer with the **ON/OFF** button **3**. The symbol **M** will flash on the display next to “- - -”. After a further short press of the **START** button **5**, memory slot number **1** will appear in the unit display, followed directly by the last stored temperature reading. The symbol **M** will flash below the unit of measurement.

With a further press of the **START** button **5**, you can call up the result stored in memory slot “**2**”. A total of 12 memory slots are available, and these will be displayed in order if you continue to press the **START** button **5**.

If a memory slot is empty, the symbol “- - -” will appear next to it in the display.

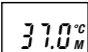
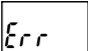
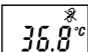
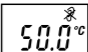
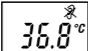
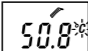
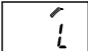
### 3.2 Changing the unit of measurement

When taking temperature, you have the choice between degrees centigrade and degrees Fahrenheit.

Proceed as follows:

1. The unit must be switched off.
2. Hold down the **START** button **5** on the back of the unit for at least 10 – 15 seconds until the current unit of measurement is shown in the display.
3. You can now change the unit of measurement with a further short press of the **START** button **5**. Once the desired unit of measurement is displayed, wait until all the display symbols flash in the illuminated display and the change is confirmed with a short tone. Directly after this, the thermometer is ready to take the next reading with the newly selected unit of measurement.
4. At the same time, all stored data is automatically converted to the new unit of measurement..

#### 4.1 Explanation of display symbols and error messages

Display	Description
	<b>SWITCHING ON</b> All symbols are displayed for 2 seconds.
	<b>Memory Function</b> Automatic display of the last reading for 3 seconds. The display remains illuminated for 2 seconds. No tone.
	<b>SELF-TEST</b> The symbol "Err" indicates an error. You will hear three short tones and the display will remain illuminated for 5 seconds. The thermometer switches off after 60 seconds.
	<b>READY FOR READING</b> You will hear a short tone. The °C or °F symbol flashes. Give the <b>START</b> button <b>5</b> a short press to start the reading.
	<b>READING COMPLETED</b> You will hear a long tone. The display remains illuminated for 5 seconds. The crossed-out ear symbol is displayed if the result is not within the temperature range 32 °C to 42,2 °C.
	If the result is greater than or equal to 37,5 °C (99,5 °F), the fever alarm will be activated (10 short tones).
	<b>FURTHER READINGS</b> When the °C or °F symbol starts to flash again, the thermometer is ready to take the next reading. The unit switches off automatically after 60 seconds if no further buttons are pressed.
	
	<b>The temperature measured is too high.</b> "H" appears in the display when the temperature reading taken is above 100 °C or 212 °F.
	<b>The temperature measured is too low.</b> "L" appears in the display when the temperature reading taken is below 0 °C or 32 °F.

*Ambient temperature is too high.*

The ambient temperature is above 40 °C (104 °F).

*Ambient temperature is too low.*

The ambient temperature is below 5 °C (41 °F).

*Error message.*

An error has occurred.

*Blank display.*

Please check the position of the battery.

*Symbol indicating flat battery.*

Replace the battery.

## 4.2 Cleaning and maintenance

- The sensor is the most sensitive part of the **MEDISANA FTO** infrared thermometer. Protect the lens from dirt and damage by always putting the protective cap on the thermometer when it is not in use.
- Should the lens get dirty, you can clean it carefully using the alcohol wipes supplied with the unit. You may also use a cotton bud soaked in alcohol or warm water to wipe the lens clean.
- Never use strong detergents or hard brushes.
- To clean the thermometer, please use a soft cloth moistened in a mild soap solution. Do not let water get into the unit. Only use the unit again when completely dry. However, if any liquid does get into the unit, remove the battery immediately and discontinue further use. In such a case, contact your dealer or the **MEDISANA** Service Department.
- Do not use or store the unit in an environment where the ambient temperature or humidity is too high or too low (see operating conditions in the "Specifications" section) , in direct sunshine, in the vicinity of static electricity or in a dusty environment.
- Remove the battery if you do not wish to use the unit for an extended period of time, otherwise there is a risk of the battery leaking.
- If the thermometer is used as described in these instructions, no regular maintenance or calibration is required.

### 4.3 Disposal

This product must not be disposed together with the domestic waste. All users are obliged to hand in all electrical or electronic devices, regardless of whether or not they contain toxic substances, at a municipal or commercial collection point so that they can be disposed of in an environmentally acceptable manner. Please remove the batteries before disposing of the equipment. Do not dispose of old batteries with your household waste, but dispose of them at a battery collection station at a recycling site or in a shop. Consult your municipal authority or your dealer for information about disposal.

### 4.4 Specifications

Name and model	: <b>MEDISANA FTO</b> Infra-red Ear Thermometer
Measuring range	: 0 °C – 100 °C (32 °F – 212 °F)
Accuracy of measurement	: (laboratory readings)
	±0,2 °C or ±0,4 °F in the region of 36 °C – 42,2 °C / 96,8 °F – 108 °F (under correct operating conditions);
	±0,3 °C or ±0,6 °F in the region of 32 °C – 35,9 °C / 89,6 °F – 96,6 °F (under correct operating conditions);
	±1 °C or ±1,8 °F in the region of <32 °C bzw. >42,2 °C / <89,6 °F bzw. >108 °F (under correct operating conditions)
Clinical repeatability	: ±0,2 °C (±0,4 °F)
Duration of measurement	: approx. 1 second
Interval between two readings	: at least 5 seconds
Fever alarm	: for temperatures > 37,5 °C (99,5 °F)
Unit of measurement	: °Centigrade or °Fahrenheit
Operating conditions	: Outside temperature 5,0 °C –40 °C (41°F – 104°F).
Storage conditions	: -25 °C to +55 °C (-13 °F to +131°F)
Dimensions	: 170 mm x 38 mm x 30 mm
Weight	: 50 g including battery
Battery	: 1 battery (CR-2032 type)
Battery life	: approx. 1000 readings
Memory capacity	: up to 12 readings
Article No.	: 76073
EAN Code	: 40 15588 76073 0

**CE 0297**

The CE-mark of this device refers to the EC-directive 93/42/EEC MDD.

The unit is designed for home use only.

Device classification : Class IIa medical product;  
protection class 

**We reserve the right to make technical and design changes in the course of continuous product improvement.**

## 5.1 Warranty and repair terms

Please contact your dealer or the service centre in case of a claim under the warranty. If you have to send in the unit, please enclose a copy of your receipt and state what the defect is.

The following warranty terms apply:

1. The warranty period for **MEDISANA** products is three years from date of purchase. In case of a warranty claim, the date of purchase has to be proven by means of the sales receipt or invoice.
2. Defects in material or workmanship will be removed free of charge within the warranty period.
3. Repairs under warranty do not extend the warranty period either for the unit or for the replacement parts.
4. The following is excluded under the warranty:
  - a. All damage which has arisen due to improper treatment, e.g. nonobservance of the user instructions.
  - b. All damage which is due to repairs or tampering by the customer or unauthorised third parties.
  - c. Damage which has arisen during transport from the manufacturer to the consumer or during transport to the service centre.
  - d. Accessories which are subject to normal wear and tear.
5. Liability for direct or indirect consequential losses caused by the unit are excluded even if the damage to the unit is accepted as a warranty claim.

### **MEDISANA AG**

Itterpark 7-9

40724 Hilden

Deutschland

Tel.: +49 (0) 2103 / 2007-60

Fax: +49 (0) 2103 / 2007-626

eMail: [info@medisana.de](mailto:info@medisana.de)

Internet: [www.medisana.de](http://www.medisana.de)

The service centre address is shown on the last page.

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Marketing Oy
Electronics AB