20021 Bollate (MI) ITALY - Viale Lombardia, 18  
Telefono 02 38302386 r.a. - Telefax 02 38302389  
e-mail: gfc.conti@tin.it

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**Istruzioni per l'uso e la manutenzione**  
***Operating and maintenance instructions***  
**Instructions pour l'emploi et l'entretien**  
***Gebrauchs- und Wartungsanleitung***  
**Manual de instrucciones de uso y mantenimiento**

# DICHIARAZIONE CE DI CONFORMITÀ CE COMPLIANCE DECLARATION / DECLARATION DE CONFORMITE CE CE - ÜBEREINSTIMMUNGSERKLÄRUNG / DECLARACION CE DE CONFORMIDAD

LA SOTTOSCRITTA / *The undersigned* / La soussignée / *Die unterzeichnende Firma* / Quien suscribe

GFC  
Viale Lombardia, 18  
I - 20021 Bollate (Milano)

DICHIARA SOTTO LA PROPRIA RESPONSABILITÀ CHE L'APPARECCHIO:  
*on his own responsibility, declares that the equipment*  
déclare sous sa propre responsabilité que l'appareil  
*erklärt unter eigener Verantwortung, daß das Gerät*  
declara bajo su responsabilidad que la maquina

Tipo / *Type* / Type / *Vom Typ* / Tipo

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## CONTENTS

<b>1</b>	<b>GENERAL RULES AND DIRECTIONS</b> .....	4
1.1	INTRODUCTION.....	4
1.2	DESCRIPTION OF MACHINE.....	4
1.3	NORMATIVE REFERENCES.....	4
1.3.1	Compulsory set of rules.....	4
1.3.2	Additional voluntary set of rules.....	4
1.4	PRE-ARRANGEMENTS TO BE CARRIED OUT BY CLIENT.....	4
1.5	REQUEST OF INTERVENTIONS AND SPARE PARTS.....	4
<b>2</b>	<b>TECHNICAL CHARACTERISTICS</b> .....	5
<b>3</b>	<b>INSTALLATION</b> .....	5
3.1	SETTING UP.....	5
3.2	CONNECTION AND STARTING.....	5
<b>4</b>	<b>OPERATION AND USE</b> .....	5
4.1	OPERATION.....	5
4.2	OPERATOR.....	6
4.3	USE FORESEEN.....	6
4.4	SAFETY WARNINGS.....	6
<b>5</b>	<b>INSTRUCTIONS FOR OPERATOR</b> .....	6
5.1	CONTROL COMMANDS AND UNITS.....	6
5.2	RELATIVE INDICATIONS FOR USE.....	7
5.2.1	Stopping and emergency operations.....	7
5.2.2	Warning.....	7
<b>6</b>	<b>MAINTENANCE</b> .....	7
6.1	CLEANING.....	7
<b>7</b>	<b>DISPOSAL</b> .....	7
<b>8</b>	<b>DIAGNOSTICS</b> .....	7
<b>9</b>	<b>CIRCUIT DIAGRAM</b> .....	9

## **1 - GENERAL RULES AND DIRECTIONS**

### **1.1 - Introduction**

The present manual belongs to GFC - Bollate. The contents of this document may not be reproduced or assigned to third parties. All rights reserved.

### **1.2 - Description of machine**

The instrument is a device for initialling control of values of multi-focal lenses, together with control of optical quality of assemblies.

### **1.3 - Normative references**

#### **1.3.1- Compulsory set of rules**

- EEC directive no. 73/23 - Low Tension (DBT) (Law no. 791/1977, Law Decree no. 626/1996, Law Decree no. 277/97).
- EEC directive no. 89/336 relative to Electromagnetic Compatibility (EMC) - (Law Decree no. 615/1996).
- Law Decree no. 547 dated 27.4.1955 "Rules for prevention of industrial accidents".

#### **1.3.2- Additional voluntary set of rules**

- EN 292 (1992) Safety of machinery - Fundamental concepts, general design principals - First Part - Basic Terminology and Methodology (EN 292-1)/Second Part - Technical specifications and principals (EN 292-2).
- EN 60204-1 (1998) Safety of machinery - Electrical equipment - Part 1. General requisites (Revision of EN 60204-1).

### **1.4 - Pre-arrangements to be carried out by client**

The user installs the JOLLY machine in a suitable surroundings equipped with an electrical system corresponding to the rules in force.

Machine must be installed in well-ventilated, dry and illuminated environments, in compliance with the current law.

### **1.5 - Instructions for the request of interventions and spare parts**

For any operation of electric maintenance, contact GFC of Bollate, Viale Lombardia, 18 or an Authorized Assistance Centre.

In case of anomalies, damage etc., give specific indication of defect encountered to GFC technicians or other authorized technicians.

## 2 - TECHNICAL CHARACTERISTICS

		JOLLY
Voltage	Volt	230
Frequency	Hz	50
Installed power	W	15
Fuse	A	0.5
Machine weight	Kg	1.9
Dimensions	mm	150x215x215

## 3 - INSTALLATION

### 3.1- Setting up

The machine is supplied already assembled and ready for use. It must only be connected to mains.

The packaging includes:

- a) instrument body;
- b) operating instructions.

Owing to its particular nature, the machine can be handled manually without necessitating further devices.

### 3.2- Connection and starting

Once instrument has been placed on a table in front of the operator, perfectly horizontal, thereby guaranteeing an adequate stability and having assured that general switch is on OFF position (1), feed cable may be connected, previously plugging in same at the side of machine (2) and subsequently plugging it to mains.

## 4 - OPERATION AND USE

### 4.1- Operation

The equipment disposes of a general ON/OFF device (1) with which one can turn the light on which illuminates the reference point both under the contrast base (6) as well as that under the Polaroids lens (4).

**Please remember that, after use, the machine must be turned off.**

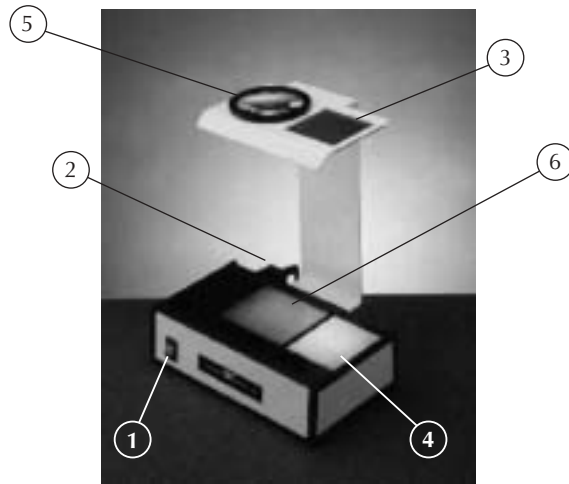
Fig. 1

#### **4.2- Operator**

The operator, after having prepared the instrument and turned on same, carries out the foreseen controls, without the necessity for any further interventions.

#### **4.3- Use foreseen**

The instrument is utilized as a tensiometer to control the strains on assembled lenses and as a Lens Reader, to check the initialling of the lenses.

#### **4.4- Safety warnings**

- The instrument must be used by qualified personnel and nevertheless same must be informed on existing dangers, with particular regard to electric shock;
- Do not use the machine near inflammable material or gas.
- Do not open or disassemble instrument.

### **5- INSTRUCTIONS FOR OPERATOR**

#### **5.1- Control commands and units**

The use of the JOLLY instrument foresees that test light turns on through main switch (1) with a voluntary action.

##### **Tensiometer**

Place the glasses between the two filters (3) and (4), checking the eventual strains present on the assembled lens.

Lenses reader

Place the eye-glass lens between the magnifying glass (5) and the contrast background (6); look through the magnifying glass and move the lens vertically until initialling is found; same can be better evidenced when one observes the contrast lens with an unclear background.

## **5.2- Relative indications for use**

### **5.2.1- Stopping and emergency operations**

The stop function may be obtained:

- placing general switch (1) in OFF position;
- removing feed cable (2) from mains.

### **5.2.2- Warning**

ISOLATE INSTRUMENT FROM ELECTRIC MAINS EACH TIME WORK IS TERMINATED.

IN CASE OF ANOMALIES DISCONNECT INSTRUMENT IMMEDIATELY FROM ELECTRIC MAINS; BEFORE RECONNECTING CHECK REASONS WHICH HAVE CAUSED THIS ANOMALY, CONTACTING THE AUTHORIZED RESELLER GFC FOR FURTHER INFORMATION.

## **6- MAINTENANCE**

### **6.1- Cleaning**

The instrument does not require any particular maintenance interventions, other than the usual cleaning operations.

Cleaning of external part of the instrument may be carried out with normal detergents.

## **7- DISPOSAL**

The JOLLY instrument utilizes materials which do not require any particular instructions for disposal.

## **8- DIAGNOSTICS**

Should the machine stop functioning, check the fuses in the rear feed connection group (2).

Assure that the switch (1) is on position 0 (OFF).

For the replacement utilize fuses according to the material list attached to the diagram; to reach the fuse group, it is necessary to insulate the machine from electric mains.

A small box can be found in the lower part of the group, in which the fuses are contained; by moving tab, it is possible to open the drawer thereby removing fuse for replacement of same.

Once small draw has been closed, it is possible to connect feed cable again.

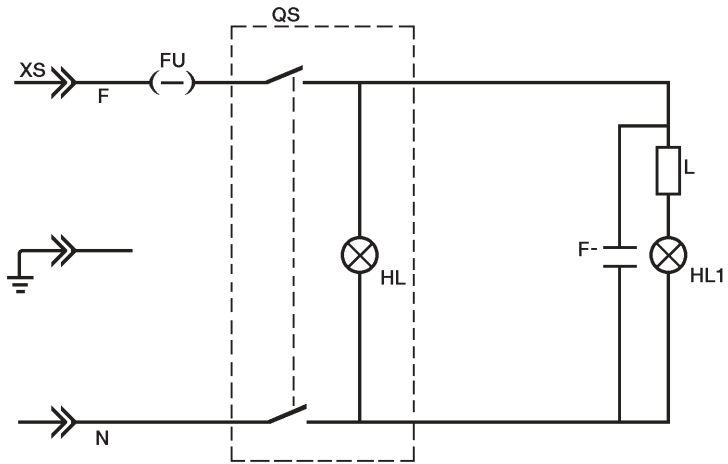
**In case of continuous malfunctionings, damages or break-downs, position equipment on OFF position and immediately disconnect from mains; subsequently contact the GFC company or an authorized service centre.**

ATTENTION!

ANY INTERVENTION ON THE MACHINE MUST BE  
REALIZED WITH THE ELECTRICITY TURNED OFF

### 9 - CIRCUIT DIAGRAM

DATE	REVISION	ITEM	COMPONENTS
A		XS	INTAKE
B		QS	GENERAL SWITCH
C		HL	LAMP
		L	REACTOR
		HL1	LAMP
		F	FILTER
WIRE SECTION 1,5 mm			
N	NEUTRAL BLUE		
F	PHASE-BLACK		

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