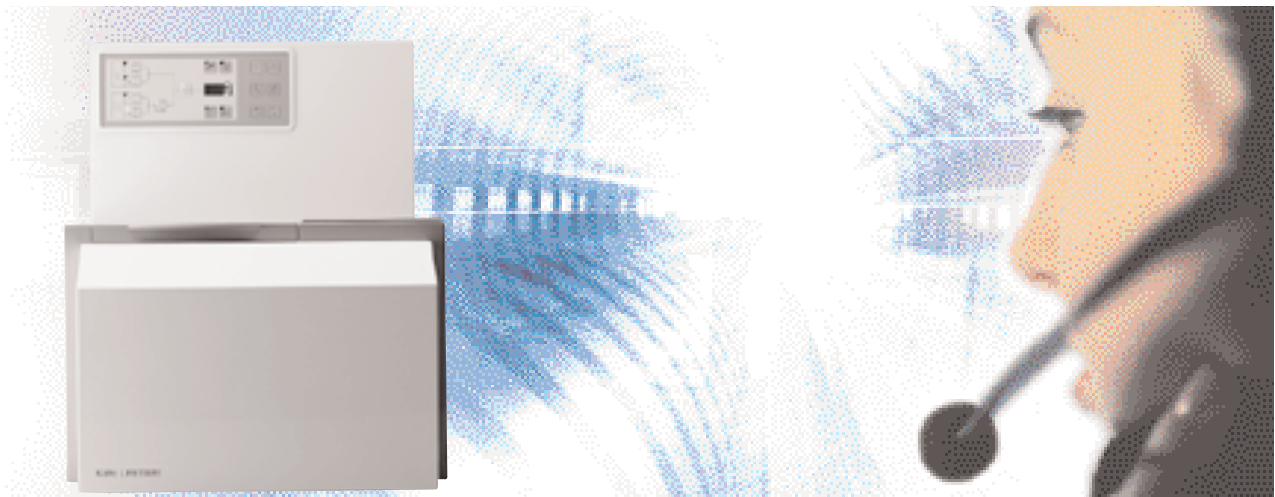
Documents

LIFETIME® 2210

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A	Operating instructions.....	3
A 1	Important notes	4
A 1.1	Introduction	4
A 1.2	Intended application	6
A 1.3	Technical customer service	6
A 1.4	Technical Data	8
A 1.5	Technical customer service	10
A 2	Description	12
A 3	Before first start-up.....	14
A 3.1	Installing the KaVo Spray bottle	14
A 3.2	Filling with KaVo LIFETIME Clean 1 and KaVo LIFETIME Clean 2 N cleaning agents.....	16
A 3.3	Filling the salt reservoir	16
A 3.4	Attaching instruments to the instrument carriers...18	
A 3.5	Changing couplings on the instrument carrier.....18	
A 3.6	Inserting the instrument carriers.....	20
A 3.7	Inserting the tool basket	20
A 3.8	Basic settings	22
A 4	Programme selection	28
A 5	Starting a programme	30
A 6	End of programme / after use	30
A 7	Troubleshooting	32
B	Assembly instructions.....	44
B 1	Transportation and storage	44
B 2	Packing list	46
B 3	Setting up the LIFETIME.....	46
B 3.1	Connecting the LIFETIME.....	48
E	Maintenance instructions	50
E 1	Maintenance.....	50
E 1.1	Cleaning the housing	50
E 1.2	Cleaning and maintenance of cleansing chamber	50
E 1.3	Cleaning and maintenance of instrument carriers	50
E 1.4	Spare parts required for cleaning and maintenance	50
K	Terms of guarantee.....	52
	EC Declaration of Conformity	55

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KaVo China
Service and Sales office
Block 2 Huiwan Overseas-Chinese Apartments
Area 3 of Huianli
Chaoyang District # 12-03
Unit 2
Beijing 100101
Tel. ++86 / 10 64 / 91 65 87
Fax ++86 / 10 64 / 91 65 87

KaVo Polska sp.z.o.o.
ul. Pomorska 251 P 405
92-214 Lodz
Polska
Tel. ++48 / 42 / 6 75 75 36
Fax ++48 / 42 / 6 75 75 36

KaVo Osaka
Yotsubashi Okawa Building
11 Fl. 1-6-23, Shinmachi
Nishi-Ku
Osaka 550-0013
Japan
Tel. ++81 / 6 / 65 38 58 50
Fax ++81 / 6 / 65 38 58 55
e-mail: gitwmike@gol.com

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A 1 Important notes

Please read the Operating, Maintenance and Assembly (OMA) instructions carefully before operating the KaVo LIFETIME, and keep the manual for future reference. The notes regarding operation, maintenance and installation of spare parts must be strictly observed to assure proper operation of the unit.

The trademarks named herein are the property of their respective owners. KaVo shall not be liable for damage resulting from:

- External influences (inadequate media quality, improper installation),
- application of incorrect information,
- usage for purposes other than the intended purpose,
- improper execution of repairs.

KaVo products may be repaired and serviced by:

- The technicians of KaVo's subsidiaries all over the world;
- technicians employed by authorized KaVo dealers and specially trained by KaVo;
- independent technicians specially trained and authorized by KaVo.

Approvals expire upon product modification by third parties. Only original spare parts may be used to operate and repair the unit.

A 1.1 Introduction

Please read these OMA instructions carefully for your own safety and to assure trouble-free operation. Particular attention should be paid to text passages marked with one of the symbols depicted below.

Meaning of symbols

Condition that may result in a hazardous situation, in damaged material or malfunction if ignored.

Important information for operators and technicians.

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A 1.2 Intended application

The LIFETIME maintenance unit is to be used exclusively for maintenance and disinfection of medical instruments. The manufacturers- maintenance instructions are to be observed.

Discolouration of instruments made of aluminium or anodized materials is possible.

The LIFETIME corresponds to the CE and DVGW regulations.

The device must not be operated in areas subject to explosion hazard (e.g., operating rooms).

Statutory provisions

The directives/guidelines and/or national law, national regulations and technical rules relevant for medical products are to be applied and fulfilled for starting up and operating the KaVo product for the intended purpose.

A 1.3 Location of nameplate

The nameplate provides information on:

Manufacturer's address
Name and type of product
Serial number
Order number
Voltage/power consumption

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A 1.5 Technical customer service

Technical service for KaVo equipment is provided primarily by the KaVo agent.

The technicians employed by the KaVo agent and the KaVo customer service technicians continuously participate in general and special training courses at the factory, and are familiar with the entire KaVo product line.

KaVo equipment should be serviced at the recommended intervals in order to guarantee continued operability and maintain the value of the equipment.

Please contact us at the following addresses regarding your inquiries:

KaVo Dental Ltd.
Industrial/Estate
Raans Road
Amersham Bucks, HP6 6JL
Tel.: 0044 - 494 - 72 15 21

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Always specify the unit type and serial number with your inquiries.

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A 2 Description

- ① Hinged front cover
- ② Control panel
- ③ Ledge
- ④ Power switch
- ⑤ Manometer
- ⑥ Salt reservoir
- ⑦ Reservoir KaVo LIFETIME Clean 2 N
- ⑧ Reservoir KaVo LIFETIME Clean 1
- ⑨ KaVo spray bottle
- ⑩ Wrench-salt reservoir
- ⑪ Power cord
- ⑫ Water supply hose
- ⑬ Compressed air hose
- ⑭ Drain hose

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Status indicators

- ⑮ Cleaning
- ⑯ Disinfection
- Cooling and drying
- ⑳ Lubrication and final cleaning

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Level indicators

- ⑱ KaVo LIFETIME Clean 1
- ⑲ KaVo LIFETIME Clean 2 N
- ⑳ KaVo LIFETIME water softener
- ㉑ KaVo LIFETIME Spray

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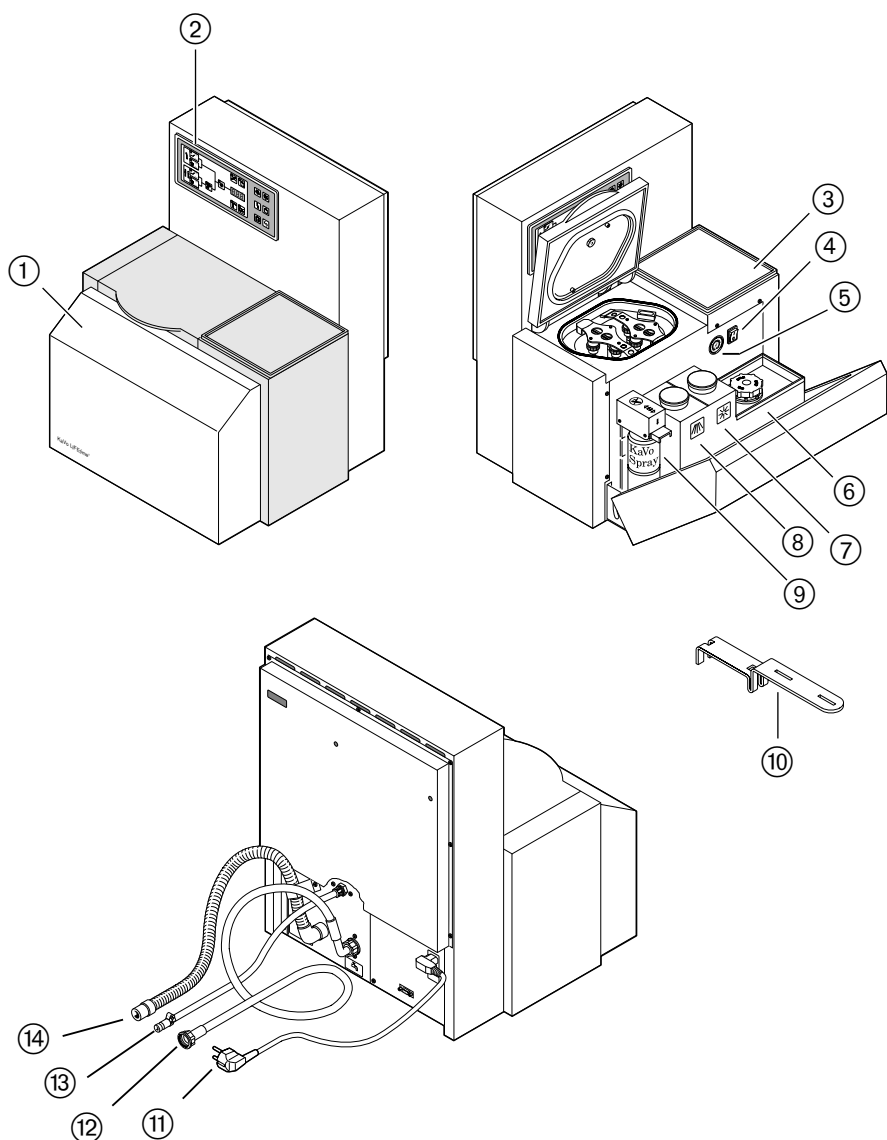
Programme keys

- ㉕ Start/Stop key
- ㉖ Hand instrument cleaning time preselection key
- ㉗ 3 min disinfection for hand instruments
- ㉘ 10 min disinfection for hand instruments
- ㉙ 3 min disinfection for driven instruments
- ㉚ 10 min disinfection for driven instruments

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A 3 Before first start-up

Make sure the LIFEtime unit is properly connected before starting it up for the first time.

 **Do not kink, pinch or entwine hoses or cables. The voltage on the nameplate must match the mains voltage.**

Fill the KaVo LIFEtime Clean 1, KaVo LIFEtime Clean 2 N and water softener reservoirs.

 **Be sure to fill the correct agents into the reservoirs.**

- Set water hardness using operator utility programs (see A 3.8), check time and date and adjust via utility programs if necessary.

A 3.1 Installing the KaVo LIFEtime Spray bottle order no. 411 9700

- Fold down the hinged front cover.
- Remove the lid from the spray bottle.
- Insert the spray bottle in the holder from below.
- Latch in the spray bottle by pulling up lever ①.
- To remove the spray bottle, push down on lever ①.

 **KaVo Spray contains a flammable propane/butane mixture as a propellant. The warning notes on the spray can and rear panel of the unit must be strictly observed.**

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A 3.2 Filling with KaVo LIFEtime Clean 1 and KaVo LIFEtime Clean 2 N cleaning agents

	Description	order no.
6l	KaVo LIFEtime Clean 1	489 3022
30l	KaVo LIFEtime Clean 1	489 3027
6l	KaVo LIFEtime Clean 2 N	489 3025
30l	KaVo LIFEtime Clean 2 N	489 3026

 **Do not use a funnel contaminated with salt to fill in KaVo LIFEtime Clean 1 or KaVo LIFEtime Clean 2 N.**

Reservoir ① KaVo LIFEtime Clean 1
Reservoir ② KaVo LIFEtime Clean 2 N

Fold down the hinged front cover. Unscrew the lid and fill cleaner into the reservoir. To do this, screw the supplied stopcocks onto the tank, hold the spout over the reservoir opening and open the stopcock. Remove any residue from around the reservoir opening and screw the lid back on.

A 3.3 Filling the salt reservoir

KaVo LIFEtime water softener
order no. 489 3024

Fold down the hinged front cover.

 **Switch on the power switch.**

When the salt reservoir ③ is opened, water overflows into a collection tank from which it is drained by a pump.

 **Do not fill in salt while a programme is running.**

Unscrew the lid and fill in salt through the supplied funnel.

If the cover is hard to open, the supplied wrench ④, order no. 665 1892, can be used.

 **Use only KaVo LIFEtime water softener. Never use table salt, cattle salt, defrosting salt or dishwasher salt.**

Remove salt deposits from the threads and sealing lip. Fill the reservoir with water for first-time operation. Close the cover. Rinse the tray with water.

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A 3.4 Placing instruments on the instrument carriers

The LIFETIME can be operated with two or only one instrument carrier.

Three turbines, SONICflex instruments, INTRAFlex instruments or handpieces and contra-angle handpieces can be placed on an instrument carrier if the required couplings have been ordered. Turbines, handpieces and contra-angle handpieces are to be plugged into couplings.

 Verify that the instruments which have been placed on couplings are properly latched in.

Press button ① to release handpieces or contra-angle handpieces.

Maintenance of SONICflex instruments in the LIFETIME requires an oil device ③, order no. 665 0601. This device prevents oil from collecting in the instrument.

Application:

Insert the oil device loop ③ between the oscillator body ④ and the housing ⑤ of the SONICflex all the way to the stop.

A 3.5 Changing couplings on the instrument carrier

There are special couplings for turbines, handpieces, contra-angle handpieces and instruments from other manufacturers which have been approved for maintenance in the LIFETIME.

Couplings for head maintenance are also available.

Loosen the union nut ② on the instrument carrier to change a coupling. Remove the coupling and install another coupling. The position of the coupling is defined by 2 pins. Replace the union nut and tighten.

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A 3.6 Inserting the instrument carriers

Open the cleansing chamber lid and push the instrument carrier with the attached instruments down on the coupling.

The heads of the turbines, handpieces and contra angle handpieces point down into the chamber.

Close the cleansing chamber lid.

Select the desired programme.

Select the desired programme.

When an instrument carrier is inserted, a line appears on the display. The line for the left instrument carrier appears on the left side ①, the line for the right instrument appears on the right side ②. Select the desired program.

Programmes can only be started when the cleansing chamber lid is closed.

A 3.7 Inserting the tool basket

Open the cleansing chamber lid and insert the tool basket.

Close the cleansing chamber lid.

Select the desired programme.

Programmes can only be started when the cleansing chamber lid is closed.

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A 3.8 Basic settings

Some basic settings must be established before starting up the unit for the first time. This is accomplished with the aid of six operator utility programmes.

- Water drainage
- Retrieval of factory defaults
- Time and date settings
- Water hardness setting
- Data transfer
- Alarm signalling

Definition of keys and indicators

T1 = key 1

L1 = LED indicator 1

LEDs **L1** to **L4** flash when the unit is powered up.

Before you can call up utility programs, you must press the following key combination:

- ① • Press and hold **T5**.
• Press and release **T1**.
• Release **T5** (**L9** is on).
- ② • Press and release **T1**
(**L7** and **L8** are on, **L9** goes off).

Now you can call up the utility programs with the corresponding key combinations.

Water drainage - ①, ②, press **T1**-

The water reservoir and cleansing chamber are drained. The cleansing chamber lid must be closed. If the lid is open, error message 17 prompts the operator to close the lid. Subsequently restart function. The water drainage process is finished after 3 min and 45 s, and the unit returns to the power-up state (LEDs **L1** to **L4** go on).

Retrieval of factory defaults

- ①, ②, press **T2** -

The factory settings are written to the auxiliary memory. The indicator remains dark for 1 s when the **T2** key is pressed, then the unit returns to the power-up state (LEDs **L1** to **L4** go on).

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Time and date settings**- ①, ②, press T3**LEDs **L1** to **L6** light up**To display time and date:**

- Press **T1** or **T4** key. The display shows the current time and date.

To change time and date:

- Select the value to be changed with **T1** or **T4**.
- Change the value with the **T2** or **T3** key.

T2 key = value plus 1**T3** key = value minus 1Store each individual value immediately with the **T5** key.

	Display
1: Hours	1HH
2: Minutes	2MM
3: Day	3DD
4: Month	4MM
5: Year	5YY

 Every hour, minute, day, month or year value that has been changed must be stored immediately with the **T5** key!

- Store by pressing **T5** key.
- Restore start-up status by pressing **T6** key (LEDs **L1** to **L4** go on).

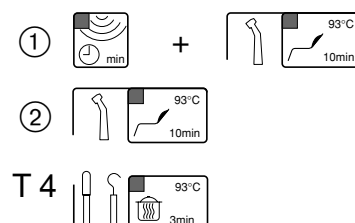
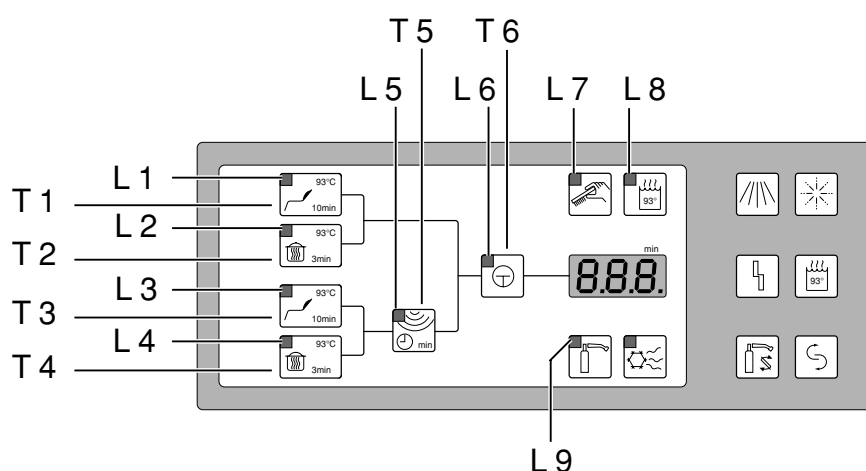
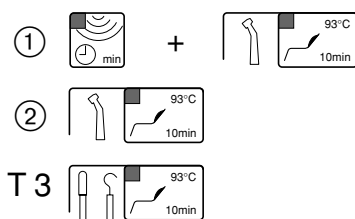
Water hardness setting**- ①, ②, press T4**

This utility is used to enter the local water hardness.

- Press key **T1** or **T4**. The number shown on the display indicates the current hardness range setting:

0: very soft
1: soft
2: medium hard
3: hard
4: very hard

- Press key **T6** to store the selected range setting and return to the power-up state (LEDs **L1** to **L4** go on).

	HB	mmol/l	°dH	°f	°e	mg/l CaCO ₃
0	0	0,2	1	1,8	1,25	18
1	1	<1,3	≤6	≤11	≤8	<107
2	2	1,3-2,5	7-14	12-25	9-18	107-249
3	3	2,6-3,8	15-21	26-37	19-26	250-374
4	4	>3,8	22-35	38-62	27-44	375-625

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Data transfer

– ①, ②, press and hold T5, press T1

The cleaning cycles logged are transferred from the data memory to a PC via the serial interface.

Alarm signalling

– ①, ②, press and hold T5, press T2

LEDs **L1, L4** and **L6** flash. Press **T1** or **T4** to change the alarm signalling function. The display indicates the function selected:

- 1: No alarm function
- 2: Alarm at end of cycle
- 3: Maximum alarm function (at end of cycle, on error, on warning). This is the delivery state (factory setting).

- Press key **T6** to store the selected setting and return to the power-up state (LEDs **L1** to **L4** go on).

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A 4 Programme selection

Four different programmes are available. These pertain to two different groups:

Group A:

Driven instruments (turbines, handpieces, contra angle handpieces), SONICflex

Group B:

Hand instruments (probes, mirrors, etc.)

The individual 'phase cycles' of the programme are indicated by fields ①, ②, ⑨ and ⑩.

Field ① - external cleansing

The cleansing chamber is filled with water mixed with a cleansing agent. The instruments are cleaned externally by means of pulsating waves.

Field ② - disinfection

The instruments are disinfected at $\geq 93^\circ\text{C}$ for 10 or 3 minutes.

Field ⑩ - drying and lubrication

Air is injected to expel steam and water. The instruments are lubricated with KaVo Spray.

Field ⑨ - cooling

Compressed air is injected to cool the instruments down to a temperature at which they can be removed by the operator. The residual heat of the instruments dries these on the outside.

Group A / programme 1:

Cleaning and disinfection programme for driven instruments with a disinfection dwell time of 10 min at 93°C .

Programme sequence:

External cleaning, internal cleaning, 10 minute disinfection, drying and lubrication, cooling.

Group A / programme 2:

Cleaning and disinfection programme for driven instruments with a disinfection dwell time of 3 min at 93°C .

Programme sequence:

External cleaning, internal cleaning, 3 minute disinfection, drying and lubrication, final cleaning, cooling. This programme is intended for optimal instrument cleaning and subsequent sterilisation.

This program provides optimal instrument maintenance and cleaning before sterilisation.

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Group B / programmes 3 and 4:

Programmes 3 and 4 correspond to those in group **A** but do not include lubrication and final cleaning.

The pulsating wave cleaning period can be adjusted in steps from 1 to 6 minutes with key (12). The last time setting is stored. The display shows the countdown of the time remaining till programme completion.

A 5 Starting a programme

When the unit is switched on for the first time, error message F 23 may be displayed approx. 30 seconds after start. Switch the unit off and then back on again.

- Switch on the power switch. (The LEDs in the four programme keys flash.)
- Select the desired programme.
- Set the pulsating wave cleaning time for group **B** programmes.
- Start the programme by pressing key (11).

Indicator fields (3) and (6) indicate possible malfunction.

Do not open the cleaning chamber lid during program execution.

Indicator fields (4), (5), (7) and (8) light up when the corresponding media need to be refilled.

Indicator field (4) - KaVo LIFETIME Clean 1

Indicator field (5) - KaVo LIFETIME Clean 2 N

Indicator field (7) - KaVo LIFETIME water softener

Indicator field (8) - KaVo LIFETIME spray (Here, you must restart the program sequence with key (11).)

A 6 End of programme / after use

Remove the instrument carriers with the instruments or the tool basket with the hand instruments from the cleaning chamber immediately upon program completion. Leave the cleaning chamber lid open after use to permit residual moisture to evaporate.

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A 7 Troubleshooting

S Unit does not power up.

- C** Power cord not connected to power outlet or power cord not properly inserted in socket on rear wall of unit.
- R** Check mains connection.

- C** Power switch is defective or fuse is blown.
- R** Check power switch and fuse.

S A programme cannot be started.

- C** Cleansing chamber lid is not closed.
- R** Close cleansing chamber lid.

S F xx Symbol shown is illuminated.

C Aborted or malfunction.

 *Disinfection has not been properly performed. Instruments are not disinfected.*

- R** If aborted by operator, restart the unit.
- R** In the event of a malfunction, see corresponding F xx and explanation on the following pages.

S Symbol shown is illuminated.

- C** Out of KaVo LIFEtime Clean 1.
- R** Refill KaVo LIFEtime Clean 1. To switch off this indicator, press the Start/Stop key when the cycle is finished.

S Symbol shown is illuminated.

- C** Out of KaVo LIFEtime Clean 2 N.
- R** Refill KaVo LIFEtime Clean 2 N. To switch off this indicator, press the Start/Stop key when the cycle is finished.

S Symbol shown is illuminated.

- C** Out of KaVo LIFEtime water softener.
- R** Refill KaVo water softener (see section A 3.3). Do not refill while a programme is running.

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S Symbol shown is illuminated.

C Out of KaVo LIFEtime Spray.

R Replace spray can. Press Start/Stop key to resume programme execution.

C KaVo LIFEtime spray can is not coupled properly.

R Couple spray can properly (see A 3.1).

S Water in cleansing chamber after completion of programme.

C Strainer clogged (see section E 1.2).

R Clean strainer (see section E 1.2)

S F 10

C Dry running protection for cleansing chamber heater.

R Call customer service. Programme is aborted, further operation is not possible.

S F 11

C Dry running protection for reservoir heater.

R Call customer service. Programme is aborted, further operation is not possible.

S F 12

C Pulsating water wave error.

R Call customer service.

S F 13

C Pressure rinse error.

R Switch unit off and back on, try again. Call customer service if error message reappears.

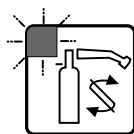
S F 14

C Pump solenoid error.

R Call customer service. Programme is aborted, further operation is not possible.

C No spray can inserted.

R Insert spray can and couple properly.

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A= Anomalie ; **C**= Cause ; **R**= Remède

S F 15

- C** No compressed air.
R Check compressed air supply. Reselect programme, start programme.

S F 16

- C** Incorrect programme selection.
R Select correct programme, start programme.

S F 17

- C** Cleansing chamber lid open.
R Close cleansing chamber lid. Select programme, start programme.

S F 20

- C** No water.
R Check water supply. Reselect programme, start programme.

S F 21

- C** Regeneration error.
R Switch unit off and back on, try again. Call customer service if error message reappears.

S F 22

- C** Pump M 6 error.
R Check routing of drain hose. Switch unit off and back on, try again. Call customer service if water leaks out of rear wall.

S F 23

- R** When error F 23 is indicated approx. 30s after power-up when the unit is switched on for the first time, switch the unit off and back on again.

- C** B19 error.
R Call customer service.

- C** Y3 is defective.
R Call customer service.

- C** No water.
R Check water connection.

- C** Drain hose installed incorrectly.
R Hose must leave unit at upward 90° angle.

- C** Strainer in cleaning chamber clogged.
R Clean strainer (see E 1.2)

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S F 24

- C** Maximum cleansing chamber heating period exceeded.
- R** Switch unit off and back on, try again. Call customer service if error message reappears.

S F 25

- C** Maximum reservoir heating period exceeded.
- R** Switch unit off and back on, try again. Call customer service if error message reappears.

S F 26

- C** Cleansing chamber level sensor is defective.
- R** Switch off unit. Clean level sensor (2 black plastic disks in side wall of cleansing chamber) with a dry cloth Switch the unit back on. Call customer service if error message reappears.

S F 27

- C** Safety circuit relay not reset.
- R** Call customer service.

S F 28

- C** ROM error
- R** Switch unit off and back on. Call customer service if error message reappears.

S F 29

- C** Built-in clock (RTC) is defective.
- R** Check time and date; adjust if necessary (see A 3.8). Selected programme can be started by pressing the START/STOP key once more after error message F 29 is displayed. Call customer service if error message reappears.

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S F 30

- C** Informs the user that another 1000 cycles can be run after this message appears for the first time until the built in flash RAM storage is full. F 30 is displayed after approx. 14,000 cycles.
- R** Selected programme can be started by pressing the START/STOP key once more after error message F 30 is displayed. Read out data and back up or have customer service perform data back-up.

S F 31

- C** Data storage flash RAM is full (after approx. 15,000 cycles). No further cycles can be run.
- R** Read out data and back up or have customer service perform data back-up.

S F 33

- C** KaVo LIFEtime spray monitoring defect.
- R** Call customer service.

S F 34

- C** Temperature sensor B4 or B9 defective.

Disinfection may be out of order.

- R** Switch programme off and back on. Restart programme. Call customer service if F 34 reappears.

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B 1 Transportation and storage**Printing on package:**

- Transport upright; the arrows point towards the top.
- Protect against shocks.
- Protect against humidity.
- **Protect against frost.**

After start-up, various assemblies are filled with water or chemicals. These assemblies must be drained to prevent spilling or freezing during transport and storage. Call customer service.

 *Drainage as described in section A 3.8 is not sufficient. Only the reservoir and cleaning chamber are drained by this procedure.*

B 1.1 What to do in case of shipping damage

Unpack the damaged item(s) in the presence of the transport company driver and have the driver acknowledge the damage on the consignment note.

KaVo must be notified of any damage within 4 days after receipt of merchandise.

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B 2 Packing list

The LIFEtime shipping carton contains the articles listed below. If one of these articles is missing, please contact your supplier immediately to request that the missing part be delivered.

- 1 LIFEtime unit
- 2 instrument carriers (each with 1 turbine coupling and 2 instrument couplings)
- 1 cleaning basket for hand instruments
- 1 tank of KaVo LIFEtime Clean 1
- 1 tank of KaVo LIFEtime Clean 2 N
- 1 tank of KaVo water softener
- 6 cans of KaVo Spray - **packed separately**
- 1 salt filling funnel
- 1 water supply hose
- 1 water drain hose
- 1 compressed air hose
- 1 hose clamp
- 2 stopcocks
- 2 O-rings 3.8 x 1.1, order no. 200 6037
- 2 O-rings 6.65 x 0.8, order no. 200 6913
- 2 O-rings 8.3 x 0.68, order no. 200 6120
- 2 O-rings 6 x 1, order no. 200 6081
- 1 power cord
- 1 Oil device for order no. 665 0601 SONICflex

B 3 Setting up the LIFEtime

 Do not hold the LIFEtime by the front door when transporting it. Place on a horizontal, plane, water-resistant surface. If the LIFEtime must be installed above other electric equipment or installations, place on a collection tray with a capacity of at least 4 litres.

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B 3.1 Connecting the LIFETIME

Water connection

The LIFETIME is hooked up to the water supply using the supplied water supply hose ③. Use the supplied seals to seal the 3/4" connections. Open water cock after connection and inspect for leakage.

Compressed air connection

The compressed air connection ② is established by means of a plug-in connector or quick-release coupling.

Water drain

The drain hose ① is attached to the drain connector on the rear wall with the supplied hose clamp. The 90° angle of the drain hose must point upward; fix in place with the sheet metal fastening bracket.

 *The drain should be a maximum of 30 cm above the surface the LIFETIME is placed on. Route the drain hose such as to prevent accidental contact (the water draining from the unit is hot).*

Power connection

Connect the power cord ④ to a grounded receptacle. The load rating of the LIFETIME is 2.2 kW.

-  **The LIFETIME may only be connected to a properly fused and grounded power outlet.**
- The power source and LIFETIME must have identical nominal voltage ratings. The nominal voltage is printed on the nameplate on the rear wall of the unit.**
- The use of multiple socket outlets is to be avoided.**
- Do not connect any devices besides the LIFETIME to a multiple socket outlet equipped with an overcurrent protection device.**
- Extension cords equipped with a protective conductor and approved for the nominal voltage specified on the nameplate can be used if necessary.**

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E 1 Maintenance

i The unit should be inspected by an authorised technician at least once a year. For this reason, KaVo recommends a maintenance agreement.

E 1.1 Cleaning the housing

Clean all external surfaces of the LIFEtime with a soft cloth and mild cleaning solution or a mild disinfectant. Do not use solvents or aggressive chemicals.

E 1.2 Cleaning and maintenance of cleansing chamber

The cleansing chamber of the LIFEtime requires regular cleaning. We recommend cleaning the inside surfaces of the cleansing chamber at least once per week with a commercially available liquid dish detergent.

To clean, open the lid of the cleansing chamber. Remove the instrument carrier(s) or instrument basket. Remove cleaning agent residue thoroughly from the chamber after cleaning.

! There is a strainer ① at the bottom of the cleansing chamber. This strainer prevents large particles of debris from entering the inside of the unit which might cause the downstream solenoid valves to malfunction. For cleaning, pull the strainer up by the vertical handle, clean under running water and reinstall.

Check the O-rings ② on the instrument carrier couplings weekly and replace if damaged.

E 1.3 Cleaning and maintenance of instrument carriers

Check at regular intervals to see if the union nuts ⑥ are firmly tightened.

E 1.4 Spare parts required for cleaning and maintenance

① strainer,	order no. 665 0452
② O-ring 6 x 1,	order no. 200 6081
③ O-ring 8,3 x 0,65,	order no. 200 6120
④ O-ring 3,8 x 1,1,	order no. 200 6037
⑤ O-ring 6,65 x 0,8,	order no. 200 6013

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K 1 Terms of guarantee

KaVo warrants its products to the end user to be properly functioning and free of defects in material and workmanship for a period of 12 months from the date of invoice under the following conditions:

In the event of justified complaints, KaVo will fulfill its obligations under this guarantee and, at its option, repair or replace the product free of charge. Any other claims, in particular for damages, shall be excluded. In case of default, serious fault or intent, this shall only apply in as far as it does not oppose binding law.

KaVo shall not be liable for any defects and the consequences thereof which have been caused or may have been caused by natural wear and tear, improper handling, cleaning or maintenance; non-observance of operating and installation instructions; calcification or corrosion, contamination in the air and water supply; and chemical or electrical influences which are unusual or impermissible according to the KaVo operating, maintenance and assembly instructions or other manufacturers' instructions. The guarantee does not, in general, cover lamps, optical waveguides made of glass or glass fibre, glassware, rubber parts and the colour fastness of plastic parts.

Any liability shall be excluded if defects or the consequences thereof are due to manipulation of or modification to the product by the customer or third parties not authorized by KaVo-DMI.

i Claims under this warranty shall only be considered if a proof of purchase in the form of a copy of the invoice or delivery note clearly showing the sales agent, date of purchase, unit number or type and factory or serial number is presented along with the product.

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