GIMA

LAMPADA GERMICIDA
GERMICIDAL LAMP

GERMI GIMA PLUS

MANUALE D'USO USER MANUAL

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DESCRIPTION

Germicidal lamp ideal for the employment in aesthetical field and ambulatory field.
It maintains the sterilization of surgical or aesthetical irons.
It isn't adapt for sterilizing. **It isn't a medical device.** Furnished in 3 different models according to the demands. Built with body in metal with elevated resistance.

PRECAUTIONS

CAREFULLY CONSULT THE INSTRUCTION MANUAL BEFORE USING THE EQUIPMET

THE EQUIPMENT MAY ONLY BE UTILIZED BY QUALIFID PERSONNEL

**DON'T OPEN THE EQUIPMENT
FOR A TECHNICAL ASSISTANCE CONTACT GIMA AFTER SALES SERVICE**

**GIMA does everything within its power to ensure that its products are safe to use.
It is, however, of fundamental importance to observe the basic rules of safety in order to avoid electric shocks or fires.**

1. On using the equipment for the first time, and on each successive occasion, check that the power cord cable is not damaged, split or peeling. **If it is, do not insert the plug in the electric socket**
2. On using the equipment for the first time, check that no parts of it have been broken, cracked or opened during transportation. **If so, do not insert the plug in the socket**
3. Do not handle the plug with wet or damp hands. Do not use the equipment in the shower or in the bath.
4. Do not immerse the equipment in water
5. Do not allow children or incapable people to use the equipment without supervision
6. Do not leave the equipment exposed to atmospheric agents or heat sources
7. Do not utilize the equipment for uses other than those foreseen by the manufacturer. Any different use must be considered incorrect and therefore dangerous; the manufacturer cannot be considered liable for damage caused by improper, incorrect and / or unreasonable use or if the appliance is used in electrical plants that are not in compliance with the regulations in force;

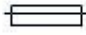
INSTRUCTIONS FOR THE CORRECT DISPOSAL OF THE PRODUCT FOR THE PURPOSE AND EFFECT OF EUROPEAN COMMUNITY DIRECTIVE 2002/96/EC::

At the end of its useful life the product must not be disposed of with domestic waste.
It may be delivered to the relevant separate waste collection centres prepared by municipal administrations, or at retailers that supply this service. Disposing of the product separately prevents possible negative consequences for the environment and for health, deriving from inadequate disposal.
It also allows the recovery of materials of witch it's made up in order to obtain an important saving of energy and resources. In order to point out the obligation to dispose of the appliances separately, the product has the sign of the crossed mobile waste container applied to it.

TECHNICAL SPECIFICATIONS

Model	GERMI GIMA PLUS
Supply	230V / 50Hz
Power Consumption	A seconda del modello 10W – 18W – 35W
Fuse	2 x 1 A
Operative Condition	Room temperature: 5 ÷ 35°C Room humidity temperature: 30 ÷ 75% RH
Storage Condition and Trasporting	Room temperature: -40 ÷ 70°C Room, humidity temperature: 10 ÷ 100% RH

SIMBOLOGY

	Mark of conformità to the European Directive 89/336/CCE; 73/23/CEE Manufactured by: CA-MI snc – Via Ugo La Malfa n°31 43010 Pilastro (PR) Italia
	Warning, consult the instruction manual
	Fuse
230V ~	AC Voltage
Hz	Frequency
	To preserve in place coolness and dry land
	Conservation temperature: -40 ÷ 70 °C

IN THE EVENT OF A FAILURE OR MALFUNCTIONING DON'T ATTEMPT TO REPAIR THE EQUIPMET, BUT CONTACT GIMA AFTER SALES SERVICE IMMEDIATELY

IF, ON INSPECTION OF THE EQUIPMET BY GIMA AFTER SALES SERVICE, THE EQUIPMENT IS FOUND TO HAVE BEEN TAMPERED WITH, THE GUARANTEE WILL BE CONSIDERED NULL AND VOID.

USER INSTRUCTIONS

Principle of operation:

Cold Sterilization through irradiation with short wave ultraviolet rays .

The vapor of mercury at to low pressure contained in the lamp, submitted to electric discharge, give out a short wave radiations that will be distribute for the 90% around the frequency of 253,7 nm, and in tall frequencies the lamp give out a visible light of bluish color.

It is just around the 253,7 nm that is developed the maximum germicidal power.

A low radiation is gived ot on the frequency of 185 nm and it provokes a small issue of ozone with deodorant effect.

Germicidal action:

The ultraviolet radiations are lethal for the microorganisms.

The quantity of radiation necessary to destroy a microorganism depends from the type of the microorganism and from the environmental conditions.

The lethal dose of radiation UV for a determined microorganism depends therefore from the intensity of the source, from the exposure time and from other factors like temperature, damp and the reflection formalities of the rays. Our apparatus has been planned and built in way to maximize the effect of the radiations and to assure an optimal result in varied uses.

It is ideal for a suitable maintenance of the tools after the sterilization.

The following chart points out the necessary times, express in second, for the destruction of some microorganisms.

Microrganism	Seconds	Microrganism	Seconds	Microrganism	Seconds
Batterius	14	Protozoi		Aspergillus Flauvus	165
Streptococco. Lactis	9	Paramecium	333	Aspergillus Flavus	550
Streptococco. Hemulyticus	10	Uova di Nematodi	36	Rhizopus Nigricans	366
Staph. Aures	10	Alghe	36	Mucor Racemosus A	58
Micrococcus Sphaeroid.	25	Virus		B di Racemosus di Mucor	58
Sarcinia Lutea	44	Bacteriophage	10	Lactis di Oospora	18
Fluorescens di Pseudomonas	18	Mosaico del tabacco	733	Fermental	
Escherichia Coli	11	Influenza	10	Elipsoideus di Saccharaomyces	22
Proteus Volgaris	13	Spore		Sp di Saccharaomyces.	29
Serratia Marcescens	10	Pencillium Requeforti	45	Cerevisiac di Saccharaomyces	22
Bacillo Subtilis	18	Pencillium Expansum	36	Fermento di birra	11
Bacillo spore di Subtilis	36	Pencillium Digitatum	146	Fermento del pane	14
Spirillum	10	Aspergillus Glaucos	146		

INSTRUCTION

- Position the appliance on a flat, stable and clean surface. **CAUTION: The ON/OFF switch on the later side of the appliance must be always easily reachable in case of emergency condition**
- Connect the electric cable to the net.
- Open the frontal cover using the special handle grip
- Insert on the intern grate the tools that it intends each other to maintain sterile.
- Close the frontal cover
- Switch ON the interrupter

The apparatus is furnished with a safety switch that prevents the operation with the frontal cover open.
Before turning on the apparatus always close the frontal counter.

MAITENANCE

- Clean the lamp with cotton soaked with alcohol.
- Perform this operation with the apparatus turned off and with cable disconnected from the alimentation.
- Clean the frontal cover with water and soap.
- Don't use solvents.
- Replace the lamp every 2-3000 ours.
- **To replace the lamp rotate it by 90° and pull it downwards. Repeat the operation in the opposite way to install the spare lamp.**

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WARRANTY

GIMA warrants it's products for **24 months** after purchasing date. In front of this warranty, GIMA will be obliged only to repair or substitute free of charge the products or parts of them that, after verification effected on our factory, or our authorized Service Center, by the Technical Service, results defective. The product must be accompanied by a description of the defect. The warranty, with exclusion of responsibility for direct and indirect damages, it is thought limited to the solos defects of material or workmanship and it stops having effect when the device results however gotten off, tampered or sheltered out of the Factory or from the Authorized Service center. The commodity always travels to risk and danger of the buyer, without any responsibility of GIMA for damages caused by the transport or dismay from the vector.

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Apparecchio tipo / Device model _____

Acquistato in data / Purchasing date _____

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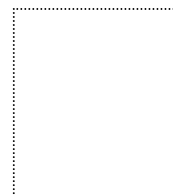
FOR WARRANTY REPARATIONS THE
POSTCARD WILL BE RETURNED WITH
THE DEVICE

TIMBRO E FIRMA
RIVENDITORE
AUTHORIZED DEALER STAMP
AND SIGNATURE

GIMA S.p.A. – Via Monza, 102 - 20060 Gessate (MI) - Italia
ITALIA : Tel. 199 400 401 (8 linee r.a.) - Fax 199 400 403
E-mail: gima@gimaitaly.com - www.gimaitaly.com
EXPORT: Tel. ++39 02 953854209 - Fax ++39 02 95380056
E-mail. export@gimaitaly.com - www.gimaitaly.com

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GIMA S.p.A. - Via Monza, 102 - 20060 Gessate (MI) - Italia