SFIGMOMANOMETRO A MERCURIO
MERCURIAL SPHYGMOMANOMETERS
TENSIOMETRES A COLONNE DE MERCURE
ESFIGMOMANÓMETROS DE MERCURIO
ESFIGMOMANÓMETRO A MERCÚRIO

Istruzioni per l'uso
Instructions for use
Mode d'emploi
Manual de instrucciones
Instruções de uso

I

GB

F

E

P

FAZZINI

Your health is our aim

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EC Certificate**No.: G3M 01 10 44963 004**

Decision according to Annex VI of Council Directive 93/42/EEC concerning medical devices.

The Certification Body of TÜV PRODUCT SERVICE certifies that

FAZZINI s.r.l.
s.s. Padana Superiore 317
20090 Vimodrone Milano
Italy

in the facility(ies)

FAZZINI s.r.l.
s.s. Padana Superiore 317
20090 Vimodrone Milano
Italy

for the product(s)/product category(ies)

Aneroid sphygmomanometers
Mercury sphygmomanometers

maintains quality assurance which ensures the conformity with the type described in the technical documentation related to the measuring function.

Reasoned assessment see audit report no.: ITA 116141.

Provided the agreed periodical surveillance is carried out, this certificate is valid until 2006-10-21.

Released with the above mentioned certificate number by the Certification Body of TÜV PRODUCT SERVICE.

Department: MHS ITA / rg-tgu
 Date: 2001-10-22

TÜV PRODUCT SERVICE GMBH is Notified Body according to Council Directive 93/42/EEC concerning medical devices with identification no. 0123.

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ENGLISH

Contents	Page
Information on the various models of Fazzini sphygmomanometers	12
Selection of suitable cuff sizes	12
How to put on the cuffs	13
How to measure blood pressure	13
How to care for the mercurial sphygmomanometer	14
Important Information on Dealing with Mercury	14
Disposal Information	15
Test of precision	15
Maintenance	16
Technical Data	16

Key to symbols on the manometer scale.

Important: please read the documentation

Information on the various models of Fazzini sphygmomanometers

Thanks for purchasing a high-quality Fazzini precision sphygmomanometer manufactured in conformity with UNI-EN standard 1060/1: "Non-invasive blood pressure instruments"

Table model

Open the case pushing the button on the front part of the case and pushing upwards the cover until it reaches vertical position. To measure put the apparatus in horizontal position.

System operating against mercury overflow : these models have a safety system, shaped as a lever, to prevent mercury overflow. When you purchase it the lever is turned towards right, that means that safety is turned off. Before using the apparatus you should turn the lever towards left until release, in order to let mercury flow from its container and go up till point zero of the box.

2-Tube cuffs : connect the tube connector of the apparatus to the tube connector of the cuff.

Selection of suitable cuff sizes

Cuffs

Our cuffs have a tufted strip on one side and a snap on the other. This ensures that the cuffs can be easily opened and closed quickly and repeatedly. On all the models the cuffs are calibrated, i.e. provided with measurement lines. To make sure that you have chosen the right cuff size, check to see whether the white index line is in the range between the arrows after the cuff has been put on. If the index line fails to reach this range, the cuff is too small. If it is beyond the range, the cuff is too large. Precise blood pressure readings can be obtained only if the correct cuff size is used.

The following cuff sizes are available for all models:

cm.22x4,5 art. 080.117.10	newborn
cm.40x5 art. 080.118.10	baby
cm.50x10 art. 080.119.10	child
cm.50x14 art. 080.120.10	adult
cm.60x17 art. 080.123.10	obese

How to put on the cuffs

Cuffs

Place the cuff 2 cm above the bend of the elbow on the naked left upper arm; make sure that the symbol “ø” is above the artery. Close the cuff by means of the velcro closure.

How to measure blood pressure

After you have put on the cuff, pump it up by pressing the bulb until the pressure in the cuff is about 20 mm/Hg above the expected systolic blood pressure (upper value). It is easy to read the values by comparing the column of mercury, which rises due to pumping, with the graduated scale.

It is very important that the person whose blood pressure is being taken is relaxed and sitting comfortably. Furthermore, the lower arm should be placed at the level of the heart with the inside of the arm facing upwards. Place the chestpiece of the stethoscope on the artery under the cuff. In the model 08.303.00 stethoscope is already included.

To be able to measure blood pressure, open the air-release screw on the bulb by turning it counter-clockwise. Ideally, the rate of air release should be 2-3 mm Hg/s. This rate can be adjusted by turning the screw very carefully. Perform a visual check of the air-release rate by looking at the scale: the mercury display must move along the scale at a speed of 1 - 1.5 graduations per second.

Once the upper blood pressure value i.e. (systolic pressure) has been reached, a rhythmic thumping sound is heard. When the lower blood pressure value (diastolic pressure) is reached, the thumping sound stops.

Now open the air-release valve completely so that air can escape from the cuff immediately.

The blood pressure measurement procedure has now been completed.

We expressly call your attention to the fact that only a physician is capable of precisely analysing the blood pressure readings you have taken.

We recommend locking the unit after every measurement and any time the unit is transported. To do this, turn the unit on its side carefully until no more mercury is visible. Then dose the mercury lock.

How to care for the mercurial sphygmomanometer

Manometer and bulb

Cleaning:

Wipe off the manometer and bulb with a damp cloth. Disinfection is non necessary, since the parts do not come into direct contact with the patient's body.

Cuffs

Cleaning:

After removing the bladder, wipe the cotton covers with a damp cloth. Alternatively, these can be washed with soap and cold water like all the other cuffs. If you decide on the latter course, rinse the cuffs with clear water afterwards and let them air dry. Wipe the bladder and tubes with a damp cloth.

IMPORTANT!

Do not iron cuffs!

Never expose the cuffs and the bulb to intensive solar radiation!

Never touch the cuff fabric or latex parts with a sharp instrument, since this could cause damage!

Important Information on Dealing with Mercury

Guidelines and Precautionary Regulations

Mercurial sphygmomanometers should be handled carefully. Take care not to drop the unit or knock it to the floor, and handle the unit gently to avoid damaging it. To ensure that no mercury escapes from your unit, you should send it to your specialised dealer for periodic inspection.

Health and Safety Issues When Dealing with Mercury

Exposure to mercury can have grave toxicological effects. Absorption of mercury leads to neuro-psychiatric disturbances and, in extreme cases, kidney disease. For this reason, precautions should be taken when servicing mercurial sphygmomanometers. Units that are being cleaned and repaired should be placed on a smooth, impermeable tablet that is tilted away from the operator at a horizontal angle of 10° and has a water-filled depression. Wear suitable gloves (e.g., latex gloves) to avoid direct skin contact. Ensure that your work area is well ventilated and avoid coming into contact with or breathing vapours. A high level of workplace hygiene is imperative in areas in which mercurial apparatus is repaired. It is known that persons who repair mercurial apparatus are subject to chronic mercury exposure.

Disposal Information

Mercury must be recycled or disposed of as waste requiring special monitoring.

Test of precision

Remove the tube from the manometer.

Open the mercury lock.

The mercury column should come to rest at "0". If the mercury column comes to rest below the 0, mercury must be added. In this case, either take it to a specialised dealer or send it to us for recalibration.

Maintenance

In accordance with the testing requirements set down in the Medical Product Law (MPG), a maintenance period of two years should be adhered to.

Technical Data

Minimal environmental conditions under which the error tolerance of +/- 3 mm/Hg must be satisfied in accordance with the UNI-EN 1060/1 standard:

Measurement conditions:	10°C to 40°C at a relative air humidity of 85% (non-condensing).
Storage conditions:	-20°C to 70°C at a relative air humidity of 85% (non-condensing).
Versions:	Desk, wall, stand, and anaesthetic models.
Type of indication:	Mercury column.
Scale graduation	Increments of 2 mm/Hg.
Range shown on scale:	0 to 300 mm/Hg.
Measurement range:	0 to 300 mm/Hg.
Tube adapters:	2
Pressure generation:	Bulb.
Pressure reduction:	Air-release valve that can be regulated.

NOTE

Please do not use or store mercurial sphygmomanometers in areas in which there is danger of explosion.

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