SAVE THESE INSTRUCTIONS

NOTE: Check ALL parts for shipping damage. If shipping damage is noted, DO NOT use. Contact Carrier/Dealer for further instruction.

Safety Summary

To ensure the safe and proper installation of the tub bars, these instructions **MUST** be followed:

⚠ WARNING

DO NOT install this equipment without first reading and understanding this instruction sheet. If you are unable to understand the warnings, cautions and instructions, contact a healthcare professional, dealer or technical personnel if applicable, before attempting to install this equipment - otherwise, injury or damage may occur.

ALWAYS check the tub bar for proper tightness before using.

The tub bar provides support and increases stability for an individual weighing up to 250 lbs (114 kg). The tub bar is not designed to support the total weight of an individual. Use the tub bar for assistance only.

Hold the tub bar securely when entering or exiting the bath tub.

DO NOT use the upper gripping position (handle) of the tub bar when lowering or raising yourself into or out of the bathtub. The upper gripping position (handle) is **NOT** designed to support an individual's total weight. Pushing or pulling the upper gripping position (handle) in a sideways motion will loosen the tub bar from its secured position and may cause injury.

Apply downward force on the lower gripping position only when lowering or raising yourself into or out of the bathtub.

Invacare products are specifically designed and manufactured for use in conjunction with Invacare accessories. Accessories designed by other manufacturers have not been tested by Invacare and are not recommended for use with Invacare products.

⚠ INSTALLATION WARNING

After any adjustments, repair or service and before use, make sure that all attaching component parts are secure.

Bathtub construction varies and this product may not be appropriate for use on all bathtub styles. Bathtubs with sloped inner walls may not provide the tub bar sufficient holding ability. Given this condition the assembly can become dislodged during use. After installation, verify that the device holds as expected. **ALWAYS** verify that the device is secure prior to use.

For more information regarding Invacare products, parts, and services, please visit www.invacare.com

⚠ INSTALLATION WARNING

DO NOT use this device on bathtubs with lipped walls.

DO NOT overtighten tension lever. Instability to the tub bar and/or bath tub damage can occur.

Installing the Tub Bar

NOTE: For this procedure, refer to FIGURE 1.

1. Remove the tub bar from the package.
2. Flip up the tension lever.
3. Rotate the tension lever counter-clockwise to adjust the internal non-skid pad.

⚠ WARNING

Ensure that inside and outside area of bath tub wall where tub bar will be placed is clean and dry. Otherwise, once installed, the tub bar may not be fully secure to the tub wall and cause injury to the user.

4. Place the tub bar over the top of the tub wall with the external non-skid pad against the outside of the tub wall and the internal non-skid pad on the inside of the tub wall.
5. Place the tub bar upright with the cutout portion flat on the top of the tub wall and turn the tension lever clockwise until considerable resistance is felt, but **DO NOT** overtighten.
6. Press down on the the tension lever to secure the tub bar to the tub wall.

NOTE: Inspect the tub bar for stability by putting a reasonable amount of weight onto the grip and then try to move it from side to side.

7. If tub bar is unstable, repeat STEPS 2 - 6.

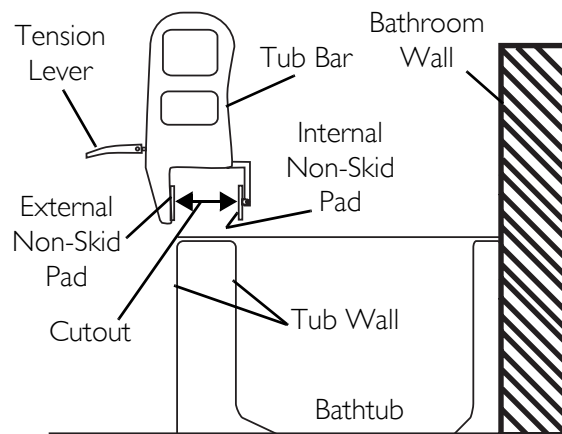
FIGURE 1 Installing the Tub Bar

Using the Tub Bar

NOTE: For this procedure, refer to FIGURE 2.

When entering or exiting the bathtub, hold the tub bar securely by applying force on the lower gripping position.

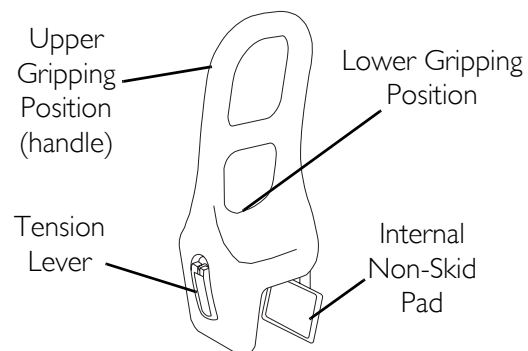
FIGURE 2 Using the Tub Bar

Care and Maintenance

1. Use mild soap and water to clean.
2. An antibacterial cleaner is recommended for decontamination.
3. Use a silicone lubricant for the threads of the tension lever to inhibit rust.
4. Periodically inspect the tub bar for wear, damage, etc. DO NOT use if any damage exists. Contact dealer.

LIMITED WARRANTY

PLEASE NOTE: THE WARRANTY BELOW HAS BEEN DRAFTED TO COMPLY WITH FEDERAL LAW APPLICABLE TO PRODUCTS MANUFACTURED AFTER JULY 4, 1975.

This warranty is extended only to the original purchaser/user of our products.

This warranty gives you specific legal rights and you may also have other legal rights which vary from state to state.

Invacare warrants its product to be free from defects in materials and workmanship for three years for the original purchaser. If within such warranty period any such product shall be proven to be defective, such product shall be repaired or replaced, at Invacare's option. This warranty does not include any labor or shipping charges incurred in replacement part installation or repair of any such product. Invacare's sole obligation and your exclusive remedy under this warranty shall be limited to such repair and/or replacement.

For warranty service, please contact the dealer from whom you purchased your Invacare product. In the event you do not receive satisfactory warranty service, please write directly to Invacare at the address on the back page. Provide dealer's name, address, model number, date of purchase, indicate nature of the defect and, if the product is serialized, indicate the serial number.

Invacare Corporation will issue a return authorization. The defective unit or parts must be returned for warranty inspection using the serial number, when applicable, as identification within thirty days of return authorization date. **DO NOT** return products to our factory without our prior consent. C.O.D. shipments will be refused; please prepay shipping charges.

LIMITATIONS AND EXCLUSIONS: THE WARRANTY SHALL NOT APPLY TO PROBLEMS ARISING FROM NORMAL WEAR OR FAILURE TO ADHERE TO THE ENCLOSED INSTRUCTIONS. IN ADDITION, THE FOREGOING WARRANTY SHALL NOT APPLY TO SERIAL NUMBERED PRODUCTS IF THE SERIAL NUMBER HAS BEEN REMOVED OR DEFACED; PRODUCTS SUBJECTED TO NEGLIGENCE, ACCIDENT, IMPROPER OPERATION, MAINTENANCE OR STORAGE; OR PRODUCTS MODIFIED WITHOUT INVACARE'S EXPRESS WRITTEN CONSENT INCLUDING, BUT NOT LIMITED TO: MODIFICATION THROUGH THE USE OF UNAUTHORIZED PARTS OR ATTACHMENTS: PRODUCTS DAMAGED BY REASON OF REPAIRS MADE TO ANY COMPONENT WITHOUT THE SPECIFIC CONSENT OF INVACARE; PRODUCTS DAMAGED BY CIRCUMSTANCES BEYOND INVACARE'S CONTROL; PRODUCTS REPAIRED BY ANYONE OTHER THAN AN INVACARE DEALER, SUCH EVALUATION SHALL BE SOLELY DETERMINED BY INVACARE. THE FOREGOING WARRANTY IS EXCLUSIVE AND IN LIEU OF ANY OTHER EXPRESS WARRANTIES, IF ANY, INCLUDING THE IMPLIED WARRANTIES OF MERCHANTABILITY AND FITNESS FOR A PARTICULAR PURPOSE.

IT SHALL NOT EXTEND BEYOND THE DURATION OF THE EXPRESSED WARRANTY PROVIDED HEREIN AND THE REMEDY FOR VIOLATIONS OF ANY IMPLIED WARRANTY SHALL BE LIMITED TO REPAIR OR REPLACEMENT OF THE DEFECTIVE PRODUCT PURSUANT TO THE TERMS CONTAINED HEREIN. INVACARE SHALL NOT BE LIABLE FOR ANY CONSEQUENTIAL OR INCIDENTAL DAMAGES WHATSOEVER.

THIS WARRANTY SHALL BE EXTENDED TO COMPLY WITH STATE/PROVINCIAL LAWS AND REQUIREMENTS.

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© 2006 Invacare Corporation
Invacare, the Medallion Design, and Yes, you can. are registered trademarks of Invacare Corporation.