**MANUALE UTENTE  
USER MANUAL  
ANWENDERHANDBUCH**

***Negativoscopio***

# **MEDicanvas**

**Art. 7072-N / 7074-N / 7076-N / 7078-N**

**CE**

**ITALIANO**

**ENGLISH**

**DEUTCH**

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Declaration of conformity	2
Warranty condition	3
Loss of warranty	4
Delivery terms	5
General warnings and safety rules	6
Useful contacts and addresses	6
Symbols	6
Product description	8
Technical specifications	9
Positioning	10
Installation	11
Description of machine operation	13
Demolition and dismantling	14
Maintenance and troubleshooting	15

**DECLARATION OF CONFORMITY**

This device compliance to Directive 93/42/EEC .

The original declaration of conformity is provided in attached to the manual.

**WARRANTY CONDITION**

- 1) **DURATION:** the product is covered by a 12-month (twelve) guarantee.
- 2) **EXCLUSION:** the guarantee excludes:
  - a) faults originating from non-performance of scheduled maintenance and due to negligence or improper use of the product by the User;
  - b) periodic checks and maintenance;
  - c) repair or replacement of parts subject to wear that are fragile or have an unpredictable life, unless they prove faulty at the time of their delivery;
  - d) faults caused by personnel or originating from technical personnel transfer and transportation;
  - e) faults or damage originating from improper use or operating errors;
  - f) faults or damage originating from contamination in the water and air infeed systems, extraordinary chemical agents or electrical events;
  - g) faults or damage originating from the use of detergents, disinfecting agents, sterilizing fluids or processes that are not described explicitly in the instruction manual for operation and maintenance;
  - h) natural color change of plastic components.
- 3) **INSTALLATION AND TESTING REPORT:** an essential prerequisite for guarantee validity is the return of the installation and testing report after all its sections have been filled in by the User and the Fitter. The report must be returned within 15 days after installation, otherwise the guarantee becomes null and void.
- 4) **LIMITS:** the guarantee entitles its holder to repairs or replacement of faulty components free of charge. The right to replace the entire machine is excluded. As for components either applied or added to TECNO-GAZ products or by third parties, having their own guarantee certificate, the conditions, limits and exclusions indicated in the relevant certificates apply.

Under no circumstances shall carrying out of one or more than one repair during the guarantee term change the guarantee expiry date.
- 5) **CLAIMS:** in the event of the Purchaser making a claim relating to the application of the guarantee and the quality or conditions of the delivered equipment, the said Purchaser shall in no way suspend and/or delay payments.
- 6) **DISPUTES:** if a dispute arises on the application and interpretation of this Certificate of guarantee, the Court of PARMA (Italy) shall be the court of jurisdiction, regardless of the location in which the purchase agreement was stipulated.
- 7) **DEROGATIONS:** any special derogations from the guarantee terms described in this section shall not imply the granting to the Purchaser of any right and shall be considered as granted for a specific case only.
- 8) **MISCELLANEOUS:** for any other issue, which is not covered by this Certificate of Guarantee, the rules described in the Civil Code and existing relevant Laws in the Republic of Italy apply.

## LOSS OF WARRANTY

The guarantee becomes null and void in the event that:

- a) the equipment presents damage caused by a fall, exposure to flames, liquid spillage, natural events, bad weather conditions or by any other cause that is not due to manufacturing defects;
- b) installation has not been performed in line with TECNO-GAZ instructions and by authorized personnel;
- c) the equipment is repaired, modified or tampered with by the Purchaser or by non-authorized third parties;
- d) when asking for an intervention under guarantee, the product serial number has been removed, deleted, counterfeited, etc.;
- e) the installation and testing report is not returned suitably filled in and signed within 15 days from installation;
- f) the Purchaser suspends and/or delays payment of amounts due, for any reason whatsoever, relating to the purchase and/or maintenance of the equipment;
- g) scheduled periodic maintenance or any other instruction envisaged in the manual for operation and maintenance is not carried out.

**DELIVERY TERMS**

1. All goods delivered must be packaged in their original packaging. If the goods are delivered in a non-genuine packaging, a charge will be applied to restore the original packaging.
2. All goods must be delivered CARRIAGE PAID.
3. The client shall ask a written authorization to TECNO-GAZ sales dept. before returning the goods by means of the "Form for Client's Returned Goods Management" (CM-M-002) in compliance with the requirements of TECNO-GAZ Quality Management System (ref. CM-P-003, CM-I-009).
4. The document accompanying the returned goods shall show:
  - the TECNO-GAZ product ode/item no.;
  - the number of the TECNO-GAZ purchase invoice or shipment document;
  - the product serial number and batch number;
  - a description of the goods returned;
  - the number of the Form for Client's Returned Goods Management" (CM-M-002) or a copy of the same attached.
5. Returned goods are considered as accepted only after they have been examined. If they prove non-conforming, they may be sent back to the consignor at TECNO-GAZ full discretion.
6. Charges will be applied to restore and test the returned product.
7. Parts replaced under guarantee must be returned, carriage paid, to TECNO-GAZ.
8. If the parts are not returned, their cost will automatically be debited to the recipient.
9. TECNO-GAZ does not accept goods returns from end users.
10. Goods sent to TECNO-GAZ for repair are managed as returned goods.

## GENERAL WARNINGS AND SAFETY RULES

-  Make sure that the equipment is supplied with the correct voltage as shown in the plate.
-  Do not remove the plate.
-  Make sure that the machine is equipped with a grounding system.
-  Use a dry cloth to clean the machine.
-  Before any operation is performed, disconnect the power cable from the socket.
-  Use original spare parts only.

*Non-compliance with the rules above relieves of all responsibility.*

## USEFUL CONTACTS AND ADDRESSES

For technical service please apply to the dealer that invoiced the device, or directly to:

TECNO-GAZ SpA  
Str. Cavalli n°4 – Sala Baganza – Parma – ITALIA.  
Tel. +39 521 8380 - Fax +39 521 833391  
e-mail : [info@tecnogaz.com](mailto:info@tecnogaz.com)  
<http://www.tecnogaz.com>

## SYMBOLS

TECNO – GAZ S.p.A.  
Strada cavalli n. 4 – 43038 – Sala Baganza – Parma - ITALY

Pursuant to DIRECTIVE 2002/96/EC, this symbol shows that the product must not be disposed of as urban waste at the end of its operating life.

General warning and information for the user.

Pay attention to the instructions provided alongside this symbol.

Conformity with the Directive 2004/108/CE and Directive 2006/95/CE and further amendments.

PE

External protection.

	Protection GROUNDING SYSTEM.
	(Operating) GROUNDING SYSTEM.
	Alternate current.
	Warning: high voltage. Risk of electrocution!
	Read the instruction manual carefully.
	Applied part.
	Press the button shown.
<b>POS.</b>	Position (number identifying a component in the figures or diagrams).
<b>Fg.</b>	Figure
$\Omega$	Ohm (Electric Resistance unit).
<b>s</b>	Seconds (time unit).
<b>W</b>	Watt (Power unit).
<b>Hz</b>	Hertz (Frequency unit).
<b>mm</b>	Millimeters (Length unit).
<b>A</b>	Ampere (Electric Current unit).
<b>V</b>	Volt (Electric Voltage unit).
<b>Pa</b>	Pascal (Atmospheric Pressure unit).
<b>°C</b>	Degrees Centigrade (Temperature unit).
<b>kg</b>	Kilograms (Weight unit).

PRODUCT DESCRIPTION

■ Accessories

● Adapter & Power cord

● Accessory for setting

Setting Tape	1ea
Plastic Anchor	3ea
Screws	3ea
Nails	3ea

◆ Optional Type

Handle (7075-N)  
Only for 7072-N and 7074-N

**TECHINICAL SPECIFICATION**

MODEL NAME		7072-N	7074-N
Dimensions (unit : mm)	Frame	498 X 482 X 28	854 X 482 X 28
	Viewing Area	14" X 17" 356 X 423	28" X 17" 711 X 423
Power Consumption		36 W	72 W
Light Source		CCFL	
Wall Mount Dimension		454.6 mm	810.8 mm
Power Supply	Input	AC 250V, 50/60Hz	
	Output	DC 12V±5%, 3.5 A	DC 12V±5%, 6.67 A
Storage Temperature		10 °C ~ 40 °C	
Relative Humidity		-15 °C ~ 60 °C	
Altitude		5 ~ 90 %	
Optional		0 ~ 10,000 Feet	
Net Weight		Handle	Handle
Storage Temperature		5.4 kg	9.6 kg

MODEL NAME		7076-N	7078-N
Dimensions (unit : mm)	Frame	1230 X 482 X 39	1606 X 482 X 39
	Viewing Area	43" X 17" 1087 X 423	58" X 17" 1463 X 423
Power Consumption		108 W	144 W
Light Source		CCFL	
Wall Mount Dimension		476 mm	554 mm X 554 mm
Power Supply	Input	AC 250V, 50/60Hz	
	Output	DC 12 V±5%, 6.67 A 2EA	
Operating Temperature		10 °C ~ 40 °C	
Storage Temperature		-15 °C ~ 60 °C	
Relative Humidity		5 ~ 90 %	
Altitude		0 ~ 10,000 Feet	
Net Weight		13.1 kg	17.1 kg

These specifications are subject to change without notice for improvement of the MEDicanvas.

**POSITIONING**

Temperature	- 20 ÷ 45	°C
Humidity	10 ÷ 90	%
Pressure	500 ÷ 1060	hPa

Before installing, check the integrity of the device; if there are signs of transit damage, please notify the dealer.

**WARNING!** Do not switch on the device.

The device must be placed far away from instruments that generate strong heat emissions and electromagnetic radiations that may damage the installed electronics.

Failure to comply with the above will terminate the guarantee.

Disposal of the packaging components must be carried out in compliance with the current regulations in the country where the device is used.

## INSTALLATION

Because of the characteristics of the device is important to follow certain precautions:

 **The device should not be used in potentially explosive atmospheres.**

 **Remove packaging, and check the status of the unit.**  
**If any signs of damage to vehicles, alert the manufacturer.**

 **Packaging must be preserved throughout the warranty period.**  
**The manufacturer does not accept returns without original packaging.**

It must be transported following the rules for the transportation of fragile goods.

The original packaging should be kept for the duration of the guarantee period, since only in this container it is possible to avoid damages during transport.

 **Place the unit on a horizontal surface, away from heat sources.**

The operator can hang it on a concrete wall or a wooden wall for use.

In the case of the material frame of the wall being strong concrete, light-weight concrete, strong natural stone, tender natural stone, processed brick, inside-empty block and wood, the operator can use a bolt and a nail.

In the case of the thickness of the finish material being less than 10mm, the operator can use a bolt and nail.

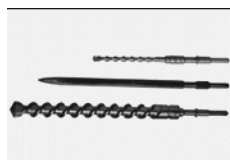
Don't install it on the wall for which only plaster board is used.

Attach installation tape in horizontal style.

Hammer  
Drill

Hammer  
Bit


FD  
plastic  
anchor

After the operator makes a hole on the wall by a hammer drill and attach FD plastic anchor to the hole, he fastens tapping nuts (ST4.0 X 30.0L) by a driver so that they can be projected 1cm out of the surface. Or after he fastens them by a concrete nail (3.5x40L) on the surface of the wall, he installs a product.

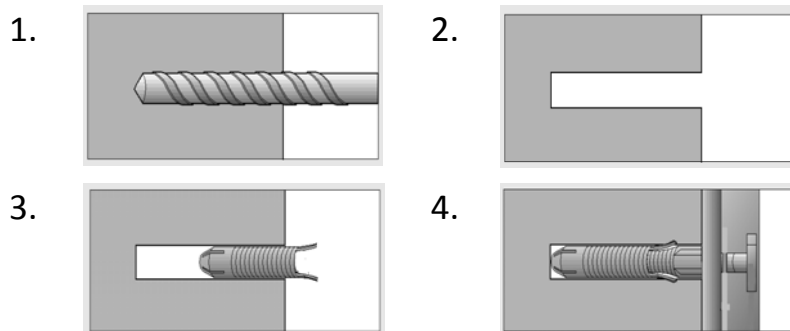Installation order:

**Notice:**

In the case of using the product, fastening it on the wooden wall, install it after fastening tapping nuts at a point where the thickness of the wood of the wall surface is more than 30mm.

In the case of using concrete nails, put them fully and fasten them tightly.

Install the product at a flat place.

Don't install the product at a place which is exposed to the sunray or at a place near hot articles such as furnace and so on. ( Be careful of the danger of fire)

Don't install at a place with a lot of water or moisture.

Install at a drafty place.

Install after securing enough space for the safe installation of the product

**DESCRIPTION OF MACHINE OPERATION**

- Plug the Adapter into the specified voltage source only.

**1) Insert female end into socket of MEDlcanvas.**

**2) Plug Adapter into wall outlet AC230V (or 110V).**

**3) Turn on the switch, the panel will illuminate.**

- Use only the adapter provided with this MEDlcanvas

- Use only the adapter provided with this MEDlcanvas after checking the adapter voltage.

7072-N : DC 12V±5%, 3.5°

7074-N : DC 12V±5%, 6.67°

7076-N, 7078-N : DC 12V±5%, 6.67A (2EA)

- Disconnect the plug from the Adapter if the product is not in use for a long period of time (over one week).

- Unit will preheat in about 1 minute once the product is switched on.

Initially, the light might not be that bright but after about a minute or two it will brighten. This may also occur when the room temperature is very low.

- In case the product is switched on and the plug is separated from the adapter. The screen may still be somewhat illuminated for a few moments. However, do not be alarmed. It does not mean it is out of order.

**ATTENTION:**

All negatoscopes Medicanvas are equipped with FAS system: if you insert the X-RAY on the top between the screen and the frame, the X-RAY is blocked and the negatoscope switches on automatically

## DEMOLITION AND ELIMINATION

Directive on waste from electrical and electronic equipment (WEEE)

Pursuant to DIRECTIVE 2002/96/EC, this symbol shows that the product must not be disposed of as urban waste at the end of its operating life.

The product must be taken to centers specialized in separate collection of electric and electronic equipment or to a dealer, when a similar product is purchased.

The machine holder shall be responsible for bringing it to collection centers.

For further information about collection systems, address the local waste disposal service.

**Correct disposal of the equipment, which is no longer used, prevents negative consequences for the environment and human health.**

**Abuses will be prosecuted by law.**

## MAINTENANCE AND TROUBLESHOOTING

**DO NOT DO MAINTENANCE OR INSIDE REPAIR TO THE DEVICE.**  
**DISMANTLE THE DEVICE MEANS BREAKING IT.**

	<i>Defect</i>	<i>Treatment</i>
1	Fails to illuminate	Check that the transformer is connected
2	Fails to illuminate	Check that the plug in power is properly inserted

If the device does not work properly or not at all, contact an authorized service centre. The fuse is non-replaceable, Please contact our distributor or service center.

### Diagnostics and troubleshooting

If the device does not work properly or not at all, contact an authorized service centre. It is not possible to work directly on the device.

The only work allowed is the following:

- Please clean the exterior of the MEDlcanvas with a clean damp cloth and then, wipe and dry with a clean soft cloth.
- Please do not use volatile organic solvents such as alcohol, thinner, and benzenes, when cleaning the frame. Otherwise, it may be discolored or the resin may melt.
- Please wipe the front acryl panel with a dry cloth on a regular basis as a dirty panel may affect film reading

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**TECNO-GAZ S.p.A.**