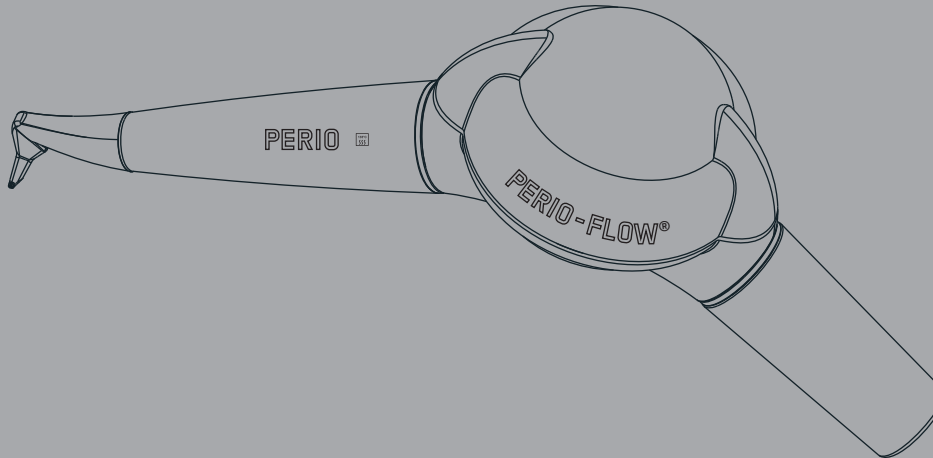
OPERATION INSTRUCTIONS
MODE D'EMPLOI
GEBRAUCHSANWEISUNG
INSTRUCCIONES DE FUNCIONAMIENTO
ISTRUZIONI PER L'USO
BRUKSANVISNING

AIR-FLOW® handy PERIO

ADDITIONAL SAFETY ADVICE FOR USA:

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DEAR CUSTOMERS,

Thank you for buying a new EMS product. It meets the highest quality and safety standards.

Adaptable on the turbine connection of the dental unit. The AIR-FLOW handy PERIO removes the subgingival biofilm during periodic periodontal treatments. Its long fine nozzle is specifically developed for easy access to the periodontal pocket.

Indication:

All patients suffering from periodontal disease and peri-implantitis.

This system is specially recommended for:

- Periodontal maintenance of deep pocket up to 10 mm following initial treatment
- Periodontal biofilm removal
- Implant cleaning

Contra-indications:

 The treatment of deep periodontal pockets can cause bacteraemia. Please apply appropriate restrictions for the treatment of risk patients:

- Endocarditis
- Pregnancy, breast feeding
- Contagious disease
- Immunodeficiency (neutropenia, agranulocytosis, diabetes, hemophilia)
- Patients under treatment (radiotherapy, chemotherapy, antibiotics)

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PLEASE READ THIS BEFORE BEGINNING WORK!

This operation instructions are to ensure the correct installation and use of this product.

Please read these operation instructions carefully as they explain all the most important details and procedures. Please pay special attention to the safety precautions.

Always keep this instruction close at hand.

To prevent injury to people and damage to property, please heed the corresponding directives. They are marked as follows:

Danger
Risk of injury

Caution
Risk of property damage or environmental harm

Please note
Useful additional information and hints

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Tillåtet

About this manual

Note that the English version of this manual is the master from which translations derive. In case of any discrepancy, the binding version is the English text.

We would be pleased to answer your questions or listen to your suggestions. We do, of course, provide support in case of technical problems. Please call directly your approved EMS repair center or your dealer.

We wish you lots of success!

EMS

TECHNICAL MODIFICATIONS

EMS reserves the right to modify the technique, accessories, operation instructions or contents of the set due to technical or scientific improvements.

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CONTENTS

General information	2
Setting the dental unit	4
Water supply	4
Compressed air supply	4
Preparing for treatment.....	6
Drying the sterilized handpiece	6
Nozzle installation and handpiece connection.....	8
Installation and setting up.....	10
Checking the turbine connection (Typical is shown).....	10
Connecting to the dental unit	12
Setting the water flow rate	14
Filling the powder chamber.....	16
Closing the cap	18
Precautions	20
Wear a mask and eye protection	20
Treatment	24
Positioning of the saliva pump and the high speed evacuator suction nozzles.....	24
Working Technique	26
End of the treatment	32
Removing the cord, the handpiece and the nozzle	32
Cleaning, disinfecting and sterilizing	34
Clean the handpiece after treatment	34
Sterilizing the handpiece.....	38
Clean the device after treatment.....	40
Daily cleaning of the device	42
Daily cleaning of the cap.....	44

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Maintenance.....	46
Replacing the dome and the O-rings.....	46
Safety precautions	48
Storing the product	52
Product disposal	54
Warranty.....	54
Accessories.....	54
EMS-Service.....	56
Symbols	58
Technical data	60
Troubleshooting	62

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GENERAL INFORMATION

 The air and powder jet may cause breathing difficulties. Please apply appropriate restrictions for the treatment of risk patients. Patients suffering from chronic bronchitis or asthma must not be treated under any circumstances with this product.

 Predisposed persons may be sensitive to the powder. If allergic reactions are observed, stop using the product and completely remove it.

 The nozzle cannot be sterilized.

 The single use nozzle must be used for one single patient only. Never reuse a nozzle because treatment will be ineffective and the risk of emphysema would increase.

 The use of any other powder than the EMS powders for subgingival application reduces the nozzle's service life. As a result the treatment would become ineffective and would increase the risk of emphysema.

 For more details about the EMS powder, please read the relative operation instructions.

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 Following the sterilization, blow the interior of the handpiece with compressed air to eliminate possible humidity.

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
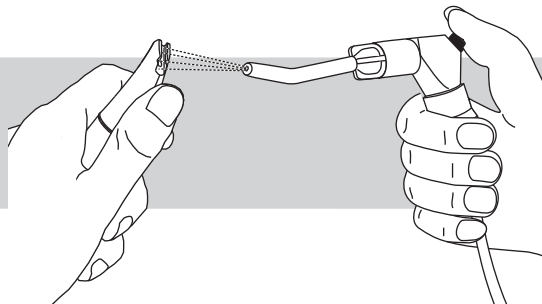
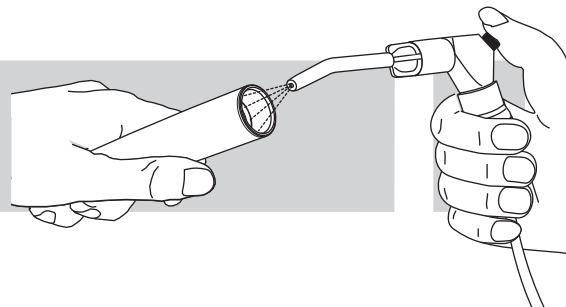
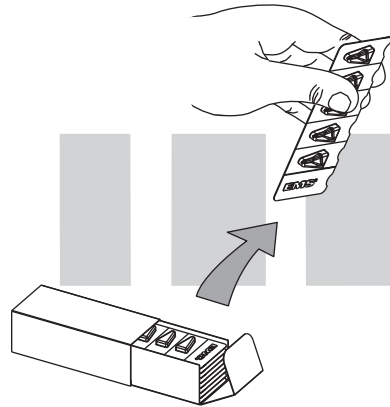
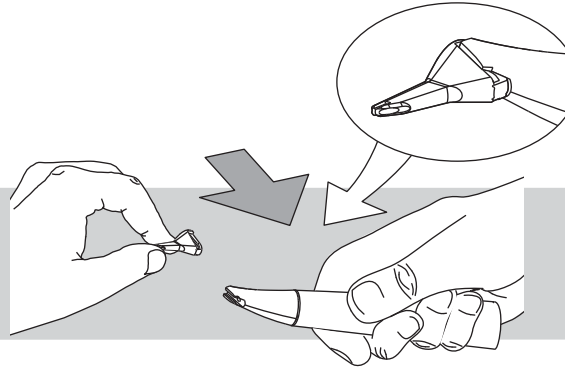
Nozzle installation and handpiece connection

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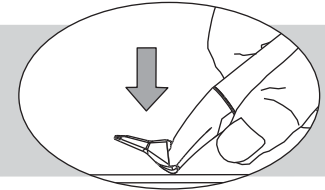 Before each use, make sure the nozzle is correctly attached. The nozzle must be fully inserted.

 Do not use the nozzle in case of doubt that package seal was broken, or that package or the nozzle is damaged.

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INSTALLATION AND SETTING UP

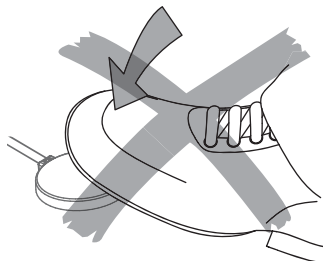
Checking the turbine connection
(Typical is shown)

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⚠ The device is equipped with a connector specially designed for the turbine connection of your dental unit. Only use the device with this special turbine connection. To connect it to another type of turbine will damage it.

⚠ The turbine of your dental unit must not be under pressure when connecting the device. Do not activate the turbine footpedal. If you have a turbine equipped with light, turn the light off.

⚠ Make sure the o-rings of your turbine connection are in good condition. A turbine connection with o-rings in bad condition can damage the device.

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
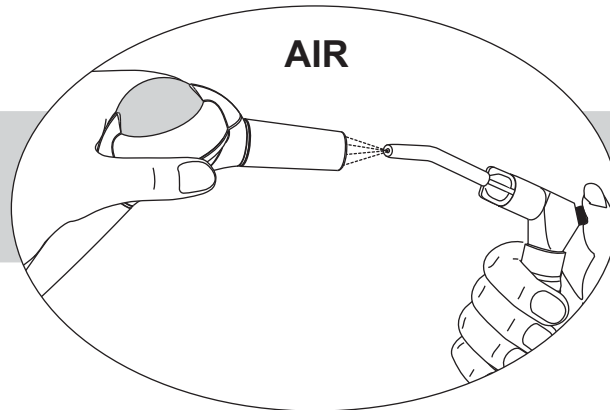
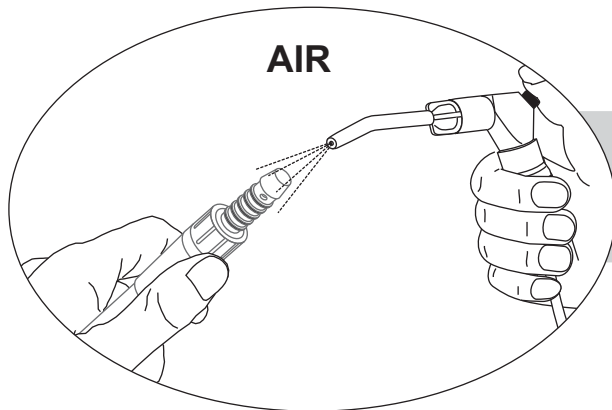
 The turbine connection and the connector must be absolutely dry. Moisture at the connection can clog the air/powder channels of the device.

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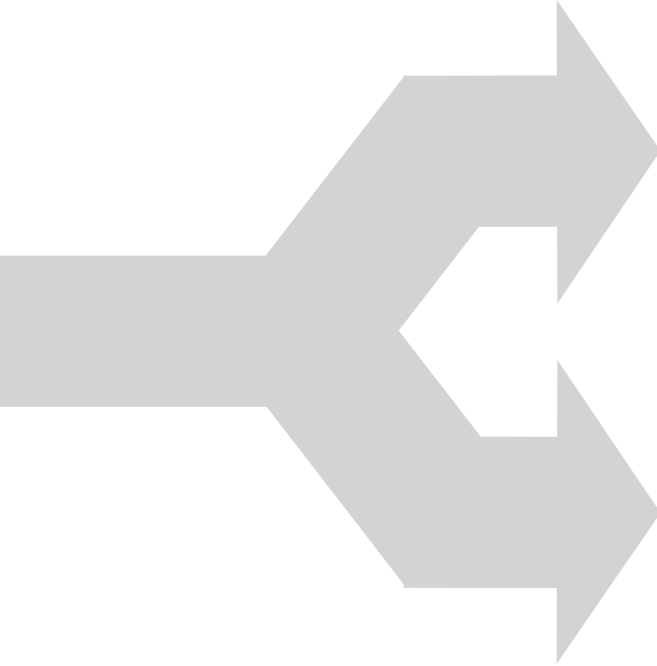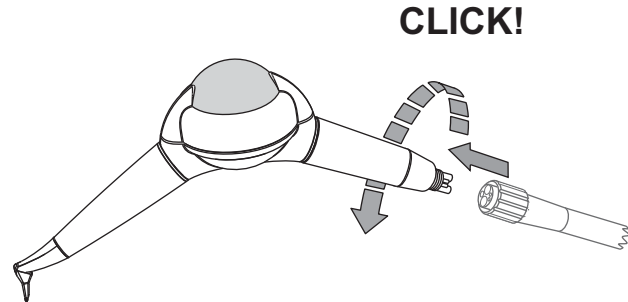
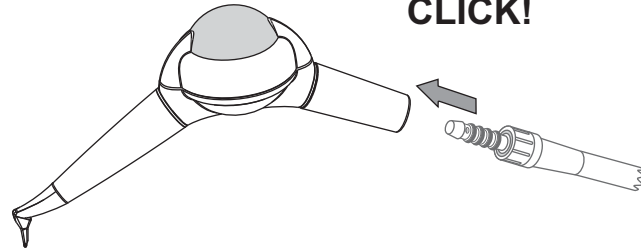
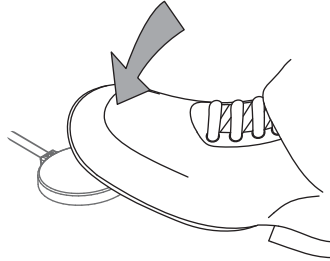
 Never use the AIR-FLOW handy PERIO handpiece without water flow.

 It is better to set the water flow rate before using the device for the first time while the powder chamber is empty.

Set the water flow rate from your handpiece to obtain a small continuous spray.

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PRECAUTIONS

Wear a mask and eye protection

 A powder jet accidentally directed into the eye can cause severe ocular injury. Therefore it is strongly recommended for all persons to wear eye protection during treatment.

 To limit the risk of bacterial or viral infections and inhalation of powder, we recommend to the dentist and the hygienist to wear a safety protection mask.

 The contact lenses or glasses of the patient may get dirty during the treatment. We recommend to take them off.

 The nozzle should never be pointed toward the patient before, during and after operation.

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
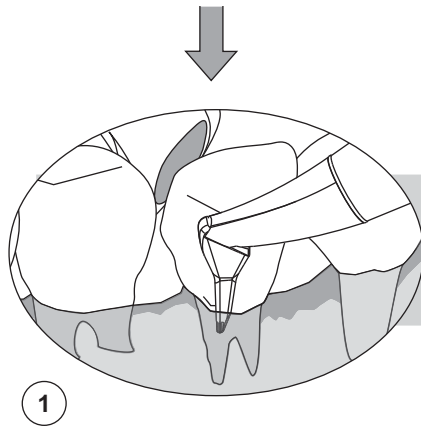
**"ON"
5 SEC**

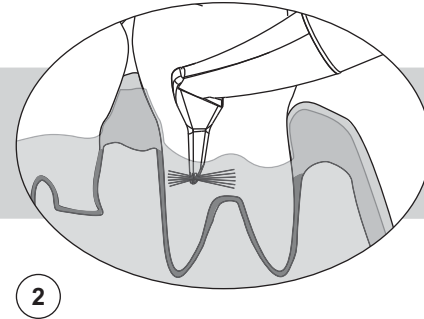1. Insert the nozzle into the periodontal pocket. The penetration depth depends on the pocket depth. Do not force the nozzle into the pocket because this could hurt soft tissues.

2. When the nozzle is inserted in the pocket, press the pedal of your device for 5 seconds. EMS powder/air mixture confined by a water jet is sprayed into the periodontal pocket. Due to the mechanical action of the air, powder and water spray, the subgingival biofilm is eliminated from the root surface.

3. Place the nozzle on the edge of the sulcus and let the spray act for 5 seconds maximum on each side of the tooth (vestibular, lingual, distal).

Carry on the treatment with short tiny rotary motions of the nozzle.

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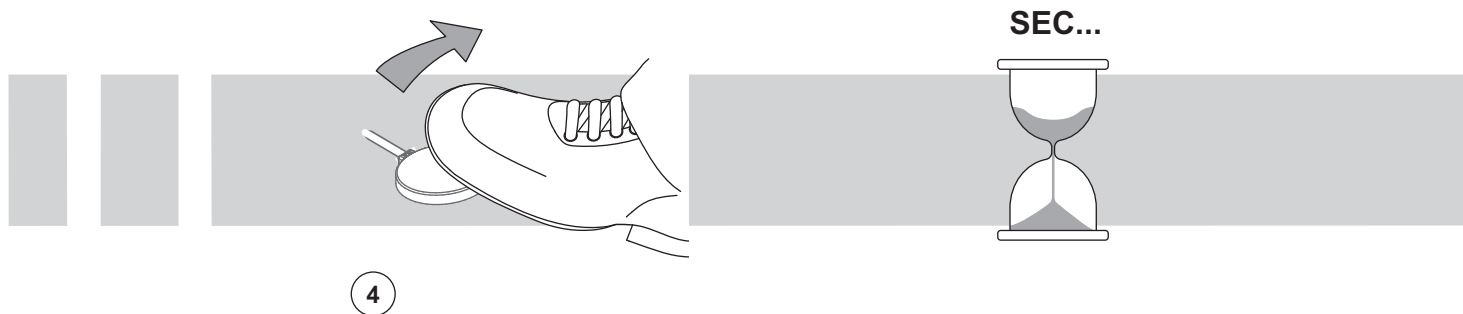
4

4. Release the pedal and wait until the spray stops.

 When removing the foot from the control pedal, the air/powder jet continues for a few more seconds. Take these seconds into account.

5. Remove the nozzle from the pocket and repeat the operation in the next pocket.

 Never spray outside periodontal pockets. External operation accelerates nozzle wear and causes a multidirectional spray jet which can be unpleasant for the patient.

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SAFETY PRECAUTIONS

EMS and the dealer of this product accept no liability for direct or consequential injury or damage resulting from improper use, arising in particular through non-observance of the operation instructions, or improper preparation and maintenance.

Use for the intended purpose only. Before using the product, make sure that the manual has been read. This also applies to any equipment used with this product. Failure to observe the operation instructions may result in the patient or user suffering serious injury or the product being damaged, possibly beyond repair.

Risk of explosion: Do not use this product in the presence of inflammable anaesthetics or gases.

This product must be used only by trained and qualified personnel (dentist or dental hygienist).

Before each use, clean, disinfect and sterilize the different pieces and accessories of this product (nozzles excepted). Please refer to the information provided in the operation instructions. Non-sterile pieces and accessories may cause bacterial or viral infections.

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 Use only the powder recommended by EMS for the subgingival application when using the AIR-FLOW handy PERIO. Any other powder could cause premature wear of the nozzle and would increase the risk of emphysema and damage to the unenameled dentine.

 Always examine the product for damage before starting the treatment. Damaged accessories or a damaged device must not be used and must be replaced. Use original EMS spare parts and accessories only.

 Have this product exclusively repaired by an approved EMS repair center.

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STORING THE PRODUCT

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 Keep the original packaging until the product is to be disposed of permanently. You can use it for shipping or storing at any time.

Should you wish to store the product for an extended period of time:

- Proceed as described in the chapter "Cleaning, disinfecting and sterilizing".
- Pack the product and all accessories in the original packaging.

Please refer to the storage and transport conditions in the "Technical data".

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PRODUCT DISPOSAL

The product, its accessories and its packaging do not contain any substances that are dangerous for the environment.

Should you wish to definitively dispose the product, please comply with the regulations which apply in your country.

WARRANTY

The warranty is valid from the date of purchase of your product for the period indicated in the Registration Card.

Damages due to non-adherence to the operation instructions or wear out of parts are excluded from warranty.

The warranty of your product will be cancelled if you try to disassemble it.

ACCESSORIES

Accessories are available from EMS or any authorized dealers. Please contact your customer service directly.

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EMS-SERVICE

Should your product need additional servicing or repairs, please send it to a certified EMS Repair Center.

In the case of non-authorized repairs or damaged due to non-adherence to the operation instructions, EMS accepts no liability whatsoever. This will also void the warranty.

It is best to ship your product in the original packaging. It protects your product against damage during shipment.

Before dispatching your product including all accessories, please clean, disinfect and sterilize as described in the operation instructions.

When sending your product directly to the approved EMS repair center, please include the name of your dealer. This simplifies processing for us.

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TECHNICAL DATA

DESCRIPTION

Manufacturer	EMS SA, CH-1260 Nyon, Switzerland
Model	AIR-FLOW® handy PERIO
Classification 93/42 EEC	Class IIa
Mode	Continuous operation
Service Pressure to the turbine connection: Water Temperature:	0.7 to 1.5 bar (700-1500 hPa) with a service flow of 20-80 ml/min. 40 °C
Service Pressure to the turbine connection: Air	Static pressure 3 to 4 bar (3000-4000 hPa)
Weight	Approx. 0.160 kg
Operation conditions	+10 °C to +40 °C 30% to 75% relative humidity
Storage and transport conditions	-10 °C to +40 °C 10% to 95% relative humidity 500 hPa to 1060 hPa air pressure

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TROUBLESHOOTING

Symptom	Solution
Water comes into the powder chamber	<ul style="list-style-type: none">• Check the connection for the water and air at the dental unit• Check the O-rings of the turbine connection• Check the O-rings on the device• Send the device to a certified EMS Repair Center
No powder/air jet coming from the device / Nozzle	<p><i>Immediately release the pedal</i></p> <p><i>Wait 1-2 minutes for pressure release in the system</i></p> <p><i>Remove the powder from the device to check the condition of the handpiece and the device</i></p> <p>→ Replace the nozzle</p> <p>→ Check if air comes out of the turbine connection</p> <ul style="list-style-type: none">• Unplug the device• Press on the pedal• If air not come out from the cord, check the dental unit <p>→ Check if the handpiece is clogged</p> <ul style="list-style-type: none">• Unplug the handpiece• Press on the pedal• If air comes out of the device, the handpiece is clogged, clean the handpiece <p>→ Check if the device is clogged</p> <ul style="list-style-type: none">• Unplug the handpiece• Press the pedal• If air does not come out of the device, the device is clogged, send the device to a certified EMS Repair Center
No water spray is coming from the nozzle	<ul style="list-style-type: none">• Change the nozzle• Remove the nozzle and verify that a water jet comes out from the handpiece• Remove any residue from the handpiece with the cleaning needle in both water & air channels• Send the device to a certified EMS Repair Center
Water leakage between the handpiece and the nozzle (unusual leak of droplets)	<ul style="list-style-type: none">• Check the position of the nozzle and its position on the handpiece• Change the nozzle• Check for residues between the nozzle and the handpiece• Check that nothing affect the alignment of the nozzle• Eliminate residual powder from the channels using cleaning needle• Send the device to a certified EMS Repair Center

Water leakage between the nose connection and handpiece	<ul style="list-style-type: none"> • Check the O-rings on the nose connection of the device, replace them if necessary • Change the handpiece • Send the device to a certified EMS Repair Center
Air and/or powder is leaking from the threads of the cap	<ul style="list-style-type: none"> • Check the seal and the cleanliness of the thread on the powder chamber and on the cap • Replace the seal if necessary
The efficiency diminishes	<ul style="list-style-type: none"> • Check the quantity of powder in your device • Change the nozzle • Eliminate residual powder from the handpiece in the channels using the cleaning needle • Send the device to a certified EMS Repair Center
If the handpiece falls	<ul style="list-style-type: none"> • Replace the nozzle and check the holding of the nozzle • If the nozzle doesn't hold or it is not waterproof, send the handpiece to a certified EMS Repair Center

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EMS Corporation