**MANUALE UTENTE
USER MANUAL
BEDIENUNGSANLEITUNG**

ALGIMIX

ITALIANO

ENGLISH

DEUTSCH

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Declaration of Conformity	2
Condition of warranty	3
Loss of warranty	4
Delivery terms	5
General warnings and safety rules	6
Useful contacts and addresses	6
Symbols	6
Product description	8
Constructive specification	9
Installation	10
Description of machine operation	11
Setting of mixing time	12
Preparation of mixture	13
Demolition and dismantling	14
Maintenance and troubleshooting	15

DECLARATION OF CONFORMITY

This device compliance to Directive 93/42/EEC .

The original declaration of conformity is provided in attached to the manual.

CONDITION OF WARRANTY

- 1) **DURATION:** the product is covered by a 12-month (twelve) guarantee.
- 2) **EXCLUSION:** the guarantee excludes:
 - a) faults originating from non-performance of scheduled maintenance and due to negligence or improper use of the product by the User;
 - b) periodic checks and maintenance;
 - c) repair or replacement of parts subject to wear that are fragile or have an unpredictable life, unless they prove faulty at the time of their delivery;
 - d) faults caused by personnel or originating from technical personnel transfer and transportation;
 - e) faults or damage originating from improper use or operating errors;
 - f) faults or damage originating from contamination in the water and air infeed systems, extraordinary chemical agents or electrical events;
 - g) faults or damage originating from the use of detergents, disinfecting agents, sterilizing fluids or processes that are not described explicitly in the instruction manual for operation and maintenance;
 - h) natural color change of plastic components.
- 3) **INSTALLATION AND TESTING REPORT:** an essential prerequisite for guarantee validity is the return of the installation and testing report after all its sections have been filled in by the User and the Fitter. The report must be returned within 15 days after installation, otherwise the guarantee becomes null and void.
- 4) **LIMITS:** the guarantee entitles its holder to repairs or replacement of faulty components free of charge. The right to replace the entire machine is excluded. As for components either applied or added to TECNO-GAZ products or by third parties, having their own guarantee certificate, the conditions, limits and exclusions indicated in the relevant certificates apply.

Under no circumstances shall carrying out of one or more than one repair during the guarantee term change the guarantee expiry date.
- 5) **CLAIMS:** in the event of the Purchaser making a claim relating to the application of the guarantee and the quality or conditions of the delivered equipment, the said Purchaser shall in no way suspend and/or delay payments.
- 6) **DISPUTES:** if a dispute arises on the application and interpretation of this Certificate of guarantee, the Court of PARMA (Italy) shall be the court of jurisdiction, regardless of the location in which the purchase agreement was stipulated.
- 7) **DEROGATIONS:** any special derogations from the guarantee terms described in this section shall not imply the granting to the Purchaser of any right and shall be considered as granted for a specific case only.

8) MISCELLANEOUS: for any other issue, which is not covered by this Certificate of Guarantee, the rules described in the Civil Code and existing relevant Laws in the Republic of Italy apply.

LOSS OF WARRANTY

The guarantee becomes null and void in the event that:

- a) the equipment presents damage caused by a fall, exposure to flames, liquid spillage, natural events, bad weather conditions or by any other cause that is not due to manufacturing defects;
- b) installation has not been performed in line with TECNO-GAZ instructions and by authorized personnel;
- c) the equipment is repaired, modified or tampered with by the Purchaser or by non-authorized third parties;
- d) when asking for an intervention under guarantee, the product serial number has been removed, deleted, counterfeited, etc.;
- e) the installation and testing report is not returned suitably filled in and signed within 15 days from installation;
- f) the Purchaser suspends and/or delays payment of amounts due, for any reason whatsoever, relating to the purchase and/or maintenance of the equipment;
- g) scheduled periodic maintenance or any other instruction envisaged in the manual for operation and maintenance is not carried out.

DELIVERY TERMS

1. All goods delivered must be packaged in their original packaging. If the goods are delivered in a non-genuine packaging, a charge will be applied to restore the original packaging.
2. All goods must be delivered CARRIAGE PAID.
3. The client shall ask a written authorization to TECNO-GAZ sales dept. before returning the goods by means of the "Form for Client's Returned Goods Management" (CM-M-002) in compliance with the requirements of TECNO-GAZ Quality Management System (ref. CM-P-003, CM-I-009).
4. The document accompanying the returned goods shall show:
 - the TECNO-GAZ product code/item no.;
 - the number of the TECNO-GAZ purchase invoice or shipment document;
 - the product serial number and batch number;
 - a description of the goods returned;
 - the number of the Form for Client's Returned Goods Management" (CM-M-002) or a copy of the same attached.
5. Returned goods are considered as accepted only after they have been examined. If they prove non-conforming, they may be sent back to the consignor at TECNO-GAZ full discretion.
6. Charges will be applied to restore and test the returned product.
7. Parts replaced under guarantee must be returned, carriage paid, to TECNO-GAZ.
8. If the parts are not returned, their cost will automatically be debited to the recipient.
9. TECNO-GAZ does not accept goods returns from end users.
10. Goods sent to TECNO-GAZ for repair are managed as returned goods.

GENERAL WARNINGS AND SAFETY RULES

-  Make sure that the equipment is supplied with the correct voltage as shown in the plate.
-  Do not remove the plate.
-  Make sure that the machine is equipped with a grounding system.
-  Use a dry cloth to clean the machine.
-  Before any operation is performed, disconnect the power cable from the wall outlet.
-  Use original spare parts only.

Non-compliance with the rules above relieves TECNO-GAZ of all responsibility.

USEFUL CONTACTS AND ADDRESSES

For technical service please apply to the dealer that invoiced the device, or directly to:

TECNO-GAZ S.p.A.

Str. Cavalli n°4 – Sala Baganza – Parma – ITALIA.

Tel. +39 521 8380 - Fax +39 521 833391

e-mail : info@tecnogaz.com

<http://www.tecnogaz.com>

SYMBOLS

Pursuant to DIRECTIVE 2002/96/EC, this symbol shows that the product must not be disposed of as urban waste at the end of its operating life.

General warning and information for the user.

Pay attention to the instructions provided alongside this symbol.

Conformity with the Directive 2004/108/CE and Directive 2006/95/CE and further amendments.

PE External protection.

Protection GROUNDING SYSTEM.

(Operating) GROUNDING SYSTEM.

Alternate current.

Warning: high voltage. Risk of electrocution!

Read the instruction manual carefully.

Applied part.

Press the button shown.

POS. Position (number identifying a component in the figures or diagrams).

Fg. Figure

Ω Ohm (Electric Resistance unit).

s Seconds (time unit).

W Watt (Power unit).

Hz Hertz (Frequency unit).

mm Millimeters (Length unit).

A Ampere (Electric Current unit).

V Volt (Electric Voltage unit).

Pa Pascal (Atmospheric Pressure unit).


°C Degrees Centigrade (Temperature unit).

kg Kilograms (Weight unit).


PRODUCT DESCRIPTION

Prior to using the product, read this user manual carefully and respect the proper maintenance to ensure the duration of the unit.

Store carefully this manual.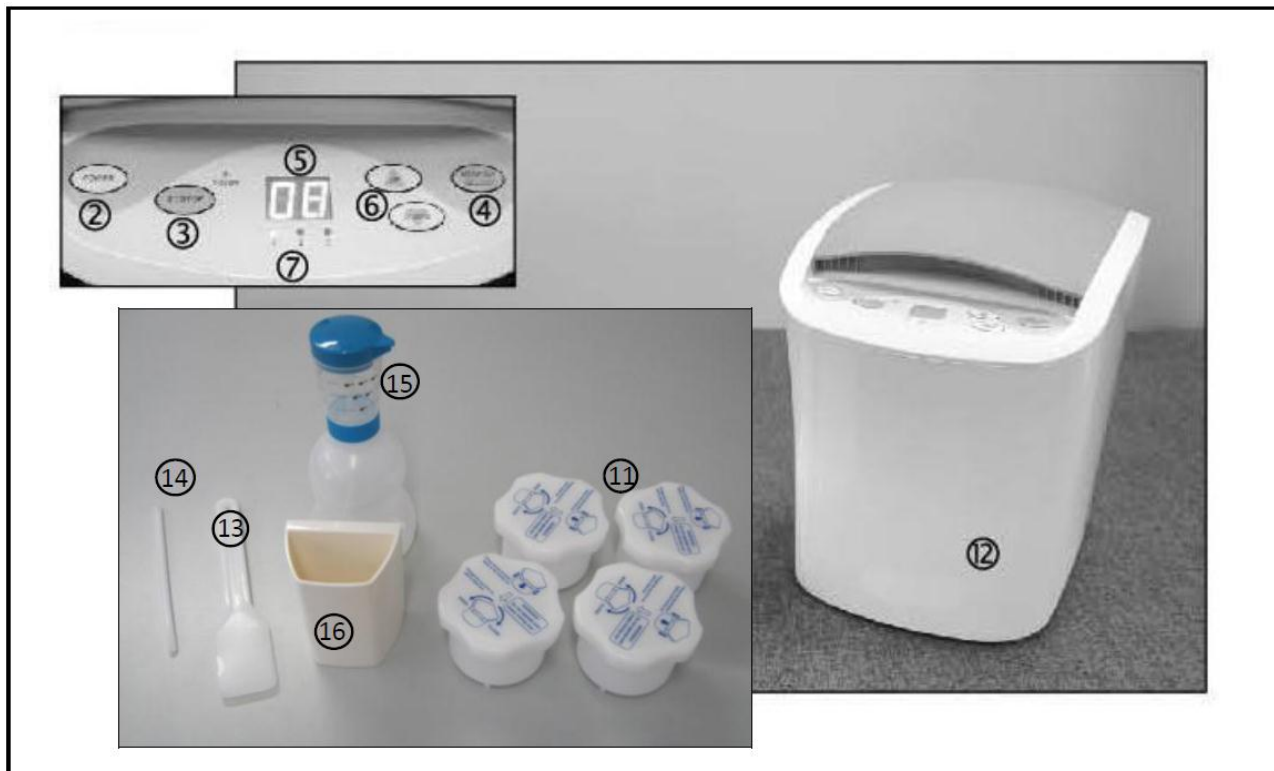

1	Main power switch (I/O)
2	POWER switch (On/Off)
3	START/STOP button (ST/STOP)
4	Memory selection button (MEMORY)
5	Display
6	Setting time buttons (UP/DOWN)
7	Memory indicator
8	Supply cable
9	Plug of Supply cable
10	Fuse carrier
11	Mixing bowl
12	Machine casing
13	Spatula
14	Stirrer
15	Plastic dispenser
16	Plastic holder with magnet

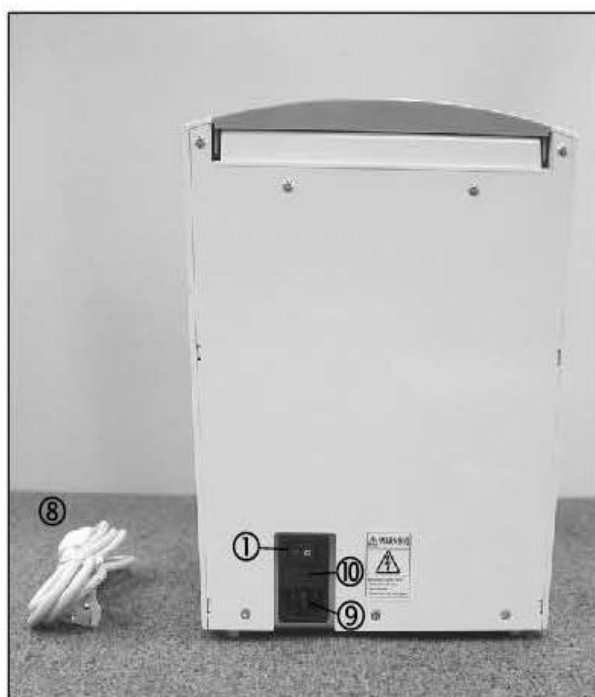
CONSTRUCTIVE SPECIFICATION

Ambient temperature of use	5°C ÷ 40
Sizes (X,Y,Z)	240 x 300 x 280 mm
Weight	15.5 kg
Packaging sizes (X,Y,Z)	420 x 410 x 300 mm
Supply voltage	AC 220V~230V (50Hz)
Number of phases	1
MAX. absorbed power	1500 W
Operating power:	400 W
Revolution	3000 r.p.m
Length of the supply cable	L = 1500 mm
Timer	1-16 sec.
Memory	3 (installed setting 8,10,12 sec.)

X = Width Y = Height Z = Depth

INSTALLATION

1. Remove the two fixing bolts at the bottom of machine (fig.1).

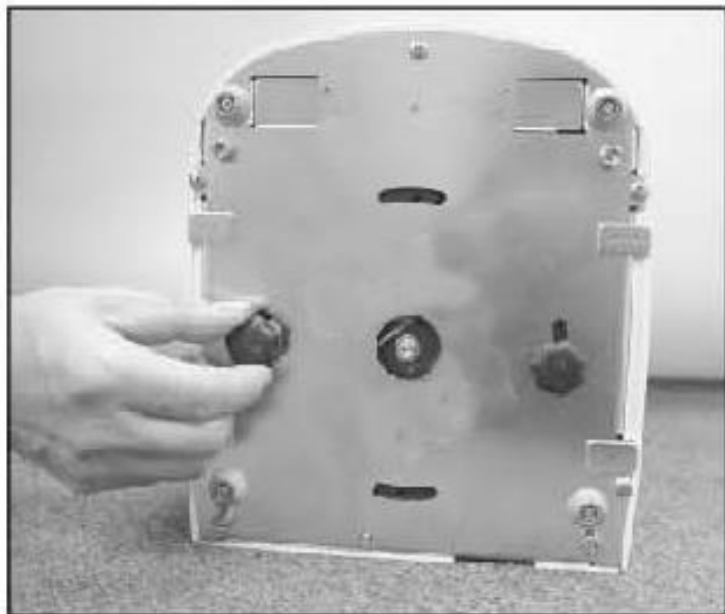*fig.1*

Store carefully the fixing bolts. In case of transport (as, for example, for a shipment of repair) reuse absolutely to avoid breakage or malfunctions

2. Place the alginate mixer on a horizontal plane to be able to support the weight.
3. Check that the supply voltage corresponds with that given in the manual and on metal data plate on the back of the unit.
4. Connect the supply cable with the socket on the machine and then to a ground outlet (fig.2)

*fig.2*

DESCRIPTION OF MACHINE OPERATION

1. Press the main power switch on the back of the machine (fig. 3)

*fig.3*

2. Press the POWER switch button that is located on the control panel, and make sure that the red led indication is lit.
3. Prepare the alginate powder and close the mixing bowl.
(see *PREPARATION OF MIXTURE*)
4. Open the upper cover. Place the mixing bowl inside the compartment mixing and close the cover (fig. 4)

*fig.4*

5. Select the mixing time (see *SETTING OF MIXING TIME*)
6. After have finished all settings, press ST/STOP button. The display will show the counting of seconds until the end of the cycle of mixing.

The upper cover of mixer is closing during the operation through a security mechanism that prevents accidental opening

If you intend to stop the mixer during the operation, press ST/STOP button.

If during the operation occur abnormal vibration or noise, turn off the machine immediately.

The use of the machine must be performed by authorized personnel

- After verifying that the mixer has stopped, open the cover and remove the mixing bowl and use the spatula to extract the compaund obtained.

SETTING OF MIXING TIME

The machine has 3 programs in memory established by the manufacturer and selectable through MEMORY SELECT button to set the appropriate mixing time.

MEMORY	MIXING TIME (sec.)
1	8
2	10
3	12

You can change the mixing time acting on the UP/DOWN button to get the value of time desired. The maximum time is 16 seconds (you can intervene for the setting of time second by second). When the cycle ends the display will reset to the originally value set.

In addition, you can change the memory of mixing time established by the manufacturer.

- Press MEMORY SELECT button to select the desired position in memory.
- Press UP/DOWN button to obtain the value of time desired.
- The mixing time chosen is stored in memory.

PREPARATION OF MIXTURE

Dose in the bowl the necessary quantity of powder (fig. 5) and add corresponding amount of water, following indications of alginate manufacturer, because the amount of water added will have effects on viscosity of the mixture of alginate.

For an optimal mixture, mix lightly with plastic stirrer.

fig.5

The total quantity of powder and water in the container should not exceed 100 g.

Close the capo of mixing bowl pressing the two sides.

	Close the cap to prevent the escape of alginate during the mixing; this could influence negatively on operation of the machine.
	If it contained only water in the container, the mixer should not be turned on.
	Never leave obstructed the hole on the capo of mixing bowl.
	The mixing bowl is an accessory subject to spoilage. If no longer close tightly, replace with a new one

DEMOLITION AND ELIMINATION

Directive on waste from electrical and electronic equipment (WEEE)

Pursuant to DIRECTIVE 2002/96/EC, this symbol shows that the product must not be disposed of as urban waste at the end of its operating life.

The product must be taken to centers specialized in separate collection of electric and electronic equipment or to a dealer, when a similar product is purchased.

The machine holder shall be responsible for bringing it to collection centers.

For further information about collection systems, address the local waste disposal service.

Correct disposal of the equipment, which is no longer used, prevents negative consequences for the environment and human health.

Abuses will be prosecuted by law.

MAINTENANCE AND TROUBLESHOOTING

For cleaning the mixer is sufficient to refer the most basic hygienic care

- First turn off the machine and pull the plug from the outlet.
- After each use, wash and dry the mixing bowl and the cap, making sure to dry well to avoid possible errors of calculation in the successive use.
- Do not use a knife or a similar object to avoid scratching the surface .
- Make sure that the hole on cap is not obstructed: could make it difficult to open and close the cap.
- Use a dry cloth with neutral detergent to clean the outer part of the mixer
- Do not use solvents or detergents, which could alter irreversible manner the appearance of plastic parts of the machine.

DO NOT DO MAINTENANCE OR INSIDE REPAIR TO THE DEVICE.
DISMANTLE THE DEVICE MEANS BREAKING IT.

<i>Defect</i>	<i>Cause</i>	<i>Intervention</i>
The unit will not turn	The power cable is not connected well	Check the connection of cable to the machine and to the wall outlet.
	Fuses burned	Check the fuses and replace if necessary. In case of new break, contact the dealer
The display not turn on	The electronic card is worth	Contact the dealer
Slow start-up of the machine	The engine or transmission are broken	Contact the dealer
	The starting condenser is broken	Contact the dealer
Noise or abnormal vibrations during operation	The mixing bowl is too load	The total content of powder and water should not exceed 100 g.
	The mixing bowl is empty	Never operate the unit when the bowl is empty
	Leak of powder and water during the mixing	Make sure to have closed the mixing bowl. Control that the bowl and the cap are not damaged
	Transmission fails	Contact the dealer
Error "E1" appears on electronic indicator	The upper closure is not in correct position	Close correctly upper cover

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TECNO-GAZ S.p.A. si riserva il diritto di apporre modifiche al presente documento senza dare alcun pre-avviso.
TECNO-GAZ S.p.A reserves the right to modify the enclosed document without notice.
TECNO-GAZ S.p.A reserves the property of the document and forbids others to use it or spread it without its approval.